


Liberty Height Adjustable T-Arm

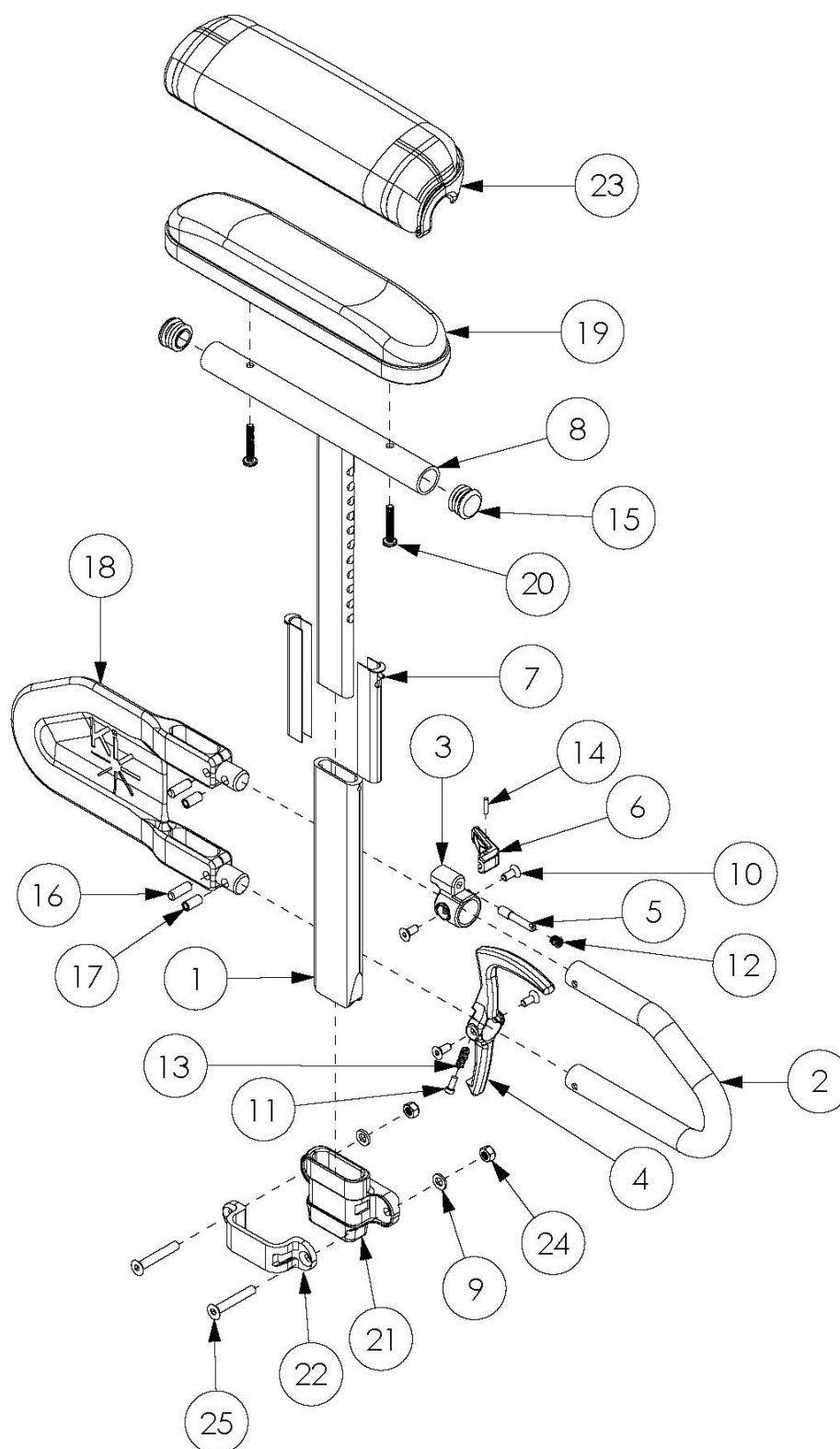

If retrofitting, Receivers are needed.

Item Number	Part Number	Description	Quantity
1-7, 8a, 9-18, 19b, 20a, 21, 22, 24, 25	506188	Height Adjustable T-Arm Assembly with Short Side Shield, Standard Pad Desk and Receiver, PAIR, Liberty	1
1-7, 8a, 9-18, 19b, 20a, 21, 22, 24, 25	506187	Height Adjustable T-Arm Assembly with Short Side Shield, Standard Pad Desk and Receiver, Liberty	1
1-7, 8b, 9-18, 19a, 20a, 21, 22, 24, 25	506190	Height Adjustable T-Arm Assembly with Short Side Shield, Standard Pad Full and Receiver, PAIR, Liberty	1
1-7, 8b, 9-18, 19a, 20a, 21, 22, 24, 25	506189	Height Adjustable T-Arm Assembly with Short Side Shield, Standard Pad Full and Receiver, Liberty	1
1-7, 8a, 9-18, 20b, 21, 22, 23b, 24, 25	506338	Height Adjustable T-Arm Assembly with Short Side Shield, Waterfall Pad Desk and Receiver, PAIR, Liberty	1
1-7, 8a, 9-18, 20b, 21, 22, 23b, 24, 25	506337	Height Adjustable T-Arm Assembly with Short Side Shield, Waterfall Pad Desk and Receiver, Liberty	1
1-7, 8b, 9-18, 20b, 21, 22, 23a, 24, 25	506340	Height Adjustable T-Arm Assembly with Short Side Shield, Waterfall Pad Full and Receiver, PAIR, Liberty	1
1-7, 8b, 9-18, 20b, 21, 22, 23a, 24, 25	506339	Height Adjustable T-Arm Assembly with Short Side Shield, Waterfall Pad Full and Receiver, Liberty	1
1-7, 8a, 10-18, 19b, 20a	114157	Height Adjustable T-Arm Assembly with Short Side Shield, Standard Pad Desk, PAIR, Liberty	1
1-7, 8a, 10-18, 19b, 20a	114156	Height Adjustable T-Arm Assembly with Short Side Shield, Standard Pad Desk, Liberty	1
1-7, 8b, 10-18, 19a, 20a	114159	Height Adjustable T-Arm Assembly with Short Side Shield, Standard Pad Full, PAIR, Liberty	1
1-7, 8b, 10-18, 19a, 20a	114158	Height Adjustable T-Arm Assembly with Short Side Shield, Standard Pad Full, Liberty	1
1-7, 8a, 10-18, 20b, 23b	114161	Height Adjustable T-Arm Assembly with Short Side Shield, Waterfall Pad Desk, PAIR, Liberty	1
1-7, 8a, 10-18, 20b, 23b	114160	Height Adjustable T-Arm Assembly with Short Side Shield, Waterfall Pad Desk, Liberty	1
1-7, 8b, 10-18, 20b, 23a	114163	Height Adjustable T-Arm Assembly with Short Side Shield, Waterfall Pad Full, PAIR, Liberty	1
1-7, 8b, 10-18, 20b, 23a	114162	Height Adjustable T-Arm Assembly with Short Side Shield, Waterfall Pad Full, Liberty	1
3, 5, 6, 10, 12, 14	505490	Height Adjustable T-Arm Upper Latch Assembly	1
4, 10, 11, 13	505507	Height Adjustable T-Arm Release Lever Assembly	1
16, 17, 18	506082	Height Adjustable T-Arm Short Shield Assembly	1
1-7, 10-14, 16-18	004160	Height Adjustable T-Arm Lower Assembly Short <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1
1-7, 8a, 10-18	003463	Height Adjustable T-Arm Short Desk <i>Does not include arm pad or arm pad screws</i>	1

Item Number	Part Number	Description	Quantity
1-7, 8b, 10-18	003464	Height Adjustable T-Arm Short Full <i>Does not include arm pad or arm pad screws</i>	1
9, 21-22, 24-25	116056	Height Adj T-Arm Receiver Assm Liberty	1
1	000089	Height Adjustable T Armrest Post	1
2	000095	Height Adjustable T Armrest Transfer Tube	1
7	000090	Height Adjustable T Armrest Shim	2
8a, 15	112906	Height Adjustable T Armrest Upper Desk w/ Tube End Plug	1
8b, 15	112907	Height Adjustable T Armrest Upper Full w/ Tube End Plug	1
9	100746	M6 Flat Washer Blk Zn	2
10	003458	M5 X 12mm FHCS BLK ZN W/ Patch	4
11	000152	Height Adjustable T Armrest Release Spring Button	1
13	000135	Height Adjustable T Armrest Release Lever Spring	1
15	100803	Tube End Plug 7/8"	2
16	003641	.25" x .75" Dowel Pin	2
17	003457	M5 x 14mm Barrel Nut	2
19a	100500	Arm Pad Full	1
19b	100501	Arm Pad Desk	1
20a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2
20b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2
21	003336	Receiver, T-Arm, Liberty	1
22	003337	Bracket, T-Arm	1
23a	102452	Waterfall Arm Pad Full	1
23b	102521	Waterfall Arm Pad Desk	1
24	100658	M6 Nylock Nut Blk Zn	2
25	101819	M6 x 40 FHCS Blk Zn	2