


Rogue2 Standard Camber Tube

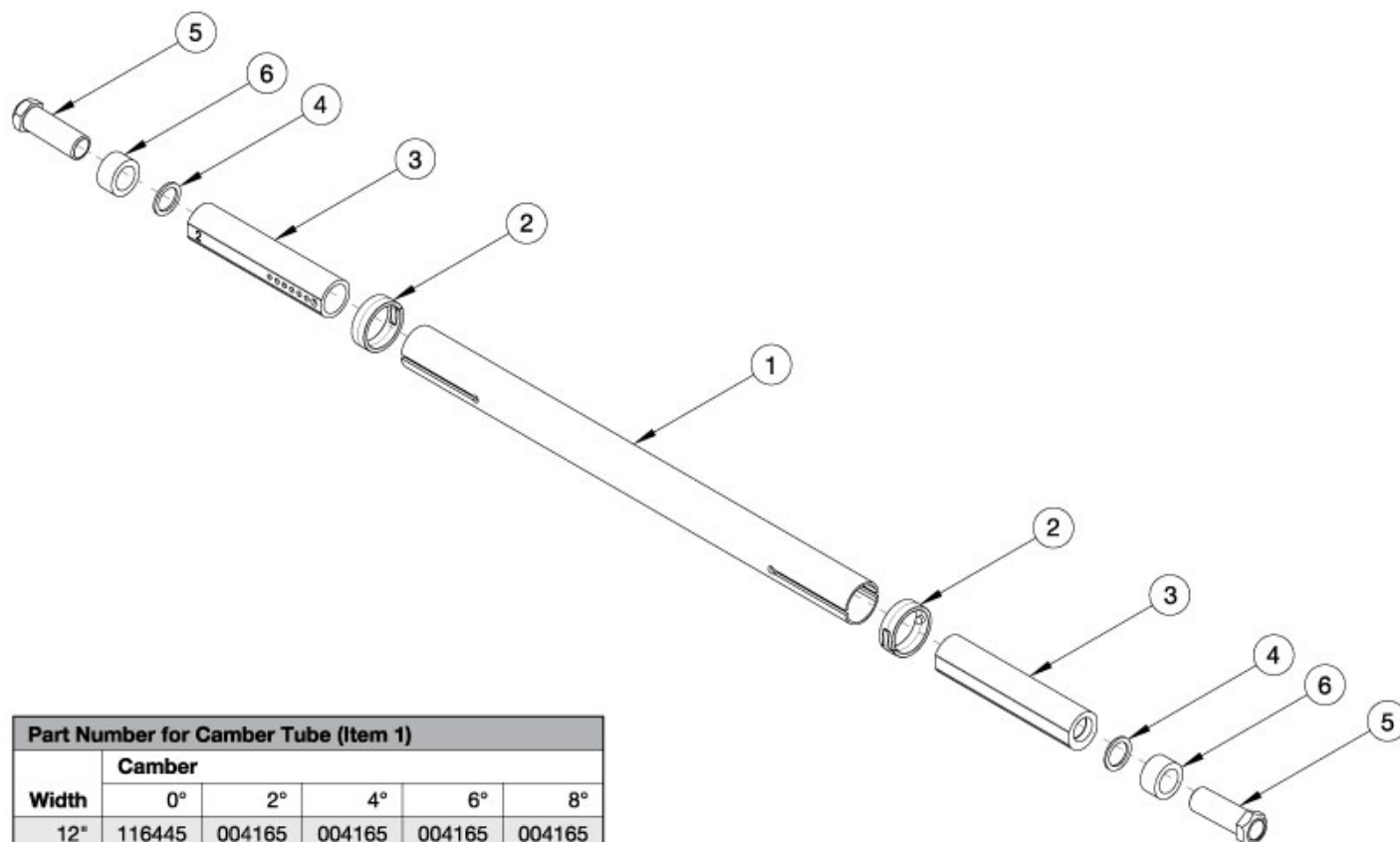

Part Number for Camber Tube (Item 1)					
	Camber				
Width	0°	2°	4°	6°	8°
12"	116445	004165	004165	004165	004165
13"	004165	004166	004166	004166	004166
14"	004166	004167	004167	004167	004167
15"	004167	004168	004168	004168	004168
16"	004168	004169	004169	004169	004169
17"	004169	004170	004170	004170	004170
18"	004170	004171	004171	004171	004171
19"	004171	004172	004172	004172	004172
20"	004172	004173	004173	004173	004173

If One Arm Drive is present, refer to the corresponding One Arm Drive parts manual page.

Item Number	Part Number	Description	Quantity
1a	116445	11" Wide Camber Tube, Black Anodized	1
1b	116445_ _ _	11" Wide Camber Tube, Color Anodized/Color Accent	1
1c	004165	12" Wide Camber Tube, Black Anodized	1
1d	004165_ _ _	12" Wide Camber Tube, Color Anodized/Color Accent	1
1e	004166	13" Wide Camber Tube, Black Anodized	1
1f	004166_ _ _	13" Wide Camber Tube, Color Anodized/Color Accent	1

Item Number	Part Number	Description	Quantity
1g	004167	14" Wide Camber Tube, Black Anodized	1
1h	004167_ _ _	14" Wide Camber Tube, Color Anodized/Color Accent	1
1i	004168	15" Wide Camber Tube, Black Anodized	1
1j	004168_ _ _	15" Wide Camber Tube, Color Anodized/Color Accent	1
1k	004169	16" Wide Camber Tube, Black Anodized	1
1l	004169_ _ _	16" Wide Camber Tube, Color Anodized/Color Accent	1
1m	004170	17" Wide Camber Tube, Black Anodized	1
1n	004170_ _ _	17" Wide Camber Tube, Color Anodized/Color Accent	1
1o	004171	18" Wide Camber Tube, Black Anodized	1
1p	004171_ _ _	18" Wide Camber Tube, Color Anodized/Color Accent	1
1q	004172	19" Wide Camber Tube, Black Anodized	1
1r	004172_ _ _	19" Wide Camber Tube, Color Anodized/Color Accent	1
1s	004173	20" Wide Camber Tube, Black Anodized	1
1t	004173_ _ _	20" Wide Camber Tube, Color Anodized/Color Accent	1
2	111808	Detent Clip For Camber Tube	2
3a	004174	Camber Plug 0 Degrees	2
3a, 4, 5	004355	Camber Plug 0 Degrees Assembly	2
2, 3a, 4, 5	108093	Camber Plug 0 Degrees Assembly with Camber Tube Clip <i>This Kit is for one side</i>	1
3b	004175	Camber Plug 2 Degrees	2
3b, 4, 5	004356	Camber Plug 2 Degrees Assembly	2
2, 3b, 4, 5	108094	Camber Plug 2 Degrees Assembly with Camber Tube Clip <i>This Kit is for one side</i>	1
3c	004176	Camber Plug 4 Degrees	2
3c, 4, 5	004357	Camber Plug 4 Degrees Assembly	2
2, 3c, 4, 5	108095	Camber Plug 4 Degrees Assembly with Camber Tube Clip <i>This Kit is for one side</i>	1
3d	004177	Camber Plug 6 Degrees	2
3d, 4, 5	004358	Camber Plug 6 Degrees Assembly	2

Item Number	Part Number	Description	Quantity
2, 3d, 4, 5	108096	Camber Plug 6 Degrees Assembly with Camber Tube Clip <i>This Kit is for one side</i>	1
3e	004178	Camber Plug 8 Degrees	2
3e, 4, 5	004359	Camber Plug 8 Degrees Assembly	2
2, 3e, 4, 5	108097	Camber Plug 8 Degrees Assembly with Camber Tube Clip <i>This Kit is for one side</i>	1
4	004180	Washer .635 ID x .895 OD x .06 T Steel Blk Zc	2
5	004179	Axle Sleeve 16mm x 1.0 X 2" Long	2
6	004182	Spacer Aluminum .635 Id X .945 Od X .500 T <i>Used as needed. Increases rear wheel spacing</i>	2

Color Anodizing Samples

