




# Catalyst 5VX Transit Wheels

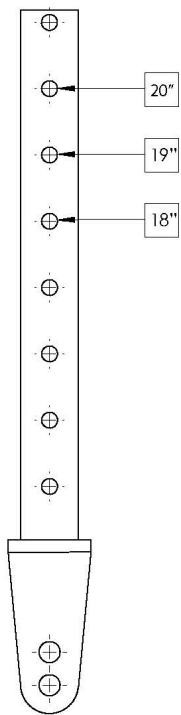


#003703 Long

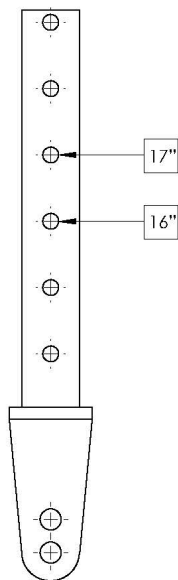

#003702 Medium

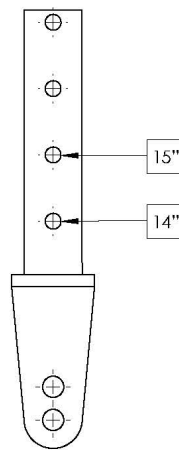

#003701 Short

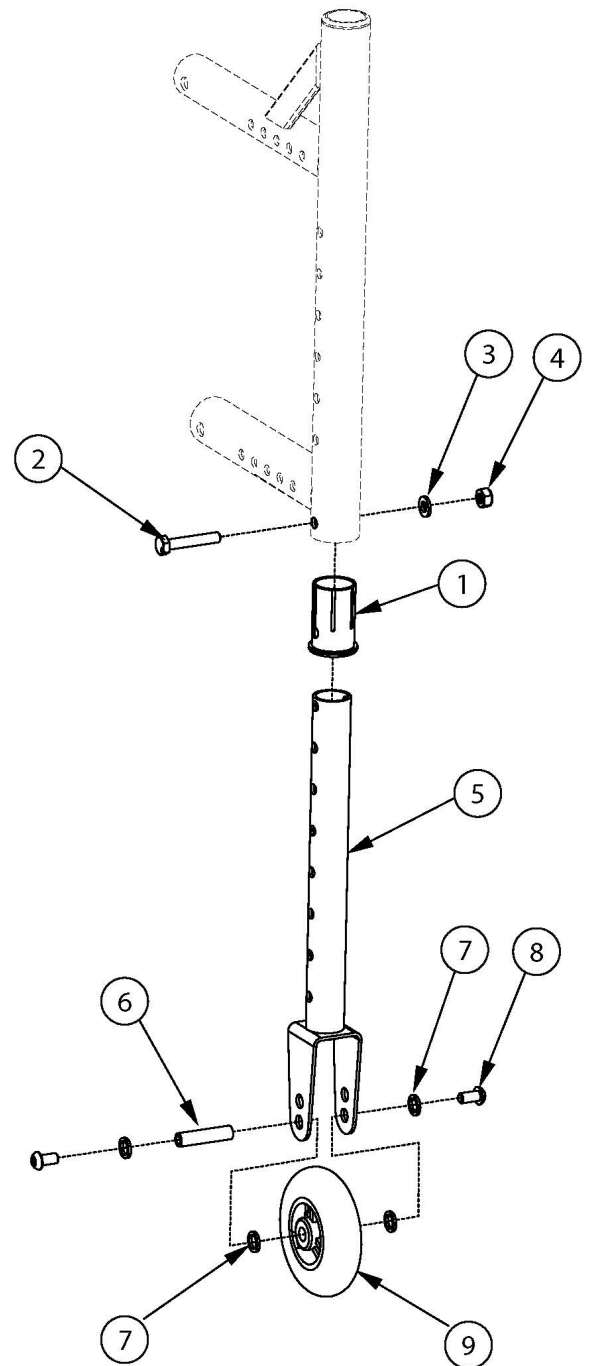

*New rear frames required with serial numbers prior to C5053765, C5HD003546, or CVX0091137 Not available with Heavy Duty Option*

Item Number	Part Number	Description	Quantity
1-4, 5a, 6-9	505421	Catalyst 5VX Transit Wheels Tower Assembly Kit, Short (PAIR) <i>Used with 14" - 15" rear seat heights</i>	1

Item Number	Part Number	Description	Quantity
1-4, 5b, 6-9	505422	Catalyst 5VX Transit Wheels Tower Assembly Kit, Medium (PAIR) <i>Used with 15.5" - 17" rear seat heights</i>	1
1-4, 5c, 6-9	505423	Catalyst 5VX Transit Wheels Tower Assembly Kit, Long (PAIR) <i>Used with 17.5" - 20" rear seat heights</i>	1
1-4, 5a, 6-9	506286	Catalyst 5VX Transit Wheels Tower Assembly Kit, Short (One Side) <i>Used with 14" - 15" rear seat heights</i>	1
1-4, 5b, 6-9	506287	Catalyst 5VX Transit Wheels Tower Assembly Kit, Medium (One Side) <i>Used with 15.5" - 17" rear seat heights</i>	1
1-4, 5c, 6-9	506288	Catalyst 5VX Transit Wheels Tower Assembly Kit, Long (One Side) <i>Used with 17.5" - 20" rear seat heights</i>	1
1	003501	Tipping Lever (Step Tube)/Transit Wheel Sleeve, CAT5VX	1
2	100860	M6 X 35 Hhcs P/Thd Zc	1
3	100746	M6 Flat Washer Blk Zn	1
4	100658	M6 Nylock Nut Blk Zn	1
5a	003701	Transit Wheel Tower Short <i>Used with 14" - 15" rear seat heights</i>	1
5b	003702	Transit Wheel Tower Medium <i>Used with 15.5" - 17" rear seat heights</i>	1
5c	003703	Transit Wheel Tower Long <i>Used with 17.5" - 20" rear seat heights</i>	1
6	000347	M6 x 1.0 Threaded Barrel	1
7	101701	Washer 3.315/.350 x .425/.525 x .059/.067 18-8 F/W	4
8	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2
9	100790	Caster 3" Rollerblade	1