


Catalyst 5 Tipping Lever

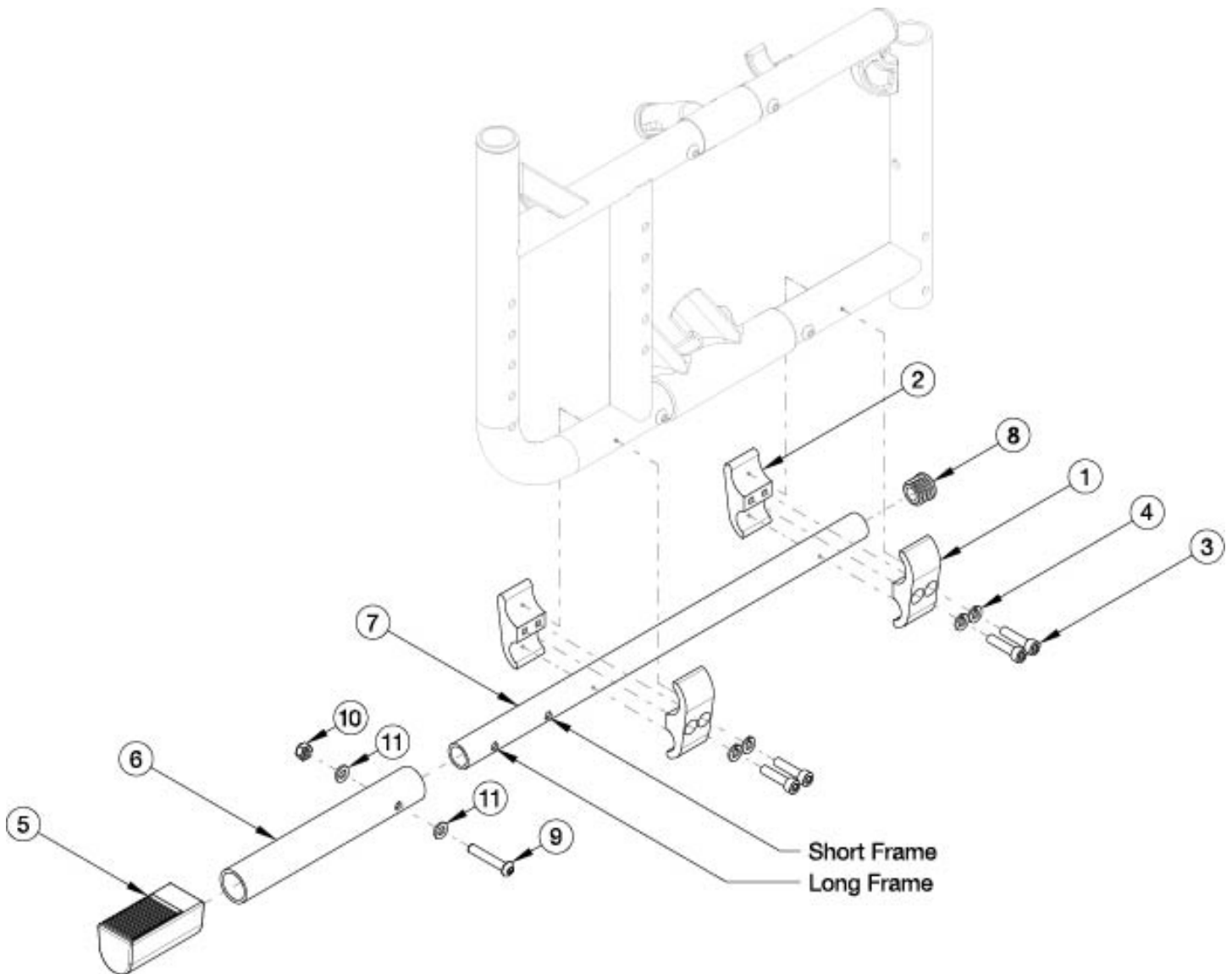

Not available with Titanium frame

Item Number	Part Number	Description	Quantity
1 - 11	120806	Tipping Lever Assembly Cat 5	1
5 - 11	121375	Tipping Lever Tube Cat 5	1
1 - 4	120808	Tipping Lever Clamps with Hardware Cat 5	2
1	002541	Clamp 1.125 to .75 C'Bored	2
2	002542	Clamp 1.125 to .75 Threaded	2
3	100780	M6 x 25 SHCS ZC	4

Item Number	Part Number	Description	Quantity
4	101835	M6 Lock Washer Blk Zn	4
6	119600	Tipping Lever, Tube, 1" OD	1
7	119601	Tipping Lever, Tube, .75" OD	1
8	100638	Tube End Plug 3/4"	1
9	101811	M6 x 35 BHCS Blk Zn	1
10	100658	M6 Nylock Nut Blk Zn	1
11	100746	M6 Flat Washer Blk Zn	2