


# Flip Non-Self Propel Push / Pull Wheel Lock

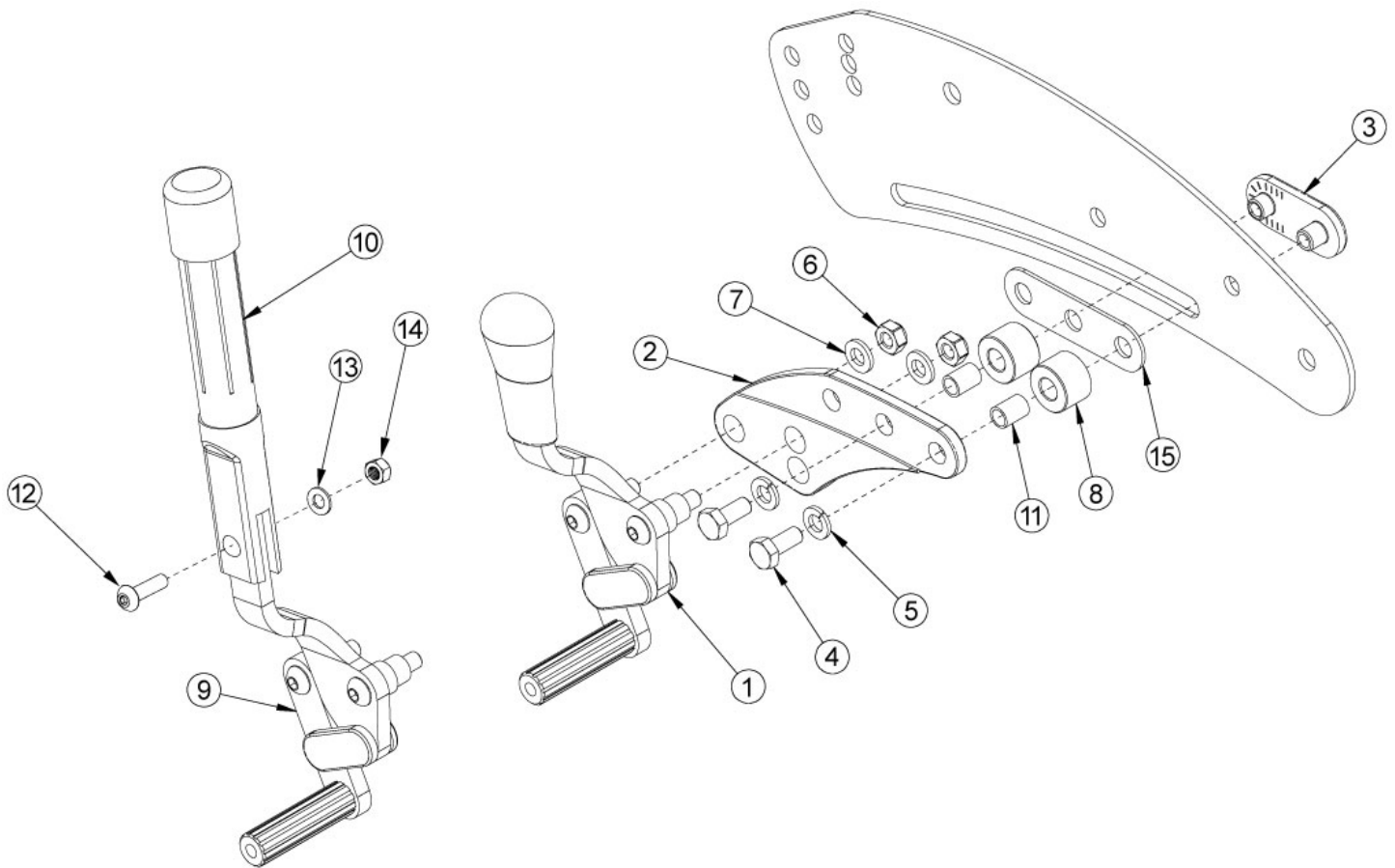

If 12" Wheels are present, use kits for corresponding wheel size.

Item Number	Part Number	Description	Quantity
1a, 2a, 3, 4b, 5-7, 15	502990	Wheel Lock Assembly, Std Push, Right, Flip	1
1b, 2a, 3, 4b, 5-7, 15	502991	Wheel Lock Assembly, Std Push, Left, Flip	1
1a, 1b, 2a, 3, 4b, 5-7, 15	502836	Wheel Lock Assembly, Std Push, PAIR, Flip	1
2a, 3, 4b, 5-7, 9a, 10, 12-14, 15	502992	Wheel Lock Assembly, Std Push w/ Extension, Right, Flip	1
2a, 3, 4b, 5-7, 9b, 10, 12-14, 15	502993	Wheel Lock Assembly, Std Push w/ Extension, Left, Flip	1
2a, 3, 4b, 5-7, 9a, 9b, 10, 12-14, 15	502837	Wheel Lock Assembly, Std Push w/ Extension, PAIR, Flip	1
1c, 2b, 3, 4b, 5-7, 15	502994	Wheel Lock Assembly, Std Pull, Right, Flip	1
1d, 2b, 3, 4b, 5-7, 15	502995	Wheel Lock Assembly, Std Pull, Left, Flip	1

Item Number	Part Number	Description	Quantity
1c, 1d, 2b, 3, 4b, 5-7, 15	502838	Wheel Lock Assembly, Std Pull, PAIR, Flip	1
2b, 3, 4b, 5-7, 9c, 10, 12-14, 15	502996	Wheel Lock Assembly, Std Pull w/ Extension, Right, Flip	1
2b, 3, 4b, 5-7, 9d, 10, 12-14, 15	502997	Wheel Lock Assembly, Std Pull w/ Extension, Left, Flip	1
2b, 3, 4b, 5-7, 9c, 9d, 10, 12-14, 15	502839	Wheel Lock Assembly, Std Pull w/ Extension, PAIR, Flip	1
1a, 2a, 3, 4a, 5-8, 11	503417	Wheel Lock Assembly, Std Push, Right, Flip, 12" Wheels	1
1b, 2a, 3, 4a, 5-8, 11	503418	Wheel Lock Assembly, Std Push, Left, Flip, 12" Wheels	1
1a, 1b, 2a, 3, 4a, 5-8, 11	503419	Wheel Lock Assembly, Std Push, PAIR, Flip, 12" Wheels	1
2a, 3, 4a, 5-8, 9a, 10-14	503420	Wheel Lock Assembly, Std Push w/ Extension, Right, Flip, 12" Wheels	1
2a, 3, 4a, 5-8, 9b, 10-14	503421	Wheel Lock Assembly, Std Push w/ Extension, Left, Flip, 12" Wheels	1
2a, 3, 4a, 5-8, 9a, 9b, 10-14	503422	Wheel Lock Assembly, Std Push w/ Extension, PAIR, Flip, 12" Wheels	1
1c, 2b, 3, 4a, 5-8, 11	503423	Wheel Lock Assembly, Std Pull, Right, Flip, 12" Wheels	1
1d, 2b, 3, 4a, 5-8, 11	503424	Wheel Lock Assembly, Std Pull, Left, Flip, 12" Wheels	1
1c, 1d, 2b, 3, 4a, 5-8, 11	503425	Wheel Lock Assembly, Std Pull, PAIR, Flip, 12" Wheels	1
2b, 3, 4a, 5-8, 9c, 10-14	503426	Wheel Lock Assembly, Std Pull w/ Extension, Right, Flip, 12" Wheels	1
2b, 3, 4a, 5-8, 9d, 10-14	503427	Wheel Lock Assembly, Std Pull w/ Extension, Left, Flip, 12" Wheels	1
2b, 3, 4a, 5-8, 9c, 9d, 10-14	503428	Wheel Lock Assembly, Std Pull w/ Extension, PAIR, Flip, 12" Wheels	1
3	001905	Nut, Slide, Flip	1
4a	102792	M6 x 25, HHCS, Blk Zn <i>Used only with 12" Wheels</i>	2
4b	102794	M6 x 14, HHCS, Blk Zn <i>Used with All Wheels Except 12" Wheels</i>	2
5	101835	M6 Lock Washer Blk Zn	2
6	100658	M6 Nylock Nut Blk Zn	2
7	100746	M6 Flat Washer Blk Zn	2
8	003156	Spacer .7"OD x .315"ID x .485" <i>Used only with 12" Wheels</i>	2
10, 12-14	100854	Brake Extension Replacement	1

Item Number	Part Number	Description	Quantity
11	001019	Bearing Shaft Upper <i>Used only with 12" Wheels</i>	2
12	101117	M5 x 22 BTN SHCS BLK ZC	1
13	101856	Flat Washer, M5, Blk Zn	1
14	100657	M5 Nylock Nut Blk Zn	1
15	002913	Wheel Lock Mount Spacer FLIP <i>Used with All Wheels Except 12" Wheels</i>	1