


Arc Front Side Frame

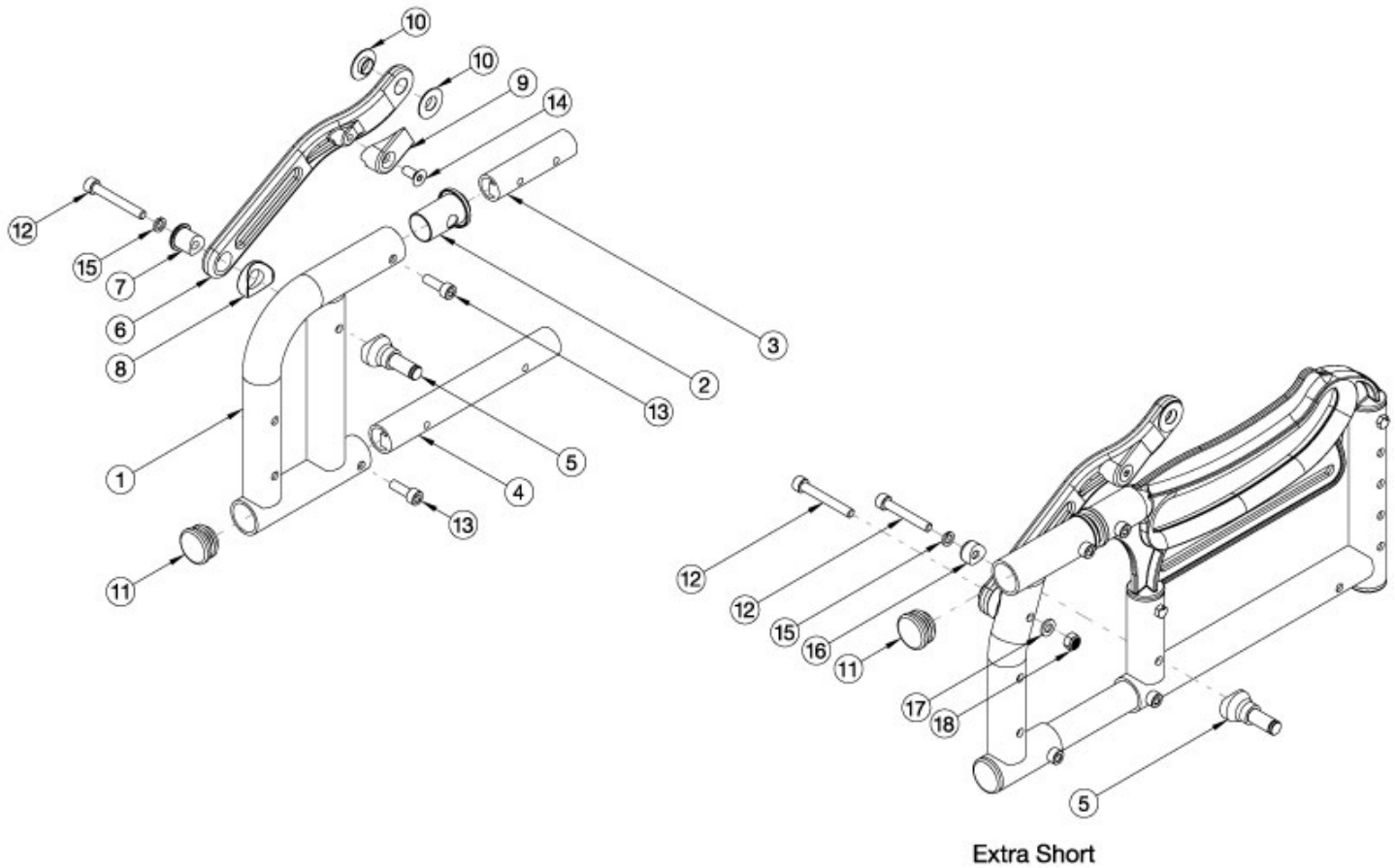

Select color of parts based on Base Frame color of the chair. When changing frame depth ranges, new seat frame assemblies are needed.

Item Number	Part Number	Description	Quantity
1a, 2, 11	113109_ _ _	Front Side Frame, Extra Short, Right, Arc <i>Accommodates 12-15" Depths</i>	1
1b, 2, 11	113110_ _ _	Front Side Frame, Extra Short, Left, Arc <i>Accommodates 12-15" Depths</i>	1
1c, 2, 11	113111_ _ _	Front Side Frame, Short, Right, Arc <i>Accommodates 14-17" Depths</i>	1
1d, 2, 11	113112_ _ _	Front Side Frame, Short, Left, Arc <i>Accommodates 14-17" Depths</i>	1
1e, 2, 11	113113_ _ _	Front Side Frame, Medium, Right, Arc <i>Accommodates 16-19" Depths</i>	1
1f, 2, 11	113114_ _ _	Front Side Frame, Medium, Left, Arc <i>Accommodates 16-19" Depths</i>	1

Item Number	Part Number	Description	Quantity
1g, 2, 11	113115_ _ _	Front Side Frame, Long, Right, Arc <i>Accommodates 18-20" Depths</i>	1
1h, 2, 11	113116_ _ _	Front Side Frame, Long, Left, Arc	1
2	110149	Bushing, Pivot, X-Hinge	1
3	110162	Side Frame Pivot Tube, Upper	1
4	110163	Side Frame Pivot Tube, Lower	1
5	110210	Mount, Mechlock, Lower	1
6	110212	Link Arm	1
7	110214	Link Arm Lower Bushing	1
8	110215	Saddle, Link Arm	1
9	110270	Bumper, Tilt Stop	1
10	110253	Bushing, Flg, .397IDx.87ODx.175Lg	2
11	100804	Tube End Plug 1"	1
12	101472	M6 x 45 SHCS BLK ZC	1
13	101844	M6 x 18 SHCS Blk Zn	2
14	101594	M6 x 14 CLS10.9 DIN7991 FSHCS BLKZC w/Patch	1
15	102810	M6 Lockwasher, Hi-Collar, ZC	1