

VENT HOLDER INSTALLATION KIT - SEDEO ERGO

Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical support at 800-333-4000.

O2 mounting parts:

1. 6 ea Socket head screws


2. 6 ea T-nuts

Tools needed:

- 1. 4mm hex key
- 2. 5mm hex key

A. INSTALL THE MOUNTING BRACKET 1

- 1. Remove the rear shroud and front cushion from the
- 1. Remove the real stroud and front cushion from the backrest. They are attached with Velcro®.
 2. Using the 5mm hex key, remove the 4 screws and washers from the backrest shell, detail 1a. Do not discard.
 3. Insert screws (1) through holes in the mounting bracket as shown in detail 1b. Repeat for opposite side.
 4. Attach the T-nuts (2) to all of the screws, 1b. Leave loose.
 side rails, detail 1c, Until If is approximately 2 inches at the seat. Slide both left and right sides simultaneously.
 Tighten hardware using 4mm hex key.
 Reattach the backrest shell using saved screws and washers. Reattach the rear shroud and front cushion shown in detail 1b. Repeat for opposite side.
 Slide vent (not included) onto vent bracket and secusively.
- 5. Slide the T-nut down into the front channels on the backrest side rails, detail 1c, until it is approximately 2 inches above the seat. Slide both left and right sides simultaneously.
- washers. Reattach the rear shroud and front cushion.
- Slide vent (not included) onto vent bracket and secure with

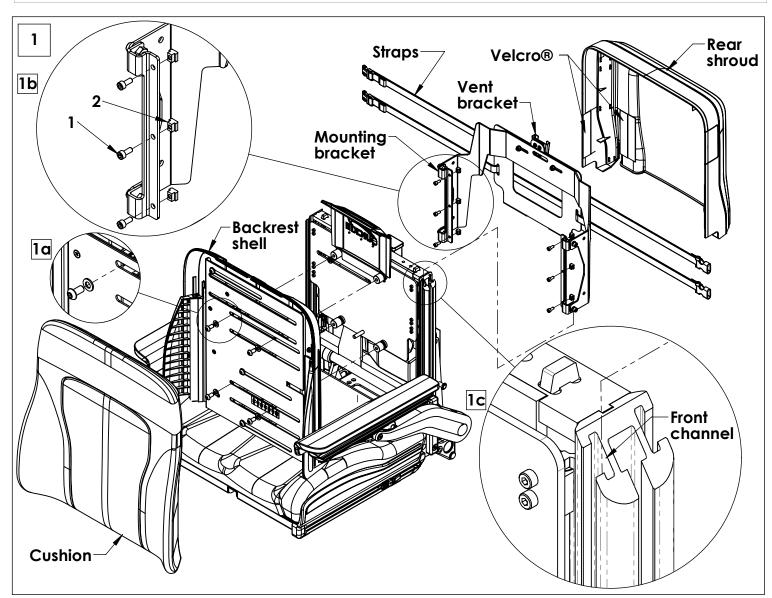

Sunrise Medical 2842 Business Park Ave Fresno, CA 93727 USA In Canada (800) 263-3390



Customer service: 800.333.4000 or visit www.SunriseMedical.com