


Discontinued 2 Rogue XP Tension Adjustable Back Upholstery 17" - 18" Wide

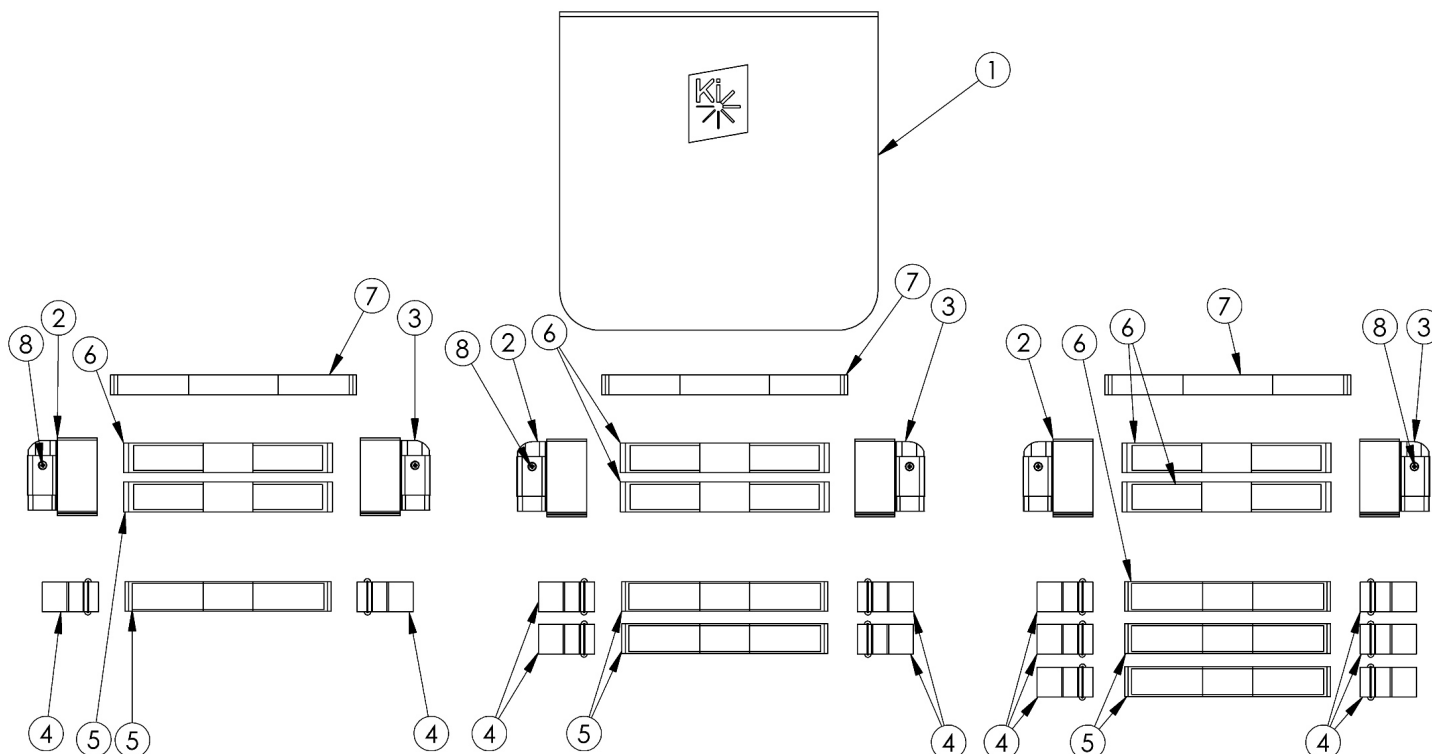

Quantity Chart			
Back Height	Item 6	Item 5	Item 4
10"-13.5"	1	2	2

Quantity Chart			
Back Height	Item 6	Item 5	Item 4
14"-16.5"	2	2	4

Quantity Chart			
Back Height	Item 6	Item 5	Item 4
17"-20"	3	2	6

Open Back Panels are used when integrated push handles are present. Closed Back Panels are when backposts without push handles are present. <p>

Tension Straps (7a-7b) are used with the combination of Fold Down Push Handle and Tension Adjustable Back Upholstery. <p>

Tension Straps 6a, 6b, 7a and 7b are based on chair width.

Item Number	Part Number	Description	Quantity	Retail
1a, 5a	504754	Backrest Cover with Two Straps 17W Rigid, 9-12H Contains 003048 cover	1	\$210.12
1b, 5a	504205	Backrest Cover with Two Straps 17W Rigid, 11-14H Contains 003049 cover	1	\$210.12
1c, 5a	504206	Backrest Cover with Two Straps 17W Rigid, 13-16H Contains 003050 cover	1	\$210.12
1d, 5a	504207	Backrest Cover with Two Straps 17W Rigid, 15-18H Contains 003051 cover	1	\$210.12

Item Number	Part Number	Description	Quantity	Retail
1e, 5a	504208	Backrest Cover with Two Straps 17W Rigid, 17-20H <i>Contains 003052 cover</i>	1	\$210.12
1f, 5b	504755	Backrest Cover with Two Straps 18W Rigid, 9-12H <i>Contains 003054 cover</i>	1	\$210.12
1g, 5b	504209	Backrest Cover with Two Straps 18W Rigid, 11-14H <i>Contains 003055 cover</i>	1	\$210.12
1h, 5b	504210	Backrest Cover with Two Straps 18W Rigid, 13-16H <i>Contains 003056 cover</i>	1	\$210.12
1i, 5b	504211	Backrest Cover with Two Straps 18W Rigid, 15-18H <i>Contains 003057 cover</i>	1	\$210.12
1j, 5b	504212	Backrest Cover with Two Straps 18W Rigid, 17-20H <i>Contains 003058 cover</i>	1	\$210.12
4	100973	Back Upholstery D-Ring	4	\$10.51
5a	002534	Back Upholstery Strap 21.3" <i>Used on 17" chair widths</i>	2	\$12.61
5b	002535	Back Upholstery Strap 23.3" <i>Used on 18" chair widths</i>	2	\$12.61
7a	001751	Tension Strap Folding Handle 16-17 Wide	1	\$21.98
7b	001752	Tension Strap Folding Handle 18-19 Wide	1	\$23.28
8	100734	M5 x 16 PHMS ZC	2	\$1.58