


Focus CR Rotary Frame

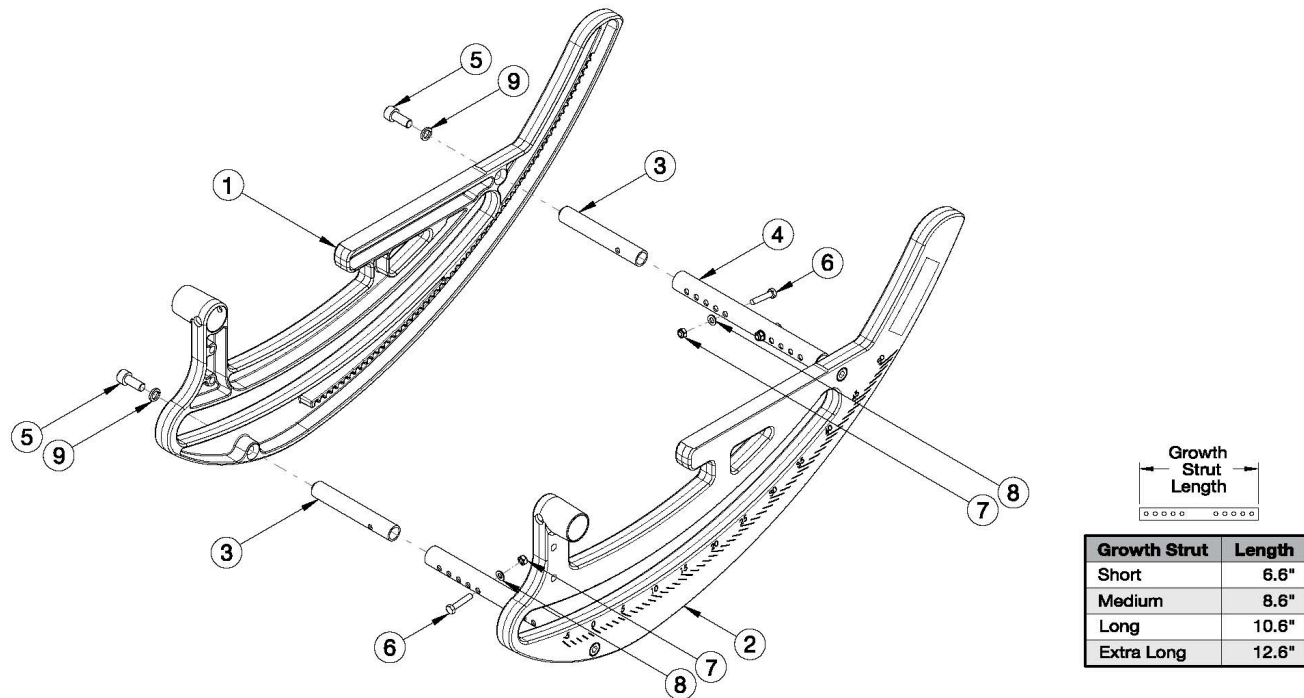

Rotary Frame Assemblies and Growth Struts based on Frame Width Range

Item Number	Part Number	Description	Quantity
1a	505488	Rotary Frame Right, Focus <i>Replacement for 001000INST</i>	1
1b	505301	Rotary Frame with Transit Right <i>Contains Rotary Frame with Transit Bracket and Retrofit Instructions (Transit not shown in image). If wanting to retrofit the Transit option, please see the Focus CR Transit page.</i>	1
2a	505489	Rotary Frame Left, Focus <i>Replacement for 001001INST</i>	1
2b	505302	Rotary Frame with Transit Left <i>Contains Rotary Frame with Transit Bracket and Retrofit Instructions (Transit not shown in image). If wanting to retrofit the Transit option, please see the Focus CR Transit page.</i>	1
1a, 2a, 3, 4a, 5 - 9	501069	Rotary Frame Assembly Small Focus	1
1a, 2a, 3, 4b, 5 - 9	501070	Rotary Frame Assembly Medium Focus	1
1a, 2a, 3, 4c, 5 - 9	501071	Rotary Frame Assembly Large Focus	1
1a, 2a, 3, 4d, 5 - 9	114274	Rotary Frame Assembly X-Large Focus	1

Item Number	Part Number	Description	Quantity
1b, 2b, 3, 4a, 5 - 9	505303	Rotary Frame with Transit Assembly Small Focus <i>If wanting to retrofit the Transit option, please see the Focus CR Transit page to add the Backrest Hardware Kit</i>	1
1b, 2b, 3, 4b, 5 - 9	505304	Rotary Frame with Transit Assembly Medium Focus <i>If wanting to retrofit the Transit option, please see the Focus CR Transit page to add the Backrest Hardware Kit</i>	1
1b, 2b, 3, 4c, 5 - 9	505305	Rotary Frame with Transit Assembly Large Focus <i>If wanting to retrofit the Transit option, please see the Focus CR Transit page to add the Backrest Hardware Kit</i>	1
1b, 2b, 3, 4d, 5 - 9	114275	Rotary Frame with Transit Assembly X-Large Focus <i>If wanting to retrofit the Transit option, please see the Focus CR Transit page to add the Backrest Hardware Kit</i>	1
3	001029	Strut End Rotary Frame TILT	4
4a	001030	Growth Strut Rotary Short TILT	2
4b	001031	Growth Strut Rotary Medium TILT	2
4c	001032	Growth Strut Rotary Long TILT	2
4d	112004	Growth Strut Rotary X-Long TILT	2
5	101850	M8 x 20 SHCS Blk Zn	4
6	101821	M5 x 25 HHCS Blk Zn	4
7	100657	M5 Nylock Nut Blk Zn	4
8	101856	Flat Washer, M5, Blk Zn	4
9	101834	M8 Hi-Collar Lockwasher Blk Zn	4