


(Discontinued) Catalyst Depth Adjustable Backrest with Push Handle

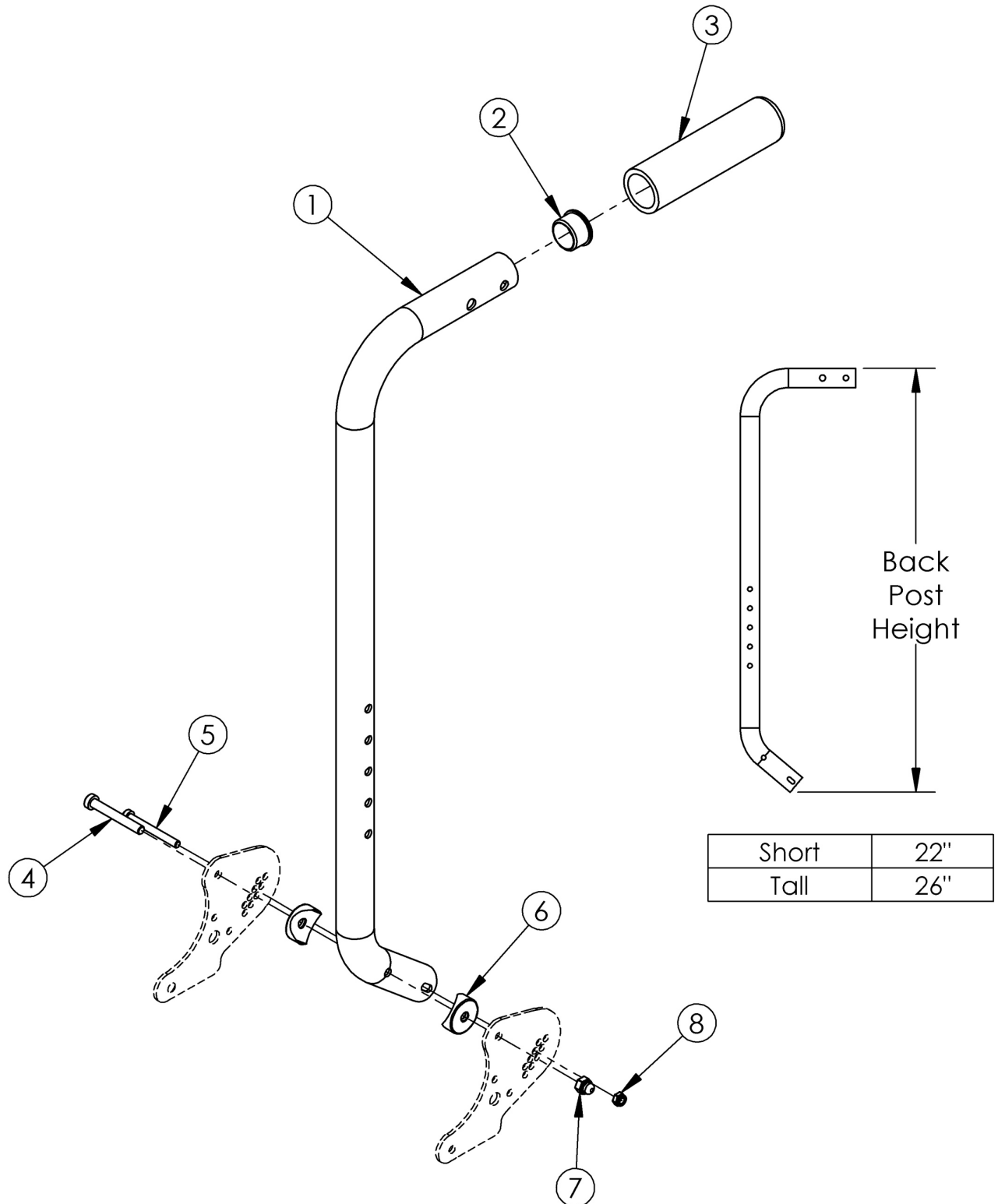

This backrest style discontinued February 8th, 2021

Item Number	Part Number	Description	Quantity
1a - 8	501206	Back Post Depth Adjustable with Push Handle Short Assm with Hardware CAT/ SPARK	1
1b - 8	501207	Back Post Depth Adjustable with Push Handle Tall Assm with Hardware CAT/ SPARK	1
1a, 2, 3	500613	Back Post Depth Adjustable with Push Handle Short Assm CAT/ SPARK	1
1b, 2, 3	500614	Back Post Depth Adjustable with Push Handle Tall Assm CAT/ SPARK	1
4-8	111164	Depth Adjustable Backrest Attaching Hardware Kit	1
4	101123	M6 x 45 Low SHCS BLK ZC	1
5	101186	M5 X 45 CLS12.9 DIN912 SHCS P/THD BLZ	1
6	000055	Saddle 1"	2
7	100558	M6 Nyl Ins Domed Cap L/N Zc	1
8	100657	M5 Nylock Nut Blk Zn	1