

# LANYARD REPLACEMENT INSTRUCTIONS QUICKIE 7R/QUICKIE 5R



Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical support at 800-333-4000.

- 1. 1 ea New lanyard assembly 2. 2 ea Screws 3mm x 16mm
- Tools needed for assembly:
- 1. 2mm hex key

## A. REMOVE OLD LANYARD

- Usinge a 2mm hex key:
   a. Remove the screw (A) from the old lanyard. Figure 1
- b. Repeat on the opposite side.
- c. Discard old lanyard assembly and mounting screws (A).

#### **B. INSTALL NEW LANYARD**

2 2a

#### 1. Using a 2mm hex key:

- a. Slide lanyard housing (B) over pin (C). Ensure that the holes line up. Figure 2
- b. Insert new screw (D) into recessed hole (E) at the bottom of the lanyard housing. Figure 2 & 2a
- c. Tighten securely.
- d. Repeat on the opposite side.

#### C. INSPECT

- 1. Ensure that new screw (D) is in the pin hole (C).
- 2. Pull on lanyard to check that the pins unlock and re-lock smoothly.


## **DEALER/TECHNICIAN WARNING**





### **WARNING**

Attention dealers and qualified technicians, do not operate or service this device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Quickie Technical Service Department before operating and/or servicing the Quickie device. Failure to do so may result in damage and/or injury.

For more information and important warnings, you can find the wheelchair owner's manual at www.Sunrisemedical.com

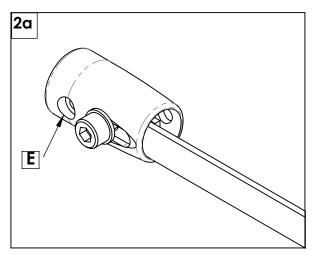

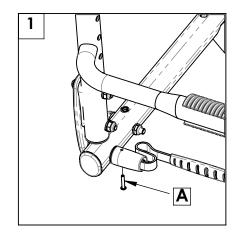

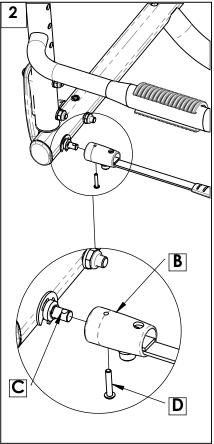

Sunrise Medical 2842 Business Park Ave Fresno, CA 93727 USA In Canada (800) 263-3390



**Customer service: 800.333.4000** or visit www.SunriseMedical.com