


# Clik Wheel Lock Clamps

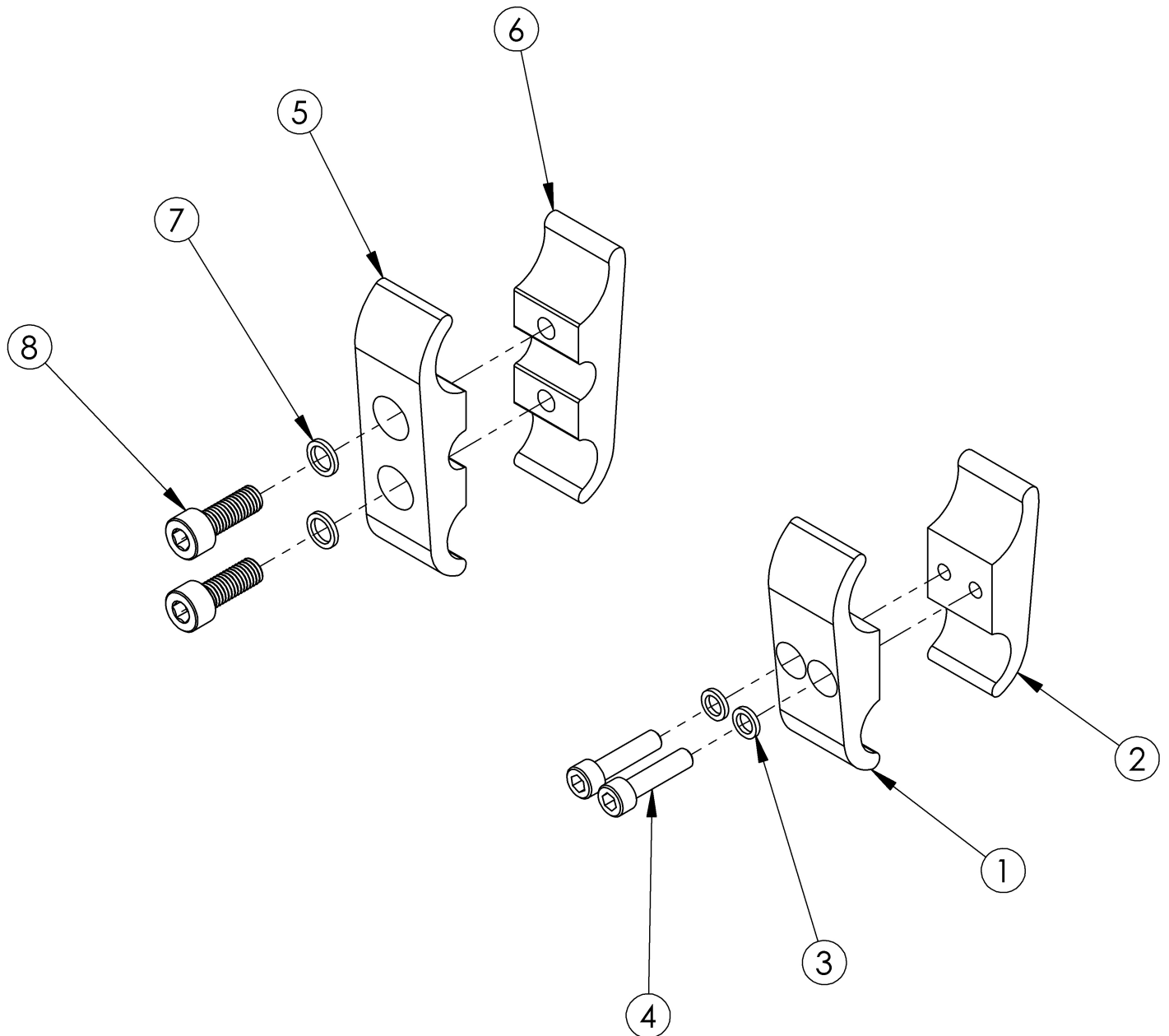

Item Number	Part Number	Description	Quantity	Retail
1a, 2a, 3, 4	502977	Clamp Assembly 1.125" Tubing, CLIK	1	\$52.53
1b, 2b, 3, 4	503460	Clamp Assembly 1.125" Tubing, Long, CLIK <i>Used with Offset Frame</i>	1	\$39.92
5 - 8	502976	Clamp Assembly 1.125" Tubing Extra Long CLIK <i>Used as needed or if Height Adjustable Low T-Arms are present</i>	1	\$40.97
1a	002541	Clamp 1.125 to .75 C'Bored	1	\$15.76

Item Number	Part Number	Description	Quantity	Retail
1b	002831	Wheel Lock Clamp 1.125 to .75 C'Bored Long Clik <i>Used with Offset Frame</i>	1	\$9.46
2a	002542	Clamp 1.125 to .75 Threaded	1	\$15.76
2b	002832	Wheel Lock Clamp 1.125 to .75 Threaded Long Clik <i>Used with Offset Frame</i>	1	\$9.46
3	102721	Washer .250 ID x .375 OD x .060 THK	2	\$6.30
4	100780	M6 x 25 SHCS ZC	2	\$2.10
5	002814	Wheel Lock Clamp 1.125 to .75 C'Bored Extra Long Clik <i>Used as needed or if Height Adjustable Low T-Arms are present</i>	1	\$10.51
6	002815	Wheel Lock Clamp 1.125 to .75 Threaded Extra Long Clik <i>Used as needed or if Height Adjustable Low T-Arms are present</i>	1	\$10.51
7	101701	Washer 3.315/.350 x .425/.525 x .059/.067 18-8 F/W	2	\$2.63
8	102864	M8 x 25mm SHCS, BLK ZC	2	\$1.58