


Rogue XP / Little Wave XP Height Adjustable T-Arm

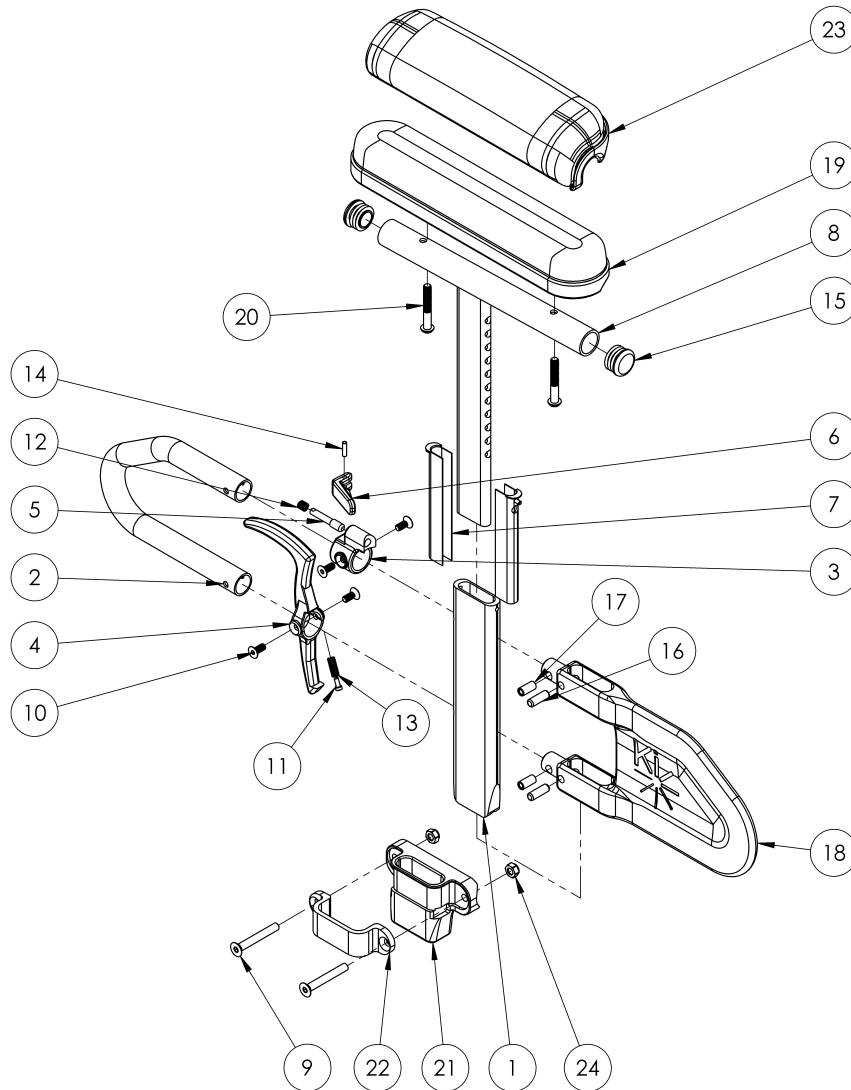

*This design effective December 6, 2017 <p>
Height Adjustable T-Arms cannot be retrofitted*

Item Number	Part Number	Description	Quantity	Retail
1-7, 8a, 10-18, 19b, 20a	505646	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk	1	\$234.78
1-7, 8b, 10-18, 19a, 20a	505644	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full	1	\$234.78
1-7, 8a, 10-18, 20b, 23b	505650	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk	1	\$243.94
1-7, 8b, 10-18, 20b, 23a	505648	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full	1	\$243.94
1-7, 8a, 10-18, 19b, 20a	505647	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, PAIR	1	\$469.57
1-7, 8b, 10-18, 19a, 20a	505645	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, PAIR	1	\$469.57

Item Number	Part Number	Description	Quantity	Retail
1-7, 8a, 10-18, 20b, 23b	505651	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, PAIR	1	\$487.89
1-7, 8b, 10-18, 20b, 23a	505649	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, PAIR	1	\$487.89
1-7, 8a, 9-18, 19b, 21, 20a, 22, 24	506081	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, 1-3/8" Receiver TSU/RG	1	\$262.96
1-7, 8b, 9-18, 19a, 21, 20a, 22, 24	506093	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, 1-3/8" Receiver TSU/RG	1	\$262.96
1-7, 8a, 9-18, 20b, 21, 22, 23b, 24	506094	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, 1-3/8" Receiver TSU/RG	1	\$272.57
1-7, 8b, 9-18, 20b, 21, 22, 23a, 24	506095	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, 1-3/8" Receiver TSU/RG	1	\$272.57
1-7, 8a, 9-18, 19b, 21, 20a, 22, 24	506091	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, 1-3/8" Receiver TSU/RG PAIR	1	\$525.91
1-7, 8b, 9-18, 19a, 21, 20a, 22, 24	506092	Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, 1-3/8" Receiver TSU/RG PAIR	1	\$525.91
1-7, 8a, 9-18, 20b, 21, 22, 23b, 24	506096	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, 1-3/8" Receiver TSU/RG PAIR	1	\$545.16
1-7, 8b, 9-18, 20b, 21, 22, 23a, 24	506097	Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, 1-3/8" Receiver TSU/RG PAIR	1	\$545.16
3, 5, 6, 10, 12, 14	505490	Height Adj T-Arm Upper Latch Assembly	1	\$40.56
4, 10, 11, 13	505507	Height Adj T-Arm Release Lever Assembly	1	\$8.49
9, 21, 22, 24	501731	Height Adj T-Arm Receiver Assembly 1-3/8" TSU/RG	1	\$48.23
16-18	506082	Height Adj T-Arm Short Shield Assembly	1	\$16.44
1-7, 10-14, 16-18	004160	Height Adjustable T-Arm Lower Assembly Short <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1	\$132.94
1-7, 8a, 10-18	003463	Height Adjustable T-Arm Short Desk <i>Does not include arm pad or arm pad screws</i>	1	\$207.98
1-7, 8b, 10-18	003464	Height Adjustable T-Arm Short Full <i>Does not include arm pad or arm pad screws</i>	1	\$207.98
1	000089	Height Adj T Armrest Post	1	\$25.10
2	000095	Height Adj T Armrest Transfer Tube	1	\$28.07
7	000090	Height Adj T Armrest Shim	2	\$2.55
8a, 15	112906	Height Adj T Armrest Upper Desk w/ Tube End Plug	1	\$76.67
8b, 15	112907	Height Adj T Armrest Upper Full w/ Tube End Plug	1	\$76.67

Item Number	Part Number	Description	Quantity	Retail
9	101130	M6 x 55 FSHCS P/THD BLKZC	2	\$2.10
10	003458	M5 X 12mm FHCS BLK ZN W/ Patch	4	\$1.05
11	000152	Height Adj T Armrest Release Spring Button	1	\$1.09
13	000135	Height Adj T Armrest Release Lever Spring	1	\$1.09
15	100803	Tube End Plug 7/8"	2	\$1.58
16	003641	.25" x .75" Dowel Pin	2	\$1.05
17	003457	M5 x 14mm Barrel Nut	2	\$1.05
19a	100500	Arm Pad Full	1	\$23.64
19b	100501	Arm Pad Desk	1	\$23.64
20a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2	\$1.58
20b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2	\$1.58
21	004578	Height Adjustable T-Arm Receiver 1.375"	1	\$15.25
22	000088	Height Adj T Armrest Bracket	1	\$27.42
23a	102452	Waterfall Arm Pad Full	1	\$32.80
23b	102521	Waterfall Arm Pad Desk	1	\$32.80
24	100658	M6 Nylock Nut Blk Zn	2	\$1.05