


# Focus CR Caster Forks and Stems

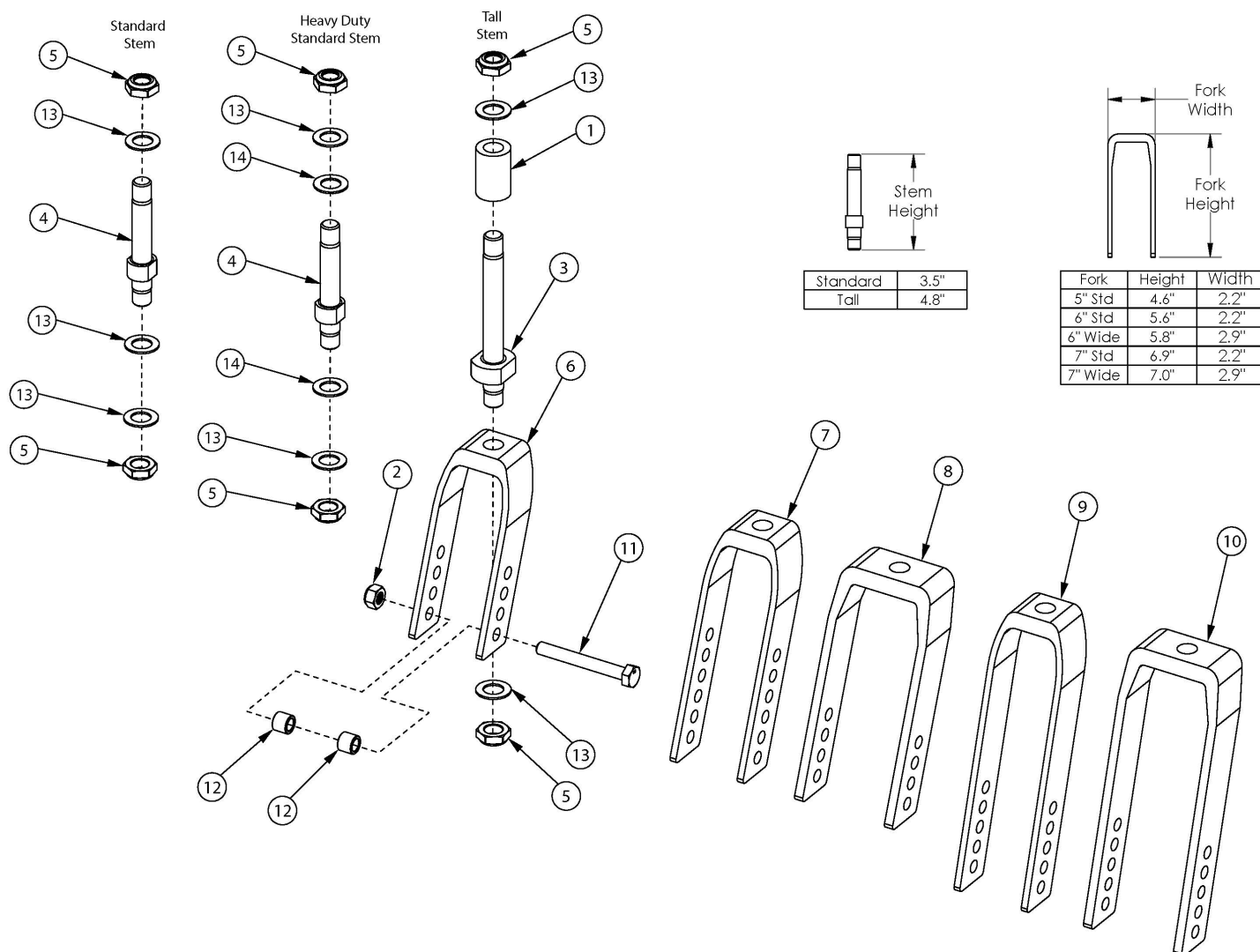

If changing to 2" casters, wide forks and hardware needed.

Item Number	Part Number	Description	Quantity
4a, 5, 6, 13	503816	Fork Assembly 5" Std Stem, Tilt <i>Used with standard weight capacity</i>	1
4a, 5, 7, 13	503814	Fork Assembly 6" Std Stem, Tilt <i>Used with standard weight capacity</i>	1
4a, 5, 8, 13	503817	Fork Assembly 6" Wide Std Stem, Tilt <i>Used with standard weight capacity. Only used with 2" wide casters.</i>	1
4a, 5, 9, 13	503815	Fork Assembly 7" Std Stem, Tilt <i>Used with standard weight capacity</i>	1
4a, 5, 10, 13	503818	Fork Assembly 7" Wide Std Stem, Tilt	1

Item Number	Part Number	Description	Quantity
		<i>Used with standard weight capacity. Only used with 2" wide casters.</i>	
1, 3, 5, 6, 13	506369	Fork Assembly 5" Tall Stem Tilt <i>Used with standard and heavy duty weight capacity</i>	1
1, 3, 5, 7, 13	506370	Fork Assembly 6" Tall Stem Tilt <i>Used with standard and heavy duty weight capacity</i>	1
1, 3, 5, 8, 13	506371	Fork Assembly 6" Wide Tall Stem Tilt <i>Used with standard and heavy duty weight capacity. Only used with 2" wide casters.</i>	1
1, 3, 5, 9, 13	506372	Fork Assembly 7" Tall Stem Tilt <i>Used with standard and heavy duty weight capacity.</i>	1
1, 3, 5, 10, 13	506373	Fork Assembly 7" Wide Tall Stem Tilt <i>Used with standard and heavy duty weight capacity. Only used with 2" wide casters.</i>	1
2, 11a, 12a	502653	Caster Attachment Hardware Kit CAT/FCS	1
2, 11b, 12b	502654	Caster Attachment Hardware Kit for 6" and 7" Wide Fork, CAT/FCS <i>Only used with 2" wide casters</i>	1
1	002476	Spacer, 3/4" OD x 1/2" ID x 1.23	1
2	100659	M8 Nylock Nut Blk Zn	1
1, 3, 5, 13	506374	Caster Stem Tall, Adult Tilt <i>Used with standard and heavy duty weight capacity</i>	1
4a, 5, 13	503819	Caster Stem Standard, Tilt <i>Used with standard weight capacity</i>	1
5	100529	M12 x 1.25mm Lock Nut Black	2
5, 6	503823	Caster Fork 5" Tilt	1
5, 7	503824	Caster Fork 6" Tilt	1
5, 8	503825	Caster Fork 6" Wide Tilt <i>Only used with 2" wide casters</i>	1
5, 9	503826	Caster Fork 7" Tilt	1
5, 10	503827	Caster Fork 7" Wide Tilt <i>Only used with 2" wide casters</i>	1
11a	101829	M8 x 65 HHCS Blk Zn	1
11b	101831	M8 x 80 HHCS Blk Zn <i>Only used with 2" wide casters</i>	1
12a	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer	2

Item Number	Part Number	Description	Quantity
		<i>Used with all casters except 6x2 and 8x2</i>	
12b	001246	Caster Spacer Black 6 x 2 Poly .472ODX.319IDX.370LG <i>Only used with 6" x 2" casters</i>	2
13	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	3