

The Lip Kit allows a therapist or supplier to ensure the desired placement of a cushion on an Adjustable Solid Seat. It is especially valuable for the JAY[®] GS Seating System for children.


ADJUSTABLE SOLID SEAT LIP KIT M-285


INSTALLING THE FRONT LIP

- 1. Using a 7/32 (.218") drill bit, drill a hole 1/2" from the middle of the front edge of the Solid Seat. See diagram.
- 2. Using the same drill bit, drill a second hole I" behind the first hole.
- 3. Place one insert into the front hole with the hex facing up and screw it into the hole using the allen wrench (provided). Turn the insert until it is flush with the surface of the Solid Seat. Repeat the operation for the second hole.
- 4. Place the Front Lip over the two holes and attach to the Solid Seat using two of the enclosed screws and washers. The tall lip goes in front and the short lip in the back. Adjust the lip to the desired position and tighten down both screws.

INSTALLING THE BACK LIP

- I. Determine the desired location of the Back Lip.
- 2. Drill two holes I" apart using the 7/32 (.218") drill bit.
- 3. Place one insert into each hole with the hex facing up and screw it into the hole using the allen wrench (provided). Screw the insert until it is flush with the surface of the Solid Seat. Repeat the operation for the second hole.
- 4. Place the Back Lip over the two holes and attach to the Solid Seat using the enclosed screws and washers. Adjust the lip to the desired position and tighten down both screws. For more range of adjustment, remove one screw and spin the lip 180° and re-insert screw.

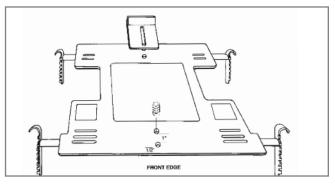

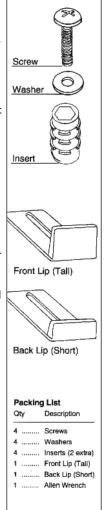

Sunrise Medical • 2842 Business Park Ave. • Fresno, CA 93727 • USA In Canada (800) 263-3390



Customer Service: 800.333.4000 www.SunriseMedical.com