


Arc Foot Tilt Mechanism

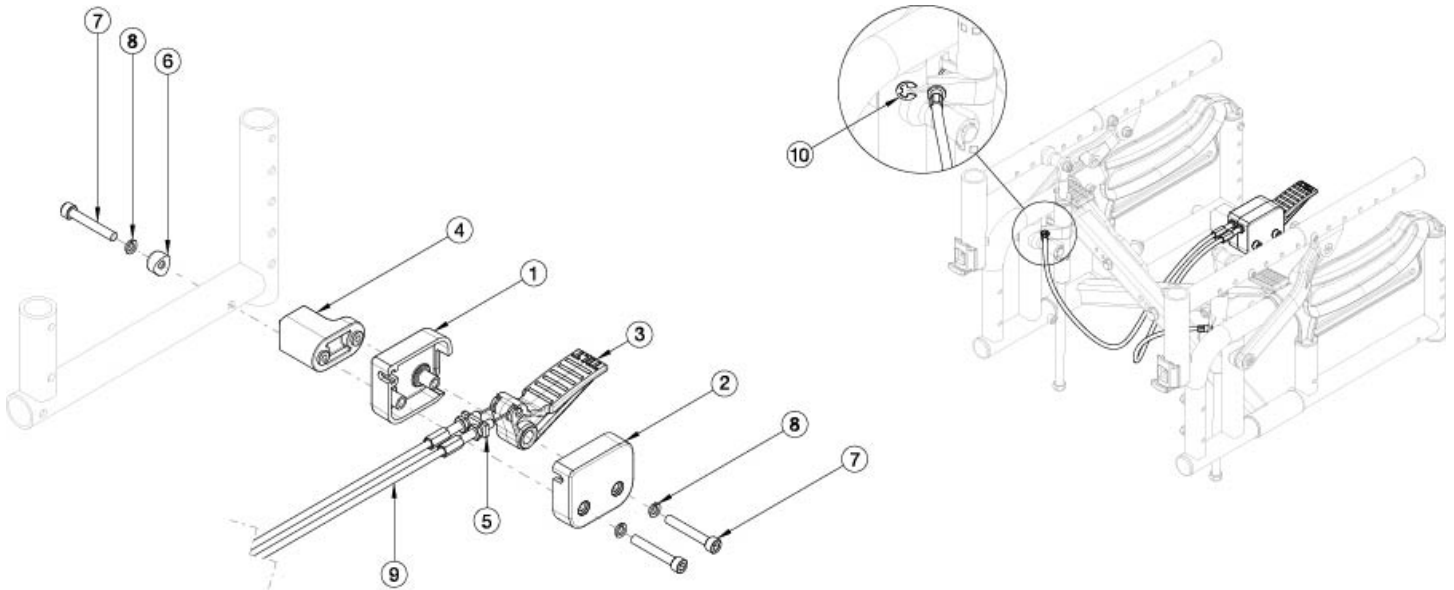

Cables (Item 9)					
Frame Style	Seat Width	Extra Short (12-15D)		Short, Medium, Long	
		Same Side	Opposite Side	Same Side	Opposite Side
Standard Frame	Small, 10-12"	113027	113028	113029	113029
	Medium, 12-14"	113027	113029	113029	113030
	Large, 14-16"	113027	113029	113029	113030
	Extra Large, 16-18"	113027	113030	113029	113031
Extended Frame	Small, 10-12"	113028	113029	113030	113030
	Medium, 12-14"	113028	113030	113030	113031
	Large, 14-16"	113028	113031	113030	113031
	Extra Large, 16-18"	113028	113030	113030	113031

Please reference the table above to determine which cables you need for your configuration. Cable kits contain one cable and one e-ring.

Item Number	Part Number	Description	Quantity
1-8	113026	Foot Tilt Actuator Assembly	1
9a, 10	113027	Cable, Tilt, Foot, Short, Arc	1
9b, 10	113028	Cable, Tilt, Foot, Medium, Arc	1
9c, 10	113029	Cable, Tilt, Foot, Long, Arc	1
9d, 10	113030	Cable, Tilt, Foot, XL, Arc	1
9e, 10	113031	Cable, Tilt, Foot, XXL, Arc	1
10	102822	E-Ring, M6, YZ	2