


Arc Caster Forks and Stems

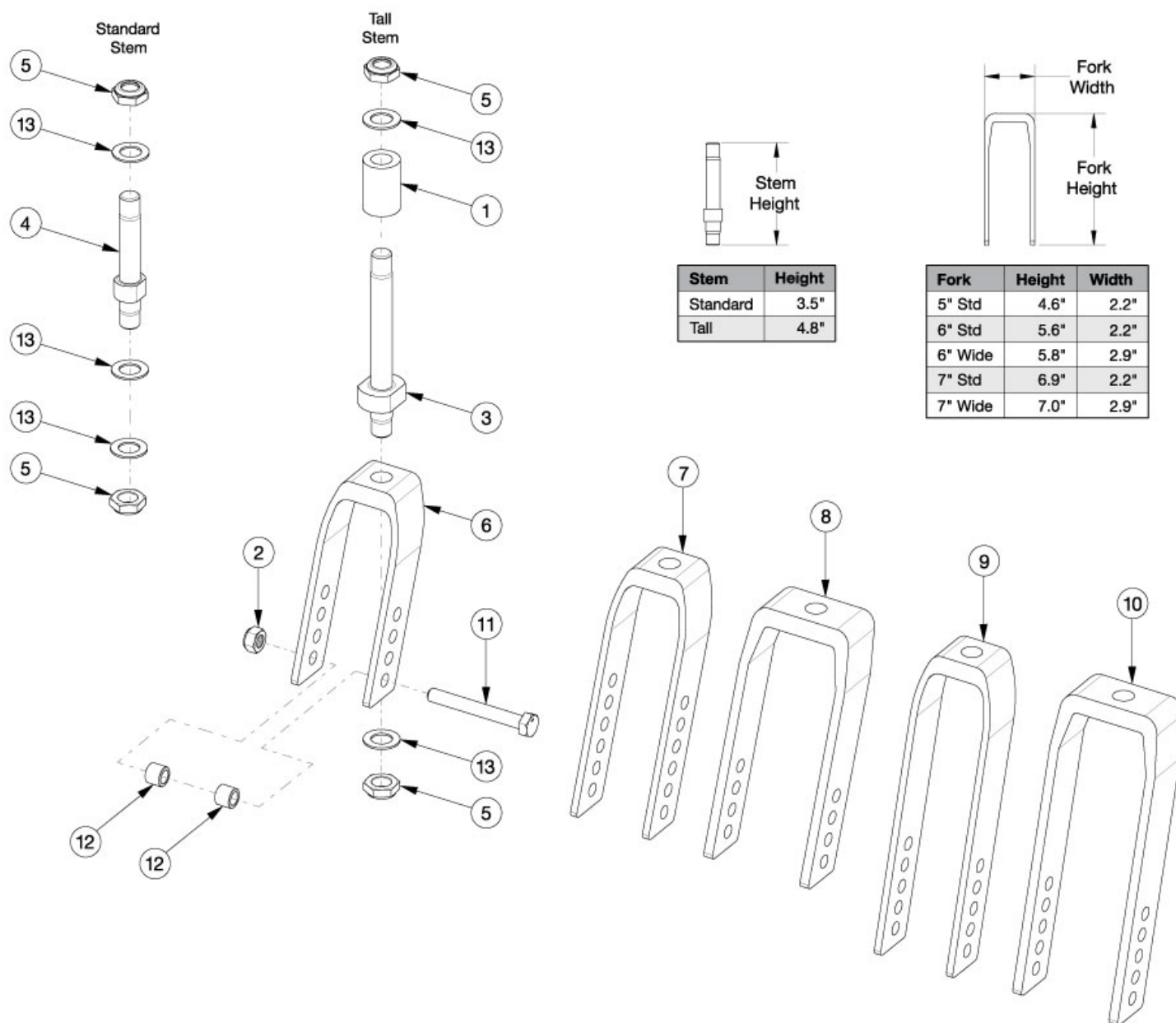

If changing to 2" casters, wide forks and hardware needed.

Item Number	Part Number	Description	Quantity
4-6, 13	113773	5" Fork Assembly, Standard Stem, Arc	1
4, 5, 7, 13	113774	6" Fork Assembly, Standard Stem, Arc	1
4, 5, 8, 13	113775	6" Wide Fork Assembly, Standard Stem, Arc <i>Used with 6 x 2" Poly caster</i>	1
4, 5, 9, 13	113776	7" Fork Assembly, Standard Stem, Arc	1

Item Number	Part Number	Description	Quantity
4, 5, 10, 13	113777	7" Wide Fork Assembly, Standard Stem, Arc <i>Used with 6 x 2" Poly caster</i>	1
1, 3, 5, 6, 13	113778	5" Fork Assembly, Tall Stem, Arc	1
1, 3, 5, 7, 13	113779	6" Fork Assembly, Tall Stem, Arc	1
1, 3, 5, 8, 13	113780	6" Wide Fork Assembly, Tall Stem, Arc <i>Used with 6 x 2" Poly caster</i>	1
1, 3, 5, 9, 13	113781	7" Fork Assembly, Tall Stem, Arc	1
1, 3, 5, 10, 13	113782	7" Wide Fork Assembly, Tall Stem, Arc <i>Used with 6 x 2" Poly caster</i>	1
1	002476	Spacer, 3/4" OD x 1/2" ID x 1.23	1
2	100659	M8 Nylock Nut Blk Zn	1
4, 5, 13	113783	Caster Stem, Standard, Arc	1
1, 3, 5, 13	113784	Caster Stem, Tall, Arc	1
5	100529	M12 x 1.25mm Lock Nut Black	2
5, 6	113785	5" Caster Fork, Arc	1
5, 7	113786	6" Caster Fork, Arc	1
5, 8	113787	6" Wide Caster Fork, Arc <i>Used with 6 x 2" Poly caster</i>	1
5, 9	113788	7" Caster Fork, Arc	1
5, 10	113789	7" Wide Caster Fork, Arc <i>Used with 6 x 2" Poly caster</i>	1
11a	101829	M8 x 65 HHCS Blk Zn	1
11b	101831	M8 x 80 HHCS Blk Zn <i>Used with 6 x 2" Poly caster</i>	1
12a	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer	2
12b	001246	Caster Spacer Black 6 x 2 Poly .472ODX.319IDX.370LG <i>Used with 6 x 2" Poly caster</i>	2
13	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	3
2, 11a, 12a	111379	Caster Attaching Hardware Kit <i>Used with all casters except the 6 x 2" Poly caster</i>	1
2, 11b, 12b	111380	Caster Attaching Hardware Kit for Wide Forks <i>Used with 6 x 2" Poly caster</i>	1