


Clik / Rogue XP Swing Away Armrest

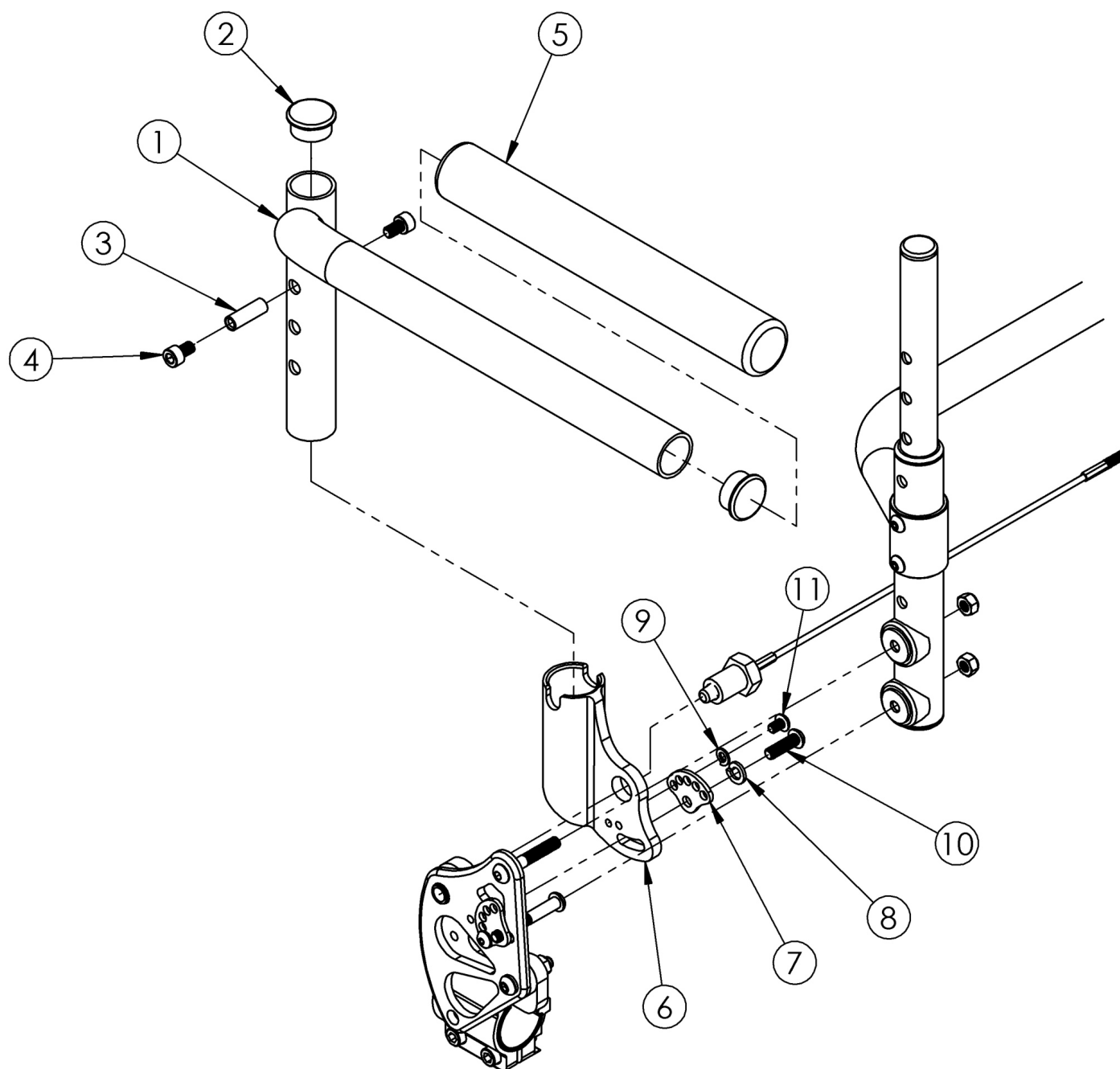

Swing Away Arm	Height Range
Short	7"-9"
Tall	9.25"-11.25"

Item Number	Part Number	Description	Quantity	Retail
1a, 2-11	502492	Swing Away Short Armrest w/ Receiver Assembly Right	1	\$110.89
1b, 2-11	502493	Swing Away Short Armrest w/ Receiver Assembly Left	1	\$110.89
1a, 1b, 2-11	502494	Swing Away Short Armrest w/ Receiver Assembly PAIR	1	\$221.78
1c, 2-11	502495	Swing Away Tall Armrest w/ Receiver Assembly Right	1	\$113.04
1d, 2-11	502496	Swing Away Tall Armrest w/ Receiver Assembly Left	1	\$113.04
1c, 1d, 2-11	502497	Swing Away Tall Armrest w/ Receiver Assembly PAIR	1	\$226.07
1a, 2 - 5	502485	Swing Away Short Armrest Assembly Right	1	\$53.17
1b, 2 - 5	502486	Swing Away Short Armrest Assembly Left	1	\$53.17
1a, 1b, 2 - 5	502487	Swing Away Short Armrest Assembly PAIR	1	\$106.35
1c, 2 - 5	502488	Swing Away Tall Armrest Assembly Right	1	\$55.31
1d, 2 - 5	502489	Swing Away Tall Armrest Assembly Left	1	\$55.31
1c, 1d, 2 - 5	502490	Swing Away Tall Armrest Assembly PAIR	1	\$110.64
6-11	502491	Swing Away Armrest Receiver Assembly One Side	1	\$50.03
2	100804	Tube End Plug 1"	2	\$1.58
3	000065	M6 x 23.68 Threaded Barrel	1	\$4.13
4	101179	M6 x 10 SHCS BLK ZC w/Patch	2	\$2.10
6	001769	Arm Receiver LW	1	\$42.88
7	001762	Arc Adjustment Cam	1	\$3.98
8	101835	M6 Lock Washer Blk Zn	1	\$0.53
9	102614	M5 Split Lock Washer BLK ZC	1	\$0.53
10	102623	M6 x 20 BHCS BLK ZC	1	\$1.05
11	102621	M5 x 8 BHCS BLK ZC	1	\$0.53