


# Clik 1" Offset Backrest Mount

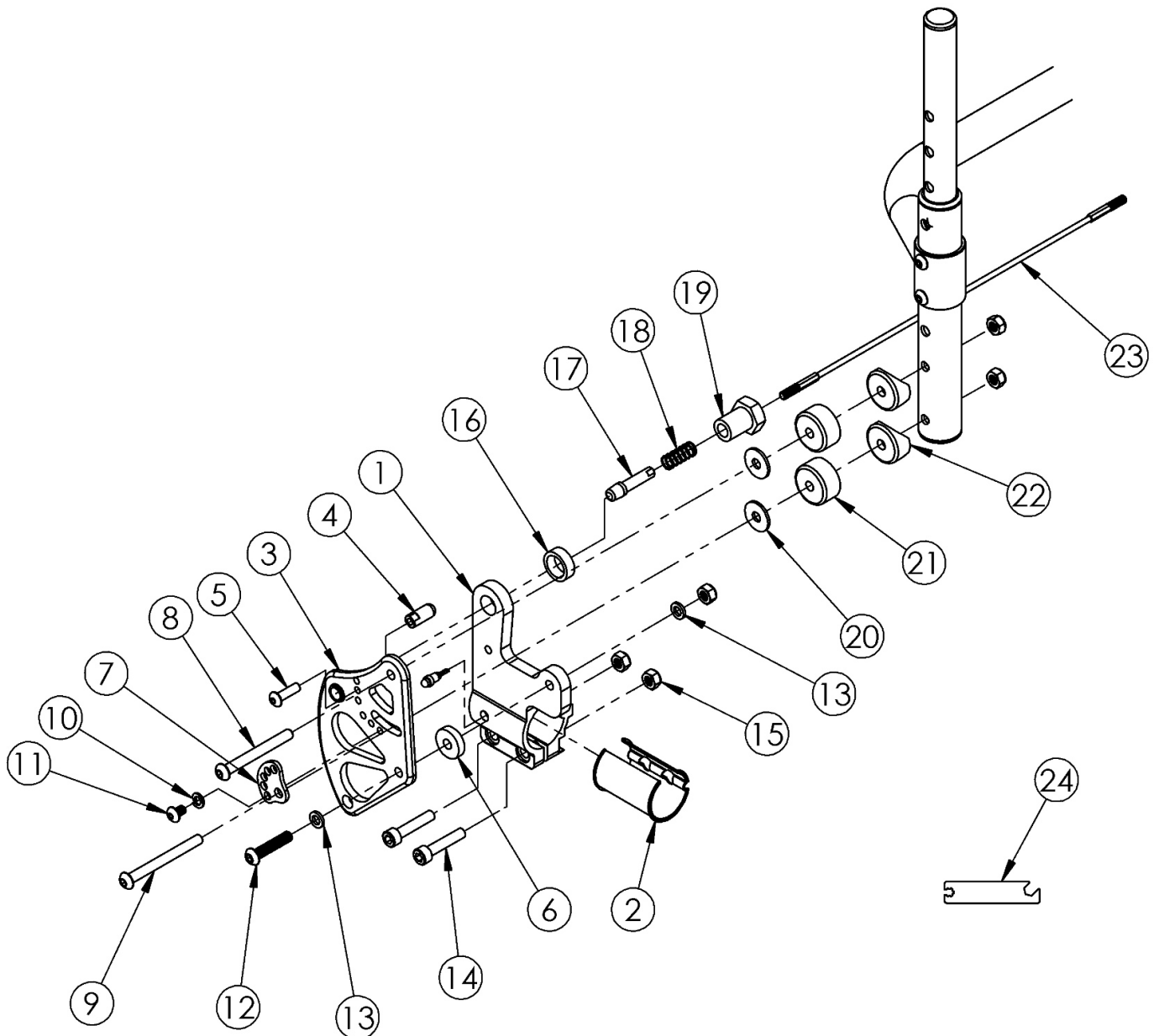

1" Offset Backrest Mount used with Height Adjustable Back Posts

<p>

Spinergy Back Release Cable based on Seat Width

Item Number	Part Number	Description	Quantity	Retail
1a, 2, 3a, 4-7, 8-22	502436	1" Inward Offset Backrest Mount Assembly Right Black CLIK	1	\$150.05
1a, 2, 3b, 4-7, 8-22	502436_---	1" Inward Offset Backrest Mount Assembly, Right, Color Anodized, CLIK	1	\$191.68
1b, 2, 3c, 4-7, 8-22	502437	1" Inward Offset Backrest Mount Assembly Left Black CLIK	1	\$150.05

Item Number	Part Number	Description	Quantity	Retail
1b, 2, 3d, 4-7, 8-22	502437_ _ _	1" Inward Offset Backrest Mount Assembly, Left, Color Anodized, CLIK	1	\$191.68
1a, 1b, 2, 3a, 3c, 4-7, 8-22	502438	1" Inward Offset Backrest Mount Assembly PAIR Black CLIK	1	\$300.10
1a, 1b, 2, 3b, 3d, 4-7, 8-22	502438_ _ _	1" Inward Offset Backrest Mount Assembly, PAIR, Color Anodized, CLIK	1	\$383.37
1a	001783	Back Mount Right LW	1	\$47.38
1b	001784	Back Mount Left LW	1	\$47.38
2	001785	Back Sleeve	2	\$1.46
3a	003229	Back Bracket Relaxed Assembly Right, Black Anodized	1	\$63.04
3b	002770_ _ _	Back Bracket Relaxed Right Color Anodized	1	\$104.67
3c	003231	Back Bracket Relaxed Assembly Left, Black Anodized	1	\$63.04
3d	002771_ _ _	Back Bracket Relaxed Left Color Anodized	1	\$104.67
4	000922	Backrest Stop RG	1	\$5.01
5	101716	M5 x 16 SHCS Blk Zn	1	\$1.05
6	102630	Washer Nylon 25 X 75 X .187 Black	1	\$1.05
7	001762	Arc Adjustment Cam	1	\$3.98
8	003232	M6 x 55 BHCS Blk Zc	1	\$8.94
9	003241	M6 x 60 BHCS P/THD Custom Blk Zc	1	\$11.56
10	102614	M5 Split Lock Washer BLK ZC	1	\$0.53
11	102621	M5 x 8 BHCS BLK ZC	1	\$0.53
12	102625	M6 X 30 BHCS BLK ZC	1	\$1.58
13	100746	M6 Flat Washer Blk Zn	2	\$1.58
14	102627	M6 X 30 SHCS BLK ZC	2	\$1.05
15	100658	M6 Nylock Nut Blk Zn	5	\$1.05
16	001770	Spacer For Pin Receiver <i>This spacer is not used if Swing Away Armrests are present</i>	1	\$3.19
17	000909	Backrest Lockout Pin RG	1	\$15.63
18	000965	Backrest Cable Release Spring RG	1	\$1.79
19	001768	Pin Receiver with Hex	1	\$5.47
20	102629	Seastrom 5610-69-062 Washer Nylon Black	2	\$1.05

Item Number	Part Number	Description	Quantity	Retail
21	002475	Offset Backrest Spacer	2	\$7.75
22	000055	Saddle 1"	2	\$1.58
23a	102556	Spinergy Back Release Cable 8" Wide Black	1	\$31.52
23b	102557	Spinergy Back Release Cable 8" Wide Blue	1	\$31.52
23c	102558	Spinergy Back Release Cable 8" Wide Red	1	\$31.52
23d	102559	Spinergy Back Release Cable 8" Wide Yellow	1	\$31.52
23e	102560	Spinergy Back Release Cable 8" Wide Green	1	\$31.52
23f	102561	Spinergy Back Release Cable 8" Wide Pink	1	\$31.52
23g	102562	Spinergy Back Release Cable 8" Wide Orange	1	\$31.52
23h	102563	Spinergy Back Release Cable 8" Wide White	1	\$31.52
23i	102564	Spinergy Back Release Cable 9" Wide Black	1	\$31.52
23j	102565	Spinergy Back Release Cable 9" Wide Blue	1	\$31.52
23k	102566	Spinergy Back Release Cable 9" Wide Red	1	\$31.52
23l	102567	Spinergy Back Release Cable 9" Wide Yellow	1	\$31.52
23m	102568	Spinergy Back Release Cable 9" Wide Green	1	\$31.52
23n	102569	Spinergy Back Release Cable 9" Wide Pink	1	\$31.52
23o	102570	Spinergy Back Release Cable 9" Wide Orange	1	\$31.52
23p	102571	Spinergy Back Release Cable 9" Wide White	1	\$31.52
23q	102572	Spinergy Back Release Cable 10" Wide Black	1	\$31.52
23r	102573	Spinergy Back Release Cable 10" Wide Blue	1	\$31.52
23s	102574	Spinergy Back Release Cable 10" Wide Red	1	\$31.52
23t	102575	Spinergy Back Release Cable 10" Wide Yellow	1	\$31.52
23u	102576	Spinergy Back Release Cable 10" Wide Green	1	\$31.52
23v	102577	Spinergy Back Release Cable 10" Wide Pink	1	\$31.52
23w	102578	Spinergy Back Release Cable 10" Wide Orange	1	\$31.52
23x	102579	Spinergy Back Release Cable 10" Wide White	1	\$31.52
23y	102580	Spinergy Back Release Cable 11" Wide Black	1	\$31.52
23z	102581	Spinergy Back Release Cable 11" Wide Blue	1	\$31.52

Item Number	Part Number	Description	Quantity	Retail
23aa	102582	Spinergy Back Release Cable 11" Wide Red	1	\$31.52
23ab	102583	Spinergy Back Release Cable 11" Wide Yellow	1	\$31.52
23ac	102584	Spinergy Back Release Cable 11" Wide Green	1	\$31.52
23ad	102585	Spinergy Back Release Cable 11" Wide Pink	1	\$31.52
23ae	102586	Spinergy Back Release Cable 11" Wide Orange	1	\$31.52
23af	102587	Spinergy Back Release Cable 11" Wide White	1	\$31.52
23ag	200260	Spinergy Back Release Cable 12" Wide Black	1	\$31.52
23ah	200261	Spinergy Back Release Cable 12" Wide Blue	1	\$31.52
23ai	200262	Spinergy Back Release Cable 12" Wide Red	1	\$31.52
23aj	200263	Spinergy Back Release Cable 12" Wide Yellow	1	\$31.52
23ak	200264	Spinergy Back Release Cable 12" Wide Green	1	\$31.52
23al	200265	Spinergy Back Release Cable 12" Wide Pink	1	\$31.52
23am	200266	Spinergy Back Release Cable 12" Wide Orange	1	\$31.52
23an	200267	Spinergy Back Release Cable 12" Wide White	1	\$31.52
23ao	200268	Spinergy Back Release Cable 13" Wide Black	1	\$31.52
23ap	200269	Spinergy Back Release Cable 13" Wide Blue	1	\$31.52
23aq	200270	Spinergy Back Release Cable 13" Wide Red	1	\$31.52
23ar	200271	Spinergy Back Release Cable 13" Wide Yellow	1	\$31.52
23as	200272	Spinergy Back Release Cable 13" Wide Green	1	\$31.52
23at	200273	Spinergy Back Release Cable 13" Wide Pink	1	\$31.52
23au	200274	Spinergy Back Release Cable 13" Wide Orange	1	\$31.52
23av	200275	Spinergy Back Release Cable 13" Wide White	1	\$31.52
23aw	200276	Spinergy Back Release Cable 14" Wide Black	1	\$31.52
23ax	200277	Spinergy Back Release Cable 14" Wide Blue	1	\$31.52
23ay	200278	Spinergy Back Release Cable 14" Wide Red	1	\$31.52
23az	200279	Spinergy Back Release Cable 14" Wide Yellow	1	\$31.52
23ba	200280	Spinergy Back Release Cable 14" Wide Green	1	\$31.52
23bb	200281	Spinergy Back Release Cable 14" Wide Pink	1	\$31.52

Item Number	Part Number	Description	Quantity	Retail
23bc	200282	Spinergy Back Release Cable 14" Wide Orange	1	\$31.52
23bd	200283	Spinergy Back Release Cable 14" Wide White	1	\$31.52
23be	200284	Spinergy Back Release Cable 15" Wide Black	1	\$31.52
23bf	200285	Spinergy Back Release Cable 15" Wide Blue	1	\$31.52
23bg	200286	Spinergy Back Release Cable 15" Wide Red	1	\$31.52
23bh	200287	Spinergy Back Release Cable 15" Wide Yellow	1	\$31.52
23bi	200288	Spinergy Back Release Cable 15" Wide Green	1	\$31.52
23bj	200289	Spinergy Back Release Cable 15" Wide Pink	1	\$31.52
23bk	200290	Spinergy Back Release Cable 15" Wide Orange	1	\$31.52
23bl	200291	Spinergy Back Release Cable 15" Wide White	1	\$31.52
23bm	200292	Spinergy Back Release Cable 16" Wide Black	1	\$31.52
23bn	200293	Spinergy Back Release Cable 16" Wide Blue	1	\$31.52
23bo	200294	Spinergy Back Release Cable 16" Wide Red	1	\$31.52
23bp	200295	Spinergy Back Release Cable 16" Wide Yellow	1	\$31.52
23bq	200296	Spinergy Back Release Cable 16" Wide Green	1	\$31.52
23br	200297	Spinergy Back Release Cable 16" Wide Pink	1	\$31.52
23bs	200298	Spinergy Back Release Cable 16" Wide Orange	1	\$31.52
23bt	200299	Spinergy Back Release Cable 16" Wide White	1	\$31.52
23bu	200300	Spinergy Back Release Cable 17" Wide Black	1	\$31.52
23bv	200301	Spinergy Back Release Cable 17" Wide Blue	1	\$31.52
23bw	200302	Spinergy Back Release Cable 17" Wide Red	1	\$31.52
23bx	200303	Spinergy Back Release Cable 17" Wide Yellow	1	\$31.52
23by	200304	Spinergy Back Release Cable 17" Wide Green	1	\$31.52
23bz	200305	Spinergy Back Release Cable 17" Wide Pink	1	\$31.52
23ca	200306	Spinergy Back Release Cable 17" Wide Orange	1	\$31.52
23cb	200307	Spinergy Back Release Cable 17" Wide White	1	\$31.52
23cc	200308	Spinergy Back Release Cable 18" Wide Black	1	\$31.52
23cd	200309	Spinergy Back Release Cable 18" Wide Blue	1	\$31.52

Item Number	Part Number	Description	Quantity	Retail
23ce	200310	Spinergy Back Release Cable 18" Wide Red	1	\$31.52
23cf	200311	Spinergy Back Release Cable 18" Wide Yellow	1	\$31.52
23cg	200312	Spinergy Back Release Cable 18" Wide Green	1	\$31.52
23ch	200313	Spinergy Back Release Cable 18" Wide Pink	1	\$31.52
23ci	200314	Spinergy Back Release Cable 18" Wide Orange	1	\$31.52
23cj	200315	Spinergy Back Release Cable 18" Wide White	1	\$31.52
24	101991	Spinergy Release Cable/SPOX Adjustment Tool <i>No charge when ordered with Spinergy Release Cable</i>	1	

# Color Anodizing Samples

