


Rigid Stroller Handle

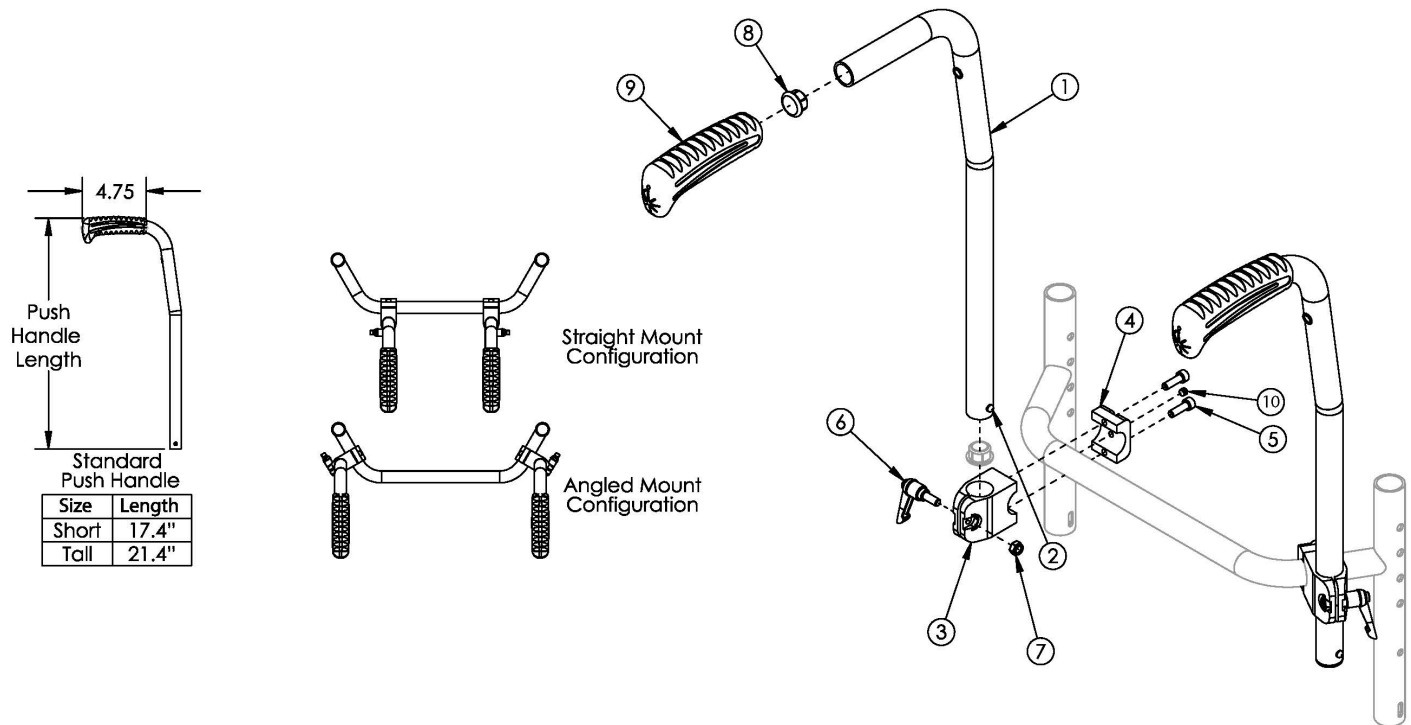

If changing from another style push handle and the Drum Brake is present, a new Drum Brake Cable Kit may be required

Item Number	Part Number	Description	Quantity
1a, 2 - 10	500700	Stroller Handle Short Assembly Each, Rigid <i>Used on chairs with 15" or greater rear seat height and drum brake is not present.</i>	1
1b, 2 - 10	500748	Stroller Handle Tall Assembly Each, Rigid <i>Used on chairs with less than 15" rear seat height and drum brake is not present.</i>	1
1c, 2 - 10	113242	Stroller Handle Short Assembly with Backcane Extension, Drum Brake, Each, Rigid <i>Used on chairs with 15" or greater rear seat height and drum brake is present. A double stroller handle is required with Drum Brake. To view the backcane extension included in this kit, see Rogue XP / Klik / Little Wave XP Drum Brake page.</i>	1
1d, 2 - 10	113243	Stroller Handle Tall Assembly with Backcane Extension, Drum Brake, Each, Rigid <i>Used on chairs with less than 15" rear seat height and drum brake is present. A double stroller handle is required with Drum Brake. To view the backcane extension included in this kit, see Rogue XP / Klik / Little Wave XP Drum Brake page.</i>	1
1a, 2, 8, 9	506137	Stroller Handle Tube Short, Rigid <i>Used on chairs with 15" or greater rear seat height and drum brake is not present. Replaced part number 102443.</i>	1
1b, 2, 8, 9	506138	Stroller Handle Tube Tall, Rigid	1

Item Number	Part Number	Description	Quantity
		<i>Used on chairs with less than 15" rear seat height and drum brake is not present. Replaced part number 102444.</i>	
1c, 2, 8, 9	113197	Stroller Handle Back Post 8° Bend Short with Backcane Extension, Drum Brake <i>Used on chairs with 15" or greater rear seat height and drum brake is present. To view the backcane extension included in this kit, see Rogue XP / Klik / Little Wave XP Drum Brake page.</i>	1
1d, 2, 8, 9	113224	Stroller Handle Back Post 8° Bend Tall with Backcane Extension, Drum Brake <i>Used on chairs with less than 15" rear seat height and drum brake is present. To view the backcane extension included in this kit, see Rogue XP / Klik / Little Wave XP Drum Brake page.</i>	1
2	100534	Detent Button Lower	1
3	000540	Stroller Handle Clamp Rigid	1
4, 5, 10	503802	Push Handle Half Clamp w/ Set Screw <i>Replaced part number 000474</i>	1
5	101180	M6 x 20 SHCS BLK ZC	2
6	102734	Stroller Handle Clamp Handle (16mm) Rigid/Flip/Tyke	1
7	100658	M6 Nylock Nut Blk Zn	1
3 - 7, 10	501468	Stroller Handle Clamp Assembly Each Rigid	1
8	100803	Tube End Plug 7/8"	2
9	109433	Ergo Push Handle Grip 7/8" Standard with Instruction	1