



Rogue ALX Fixed Height Backrest with Adjustable Height Rigidizer Bar - Growth (Formerly Tsunami)

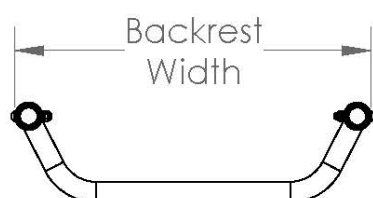

Backrest 14"	14.3"
Backrest 15"	15.3"
Backrest 16"	16.3"
Backrest 17"	17.3"
Backrest 18"	18.3"
Backrest 19"	19.3"
Backrest 20"	20.3"
Backrest 21"	21.3"
Backrest 22"	22.3"

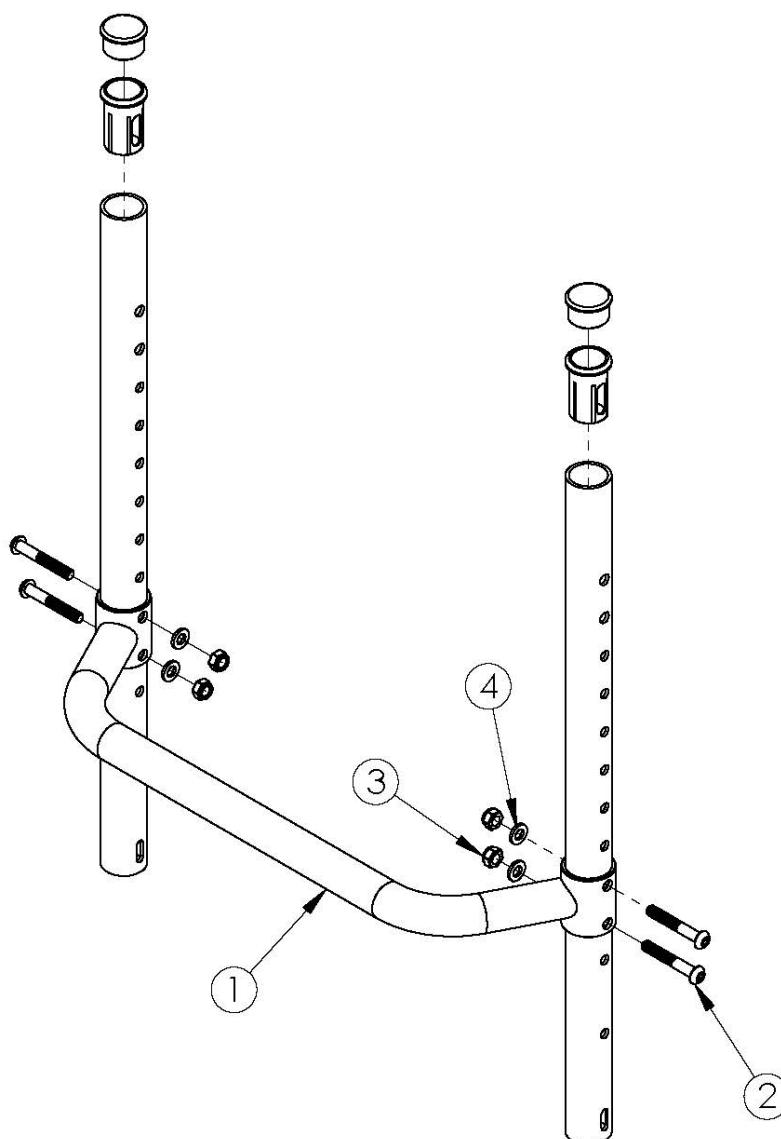

Item Number	Part Number	Description	Quantity
1a	000185	Adjustable Height Rigidizer Bar 14" Wide	1
1b	000186	Adjustable Height Rigidizer Bar 15" Wide	1
1c	000187	Adjustable Height Rigidizer Bar 16" Wide	1
1d	000188	Adjustable Height Rigidizer Bar 17" Wide	1
1e	000189	Adjustable Height Rigidizer Bar 18" Wide	1

Item Number	Part Number	Description	Quantity
1f	000190	Adjustable Height Rigidizer Bar 19" Wide	1
1g	000191	Adjustable Height Rigidizer Bar 20" Wide	1
1h	000529	Adjustable Height Rigidizer Bar 21"	1
1i	000530	Adjustable Height Rigidizer Bar 22"	1
2	100797	M6 x 40 BHCS BLK ZC	4
3	100658	M6 Nylock Nut Blk Zn	4
4	100746	M6 Flat Washer Blk Zn	4