


## INSTRUCTION SHEET OVAL TUBE TRANSIT KIT


## (Please read and understand instructions before beginning installation)

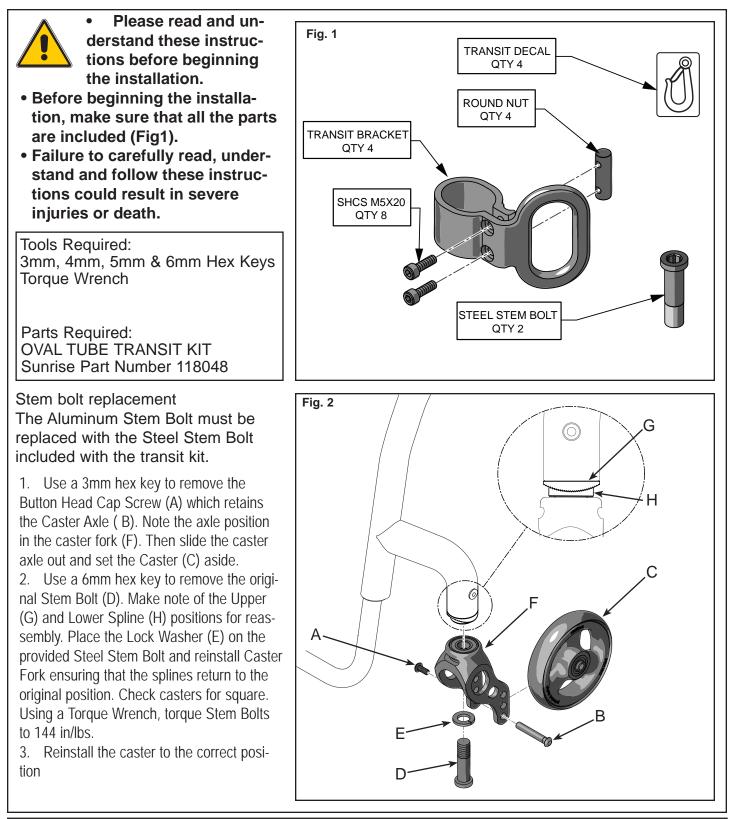

SEPTEMBER 2010

4. Use a 4mm hex key (J) to remove the Set Screws that retains the Footrest (L). Slide the Footrest out .

Install the Front Transit Bracket
Assemblies (K) on the Frame as shown in
Fig 3. Slide up to the bottom of the caster arm of the frame. Use a 5mm hex key to tighten the Button Head Cap Screws (M),
Using a Torque Wrench, torque to 62 in/lbs.
Re install the footrest to the correct height. Ensure that the footrest is parrallel to the floor.

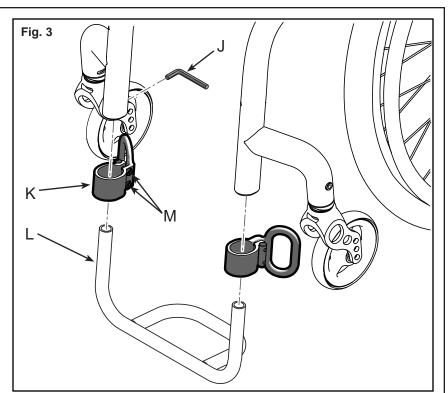



7. Install the Rear Transit Bracket Assemblies (K) onto the rear of the frame as shown in Fig 4.


8. The measurement from the rear of the frame to the clamp of the Transit Bracket should be 0" to 1/4".(Fig 5)

9. Using a Torque Wrench, torque the Button Head Cap Screws (M) to 62 in/lbs. (Fig 4)

10. Affix the four Transit Decals (N) as shown in Fig 6.

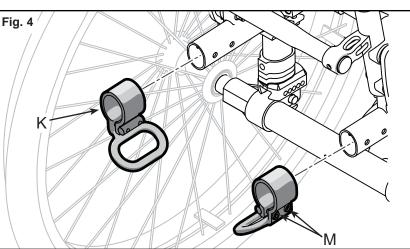

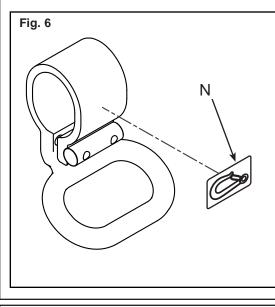

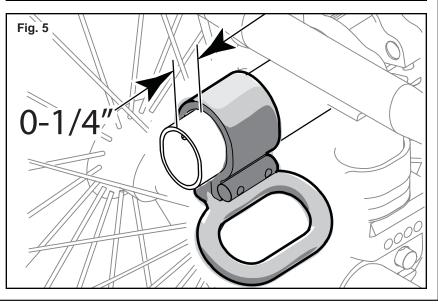

SUNRISE MEDICAL

IS00103 REV. A

SEPTEMBER 2010

PAGE 2 OF 2