


Dynamic 5th Wheel

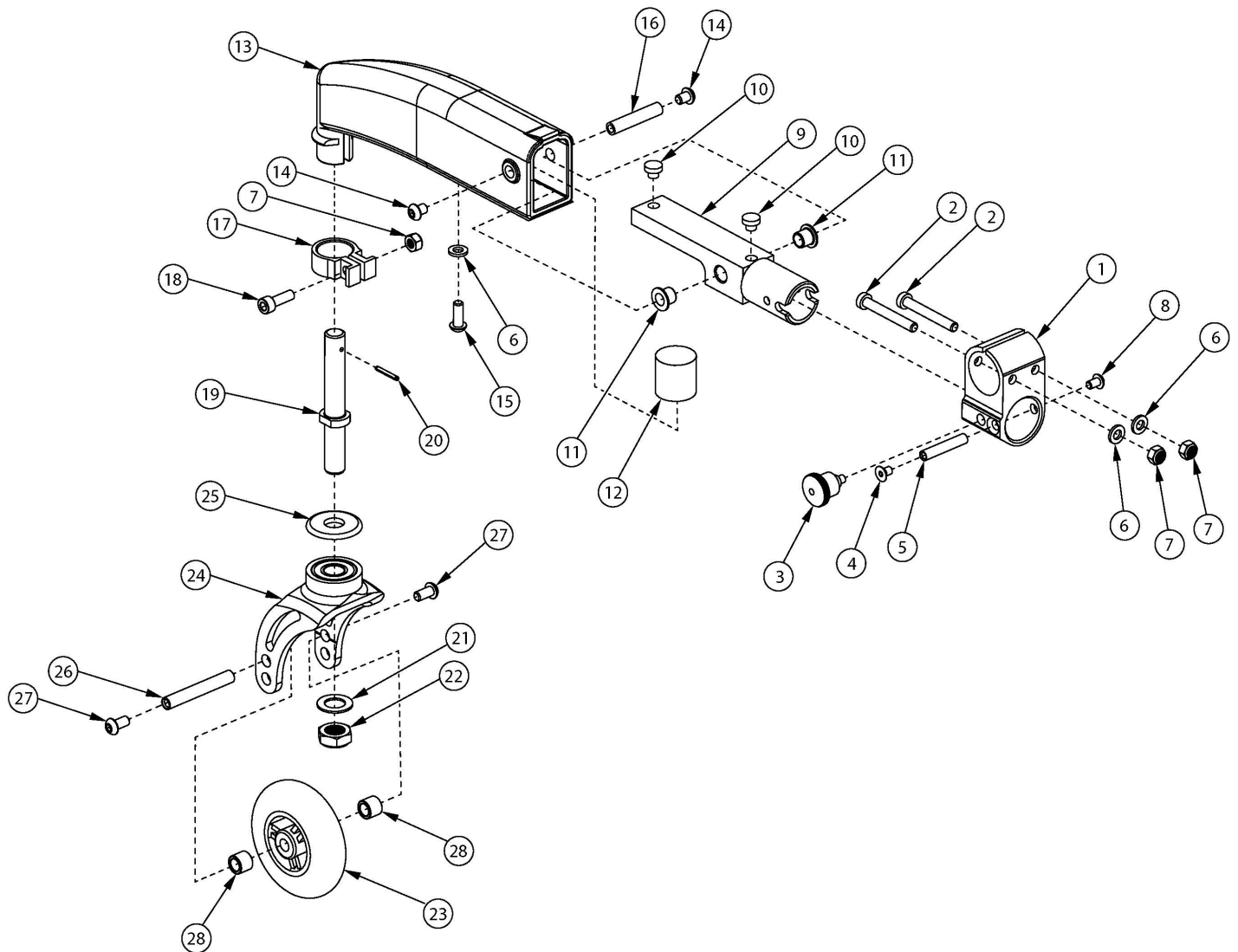

A new camber tube is needed if retrofitting to the Dynamic 5th Wheel. The Dynamic 5th wheel assemblies contain two of the camber tube mounting clamp set screw, part number 105015. For a complete Dynamic 5th Wheel Assembly, select the Dynamic 5th Wheel Clamp Assembly (502464) and the desired Dynamic 5th Wheel Assembly with Caster.

Item Number	Part Number	Description	Quantity
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23a, 24a, 25 - 28	107343	Dynamic 5th Wheel Assembly w/ 3" Rollerblade for 18" Rear Wheels	1
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23a, 24a, 25 - 28	107343_---	Dynamic 5th Wheel Assembly w/ 3" Rollerblade for 18" Rear Wheels Anodized	1

Item Number	Part Number	Description	Quantity
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23a, 24b, 25 - 28	107344	Dynamic 5th Wheel Assembly w/ 3" Rollerblade for 20" Rear Wheels	1
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23a, 24b, 25 - 28	107344_ _ _	Dynamic 5th Wheel Assembly w/ 3" Rollerblade for 20" Rear Wheels Anodized	1
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23a, 24c, 25 - 28	107345	Dynamic 5th Wheel Assembly w/ 3" Rollerblade for 22" Rear Wheels	1
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23a, 24c, 25 - 28	107345_ _ _	Dynamic 5th Wheel Assembly w/ 3" Rollerblade for 22" Rear Wheels Anodized	1
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23b, 24a, 25 - 28	107346	Dynamic 5th Wheel Assembly w/ 3" Lighted Rollerblade for 18" Rear Wheels	1
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23b, 24a, 25 - 28	107346_ _ _	Dynamic 5th Wheel Assembly w/ 3" Lighted Rollerblade for 18" Rear Wheels Anodized	1
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23b, 24b, 25 - 28	107347	Dynamic 5th Wheel Assembly w/ 3" Lighted Rollerblade for 20" Rear Wheels	1
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23b, 24b, 25 - 28	107347_ _ _	Dynamic 5th Wheel Assembly w/ 3" Lighted Rollerblade for 20" Rear Wheels Anodized	1
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23b, 24c, 25 - 28	107348	Dynamic 5th Wheel Assembly w/ 3" Lighted Rollerblade for 22" Rear Wheels	1
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23b, 24c, 25 - 28	107348_ _ _	Dynamic 5th Wheel Assembly w/ 3" Lighted Rollerblade for 22" Rear Wheels Anodized	1
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23c, 24a, 25 - 28	107349	Dynamic 5th Wheel Assembly w/ 4" Rollerblade for 20" Rear Wheels	1

Item Number	Part Number	Description	Quantity
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23c, 24a, 25 - 28	107349_ _ _	Dynamic 5th Wheel Assembly w/ 4" Rollerblade for 20" Rear Wheels Anodized	1
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23c, 24c, 25 - 28	107350	Dynamic 5th Wheel Assembly w/ 4" Rollerblade for 22" Rear Wheels	1
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23c, 24c, 25 - 28	107350_ _ _	Dynamic 5th Wheel Assembly w/ 4" Rollerblade for 22" Rear Wheels Anodized	1
9b, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23c, 24c, 25 - 28	107351	Dynamic 5th Wheel Assembly w/ 4" Rollerblade for 24" Rear Wheels	1
9b, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23c, 24c, 25 - 28	107351_ _ _	Dynamic 5th Wheel Assembly w/ 4" Rollerblade for 24" Rear Wheels Anodized	1
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23d, 24a, 25 - 28	107352	Dynamic 5th Wheel Assembly w/ 4" Lighted Rollerblade for 20" Rear Wheels	1
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23d, 24a, 25 - 28	107352_ _ _	Dynamic 5th Wheel Assembly w/ 4" Lighted Rollerblade for 20" Rear Wheels Anodized	1
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23d, 24c, 25 - 28	107353	Dynamic 5th Wheel Assembly w/ 4" Lighted Rollerblade for 22" Rear Wheels	1
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23d, 24c, 25 - 28	107353_ _ _	Dynamic 5th Wheel Assembly w/ 4" Lighted Rollerblade for 22" Rear Wheels Anodized	1
9b, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23d, 24c, 25 - 28	107354	Dynamic 5th Wheel Assembly w/ 4" Lighted Rollerblade for 24" Rear Wheels	1
9b, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23d, 24c, 25 - 28	107354_ _ _	Dynamic 5th Wheel Assembly w/ 4" Lighted Rollerblade for 24" Rear Wheels Anodized	1

Item Number	Part Number	Description	Quantity
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23e, 24a, 25 - 28	107355	Dynamic 5th Wheel Assembly w/ 4 x 1" Poly for 20" Rear Wheels	1
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23e, 24a, 25 - 28	107355_ _ _	Dynamic 5th Wheel Assembly w/ 4 x 1" Poly for 20" Rear Wheels Anodized	1
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23e, 24c, 25 - 28	107356	Dynamic 5th Wheel Assembly w/ 4 x 1" Poly for 22" Rear Wheels	1
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23e, 24c, 25 - 28	107356_ _ _	Dynamic 5th Wheel Assembly w/ 4 x 1" Poly for 22" Rear Wheels Anodized	1
9b, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23e, 24c, 25 - 28	107357	Dynamic 5th Wheel Assembly w/ 4 x 1" Poly for 24" Rear Wheels	1
9b, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23e, 24c, 25 - 28	107357_ _ _	Dynamic 5th Wheel Assembly w/ 4 x 1" Poly for 24" Rear Wheels Anodized	1
9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23f, 24c, 25 - 28	107358	Dynamic 5th Wheel Assembly w/ 5 x 1" Poly for 22" Rear Wheels	1
9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23f, 24c, 25 - 29	107358_ _ _	Dynamic 5th Wheel Assembly w/ 5 x 1" Poly for 22" Rear Wheels Anodized	1
9b, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23f, 24b, 25 - 28	107359	Dynamic 5th Wheel Assembly w/ 5 x 1" Poly for 24" Rear Wheels	1
9b, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23f, 24b, 25 - 28	107359_ _ _	Dynamic 5th Wheel Assembly w/ 5 x 1" Poly for 24" Rear Wheels Anodized	1
1 - 8	502464	Dynamic 5th Wheel Clamp Assembly	1
1	001807	5th Wheel Mount	1
2	101123	M6 x 45 Low SHCS BLK ZC	2
3	002050	Mini Indexing Plunger	1

Item Number	Part Number	Description	Quantity
4	102308	M5 x 8mm FHCS Blk Zc	1
5	001813	Barrel 5th Wheel	1
6	100746	M6 Flat Washer Blk Zn	3
7	100658	M6 Nylock Nut Blk Zn	3
8	102621	M5 x 8 BHCS BLK ZC	1
9a	001808	Bump Stop, 5th Wheel <i>Used with 18" - 22" rear wheels</i>	1
9b	002362	Bump Stop Long, 5th Wheel <i>Used with 24" rear wheels</i>	1
10	102631	Stop Bumper	2
11	002041	Flange Bushing For M6	2
12a	002409	5th Wheel Bumper, 40 Durometer, Yellow, Soft Elastomer	1
12b	002410	5th Wheel Bumper, 60 Durometer, Black, Firm Elastomer	1
13	001809	Swing Arm Composite, 5th Wheel	1
14	101700	M6 x 8 BTN SHCS BLK ZC	2
15	102634	M6 X 1.0 X 16 BHCS BLK ZC	1
16	001814	Barrel Swing Arm, 5th Wheel	1
17a	001810	Split Clamp 3/4" Black	1
17b	001810_ _ _	Color Anodized Split Clamp 3/4"	1
18	101844	M6 x 18 SHCS Blk Zn	1
19	001811INST	Stem 5th Wheel	1
20	102613	.094 X 75 SPRING PIN ZINC PLATED	1
21	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	1
22	101459	1/2-20 NTE JAM NYL INS L/N BLK ZC	1
23a	100790	Caster 3" Rollerblade	1
23b	101140	Caster 3" Lighted Rollerblade	1
23c	100791	Caster 4" Rollerblade	1
23d	100952	Caster 4" Lighted Rollerblade	1

Item Number	Part Number	Description	Quantity
23e	100809	Caster 4 x 1" Poly (1.25")	1
23f	100807	Caster 5 x 1" Poly (1.25")	1
24a	004361INST	Short Rigid Fork Streamline Assembly, Black Anodized with Instruction <i>Replaced 001812INST</i>	1
24b	004354INST	Standard Rigid Fork Streamline Assembly, Black Anodized with Instruction <i>Replaced 000927INST</i>	1
24c	004362INST	Tall Rigid Fork Streamline Assembly, Black Anodized with Instruction <i>Replaced 000928INST</i>	1
25	001355	Caster Bearing Cap Black Anodized	1
26	000422	M6 x 56 Threaded Barrel	1
27	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2
28	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer	2

Color Anodizing Samples

