


Liberty Push to Lock Wheel Locks

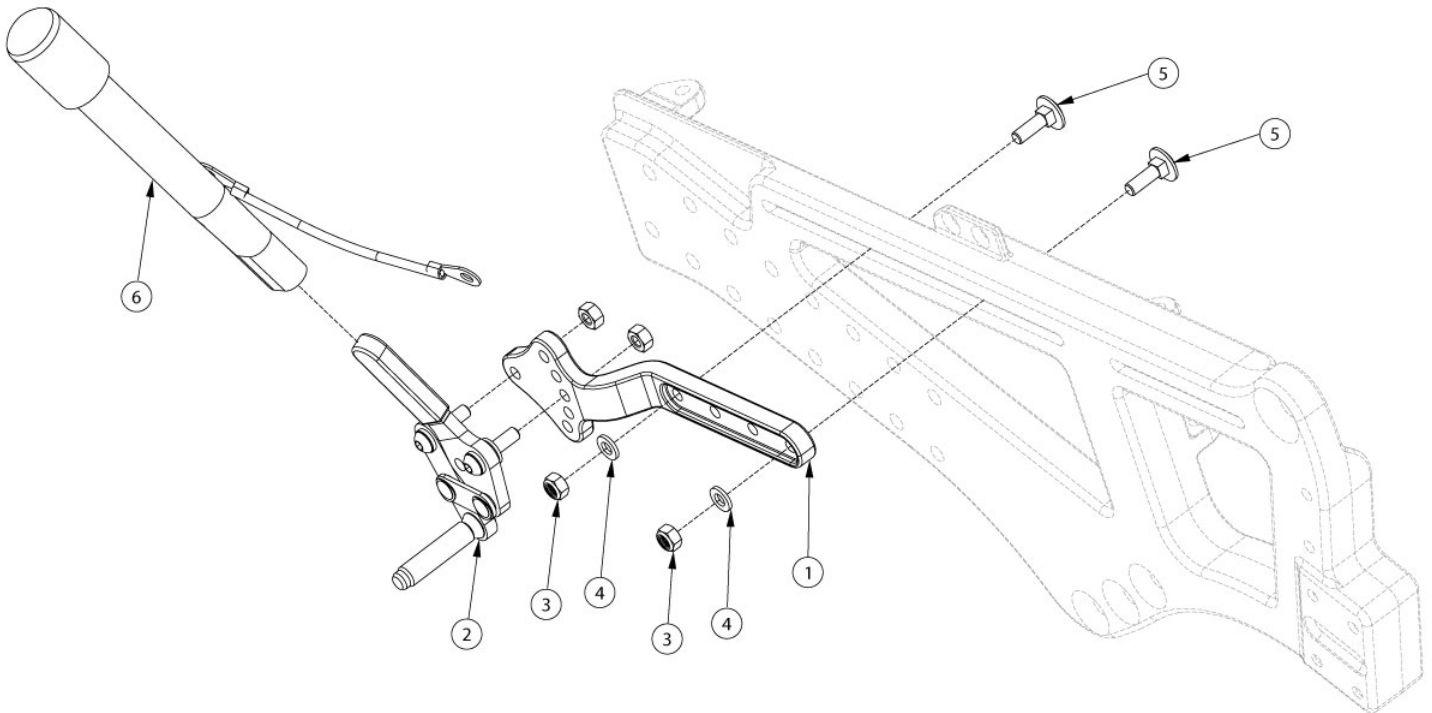

Item Number	Part Number	Description	Quantity
1, 2a, 3-5	506175	Wheel Lock Assembly, Push Right	1
1, 2b, 3-5	506176	Wheel Lock Assembly, Push Left	1
1, 2a, 3-6	506177	Wheel Lock Assembly, Push with Extension Right	1
1, 2b, 3-6	506178	Wheel Lock Assembly, Push with Extension Left	1
1	003329	Mount, Wheel Lock	1
2a	003668	Wheel Lock, Push, Right	1
2b	003669	Wheel Lock, Push, Left	1
2a, 6	003670	Wheel Lock, Push with Extension, Right	1
2b, 6	003671	Wheel Lock, Push with Extension, Left	1
3	100658	M6 Nylock Nut Blk Zn	2
4	100746	M6 Flat Washer Blk Zn	2
5	103032	M6 x 20 Carriage Bolt, Blk Zn	2
6	200369	Wheel Lock Extension	1