


## Catalyst 5VX Anti-Tip Receiver

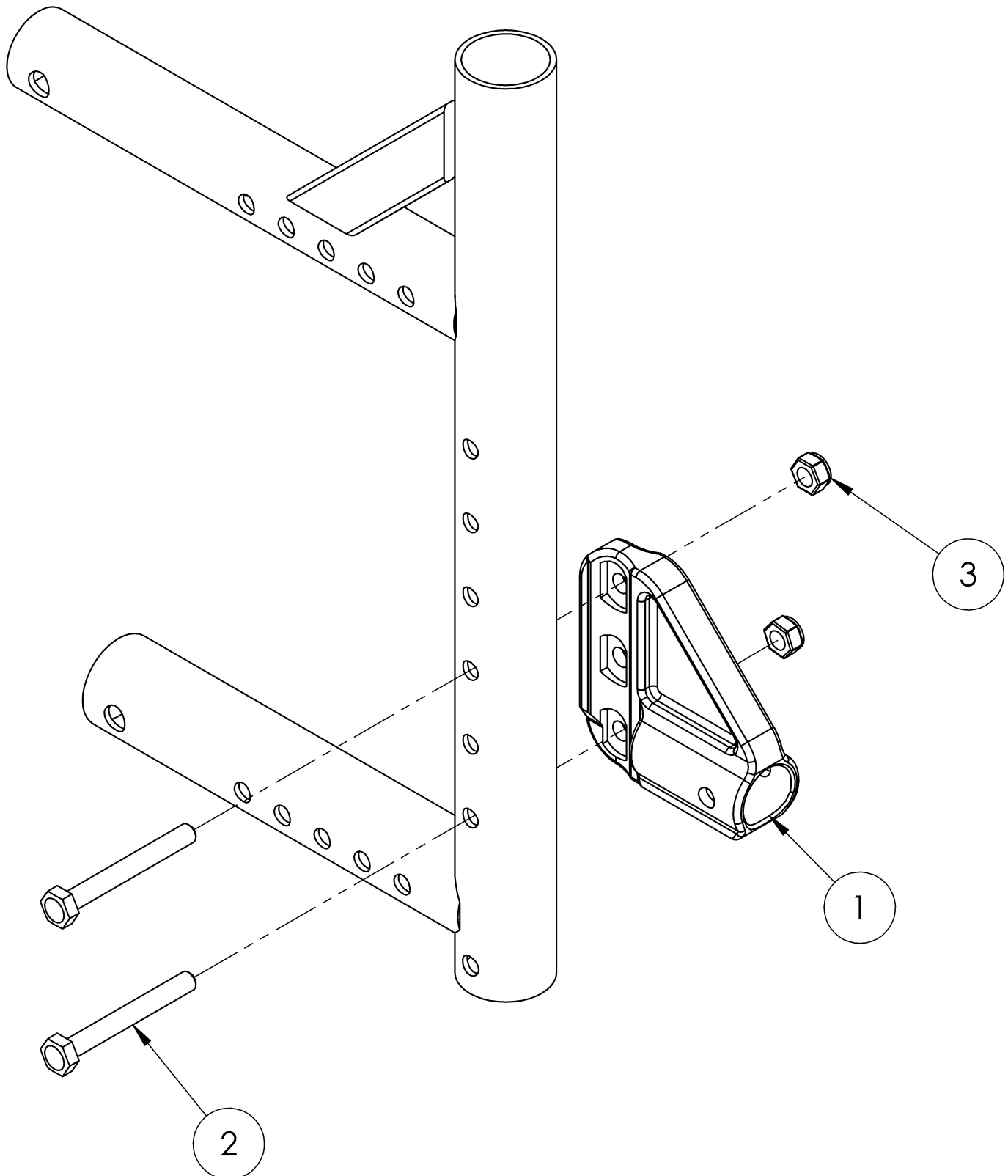

Please see the Catalyst 5VX Swing-In Anti-Tip page for Swing-In Anti-Tip replacements

| Item Number | Part Number | Description  | Quantity |
|-------------|-------------|--|----------|
| 1b          | 004539      | Anti Tip Receiver, Catalyst<br><i>Used if chair has an invoice date after November 13, 2018.<br/>Replaced 000046</i>   | 1        |
| 2a          | 101826      | M6 x 50 HHCS Blk Zn<br><i>Use quantity 2 per side if only Anti-Tips are present. Use<br/>quantity 1 per side if Transit and Anti-Tips are present</i>  | 2        |
| 2b          | 100676      | M6 x 55 CLS10.9 DIN931 HHCS ZC<br><i>Use quantity 2 per side if Transit and Anti-Tip are present.<br/>Use quantity 1 per side if Swing Away Arm bracket, Transit,<br/>and Anti-Tip are present</i> | 2        |
| 2c          | 100677      | M6 x 60 CLS10.9 DIN931 HHCS ZC<br><i>Use quantity 2 per side if Transit, Anti-Tip and Swing Away<br/>Arm bracket are present</i>   | 2        |
| 3           | 100658      | M6 Nylock Nut Blk Zn   | 2        |