


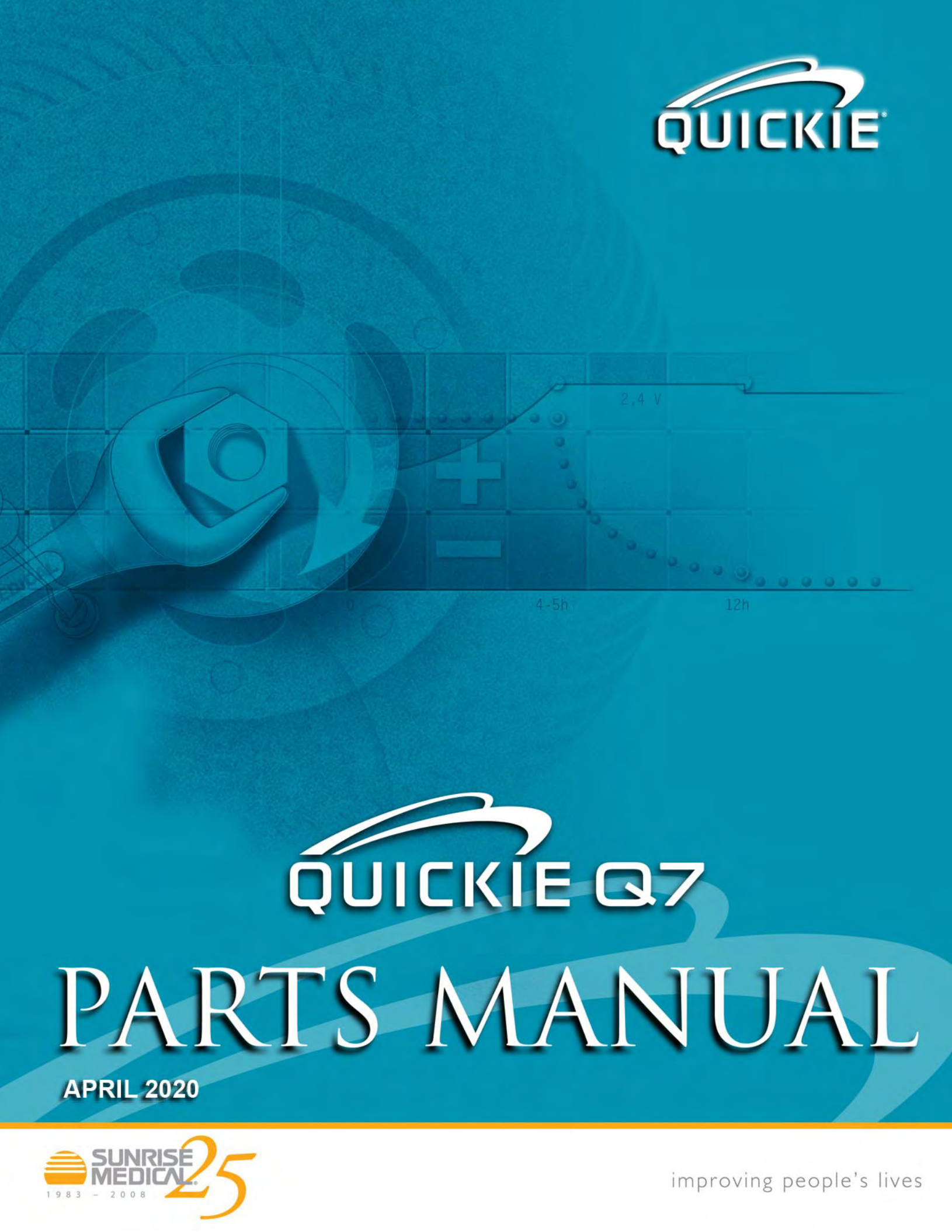

QUICKIE Q7

PARTS MANUAL

APRIL 2020

COVER.....	1
Multimedia	1
QUICKIE Q7	6
GLOSSARY OF ABBREVIATIONS.....	6
BASE FRAME	6
FRAME	6
STANDARD ADJUSTABLE FRAME prior to S/N R4-023230	7
STANDARD ADJUSTABLE FRAME after S/N R4-023230	9
ULTRA LIGHT SEAT ADJUSTABLE FRAME prior to S/N R4-023230	14
ULTRA LIGHT SEAT ADJUSTABLE FRAME after S/N R4-023230	16
ACTIVE RIGID FRAME prior to S/N R4-023230	21
ACTIVE RIGID FRAME after S/N R4-023230	22
Q7 TRANSIT KIT - WC19 after S/N R4-023230	23
TRANSIT KIT - WC19 prior to S/N R4-023230	24
AXLE PLATES AND AXLES	24
AXLE PLATE	25
ACTIVE RIGID AXLE PLATE	27
REAR SUSPENSION AXLE BRACKET	28
QUICK RELEASE AXLES MATRIX-Q7/QRI/ZONE	33
CAMBER TUBE	33
CAMBER TUBE ASSM	34
CASTER AND FORKS	35
CASTERS	35
MICRO CASTERS.....	36
3 SPOKE CASTERS	37
SOFT ROLL CASTERS.....	39
ALUMINUM CASTERS	40
ALUMINUM SOFT ROLL CASTERS	41
FORK AND STEM ASSEMBLY.....	41
STANDARD MULTIPLE POSITION FORK	42
SINGLE SIDED MULTIPLE POSITION FORK.....	46
FROG LEG SUSPENSION FORK	50
ANTI TIPS	51
ANTI TIPS	52
WHEEL LOCKS	53
HIGH MOUNT PUSH WHEEL LOCK	54
HIGH MOUNT PULL WHEEL LOCK.....	55
ERGO SCISSOR WHEEL LOCK	56
COMPACT WHEEL LOCK	57
COMPACT WHEEL LOCK - DIRECT MOUNT	58
WHEEL LOCK EXTENSION HANDLE.....	59
WHEELS & TIRES	59
WHEELS	59
TRADITIONAL 6 SPOKE MAG	60
5 SPOKE MAG	61
LITE SPOKE.....	63
NEW ULTRA LITE SPOKE	65
SPINERGY SPOX (18 SPOKE)	67
SPINERGY LX (12 SPOKE).....	68
SPINERGY SPORT LT EX (18 SPOKE).....	70
MOUNTAIN	72
TIRES.....	72
PNEUMATIC & PNEU. AIRLESS TIRE.....	73
FULL POLY TIRE	74
HIGH PRESSURE CLINCHER TIRE	75
KNOBBY V-TRACK TIRE.....	76
MOUNTAIN TIRE	77
SCHWALBE MARATHON PLUS TIRE	78
SCHWALBE SPEED RUN TIRE	79
SHOX TIRE	80
HANDRIMS	80
ALUMINUM ANODIZED (SCREW MOUNT).....	81
ULTRA LITE ANODIZED (TAB MOUNT)	82
ALUMINUM ANODIZED (TAB MOUNT)	83
PLASTIC COATED (SCREW MOUNT).....	84
PLASTIC COATED (REG TAB MOUNT)	85
NATURAL FIT (SCREW MOUNT)	86
NATURAL FIT (TAB MOUNT).....	88
PROJECTION	90
PAINTED (CROSS SPOKE WHEELS ONLY)	92

CHROME (MAG WHEELS ONLY)	93
STAINLESS STEEL (QP WHEELS ONLY)	94
TITANIUM (QP WHEELS ONLY)	95
COMPOSITE (SCREW MOUNT)	96
PAINTED (SCREW MOUNT)	97
PLASTIC COATED 5 SPOKE MAG (SCREW MOUNT)	98
ALUMINUM ANODIZED 5 SPOKE MAG (SCREW MOUNT)	99
Q-GRIP (TAB MOUNT)	100
Q-GRIP (SCREW MOUNT)	101
THE SURGE (TAB MOUNT)	102
THE SURGE (SCREW MOUNT)	103
NATURAL FIT 5 SPOKE MAG (SCREW MOUNT)	104
ALUMINUM POWDER COATED (SCREW MOUNT)	105
ULTRA LITE POWDER COATED (TAB MOUNT)	106
ALUMINUM POWDER COATED (TAB MOUNT)	107
TUBES	109
TUBES AND AIRLESS INSERTS	109
SEATING AND POSITIONING	109
SEAT	109
STANDARD SEAT SLING & CUSHION prior to S/N R4-023230	110
STANDARD SEAT SLING & CUSHION after S/N R4-023230	113
SEAT SLING ULTRA LITE	118
BACK	119
BACK FRAMES	119
FOLDING ANGLE ADJUSTABLE BACK FRAMES	119
HEIGHT ADJUSTABLE FOLDING ANGLE ADJ BACK FRAME	120
FIXED HEIGHT FOLDING ANGLE ADJ BACK FRAME	123
HEIGHT ADJUSTABLE FOLDING INSET ANGLE ADJ BACK FRAME	132
FIXED HEIGHT FOLDING INSET ANGLE ADJ BACK FRAME	135
NON-FOLDING ANGLE ADJUSTABLE BACK FRAMES	143
HEIGHT ADJUSTABLE NON-FOLDING ANGLE ADJ BACK FRAME	144
FIXED HEIGHT NON-FOLDING ANGLE ADJ BACK FRAME	146
HEIGHT ADJUSTABLE NON-FOLDING INSET ANGLE ADJ BACK FRAME	155
FIXED HEIGHT NON-FOLDING INSET ANGLE ADJ BACK FRAME	157
HEIGHT ADJUSTABLE BACKREST HANDLES	161
TELESCOPING PUSH HANDLES (BOLT ON)	163
FOLDING PUSH HANDLES-NON HGT ADJ	165
BACKREST (Q7 ACTIVE RIGID/7RS WELDED)	166
BACK UPHOLSTERY	166
STD BACK UPH USED W/ OMIT HANDLES	167
STD BACK UPH USED W/ INTEGRAL PUSH HANDLES	169
TENSION ADJUSTABLE BALLISTIC NYLON UPHOLSTERY	171
TENSION ADJUSTABLE 3DX VENTED UPHOLSTERY	174
FOOTREST	176
ALUMINUM TUBULAR prior to S/N R4-023230	177
ALUMINUM TUBULAR for 12-16" frame width after S/N R4-023230	180
ALUMINUM TUBULAR for 17-20" frame width after S/N R4-023230	182
FOOTREST COVER & WEAR PADS	184
ULTRA-LITE ALUMINUM PLATFORM FOOTREST	185
ALUMINUM PLATFORM FOOTREST	187
ALUMINUM FLIP-BACK FOOTREST	192
HIGH MOUNT FOOTREST CLAMP	195
ALUMINUM ADAPTIVE FLIP-BACK FOOTREST	196
ARMREST AND RECEIVERS	198
PADDED SWING-AWAY ARMREST	199
SINGLE POST HEIGHT ADJUSTABLE ARMREST	200
ACCESSORIES	201
XTENDER	201
4X XTENDER- discontinued	201
XTENDER RETRO FIT PARTS (7 SERIES)	202
4X XTENDER AXLE PLATE ASSM (7 SERIES)	203
XTENDER POWER ASSIST 4X PARTS	204
XTENDER INTERNAL REPLACEMENT PARTS	206
JWX-2 XTENDER	206
GEN 2 XTENDER RETRO FIT PARTS (7 SERIES)	207
XTENDER GEN 2 WHEEL PARTS	209
DRIVE UNIT INTERNAL PARTS	210
XTENDER RIGHT HAND CONTROL SPARE PARTS	211
BATTERY AND CHARGER	212
BATTERY BACKPACK PARTS	213

XTENDER SPOKE GUARD	214
SIDE GUARDS.....	214
PLASTIC SIDE GUARD	215
ALUMINUM SIDE GUARD	216
CARBON SIDE GUARD NO FENDER.....	218
POSITIONING BELTS	218
AIRCRAFT BUCKLE POSITIONING BELTS & ATTACHMENT KITS	219
AUTO BUCKLE POSITIONING BELT & ATTACHMENT KITS.....	221
BACKPACKS AND SEAT POUCHES.....	222
QUICKIE SIDE POCKET.....	223
CELL PHONE PACK.....	224
BACKPACKS AND SEAT POUCHES.....	225
SPOKE GUARDS.....	225
SPOKE GUARDS.....	226
CRUTCH HOLDER (RIGID CHAIRS)	228
QUICKIE CADDY	229
TOOL KIT	230
IMPACT GUARD NEOPRENE & GEL VELCRO PADS.....	231
TIRE PUMP	232
Back Cover.....	232
Product	233

Note: see Informations Panel for file.

1AD	One Arm Drive	CZ	Cozy	INH	Inhibit	PCB	Pediatric Center of Gravity (AKA: Zippie Iris)	S/A	Swing Away
3DCR	3DX Microclimatic cover for Cryo style inserts	DAB	Depth-adjustable Back	INORG	Inorganic coating	PCM	Power Center Mount	S/F	Side Frame
AAA	Adult Angle Adjustable	DB	Dynamic Back	INSRT	Insert	PED	Pediatric	SCW	Screw
AACM	Angle Adjustable Center Mount	DBL	Double	I/O	Swing In Out	PEL	Pelvic	SDLE	Saddle
ABD	Abductor	DC	Deep Contour	IRIS	Intelligent Rotation in Space	PH	Push Handle	SG	Side Guard
AC	All Court	DL	Desk Length	ISM	Intelligent Seating Module	PHIL	Phillips	SGTG	Super Grip Thumb Guard
AD	Arm Drive	DP	Dual Post Armrest	JA	Jay Active	P/L	Pin Lock	SHCS	Socket Head Cap Screw
ADJ	Adjustable	D-Post	Dual Post Armrest	JNR	Junior	PL	Plated	SHSS	Socket Head Set Screw
ADPT	Adapter	D-PST	Dual Post Armrest	KAA	Kids Angle Adjustable	PLL	Pull	SHLDR	Shoulder
AE	Air Exchange	DRP	Drop (seat)	KDS	Kids	PLT	Plate	SHT	Short
AHAB	ASAP II	DRV	Drive	KT	Kit (usually consists of all mounting hardware for one side)	PLST	Plastic	SLD	Solid
ANG	Angle	DTT	Diagnostic Test Tool	KVLR	Kevlar	PLY	Plywood	SLNG	Sling
ALR	Articulating Legrest	DY	Dynamic Back	KZ	Quickie Kidz/Zippie Kidz	PNEU	Pneumatic	SLTD	Slotted
ALUM	Aluminum	EA	Each	L	Left	PNK	Pink	SLV	Sleeve
AMH	Adjustable Mounting Hardware	ELR	Elevating Legrest	LAT	Left	PNT	Paint	SM	Small
ANO	Anodized	EXT	Extension	LCK	Lock	PNTD	Painted	SNGL	Single
ASAP	Adjustable Seating and Positioning	EXTND	Extended	LFH	Left Front Horizontal	POLY	Polyurethane	SOC	Socket
ASL	Adaptive Switch Laboratories	F	Folding Frame	LFT	Left	POST	Posterior	SPCR	Spacer
ASM	Assembly (includes other parts)	FHCS	Flat Head Cap Screw	LFV	Left Front Vertical	PR	Pressure Relieving	SPEC CLR	Specify Color
ASSM	Assembly	FL	Full Length	LH	Left Hand	PR	Power Recline	SFHA	Single Post Height Adjustable
ASSMLY	Assembly	FLD	Folding	LH	Left Horizontal	PR	Pair	SPK	Spoke
ASSY	Assembly	FL FT	Flip-up Footrest	LK	Lock	PR	Pair	SPR	Super Grip Handrim
AT	Anti-tip	FLP	Flip	LNG	Long	(PR)	Pair	SQZ	Squeeze
ATT	Attachment	FLRD	Flared	L/O	Lift Off	PRCR	Premium Leatherette cover for Cryo style inserts	SS	Stainless Steel
BB	Basketball	FM	Foam	L/O	Lift Off	PRE-ISC	Pre-Ischial	ST	Seat
BC	Breezy Chrome	FM	Front Mount	LP	Loop	PREM	Premium Leatherette cover material	S/TAP	Self Tapping
BCK	Back	FNDR	Fender	LPS	Loops	PROJ	Projection	STCR	Stretch Cover for Cryo Style inserts
BCKBRD	Backboard	FPH	Folding Push Handle	LRG	Large	PRPL	Purple	STD	Standard
BH	Button Head	FT	Foot	LRH	Left Rear Horizontal	PT	Point	STL	Style
BHCS	Button Head Cap Screw	F/REST	Footrest	LRV	Left Rear Vertical	PTTRN	Pattern	STR	Stretch cover material
BKL	Buckle	FRNT	Front	LS	Low Sheer	PWR	Power	STRLR	Stroller
BKRST	Backrest	FT PLT	Footplate	LT	Long Tab	Q1	Quickie 1	STRT	Straight
B-LOCK	Back Lock	FTRST	Footrest	LT	Natural Fit Lite Handrims	Q2	Quickie 2	SUP	Support
BLD	Build Up	F/UP	Flip-up	LTRL	Lateral	Q2HP	Quickie 2 High Performance	SYS	System
BLK	Black	FWD	Front Wheel Drive	LUMB	Lumbar	QC	Quickie Carbon	T	Tab
BLU	Blue	FXT	Fixed	LV	Left Vertical	QR	Quick Release	TG	Thumb Guard (std)
BR	Breezy	G8	Grade 8 higher grade = stronger	LW	Low	QRx	Quickie RX	THRD	Threaded
BRD	Board	GP	Quickie GP	LX	Quickie LX	QT	Quickie Titanium	Ti	Titanium
BRNG	Bearing	GPS	Quickie GPS	LXI	Quickie LXi	QT/QTS	Quickie Tilt & Space	TL	Tall
BRKT	Bracket	GPV	Quickie GPV	MACH	Machine	QUAD	Quadrant Pad	TLC	Tilt Lift Comb
BSHNG	Bushing	GNRC	Generic	MECH	Mechanism	R	Right	TLT	Tilt
BTTN	Button	H	Hole	MED	Medium	RCLNR	Recliner	TNNS	Tennis
BTRY	Battery	HB	High Brake	mm	Millimeter	REC	Receiver	TS	Tilt-in-Space
BZ	Breezy	HD	Head or Heavy Duty	MNT	Mount	REG	Regular	TTS	Tilt-in-Space
CAA	Composite Angle Adjustable	H/D	Heavy Duty	MNTG	Mounting	REL	Release	UNIV	Universal
CG	Center of Gravity	HDRST	Headrest	MOD	Modify or Modification	REV	Reverse	UNTHRD	Unthreaded
CGT	Center of Gravity Tilt (AKA: IRIS)	HDWR	Hardware	M/S	Machine Screw	RH	Right Hand	UPH	Upholstery
CHRG	Charger	HEX	Hexagon	MWD	Mid Wheel Drive	RH	Right Horizontal	VERT	Vertical
CJSM	Color Joystick Module	HH	Hemi-Height or Hex Head	Nat.Fit	Natural Fit Handrim	RFH	Right Front Horizontal	W	With
CLMP	Clamp	HK	Hook	NAUG	Naugahyde	RFV	Right Front Vertical	W/	With
CLNCHR	Clincher	HL	Heel Loops	NF	Non-folding	RNUT(S)	Rivnut(s)	WD	Wide
CLR	Clear	HM	High Mount	NF	Natural Fit Handrim	RoHS	Restriction of Hazardous Substances	W/F	Water Fall
COMP	Complete	HNDL	Handle	NFLT	Natural Fit Lite Handrim	RR	Rear	W/O	Without
CMPST	Composite	HNGR	Hanger	NR	Non-recessed	RRH	Right Rear Horizontal	WHL	Wheel
CRM	Chrome	HP	High Pressure	OBLIQ	Obliquity	RRV	Right Rear Vertical	WLDMNT	Weldment
C/S	Cap Screw	HR	Handrim	OBLQ	Oblique	RT	Right	XS	Cross Spoke
C-S	Countersunk	HRDWR	Hardware	OD	Outer/Outside Dimension	RV	Right Vertical	YLW	Yellow
CSTR	Caster	HRNSS	Harness	O/S	Old Style	RWD	Rear Wheel Drive	Z2	Zippie 2
CUSH	Cushion	HS	High Speed	OUTBRD	Outboard	SA	Swing Away	ZN	Zinc
CV	Conventional	HT	Height or Heat Treated	P1	Quickie P100			ZP	Zippie
CVR	Cover	ID	Inner/Inside Dimension Diameter	P2	Quickie P200			ZP2	Zippie 2
CVRS	Covers	I N	In/Out (for swing in/out hangers)	P3	Quickie P300				
CYL	Cylinder	INBRD	Inboard	P5	Quickie/Zippie P500				
		INC	Incontinent						


Pos.	Item Number	Description	Remarks
		GLOSSARY OF ABBREVIATIONS	see attachment in Informations Panel

Note: Frame design update, please order replacement frame from "STANDARD ADJUSTABLE FRAME after S/N R4-002330" page. See Frame Replacement Guide for additional parts needed.

Note: See Back Frame pages for Depth Adjustable Back or Padded Swing-Away Armrest inserts.

Note: For growing the Q7 frame please contact customer service to ensure proper frame is ordered.

Note: Frame Transit Ready as of S/N R4-019761. Replacement frames are retrofitable and ship with hardware for Transit or Non-Transit.

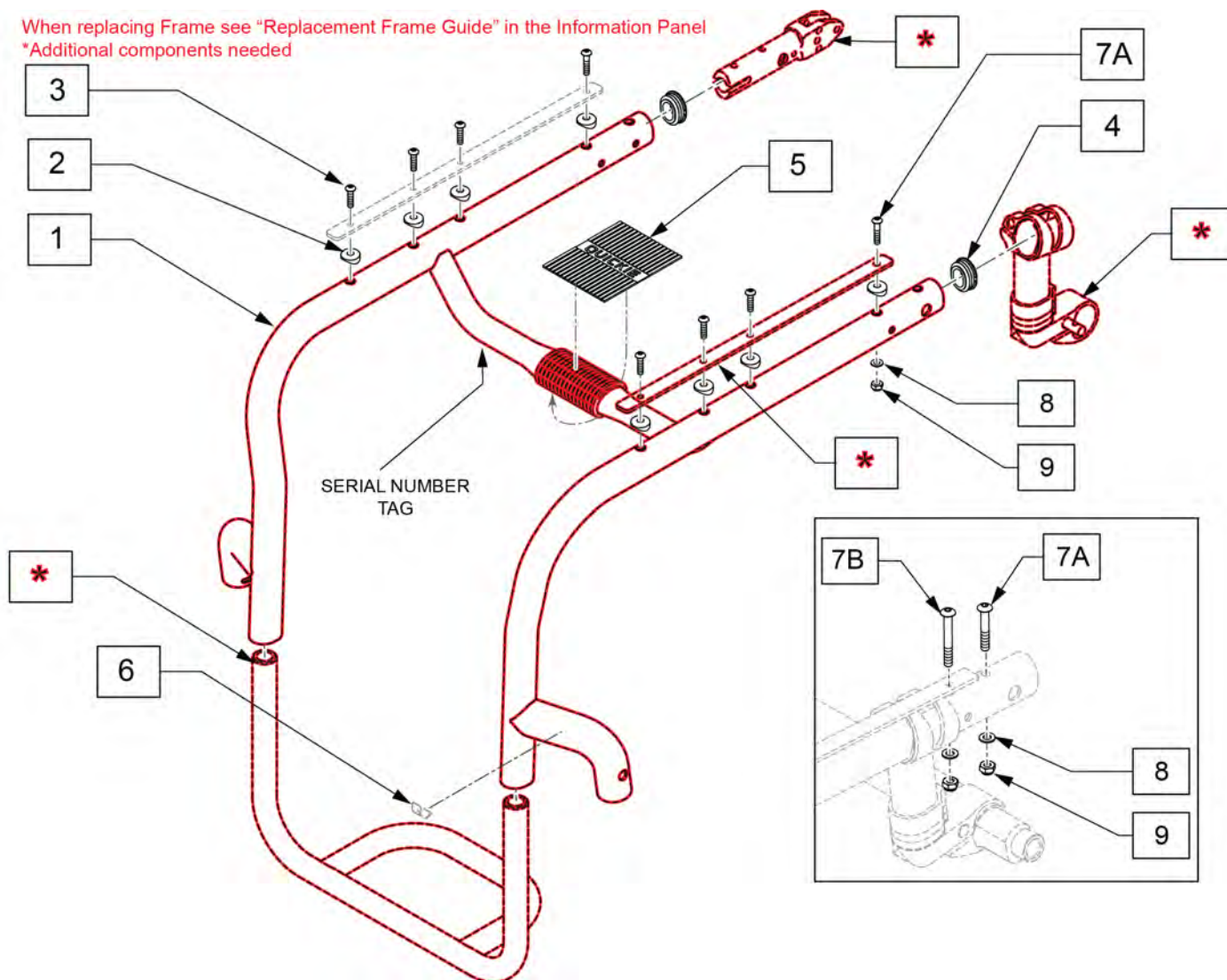


Pos.	Item Number	Description	Remarks
2	482000	SADDLE, SMALL	
3	102259-002	BHCS, M5 X 20	
4	110161	FRAME INSERT, REAR SEAT TUBE	
5	112310	GRIP FRAME CROSS BAR	
6	125519	LABEL QUICKIE Q7 SILVER	
6	125520	LABEL QUICKIE Q7 BLACK	
7A	122916	BHCS M5 X 0.8 X 50 FULL PL	
7B	122917	BHCS M5 X 0.8 X 55 FULL PL	Used with Transit
8	124956	WASHER M5	
9	220121	NUT HEX NYLOCK M5 BLACK	

NOTE: When ordering replacement frames you must provide the seat depth, frame style, front seat height, frame color and if the chair was equipped with Depth Adjustable Back or Ultra Light Wheel Lock.

NOTE: See Back Frame pages for Depth Adjustable Back or Padded Swing-Away Armrest inserts.

NOTE: For growing the Q7 frame please contact customer service to ensure proper frame is ordered.

Note: Frame Transit Ready as of S/N R4-019761. Replacement frames are retrofitable and ship with hardware for Transit or Non-Transit.

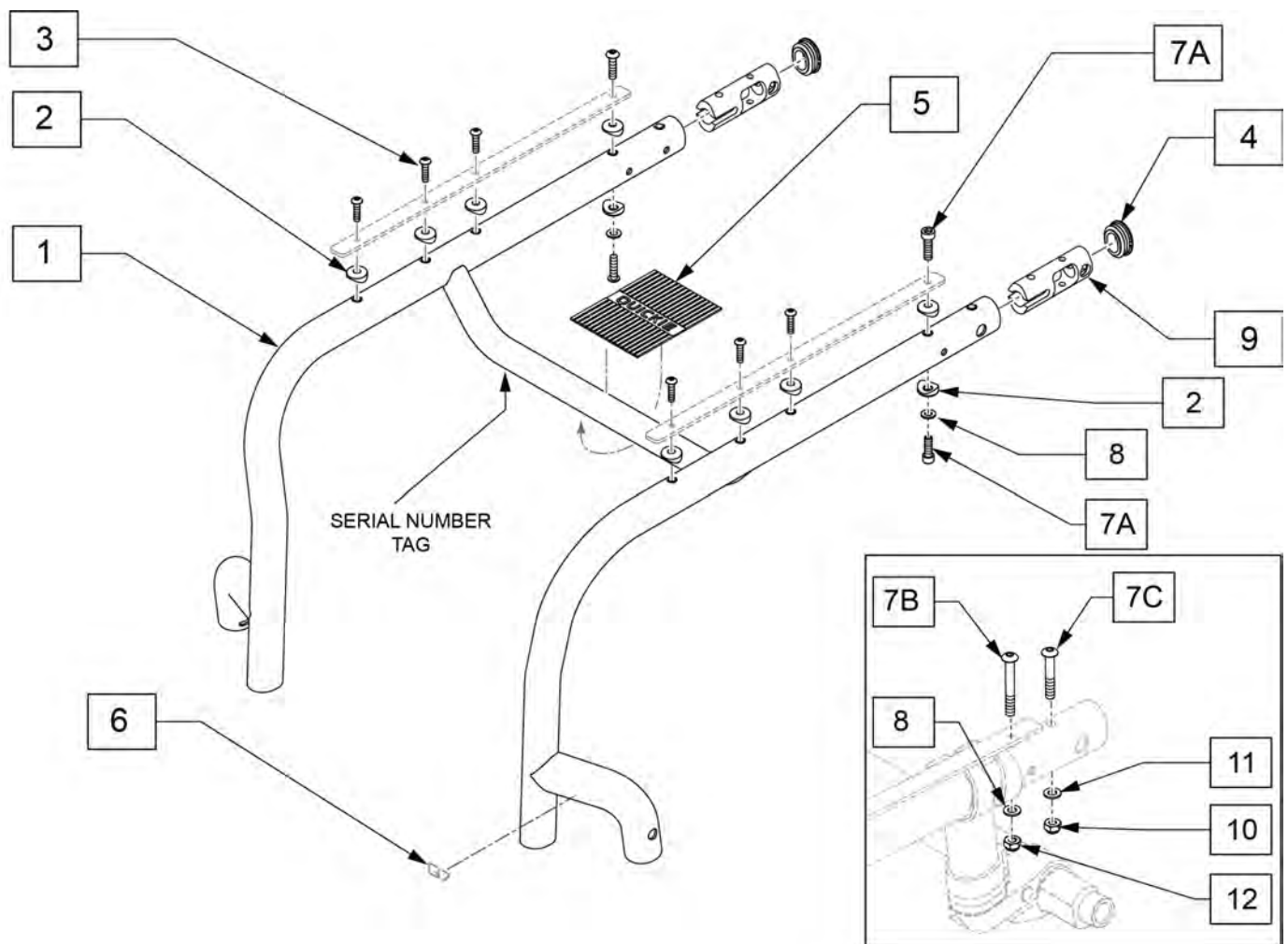

Pos.	Item Number	Description	Remarks
2	482000	SADDLE, SMALL	
3	102259-002	BHCS, M5 X 20	
4	110161	FRAME INSERT, REAR SEAT TUBE	
5	128109	ROUND GRIP FRAME CROSS BAR	

Pos.	Item Number	Description	Remarks
6	125519	LABEL QUICKIE Q7 SILVER	
6	125520	LABEL QUICKIE Q7 BLACK	
7A	102263-005	SHCS, M6X1X20, FULL, PL	
7B	102262-008	BHCS,M6X1X50,PART,NP	Used w/ Padded S/A arms & DAB
7C	122916	BHCS M5 X 0.8 X 50 FULL PL	Used w/ Transit
8	250104	WASHER FLAT M6	
9	126803	STANDARD FRAME INSERT OVAL	
10	220121	NUT HEX NYLOCK M5 BLACK	Used w/ Transit
11	124956	WASHER M5	Used w/ Transit
12	104381	NUT HEX NYLOCK M6X1.0 INORG	Used w/ Padded S/A arms & DAB
		"A" FAME ANGLE	
1-7A,8,9	114116S	FRAME A 0I SET1 12	12W,A Frame Ang, 0" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114115S	FRAME A 0I SET2 12	12W,A Frame Ang, 0" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125081S	FRAME A 0I SET3 12W	12W,A Frame Ang, 0" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114118S	FRAME A 1I SET1 12	12W,A Frame Ang, 1" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114117S	FRAME A 1I SET2 12	12W,A Frame Ang, 1" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125082S	FRAME A 1I SET3 12W	12W,A Frame Ang, 1" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114128S	FRAME A 0I SET1 13	13W,A Frame Ang, 0" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114127S	FRAME A 0I SET2 13	13W,A Frame Ang, 0" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125087S	FRAME A 0I SET3 13W	13W,A Frame Ang, 0" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114130S	FRAME A 1I SET1 13	13W,A Frame Ang, 1" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114129S	FRAME A 1I SET2 13	13W,A Frame Ang, 1" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125088S	FRAME A 1I SET3 13W	13W,A Frame Ang, 1" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114140S	FRAME A 0I SET1 14	14W,A Frame Ang, 0" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114139S	FRAME A 0I SET2 14	14W,A Frame Ang, 0" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125234S	FRAME A 0I SET3 14W	14W,A Frame Ang, 0" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114142S	FRAME A 1I SET1 14	14W,A Frame Ang, 1" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114141S	FRAME A 1I SET2 14	14W,A Frame Ang, 1" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125235S	FRAME A 1I SET3 14W	14W,A Frame Ang, 1" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114143S	FRAME A 2I SET2 14	14W,A Frame Ang, 2" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125236S	FRAME A 2I SET3 14W	14W,A Frame Ang, 2" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114155S	FRAME A 0I SET1 15	15W,A Frame Ang, 0" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114154S	FRAME A 0I SET2 15	15W,A Frame Ang, 0" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125243S	FRAME A 0I SET3 15W	15W,A Frame Ang, 0" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114157S	FRAME A 1I SET1 15	15W,A Frame Ang, 1" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114156S	FRAME A 1I SET2 15	15W,A Frame Ang, 1" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125244S	FRAME A 1I SET3 15W	15W,A Frame Ang, 1" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	125245S	FRAME A 2I SET3 15W	15W,A Frame Ang, 2" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114158S	FRAME A 2I SET2 15	15W,A Frame Ang, 2" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	114170S	FRAME A 0I SET1 16	16W,A Frame Ang, 0" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114169S	FRAME A 0I SET2 16	16W,A Frame Ang, 0" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125252S	FRAME A 0I SET3 16W	16W,A Frame Ang, 0" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114172S	FRAME A 1I SET1 16	16W,A Frame Ang, 1" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114171S	FRAME A 1I SET2 16	16W,A Frame Ang, 1" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125253S	FRAME A 1I SET3 16W	16W,A Frame Ang, 1" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114173S	FRAME A 2I SET2 16	16W,A Frame Ang, 2" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125254S	FRAME A 2I SET3 16W	16W,A Frame Ang, 2" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114185S	FRAME A 0I SET1 17	17W,A Frame Ang, 0" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114184S	FRAME A 0I SET2 17	17W,A Frame Ang, 0" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125261S	FRAME A 0I SET3 17W	17W,A Frame Ang, 0" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114187S	FRAME A 1I SET1 17	17W,A Frame Ang, 1" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114186S	FRAME A 1I SET2 17	17W,A Frame Ang, 1" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125262S	FRAME A 1I SET3 17W	17W,A Frame Ang, 1" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114188S	FRAME A 2I SET2 17	17W,A Frame Ang, 2" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125263S	FRAME A 2I SET3 17W	17W,A Frame Ang, 2" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114200S	FRAME A 0I SET1 18	18W,A Frame Ang, 0" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114199S	FRAME A 0I SET2 18	18W,A Frame Ang, 0" Frame Inset, Caster setting 2 - STD Seat Sling

QUICKIE Q7 PAGE 11

QUICKIE Q7

Pos.	Item Number	Description	Remarks
1-7A,8,9	114167S	FRAME C 1I SET1 15	15W,C Frame Ang, 1" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114166S	FRAME C 1I SET2 15	15W,C Frame Ang, 1" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125250S	FRAME C 1I SET3 15W	15W,C Frame Ang, 1" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114168S	FRAME C 2I SET2 15	15W,C Frame Ang, 2" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125251S	FRAME C 2I SET3 15W	15W,C Frame Ang, 2" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114180S	FRAME C 0I SET1 16	16W,C Frame Ang, 0" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114179S	FRAME C 0I SET2 16	16W,C Frame Ang, 0" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125258S	FRAME C 0I SET3 16W	16W,C Frame Ang, 0" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114182S	FRAME C 1I SET1 16	16W,C Frame Ang, 1" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114181S	FRAME C 1I SET2 16	16W,C Frame Ang, 1" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125259S	FRAME C 1I SET3 16W	16W,C Frame Ang, 1" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114183S	FRAME C 2I SET2 16	16W,C Frame Ang, 2" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125260S	FRAME C 2I SET3 16W	16W,C Frame Ang, 2" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114195S	FRAME C 0I SET1 17	17W,C Frame Ang, 0" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114194S	FRAME C 0I SET2 17	17W,C Frame Ang, 0" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125267S	FRAME C 0I SET3 17W	17W,C Frame Ang, 0" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114197S	FRAME C 1I SET1 17	17W,C Frame Ang, 1" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114196S	FRAME C 1I SET2 17	17W,C Frame Ang, 1" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125268S	FRAME C 1I SET3 17W	17W,C Frame Ang, 1" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114198S	FRAME C 2I SET2 17	17W,C Frame Ang, 2" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125269S	FRAME C 2I SET3 17W	17W,C Frame Ang, 2" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114210S	FRAME C 0I SET1 18	18W,C Frame Ang, 0" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114209S	FRAME C 0I SET2 18	18W,C Frame Ang, 0" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125135S	FRAME C 0I SET3 18W	18W,C Frame Ang, 0" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114212S	FRAME C 1I SET1 18	18W,C Frame Ang, 1" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114211S	FRAME C 1I SET2 18	18W,C Frame Ang, 1" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125136S	FRAME C 1I SET3 18W	18W,C Frame Ang, 1" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114213S	FRAME C 2I SET2 18	18W,C Frame Ang, 2" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125137S	FRAME C 2I SET3 18W	18W,C Frame Ang, 2" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114225S	FRAME C 0I SET1 19	19W,C Frame Ang, 0" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114224S	FRAME C 0I SET2 19	19W,C Frame Ang, 0" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125144S	FRAME C 0I SET3 19W	19W,C Frame Ang, 0" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114227S	FRAME C 1I SET1 19	19W,C Frame Ang, 1" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114226S	FRAME C 1I SET2 19	19W,C Frame Ang, 1" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125145S	FRAME C 1I SET3 19W	19W,C Frame Ang, 1" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114228S	FRAME C 2I SET2 19	19W,C Frame Ang, 2" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125146S	FRAME C 2I SET3 19W	19W,C Frame Ang, 2" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114240S	FRAME C 0I SET1 20	20W,C Frame Ang, 0" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114239S	FRAME C 0I SET2 20	20W,C Frame Ang, 0" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125153S	FRAME C 0I SET3 20W	20W,C Frame Ang, 0" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114242S	FRAME C 1I SET1 20	20W,C Frame Ang, 1" Frame Inset, Caster setting 1 - STD Seat Sling
1-7A,8,9	114241S	FRAME C 1I SET2 20	20W,C Frame Ang, 1" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125154S	FRAME C 1I SET3 20W	20W,C Frame Ang, 1" Frame Inset, Caster setting 3 - STD Seat Sling
1-7A,8,9	114243S	FRAME C 2I SET2 20	20W,C Frame Ang, 2" Frame Inset, Caster setting 2 - STD Seat Sling
1-7A,8,9	125155S	FRAME C 2I SET3 20W	20W,C Frame Ang, 2" Frame Inset, Caster setting 3 - STD Seat Sling

Note: Frame design update, please order replacement frame from "STANDARD ADJUSTABLE FRAME after S/N R4-002330" page. See Frame Replacement Guide for additional parts needed.


NOTE: See Back Frame pages for Depth Adjustable Back or Padded Swing-Away Armrest inserts.

NOTE: To ensure order accuracy Please use the PART FINDER by SERIAL NUMBER to get the correct replacement Frame for the Q7.

NOTE: For growing the Q7 frame please contact customer service to ensure proper frame is ordered.

NOTE: Frame replacements for chairs prior to R4-011176 may need item #118125. Pivot Pin Replacement Kit (Qty 1) if not previously upgraded.

When replacing Frame see "Replacement Frame Guide" in the Information Panel
*Additional components needed

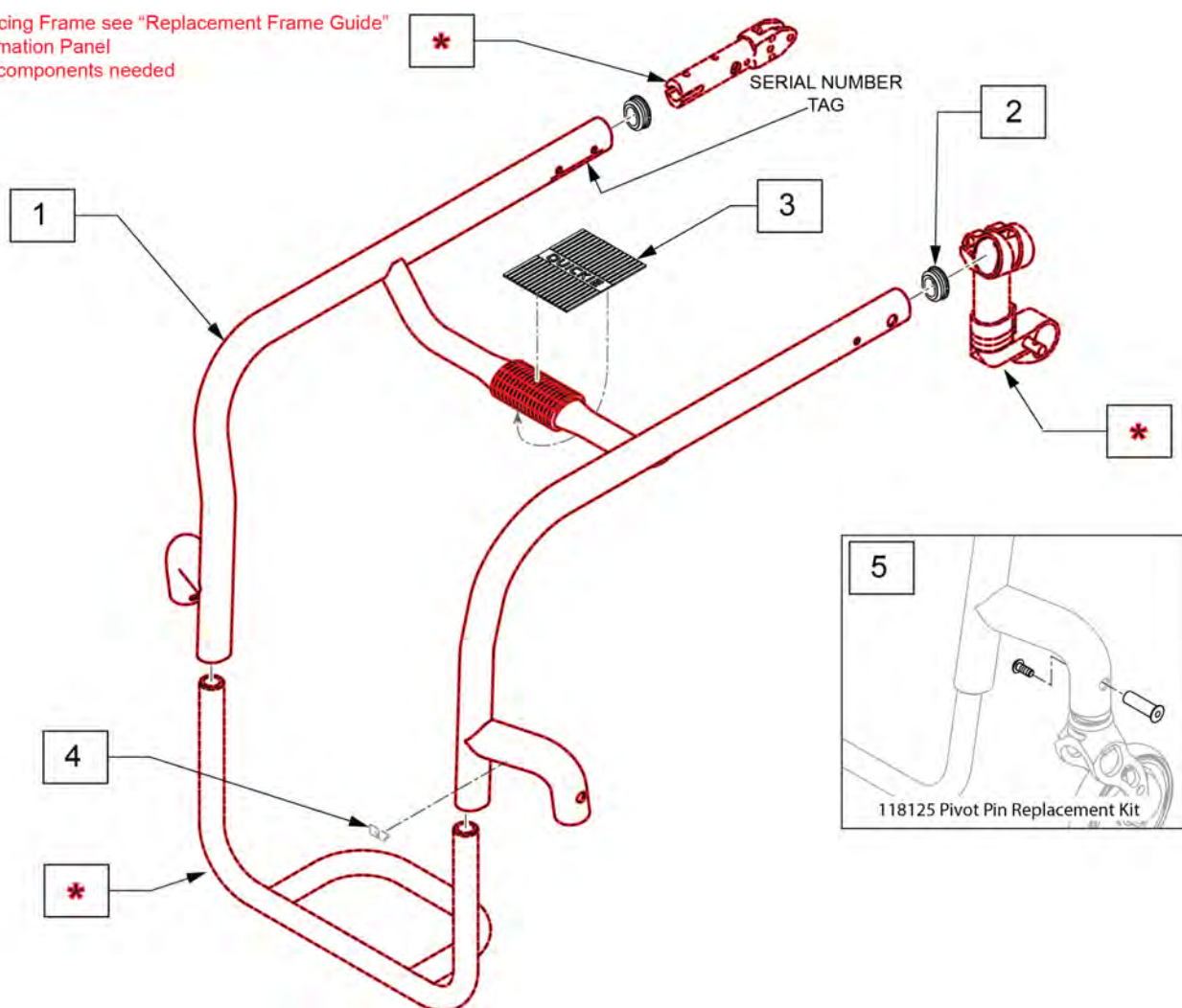

Pos.	Item Number	Description	Remarks
2	110161	FRAME INSERT, REAR SEAT TUBE	
3	112310	GRIP FRAME CROSS BAR	
4	113355	DECAL QUICKIE Q7 CAST BLACK	
4	113356	DECAL QUICKIE Q7 CAST SILVER	
5	118125	PIVOT PIN REPLACEMENT KIT	Pair


NOTE: When ordering replacement frames you must provide the seat depth, frame style, front seat height, frame color and if the chair was equipped with Depth Adjustable Back or Ultra Light Wheel Lock.

NOTE: See Back Frame pages for Depth Adjustable Back or Padded Swing-Away Armrest inserts.

NOTE: To ensure order accuracy Please use the PART FINDER by SERIAL NUMBER to get the correct replacement Frame for the Q7.

NOTE: For growing the Q7 frame please contact customer service to ensure proper frame is ordered.

NOTE: Frame replacements for chairs prior to R4-011176 may need item #118125. Pivot Pin Replacement Kit (Qty 1) if not previously upgraded.

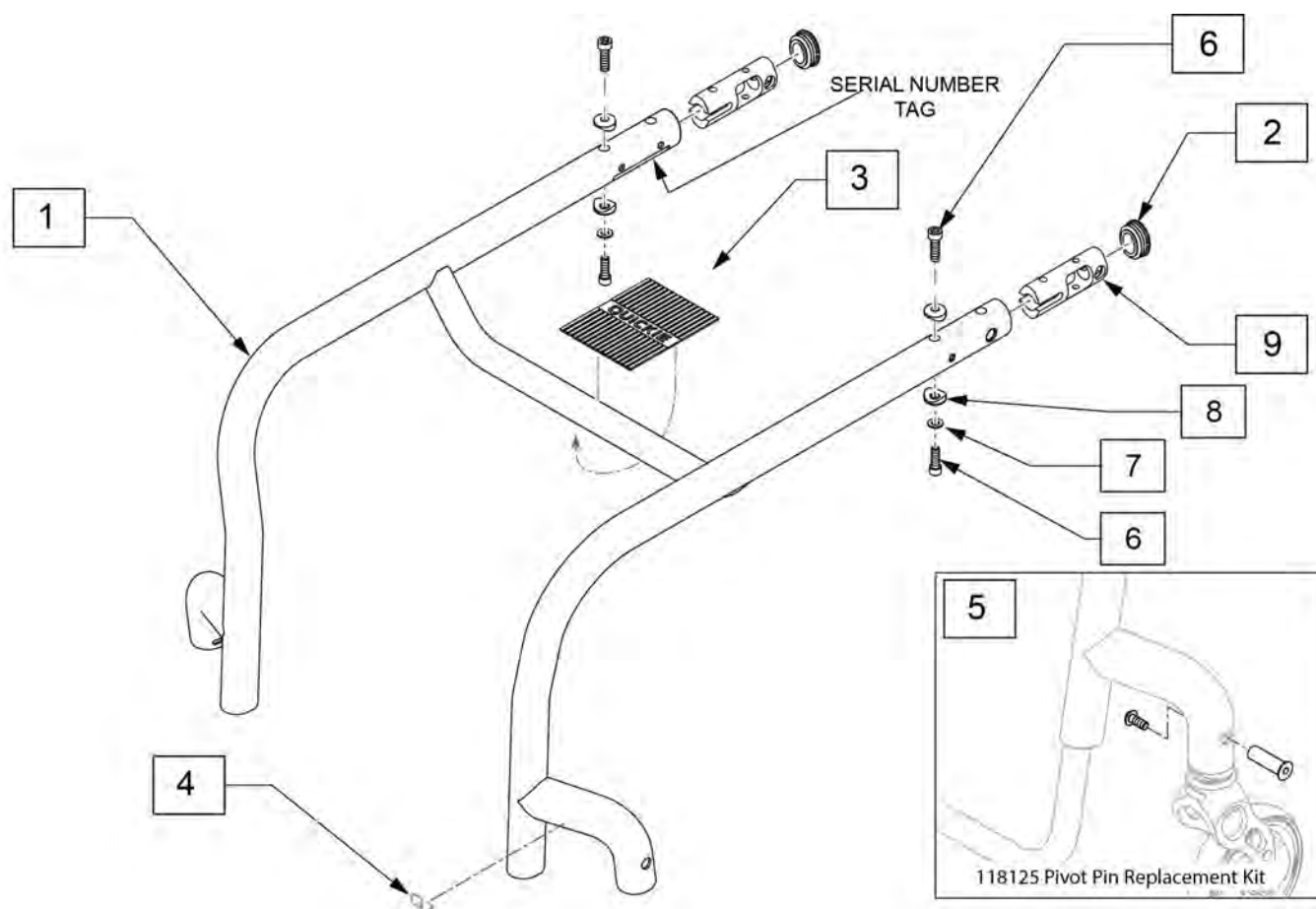


Pos.	Item Number	Description	Remarks
2	110161	FRAME INSERT, REAR SEAT TUBE	
3	128109	ROUND GRIP FRAME CROSS BAR	
4	113355	DECAL QUICKIE Q7 CAST BLACK	

Pos.	Item Number	Description	Remarks
4	113356	DECAL QUICKIE Q7 CAST SILVER	
5	118125	PIVOT PIN REPLACEMENT KIT	Pair
6	102263-005	SHCS, M6X1X20, FULL, PL	
7	250104	WASHER FLAT M6	
8	482000	SADDLE, SMALL	
9	126803	STANDARD FRAME INSERT OVAL	
		"A" FRAME ANGLE	
1-4	114245S	FRM A W/O RIV 0I SET1 12	Ultra lite seat sling only-12W,A Frame Ang, 0" Frame Inset, Caster setting 1
1-4	114244S	FRM A W/O RIV 0I SET2 12	Ultra lite seat sling only-12W,A Frame Ang, 0" Frame Inset, Caster setting 2
1-4	125156S	FRM A W/O RIV 0I SET3 12W	Ultra lite seat sling only-12W,A Frame Ang, 0" Frame Inset, Caster setting 3
1-4	114247S	FRM A W/O RIV 1I SET1 12	Ultra lite seat sling only-12W,A Frame Ang, 1" Frame Inset, Caster setting 1
1-4	114246S	FRM A W/O RIV 1I SET2 12	Ultra lite seat sling only-12W,A Frame Ang, 1" Frame Inset, Caster setting 2
1-4	125157S	FRM A W/O RIV 1I SET3 12W	Ultra lite seat sling only-12W,A Frame Ang, 1" Frame Inset, Caster setting 3
1-4	114257S	FRM A W/O RIV 0I SET1 13	Ultra lite seat sling only-13W,A Frame Ang, 0" Frame Inset, Caster setting 1
1-4	114256S	FRM A W/O RIV 0I SET2 13	Ultra lite seat sling only-13W,A Frame Ang, 0" Frame Inset, Caster setting 2
1-4	125162S	FRM A W/O RIV 0I SET3 13W	Ultra lite seat sling only-13W,A Frame Ang, 0" Frame Inset, Caster setting 3
1-4	114259S	FRM A W/O RIV 1I SET1 13	Ultra lite seat sling only-13W,A Frame Ang, 1" Frame Inset, Caster setting 1
1-4	114258S	FRM A W/O RIV 1I SET2 13	Ultra lite seat sling only-13W,A Frame Ang, 1" Frame Inset, Caster setting 2
1-4	125163S	FRM A W/O RIV 1I SET3 13W	Ultra lite seat sling only-13W,A Frame Ang, 1" Frame Inset, Caster setting 3
1-4	114269S	FRM A W/O RIV 0I SET1 14	Ultra lite seat sling only-14W,A Frame Ang, 0" Frame Inset, Caster setting 1
1-4	114268S	FRM A W/O RIV 0I SET2 14	Ultra lite seat sling only-14W,A Frame Ang, 0" Frame Inset, Caster setting 2
1-4	125168S	FRM A W/O RIV 0I SET3 14W	Ultra lite seat sling only-14W,A Frame Ang, 0" Frame Inset, Caster setting 3
1-4	114271S	FRM A W/O RIV 1I SET1 14	Ultra lite seat sling only-14W,A Frame Ang, 1" Frame Inset, Caster setting 1
1-4	114270S	FRM A W/O RIV 1I SET2 14	Ultra lite seat sling only-14W,A Frame Ang, 1" Frame Inset, Caster setting 2
1-4	125169S	FRM A W/O RIV 1I SET3 14W	Ultra lite seat sling only-14W,A Frame Ang, 1" Frame Inset, Caster setting 3
1-4	114272S	FRM A W/O RIV 2I SET2 14	Ultra lite seat sling only-14W,A Frame Ang, 2" Frame Inset, Caster setting 2
1-4	125170S	FRM A W/O RIV 2I SET3 14W	Ultra lite seat sling only-14W,A Frame Ang, 2" Frame Inset, Caster setting 3
1-4	114284S	FRM A W/O RIV 0I SET1 15	Ultra lite seat sling only-15W,A Frame Ang, 0" Frame Inset, Caster setting 1
1-4	114283S	FRM A W/O RIV 0I SET2 15	Ultra lite seat sling only-15W,A Frame Ang, 0" Frame Inset, Caster setting 2
1-4	125177S	FRM A W/O RIV 0I SET3 15W	Ultra lite seat sling only-15W,A Frame Ang, 0" Frame Inset, Caster setting 3
1-4	114286S	FRM A W/O RIV 1I SET1 15	Ultra lite seat sling only-15W,A Frame Ang, 1" Frame Inset, Caster setting 1
1-4	114285S	FRM A W/O RIV 1I SET2 15	Ultra lite seat sling only-15W,A Frame Ang, 1" Frame Inset, Caster setting 2
1-4	125178S	FRM A W/O RIV 1I SET3 15W	Ultra lite seat sling only-15W,A Frame Ang, 1" Frame Inset, Caster setting 3
1-4	114287S	FRM A W/O RIV 2I SET2 15	Ultra lite seat sling only-15W,A Frame Ang, 2" Frame Inset, Caster setting 2
1-4	125179S	FRM A W/O RIV 2I SET3 15W	Ultra lite seat sling only-15W,A Frame Ang, 2" Frame Inset, Caster setting 3
1-4	114299S	FRM A W/O RIV 0I SET1 16	Ultra lite seat sling only-16W,A Frame Ang, 0" Frame Inset, Caster setting 1
1-4	114298S	FRM A W/O RIV 0I SET2 16	Ultra lite seat sling only-16W,A Frame Ang, 0" Frame Inset, Caster setting 2
1-4	125186S	FRM A W/O RIV 0I SET3 16W	Ultra lite seat sling only-16W,A Frame Ang, 0" Frame Inset, Caster setting 3
1-4	114301S	FRM A W/O RIV 1I SET1 16	Ultra lite seat sling only-16W,A Frame Ang, 1" Frame Inset, Caster setting 1
1-4	114300S	FRM A W/O RIV 1I SET2 16	Ultra lite seat sling only-16W,A Frame Ang, 1" Frame Inset, Caster setting 2
1-4	125187S	FRM A W/O RIV 1I SET3 16W	Ultra lite seat sling only-16W,A Frame Ang, 1" Frame Inset, Caster setting 3
1-4	114314S	FRM A W/O RIV 0I SET1 17	Ultra lite seat sling only-17W,A Frame Ang, 0" Frame Inset, Caster setting 1
1-4	114313S	FRM A W/O RIV 0I SET2 17	Ultra lite seat sling only-17W,A Frame Ang, 0" Frame Inset, Caster setting 2
1-4	125195S	FRM A W/O RIV 0I SET3 17W	Ultra lite seat sling only-17W,A Frame Ang, 0" Frame Inset, Caster setting 3
1-4	114316S	FRM A W/O RIV 1I SET1 17	Ultra lite seat sling only-17W,A Frame Ang, 1" Frame Inset, Caster setting 1
1-4	114315S	FRM A W/O RIV 1I SET2 17	Ultra lite seat sling only-17W,A Frame Ang, 1" Frame Inset, Caster setting 2
1-4	125196S	FRM A W/O RIV 1I SET3 17W	Ultra lite seat sling only-17W,A Frame Ang, 1" Frame Inset, Caster setting 3
1-4	114317S	FRM A W/O RIV 2I SET2 17	Ultra lite seat sling only-17W,A Frame Ang, 2" Frame Inset, Caster setting 2
1-4	125197S	FRM A W/O RIV 2I SET3 17W	Ultra lite seat sling only-17W,A Frame Ang, 2" Frame Inset, Caster setting 3
1-4	114329S	FRM A W/O RIV 0I SET1 18	Ultra lite seat sling only-18W,A Frame Ang, 0" Frame Inset, Caster setting 1
1-4	114328S	FRM A W/O RIV 0I SET2 18	Ultra lite seat sling only-18W,A Frame Ang, 0" Frame Inset, Caster setting 2
1-4	125204S	FRM A W/O RIV 0I SET3 18W	Ultra lite seat sling only-18W,A Frame Ang, 0" Frame Inset, Caster setting 3
1-4	114331S	FRM A W/O RIV 1I SET1 18	Ultra lite seat sling only-18W,A Frame Ang, 1" Frame Inset, Caster setting 1
1-4	114330S	FRM A W/O RIV 1I SET2 18	Ultra lite seat sling only-18W,A Frame Ang, 1" Frame Inset, Caster setting 2
1-4	125205S	FRM A W/O RIV 1I SET3 18W	Ultra lite seat sling only-18W,A Frame Ang, 1" Frame Inset, Caster setting 3
1-4	114332S	FRM A W/O RIV 2I SET2 18	Ultra lite seat sling only-18W,A Frame Ang, 2" Frame Inset, Caster setting 2
1-4	125206S	FRM A W/O RIV 2I SET3 18W	Ultra lite seat sling only-18W,A Frame Ang, 2" Frame Inset, Caster setting 3

QUICKIE Q7


QUICKIE Q7

Note: Frame not available as replacement part; for warranty replacement see Customer Service. CS see warranty matrix in PRISM.

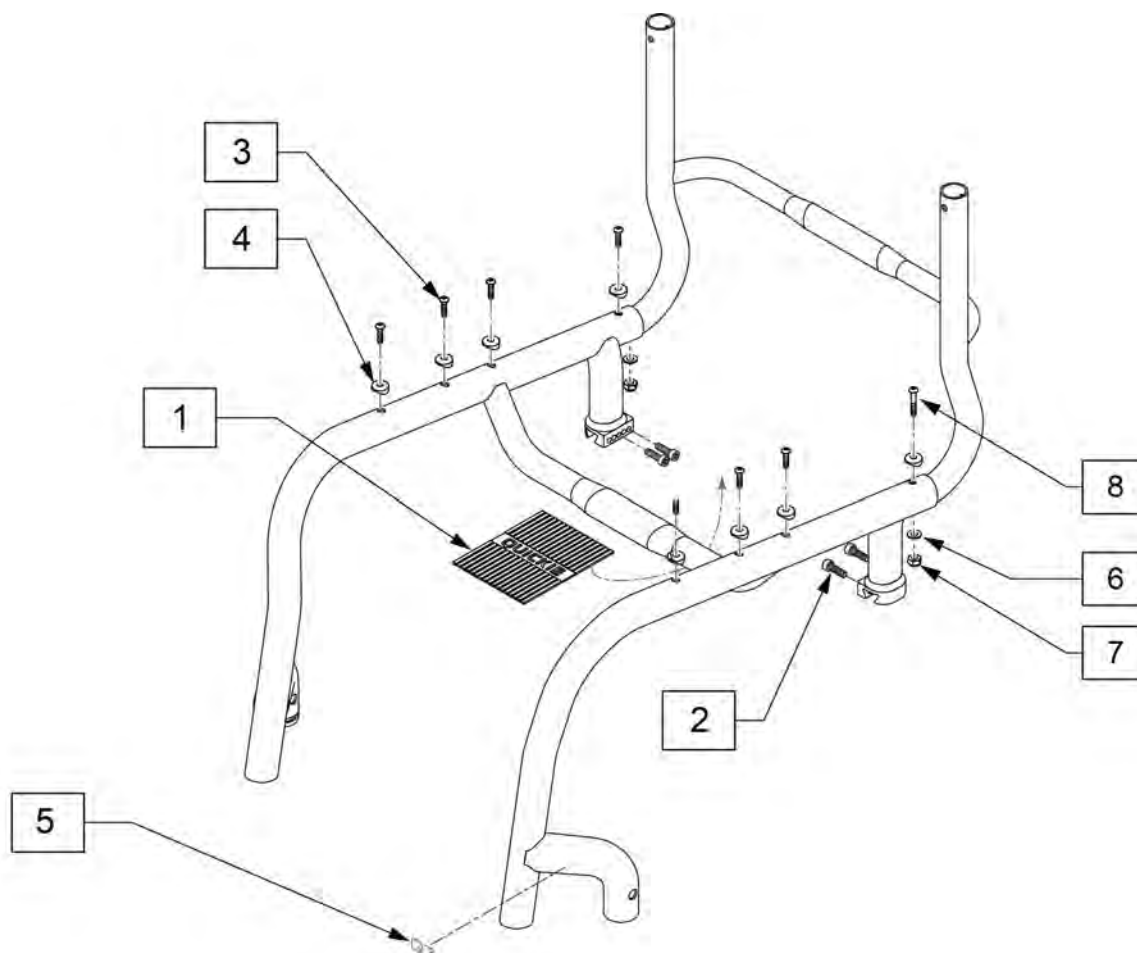


Pos.	Item Number	Description
1	112310	GRIP FRAME CROSS BAR
2	113317	SHCS M5X0.8X25 FULL PL
3	482000	SADDLE, SMALL
4	102259-002	BHCS, M5 X 20
5	113355	DECAL QUICKIE Q7 CAST BLACK
5	113356	DECAL QUICKIE Q7 CAST SILVER
6	124956	WASHER M5
7	220121	NUT HEX NYLOCK M5 BLACK
8	122916	BHCSM5X0.8X50FULLPL

Note: Frame not available as replacement part; for warranty replacement see Customer Service. CS see warranty matrix in PRISM.

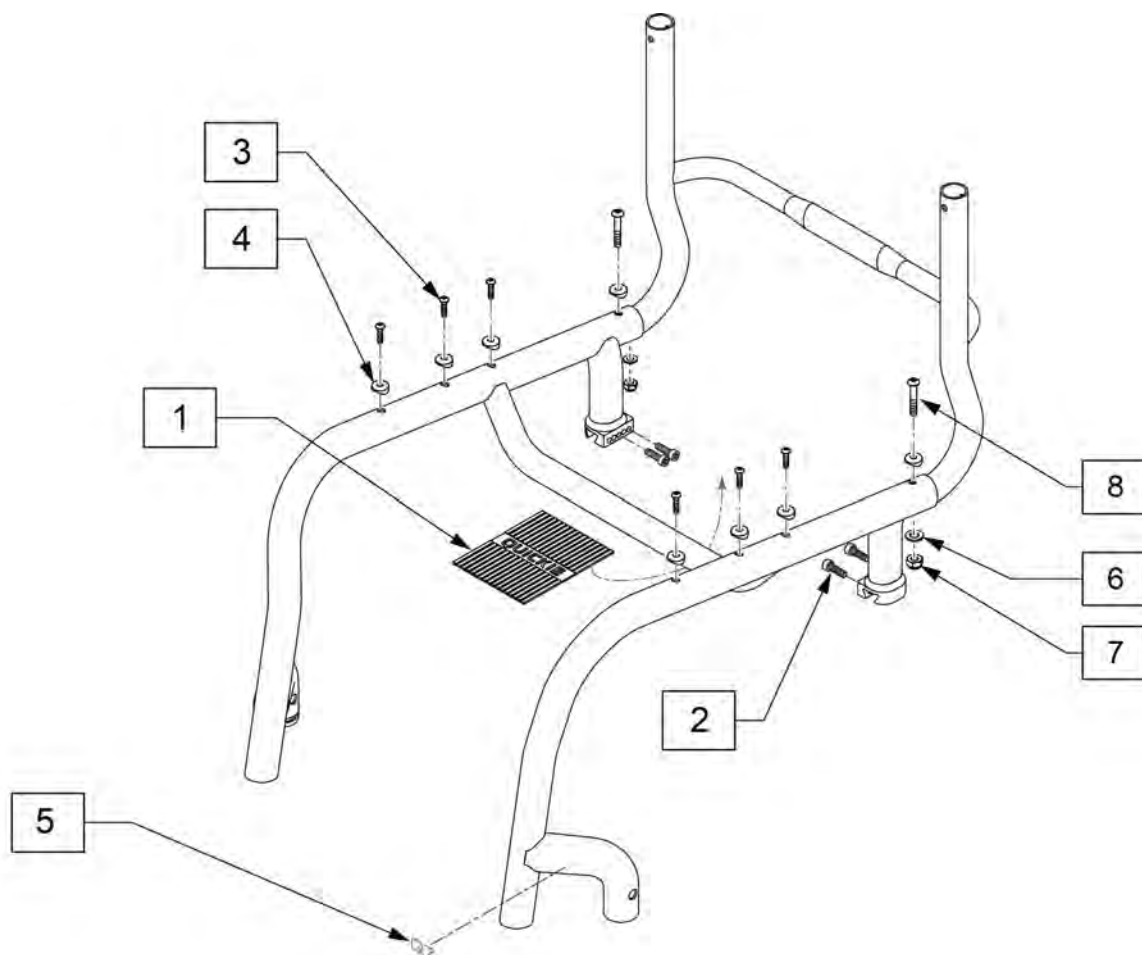


Pos.	Item Number	Description
1	128109	ROUND GRIP FRAME CROSS BAR
2	113317	SHCS M5X0.8X25 FULL PL
3	102259-002	BHCS, M5 X 20
4	482000	SADDLE, SMALL
5	113355	DECAL QUICKIE Q7 CAST BLACK
5	113356	DECAL QUICKIE Q7 CAST SILVER
6	250104	WASHER FLAT M6
7	104381	NUT HEX NYLOCK M6X1.0 INORG
8	102262-008	BHCS,M6X1X50,PART,NP

NOTE: Transit option not available with Xtender.

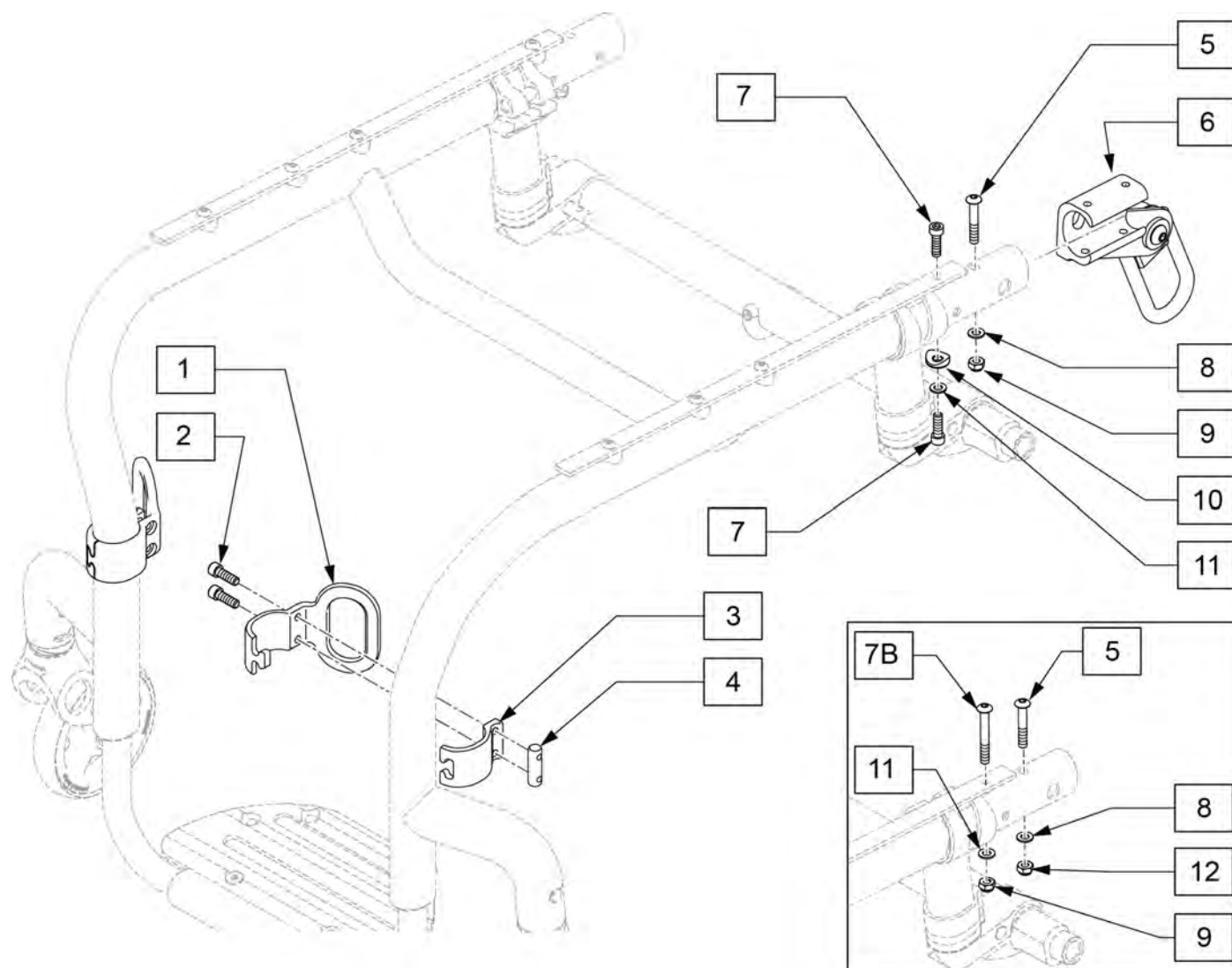

Pos.	Item Number	Description	Remarks
2	112341	SHCS M5X0.8X16 FULL PL	
5	122916	BHCSM5X0.8X50FULLPL	
7A	102263-005	SHCS, M6 x 1 x 20, FULL, P	
7B	102262-008	BHCS,M6X1X50,PART,NP	Used w/ Padded S/A arms & DAB
8	124956	WASHER M5	
9	220121	NUT HEX NYLOCK M5 BLACK	
10	482000	SADDLE SMALL 1.00 TUBE .25 HOLE	
11	250104	WASHER FLAT M6	
1-11	125329	TRANSIT KIT Q7 WC19	
12	104381	NUT HEX NYLOCK M6X1.0 INORG	


Note: Transit Kit is only retrofitable on chairs built prior to S/N R4-023230; if you replace the original frame with the after S/N R4-023230 frame, please see Frame Replacement Guide in the Informations Panel and order the after WC-19 Transit Tie Down Kit p/n 125329.

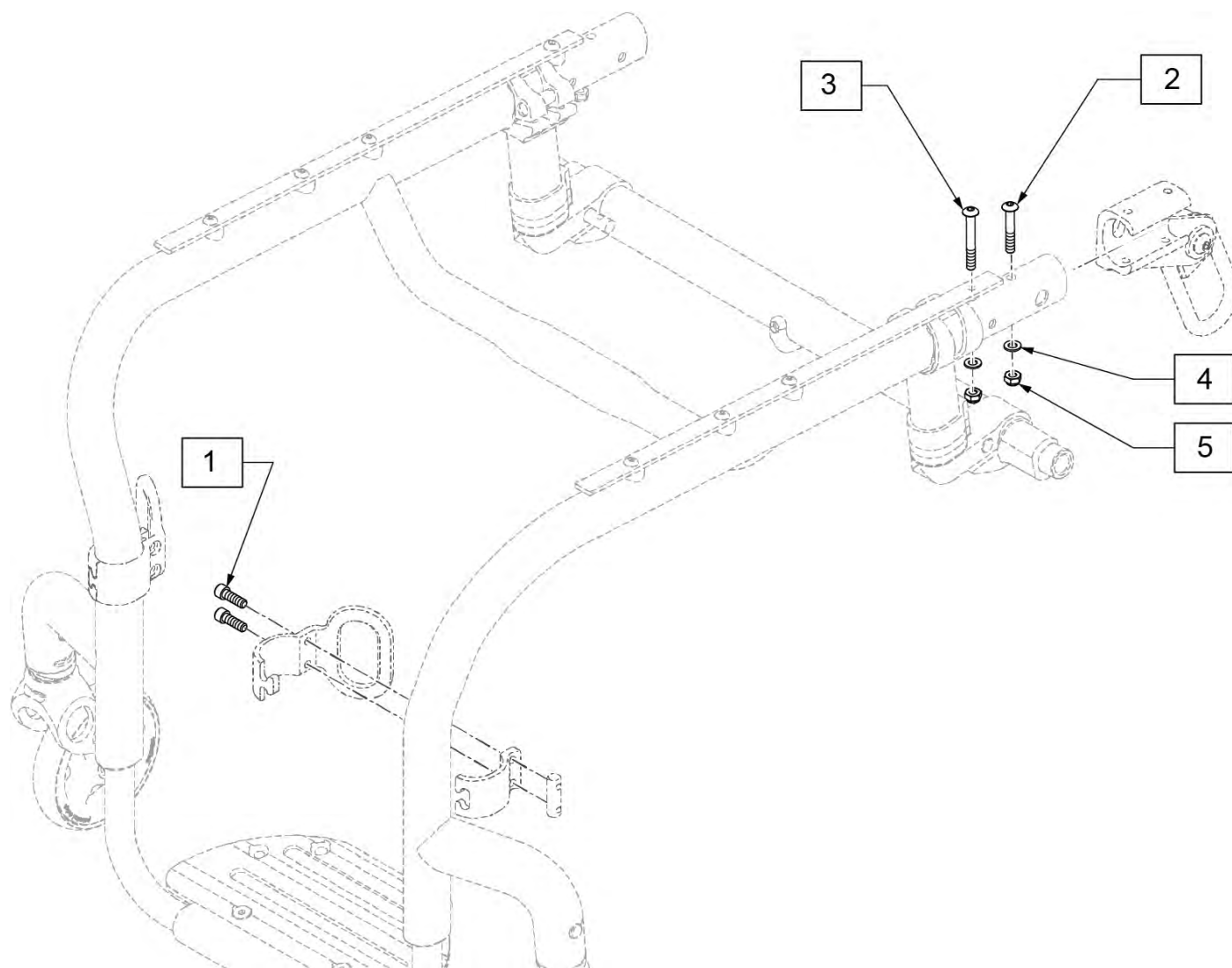

Pos.	Item Number	Description
1	112341	SHCS M5X0.8X16 FULL PL
2	122916	BHCSM5X0.8X50FULLPL
3	122917	BHCSM5X0.8X55FULLPL
4	124956	WASHER M5
5	220121	NUT HEX NYLOCK M5 BLACK


Note: For chairs with S/N prior to R4-023230 no individual parts are available, please order complete Axle Plate assembly.

NOTE: When changing the chairs REAR SEAT to Floor Height, refer to Dimension "C" below. If 'increasing' or 'decreasing' the height determine the current length "C" and then move up or down the chart until you 'increase' or 'decrease' to get to the desired height. Then order the corresponding Axle Plate size. EXAMPLE: If the dimension "C" was 3.70" and I wanted to increase the Rear Seat to Floor height by 2", I would find a 5.70" dimension "C" which would correspond to a number 8 Axle Plate and utilize 1 spacer.

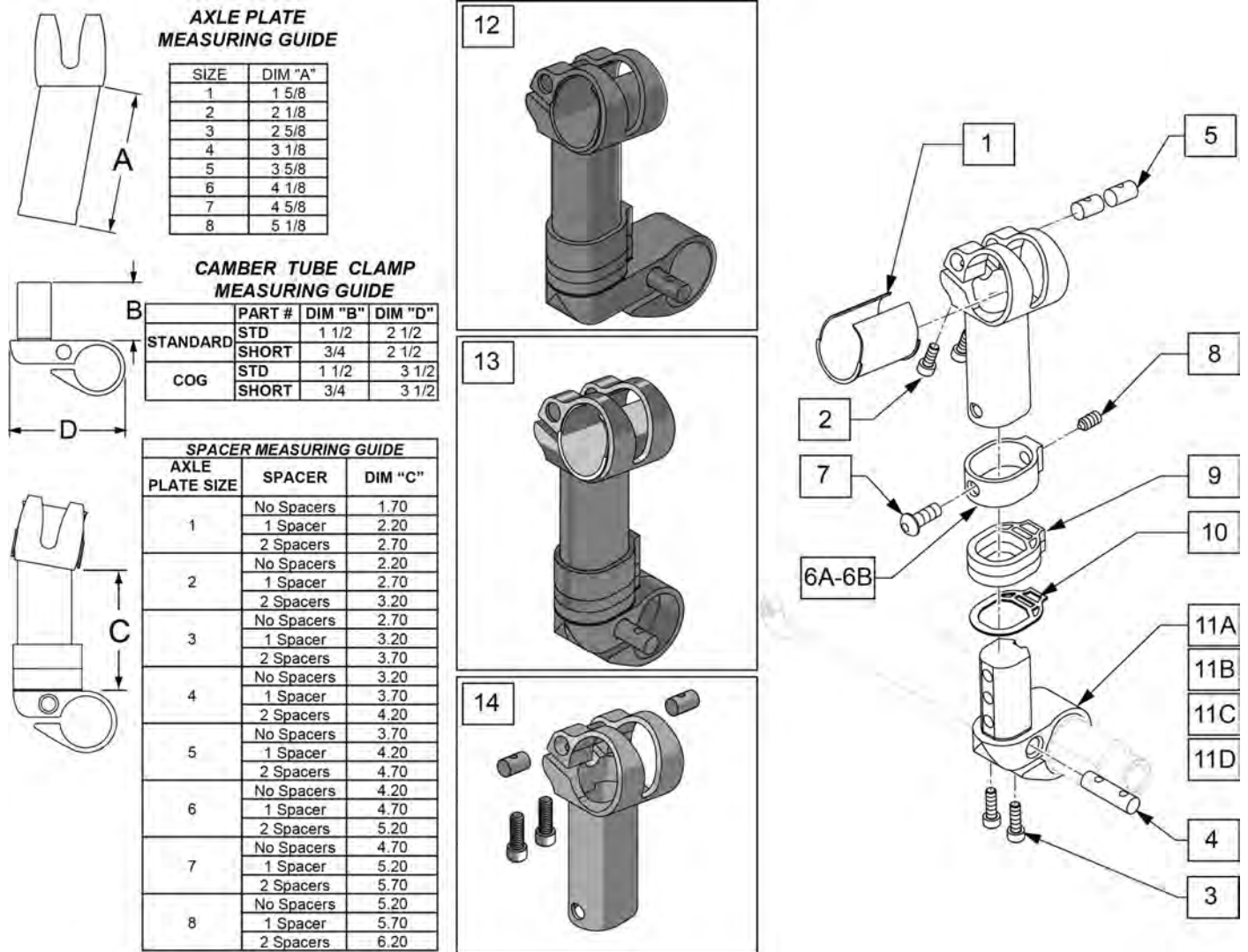


Pos.	Item Number	Description	Remarks
1	066505	AXLE STEM SLEEVE	
2	103181	SHCS M5 x 0.8 x 14 w/ PL	
3	112341	SHCS M5X0.8X16,FULL,PL	
4	112655	ROUND NUT SIZE 4	
5	112653	ROUND NUT SIZE 2	
6A,7,8	114382	AXLE TUBE CLAMP W/SCRW (PR)	Pair - black
6B	117171S	AXLE TUBE SPCR CSTM ANO	Each-Choose custom anodized color
7	230113	BHCS M6 X 1 X 12	

Pos.	Item Number	Description	Remarks
8	102267	SET SCREW M8 X 16 DOG PT	
9	126800	AXLE TUBE SPACER	See note. Not used with short camber tube clamp
10	126801	AXLE TUBE SPACER, THIN	This part used in all configurations
11A	126787	CAMBER TUBE CLAMP	See note.
11B	126788	CAMBER TUBE CLAMP SHORT	See note. Short, only used w/ axle plate 1
11C	126789	CAMBER CLAMP COG	See note.
11D	126790	CAMBER CLAMP COG SHORT	See note. Short, only used w/ axle plate 1
3,4,11A	114399	CAMBER TUBE CLAMP STD KIT	Standard
3,4,11B	114400	CAMBER TUBE CLAMP SHT KIT	Short, only used with Axle Plate 1
3,4,11C	125786	EXTENDED COG CAMBER CLAMP STD	
3,4,11D	125787	EXTENDED COG CAMBER CLAMP SHT	Short, only used w/ axle plate 1
12	128245	AXLE PLATE 1COG KIT Q7	Pair-COG Clamp-Axle Plate 1; Extended Axle Plate
12	128246	AXLE PLATE 2 COG KIT Q7	Pair-COG Clamp-Axle Plate 2; Extended Axle Plate
12	128247	AXLE PLATE 3 COG KIT Q7	Pair-COG Clamp-Axle Plate 3; Extended Axle Plate
12	128248	AXLE PLATE 4 COG KIT Q7	Pair-COG Clamp-Axle Plate 4; Extended Axle Plate
12	128249	AXLE PLATE 5 COG KIT Q7	Pair-COG Clamp-Axle Plate 5; Extended Axle Plate
12	128250	AXLE PLATE 6 COG KIT Q7	Pair-COG Clamp-Axle Plate 6; Extended Axle Plate
12	128251	AXLE PLATE 7 COG KIT Q7	Pair-COG Clamp-Axle Plate 7; Extended Axle Plate
12	128252	AXLE PLATE 8 COG KIT Q7	Pair-COG Clamp-Axle Plate 8; Extended Axle Plate
12	128253	AXLE PLATE 1COG SHORT KIT Q7	Pair-COG Short Clamp-Axle Plate 1; Extended Axle Plate
13	128254	AXLE PLATE 1 STD KIT Q7	Pair-Standard Clamp-Axle Plate 1
13	128255	AXLE PLATE 2 STD KIT Q7	Pair-Standard Clamp-Axle Plate 2
13	128256	AXLE PLATE 3 STD KIT Q7	Pair-Standard Clamp-Axle Plate 3
13	128257	AXLE PLATE 4 STD KIT Q7	Pair-Standard Clamp-Axle Plate 4
13	128258	AXLE PLATE 5 STD KIT Q7	Pair-Standard Clamp-Axle Plate 5
13	128259	AXLE PLATE 6 STD KIT Q7	Pair-Standard Clamp-Axle Plate 6
13	128260	AXLE PLATE 7 STD KIT Q7	Pair-Standard Clamp-Axle Plate 7
13	128261	AXLE PLATE 8 STD KIT Q7	Pair-Standard Clamp-Axle Plate 8
13	128262	AXLE PLATE 1 SHT KIT Q7	Pair-Short Clamp-Axle Plate 1
14	128263	AXLE PLATE LH SIZE 1 KIT	Left-Axle Plate Size 1
14	128264	AXLE PLATE RH SIZE 1 KIT	Right-Axle Plate Size 1
14	128265	AXLE PLATE LH SIZE 2 KIT	Left-Axle Plate Size 2
14	128266	AXLE PLATE RH SIZE 2 KIT	Right-Axle Plate Size 2
14	128267	AXLE PLATE LH SIZE 3 KIT	Left-Axle Plate Size 3
14	128268	AXLE PLATE RH SIZE 3 KIT	Right-Axle Plate Size 3
14	128269	AXLE PLATE LH SIZE 4 KIT	Left-Axle Plate Size 4
14	128270	AXLE PLATE RH SIZE 4 KIT	Right-Axle Plate Size 4
14	128271	AXLE PLATE LH SIZE 5 KIT	Left-Axle Plate Size 5
14	128272	AXLE PLATE RH SIZE 5 KIT	Right-Axle Plate Size 5
14	128273	AXLE PLATE LH SIZE 6 KIT	Left-Axle Plate Size 6
14	128274	AXLE PLATE RH SIZE 6 KIT	Right-Axle Plate Size 6
14	128275	AXLE PLATE LH SIZE 7 KIT	Left-Axle Plate Size 7
14	128276	AXLE PLATE RH SIZE 7 KIT	Right-Axle Plate Size 7
14	128277	AXLE PLATE LH SIZE 8 KIT	Left-Axle Plate Size 8
14	128278	AXLE PLATE RH SIZE 8 KIT	Right-Axle Plate Size 8

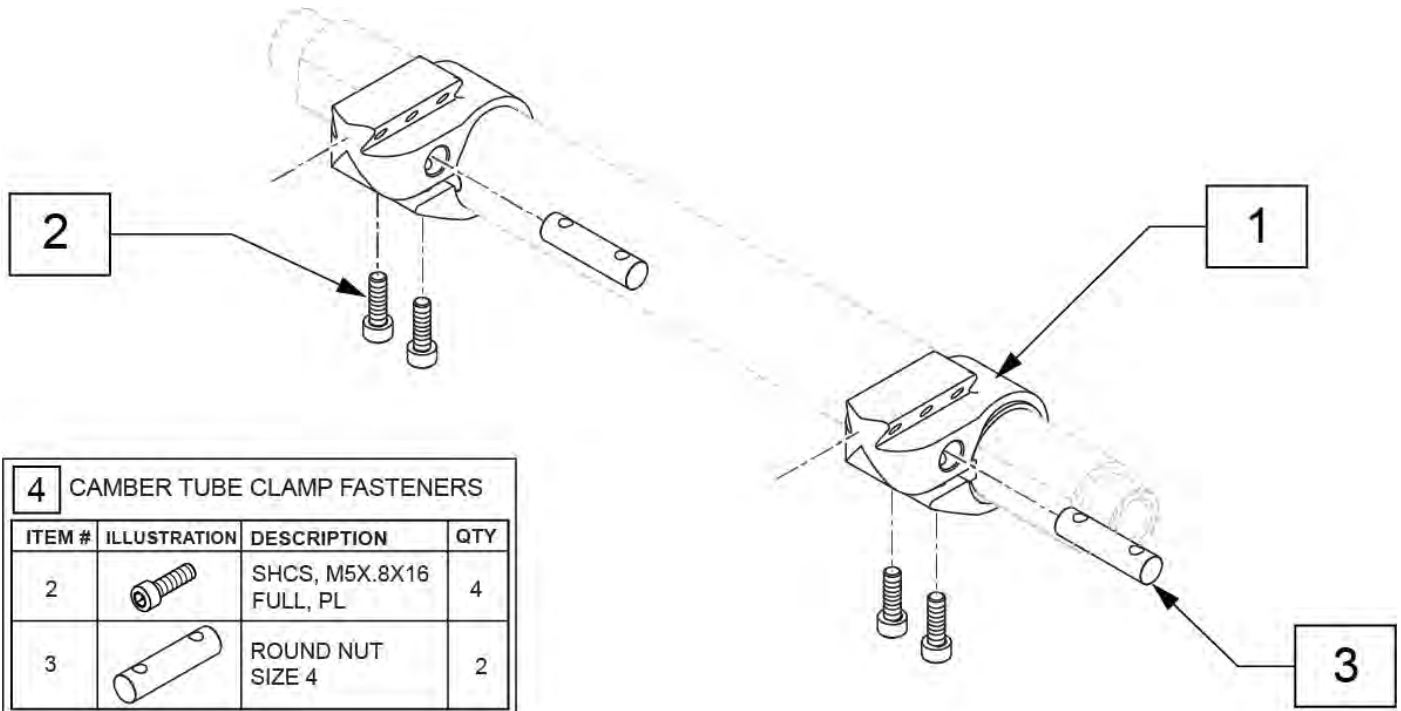

Pos.	Item Number	Description
2	112341	SHCS M5X0.8X16,FULL,PL
3	112655	ROUND NUT SIZE 4
1-3	116684	CAMBER TUBE CLAMP W/HDWR

Note: If adding suspension to a Q7 chair with S/N prior to R4-023230 you must order new Axle Plate Assemblies. If replacing the Suspension Ribs, and New Suspension Insert on a chair with S/N after S/N R4-023230 you must order the proper size Suspension Rib and Insert for the Axle Plate.

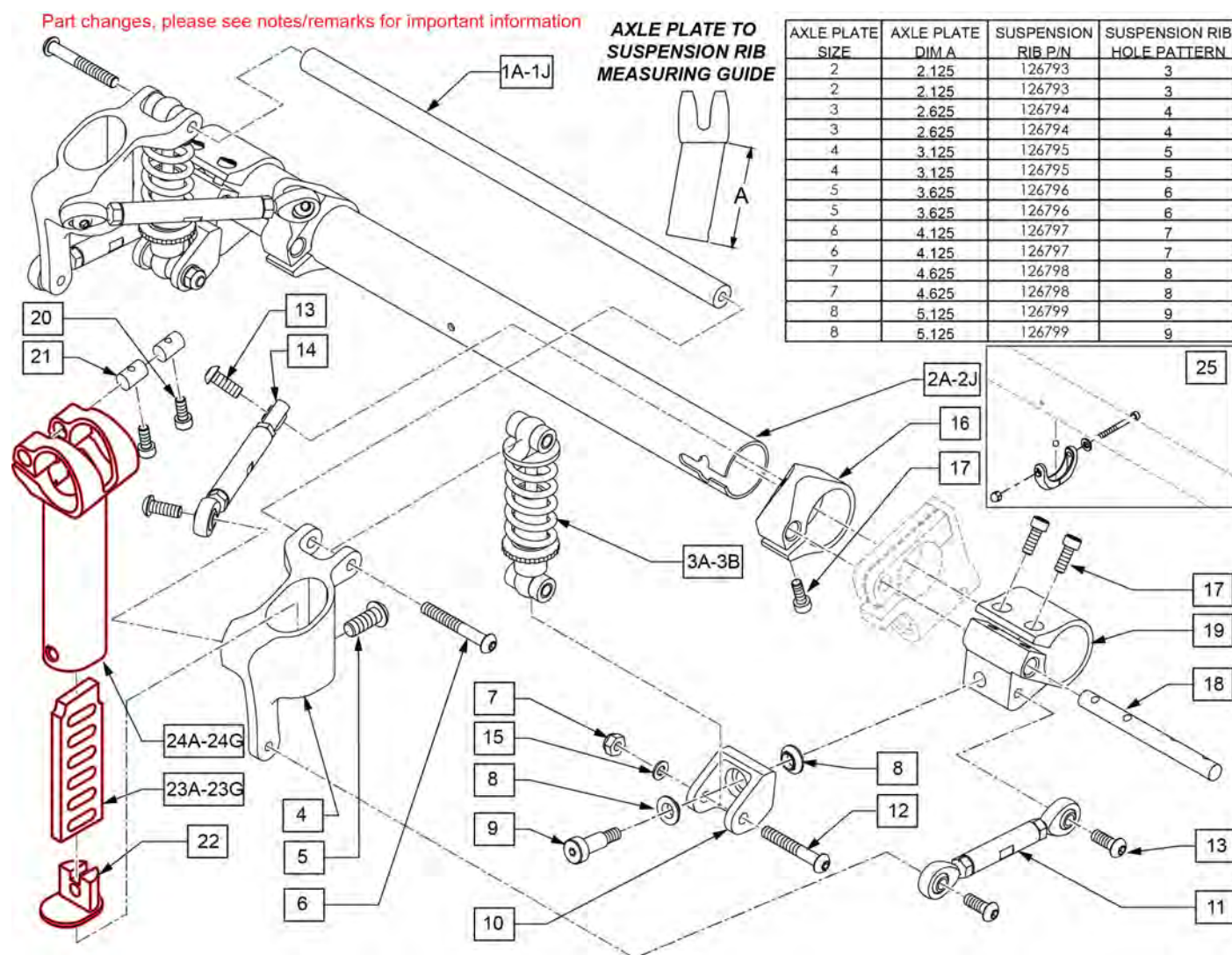






Pos.	Item Number	Description	Remarks
1A	113421	SUSP CROSS BAR 12"	
1B	113422	SUSP CROSS BAR 13"	
1C	113423	SUSP CROSS BAR 14"	
1D	113424	SUSP CROSS BAR 15"	
1E	113425	SUSP CROSS BAR 16"	
1F	113426	SUSP CROSS BAR 17"	
1G	113427	SUSP CROSS BAR 18"	
1H	113428	SUSP CROSS BAR 19"	
1J	113429	SUSP CROSS BAR 20"	
2A	110206	CAMBER TUBE 12"W	
2B	110207	CAMBER TUBE 13"W	
2C	110208	CAMBER TUBE 14"W	
2D	110209	CAMBER TUBE 15"W	
2E	110210	CAMBER TUBE 16"W	

Pos.	Item Number	Description	Remarks
2F	110211	CAMBER TUBE 17"W	
2G	110212	CAMBER TUBE 18"W	
2H	110213	CAMBER TUBE 19"W	
2J	110214	CAMBER TUBE 20"W	
3A	113419	SUSP SHOCK DV-6S 0-145LB	0-175lbs
3B	113420	SUSP SHOCK DV-6S 150LB	176-265lbs
5	230121	BHCS M8 X 1.25 X 20	
6	102262-002	BHCS M6 X 1 X 10 W/PL INORG	
7	104381	NUT HEX NYLOCK M6X1.0	
8	109198	IGUS BEARING GFM-0810-04	
9	011649-001	SHLDR SCREW 8MM OD X 16MM LG	
11	113417	SUSP LINK ARM SHORT	91.4 mm from eyelet to eyelet
12	102262-006	BHCS M6 X 1 X 40 INORGANIC	
13	103185-006	BHCSM6X1X16FULLNP	
14	113418	SUSP LINK ARM LONG LH	Left - 91.4 mm from eyelet to eyelet
14	116603	SUSP LINK ARM LONG RH	Right - 91.4 mm from eyelet to eyelet
15	250104	WASHER FLAT M6	
17	114497	SHCS, M5X0.8X14, FULL PL	
18	113413	SUSP ROUND NUT LINK CONNECT LH	
18	114108	SUSP ROUND NUT LINK CONNECT RH	
20	103181	SHCS M5 x 0.8 x 14 w/ PL	
21	112653	ROUND NUT SIZE 2	
22	126792	SUSP AXLE PLATE INSERT	Order kit if S/N is prior to R4-023230
23A	126793	SUSP RIB SIZE 2	Order kit if S/N is prior to R4-023230
23B	126794	SUSP RIB SIZE 3	Order kit if S/N is prior to R4-023230
23C	126795	SUSP RIB SIZE 4	Order kit if S/N is prior to R4-023230
23D	126796	SUSP RIB SIZE 5	Order kit if S/N is prior to R4-023230
23E	126797	SUSP RIB SIZE 6	Order kit if S/N is prior to R4-023230
23F	126798	SUSP RIB SIZE 7	Order kit if S/N is prior to R4-023230
23G	126799	SUSP RIB SIZE 8	Order kit if S/N is prior to R4-023230
25	050190	BEARING LEVEL CAMBER TUBE	Pair
20,21,24A	114385	AXLE PLATE LH SIZE 2 KIT	Left-Axle Plate Size 2
20,21,24A	114386	AXLE PLATE RH SIZE 2 KIT	Right-Axle Plate Size 2
20,21,24B	114387	AXLE PLATE LH SIZE 3 KIT	Left-Axle Plate Size 3
20,21,24B	114388	AXLE PLATE RH SIZE 3 KIT	Right-Axle Plate Size 3
20,21,24C	114389	AXLE PLATE LH SIZE 4 KIT	Left-Axle Plate Size 4
20,21,24C	114390	AXLE PLATE RH SIZE 4 KIT	Right-Axle Plate Size 4
20,21,24D	114391	AXLE PLATE LH SIZE 5 KIT	Left-Axle Plate Size 5
20,21,24D	114392	AXLE PLATE RH SIZE 5 KIT	Right-Axle Plate Size 5
20,21,24E	114393	AXLE PLATE LH SIZE 6 KIT	Left-Axle Plate Size 6
20,21,24E	114394	AXLE PLATE RH SIZE 6 KIT	Right-Axle Plate Size 6
20,21,24F	114395	AXLE PLATE LH SIZE 7 KIT	Left-Axle Plate Size 7
20,21,24F	114396	AXLE PLATE RH SIZE 7 KIT	Right-Axle Plate Size 7
20,21,24G	114397	AXLE PLATE LH SIZE 8 KIT	Left-Axle Plate Size 8
20,21,24G	114398	AXLE PLATE RH SIZE 8 KIT	Right-Axle Plate Size 8
3A-19	116308	ASSY SUSP 4LINK 100LB RH	
3A-19	116309	ASSY SUSP 4LINK 100LB LH	
3B-19	116310	ASSY SUSP 4LINK 150LB RH	
3B-19	116311	ASSY SUSP 4LINK 150LB LH	
1A,2A,3A-22,23A,24A,25	117698	SUSP ASSM 12" 0-175LBS AP2	
1B,2B,3A-22,23A,24A,25	117699	SUSP ASSM 13" 0-175LBS AP2	
1C,2C,3A-22,23A,24A,25	117700	SUSP ASSM 14" 0-175LBS AP2	
1D,2D,3A-22,23A,24A,25	117701	SUSP ASSM 15" 0-175LBS AP2	
1E,2E,3A-22,23A,24A,25	117702	SUSP ASSM 16" 0-175LBS AP2	
1F,2F,3A-22,23A,24A,25	117703	SUSP ASSM 17" 0-175LBS AP2	
1G,2G,3A-22,23A,24A,25	117704	SUSP ASSM 18" 0-175LBS AP2	
1H,2H,3A-22,23A,24A,25	117705	SUSP ASSM 19" 0-175LBS AP2	


Pos.	Item Number	Description	Remarks
1J,2J,3A-22,23A,24A,25	117706	SUSP ASSM 20" 0-175LBS AP2	
1A,2A,3B-22,23A,24A,25	117707	SUSP ASSM 12" 176-265LBS AP2	
1B,2B,3B-22,23A,24A,25	117708	SUSP ASSM 13" 176-265LBS AP2	
1C,2C,3B-22,23A,24A,25	117709	SUSP ASSM 14" 176-265LBS AP2	
1D,2D,3B-22,23A,24A,25	117710	SUSP ASSM 15" 176-265LBS AP2	
1E,2E,3B-22,23A,24A,25	117711	SUSP ASSM 16" 176-265LBS AP2	
1F,2F,3B-22,23A,24A,25	117712	SUSP ASSM 17" 176-265LBS AP2	
1G,2G,3B-22,23A,24A,25	117713	SUSP ASSM 18" 176-265LBS AP2	
1H,2H,3B-22,23A,24A,25	117714	SUSP ASSM 19" 176-265LBS AP2	
1J,2J,3B-22,23A,24A,25	117715	SUSP ASSM 20" 176-265LBS AP2	
1A,2A,3A-22,23B,24B,25	117716	SUSP ASSM 12" 0-175LBS AP3	
1B,2B,3A-22,23B,24B,25	117717	SUSP ASSM 13" 0-175LBS AP3	
1C,2C,3A-22,23B,24B,25	117718	SUSP ASSM 14" 0-175LBS AP3	
1D,2D,3A-22,23B,24B,25	117719	SUSP ASSM 15" 0-175LBS AP3	
1E,2E,3A-22,23B,24B,25	117720	SUSP ASSM 16" 0-175LBS AP3	
1F,2F,3A-22,23B,24B,25	117721	SUSP ASSM 17" 0-175LBS AP3	
1G,2G,3A-22,23B,24B,25	117722	SUSP ASSM 18" 0-175LBS AP3	
1H,2H,3A-22,23B,24B,25	117723	SUSP ASSM 19" 0-175LBS AP3	
1J,2J,3A-22,23B,24B,25	117724	SUSP ASSM 20" 0-175LBS AP3	
1A,2A,3B-22,23B,24B,25	117725	SUSP ASSM 12" 176-265LBS AP3	
1B,2B,3B-22,23B,24B,25	117726	SUSP ASSM 13" 176-265LBS AP3	
1C,2C,3B-22,23B,24B,25	117727	SUSP ASSM 14" 176-265LBS AP3	
1D,2D,3B-22,23B,24B,25	117728	SUSP ASSM 15" 176-265LBS AP3	
1E,2E,3B-22,23B,24B,25	117729	SUSP ASSM 16" 176-265LBS AP3	
1F,2F,3B-22,23B,24B,25	117730	SUSP ASSM 17" 176-265LBS AP3	
1G,2G,3B-22,23B,24B,25	117731	SUSP ASSM 18" 176-265LBS AP3	
1H,2H,3B-22,23B,24AB,25	117732	SUSP ASSM 19" 176-265LBS AP3	
1J,2J,3B-22,23B,24B,25	117733	SUSP ASSM 20" 176-265LBS AP3	
1A,2A,3A-22,23C,24C,25	117734	SUSP ASSM 12" 0-175LBS AP4	
1B,2B,3A-22,23C,24C,25	117735	SUSP ASSM 13" 0-175LBS AP4	
1C,2C,3A-22,23C,24C,25	117736	SUSP ASSM 14" 0-175LBS AP4	
1D,2D,3A-22,23C,24C,25	117737	SUSP ASSM 15" 0-175LBS AP4	
1E,2E,3A-22,23C,24C,25	117738	SUSP ASSM 16" 0-175LBS AP4	
1F,2F,3A-22,23C,24C,25	117739	SUSP ASSM 17" 0-175LBS AP4	
1G,2G,3A-22,23C,24C,25	117740	SUSP ASSM 18" 0-175LBS AP4	
1H,2H,3A-22,23C,24C,25	117741	SUSP ASSM 19" 0-175LBS AP4	
1J,2J,3A-22,23C,24C,25	117742	SUSP ASSM 20" 0-175LBS AP4	
1A,2A,3B-22,23C,24C,25	117743	SUSP ASSM 12" 176-265LBS AP4	
1B,2B,3B-22,23C,24C,25	117744	SUSP ASSM 13" 176-265LBS AP4	
1C,2C,3B-22,23C,24C,25	117745	SUSP ASSM 14" 176-265LBS AP4	
1D,2D,3B-22,23C,24C,25	117746	SUSP ASSM 15" 176-265LBS AP4	
1E,2E,3B-22,23C,24C,25	117747	SUSP ASSM 16" 176-265LBS AP4	
1F,2F,3B-22,23C,24C,25	117748	SUSP ASSM 17" 176-265LBS AP4	
1G,2G,3B-22,23C,24C,25	117749	SUSP ASSM 18" 176-265LBS AP4	
1H,2H,3B-22,23C,24C,25	117750	SUSP ASSM 19" 176-265LBS AP4	
1J,2J,3B-22,23C,24C,25	117751	SUSP ASSM 20" 176-265LBS AP4	
1A,2A,3A-22,23D,24D,25	117752	SUSP ASSM 12" 0-175LBS AP5	
1B,2B,3A-22,23D,24D,25	117753	SUSP ASSM 13" 0-175LBS AP5	
1C,2C,3A-22,23D,24D,25	117754	SUSP ASSM 14" 0-175LBS AP5	
1D,2D,3A-22,23D,24D,25	117755	SUSP ASSM 15" 0-175LBS AP5	
1E,2E,3A-22,23D,24D,25	117756	SUSP ASSM 16" 0-175LBS AP5	
1F,2F,3A-22,23D,24D,25	117757	SUSP ASSM 17" 0-175LBS AP5	
1G,2G,3A-22,23D,24D,25	117758	SUSP ASSM 18" 0-175LBS AP5	
1H,2H,3A-22,23D,24D,25	117759	SUSP ASSM 19" 0-175LBS AP5	
1J,2J,3A-22,23D,24D,25	117760	SUSP ASSM 20" 0-175LBS AP5	
1A,2A,3B-22,23D,24D,25	117761	SUSP ASSM 12" 176-265LBS AP5	
1B,2B,3B-22,23D,24D,25	117762	SUSP ASSM 13" 176-265LBS AP5	

Pos.	Item Number	Description	Remarks
1C,2C,3B-22,23D,24D,25	117763	SUSP ASSM 14" 176-265LBS AP5	
1D,2D,3B-22,23D,24D,25	117764	SUSP ASSM 15" 176-265LBS AP5	
1E,2E,3B-22,23D,24D,25	117765	SUSP ASSM 16" 176-265LBS AP5	
1F,2F,3B-22,23D,24D,25	117766	SUSP ASSM 17" 176-265LBS AP5	
1G,2G,3B-22,23D,24D,25	117767	SUSP ASSM 18" 176-265LBS AP5	
1H,2H,3B-22,23D,24D,25	117768	SUSP ASSM 19" 176-265LBS AP5	
1J,2J,3B-22,23D,24D,25	117769	SUSP ASSM 20" 176-265LBS AP5	
1A,2A,3A-22,23E,24E,25	117770	SUSP ASSM 12" 0-175LBS AP6	
1B,2B,3A-22,23E,24E,25	117771	SUSP ASSM 13" 0-175LBS AP6	
1C,2C,3A-22,23E,24E,25	117772	SUSP ASSM 14" 0-175LBS AP6	
1D,2D,3A-22,23E,24E,25	117773	SUSP ASSM 15" 0-175LBS AP6	
1E,2E,3A-22,23E,24E,25	117774	SUSP ASSM 16" 0-175LBS AP6	
1F,2F,3A-22,23E,24E,25	117775	SUSP ASSM 17" 0-175LBS AP6	
1G,2G,3A-22,23E,24E,25	117776	SUSP ASSM 18" 0-175LBS AP6	
1H,2H,3A-22,23E,24E,25	117777	SUSP ASSM 19" 0-175LBS AP6	
1J,2J,3A-22,23E,24E,25	117778	SUSP ASSM 20" 0-175LBS AP6	
1A,2A,3B-22,23E,24E,25	117779	SUSP ASSM 12" 176-265LBS AP6	
1B,2B,3B-22,23E,24E,25	117780	SUSP ASSM 13" 176-265LBS AP6	
1C,2C,3B-22,23E,24E,25	117781	SUSP ASSM 14" 176-265LBS AP6	
1D,2D,3B-22,23E,24E,25	117782	SUSP ASSM 15" 176-265LBS AP6	
1E,2E,3B-22,23E,24E,25	117783	SUSP ASSM 16" 176-265LBS AP6	
1F,2F,3B-22,23E,24E,25	117784	SUSP ASSM 17" 176-265LBS AP6	
1G,2G,3B-22,23E,24E,25	117785	SUSP ASSM 18" 176-265LBS AP6	
1H,2H,3B-22,23E,24E,25	117786	SUSP ASSM 19" 176-265LBS AP6	
1J,2J,3B-22,23E,24E,25	117787	SUSP ASSM 20" 176-265LBS AP6	
1A,2A,3A-22,23F,24F,25	117788	SUSP ASSM 12" 0-175LBS AP7	
1B,2B,3A-22,23F,24F,25	117789	SUSP ASSM 13" 0-175LBS AP7	
1C,2C,3A-22,23F,24F,25	117790	SUSP ASSM 14" 0-175LBS AP7	
1D,2D,3A-22,23F,24F,25	117791	SUSP ASSM 15" 0-175LBS AP7	
1E,2E,3A-22,23F,24F,25	117792	SUSP ASSM 16" 0-175LBS AP7	
1F,2F,3A-22,23F,24F,25	117793	SUSP ASSM 17" 0-175LBS AP7	
1G,2G,3A-22,23F,24F,25	117794	SUSP ASSM 18" 0-175LBS AP7	
1H,2H,3A-22,23F,24F,25	117795	SUSP ASSM 19" 0-175LBS AP7	
1J,2J,3A-22,23F,24F,25	117796	SUSP ASSM 20" 0-175LBS AP7	
1A,2A,3B-22,23F,24F,25	117797	SUSP ASSM 12" 176-265LBS AP7	
1B,2B,3B-22,23F,24F,25	117798	SUSP ASSM 13" 176-265LBS AP7	
1C,2C,3B-22,23F,24F,25	117799	SUSP ASSM 14" 176-265LBS AP7	
1D,2D,3B-22,23F,24F,25	117800	SUSP ASSM 15" 176-265LBS AP7	
1E,2E,3B-22,23F,24F,25	117801	SUSP ASSM 16" 176-265LBS AP7	
1F,2F,3B-22,23F,24F,25	117802	SUSP ASSM 17" 176-265LBS AP7	
1G,2G,3B-22,23F,24F,25	117803	SUSP ASSM 18" 176-265LBS AP7	
1H,2H,3B-22,23F,24F,25	117804	SUSP ASSM 19" 176-265LBS AP7	
1J,2J,3B-22,23F,24F,25	117805	SUSP ASSM 20" 176-265LBS AP7	
1A,2A,3A-22,23G,24G,25	117806	SUSP ASSM 12" 0-175LBS AP8	
1B,2B,3A-22,23G,24G,25	117807	SUSP ASSM 13" 0-175LBS AP8	
1C,2C,3A-22,23G,24G,25	117808	SUSP ASSM 14" 0-175LBS AP8	
1D,2D,3A-22,23G,24G,25	117809	SUSP ASSM 15" 0-175LBS AP8	
1E,2E,3A-22,23G,24G,25	117810	SUSP ASSM 16" 0-175LBS AP8	
1F,2F,3A-22,23G,24G,25	117811	SUSP ASSM 17" 0-175LBS AP8	
1G,2G,3A-22,23G,24G,25	117812	SUSP ASSM 18" 0-175LBS AP8	
1H,2H,3A-22,23G,24G,25	117813	SUSP ASSM 19" 0-175LBS AP8	
1J,2J,3A-22,23G,24G,25	117796	SUSP ASSM 20" 0-175LBS AP7	
1A,2A,3B-22,23G,24G,25	117815	SUSP ASSM 12" 176-265LBS AP8	
1B,2B,3B-22,23G,24G,25	117816	SUSP ASSM 13" 176-265LBS AP8	
1C,2C,3B-22,23G,24G,25	117817	SUSP ASSM 14" 176-265LBS AP8	
1D,2D,3B-22,23G,24G,25	117818	SUSP ASSM 15" 176-265LBS AP8	
1E,2E,3B-22,23G,24G,25	117819	SUSP ASSM 16" 176-265LBS AP8	


Pos.	Item Number	Description	Remarks
1F,2F,3B-22,23G,24G,25	117820	SUSP ASSM 17" 176-265LBS AP8	
1G,2G,3B-22,23G,24G,25	117821	SUSP ASSM 18" 176-265LBS AP8	
1H,2H,3B-22,23G,24G,25	117822	SUSP ASSM 19" 176-265LBS AP8	
1J,2J,3B-22,23G,24G,25	117823	SUSP ASSM 20" 176-265LBS AP8	

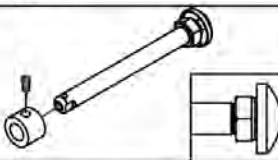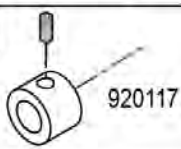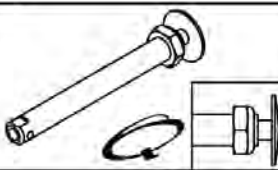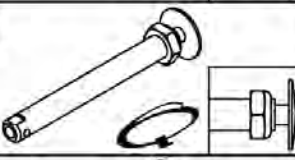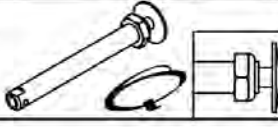Wheel Type	Axle Type		Axle P/N	Spacer P/N
6 Spoke Mag Wheels	Quick Release	1A 	922356 Includes Spacer A=4 1/4"	2  920117
	Quad Quick Release	1B 	116390 Includes Spacer A=4 1/4"	2  920117
Spinergy Wheels 18" Standard Spoke, Mountain Wheels, 5 Spoke Mag Wheels	Quick Release 	1C 	109385 A=4 1/2"	N/A
Ultra Light Wheels (N/A with Quad Quick Release)	Quick Release	1D 	111843 A=4"	N/A
Lite Spoke	Quick Release	1E 	107268 A=4 1/4"	N/A

Quick Release Measuring Guide

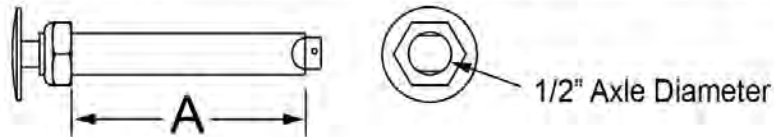


Pos.	Item Number	Description	Remarks
1A	922356	QR AXLE BIG BUTTON W/ SPACER	Includes spacer
1B	116390	QR AXLE W/SPACER FOR 6S MAG	Includes spacer
1C	109385	QR AXLE/NUT LONG BG BTTN	
1D	111843	QR AXLE EX SHORT BIG BTN	
1E	107268	QR AXLE AND NUT SHORT BIG BTTN	
2	920117	MAG SPACER 6S W/SET SCREW	

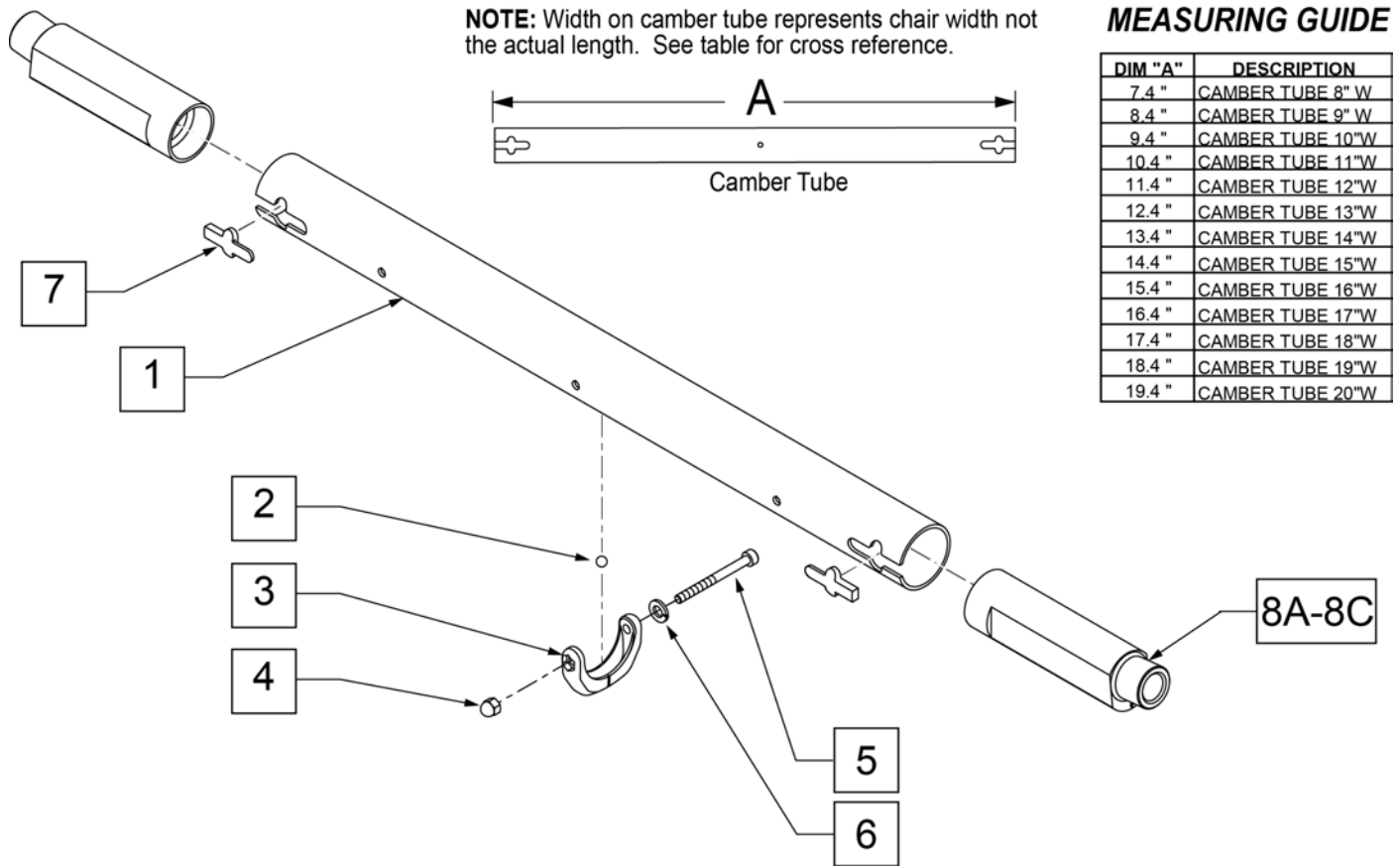

Pos.	Item Number	Description	Remarks
1	110206	CAMBER TUBE 12"W	
1	110207	CAMBER TUBE 13"W	
1	110208	CAMBER TUBE 14"W	
1	110209	CAMBER TUBE 15"W	
1	110210	CAMBER TUBE 16"W	
1	110211	CAMBER TUBE 17"W	
1	110212	CAMBER TUBE 18"W	
1	110213	CAMBER TUBE 19"W	
1	110214	CAMBER TUBE 20"W	
2-6	050190	BEARING LEVEL CAMBER TUBE	
4	115918	M4 NUT ACORN PLASTIC BLACK	
6	050129-002	LOCKWASHER HIGH COLLAR M4	
7	114428	CAMBER BLOCK KIT (10)	QTY 10
8A	110847	CAMBER PLUG 0 DEG	0 DEGREES
8B	110848	CAMBER PLUG 3 DEG	3 DEGREES
8C	110849	CAMBER PLUG 6 DEG	6 DEGREES
1-7,8A	114401	CAMBER TUBE 12"W 0D ASSM	
1-7,8A	114402	CAMBER TUBE 13"W 0D ASSM	
1-7,8A	114403	CAMBER TUBE 14"W 0D ASSM	
1-7,8A	114404	CAMBER TUBE 15"W 0D ASSM	
1-7,8A	114405	CAMBER TUBE 16"W 0D ASSM	
1-7,8A	114406	CAMBER TUBE 17"W 0D ASSM	
1-7,8A	114407	CAMBER TUBE 18"W 0D ASSM	
1-7,8A	114408	CAMBER TUBE 19"W 0D ASSM	
1-7,8A	114409	CAMBER TUBE 20"W 0D ASSM	


Pos.	Item Number	Description	Remarks
1-7,8B	114410	CAMBER TUBE 12"W 3D ASSM	
1-7,8B	114411	CAMBER TUBE 13"W 3D ASSM	
1-7,8B	114412	CAMBER TUBE 14"W 3D ASSM	
1-7,8B	114413	CAMBER TUBE 15"W 3D ASSM	
1-7,8B	114414	CAMBER TUBE 16"W 3D ASSM	
1-7,8B	114415	CAMBER TUBE 17"W 3D ASSM	
1-7,8B	114416	CAMBER TUBE 18"W 3D ASSM	
1-7,8B	114417	CAMBER TUBE 19"W 3D ASSM	
1-7,8B	114418	CAMBER TUBE 20"W 3D ASSM	
1-7,8C	114419	CAMBER TUBE 12"W 6D ASSM	
1-7,8C	114420	CAMBER TUBE 13"W 6D ASSM	
1-7,8C	114421	CAMBER TUBE 14"W 6D ASSM	
1-7,8C	114422	CAMBER TUBE 15"W 6D ASSM	
1-7,8C	114423	CAMBER TUBE 16"W 6D ASSM	
1-7,8C	114424	CAMBER TUBE 17"W 6D ASSM	
1-7,8C	114425	CAMBER TUBE 18"W 6D ASSM	
1-7,8C	114426	CAMBER TUBE 19"W 6D ASSM	
1-7,8C	114427	CAMBER TUBE 20"W 6D ASSM	

Note: to ensure that the lights will function in the Micro Light Up Caster, qty 2 each of p/n 106947 will be required per caster when retrofitting Micro Light Up Casters to the chair.

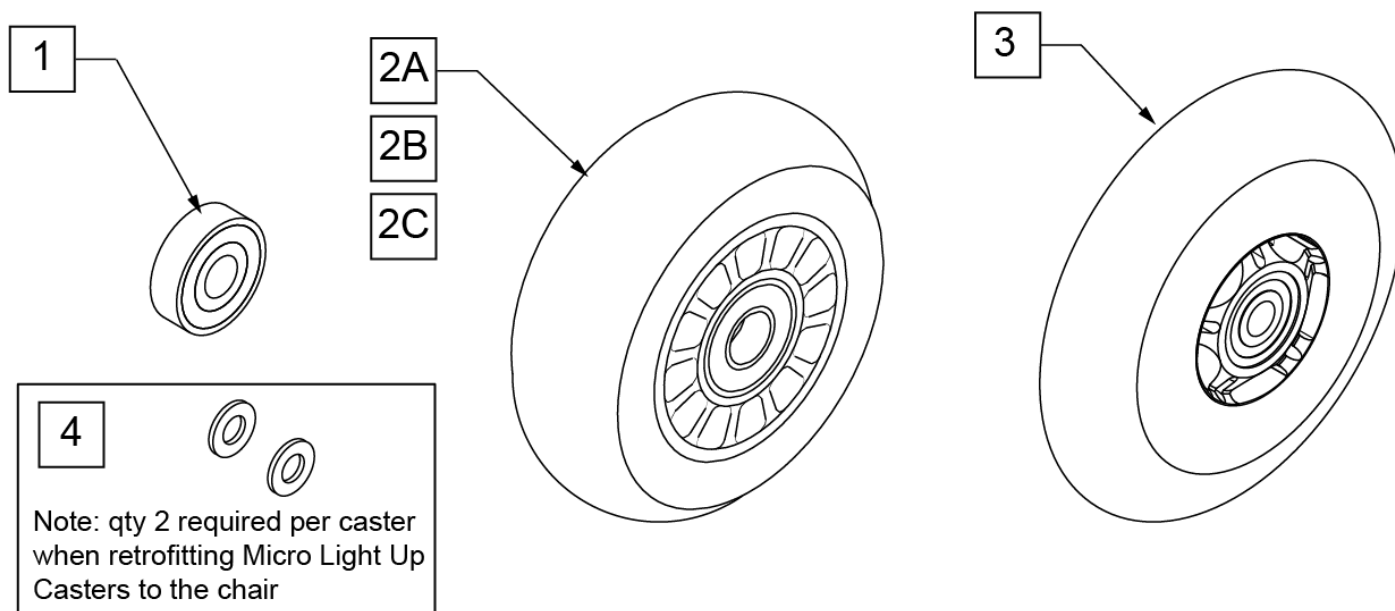

CASTER MEASURING GUIDE


PART NUMBER	"A" TIRE DIAMETER	"B" HUB WIDTH
400108	4.0	1.0
400107	3.0	1.0
052129	4.7	1.0

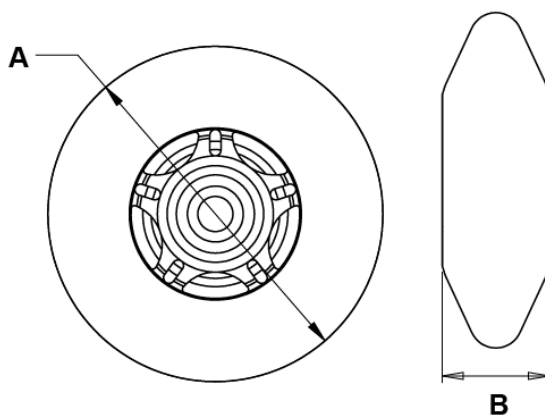

Pos.	Item Number	Description	HCPC	Remarks
1	349502	7/8"x 5/16" BEARING		
2A	400108	4" MICRO CASTER LIGHTED	K0077	4" MICRO LIGHTED
2B	920025	3" MICRO CASTER W/ BEARING -PAIR	K0077	PAIR 3" MICRO
2C	400107	3" MICRO CASTER LIGHTED	K0077	3" MICRO LIGHTED
3	052129	5" MICRO CASTER LIGHTED	K0077	5" MICRO LIGHTED
4	106947	WASHER FLAT THIN AN 5/16 ROH		


NOTE: 7" and 8" Casters are not available with Integral Bearing Forks.

Note: When 6" casters or larger casters are used, Anti Flutter Kit must be installed. See Standard Multi Position Fork page.

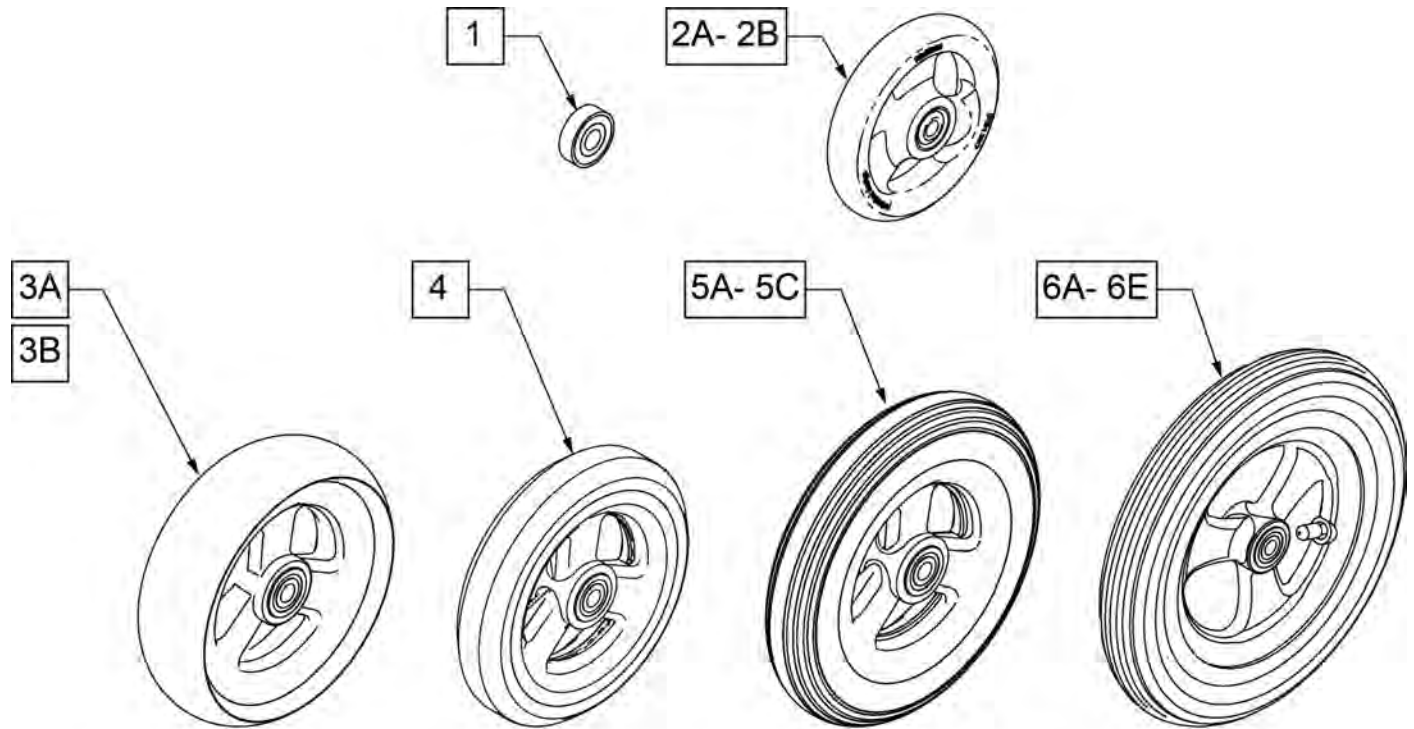


**CASTER
MEASURING GUIDE**


PART NUMBER	"A" TIRE DIAMETER	"B" HUB WIDTH	PART NUMBER	"A" TIRE DIAMETER	"B" HUB WIDTH
101957	4.3	1.0	101256	5.75	1.0
101098	4.0	1.0	101257	5.9	1.0
101259	4.9	1.0	400212	8.0	1.0
101255	5.7	1.0	400213	7.8	1.0
101254	4.9	1.0	400214	6.9	1.0
			400215	7.25	1.0

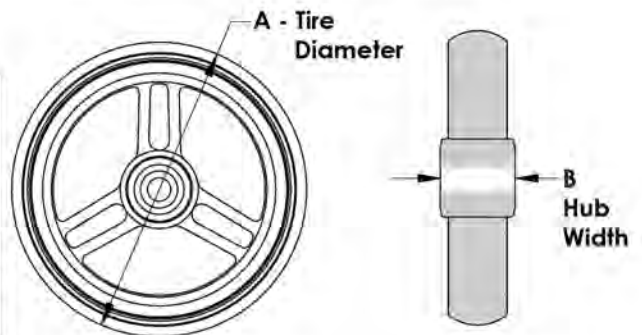

Pos.	Item Number	Description	Remarks
1	349502	7/8"x 5/16" BEARING	
2A	101957	4"x1-1/4 POLY 3 SPOKE CSTR	4" X 1.25 SEMI-PNEUMATIC
2B	101098	CASTER 4" PLASTIC/POLY W/BRG	4" LOW PROFILE POLY
3A	101259	5"x1-1/2" POLY 3 SPOKE CSTR	5" x 1.5" SEMI-PNEUMATIC
3B	101255	6"x1-1/2" POLY 3 SPOKE CSTR	6" x 1.5" SEMI-PNEUMATIC
4	101254	5" LOW POLY 3 SPOKE CASTER	5" LOW PROFILE POLY
5A	101256	6" SEMI PNEU 3 SPOKE CASTER	6" POLY
5B	101257	6" PNEUMATIC 3 SPOKE CASTER	6" PNEUMATIC
5C	383245	6" PNEUMATIC TIRE & TUBE	
6A	400212	CASTER 8" PNEU 3 SPK ASSM	8" PNEUMATIC
6B	400213	CASTER 8" POLY 3 SPK ASSM	8" POLY
6C	400214	CASTER 7" LOW POLY 3 SPK ASSM	7" x LOW PROFILE POLY
6D	383255	8" PNEUMATIC TIRE & TUBE	
6E	400215	CASTER 8" WIDE POLY 3 SPK ASSM	8" x 1.5" LOW POLY


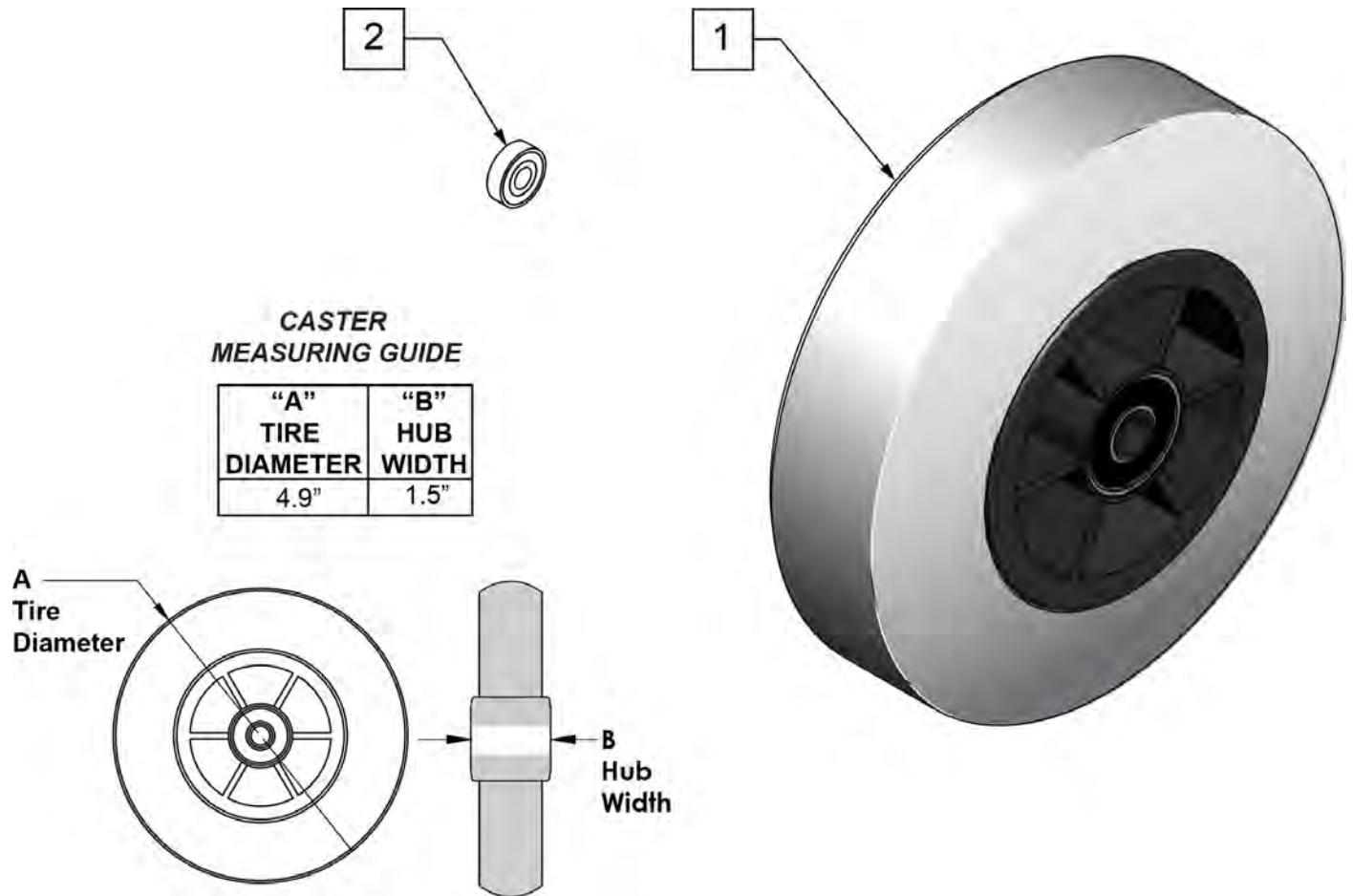


Pos.	Item Number	Description	HCPC	Remarks
1	400103	CASTER 5" SOFT ROLL W/BRNGS	K0077	5" SOFT ROLL SEMI PNEU (GREY)
2	349502	7/8"x 5/16" BEARING		

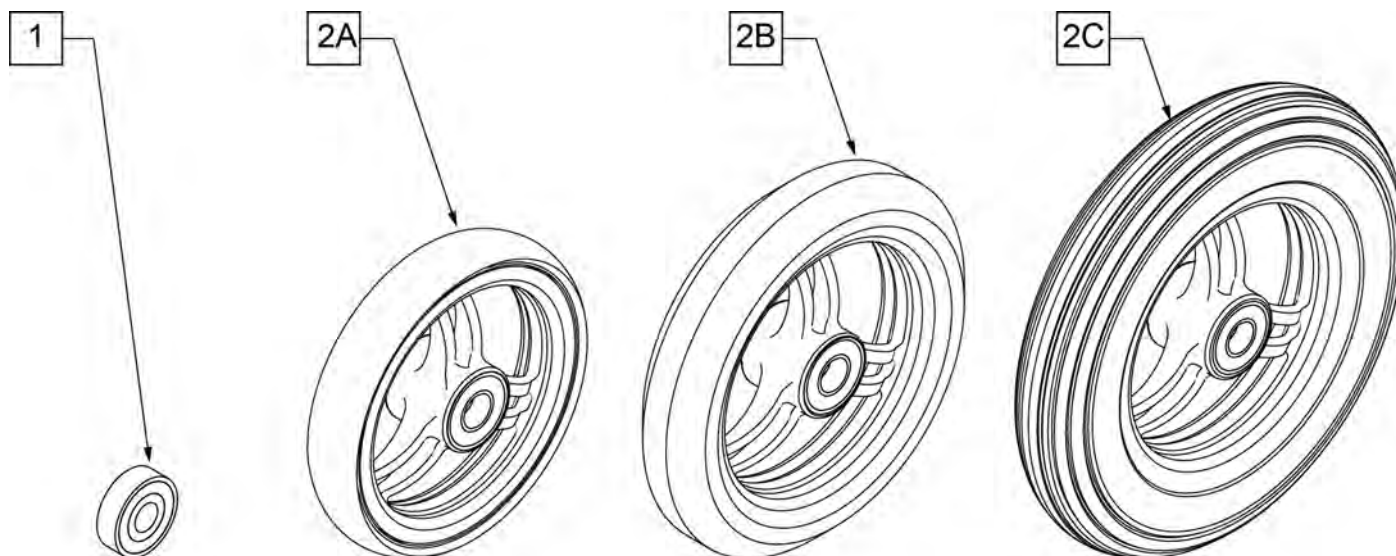

**CASTER
MEASURING GUIDE**


"A" TIRE DIAMETER	"B" HUB WIDTH
4"	1"
5"	1"
6"	1"

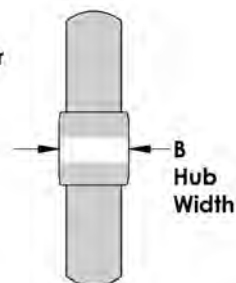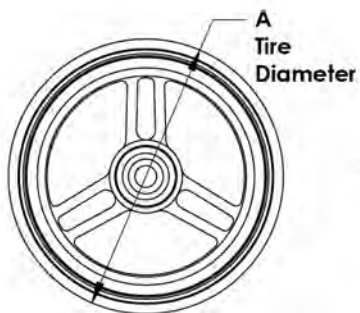


Pos.	Item Number	Description	HCPC	Remarks
1	349502	7/8"x 5/16" BEARING		
2A	101946-001	CASTR 4X1 ALUM/POLY W/BRG CLR	K0077	4" x 1" ALUMINUM POLY
2B	101946-002	CASTR 5X1 ALUM/POLY W/BRG CLR	K0077	5" x 1" ALUMINUM POLY
2C	101946-003	CASTR 6X1 ALUM/POLY W/BRG CLR	K0077	6" x 1" ALUMINUM POLY



SIDE VIEW

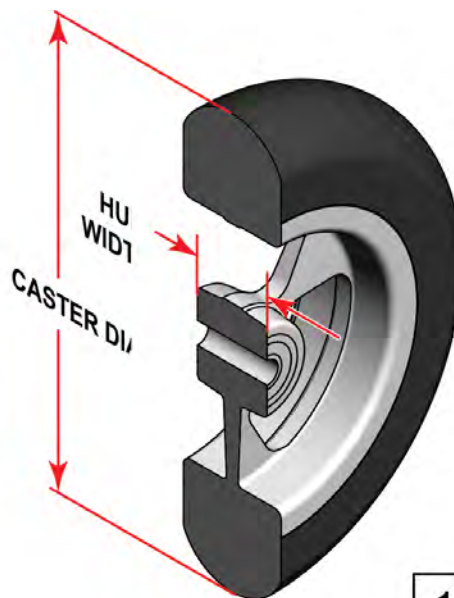**CASTER
MEASURING GUIDE**


TIRE DIAMETER	HUB WIDTH
3	1
4 1/8	1
5	1
6	1

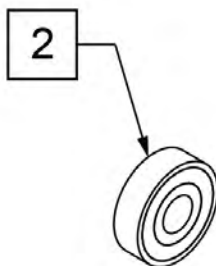

Pos.	Item Number	Description	HCPC	Remarks
1	113394	WHEEL CASTER 3" ULTRALITE SFTRLL	K0077	3" x 1.5" Aluminum Soft Roll - Black Tire - Silver Hub
1	113395	WHEEL CASTER 4" ULTRALITE SFTRLL	K0077	4" x 1.5" Aluminum Soft Roll - Black Tire - Silver Hub
1	113396	WHEEL CASTER 5" ULTRALITE SFTRLL	K0077	5" x 1.5" Aluminum Soft Roll - Black Tire - Silver Hub
1	113397	WHEEL CASTER 6" ULTRALITE SFTRLL	K0077	6" x 1.5" Aluminum Soft Roll - Black Tire - Silver Hub
2	349502	7/8" X 5/16" BEARING		

Note: After S/N R4-019625 for Q7 and S/N RZ-007453 for Zone, Fork Stem Assembly was updated. Please replace in pairs if not previously replaced.

Note: If using 6" casters on Q7, Anti Flutter should be installed.

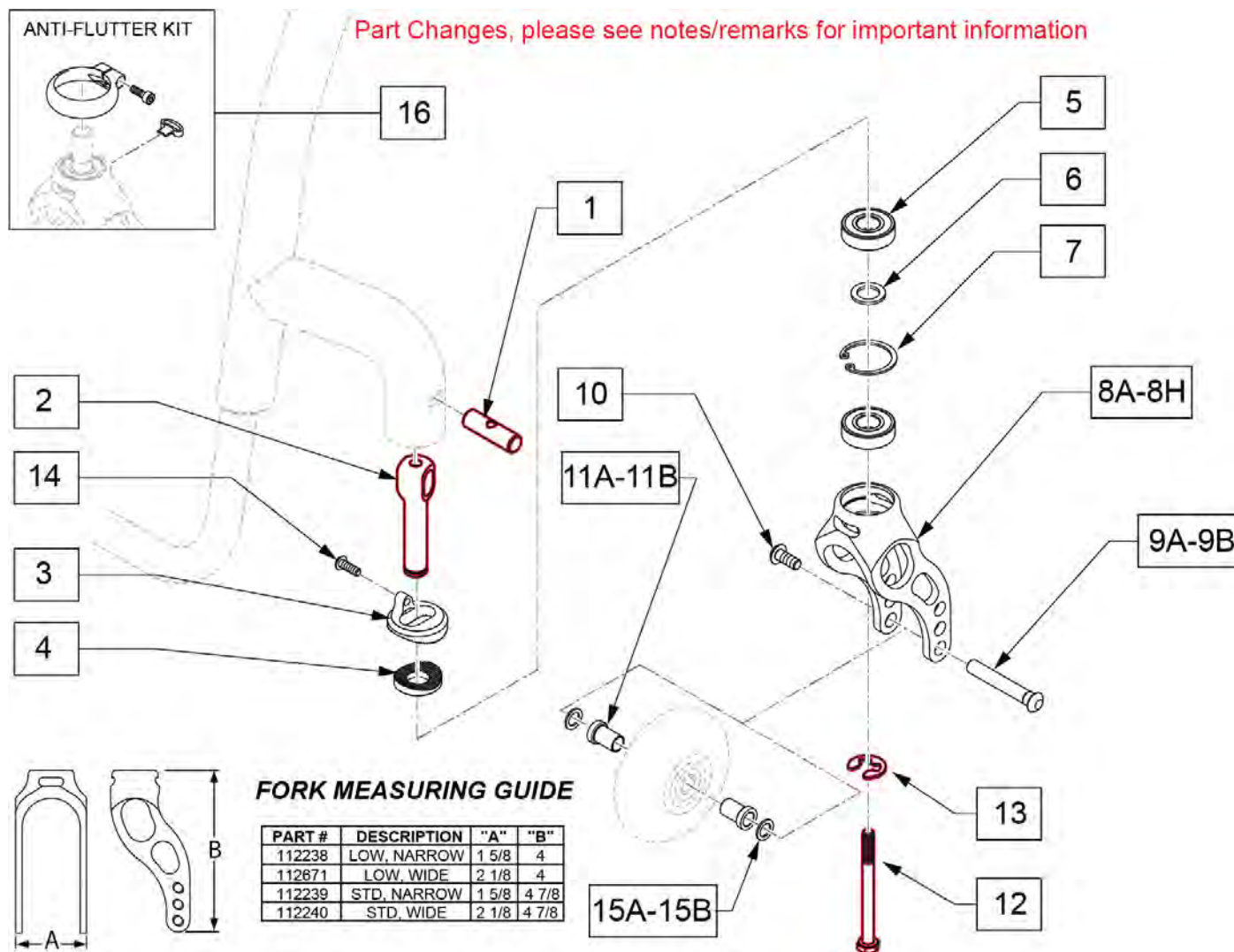

Pos.	Item Number	Description	Remarks
1	123396	PIVOT PIN, SOLID	See Fork Stem & Pin Kits for orderable parts

Pos.	Item Number	Description	Remarks
2	123397	CASTER STEM, ADJ, HOLLOW	See Fork Stem & Pin Kits for orderable parts
3	110163	HELIUM COMBI SPLINE	
4	502051	SPLINE CASTER ANG ADJ LOWER	
5	349104	BRNG 28MM OD 12MM ID 8MM THK	
6	265162	WSHR FLT 18MM OD 12MM ID 1.5MM	
7	342093	RING SNAP INTERNAL 28MM	
5-7,8A	112238	ASSM FORK LOW NARROW INTG BRG	Black-Fits 3" Micro, 3" Micro lighted, 4" Low Profile Poly, 4" Micro Lighted, 4x1 Alu Poly
5-7,8B	112671	ASSM FORK LOW WIDE INTG BRG	Black-Fits 3x1.5 Alu Soft Roll, 4x1.25 Semi Pneu, 4x1.5 Alu Soft Roll
5-7,8C	112239	ASSM FORK STD NARROW INTG BRG	Black-Fits 5" Low Profile Poly, 5" Micro Lighted
5-7,8D	112240	ASSM FORK STD WIDE INTG BRG	Black-Fits 5x1 Alu Poly, 5x1.5 Semi Pneu, 5' Soft Roll, 5x1.5 Alu Soft Roll, 6" Pneu, 6x1.5 Semi Pneu, 6x1.5 Alu Soft Roll
5-7,8E	117172S	FORK LOW NRW CSTM ANO W/BRG	Choose custom anodized color-Fits 3" Micro, 3" Micro lighted, 4" Low Profile Poly, 4" Micro Lighted, 4x1 Alu Poly

Pos.	Item Number	Description	Remarks
5-7,8F	117173S	FORK LOW WD CSTM ANO W/BRG	Choose custom anodized color-Fits 3x1.5 Alu Soft Roll
5-7,8G	117174S	FORK STD NRW CSTM ANO W/BRG	Choose custom anodized color-Fits 5x1 Alu Poly
5-7,8H	117175S	FORK STD WD CSTM ANO W/BRG	Choose custom anodized color-Fits 5x1.5 Semi Pneu, 5' Soft Roll, 5x1.5 Alu Soft Roll
9A	112783	AXLE CASTER NARROW 1/4"	Used w/ Narrow Forks Only.
9B	112782	AXLE CASTER WIDE 1/4"	Used w/ Wide Forks Only.
10	102259-001	BHCS M5 X 0.8 X 12 FULLNP	
11A	112781	SPACER, CASTER NARROW, INTEGRAL	Used w/ Narrow Forks Only.
11B	112780	SPACER, CASTER WIDE, INTEGRAL	Used w/ Wide Forks Only.
12	230063	HHCS M6 X 1.0 X 70	See Fork Stem & Pin Kits for orderable parts
13	114479	E-STYLE BOWED CLIP	See Fork Stem & Pin Kits for orderable parts
14	102257	BHCS M4 X .7 X 8 INORGANIC	
15A	107357	WASHER SPLIT LOCK M5 ROHS	
15B	106947	WASHER FLAT THIN AN 5/16 ROHS	Use w/ micro light casters only
16	120957	ANTI FLUTTER KIT W/ INSTR	Pair - Used on Standard Wide and Standard Narrow Fork only.


Pos.	Item Number	Description	Remarks
17	113403	6MM ALLEN WRENCH L, 7MM SHORT	Required to remove old stem bolt
1-8A,9A,10,11A-16	114429	FORK LOW NARROW Q7 ASSM	Black-Fits 3" Micro,3" Micro lighted,4" Low Profile Poly,4" Micro Lighted,4x1 Alu Poly
1-8B,9B,10,11B-16	114430	FORK LOW WIDE Q7 ASSM	Black-Fits 3x1.5 Alu Soft Roll,4x1.25 Semi Pneu,4x1.5 Alu Soft Roll
1-8C,9A,10,11A-16	114431	FORK STD NARROW Q7 ASSM	Black-Fits 5" Low Profile Poly,5" Micro Lighted,6" Pneu,6" Poly
1-8D,9B,10,11B-16	114432	FORK STD WIDE Q7 ASSM	Black-Fits 5x1 Alu Poly, 5x1.5 Semi Pneu, 5' Soft Roll,5x1.5 Alu Soft Roll, 6x1.5 Semi Pneu,6x1.5 Alu Soft Roll
1,2,12,13	124999	FORK STEM & PIN KIT PAIR	Pair
1,2,12,13	125045	FORK STEM & PIN KIT 1 SIDE	one side only
3,4,14	125046	FORK SPLINE ADJ KIT 1 SIDE	one side only

Note: After S/N R4-019625 for Q7 and S/N RZ-007453 for Zone, Fork Stem Assembly was updated. Please replace in pairs if not previously replaced.

Part Changes, please see notes/remarks for important information

FORK MEASURING GUIDE

PART #	DESCRIPTION	"A"
112672	LOW, LH	4
112673	LOW, RH	4
112242	STD, LH	4 7/8
112243	STD, RH	4 7/8

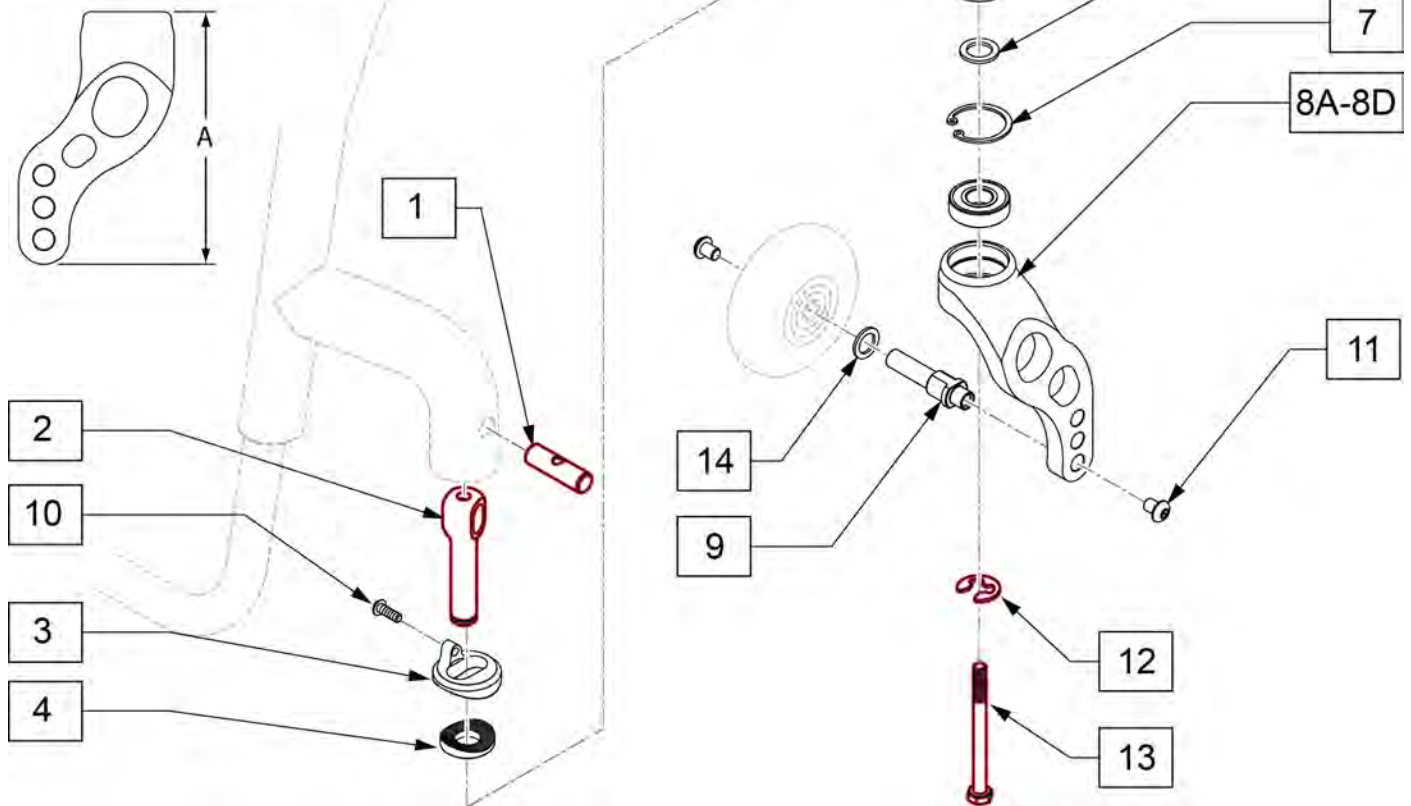

Pos.	Item Number	Description	Remarks
1	123396	PIVOT PIN, SOLID	See Fork Stem & Pin Kits for orderable parts
2	123397	CASTER STEM, ADJ, HOLLOW	See Fork Stem & Pin Kits for orderable parts
3	110163	HELIUM COMBI SPLINE	
4	502051	SPLINE CASTER ANG ADJ LOWER	


Pos.	Item Number	Description	Remarks
5	349104	BRNG 28MM OD 12MM ID 8MM THK	
6	265162	WSHR FLT 18MM OD 12MM ID 1.5MM	
7	342093	RING SNAP INTERNAL 28MM	
5-8A	112672	ASSM FORK LOW SNGL SD LH W/BRG	Black-Fits 3" Micro,3" Micro lighted,4" Low Profile Poly,4" Micro Lighted,4x1 Alu Poly,3x1.5 Alu Soft Roll,4x1.25 Semi Pneu,4x1.5 Alu Soft Roll
5-8A	112673	ASSM FORK LOW SNGL SD RH W/BRG	Black-Fits 3" Micro,3" Micro lighted,4" Low Profile Poly,4" Micro Lighted,4x1 Alu Poly,3x1.5 Alu Soft Roll,4x1.25 Semi Pneu,4x1.5 Alu Soft Roll
5-8B	112242	ASSM FORK STD SNGL SD LH W/BRG	Black-Fits 5" Low Profile Poly,5" Micro Lighted,5x1 Alu Poly, 5x1.5 Semi Pneu, 5x1.5 Alu Soft Roll,6" Poly,6"Pneu, 6X1 Alu Poly, 6x1.5 Semi Pneu, 6x1.5 Alu Soft Roll
5-8B	112243	ASSM FORK STD SNGL SD RH W/BRG	Black-Fits 5" Low Profile Poly,5" Micro Lighted,5x1 Alu Poly, 5x1.5 Semi Pneu, 5x1.5 Alu Soft Roll,6" Poly, 6"Pneu, 6X1 Alu Poly, 6x1.5 Semi Pneu, 6x1.5 Alu Soft Roll

Pos.	Item Number	Description	Remarks
5-8C	117176S	FORK SNGL LOW CSTM ANO L	Choose custom anodized color-Fits 3" Micro,3" Micro lighted,4' Low Profile Poly,4" Micro Lighted,4x1 Alu Poly
5-8C	117177S	FORK SNGL LOW CSTM ANO R	Choose custom anodized color-Fits 3x1.5 Alu Soft Roll
5-8D	117178S	FORK SNGL STD CSTM ANO L	Choose custom anodized color-Fits 5x1 Alu Poly, 5x1.5 Semi Pneu, 5' Soft Roll,5x1.5 Alu Soft Roll
5-8D	117179S	FORK SNGL STD CSTM ANO R	Choose custom anodized color-Fits 5x1 Alu Poly, 5x1.5 Semi Pneu, 5' Soft Roll,5x1.5 Alu Soft Roll
9	112674	AXLE FORK SINGLE SIDE WIDE	
10	102257	BHCS M4 X .7 X 8 INORGANIC	
11	103185-005	BHCS M6X0.8X12 W/PL INORG	
12	114479	E-STYLE BOWED CLIP	See Fork Stem & Pin Kits for orderable parts
13	230063	HHCS M6 X 1.0 X 70	See Fork Stem & Pin Kits for orderable parts
14	106947	WASHER FLAT THIN AN 5/16 ROHS	Use w/ micro light casters only
15	113403	6MM ALLEN WRENCH L, 7MM SHORT	For adjusting stem bolt
1-8A,9-14	114434	FORK SS LOW Q7 L ASSM	Left
1-8A,9-14	114433	FORK SS LOW Q7 R ASSM	Right
1-8B,9-14	114436	FORK SS STD Q7 L ASSM	Left

Pos.	Item Number	Description	Remarks
1-8B,9-14	114435	FORK SS STD Q7 R ASSM	Right
1,2,12,13	124999	FORK STEM & PIN KIT PAIR	Pair
1,2,12,13	125045	FORK STEM & PIN KIT 1 SIDE	one side only
3,4,10	125046	FORK SPLINE ADJ KIT 1 SIDE	one side only

Note: Item 10(Insert Tool) is used to aid in aligning the upper and lower fork assembly during Frog Leg assembly and elastomer replacement. See instruction sheet for details.


Note: After S/N R4-019625 for Q7 and S/N RZ-007453 for Zone, Fork Stem Assembly was updated. Please replace in pairs if not previously replaced.

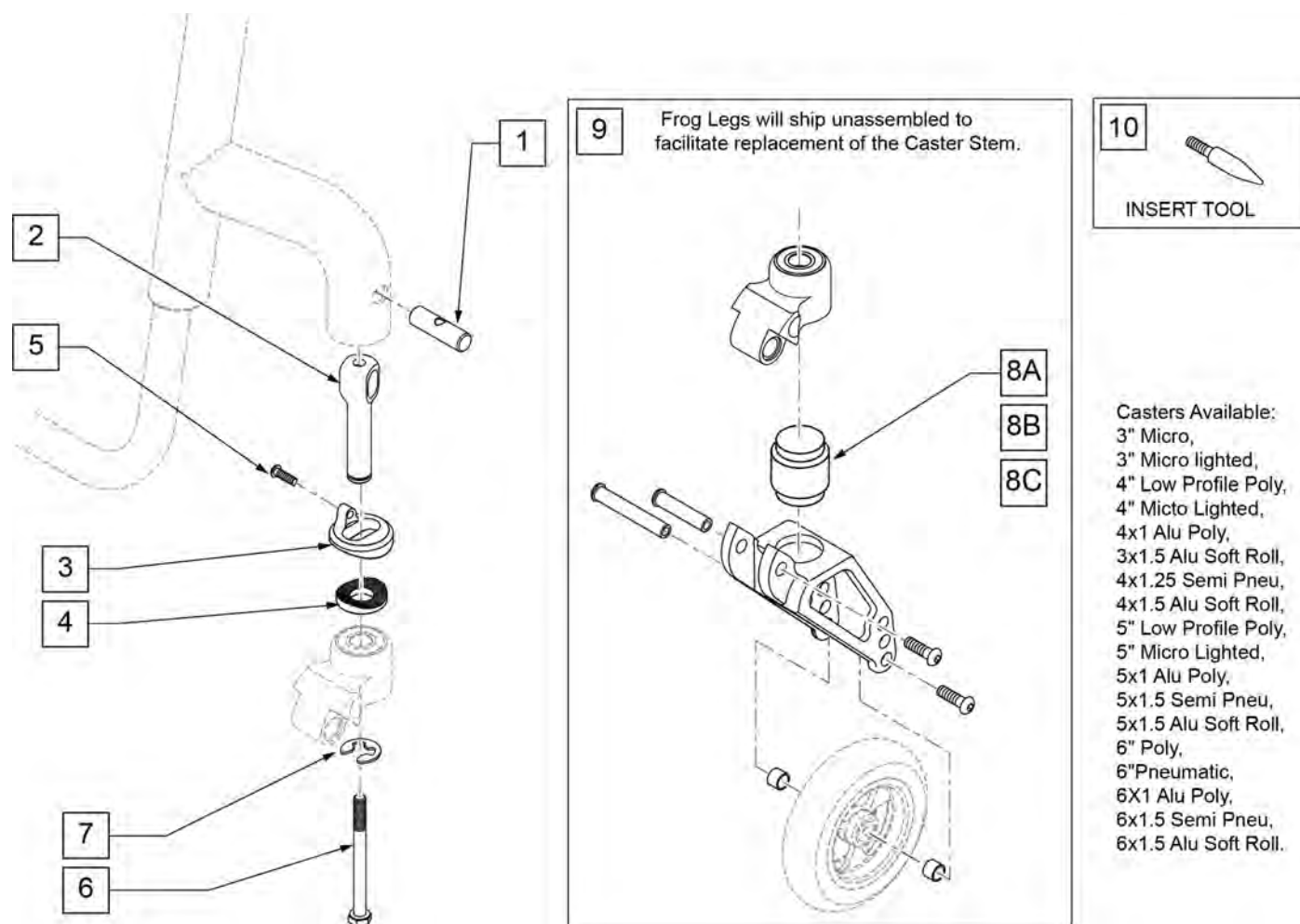

Pos.	Item Number	Description	Remarks
1	123396	PIVOT PIN, SOLID	See Fork Stem & Pin Kits for orderable parts
2	123397	CASTER STEM, ADJ, HOLLOW	See Fork Stem & Pin Kits for orderable parts
3	110163	HELIUM COMBI SPLINE	
4	502051	SPLINE CASTER ANG ADJ LOWER	
5	102257	BHCS M4 X .7 X 8 INORGANIC	
6	230063	HHCS M6 X 1.0 X 70	See Fork Stem & Pin Kits for orderable parts
7	114479	E-STYLE BOWED CLIP	See Fork Stem & Pin Kits for orderable parts
8A	050156-001	ELASTOMER FROGLEGS SOFT	0 -100 Lbs
8B	050156-002	ELASTOMER FROGLEGS MEDIUM	101 -190 Lbs
8C	050156-003	ELASTOMER FROGLEGS FIRM	191 -265 Lbs (not used w/ ZippieZone)
9	050155	FORK FROG LEGS INT BRNG	
10	116351	FROG LEG FORK ASSEMBLY TOOL	

Pos.	Item Number	Description	Remarks
8A,9	050225-001	FORK FROGLEGS INT BRNG SOFT	
8B,9	050225-002	FORK FROGLEGS INT BRNG MED	
8C,9	050225-003	FORK FROGLEGS INT BRNG FIRM	
1-7,10	120025	FL CASTER STEM RPLC KIT	
1-7,8A,9	114437	FORK FROG SOFT Q7 ASSM	
1-7,8B,9	114438	FORK FROG MED Q7 ASSM	
1-7,8C,9	114439	FORK FROG FIRM Q7 ASSM	
1,2,6,7	124999	FORK STEM & PIN KIT PAIR	Pair
1,2,6,7	125045	FORK STEM & PIN KIT 1 SIDE	one side only
3-5	125046	FORK SPLINE ADJ KIT 1 SIDE	one side only


Note: If adding Anti-tips to a chair it is not required to order item 10 as it is included in the Camber Tube Clamp. During installation slide item 10 inward and insert item 7 in the outside of the clamp.

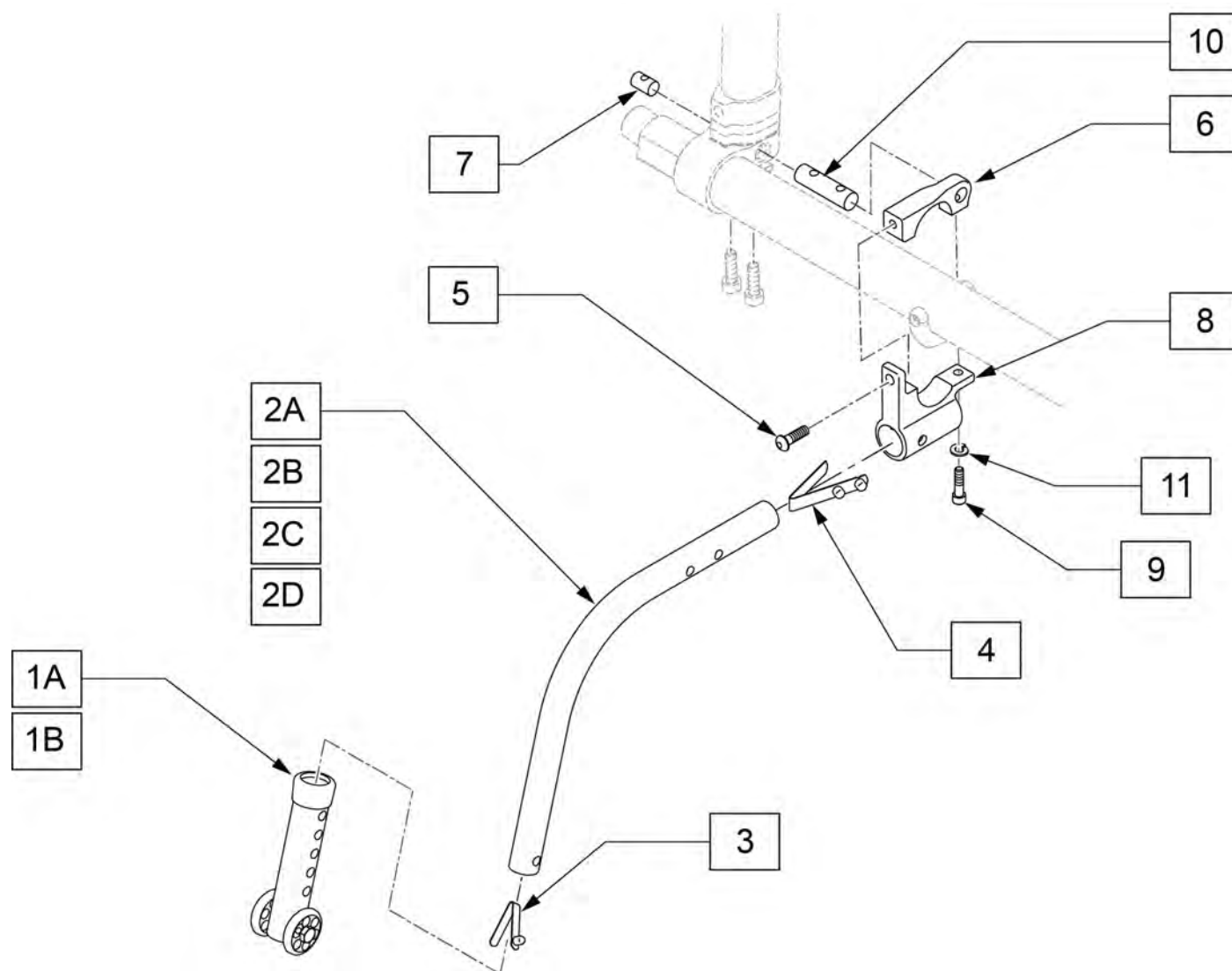

Pos.	Item Number	Description	Remarks
1A	104157	ANTI TIP ARM ADJUSTER	Used with 20" & 22" wheels
1B	104639	ANTI TIP ARM ADJ. MED	Used with 24", 25" & 26" wheels
5	112346	BHCS, M6X1X16, FULL, PL	
7	112653	ROUND NUT SIZE 2	
9	103189-008	SHCS, M5X0.8X30, FULL, NP	
10	112655	ROUND NUT SIZE 4	
11	107357	WASHER SPLIT LOCK M5 ROHS	
2A-4	104219-002	ANTI TIP TUBE ASSY CRVD RH	Right-Used with 20" wheels
2A-4	104219-001	ANTI TIP TUBE ASSY CRVD LH	Left-Used with 20" wheels
2B-4	104221-002	ANTI TIP ARM SML RGT	Right-Used with 22" wheels
2B-4	104221-001	ANTI TIP ARM SML LFT	Left-Used with 22" wheels
2C-4	104221-004	ANTI TIP ARM MED RGT	Right-Used with 24 & 25" wheels
2C-4	104221-003	ANTI TIP ARM MED LFT	Left-Used with 24 & 25" wheels


Pos.	Item Number	Description	Remarks
2D-4	104221-006	ANTI TIP ARM LRG RGT	Right-Used with 26" wheels
2D-4	104221-005	ANTI TIP ARM LRG LFT	Left-Used with 26" wheels
1A,2A-4	050172-001	REAR ANTI TIP ASSM 20" RIGHT	Used with 20" wheels
1A,2A-4	050173-001	REAR ANTI TIP ASSM 20" LEFT	Used with 20" wheels
1A,2B-4	050172-002	REAR ANTI TIP ASSM 22" RIGHT	Used with 22" wheels
1A,2B-4	050173-002	REAR ANTI TIP ASSM 22" LEFT	Used with 22" wheels
1B,2C-4	050172-003	REAR ANTI TIP ASSM 25" RIGHT	Used with 24 & 25" wheels
1B,2C-4	050173-003	REAR ANTI TIP ASSM 25" LEFT	Used with 24 & 25" wheels
1B,2D-4	050172-004	REAR ANTI TIP ASSM 26" RIGHT	Used with 26" wheels
1B,2D-4	050173-004	REAR ANTI TIP ASSM 26" LEFT	Used with 26" wheels
5-11	114440	ANTI TIP REC Q7 KIT	

Note: the Wheel Locks for Chair or Replacement Part orders shipped on or after 6/27/17 have upgraded mounting hardware; please replace in pairs for chairs ordered prior to 6/27/17.

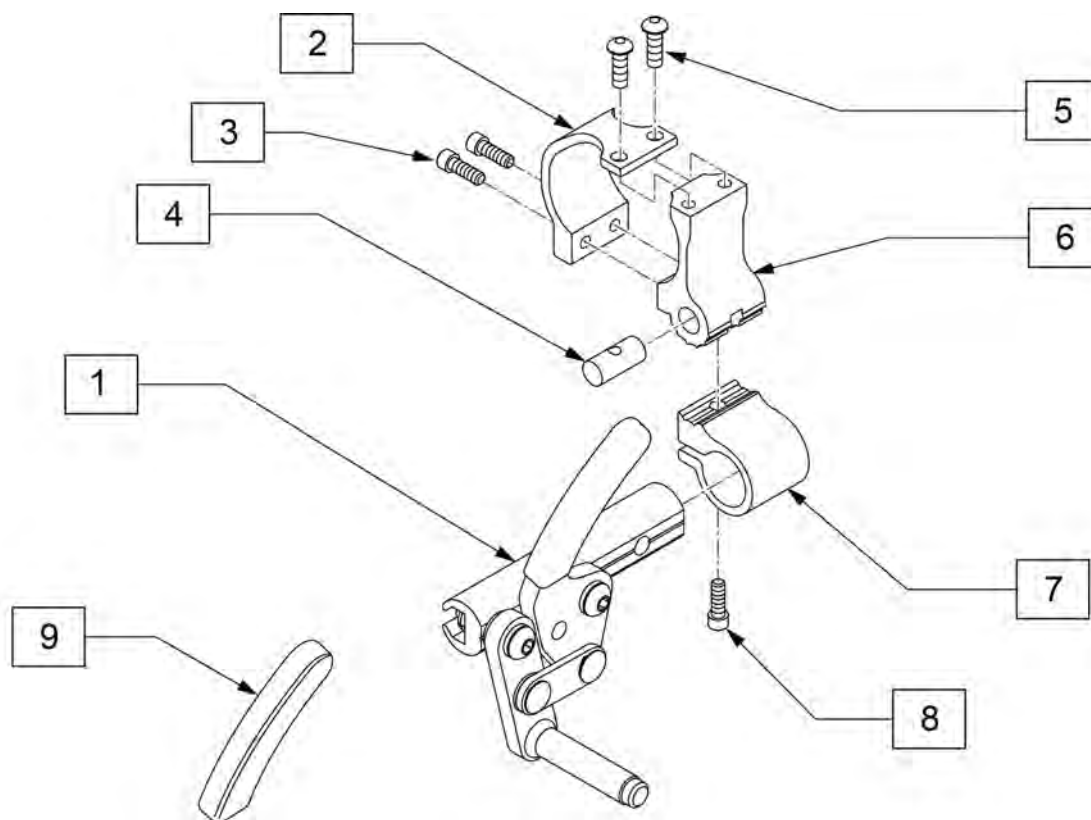


Pos.	Item Number	Description	Remarks
1	104216-001	HIGH MOUNT PUSH, RH	RIGHT
1	104216-002	HIGH MOUNT PUSH, LH	LEFT
3	102259-002	BHCS M5 x 0.8 x 20 W/ PL INORG	
4	112652	ROUND NUT SIZE 1	
5	103184-002	BHCS M5X0.8X8 FULL PL	
8	113317	SHCS M5X0.8X25 FULL PL	
2-8	114441	WHEEL LOCK CLAMP KIT	
9	104141	COVER CURVED WHEEL LOCK VINYL	


Note: the Wheel Locks for Chair or Replacement Part orders shipped on or after 6/27/17 have upgraded mounting hardware; please replace in pairs for chairs ordered prior to 6/27/17.

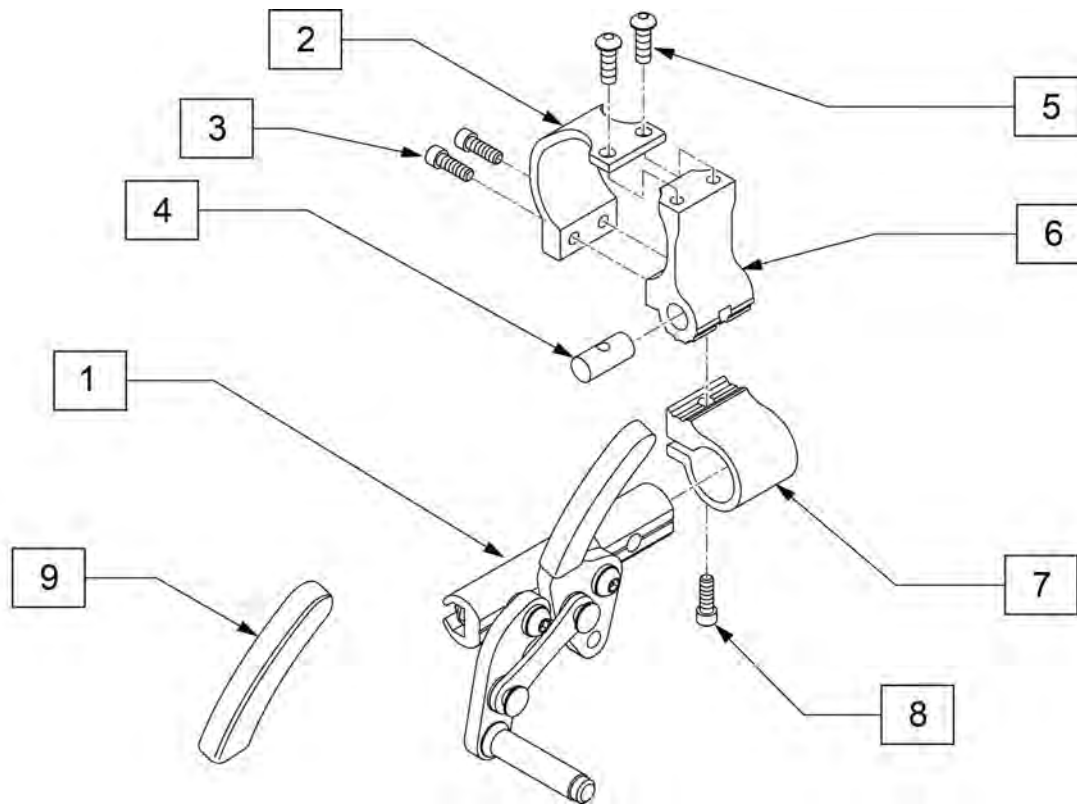

Pos.	Item Number	Description	Remarks
1	104227-001	HIGH MOUNT PULL, RH	RIGHT
1	104227-002	HIGH MOUNT PULL, LH	LEFT
3	102259-002	BHCS M5 x 0.8 x 20 W/ PL INORG	
4	112652	ROUND NUT SIZE 1	
5	103184-002	BHCS M5X0.8X8 FULL PL	
8	113317	SHCS M5X0.8X25 FULL PL	
2-8	114441	WHEEL LOCK CLAMP KIT	
9	104141	COVER CURVED WHEEL LOCK VINYL	

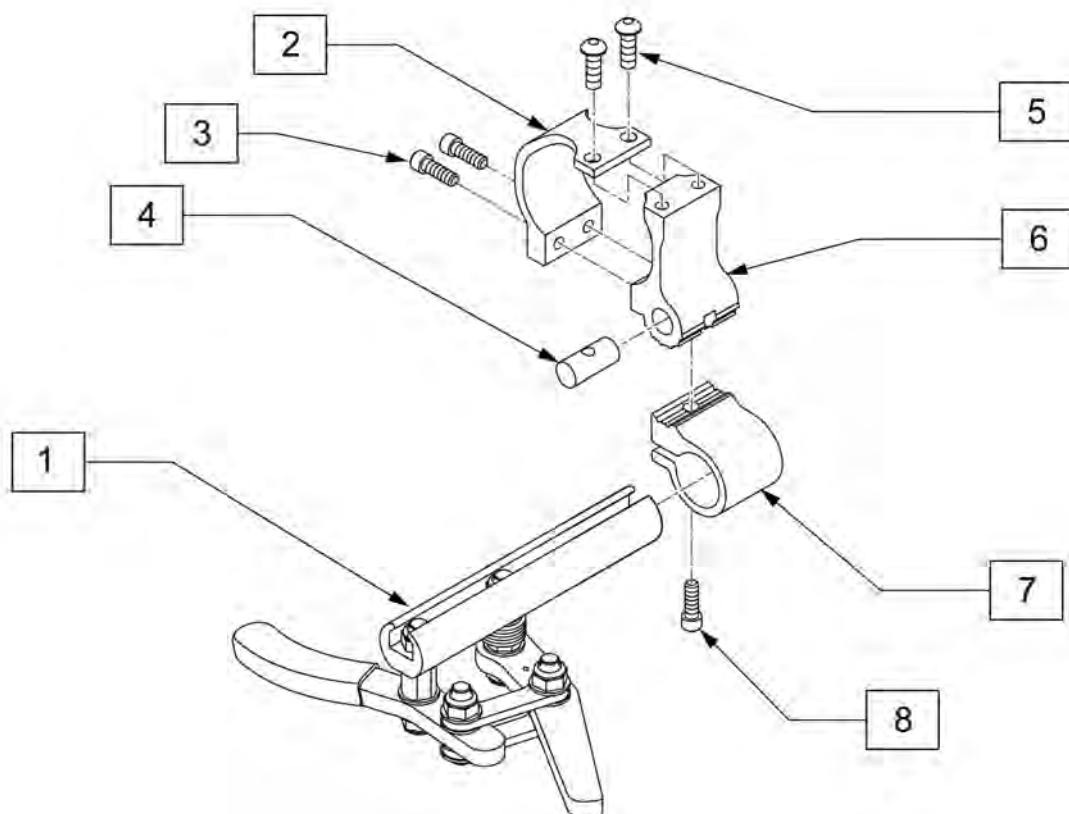


Pos.	Item Number	Description	Remarks
1	104726-001	ERGOSCISSOR, RH	RIGHT
1	104726-002	ERGOSCISSOR, LH	LEFT
3	102259-002	BHCS M5 x 0.8 x 20 W/ PL INORG	
4	112652	ROUND NUT SIZE 1	
5	103184-002	BHCS M5X0.8X8 FULL PL	
8	113317	SHCS M5X0.8X25 FULL PL	
9	104141	COVER CURVED WHEELLOCK VIN	
2-9	114441	WHEEL LOCK CLAMP KIT	

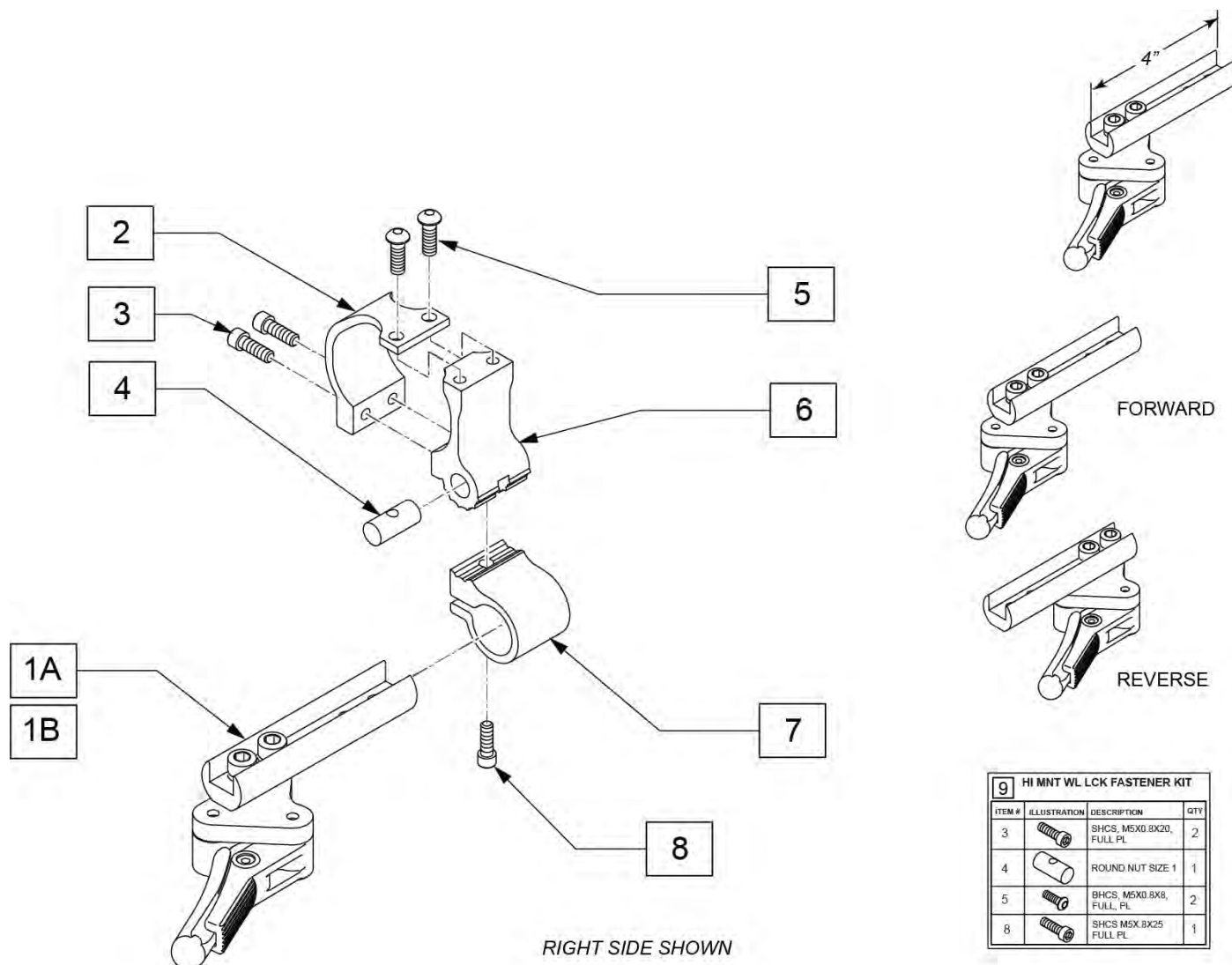


Pos.	Item Number	Description	Remarks
1A	066608-001	LIGHTWEIGHT C WHEEL LOCK LH	Left-Forward
1A	066608-002	LIGHTWEIGHT C WHEEL LOCK RH	Right-Forward
1B	119346	COMPACT WHL LCK REV LH	Left-Reverse
1B	119347	COMPACT WHL LCK REV RH	Right-Reverse
2-8	114441	WHEEL LOCK CLAMP KIT	
9	159464	HI MNT WL LCK FASTENER KIT	
1A-8	159475	COMPACT WHL LCK FWD L KIT	Left-Forward
1A-8	159476	COMPACT WHL LCK FWD R KIT	Right-Forward
1B-8	159477	COMPACT WHL LCK REV L KIT	Left-Reverse
1B-8	159478	COMPACT WHL LCK REV R KIT	Right-Reverse

NOTE: In order to mount the Compact Wheel Lock for the Ultra Lite package on a Q7, three rivnuts must exist on the underside of the frame rail. If they are not there a new frame must be ordered to add this option.

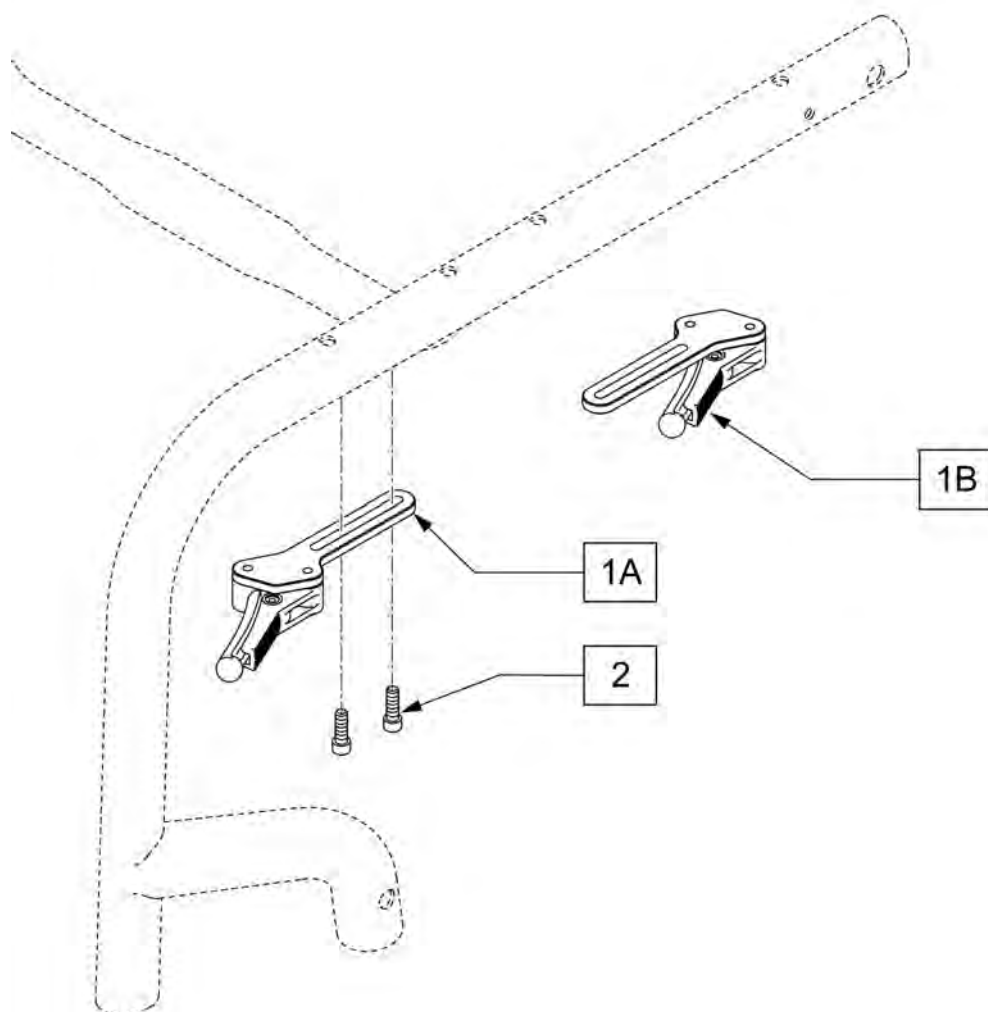

Pos.	Item Number	Description	Remarks
1A	066600-101	LIGHTWEIGHT C WHEEL LOCK LH STD	Left
1A	066600-102	LIGHTWEIGHT C WHEEL LOCK RH STD	Right
1A	066600-100	CMPCT WHLLCK LW STD ASSY PR	pair
1B	066600-201	LIGHTWEIGHT C WHEEL LOCK LH FRONT MOUNT	Left
1B	066600-202	LIGHTWEIGHT C WHEEL LOCK RH FRONT MOUNT	Right
1B	066600-200	CMPCT WHLLCK LW FRONT MOUNT ASSY PR	pair
2	102259-002	BHCS M5 X .8 X 20 W/PL INORG	

Note: used on High Mount Push and Pull Wheel Locks only.

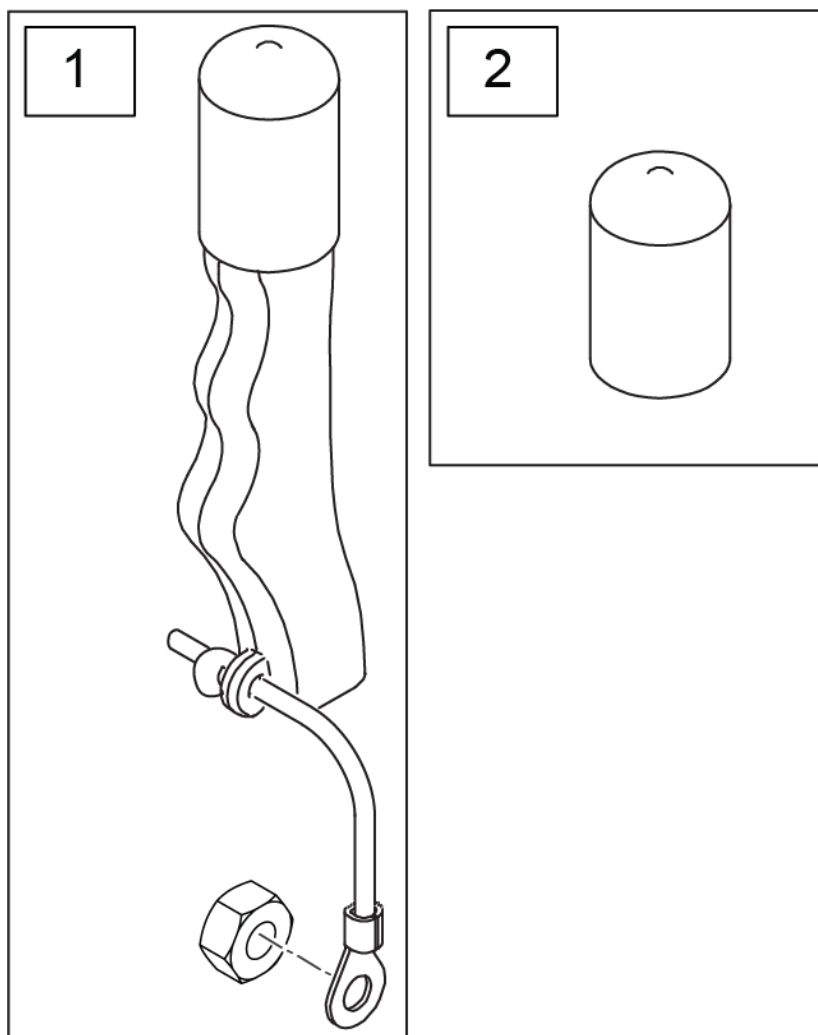

Pos.	Item Number	Description	Remarks
1	161930	WHEEL LOCK EXT ASSY (PAIR) KIT	Pair
2	560600	FOAM GRIP 1 1/2 X 0.705 X 0.1	

Note: this wheel utilizes Screw Mount Handrims - the handrim has a threaded hole that mounts the handrim to the wheel through a spacer and fastener.

Note: if Mag Wheel Assembly was ordered without Handrim and a Handrim is added to the wheel, qty 6 of 101932 and 281175 ARE REQUIRED to properly mount the handrim.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

Note: for Zippie X'Cape 6 Spoke Mags are only available with the One Arm Drive.



Pos.	Item Number	Description	Remarks
1-3	401094	20" MAG W/BRNG 6 SPOKE	20"
1-3	401093	22" MAG W/BRNG 6 SPOKE	22"(501mm)
1-3	401092	24" MAG W/BRNG 6 SPOKE	24"(540mm)
2	349500	1 1/8 X 1/2 BEARING	
3	521564	MAG BEARING SPACER	
4	101932	SPCR .194ID X .375 OD X .750	Used when previously ordered with Omit handrims or Natural Fit Handrims

Note: 5 Spoke Mag is not compatible with One Arm Drive or Projection Handrims.

Note: 5 Spoke Mag utilizes exclusive screw mounted handrims.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.



Pos.	Item Number	Description	Remarks
1,2	128985	20" 5 SPOKE MAG WHEEL	20" (451mm)
1,2	128986	22" 5 SPOKE MAG WHEEL	22" (501mm)
1,2	126405	24" MAG W/BRNG 5 SPOKE	24" (540mm)
2	349500	1 1/8 X 1/2 BEARING	
3	129325	20" MAG RIM STRIP	
3	129324	22" MAG RIM STRIP	
3	127912	24" MAG RIM STRIP	

Note: this Wheel utilizes Screw Mount Handrims; the Handrim has a threaded hole that mounts the Handrim to the Wheel through a spacer and fastener.

Note: the Lite Spoke Wheel has a total of 30 spokes.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

Note: when ordering replacement bearings, if dust shield on bearing has the following markings: MBL-Z2 6001 2RS, then order P/N 114097.



Pos.	Item Number	Description	Remarks
1A	118755	WHEEL ASSM GALLOP 20IN	20" (451)
1B	118756	WHEEL ASSM GALLOP 22IN	22" (501)
1C	107131	WHEEL RIM/SPOKE 24" ASSM	24" (540)
1D	118764	25 LT SPK W-HNDRM ANO ASSM GLP	25" (559)
1E	118760	WHEEL ASSM GALLOP 26IN	26" (590)
2A	114097	BEARING, REPLACEMENT, MBL	12.7mm i.d. - 28mm o.d. - 8mm wide; used prior to 11/29/10
2B	349500	1_125 X .5 BEARING	Beginning 11/29/10
3A	118100	MBL LT 20IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3B	118101	MBL LT 22IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3C	118102	MBL LT 24IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3D	118103	MBL LT 25IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes
3E	118104	MBL LT 26IN SPOKE RPLMNT KIT	Kit contains 6 inner and 6 outer spokes

Note: Wheels do not include rim strips, see "RIM STRIP" page to order correct rim strip.

Note: this wheel utilizes Tab Mount Handrims - the handrim has a straight tab that mounts to the wheel through the rim.

Note: 20" wheels have 18 spokes per wheel; 22", 24" & 25" wheels have 24 spokes per wheel; spokes are black.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.



Pos.	Item Number	Description	Remarks
1A	112857	ULTRA LIGHTWEIGHT WHEEL 20"	20"
1B	112858	ULTRA LIGHT WHEEL 22"	22"(501mm)
1C	112859	ULTRA LIGHTWEIGHT WHEEL 24"	24"(540mm)
1D	112860	ULTRA LIGHTWEIGHT WHEEL 25"	25"(559mm)
2	349500	1 1/8 X 1/2 BEARING	
3A	118105	ULTRA LT 20IN SPOKE RPLMNT KIT	Kit contains 12 spokes
3B	118106	ULTRA LT 22IN SPOKE RPLMNT KIT	Kit contains 12 spokes
3C	118107	ULTRA LT 24IN SPOKE RPLMNT KIT	Kit contains 12 spokes
3D	118108	ULTRA LT 25IN SPOKE RPLMNT KIT	Kit contains 12 spokes

Note: this wheel utilizes tab mount handrims - the handrim has a straight tab that mounts to the wheel through the rim.

Note: please see the Rear Wheel Finder for the complete Wheel Assembly with the Flex Rim.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.



Pos.	Item Number	Description	Remarks
1	109242	SPINERGY STD WHL 22" BLK SPK	22"(489mm)
1	762438	SPINERGY STD WHEEL 540 BLK SPK	24"(540mm)
1	762441	SPINERGY STD WHL 559 BLK SPK	25"(559mm)
1	762437	SPINERGY STD WHEEL 590 BLK SPK	26"(590mm)
1	762442	SPINERGY STD WHEEL 540 BLUE	24"(540mm)
1	762444	SPINERGY STD WHEEL 559 BLU	25"(559mm)
1	762446	SPINERGY STD WHEEL 590 BLUE	26"(590mm)
1	762443	SPINERGY STD WHEEL 540 RED	24"(540mm)
1	762445	SPINERGY STD WHEEL 559 RED	25"(559mm)
1	762447	SPINERGY STD WHEEL 590 RED	26"(590mm)
1	762456	SPINERGY STD WHEEL 540 WHT	24"(540mm)
1	762433	SPINERGY STD WHEEL 540 YELLOW	24"(540mm)
1	762462	SPINERGY STD WHL 559 YLW SPK	25"(559mm)
1	762431	SPINERGY STD WHEEL 590 YELLOW	26"(590mm)
2	349500	1 1/8 X 1/2 BEARING	

Note: this wheel utilizes tab mount handrims - the handrim has a straight tab that mounts to the wheel through the rim.

Note: please see the Rear Wheel Finder for the complete Wheel Assembly with the Flex Rim.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.



Pos.	Item Number	Description	Remarks
1	108858	SPINERGY LT EX WHL 22" BLK	22"(489mm); 12 Spoke
1	109215	SPINERGY LT EX WHL 22" BLU	22"(489mm); 12 Spoke
1	109216	SPINERGY LT EX WHL 22" RED	22"(489mm); 12 Spoke
1	109217	SPINERGY LT EX WHL 22" WHT	22"(489mm); 12 Spoke
1	109218	SPINERGY LT EX WHL 22" YLW	22"(489mm); 12 Spoke
1	101989-001	SPINERGY LT EX WHEEL 540 BLK	24"(540mm); 12 Spoke
1	101989-002	SPINERGY LT EX WHEEL 540 BLU	24"(540mm); 12 Spoke
1	101989-003	SPINERGY LT EX WHEEL 540 RED	24"(540mm); 12 Spoke
1	101989-005	SPINERGY LT EX WHEEL 540 WHT	24"(540mm); 12 Spoke
1	101989-004	SPINERGY LT EX WHEEL 540 YLW	24"(540mm); 12 Spoke
1	101991-001	SPINERGY LT EX WHEEL 559 BLK	25"(559mm); 12 Spoke
1	101991-002	SPINERGY LT EX WHEEL 559 BLU	25"(559mm); 12 Spoke
1	101991-003	SPINERGY LT EX WHEEL 559 RED	25"(559mm); 12 Spoke
1	101991-004	SPINERGY LT EX WHEEL 559 YLW	25"(559mm); 12 Spoke
1	101993-001	SPINERGY LT EX WHEEL 590 BLK	26"(590mm); 12 Spoke
1	101993-004	SPINERGY LT EX WHEEL 590 YLW	26"(590mm); 12 Spoke
2	349500	1 1/8 X 1/2 BEARING	

Note: this wheel utilizes tab mount handrims - the handrim has a straight tab that mounts to the wheel through the rim.

Note: please see the Rear Wheel Finder for the complete Wheel Assembly with the Flex Rim.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.




Pos.	Item Number	Description	Remarks
1	101988-001	SPINRGY SPT LT EX WHL 540 BLK	24"(540mm), 18 spoke
1	101988-002	SPINRGY SPT LT EX WHL 540 BLU	24"(540mm), 18 spoke
1	101988-003	SPINRGY SPT LT EX WHL 540 RED	24"(540mm), 18 spoke
1	101988-004	SPINRGY SPT LT EX WHL 540 YLW	24"(540mm), 18 spoke
1	101988-005	SPINRGY SPT LT EX WHL 540 WHT	24"(540mm), 18 spoke
1	101990-001	SPINRGY SPT LT EX WHL 559 BLK	25"(559mm), 18 spoke
1	101990-002	SPINRGY SPT LT EX WHL 559 BLU	25"(559mm), 18 spoke
1	101990-003	SPINRGY SPT LT EX WHL 559 RED	25"(559mm), 18 spoke
1	101990-004	SPINRGY SPT LT EX WHL 559 YLW	25"(559mm), 18 spoke
1	101990-005	SPINRGY SPT LT EX WHL 559 WHT	25"(559mm), 18 spoke
1	101992-001	SPINRGY SPT LT EX WHL 590 BLK	26"(590mm), 18 spoke
1	101992-002	SPINRGY SPT LT EX WHL 590 BLU	26"(590mm), 18 spoke
1	101992-003	SPINRGY SPT LT EX WHL 590 RED	26"(590mm), 18 spoke
1	101992-004	SPINRGY SPT LT EX WHL 590 YLW	26"(590mm), 18 spoke
1	101992-005	SPINRGY SPT LT EX WHL 590 WHT	26"(590mm), 18 spoke
2	349500	1 1/8 X 1/2 BEARING	

Note: this wheel utilizes Tab Mount Handrims - the handrim has a straight tab that mounts to the wheel through the rim.

Note: if changing the Rear Wheel Size on a chair with Anti-tips, new size Anti-tip Assemblies ARE REQUIRED to accommodate the new Rear Wheel Size.

Note: if retrofitting Natural Fit Handrims to a Mountain wheel assembly, the complete wheel assemblies will be required; see Rear Wheel Finder.

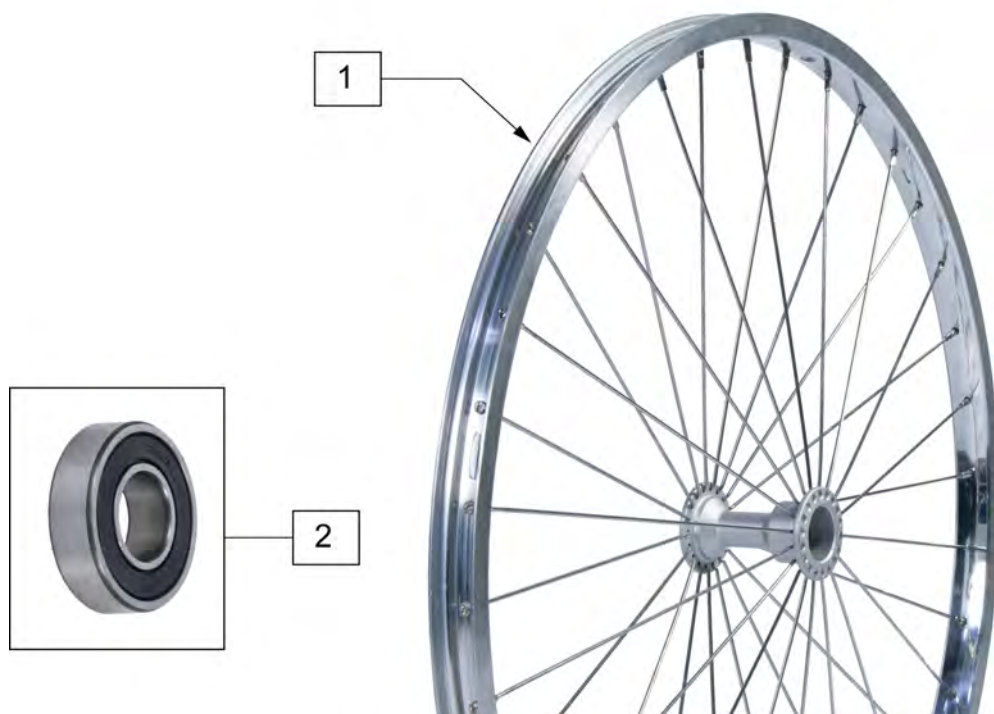


Pos.	Item Number	Description	Remarks
1	401075	24 RIM ASSM X 1.5 SUN PSH BTTN	22" (507mm); 4 Tabs; overall whl size w/tire = to 24"(540mm)
1	401076	24" HT RIM ASSM SUN PUSH BTTN	25" (559mm); 6 Tabs; overall whl size w/tire = to 26"(590mm)
1	108304	6 TAB RIM ASSM 24IN X 1.5	22" (507mm); use w/ Natural Fit HR's; overall whl size w/tire = to 24"(540mm)
2	349500	1 1/8 X 1/2 BEARING	




Pos.	Item Number	Description	Remarks
1	385237	16" X 1.75" PNEU TIRE	
1	385250	20" PNEUMATIC TIRE	
1	385275	22" PNEUMATIC TIRE	22" x 1 3/8 (501mm)
1	385300	24" PNEUMATIC TIRE	24" x 1 3/8 (540mm)
1	385325	26" PNEUMATIC TIRE	26" x 1 3/8 (590mm)
1	384614	24" x 2.125 PNEUMATIC TIRE	24"x2 1/8 (540mm) used w/ M6



Pos.	Item Number	Description	Remarks
1	386232	20" POLY TIRE STD	20"
1	386242	22" POLY TIRE STD	22" (501mm)
1	386252	24" POLY TIRE STD	24" (540mm)

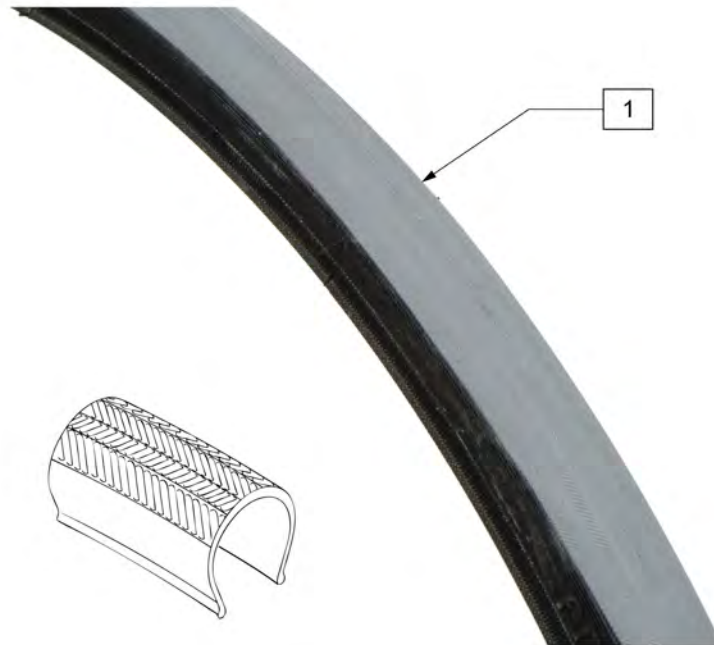

Pos.	Item Number	Description	Remarks
1	385280	22" x 1" HI PRES CLINCHER	22" (501mm)
1	389020	24" x 1" HI PRES CLNCHR TIRE	24" (540mm)
1	389012	QP 24" x 1" HT TURB BLK WR/BD 559	25" (559mm)
1	389030	26" x 1" HI PRES CLNCHR TIRE	26" (590mm)
1	105919	22" x 1" HI PRESS CLNCHR ULTRALIGHT	22" (489mm)

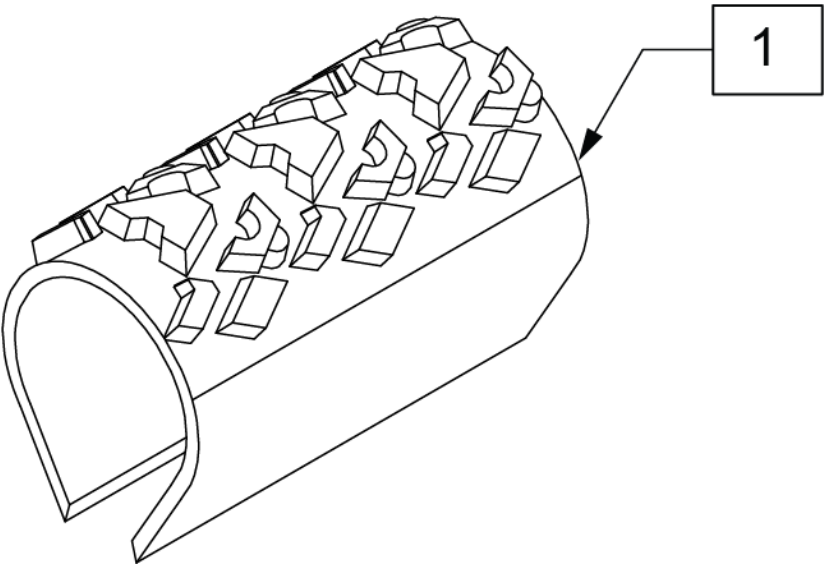

Pos.	Item Number	Description	Remarks
1	385354	24" X 1-3/8" KNOBBIE TIRE	24"(540mm)



Pos.	Item Number	Description	Remarks
1	384604	TIRE CS 24 X 2.1 SMOKER 507MM	22"(501mm)
1	384606	TIRE CS 26 X 2.1 SMOKER 559MM	25"(559mm)




Pos.	Item Number	Description	Remarks
1	102066-004	TIRE SCHWALBE MP 489 BLK	22" (489mm)
1	102066-001	TIRE SCHWBL MPE 540 BLK	24" (540mm)
1	102066-002	TIRE SCHWBL MPE 559 BLK	25" (559mm)
1	102066-003	TIRE SCHWBL MPE 590 BLK	26" (590mm)




Pos.	Item Number	Description	Remarks
1	113247	TIRE SCHWALBE SPEED RUN 24"	24"(540mm)
1	113248	TIRE SCHWALBE SPEED RUN 25"	25"(559mm)


Note: Shox Tire do not fit Topolino Wheels due to width.

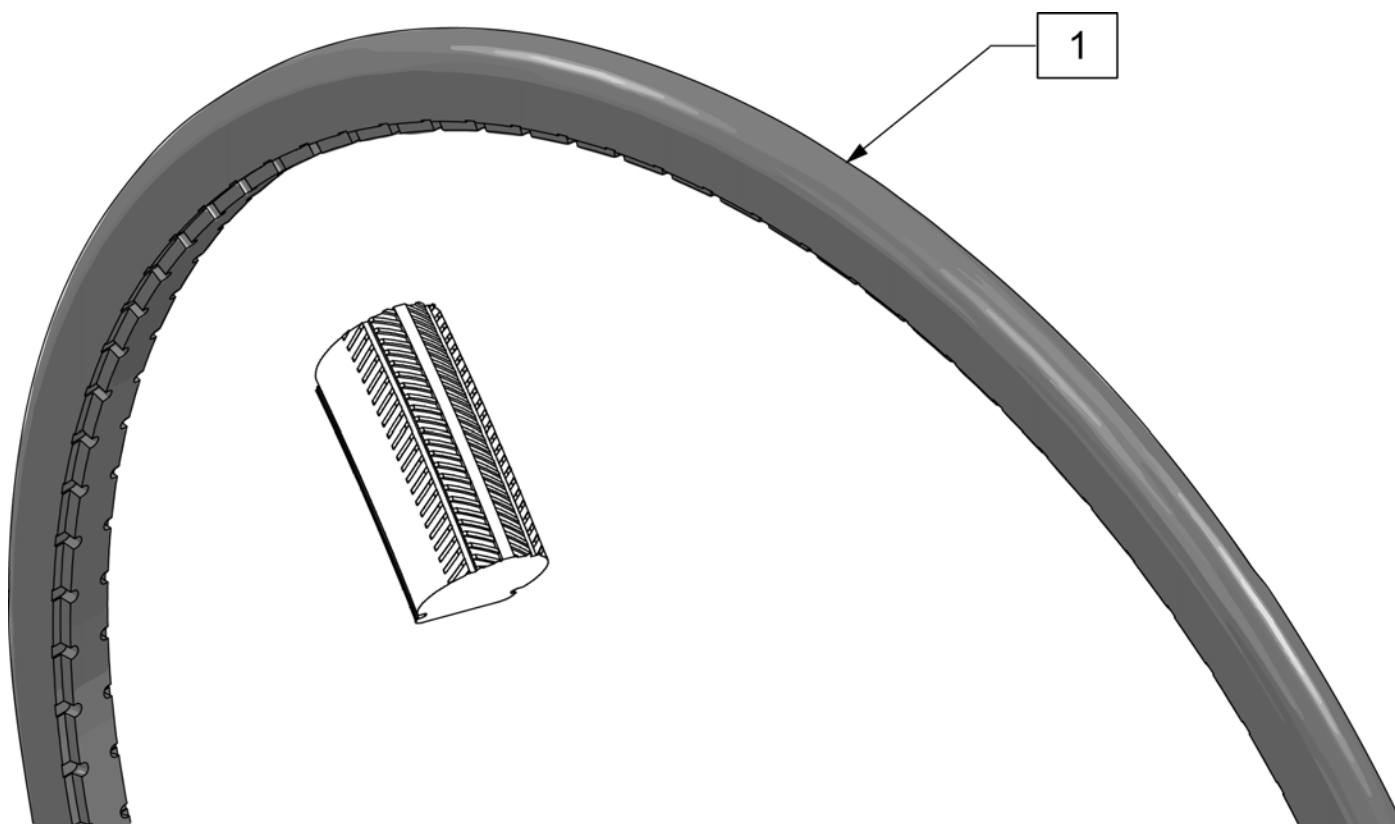




Pos.	Item Number	Description	Remarks
1	129924	TIRE SHOX CEW20"	20"(457mm) Black
1	129925	TIRE SHOX CEW22"	22"(501mm) Black
1	129926	TIRE SHOX CEW24"	24"(540mm) Black
1	129927	TIRE SHOX CEW25"	25"(559mm) Black

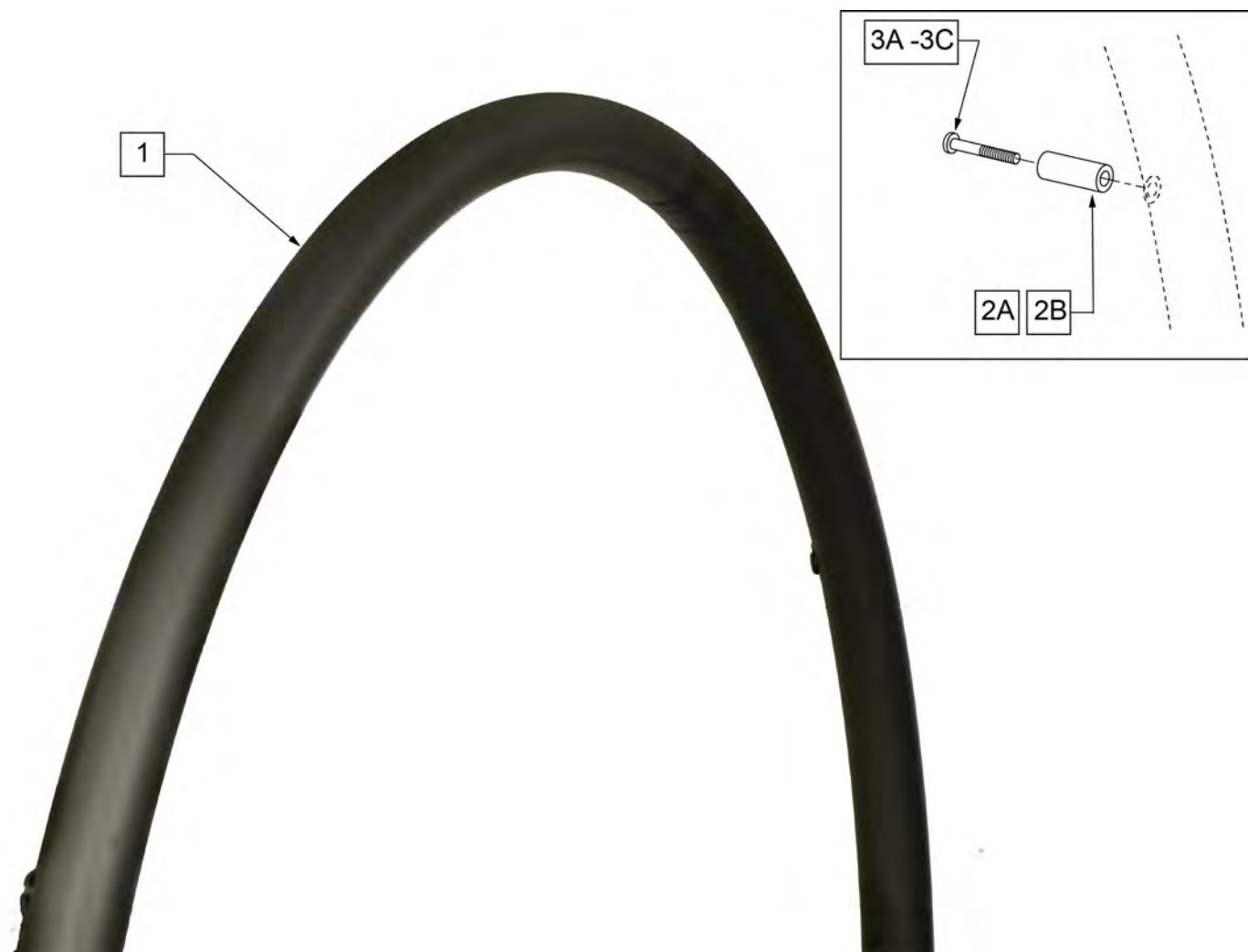


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	LITE SPOKE	>>>>>
1,2A,3A	159508	20IN LT SPK AA HNDRIM W/HDWR	20" LITE SPOKE
1,2A,3A	159509	22IN LT SPK AA HNDRIM W/HDWR	22" (501) LITE SPOKE
1,2A,3A	159510	24IN LT SPK AA HNDRIM W/HDWR	24" (540) LITE SPOKE
1,2A,3A	159511	25IN LT SPK AA HNDRIM W/HDWR	25" (559) LITE SPOKE
1,2A,3A	159512	26IN LT SPK AA HNDRIM W/HDWR	26" (590) LITE SPOKE
2A,3A	159513	LT SPK AL AN HNDRIM HDWR	LITE SPOKE, SET OF 6
>>>>>	>>>>>>>>>	SPOKE	>>>>>
1,2B,3B	425931	20" HR 6SMT W/HDWR	20" SPOKE
1,2B,3B	425941	22" HR 6SMT W/HDWR	22" (501) SPOKE
1,2B,3B	426000	24" HR 6SMT W/ HDWR	24" (540) SPOKE
2B,3B	159514	SPK AL AN HNDRIM HDWR	SPOKE, SET OF 6
>>>>>	>>>>>>>>>	MAG	>>>>>
1,3C	425934	20" HR 6SMT 6MAG W/HDWR	20" MAG
1,3C	425951	22" HR 6SMT 6MAG W/HDWR	22" (501) MAG
1,3C	426009	24" HR 6SMT 6MAG W/HDWR	24" (540) MAG
3C	281175	10-32 X 2 1/8 PAN PHIL SS	MAG
>>>>>	>>>>>>>>>	M6 SPOKE	>>>>>
1,2B,3B	425941	22" HR 6SMT W/HDWR	24" (540) SPOKE

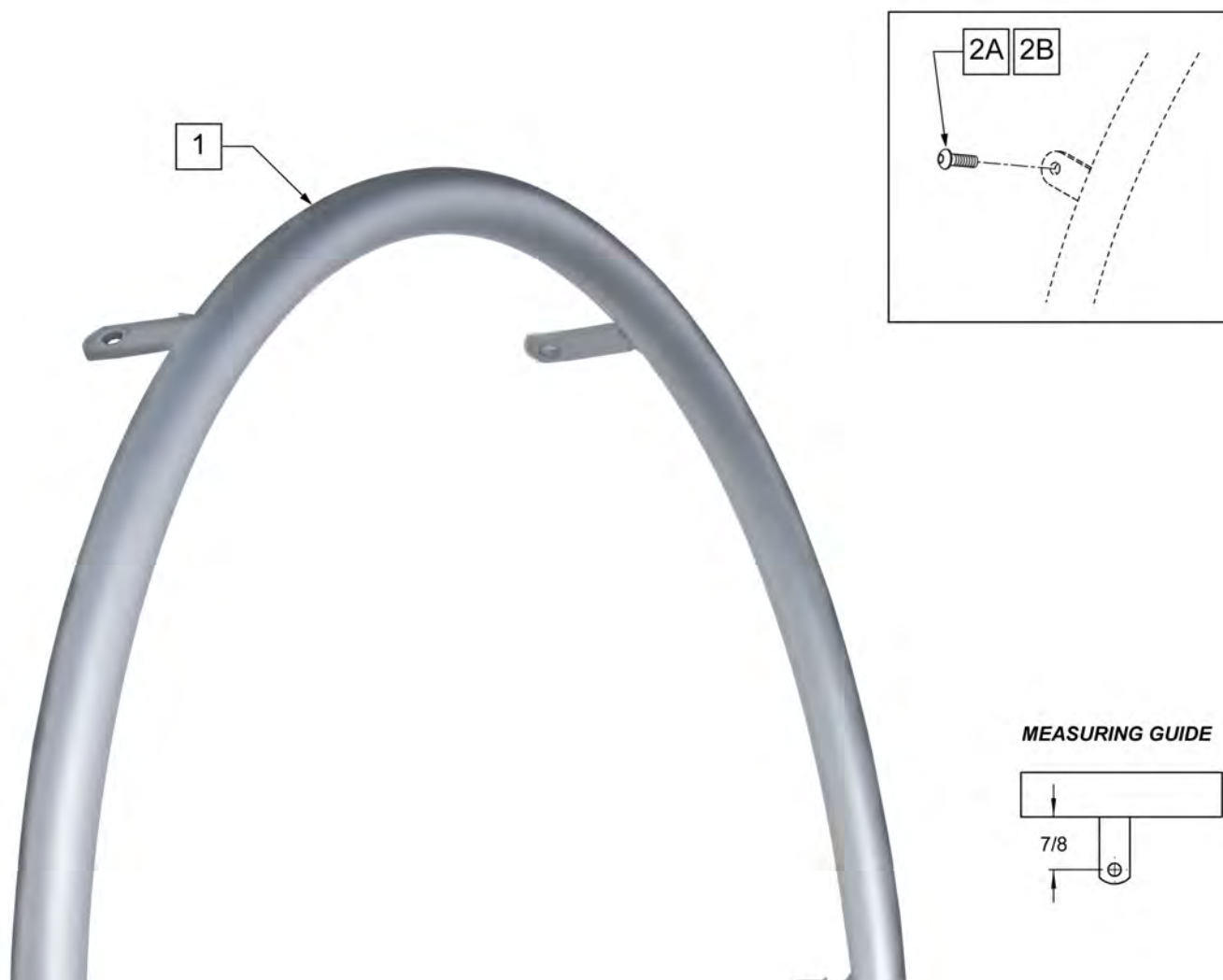

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>	ULTRA LITE SPOKE	>>>>>
1	100739-001	HANDRIM 22" WHL MBL 1260	22" (489)
1	100739-002	HANDRIM 24" WHL MBL 1260	24" (540)
1	100739-003	HANDRIM 26" WHL MBL 1260	26" (590)
2A	102259-001	BHCS M5 X .8 X 12 W/PL INORG	QTY 6 REG
>>>>>	>>>>>>>>	NEW ULTRA LITE SPOKE	>>>>>
1	066516-002	LIGHTWEIGHT HANDRIM 22"	22" (489)
1	066516-003	LIGHTWEIGHT HANDRIM 24"	24" (540)
1	066516-004	LIGHTWEIGHT HANDRIM 25"	26" (559)
2B	012931	BHCS M5 X 10 W/PL BLK CL 10.9	QTY 6 REG

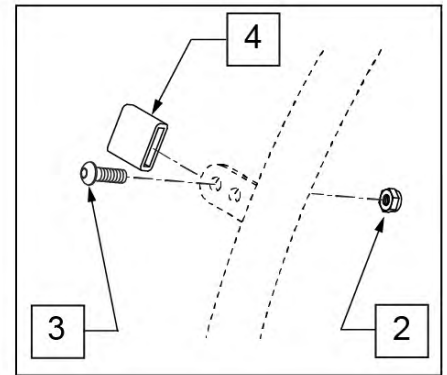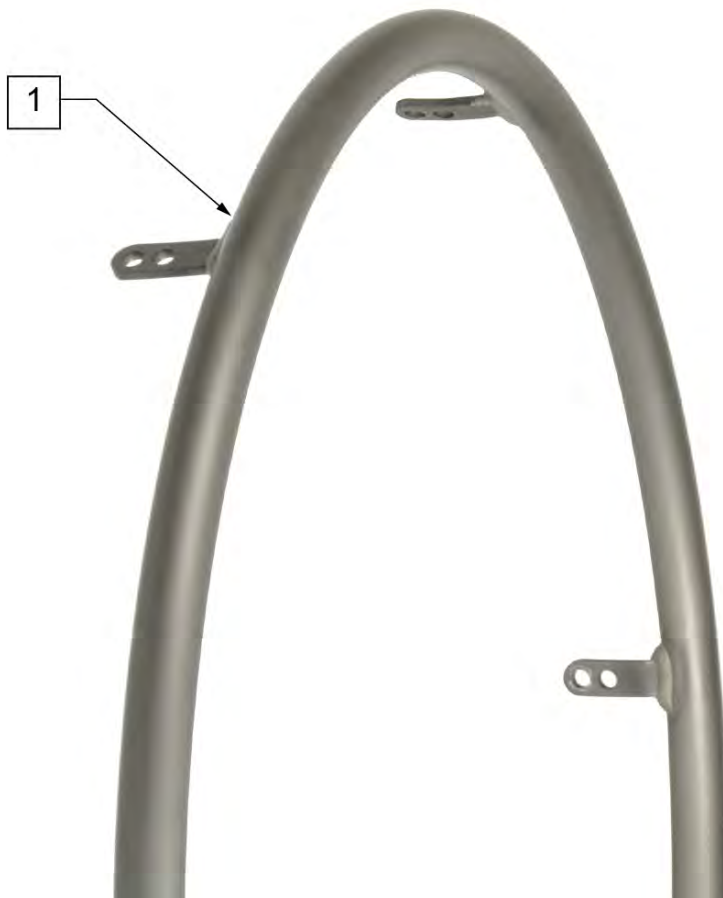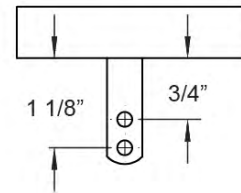**MEASURING GUIDE**

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>>	CROSS SPOKE & SPOKE	>>>>>
1 -4	425095	HANDRIM 18" 4T W/HDWR ANO	18" SPOKE
1	529529	QP 24" HR .065 (6) TAB ANO	24" (540) CROSS SPOKE -SHORT & REG TAB
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	25" (590) CROSS SPOKE
>>>>>	>>>>>>>>>>	CROSS SPOKE QP (QUICKIE PERFORMANCE)	>>>>>
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) CROSS SPOKE TENNIS & BBALL
1 -4	425045	24" HR SHT(6T)ANO W/X/S HDWR	25" (590) CROSS SPOKE TENNIS & BBALL-SHORT TAB
1 -4	425048	26" HR (6T)ANO W/X/S HDWR	26" (700) CROSS SPOKE TENNIS & BBALL -REG TAB
1 -4	425055	26" HR SHT(6T)ANO W/X/S HDWR	26" (700) CROSS SPOKE TENNIS & BBALL-SHORT TAB
1 -4	426015	QP 24" HR .049 W/HDWR SMT	24" (540) TENNIS
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	25" (559) TENNIS & BASKETBALL
1 -4	426050	QP 26" HR .065 W/HDWR (6T)	26" (700) TENNIS & BASKETBALL
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) TENNIS & BASKETBALL
>>>>>	>>>>>>>>>>	SPINERGY	>>>>>
1	529529	QP 24" HR .065 (6) TAB ANO	24" (540) SPINERGY
1 -4	425038	QP 24" HR .049 W/HDWR (6T)	25" (559) SPINERGY
1 -4	425050	26" HR 6T BB/TN W/HDWR	26" (590) SPINERGY
>>>>>	>>>>>>>>>>	MOUNTAIN	>>>>>
1 -4	425020	22" HR 4T W/HDWR	24" (540) MOUNTAIN
1 -4	425030	24" HR 6T BB/HTBB W/HDWR	26" (590) MOUNTAIN
2-4	159515	AA TAB MOUNT HANDRIM HARDWARE	SET OF 6

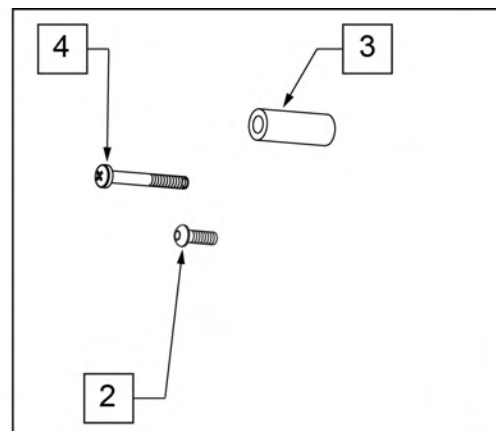


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>>	SPOKE & LITE SPOKE	>>>>>
1 -4	425933	20" HR 6SMT PLST W/HDWR	20" SPOKE
1 -4	425943	22" HR 6SMT PLST W/HDWR	22" (501) SPOKE
1 -4	426004	24" HR 6SMT PLST W/HDWR	24" (540) SPOKE
>>>>>	>>>>>>>>>>	MAG	>>>>>
1 -4	425935	20" HR 6SMT 6MAG PLST W/HDWR	20" MAG
1 -4	425953	22" HR 6SMT 6MAG PLST W/HDWR	22" (501) MAG
1 -4	426013	24" HR 6SMT 6MAG PLST W/HDWR	24" (540) MAG
>>>>>	>>>>>>>>>>	M6 SPOKE	>>>>>
1 -4	425953	22" HR 6SMT 6 MAG PLST W/ HDWR	24" (540) M6 SPOKE
2,3	159534	PLST COATED HNDRM HARDWARE	SPOKE; SET OF 6
4	281175	10 -32 x 2 1/8 PAN PHIL SS	6 SPOKE MAG



Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	MOUNTAIN	>>>>>
1,2	425022	22" HR 4T PLST W/ HDWR	24" (540)
1,2	425032	24"HR 6T PLST BB/NTBB/TN W/HDWR	26" (590)
>>>>>	>>>>>>>>>	SPINERGY	>>>>>
1,2	425039	24"HR 6T BLK PLST W/HDWR	24" (540)
1,2	425032	24"HR 6T PLST BB/NTBB/TN W/HDWR	25" (559)
1,2	425052	26" HR 6T PLST BB/TN W/HDWR	26" (590)
>>>>>	>>>>>>>>>	QP (QUICKIE PERFORMANCE)	>>>>>
1,2	425032	24"HR 6T PLST BB/NTBB/TN W/HDWR	25" (559)
1,2	425052	26" HR 6T PLST BB/TN W/HDWR	26" (590)
1,2	426060	QP 26" HR W/ HDWR (6T) PLASTIC	26" (700)
2	159535	PLST COATED TAB HNDRM HDWR	


Note: Screw Mount Handrims are utilized with Lite Spoke, Spoke and 6 Spoke Mag Wheels.

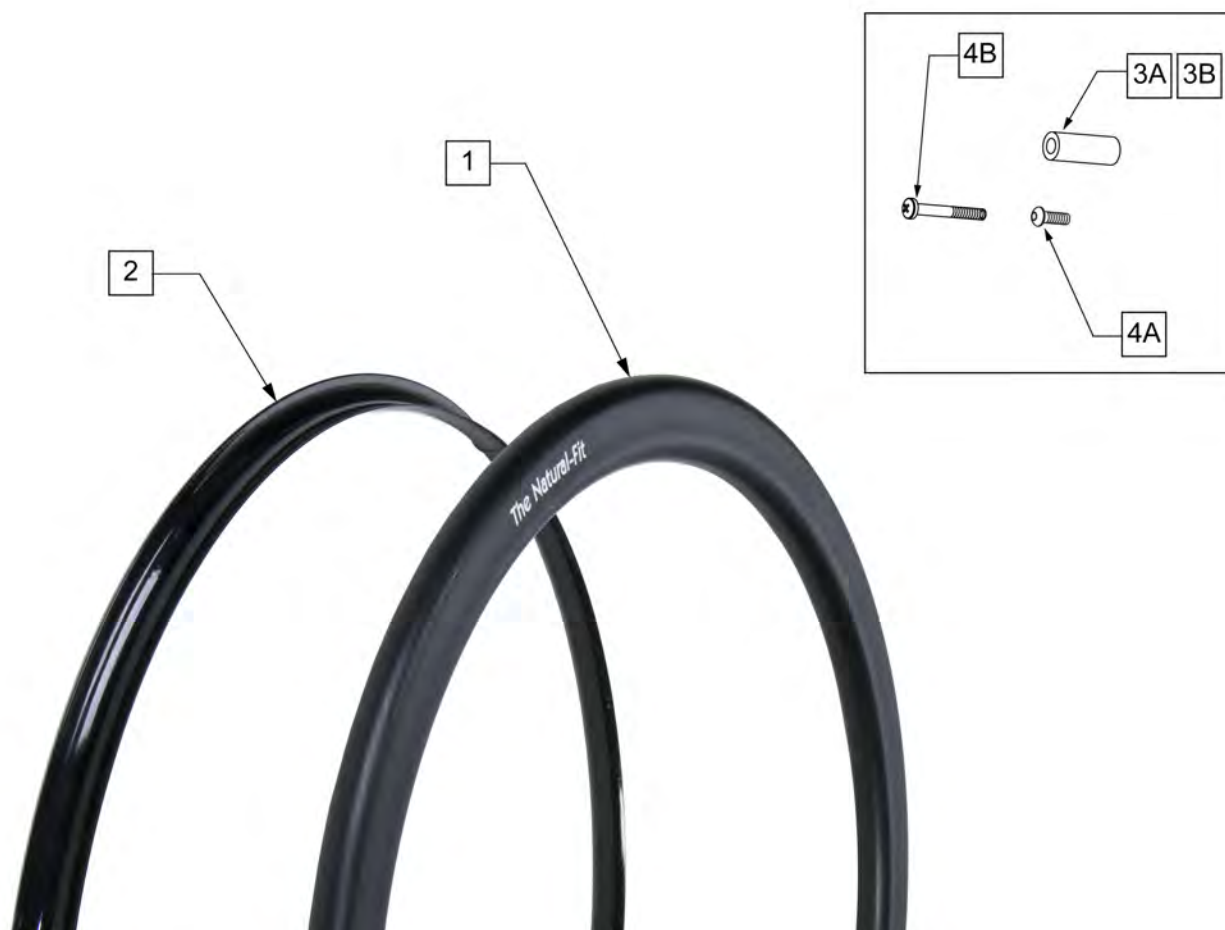

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	STD GRIP THUMB GUARD	>>>>>
2	102168-001	GUARD THUMB NAT FIT 22 PC BLK	22" (501)- Std Grip Thumb Guard
2	102168-003	GUARD THUMB NAT FIT 24 PC BLK	24" (540)- Std Grip Thumb Guard
2	102168-005	GUARD THUMB NAT FIT 25 PC BLK	25" (559)- Std Grip Thumb Guard
2	102168-007	GUARD THUMB NAT FIT 26 PC BLK	26" (590)- Std Grip Thumb Guard
>>>>>	>>>>>>>>>	SUPER GRIP THUMB GUARD	>>>>>
2	102168-002	GUARD THUMB NAT FIT 22 SG BLK	22" (501)- Super Grip Thumb Guard
2	102168-004	GUARD THUMB NAT FIT 24 SG BLK	24" (540)- Super Grip Thumb Guard
2	102168-006	GUARD THUMB NAT FIT 25 SG BLK	25" (559)- Super Grip Thumb Guard
2	102168-008	THUMB GUARD NAT FIT 26 SG BLK	26" (590)- Super Grip Thumb Guard
>>>>>	>>>>>>>>>	NATURAL FIT LITE HANDRIM	>>>>>
1,3A,4A	159516	22 NAT-FIT LT HNDRIM W/HDWR	22" (501)
1,3A,4A	159517	24 NAT-FIT LT HNDRIM W/HDWR	24" (540)
1,3A,4A	159518	25 NAT-FIT LT HNDRIM W/HDWR	25" (559)
1,3A,4A	159519	26 NAT-FIT LT HNDRIM W/HDWR	26" (590)
>>>>>	>>>>>>>>>	NATURAL FIT HANDRIM	>>>>>
1,3A,4A	159520	22 NAT-FIT HNDRIM W/HDWR	22" (501)
1,3A,4A	159521	24 NAT-FIT HNDRIM W/HDWR	24" (540)

Pos.	Item Number	Description	Remarks
1,3A,4A	159522	25 NAT-FIT HNDRIM W/HDWR	25" (559)
1,3A,4A	159523	26 NAT-FIT HNDRIM W/HDWR	26" (590)
3A,4A	159524	NAT-FIT HNDRIM LT HDWR SET 6	USED W/ LITE SPOKE & SPOKE; SET OF 6
3B	101932	SPCR .194ID X .375 OD X .750	USED W/ MAG
4B	281175	10-32 X 2 1/8 PAN PHIL SS	USED W/ MAG

Note: Tab Mount Hand Rims are utilized with Spinergy & Toplino Wheels.

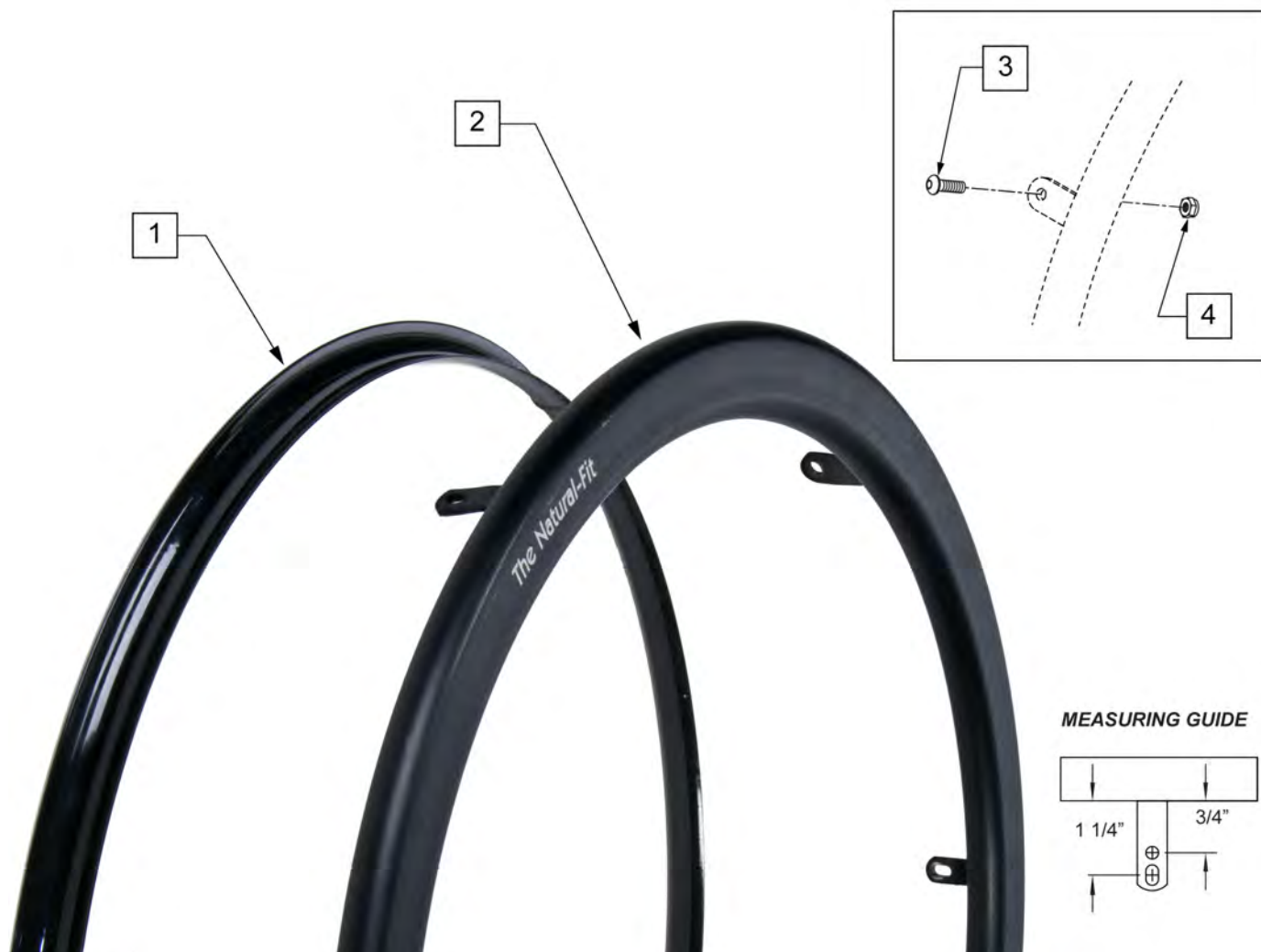


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	STD GRIP THUMB GRIP	>>>>>
1	102168-001	GUARD THUMB NAT FIT 22 PC BLK	22" (501)
1	102168-003	GUARD THUMB NAT FIT 24 PC BLK	24" (540)
1	102168-005	GUARD THUMB NAT FIT 25 PC BLK	25" (559)
1	102168-007	GUARD THUMB NAT FIT 26 PC BLK	26" (590)
>>>>>	>>>>>>>>>	SUPER GRIP THUMB GUARD	>>>>>
1	102168-002	GUARD THUMB NAT FIT 22 SG BLK	22" (501)
1	102168-004	GUARD THUMB NAT FIT 24 SG BLK	24" (540)
1	102168-006	GUARD THUMB NAT FIT 25 SG BLK	25" (559)
1	102168-008	THUMB GUARD NAT FIT 26 SG BLK	26" (590)
>>>>>	>>>>>>>>>	NATURAL FIT LITE HANDRIM	>>>>>
2-4	159525	22 NAT-FIT LT TB HNDRIM W/HDWR	22" (501)
2-4	159526	24 NAT-FIT LT TB HNDRIM W/HDWR	24" (540)
2-4	159527	25 NAT-FIT LT TB HNDRIM W/HDWR	25" (559)


Pos.	Item Number	Description	Remarks
2-4	159528	26 NAT-FIT LT TB HNDRI W/HDWR	26" (590)
>>>>>	>>>>>>>>>	NATURAL FIT -W/O THUMB GRIP	>>>>>
2-4	159529	22 NAT-FIT TB HNDRI W/HDWR	22" (501)
2-4	159530	24 NAT-FIT TB HNDRI W/HDWR	24" (540)
2-4	159531	25 NAT-FIT TB HNDRI W/HDWR	25" (559)
2-4	159532	26 NAT-FIT TB HNDRI W/HDWR	26" (590)
3,4	159533	NAT-FIT TAB MNT HARDWARE	SET OF 6

NOTE: Not all Projection Handrims are offered on all chairs, check the order form for compatibility.


NOTE: Projection Handrims are only available with the Traditional 6 Spoke Mag and Quickie Spoke Wheels.

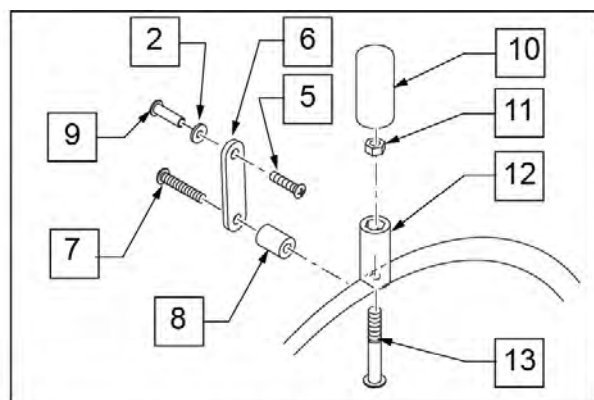

MAG VIEW

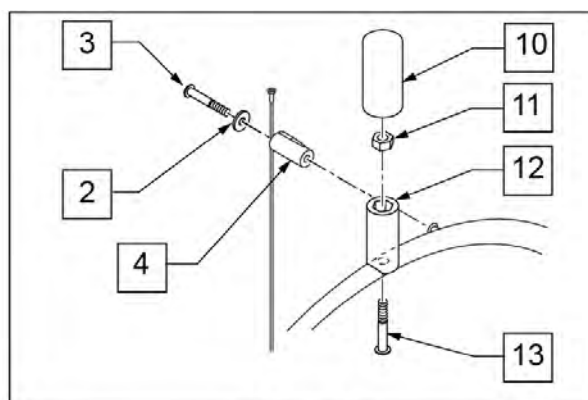

SPOKE VIEW

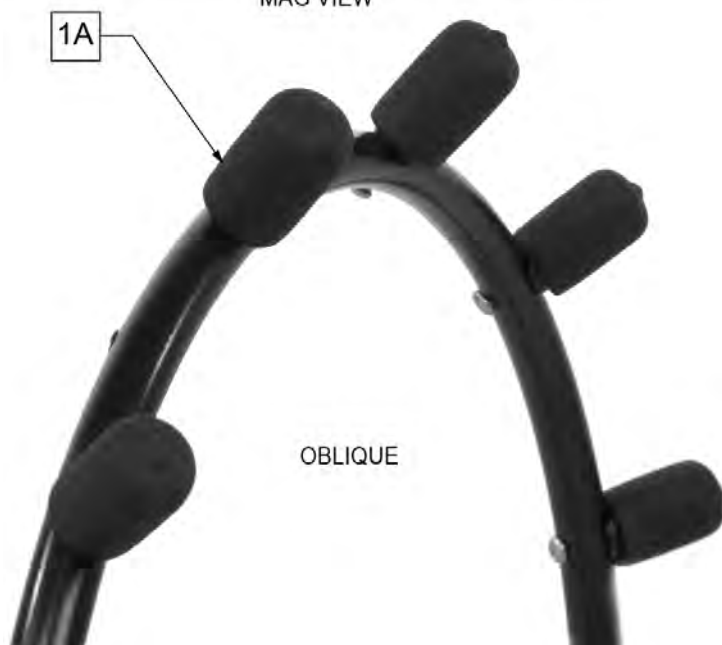


OBLIQUE




VERTICAL

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	8" OBLIQUE	>>>>>
1A,2-4,10-13	427003	PROJ 24/26" HR 8OBLQ W/HDWR	24" (540), 25" (559), 26" (590) SPOKE
1A,2,3-13	427511	PROJ 24" HR 6MAG 8OBLQ W/HDWR	24" (540) MAG
>>>>>	>>>>>>>>>	12" OBLIQUE	>>>>>
1A,2-4,10-13	427007	PROJ 24/26" HR 12OBLQ W/HDWR	24" (540), 25" (559), 26" (590) SPOKE
1A,2,5-13	427515	PROJ 24" HR 6MAG 12OBLQ W/HDWR	24" (540) MAG
>>>>>	>>>>>>>>>	8" VERTICAL	>>>>>
1B,4,10-13	427002	PROJ 24/26" HR 8VERT W/HDWR	24" (540), 25" (559), 26" (590) SPOKE
1B,2,5-13	427510	PROJ 24" HR 6 MAG 8 VERT W/ HDWR	24" (540) MAG
>>>>>	>>>>>>>>>	12" VERTICAL	>>>>>
1B,4,10-13	427006	PROJ 24/26" HR 12VERT W/HDWR	24" (540), 25" (559), 26" (590) SPOKE

Pos.	Item Number	Description	Remarks
1B,2,5-13	427514	PROJ 24" HR 6MAG 12VERT W/HDWR	24" (540) MAG
2	261300	3/16 WASHER FLAT	
3	281170	10-32 X 1 5/8 PAN PHIL SS	SPOKE ONLY
4	561046	PROJECTION SPOKE MOUNT	SPOKE ONLY
5	281120	10/32 X 5/8 PAN HD PHIL	MAG ONLY
6	505004	TOGGLE 2 - HOLE	MAG ONLY
7	281147	10-32 X 1 1/4 PAN PHIL	MAG ONLY
8	561037	HANDRIM SPACER MAG	MAG ONLY
10	560600	1 1/2 x .705 x .187 FOAM GRIP	
11	106935	NUT NYLOCK THIN 1/4-28 ROHS	
12	561044	PROJECTION PEG	
13	226155	1 1/4 -28 x 2 1/4 BH C/S	

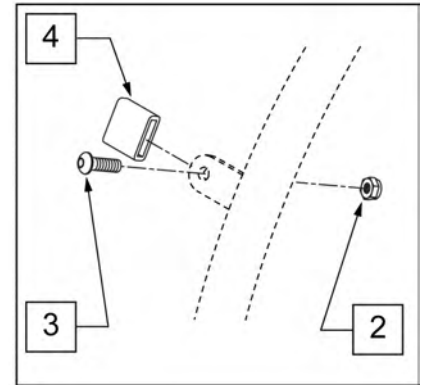


MEASURING GUIDE

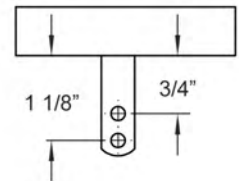


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	PAINTED REG TAB	>>>>>
1	529528	QP 24" HR .065 (6)TAB BLK PNTD	24" (540)
1-4	425041	24" HR (6T)BLK W/X/S HDWR	25" (559)
1	529527	QP 26" HR .065 (6)TAB BLK PNTD	26" (590)
1-4	425047	26" HR (6T)BLK W/X/S HDWR	26" (700)
>>>>>	>>>>>>>>>	PAINTED SHORT TAB	>>>>>
1	529528	QP 24" HR .065 (6)TAB BLK PNTD	24" (540)
1-4	425044	24" HR SHT(6T)BLK W/X/S HDWR	25" (559)
1	529527	QP 26" HR .065 (6)TAB BLK PNTD	26" (590)
1-4	425053	26" HR SHT(6T)BLK W/X/S HDWR	26" (700)
2	200100-QU	10-32 NYLOCK NUT	
3	281110	10-32 X 1/2 PAN PHIL	
4	580301	HR TAB COVER BLK	



NOTE: IF MAG WHEEL ASSEMBLY WAS ORDERED WITHOUT HANDRIM AND THIS HANDRIM IS ADDED TO THE WHEEL QTY 6 OF 101932 AND 281175 MUST BE ORDERED TO PROPERLY MOUNT THE HANDRIM.

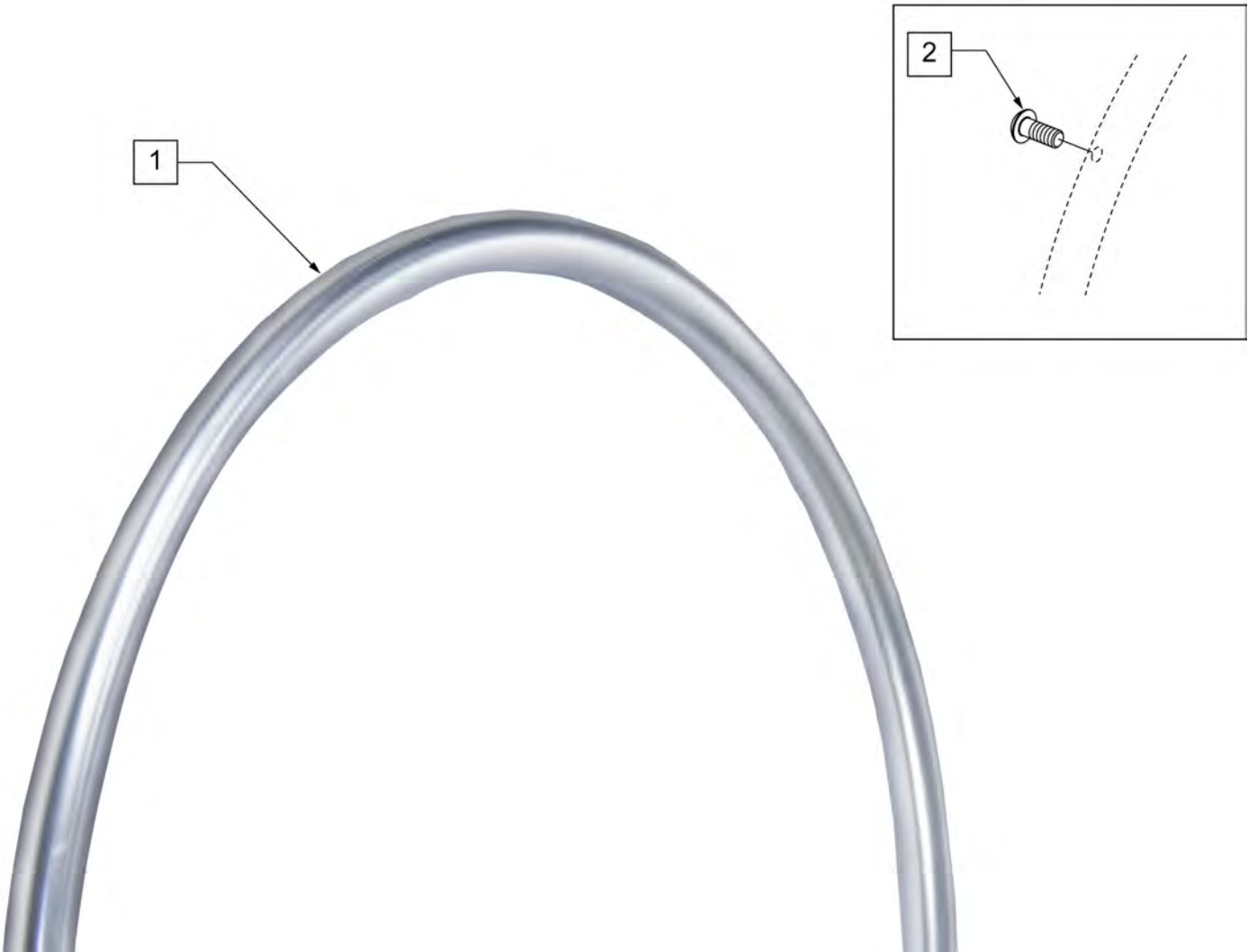


Pos.	Item Number	Description	Remarks
1,2	426026	20" SCW MT HR CHRM 6S W/HDWR	20"
1,2	426024	22" SCW MT HR CHRM 6S W/HDWR	22" (501)
1,2	426011	24 IN SCW MT HR CHRM 6S W/HDWR	24" (540)
2	281175	10-32 X 2 1/8 PAN PHIL SS	QTY 6 REG

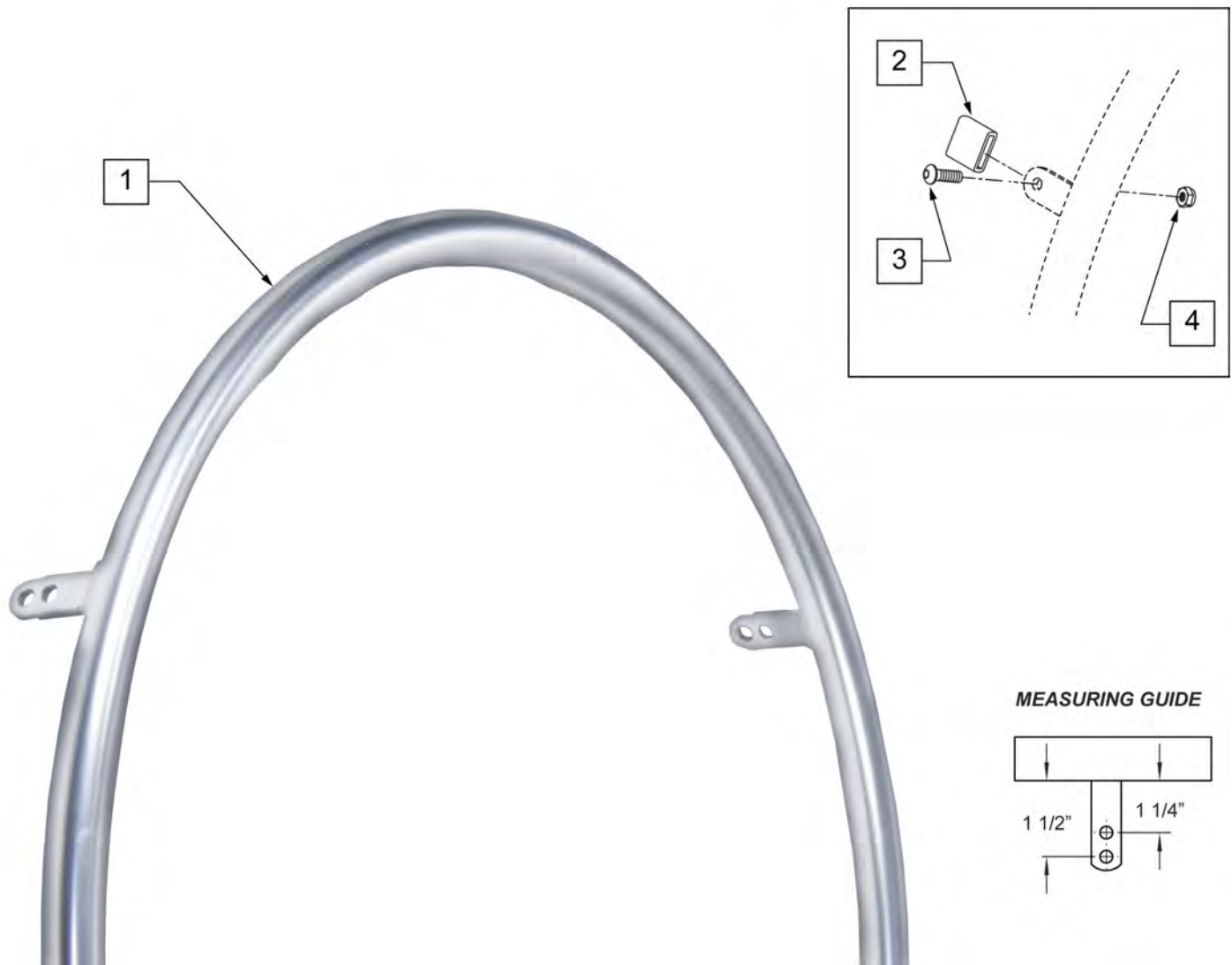


Pos.	Item Number	Description
1 -4		Assembly Obsolete
2	580301	HR TAB COVER BLK
3	281110	10-32 X 1/2 PAN PHIL
4	200100-QU	10-32 NYLOCK NUT

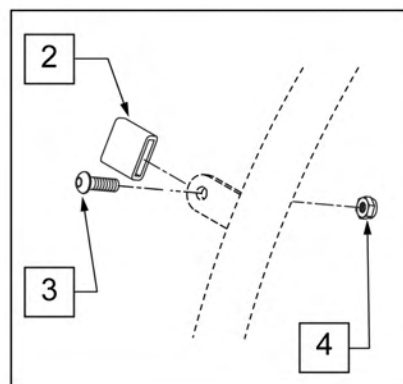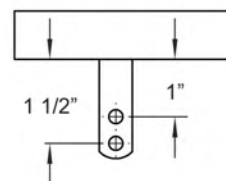**MEASURING GUIDE**

Pos.	Item Number	Description
1		handrim no longer available
2	580301	HR TAB COVER BLK
3	281110	10-32 X 1/2 PAN PHIL
4	200100-QU	10-32 NYLOCK NUT

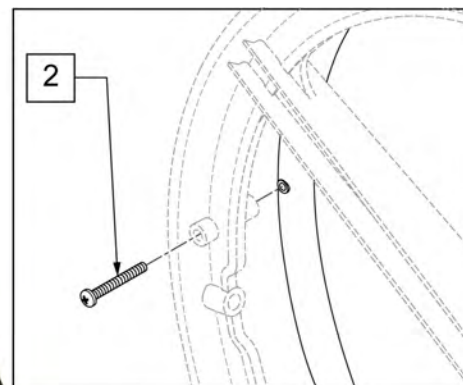


Pos.	Item Number	Description
1,2	426020	24IN HR 6SMT COMPOSITE W/HDWR
1,2	426021	22" HR 6SMT COMPOSITE W/HDWR
1,2	426022	20" HR 6SMT COMPOSITE W/HDWR
2	220100	#10 X 2 1/4" SELF TAPPNG SCREW

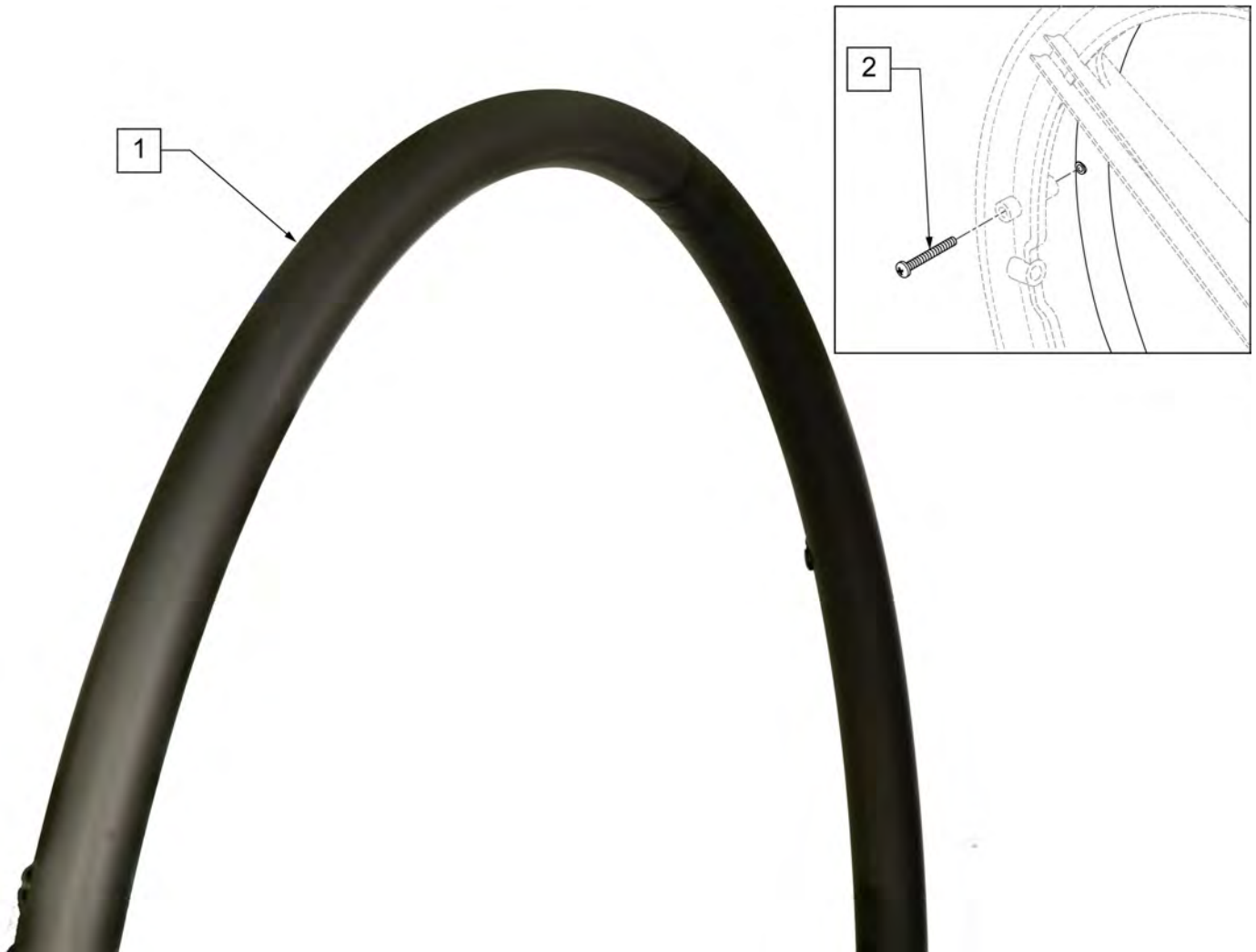

Pos.	Item Number	Description
1,2	425016	20" SCW MT HR PNTD 6S W/HDWR
1,2	425026	22" HR 6SMT PNTD 6MAG W/HDWR
1,2	426005	24" SCW MT HR PNTD 6S W/HDWR
2	220100	#10 X 2 1/4" SELF TAPPNG SCREW



Pos.	Item Number	Description
2,3	159507	5 SPK HNDRM HDWR
1-3	136155	20" PLSTC COATED HND RIM KIT
1-3	136154	22" PLSTC COATED HND RIM KIT
1-3	136153	24" PLSTC COATED HND RIM KIT

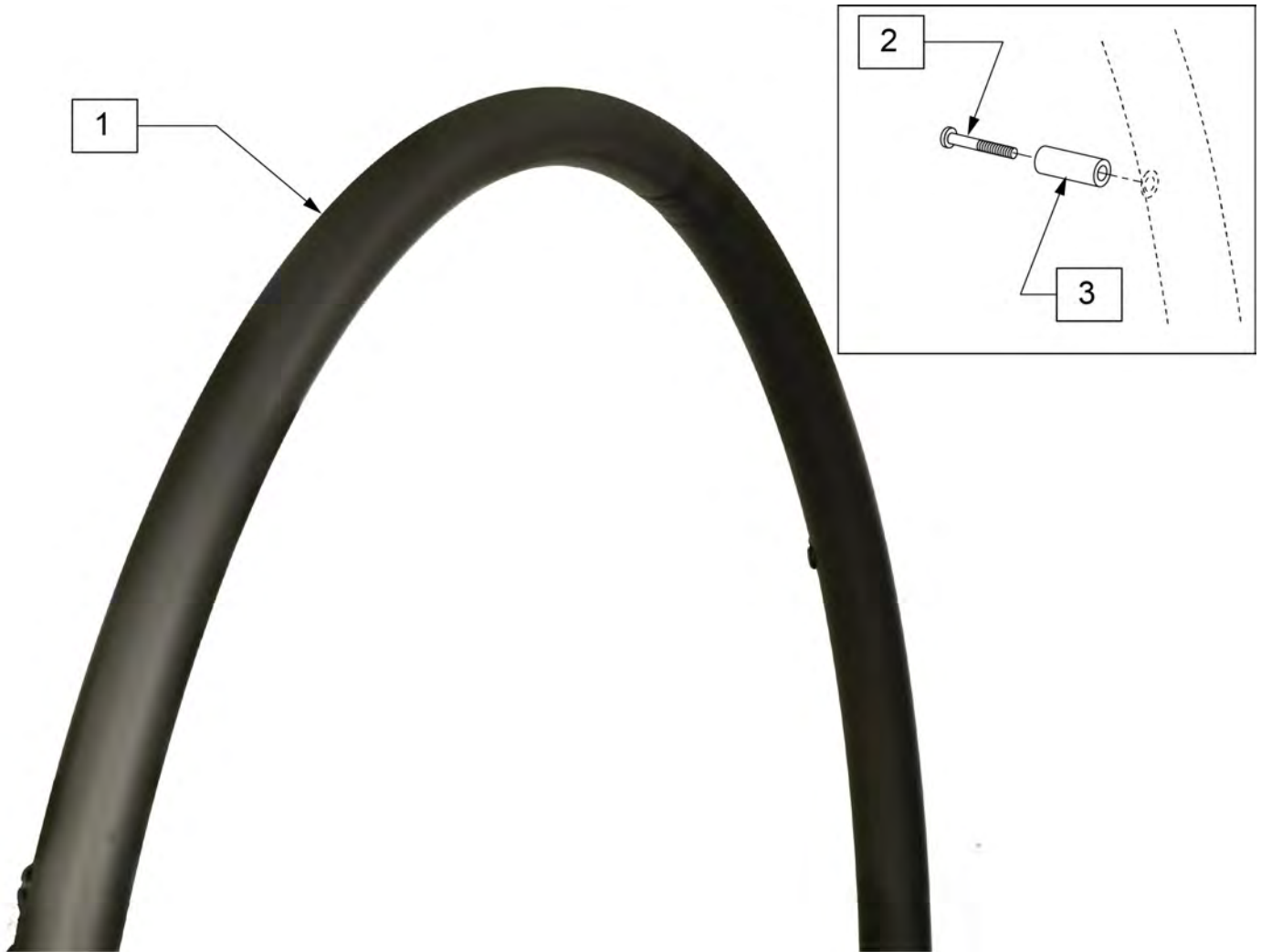

Pos.	Item Number	Description	Remarks
1-3	136158	20" ALU ANO HND RIM KIT	
1-3	136157	22" ALU ANO HND RIM KIT	
1-3	136156	24" ALU ANO HND RIM KIT	
2,3	159507	5 SPK HNDRM HDWR	set of 5

Note: Handrim is Latex free.


Note: used on Spinergy SPOX and LX wheels.



Pos.	Item Number	Description	Remarks
1-4	159501	24IN HANDRIM TAB Q-GRIP W/HDWR	
1-4	159502	25IN HANDRIM TAB Q-GRIP W/HDWR	
2-4	159503	HANDRIM TAB Q-GRIP HARDWARE	set of 6


Note: Handrim is Latex free.

Note: used on Lite Spoke Wheels.

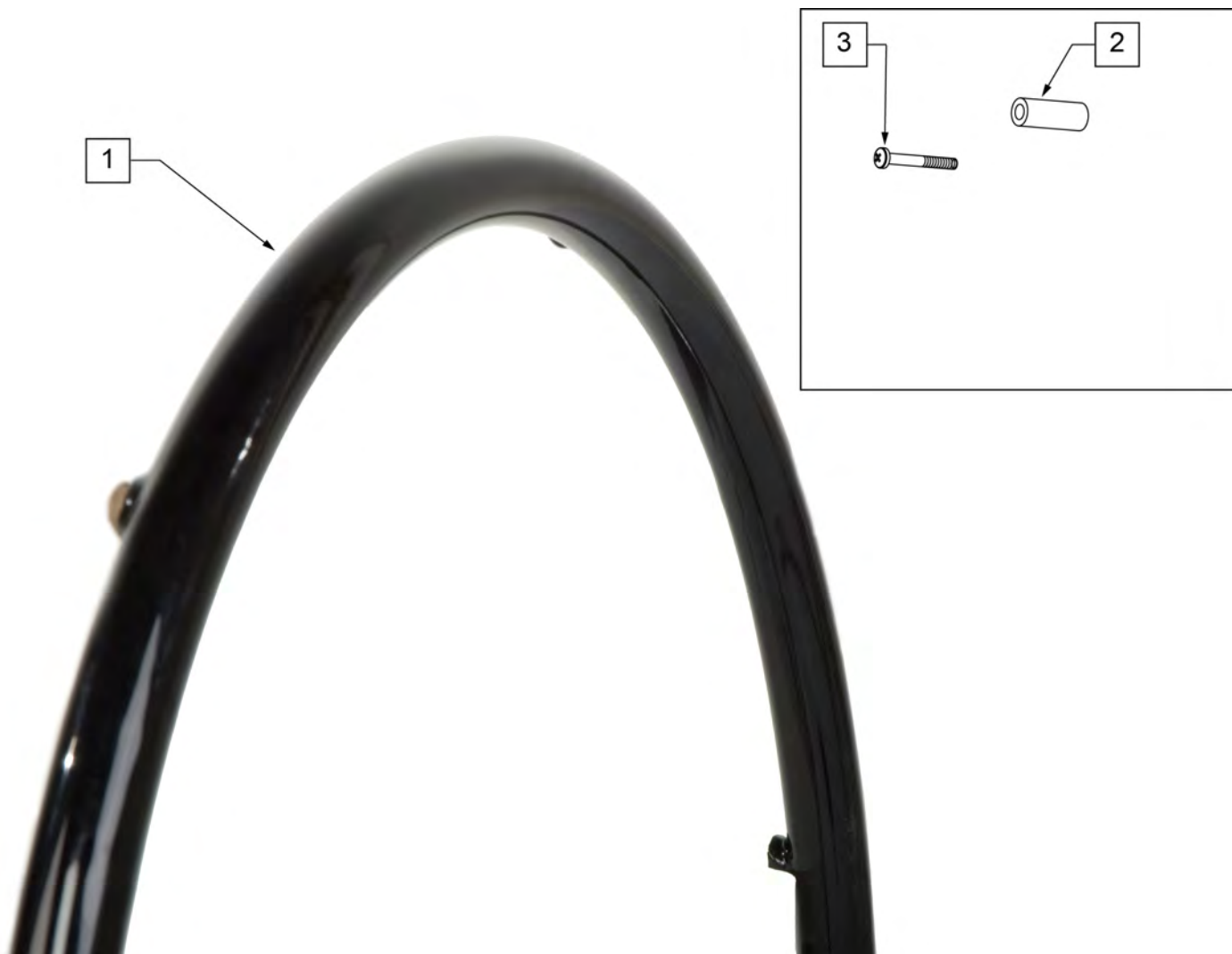


Pos.	Item Number	Description	Remarks
1-3	159504	24 HNDRIM RVNUT Q-GRIP W/HDWR	
1-3	159505	25 HNDRIM RVNUT Q-GRIP W/HDWR	
2,3	159506	Q-GRIP HARDWARE	set of 6

Note: used on Spinergy SPOX and LX wheels.

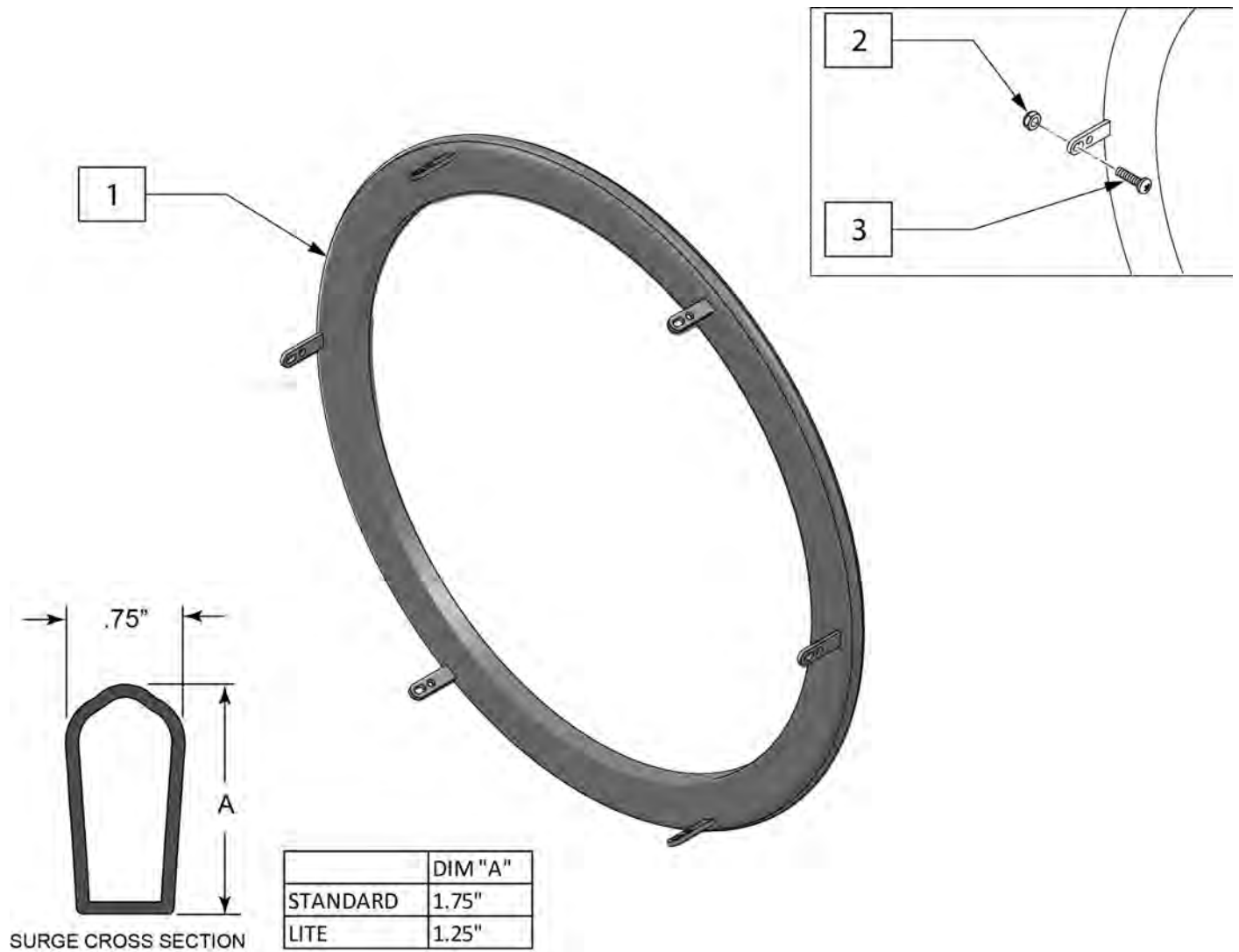

Pos.	Item Number	Description
1-3	159492	22IN TAB SURGE HANDRIM W/HDWR
1-3	159493	24IN TAB SURGE HANDRIM W/HDWR
1-3	159494	25IN TAB SURGE HANDRIM W/HDWR
1-3	159495	26IN TAB SURGE HANDRIM W/HDWR
1-3	159496	22IN TAB SURGE HANDRIM LT W/HDWR
1-3	159497	24IN TAB SURGE HANDRIM LT W/HDWR
1-3	159498	25IN TAB SURGE HANDRIM LT W/HDWR
1-3	159499	26IN TAB SURGE HANDRIM LT W/HDWR
2,3	159500	SURGE HARDWARE TAB MNT

Note: used on Lite Spoke wheels.

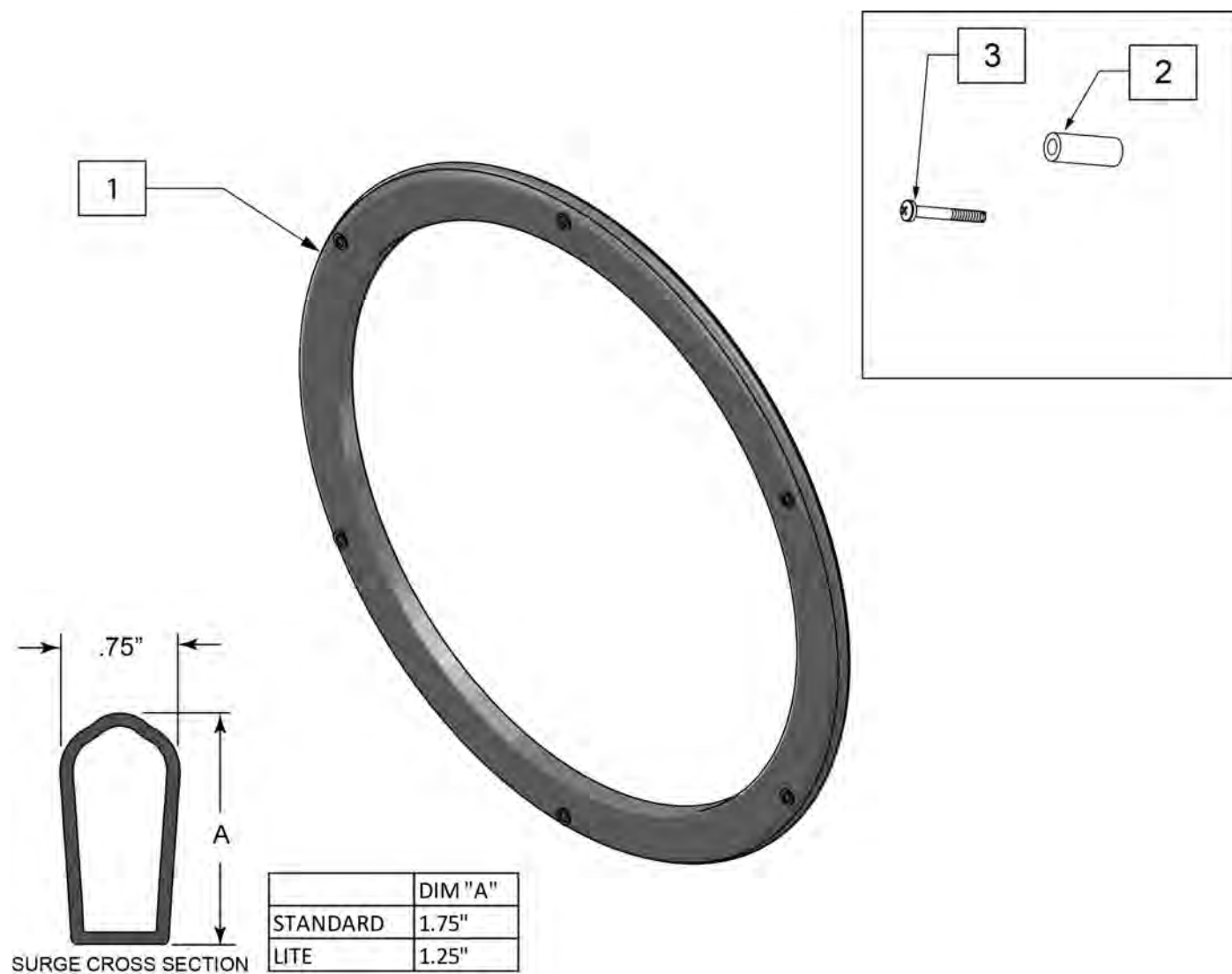

Pos.	Item Number	Description	Remarks
1-3	159483	22IN HANDRIM SURGE W/HDWR	
1-3	159484	24IN HANDRIM SURGE W/HDWR	
1-3	159485	25IN HANDRIM SURGE W/HDWR	
1-3	159486	26IN HANDRIM SURGE W/HDWR	
1-3	159487	22IN HANDRIM LT SURGE W/HDWR	
1-3	159488	24IN HANDRIM LT SURGE W/HDWR	
1-3	159489	25IN HANDRIM LT SURGE W/HDWR	
1-3	159490	26IN HANDRIM LT SURGE W/HDWR	
2,3	159491	SURGE HARDWARE	set of 6


NOTE: fits 24" 5 Spoke Wheel only.





Pos.	Item Number	Description	Remarks
1A	126558	HNDRM NAT FIT LT OVAL 5 RVT	24" Light
1B	126559	HNDRM NATURAL FIT OVAL 5 RVT	24" Standard
2A	102168-004	GUARD THUMB NAT FIT 24 SG BLK	24" (540)- Super Grip Thumb Guard
2B	102168-003	GUARD THUMB NAT FIT 24 PC BLK	24" (540)- Standard Grip Thumb Guard
3	101932	SPCR .194ID X .375 OD X .750	
4	281175	10-32 X 2 1/8 PAN PHIL SS	



Pos.	Item Number	Description	Remarks
>>>>	>>>>>>>>>	LITE SPOKE	>>>>
1,2A,3A	163434S	20IN LT SPK AA PWDR CT HNDRIM	20" LITE SPOKE
1,2A,3A	163435S	22IN LT SPK AA PWDR CT HNDRIM	22" (501) LITE SPOKE
1,2A,3A	163436S	24IN LT SPK AA PWDR CT HNDRIM	24" (540) LITE SPOKE
1,2A,3A	163437S	25IN LT SPK AA PWDR CT HNDRIM	25" (559) LITE SPOKE
1,2A,3A	163438S	26IN LT SPK AA PWDR CT HNDRIM	26" (590) LITE SPOKE
2A,3A	159513	LT SPK AL AN HNDRIM HDWR	LITE SPOKE, SET OF 6
>>>>	>>>>>>>>>	SPOKE	>>>>
1,2B,3B	425931S	20" HR 6SMT W/HDWR	20" SPOKE
1,2B,3B	425941S	22" HR 6SMT W/HDWR	22" (501) SPOKE
1,2B,3B	426000S	24" HR 6SMT W/ HDWR	24" (540) SPOKE
2B,3B	159514	SPK AL AN HNDRIM HDWR	SPOKE, SET OF 6
>>>>	>>>>>>>>>	MAG	>>>>
1,3C	425934S	20" HR 6SMT 6MAG W/HDWR	20" MAG
1,3C	425951S	22" HR 6SMT 6MAG W/HDWR	22" (501) MAG
1,3C	426009S	24" HR 6SMT 6MAG W/HDWR	24" (540) MAG
3C	281175	10-32 X 2 1/8 PAN PHIL SS	MAG
>>>>	>>>>>>>>>	M6 SPOKE	>>>>
1,2B,3B	425941S	22" HR 6SMT W/HDWR	24" (540) SPOKE

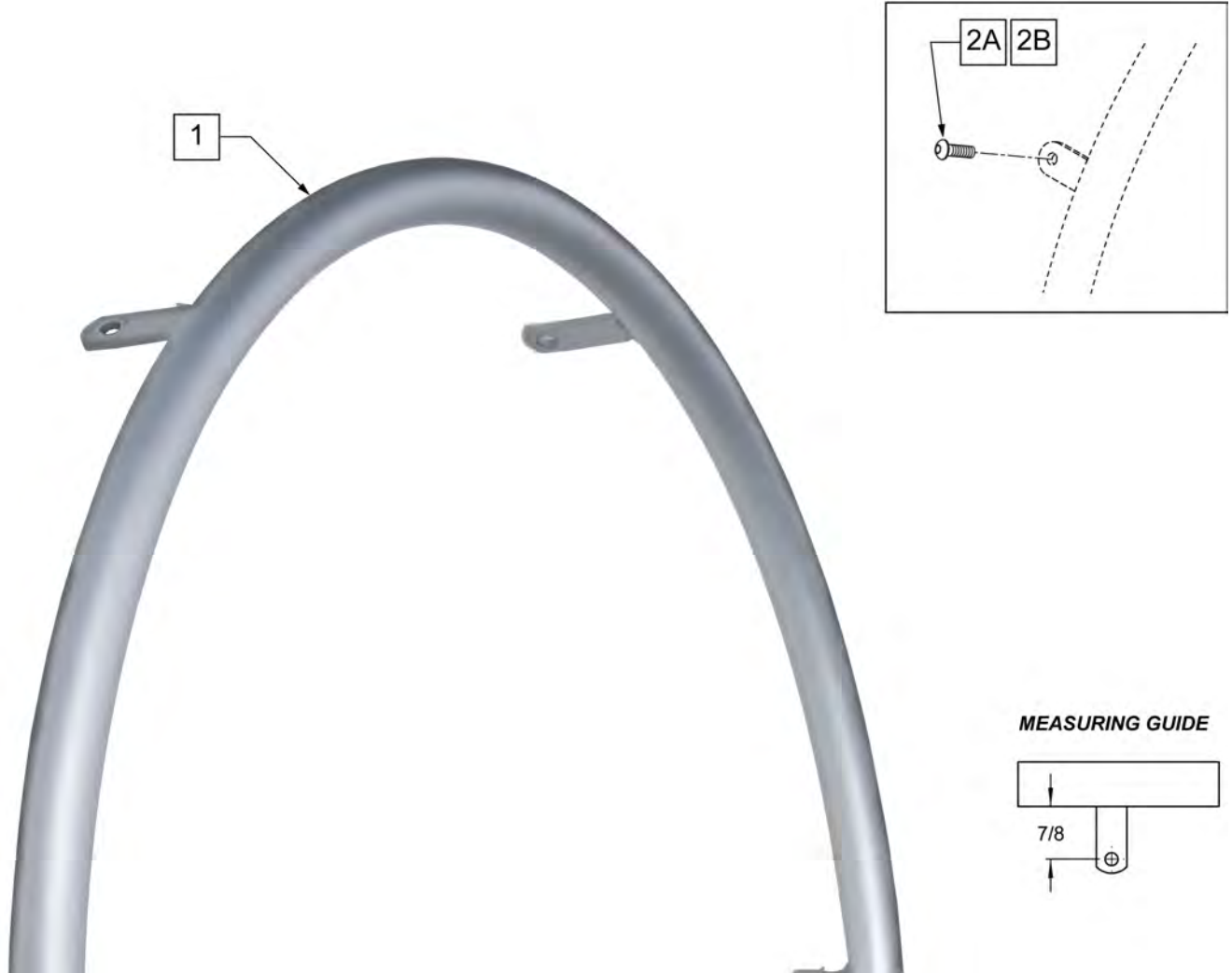

Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>	NEW ULTRA LITE SPOKE	>>>>>
1	163444S	24IN ULT LT SPK PWDRCT HDRM	24" (540)
1	163445S	25IN ULT LT SPK PWDRCT HDRM	26" (559)
2B	012931	BHCS M5 X 10 W/PL BLK CL 10.9	QTY 6 REG

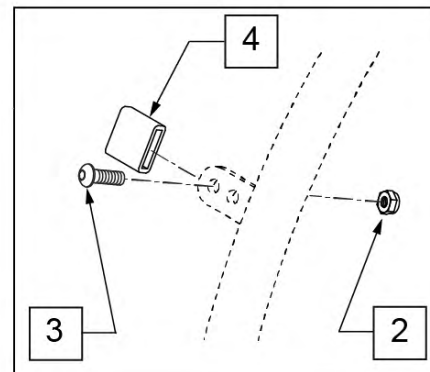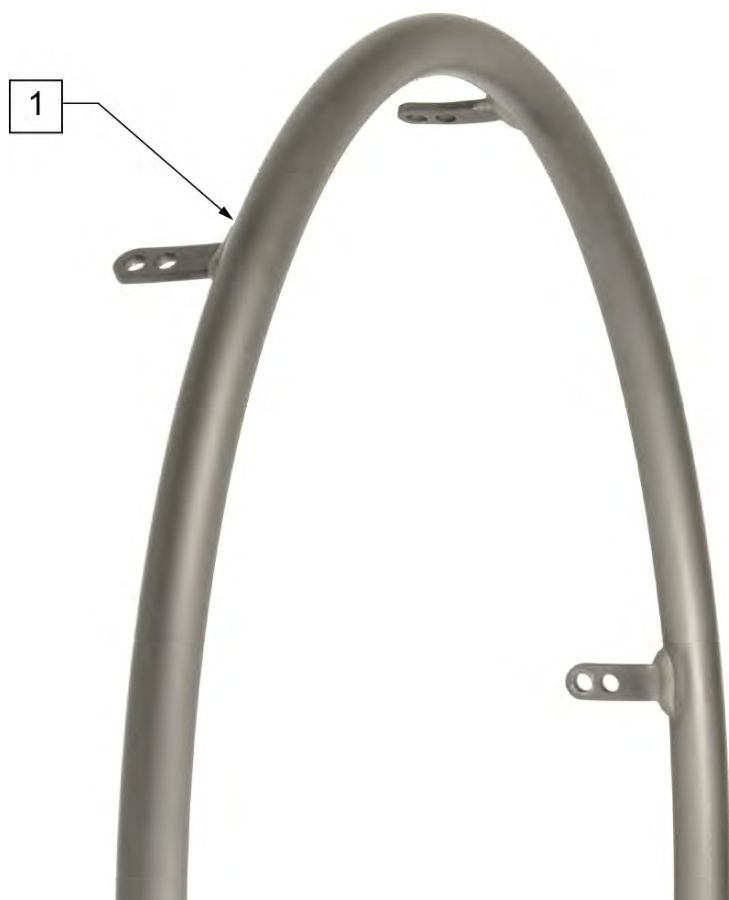


Pos.	Item Number	Description	Remarks
>>>>>	>>>>>>>>>>	CROSS SPOKE & SPOKE	>>>>>
1 -4	425095S	HANDRIM 18" 4T W/HDWR ANO	18" SPOKE
1	529529S	QP 24" HR .065 (6) TAB ANO	24" (540) CROSS SPOKE -SHORT & REG TAB
1 -4	425030S	24" HR 6T BB/HTBB W/HDWR	25" (590) CROSS SPOKE
>>>>>	>>>>>>>>>>	CROSS SPOKE QP (QUICKIE PERFORMANCE)	>>>>>
1 -4	425050S	26" HR 6T BB/TN W/HDWR	26" (590) CROSS SPOKE TENNIS & BBALL
1 -4	425045S	24" HR SHT(6T)ANO W/X/S HDWR	25" (590) CROSS SPOKE TENNIS & BBALL-SHORT TAB
1 -4	425048S	26" HR (6T)ANO W/X/S HDWR	26" (700) CROSS SPOKE TENNIS & BBALL -REG TAB
1 -4	425055S	26" HR SHT(6T)ANO W/X/S HDWR	26" (700) CROSS SPOKE TENNIS & BBALL-SHORT TAB
1 -4	426015S	QP 24" HR .049 W/HDWR SMT	24" (540) TENNIS
1 -4	425030S	24" HR 6T BB/HTBB W/HDWR	25" (559) TENNIS & BASKETBALL
1 -4	426050S	QP 26" HR .065 W/HDWR (6T)	26" (700) TENNIS & BASKETBALL
1 -4	425050S	26" HR 6T BB/TN W/HDWR	26" (590) TENNIS & BASKETBALL
>>>>>	>>>>>>>>>>	SPINERGY	>>>>>
1	529529S	QP 24" HR .065 (6) TAB ANO	24" (540) SPINERGY
1 -4	425038S	QP 24" HR .049 W/HDWR (6T)	25" (559) SPINERGY
1 -4	425050S	26" HR 6T BB/TN W/HDWR	26" (590) SPINERGY
>>>>>	>>>>>>>>>>	MOUNTAIN	>>>>>
1 -4	425020S	22" HR 4T W/HDWR	24" (540) MOUNTAIN
1 -4	425030S	24" HR 6T BB/HTBB W/HDWR	26" (590) MOUNTAIN
2-4	159515	AA TAB MOUNT HANDRIM HARDWARE	SET OF 6



Pos.	Item Number	Description	Remarks
1A	Discontinued	Presta Valves Discontinued	
1A	384736	INNER TUBE 12-1/2" X 2-1/4"	
1A	384738	16" X 1.75" INNER TUBE	
1A	382255	TUBE 18" TIRE	18" X 1-3/4" TUBE
1A	382275	20" TUBE	20" 1-3/8" TUBE
1A	382300	22" TUBE	22" X 1-3/8" TUBE
1A	382336	TUBE PRIMO 24" 25-540 SV	24" X 1" TUBE
1A	382325	24 X 1.3/8IN TUBE REG	
1A	382334	TUBE INLINE 24 X 1.75	
1A	384551	26" X 1" TUBE SCHRADER	
1A	382350	26" TUBE	26" X 1-3/8" TUBE
1A	382361	TUBE 26 X 1.5/1.95/ S/V	
1A	382365	26 X 1.25/1.5 TUBE SHRADER	
1A	382375	24" x 2.125 TUBE	
1B	102972-001	12 1/2 IN AIRLESS INSERT	12-1/2" X 2-1/4"
1B	102973-001	16IN AIRLESS INSERT	16" X 1-3/4"
1B	102973-002	18IN AIRLESS INSERT	18" X 1-3/4"
1B	102973-003	20IN AIRLESS INSERT	20" X 1-3/8"
1B	102972-002	AIRLESS INSERT 20 X 2-1/8 P3	
1B	102973-004	22IN AIRLESS INSERT	22" X 1-3/8"
1B	102973-005	24IN AIRLESS INSERT	24" X 1-3/8"
1B	102972-003	AIRLESS INSERT 24 X 2-1/8	
1B	108259	26" AIRLESS INSERT	26" X 1-3/8"

Note: There is a 1" gap between the rear of the seat sling and the front edge of the backpost, for this reason all Standard Seat Slings and Seat Rails should be ordered 1" less than the chair depth.

Note: If changing from a Ultra Lite seat sling to a standard seat sling a new frame must be ordered, as frame does not have necessary mounting locations.

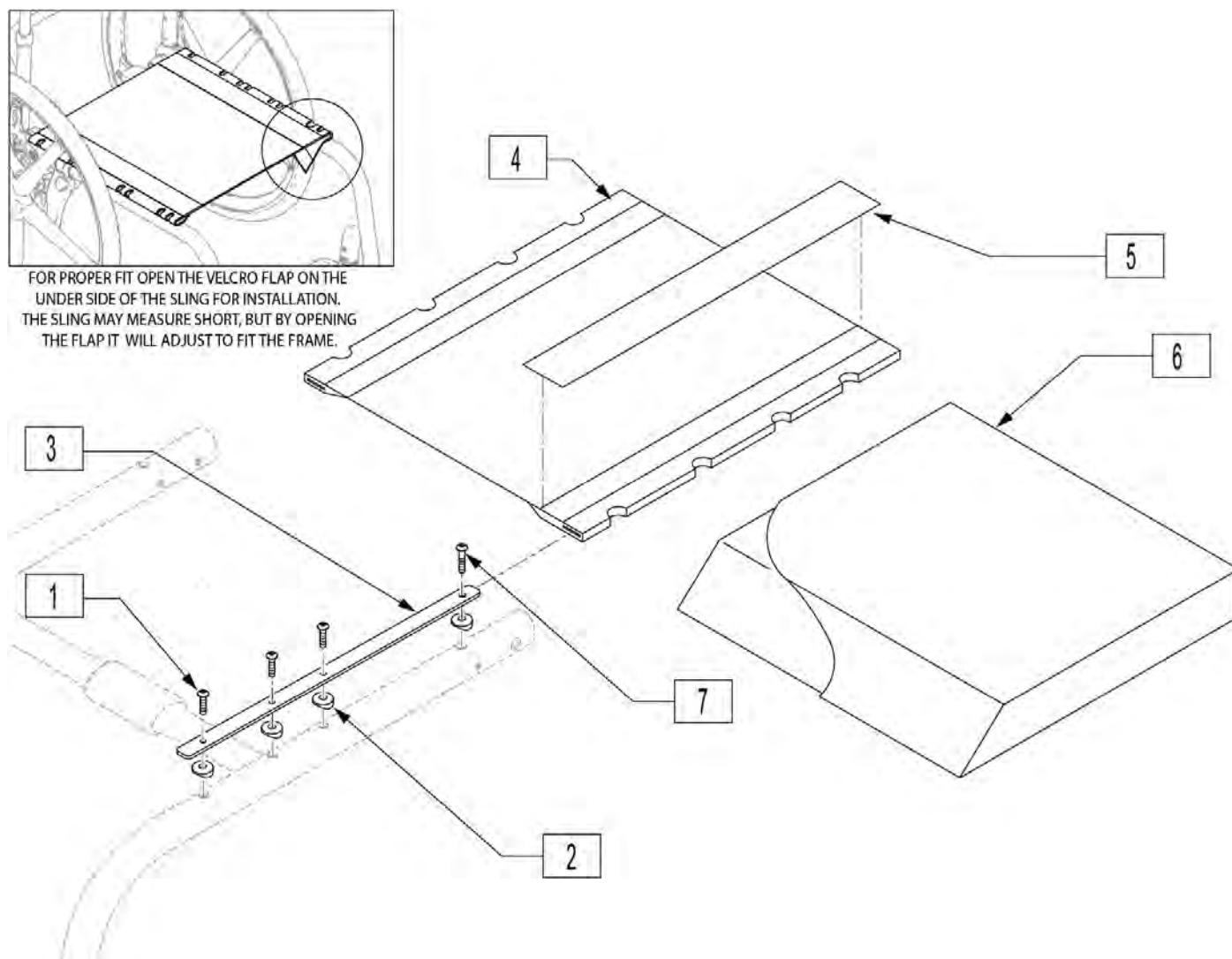


Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
1	102259-002	BHCS, M5 X 20		3	110224	SEAT RAIL, LITE WEIGHT, 18"	For 19" depth
2	482000	SADDLE, SMALL		3	110225	SEAT RAIL, LITE WEIGHT, 19"	For 20" depth
3	110217	SEAT RAIL, LITE WEIGHT, 11"	For 12" depth	4	109466	SEAT SLING, STD, 12 x 11	for 12"x12'
3	110218	SEAT RAIL, LITE WEIGHT, 12"	For 13" depth	4	109467	SEAT SLING, STD, 12 x 12	for 12"x13"
3	110219	SEAT RAIL, LITE WEIGHT, 13"	For 14" depth	4	109468	SEAT SLING, STD, 12 x 13	for 12"x14"
3	110220	SEAT RAIL, LITE WEIGHT, 14"	For 15" depth	4	109469	SEAT SLING, STD, 12 x 14	for 12"x15"
3	110221	SEAT RAIL, LITE WEIGHT, 15"	For 16" depth	4	109470	SEAT SLING, STD, 12 x 15	for 12"x16"
3	110222	SEAT RAIL, LITE WEIGHT, 16"	For 17" depth	4	109471	SEAT SLING, STD, 12 x 16	for 12"x17"
3	110223	SEAT RAIL, LITE WEIGHT, 17"	For 18" depth	4	109472	SEAT SLING, STD, 12 x 17	for 12"x18"

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
4	109473	SEAT SLING, STD, 12 x 18	for 12"x19"	4	109530	SEAT SLING, STD, 19 x 14	for 19"x15"
4	109474	SEAT SLING, STD, 13 x 11	for 13"x12"	4	109531	SEAT SLING, STD, 19 x 15	for 19"x16"
4	109475	SEAT SLING, STD, 13 x 12	for 13"x13"	4	109532	SEAT SLING, STD, 19 x 16	for 19"x17"
4	109476	SEAT SLING, STD, 13 x 13	for 13"x14"	4	109533	SEAT SLING, STD, 19 x 17	for 19"x18"
4	109477	SEAT SLING, STD, 13 x 14	for 13"x15"	4	109534	SEAT SLING, STD, 19 x 18	for 19"x19"
4	109478	SEAT SLING, STD, 13 x 15	for 13"x16"	4	109535	SEAT SLING, STD, 19 x 19	for 19"x20"
4	109479	SEAT SLING, STD, 13 x 16	for 13"x17"	4	109536	SEAT SLING, STD, 20 x 11	for 20"x12"
4	109480	SEAT SLING, STD, 13 x 17	for 13"x18"	4	109537	SEAT SLING, STD, 20 x 12	for 20"x13"
4	109481	SEAT SLING, STD, 13 x 18	for 13"x19"	4	109538	SEAT SLING, STD, 20 x 13	for 20"x14"
4	109482	SEAT SLING, STD, 14 x 11	for 14"x12"	4	109539	SEAT SLING, STD, 20 x 14	for 20"x15"
4	109483	SEAT SLING, STD, 14 x 12	for 14"x13"	4	109540	SEAT SLING, STD, 20 x 15	for 20"x16"
4	109484	SEAT SLING, STD, 14 x 13	for 14"x14"	4	109541	SEAT SLING, STD, 20 x 16	for 20"x17"
4	109485	SEAT SLING, STD, 14 x 14	for 14"x15"	4	109542	SEAT SLING, STD, 20 x 17	for 20"x18"
4	109486	SEAT SLING, STD, 14 x 15	for 14"x16"	4	109543	SEAT SLING, STD, 20 x 18	for 20"x19"
4	109487	SEAT SLING, STD, 14 x 16	for 14"x17"	4	109544	SEAT SLING, STD, 20 x 19	for 20"x20"
4	109488	SEAT SLING, STD, 14 x 17	for 14"x18"	5	901169	SEAT SLING VELCRO STRIP ® 2"	
4	109489	SEAT SLING, STD, 14 x 18	for 14"x19"	6	700122	SEAT CUSHION 12" x 12" x 2"	
4	109490	SEAT SLING, STD, 14 x 19	for 14"x20"	6	700132	SEAT CUSHION 12" x 13" x 2"	
4	109491	SEAT SLING, STD, 15 x 11	for 15"x12"	6	700142	SEAT CUSHION 12" x 14" x 2"	
4	109492	SEAT SLING, STD, 15 x 12	for 15"x13"	6	700152	SEAT CUSHION 12" x 15" x 2"	
4	109493	SEAT SLING, STD, 15 x 13	for 15"x14"	6	700162	SEAT CUSHION 12" x 16" x 2"	
4	109494	SEAT SLING, STD, 15 x 14	for 15"x15"	6	700172	SEAT CUSHION 12" x 17" x 2"	
4	109495	SEAT SLING, STD, 15 x 15	for 15"x16"	6	700182	SEAT CUSHION 12" x 18" x 2"	
4	109496	SEAT SLING, STD, 15 x 16	for 15"x17"	6	700192	SEAT CUSHION 12" x 19" x 2"	
4	109497	SEAT SLING, STD, 15 x 17	for 15"x18"	6	700222	SEAT CUSHION 13" x 12" x 2"	
4	109498	SEAT SLING, STD, 15 x 18	for 15"x19"	6	700232	SEAT CUSHION 13" x 13" x 2"	
4	109499	SEAT SLING, STD, 15 x 19	for 15"x20"	6	700242	SEAT CUSHION 13" x 14" x 2"	
4	109500	SEAT SLING, STD, 16 x 11	for 16"x12"	6	700252	SEAT CUSHION 13" x 15" x 2"	
4	109501	SEAT SLING, STD, 16 x 12	for 16"x13"	6	700262	SEAT CUSHION 13" x 16" x 2"	
4	109502	SEAT SLING, STD, 16 x 13	for 16"x14"	6	700272	SEAT CUSHION 13" x 17" x 2"	
4	109503	SEAT SLING, STD, 16 x 14	for 16"x15"	6	700277	SEAT CUSHION 13" x 18" x 2"	
4	109504	SEAT SLING, STD, 16 x 15	for 16"x16"	6	700267	SEAT CUSHION 13" x 19" x 2"	
4	109505	SEAT SLING, STD, 16 x 16	for 16"x17"	6	700302	SEAT CUSHION 14" x 12" x 2"	
4	109506	SEAT SLING, STD, 16 x 17	for 16"x18"	6	700312	SEAT CUSHION 14" x 13" x 2"	
4	109507	SEAT SLING, STD, 16 x 18	for 16"x19"	6	700322	SEAT CUSHION 14" x 14" x 2"	
4	109508	SEAT SLING, STD, 16 x 19	for 16"x20"	6	700332	SEAT CUSHION 14" x 15" x 2"	
4	109509	SEAT SLING, STD, 17 x 11	for 17"x12"	6	700342	SEAT CUSHION 14" x 16" x 2"	
4	109510	SEAT SLING, STD, 17 x 12	for 17"x13"	6	700352	SEAT CUSHION 14" x 17" x 2"	
4	109511	SEAT SLING, STD, 17 x 13	for 17"x14"	6	700362	SEAT CUSHION 14" x 18" x 2"	
4	109512	SEAT SLING, STD, 17 x 14	for 17"x15"	6	700372	SEAT CUSHION 14" x 19" x 2"	
4	109513	SEAT SLING, STD, 17 x 15	for 17"x16"	6	700382	SEAT CUSHION 14" x 20" x 2"	
4	109514	SEAT SLING, STD, 17 x 16	for 17"x17"	6	700402	SEAT CUSHION 15" x 12" x 2"	
4	109515	SEAT SLING, STD, 17 x 17	for 17"x18"	6	700412	SEAT CUSHION 15" x 13" x 2"	
4	109516	SEAT SLING, STD, 17 x 18	for 17"x19"	6	700422	SEAT CUSHION 15" x 14" x 2"	
4	109517	SEAT SLING, STD, 17 x 19	for 17"x20"	6	700432	SEAT CUSHION 15" x 15" x 2"	
4	109518	SEAT SLING, STD, 18 x 11	for 18"x12"	6	700442	SEAT CUSHION 15" x 16" x 2"	
4	109519	SEAT SLING, STD, 18 x 12	for 18"x13"	6	700452	SEAT CUSHION 15" x 17" x 2"	
4	109520	SEAT SLING, STD, 18 x 13	for 18"x14"	6	700462	SEAT CUSHION 15" x 18" x 2"	
4	109521	SEAT SLING, STD, 18 x 14	for 18"x15"	6	700472	SEAT CUSHION 15" x 19" x 2"	
4	109522	SEAT SLING, STD, 18 x 15	for 18"x16"	6	700482	SEAT CUSHION 15" x 20" x 2"	
4	109523	SEAT SLING, STD, 18 x 16	for 18"x17"	6	700502	SEAT CUSHION 16" x 12" x 2"	
4	109524	SEAT SLING, STD, 18 x 17	for 18"x18"	6	700512	SEAT CUSHION 16" x 13" x 2"	
4	109525	SEAT SLING, STD, 18 x 18	for 18"x19"	6	700522	SEAT CUSHION 16" x 14" x 2"	
4	109526	SEAT SLING, STD, 18 x 19	for 18"x20"	6	700532	SEAT CUSHION 16" x 15" x 2"	
4	109527	SEAT SLING, STD, 19 x 11	for 19"x12"	6	700542	SEAT CUSHION 16" x 16" x 2"	
4	109528	SEAT SLING, STD, 19 x 12	for 19"x13"	6	700552	SEAT CUSHION 16" x 17" x 2"	
4	109529	SEAT SLING, STD, 19 x 13	for 19"x14"	6	700562	SEAT CUSHION 16" x 18" x 2"	

Pos.	Item Number	Description	Remarks	Pos.	Item Number	Description	Remarks
6	700572	SEAT CUSHION 16" x 19" x 2"		6	700832	SEAT CUSHION 19" x 17" x 2"	
6	700582	SEAT CUSHION 16" x 20" x 2"		6	700842	SEAT CUSHION 19" x 18" x 2"	
6	700606	SEAT CUSHION 17" x 12" x 2"		6	700852	SEAT CUSHION 19" x 19" x 2"	
6	700597	SEAT CUSHION 17" x 13" x 2"		6	700862	SEAT CUSHION 19" x 20" x 2"	
6	700602	SEAT CUSHION 17" x 14" x 2"		6	700882	SEAT CUSHION 20" x 12" x 2"	
6	700612	SEAT CUSHION 17" x 15" x 2"		6	700892	SEAT CUSHION 20" x 13" x 2"	
6	700622	SEAT CUSHION 17" x 16" x 2"		6	700902	SEAT CUSHION 20" x 14" x 2"	
6	700632	SEAT CUSHION 17" x 17" x 2"		6	700912	SEAT CUSHION 20" x 15" x 2"	
6	700642	SEAT CUSHION 17" x 18" x 2"		6	700922	SEAT CUSHION 20" x 16" x 2"	
6	700652	SEAT CUSHION 17" x 19" x 2"		6	700932	SEAT CUSHION 20" x 17" x 2"	
6	700662	SEAT CUSHION 17" x 20" x 2"		6	700942	SEAT CUSHION 20" x 18" x 2"	
6	700682	SEAT CUSHION 18" x 12" x 2"		6	700952	SEAT CUSHION 20" x 19" x 2"	
6	700692	SEAT CUSHION 18" x 13" x 2"		6	700962	SEAT CUSHION 20" x 20" x 2"	
6	700702	SEAT CUSHION 18" x 14" x 2"		7	122916	BHCS M5 X 0.8 X 50 PART NP	
6	700712	SEAT CUSHION 18" x 15" x 2"		1-3,7	114442	SEAT RAIL LITE 11" KIT (PR)	Pair
6	700722	SEAT CUSHION 18" x 16" x 2"		1-3,7	114443	SEAT RAIL LITE 12" KIT (PR)	Pair
6	700732	SEAT CUSHION 18" x 17" x 2"		1-3,7	114444	SEAT RAIL LITE 13" KIT (PR)	Pair
6	700742	SEAT CUSHION 18" x 18" x 2"		1-3,7	114445	SEAT RAIL LITE 14" KIT (PR)	Pair
6	700752	SEAT CUSHION 18" x 19" x 2"		1-3,7	114446	SEAT RAIL LITE 15" KIT (PR)	Pair
6	700762	SEAT CUSHION 18" x 20" x 2"		1-3,7	114447	SEAT RAIL LITE 16" KIT (PR)	Pair
6	700782	SEAT CUSHION 19" x 12" x 2"		1-3,7	114448	SEAT RAIL LITE 17" KIT (PR)	Pair
6	700792	SEAT CUSHION 19" x 13" x 2"		1-3,7	114449	SEAT RAIL LITE 18" KIT (PR)	Pair
6	700802	SEAT CUSHION 19" x 14" x 2"		1-3,7	114450	SEAT RAIL LITE 19" KIT (PR)	Pair
6	700812	SEAT CUSHION 19" x 15" x 2"		1,2,7	114522	SEAT SLING HDWR KIT (10)	
6	700822	SEAT CUSHION 19" x 16" x 2"					

Note: There is a 1" gap between the rear of the seat sling and the front edge of the backpost, for this reason all Standard Seat Slings and Seat Rails should be ordered 1" less than the chair depth.

Note: If changing from a Ultra Lite seat sling to a standard seat sling a new frame must be ordered, as frame does not have necessary mounting locations.

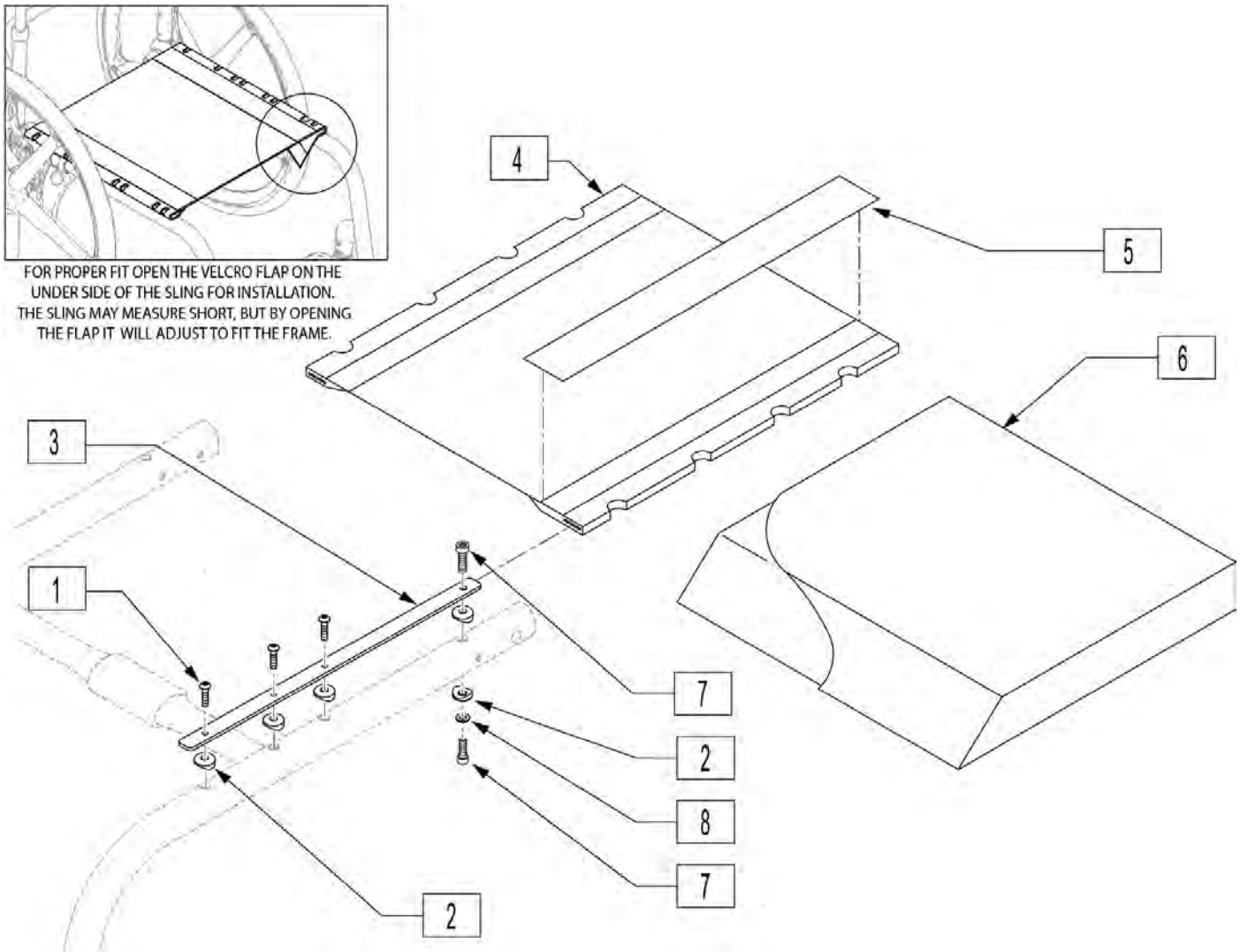

Pos.	Item Number	Description	Remarks
1	102259-002	BHCS, M5 X 20	
2	482000	SADDLE, SMALL	
3	128206	SEAT RAIL LITE WEIGHT 11"	For 12" depth
3	128207	SEAT RAIL LITE WEIGHT 12"	For 13" depth
3	128208	SEAT RAIL LITE WEIGHT 13"	For 14" depth
3	128209	SEAT RAIL LITE WEIGHT 14"	For 15" depth
3	128210	SEAT RAIL LITE WEIGHT 15"	For 16" depth
3	128211	SEAT RAIL LITE WEIGHT 16"	For 17" depth
3	128212	SEAT RAIL LITE WEIGHT 17"	For 18" depth

Pos.	Item Number	Description	Remarks
3	128213	SEAT RAIL LITE WEIGHT 18"	For 19" depth
3	128214	SEAT RAIL LITE WEIGHT 19"	For 20" depth
4	109466	SEAT SLING, STD, 12 x 11	for 12"x12'
4	109467	SEAT SLING, STD, 12 x 12	for 12"x13"
4	109468	SEAT SLING, STD, 12 x 13	for 12"x14"
4	109469	SEAT SLING, STD, 12 x 14	for 12"x15"
4	109470	SEAT SLING, STD, 12 x 15	for 12"x16"
4	109471	SEAT SLING, STD, 12 x 16	for 12"x17"
4	109472	SEAT SLING, STD, 12 x 17	for 12"x18"
4	109473	SEAT SLING, STD, 12 x 18	for 12"x19"
4	109474	SEAT SLING, STD, 13 x 11	for 13"x12"
4	109475	SEAT SLING, STD, 13 x 12	for 13"x13"
4	109476	SEAT SLING, STD, 13 x 13	for 13"x14"
4	109477	SEAT SLING, STD, 13 x 14	for 13"x15"
4	109478	SEAT SLING, STD, 13 x 15	for 13"x16"
4	109479	SEAT SLING, STD, 13 x 16	for 13"x17"
4	109480	SEAT SLING, STD, 13 x 17	for 13"x18"
4	109481	SEAT SLING, STD, 13 x 18	for 13"x19"
4	109482	SEAT SLING, STD, 14 x 11	for 14"x12"
4	109483	SEAT SLING, STD, 14 x 12	for 14"x13"
4	109484	SEAT SLING, STD, 14 x 13	for 14"x14"
4	109485	SEAT SLING, STD, 14 x 14	for 14"x15"
4	109486	SEAT SLING, STD, 14 x 15	for 14"x16"
4	109487	SEAT SLING, STD, 14 x 16	for 14"x17"
4	109488	SEAT SLING, STD, 14 x 17	for 14"x18"
4	109489	SEAT SLING, STD, 14 x 18	for 14"x19"
4	109490	SEAT SLING, STD, 14 x 19	for 14"x20"
4	109491	SEAT SLING, STD, 15 x 11	for 15"x12"
4	109492	SEAT SLING, STD, 15 x 12	for 15"x13"
4	109493	SEAT SLING, STD, 15 x 13	for 15"x14"
4	109494	SEAT SLING, STD, 15 x 14	for 15"x15"
4	109495	SEAT SLING, STD, 15 x 15	for 15"x16"
4	109496	SEAT SLING, STD, 15 x 16	for 15"x17"
4	109497	SEAT SLING, STD, 15 x 17	for 15"x18"
4	109498	SEAT SLING, STD, 15 x 18	for 15"x19"
4	109499	SEAT SLING, STD, 15 x 19	for 15"x20"
4	109500	SEAT SLING, STD, 16 x 11	for 16"x12"
4	109501	SEAT SLING, STD, 16 x 12	for 16"x13"
4	109502	SEAT SLING, STD, 16 x 13	for 16"x14"
4	109503	SEAT SLING, STD, 16 x 14	for 16"x15"
4	109504	SEAT SLING, STD, 16 x 15	for 16"x16"
4	109505	SEAT SLING, STD, 16 x 16	for 16"x17"
4	109506	SEAT SLING, STD, 16 x 17	for 16"x18"
4	109507	SEAT SLING, STD, 16 x 18	for 16"x19"
4	109508	SEAT SLING, STD, 16 x 19	for 16"x20"
4	109509	SEAT SLING, STD, 17 x 11	for 17"x12"
4	109510	SEAT SLING, STD, 17 x 12	for 17"x13"
4	109511	SEAT SLING, STD, 17 x 13	for 17"x14"
4	109512	SEAT SLING, STD, 17 x 14	for 17"x15"
4	109513	SEAT SLING, STD, 17 x 15	for 17"x16"
4	109514	SEAT SLING, STD, 17 x 16	for 17"x17"
4	109515	SEAT SLING, STD, 17 x 17	for 17"x18"
4	109516	SEAT SLING, STD, 17 x 18	for 17"x19"
4	109517	SEAT SLING, STD, 17 x 19	for 17"x20"
4	109518	SEAT SLING, STD, 18 x 11	for 18"x12"
4	109519	SEAT SLING, STD, 18 x 12	for 18"x13"
4	109520	SEAT SLING, STD, 18 x 13	for 18"x14"

Pos.	Item Number	Description	Remarks
4	109521	SEAT SLING, STD, 18 x 14	for 18"x15"
4	109522	SEAT SLING, STD, 18 x 15	for 18"x16"
4	109523	SEAT SLING, STD, 18 x 16	for 18"x17"
4	109524	SEAT SLING, STD, 18 x 17	for 18"x18"
4	109525	SEAT SLING, STD, 18 x 18	for 18"x19"
4	109526	SEAT SLING, STD, 18 x 19	for 18"x20"
4	109527	SEAT SLING, STD, 19 x 11	for 19"x12"
4	109528	SEAT SLING, STD, 19 x 12	for 19"x13"
4	109529	SEAT SLING, STD, 19 x 13	for 19"x14"
4	109530	SEAT SLING, STD, 19 x 14	for 19"x15"
4	109531	SEAT SLING, STD, 19 x 15	for 19"x16"
4	109532	SEAT SLING, STD, 19 x 16	for 19"x17"
4	109533	SEAT SLING, STD, 19 x 17	for 19"x18"
4	109534	SEAT SLING, STD, 19 x 18	for 19"x19"
4	109535	SEAT SLING, STD, 19 x 19	for 19"x20"
4	109536	SEAT SLING, STD, 20 x 11	for 20"x12"
4	109537	SEAT SLING, STD, 20 x 12	for 20"x13"
4	109538	SEAT SLING, STD, 20 x 13	for 20"x14"
4	109539	SEAT SLING, STD, 20 x 14	for 20"x15"
4	109540	SEAT SLING, STD, 20 x 15	for 20"x16"
4	109541	SEAT SLING, STD, 20 x 16	for 20"x17"
4	109542	SEAT SLING, STD, 20 x 17	for 20"x18"
4	109543	SEAT SLING, STD, 20 x 18	for 20"x19"
4	109544	SEAT SLING, STD, 20 x 19	for 20"x20"
5	901169	SEAT SLING VELCRO STRIP ® 2"	
6	700122	SEAT CUSHION 12" x 12" x 2"	
6	700132	SEAT CUSHION 12" x 13" x 2"	
6	700142	SEAT CUSHION 12" x 14" x 2"	
6	700152	SEAT CUSHION 12" x 15" x 2"	
6	700162	SEAT CUSHION 12" x 16" x 2"	
6	700172	SEAT CUSHION 12" x 17" x 2"	
6	700182	SEAT CUSHION 12" x 18" x 2"	
6	700192	SEAT CUSHION 12" x 19" x 2"	
6	700222	SEAT CUSHION 13" x 12" x 2"	
6	700232	SEAT CUSHION 13" x 13" x 2"	
6	700242	SEAT CUSHION 13" x 14" x 2"	
6	700252	SEAT CUSHION 13" x 15" x 2"	
6	700262	SEAT CUSHION 13" x 16" x 2"	
6	700272	SEAT CUSHION 13" x 17" x 2"	
6	700277	SEAT CUSHION 13" x 18" x 2"	
6	700267	SEAT CUSHION 13" x 19" x 2"	
6	700302	SEAT CUSHION 14" x 12" x 2"	
6	700312	SEAT CUSHION 14" x 13" x 2"	
6	700322	SEAT CUSHION 14" x 14" x 2"	
6	700332	SEAT CUSHION 14" x 15" x 2"	
6	700342	SEAT CUSHION 14" x 16" x 2"	
6	700352	SEAT CUSHION 14" x 17" x 2"	
6	700362	SEAT CUSHION 14" x 18" x 2"	
6	700372	SEAT CUSHION 14" x 19" x 2"	
6	700382	SEAT CUSHION 14" x 20" x 2"	
6	700402	SEAT CUSHION 15" x 12" x 2"	
6	700412	SEAT CUSHION 15" x 13" x 2"	
6	700422	SEAT CUSHION 15" x 14" x 2"	
6	700432	SEAT CUSHION 15" x 15" x 2"	
6	700442	SEAT CUSHION 15" x 16" x 2"	
6	700452	SEAT CUSHION 15" x 17" x 2"	
6	700462	SEAT CUSHION 15" x 18" x 2"	

Pos.	Item Number	Description	Remarks
6	700472	SEAT CUSHION 15" x 19" x 2"	
6	700482	SEAT CUSHION 15" x 20" x 2"	
6	700502	SEAT CUSHION 16" x 12" x 2"	
6	700512	SEAT CUSHION 16" x 13" x 2"	
6	700522	SEAT CUSHION 16" x 14" x 2"	
6	700532	SEAT CUSHION 16" x 15" x 2"	
6	700542	SEAT CUSHION 16" x 16" x 2"	
6	700552	SEAT CUSHION 16" x 17" x 2"	
6	700562	SEAT CUSHION 16" x 18" x 2"	
6	700572	SEAT CUSHION 16" x 19" x 2"	
6	700582	SEAT CUSHION 16" x 20" x 2"	
6	700606	SEAT CUSHION 17" x 12" x 2"	
6	700597	SEAT CUSHION 17" x 13" x 2"	
6	700602	SEAT CUSHION 17" x 14" x 2"	
6	700612	SEAT CUSHION 17" x 15" x 2"	
6	700622	SEAT CUSHION 17" x 16" x 2"	
6	700632	SEAT CUSHION 17" x 17" x 2"	
6	700642	SEAT CUSHION 17" x 18" x 2"	
6	700652	SEAT CUSHION 17" x 19" x 2"	
6	700662	SEAT CUSHION 17" x 20" x 2"	
6	700682	SEAT CUSHION 18" x 12" x 2"	
6	700692	SEAT CUSHION 18" x 13" x 2"	
6	700702	SEAT CUSHION 18" x 14" x 2"	
6	700712	SEAT CUSHION 18" x 15" x 2"	
6	700722	SEAT CUSHION 18" x 16" x 2"	
6	700732	SEAT CUSHION 18" x 17" x 2"	
6	700742	SEAT CUSHION 18" x 18" x 2"	
6	700752	SEAT CUSHION 18" x 19" x 2"	
6	700762	SEAT CUSHION 18" x 20" x 2"	
6	700782	SEAT CUSHION 19" x 12" x 2"	
6	700792	SEAT CUSHION 19" x 13" x 2"	
6	700802	SEAT CUSHION 19" x 14" x 2"	
6	700812	SEAT CUSHION 19" x 15" x 2"	
6	700822	SEAT CUSHION 19" x 16" x 2"	
6	700832	SEAT CUSHION 19" x 17" x 2"	
6	700842	SEAT CUSHION 19" x 18" x 2"	
6	700852	SEAT CUSHION 19" x 19" x 2"	
6	700862	SEAT CUSHION 19" x 20" x 2"	
6	700882	SEAT CUSHION 20" x 12" x 2"	
6	700892	SEAT CUSHION 20" x 13" x 2"	
6	700902	SEAT CUSHION 20" x 14" x 2"	
6	700912	SEAT CUSHION 20" x 15" x 2"	
6	700922	SEAT CUSHION 20" x 16" x 2"	
6	700932	SEAT CUSHION 20" x 17" x 2"	
6	700942	SEAT CUSHION 20" x 18" x 2"	
6	700952	SEAT CUSHION 20" x 19" x 2"	
6	700962	SEAT CUSHION 20" x 20" x 2"	
7	102263-005	SHCS, M6X1X20, FULL, PL	
8	250104	WASHER FLAT M6	
1-3,7,8	128327	SEAT RAIL LITE 12" KIT	Pair; use for 12" Seat Depth
1-3,7,8	128328	SEAT RAIL LITE 13" KIT	Pair; use for 13" Seat Depth
1-3,7,8	128329	SEAT RAIL LITE 14" KIT	Pair; use for 14" Seat Depth
1-3,7,8	128330	SEAT RAIL LITE 15" KIT	Pair; use for 15" Seat Depth
1-3,7,8	128331	SEAT RAIL LITE 16" KIT	Pair; use for 16" Seat Depth
1-3,7,8	128332	SEAT RAIL LITE 17" KIT	Pair; use for 17" Seat Depth
1-3,7,8	128333	SEAT RAIL LITE 18" KIT	Pair; use for 18" Seat Depth
1-3,7,8	128334	SEAT RAIL LITE 19" KIT	Pair; use for 19" Seat Depth


Pos.	Item Number	Description	Remarks
1-3,7,8	128335	SEAT RAIL LITE 20" KIT	Pair; use for 20" Seat Depth

NOTE: There is a 1" gap between the rear of the Ultra Lite Seat Sling and the front edge of the backpost, for this reason all Ultra Lite Seat Sling will measure 1" less in depth than the description indicates.

NOTE: Strap Kits contain Qty (5) 2" straps and Qty (3) 1" straps.

NOTE: Ultra Seat Sling only offered in 14 x 14 through 18 x 18 sizes.

NOTE: If changing from a Ultra Lite seat sling to a standard seat sling a new frame must be ordered, as frame does not have necessary mounting locations.

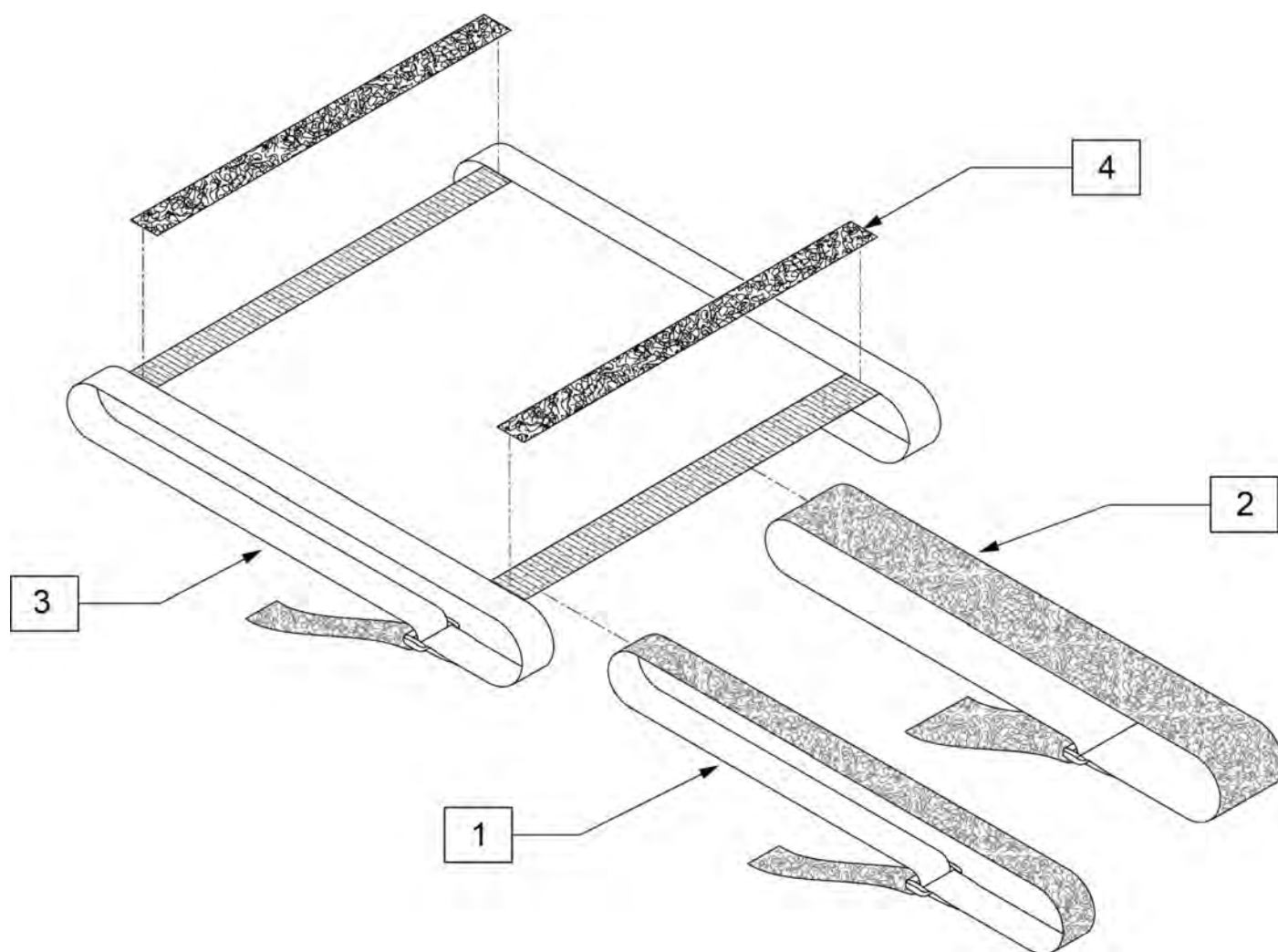


Pos.	Item Number	Description	Remarks
1-2	114525	SLING TEN ADJ STRAP KIT 14W KIT	See note 2
1-2	114526	SLING TEN ADJ STRAP KIT 15W KIT	See note 2

Pos.	Item Number	Description	Remarks
1-2	114527	SLING TEN ADJ STRAP KIT 16W KIT	See note 2
1-2	114528	SLING TEN ADJ STRAP KIT 17W KIT	See note 2
1-2	114529	SLING TEN ADJ STRAP KIT 18W KIT	See note 2
1-3	114552	SLING TEN ADJ LITE 14WX14D KIT	
1-3	114553	SLING TEN ADJ LITE 14WX15D KIT	
1-3	114554	SLING TEN ADJ LITE 14WX16D KIT	
1-3	114555	SLING TEN ADJ LITE 14WX17D KIT	
1-3	114556	SLING TEN ADJ LITE 14WX18D KIT	
1-3	114561	SLING TEN ADJ LITE 15WX14D KIT	
1-3	114562	SLING TEN ADJ LITE 15WX15D KIT	
1-3	114563	SLING TEN ADJ LITE 15WX16D KIT	
1-3	114564	SLING TEN ADJ LITE 15WX17D KIT	
1-3	114565	SLING TEN ADJ LITE 15WX18D KIT	
1-3	114570	SLING TEN ADJ LITE 16WX14D KIT	
1-3	114571	SLING TEN ADJ LITE 16WX15D KIT	
1-3	114572	SLING TEN ADJ LITE 16WX16D KIT	
1-3	114573	SLING TEN ADJ LITE 16WX17D KIT	
1-3	114574	SLING TEN ADJ LITE 16WX18D KIT	
1-3	114579	SLING TEN ADJ LITE 17WX14D KIT	
1-3	114580	SLING TEN ADJ LITE 17WX15D KIT	
1-3	114581	SLING TEN ADJ LITE 17WX16D KIT	
1-3	114582	SLING TEN ADJ LITE 17WX17D KIT	
1-3	114583	SLING TEN ADJ LITE 17WX18D KIT	
1-3	114588	SLING TEN ADJ LITE 18WX14D KIT	
1-3	114589	SLING TEN ADJ LITE 18WX15D KIT	
1-3	114590	SLING TEN ADJ LITE 18WX16D KIT	
1-3	114591	SLING TEN ADJ LITE 18WX17D KIT	
1-3	114592	SLING TEN ADJ LITE 18WX18D KIT	
4	901169	SEAT SLING VELCRO STRIP ® 2"	

Note: the Q7 has 8 degrees rearward to 27 degrees forward adjustment in 5 degree increments.

Note: when moving backrest rearward with a Depth Adjustable Back item 18 will be used as a bushing for item 14.

Note: Lanyard not included in kits; order separately.

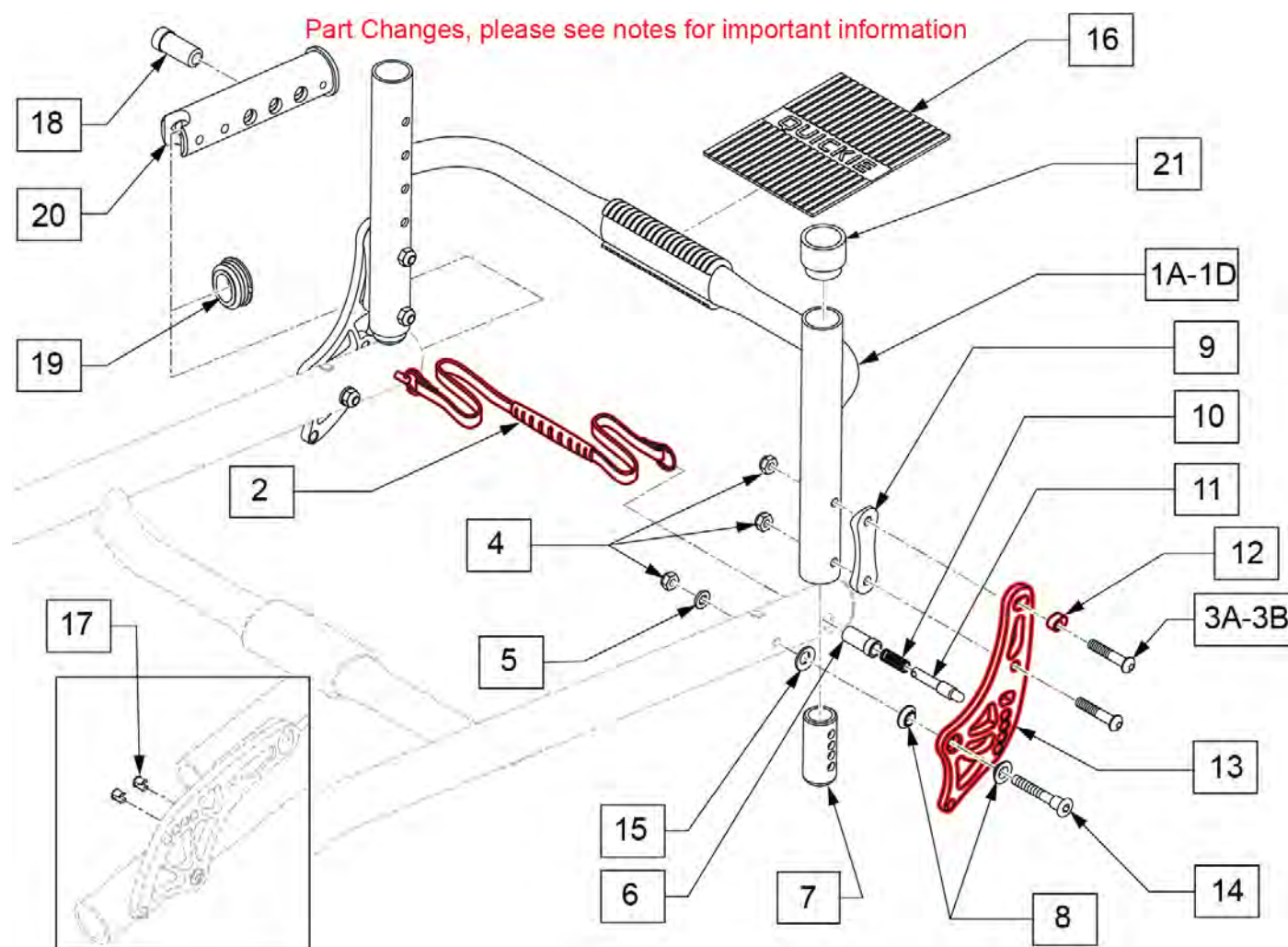

Pos.	Item Number	Description	Remarks
2	109765	LANYARD THERMO-PRESS 12W	
2	109766	LANYARD THERMO-PRESS 13W	
2	109767	LANYARD THERMO-PRESS 14W	

Pos.	Item Number	Description	Remarks
2	109768	LANYARD THERMO-PRESS 15W	
2	109769	LANYARD THERMO-PRESS 16W	
2	109770	LANYARD THERMO-PRESS 17W	
2	109771	LANYARD THERMO-PRESS 18W	
2	109772	LANYARD THERMO-PRESS 19W	
2	109773	LANYARD THERMO-PRESS 20W	
3A	102262-006	BHCS,M6X1X40,PART,NL	Used w/o Ht Adj Push Handles
3B	102262-007	BHCS,M6X1X45,PART,N	Used w/ Ht Adj Push Handles
4	104381	NUT NYLOCK M6 X 1	
5	250104	WASHER FLAT M6	
7	112317	BACKREST ADJUSTER STOP 2MM	Used w/ Ht Adj Backrest
8	109198	BACK BRACKET BUSHING	
9	110160	BACKREST BRACKET SADDLE	
12	109197	ADJUSTER BACKREST INSERT	
14	109204	BACKREST BOLT	
15	010963	SPRING BELLEVILLE 8MM	Special concave washer
16	112311	GRIP TOWEL BAR QUICKIE	
17	109822	BACKREST ADJ DELETE PLUG	PKG of 8
18	109730	PIN GUIDE BACKREST FIXED	See Note
19	110161	FRAME INSERT, REAR SEAT TUBE	
20	126833	FRAME INSERT OVAL DAB	Used w/ Depth Adj Back only for S/N after R4-023230
21	561082	1" TUBE END BUSHING THIN	
6,10,11	114875	FLD BCK PIN REPLC KIT	
3A,3B,4,5,8,9,12-15	159634S	BACK BRACKET KIT STD ANO	See note; Pair Brackets&Hdwr
1A,7,16,21	114615S	BCK FRM 4" ADJ STD 12W ASM	4"deep towel bar-12w-fits 10"-14",12"-16"&14"-18" back hts
1A,7,16,21	114616S	BCK FRM 4" ADJ STD 13W ASM	4"deep towel bar-13w-fits 10"-14",12"-16"&14"-18" back hts
1A,7,16,21	114617S	BCK FRM 4" ADJ STD 14W ASM	4"deep towel bar-14w-fits 10"-14",12"-16"&14"-18" back hts
1A,7,16,21	114618S	BCK FRM 4" ADJ STD 15W ASM	4"deep towel bar-15w-fits 10"-14",12"-16"&14"-18" back hts
1A,7,16,21	114619S	BCK FRM 4" ADJ STD 16W ASM	4"deep towel bar-16w-fits 10"-14",12"-16"&14"-18" back hts
1A,7,16,21	114620S	BCK FRM 4" ADJ STD 17W ASM	4"deep towel bar-17w-fits 10"-14",12"-16"&14"-18" back hts
1A,7,16,21	114621S	BCK FRM 4" ADJ STD 18W ASM	4"deep towel bar-18w-fits 10"-14",12"-16"&14"-18" back hts
1A,7,16,21	114622S	BCK FRM 4" ADJ STD 19W ASM	4"deep towel bar-19w-fits 10"-14",12"-16"&14"-18" back hts
1A,7,16,21	114623S	BCK FRM 4" ADJ STD 20W ASM	4"deep towel bar-20w-fits 10"-14",12"-16"&14"-18" back hts
1B,7,16,21	114626S	BCK FRM 4" ADJ TAL 12W ASM	4"deep towel bar-12w-fits 16"-20" back hts
1B,7,16,21	114627S	BCK FRM 4" ADJ TAL 13W ASM	4"deep towel bar-13w-fits 16"-20" back hts
1B,7,16,21	114628S	BCK FRM 4" ADJ TAL 14W ASM	4"deep towel bar-14w-fits 16"-20" back hts
1B,7,16,21	114629S	BCK FRM 4" ADJ TAL 15W ASM	4"deep towel bar-15w-fits 16"-20" back hts
1B,7,16,21	114630S	BCK FRM 4" ADJ TAL 16W ASM	4"deep towel bar-16w-fits 16"-20" back hts
1B,7,16,21	114631S	BCK FRM 4" ADJ TAL 17W ASM	4"deep towel bar-17w-fits 16"-20" back hts
1B,7,16,21	114632S	BCK FRM 4" ADJ TAL 18W ASM	4"deep towel bar-18w-fits 16"-20" back hts
1B,7,16,21	114633S	BCK FRM 4" ADJ TAL 19W ASM	4"deep towel bar-19w-fits 16"-20" back hts
1B,7,16,21	114634S	BCK FRM 4" ADJ TAL 20W ASM	4"deep towel bar-20w-fits 16"-20" back hts
1C,7,16,21	114637S	BCK FRM 2" ADJ STD 12W ASM	2"deep towel bar-12w-fits 10"-14",12"-16"&14"-18" back hts
1C,7,16,21	114638S	BCK FRM 2" ADJ STD 13W ASM	2"deep towel bar-13w-fits 10"-14",12"-16"&14"-18" back hts
1C,7,16,21	114639S	BCK FRM 2" ADJ STD 14W ASM	2"deep towel bar-14w-fits 10"-14",12"-16"&14"-18" back hts
1C,7,16,21	114640S	BCK FRM 2" ADJ STD 15W ASM	2"deep towel bar-15w-fits 10"-14",12"-16"&14"-18" back hts
1C,7,16,21	114641S	BCK FRM 2" ADJ STD 16W ASM	2"deep towel bar-16w-fits 10"-14",12"-16"&14"-18" back hts
1C,7,16,21	114642S	BCK FRM 2" ADJ STD 17W ASM	2"deep towel bar-17w-fits 10"-14",12"-16"&14"-18" back hts
1C,7,16,21	114643S	BCK FRM 2" ADJ STD 18W ASM	2"deep towel bar-18w-fits 10"-14",12"-16"&14"-18" back hts
1C,7,16,21	114644S	BCK FRM 2" ADJ STD 19W ASM	2"deep towel bar-19w-fits 10"-14",12"-16"&14"-18" back hts
1C,7,16,21	114645S	BCK FRM 2" ADJ STD 20W ASM	2"deep towel bar-20w-fits 10"-14",12"-16"&14"-18" back hts
1D,7,16,21	114648S	BCK FRM 2" ADJ TAL 12W ASM	2"deep towel bar-12w-fits 16"-20" back hts
1D,7,16,21	114649S	BCK FRM 2" ADJ TAL 13W ASM	2"deep towel bar-13w-fits 16"-20" back hts
1D,7,16,21	114650S	BCK FRM 2" ADJ TAL 14W ASM	2"deep towel bar-14w-fits 16"-20" back hts
1D,7,16,21	114651S	BCK FRM 2" ADJ TAL 15W ASM	2"deep towel bar-15w-fits 16"-20" back hts
1D,7,16,21	114652S	BCK FRM 2" ADJ TAL 16W ASM	2"deep towel bar-16w-fits 16"-20" back hts
1D,7,16,21	114653S	BCK FRM 2" ADJ TAL 17W ASM	2"deep towel bar-17w-fits 16"-20" back hts

Pos.	Item Number	Description	Remarks
1D,7,16,21	114654S	BCK FRM 2" ADJ TAL 18W ASM	2"deep towel bar-18w-fits 16"-20" back hts
1D,7,16,21	114655S	BCK FRM 2" ADJ TAL 19W ASM	2"deep towel bar-19w-fits 16"-20" back hts
1D,7,16,21	114656S	BCK FRM 2" ADJ TAL 20W ASM	2"deep towel bar-20w-fits 16"-20" back hts

Pos.	Item Number	Description	Remarks
2	109765	LANYARD THERMO-PRESS 12W	
2	109766	LANYARD THERMO-PRESS 13W	
2	109767	LANYARD THERMO-PRESS 14W	
2	109768	LANYARD THERMO-PRESS 15W	
2	109769	LANYARD THERMO-PRESS 16W	
2	109770	LANYARD THERMO-PRESS 17W	
2	109771	LANYARD THERMO-PRESS 18W	
2	109772	LANYARD THERMO-PRESS 19W	
2	109773	LANYARD THERMO-PRESS 20W	
3A	102262-006	BHCS,M6X1X40,PART,NL	Used w/o Ht Adj Push Handles
3B	102262-007	BHCS,M6X1X45,PART,N	Used w/ Ht Adj Push Handles
4	104381	NUT NYLOCK M6 X 1	
5	250104	WASHER FLAT M6	
7	112318	BACKREST ADJUSTER STOP 1MM	
8	109198	BACK BRACKET BUSHING	
9	110160	BACKREST BRACKET SADDLE	
12	109197	ADJUSTER BACKREST INSERT	
14	109204	BACKREST BOLT	
15	010963	SPRING BELLEVILLE 8MM	Special concave washer
16	112311	GRIP TOWEL BAR QUICKIE	
17	109822	BACKREST ADJ DELETE PLUG	PKG of 8
18	109730	PIN GUIDE BACKREST FIXED	See Note
19	110161	FRAME INSERT, REAR SEAT TUBE	
20	126833	FRAME INSERT OVAL DAB	Used w/ Depth Adj Back only for S/N after R4-023230
21A	561014	1" MED CAP	Omit Push Handle
21B	520148	ADAPTER, FLDG PUSH HANDLE	Used w/Fold Down Push Handle option
22	102264-004	BHCS, M6 X 1 X 25, FULL, PL	Used w/Fold Down Push Handle option
6,10,11	114875	FLD BCK PIN REPLC KIT	
3A,3B,4,5,8,9,12-15	159634S	BACK BRACKET KIT STD ANO	See note; Pair Brackets&Hdwr
1A,7,16,21A	114675S	BCK FRM 4" FIX 12WX8T ASM	4"deep towel bar-12w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,7,16,21A	114676S	BCK FRM 4" FIX 12WX9T ASM	4" deep towel bar-12"w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,7,16,21A	114677S	BCK FRM 4" FIX 12WX10T ASM	4"deep towel bar-12w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,7,16,21A	114678S	BCK FRM 4" FIX 12WX11T ASM	4"deep towel bar-12w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,7,16,21A	114679S	BCK FRM 4" FIX 12WX12T ASM	4"deep towel bar-12w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,7,16,21A	114680S	BCK FRM 4" FIX 12WX13T ASM	4"deep towel bar-12w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,7,16,21A	114681S	BCK FRM 4" FIX 12WX14T ASM	4"deep towel bar-12w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,7,16,21A	114682S	BCK FRM 4" FIX 12WX15T ASM	4"deep towel bar-12w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,7,16,21A	114683S	BCK FRM 4" FIX 12WX16T ASM	4"deep towel bar-12w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,7,16,21A	114684S	BCK FRM 4" FIX 13WX8T ASM	4"deep towel bar-13w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,7,16,21A	114685S	BCK FRM 4" FIX 13WX9T ASM	4"deep towel bar-13w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1A,7,16,21A	114686S	BCK FRM 4" FIX 13WX10T ASM	4"deep towel bar-13w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,7,16,21A	114687S	BCK FRM 4" FIX 13WX11T ASM	4"deep towel bar-13w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,7,16,21A	114688S	BCK FRM 4" FIX 13WX12T ASM	4"deep towel bar-13w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,7,16,21A	114689S	BCK FRM 4" FIX 13WX13T ASM	4"deep towel bar-13w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,7,16,21A	114690S	BCK FRM 4" FIX 13WX14T ASM	4"deep towel bar-13w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,7,16,21A	114691S	BCK FRM 4" FIX 13WX15T ASM	4"deep towel bar-13w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,7,16,21A	114692S	BCK FRM 4" FIX 13WX16T ASM	4"deep towel bar-13w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,7,16,21A	114693S	BCK FRM 4" FIX 14WX8T ASM	4"deep towel bar-14w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,7,16,21A	114694S	BCK FRM 4" FIX 14WX9T ASM	4"deep towel bar-14w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,7,16,21A	114695S	BCK FRM 4" FIX 14WX10T ASM	4"deep towel bar-14w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,7,16,21A	114696S	BCK FRM 4" FIX 14WX11T ASM	4"deep towel bar-14w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,7,16,21A	114697S	BCK FRM 4" FIX 14WX12T ASM	4"deep towel bar-14w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,7,16,21A	114698S	BCK FRM 4" FIX 14WX13T ASM	4"deep towel bar-14w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,7,16,21A	114699S	BCK FRM 4" FIX 14WX14T ASM	4"deep towel bar-14w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,7,16,21A	114700S	BCK FRM 4" FIX 14WX15T ASM	4"deep towel bar-14w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,7,16,21A	114701S	BCK FRM 4" FIX 14WX16T ASM	4"deep towel bar-14w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,7,16,21A	114702S	BCK FRM 4" FIX 15WX8T ASM	4"deep towel bar-15w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,7,16,21A	114703S	BCK FRM 4" FIX 15WX9T ASM	4"deep towel bar-15w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,7,16,21A	114704S	BCK FRM 4" FIX 15WX10T ASM	4"deep towel bar-15w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,7,16,21A	114705S	BCK FRM 4" FIX 15WX11T ASM	4"deep towel bar-15w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,7,16,21A	114706S	BCK FRM 4" FIX 15WX12T ASM	4"deep towel bar-15w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,7,16,21A	114707S	BCK FRM 4" FIX 15WX13T ASM	4"deep towel bar-15w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,7,16,21A	114708S	BCK FRM 4" FIX 15WX14T ASM	4"deep towel bar-15w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,7,16,21A	114709S	BCK FRM 4" FIX 15WX15T ASM	4"deep towel bar-15w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1A,7,16,21A	114710S	BCK FRM 4" FIX 15WX16T ASM	4"deep towel bar-15w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,7,16,21A	114711S	BCK FRM 4" FIX 16WX8T ASM	4"deep towel bar-16w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,7,16,21A	114712S	BCK FRM 4" FIX 16WX9T ASM	4"deep towel bar-16w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,7,16,21A	114713S	BCK FRM 4" FIX 16WX10T ASM	4"deep towel bar-16w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,7,16,21A	114714S	BCK FRM 4" FIX 16WX11T ASM	4"deep towel bar-16w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,7,16,21A	114715S	BCK FRM 4" FIX 16WX12T ASM	4"deep towel bar-16w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,7,16,21A	114716S	BCK FRM 4" FIX 16WX13T ASM	4"deep towel bar-16w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,7,16,21A	114717S	BCK FRM 4" FIX 16WX14T ASM	4"deep towel bar-16w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,7,16,21A	114718S	BCK FRM 4" FIX 16WX15T ASM	4"deep towel bar-16w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,7,16,21A	114719S	BCK FRM 4" FIX 16WX16T ASM	4"deep towel bar-16w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,7,16,21A	114720S	BCK FRM 4" FIX 17WX8T ASM	4"deep towel bar-17w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,7,16,21A	114721S	BCK FRM 4" FIX 17WX9T ASM	4"deep towel bar-17w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,7,16,21A	114722S	BCK FRM 4" FIX 17WX10T ASM	4"deep towel bar-17w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,7,16,21A	114723S	BCK FRM 4" FIX 17WX11T ASM	4"deep towel bar-17w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,7,16,21A	114724S	BCK FRM 4" FIX 17WX12T ASM	4"deep towel bar-17w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,7,16,21A	114725S	BCK FRM 4" FIX 17WX13T ASM	4"deep towel bar-17w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,7,16,21A	114726S	BCK FRM 4" FIX 17WX14T ASM	4"deep towel bar-17w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,7,16,21A	114727S	BCK FRM 4" FIX 17WX15T ASM	4"deep towel bar-17w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,7,16,21A	114728S	BCK FRM 4" FIX 17WX16T ASM	4"deep towel bar-17w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,7,16,21A	114729S	BCK FRM 4" FIX 18WX8T ASM	4"deep towel bar-18w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,7,16,21A	114730S	BCK FRM 4" FIX 18WX9T ASM	4"deep towel bar-18w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,7,16,21A	114731S	BCK FRM 4" FIX 18WX10T ASM	4"deep towel bar-18w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,7,16,21A	114732S	BCK FRM 4" FIX 18WX11T ASM	4"deep towel bar-18w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,7,16,21A	114733S	BCK FRM 4" FIX 18WX12T ASM	4"deep towel bar-18w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1A,7,16,21A	114734S	BCK FRM 4" FIX 18WX13T ASM	4"deep towel bar-18w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,7,16,21A	114735S	BCK FRM 4" FIX 18WX14T ASM	4"deep towel bar-18w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,7,16,21A	114736S	BCK FRM 4" FIX 18WX15T ASM	4"deep towel bar-18w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,7,16,21A	114737S	BCK FRM 4" FIX 18WX16T ASM	4"deep towel bar-18w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,7,16,21A	114738S	BCK FRM 4" FIX 19WX8T ASM	4"deep towel bar-19w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,7,16,21A	114739S	BCK FRM 4" FIX 19WX9T ASM	4"deep towel bar-19w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,7,16,21A	114740S	BCK FRM 4" FIX 19WX10T ASM	4"deep towel bar-19w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,7,16,21A	114741S	BCK FRM 4" FIX 19WX11T ASM	4"deep towel bar-19w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,7,16,21A	114742S	BCK FRM 4" FIX 19WX12T ASM	4"deep towel bar-19w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,7,16,21A	114743S	BCK FRM 4" FIX 19WX13T ASM	4"deep towel bar-19w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,7,16,21A	114744S	BCK FRM 4" FIX 19WX14T ASM	4"deep towel bar-19w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,7,16,21A	114745S	BCK FRM 4" FIX 19WX15T ASM	4"deep towel bar-19w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,7,16,21A	114746S	BCK FRM 4" FIX 19WX16T ASM	4"deep towel bar-19w-fits 16" fi ht Omit PH & 19" fix ht w/ Fold Down PH
1A,7,16,21A	114747S	BCK FRM 4" FIX 20WX8T ASM	4"deep towel bar-20w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,7,16,21A	114748S	BCK FRM 4" FIX 20WX9T ASM	4"deep towel bar-20w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,7,16,21A	114749S	BCK FRM 4" FIX 20WX10T ASM	4"deep towel bar-20w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,7,16,21A	114750S	BCK FRM 4" FIX 20WX11T ASM	4"deep towel bar-20w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,7,16,21A	114751S	BCK FRM 4" FIX 20WX12T ASM	4"deep towel bar-20w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,7,16,21A	114752S	BCK FRM 4" FIX 20WX13T ASM	4"deep towel bar-20w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,7,16,21A	114753S	BCK FRM 4" FIX 20WX14T ASM	4"deep towel bar-20w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,7,16,21A	114754S	BCK FRM 4" FIX 20WX15T ASM	4"deep towel bar-20w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,7,16,21A	114755S	BCK FRM 4" FIX 20WX16T ASM	4"deep towel bar-20w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,7,16,21A	114774S	BCK FRM 2" FIX 12WX8T ASM	2"deep towel bar-12w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,7,16,21A	114775S	BCK FRM 2" FIX 12WX9T ASM	2"deep towel bar-12w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1B,7,16,21A	114776S	BCK FRM 2" FIX 12WX10T ASM	2"deep towel bar-12w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,7,16,21A	114777S	BCK FRM 2" FIX 12WX11T ASM	2"deep towel bar-12w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,7,16,21A	114778S	BCK FRM 2" FIX 12WX12T ASM	2"deep towel bar-12w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,7,16,21A	114779S	BCK FRM 2" FIX 12WX13T ASM	2"deep towel bar-12w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,7,16,21A	114780S	BCK FRM 2" FIX 12WX14T ASM	2"deep towel bar-12w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,7,16,21A	114781S	BCK FRM 2" FIX 12WX15T ASM	2"deep towel bar-12w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,7,16,21A	114782S	BCK FRM 2" FIX 12WX16T ASM	2"deep towel bar-12w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,7,16,21A	114783S	BCK FRM 2" FIX 13WX8T ASM	2"deep towel bar-13w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,7,16,21A	114784S	BCK FRM 2" FIX 13WX9T ASM	2"deep towel bar-13w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,7,16,21A	114785S	BCK FRM 2" FIX 13WX10T ASM	2"deep towel bar-13w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,7,16,21A	114786S	BCK FRM 2" FIX 13WX11T ASM	2"deep towel bar-13w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,7,16,21A	114787S	BCK FRM 2" FIX 13WX12T ASM	2"deep towel bar-13w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,7,16,21A	114788S	BCK FRM 2" FIX 13WX13T ASM	2"deep towel bar-13w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,7,16,21A	114789S	BCK FRM 2" FIX 13WX14T ASM	2"deep towel bar-13w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,7,16,21A	114790S	BCK FRM 2" FIX 13WX15T ASM	2"deep towel bar-13w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,7,16,21A	114791S	BCK FRM 2" FIX 13WX16T ASM	2"deep towel bar-13w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,7,16,21A	114792S	BCK FRM 2" FIX 14WX8T ASM	2"deep towel bar-14w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,7,16,21A	114793S	BCK FRM 2" FIX 14WX9T ASM	2"deep towel bar-14w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,7,16,21A	114794S	BCK FRM 2" FIX 14WX10T ASM	2"deep towel bar-14w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,7,16,21A	114795S	BCK FRM 2" FIX 14WX11T ASM	2"deep towel bar-14w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,7,16,21A	114796S	BCK FRM 2" FIX 14WX12T ASM	2"deep towel bar-14w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,7,16,21A	114797S	BCK FRM 2" FIX 14WX13T ASM	2"deep towel bar-14w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,7,16,21A	114798S	BCK FRM 2" FIX 14WX14T ASM	2"deep towel bar-14w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,7,16,21A	114799S	BCK FRM 2" FIX 14WX15T ASM	2"deep towel bar-14w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1B,7,16,21A	114800S	BCK FRM 2" FIX 14WX16T ASM	2"deep towel bar-14w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,7,16,21A	114801S	BCK FRM 2" FIX 15WX8T ASM	2"deep towel bar-15w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,7,16,21A	114802S	BCK FRM 2" FIX 15WX9T ASM	2"deep towel bar-15w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,7,16,21A	114803S	BCK FRM 2" FIX 15WX10T ASM	2"deep towel bar-15w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,7,16,21A	114804S	BCK FRM 2" FIX 15WX11T ASM	2"deep towel bar-15w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,7,16,21A	114805S	BCK FRM 2" FIX 15WX12T ASM	2"deep towel bar-15w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,7,16,21A	114806S	BCK FRM 2" FIX 15WX13T ASM	2"deep towel bar-15w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,7,16,21A	114807S	BCK FRM 2" FIX 15WX14T ASM	2"deep towel bar-15w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,7,16,21A	114808S	BCK FRM 2" FIX 15WX15T ASM	2"deep towel bar-15w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,7,16,21A	114809S	BCK FRM 2" FIX 15WX16T ASM	2"deep towel bar-15w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,7,16,21A	114810S	BCK FRM 2" FIX 16WX8T ASM	2"deep towel bar-16w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,7,16,21A	114811S	BCK FRM 2" FIX 16WX9T ASM	2"deep towel bar-16w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,7,16,21A	114812S	BCK FRM 2" FIX 16WX10T ASM	2"deep towel bar-16w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,7,16,21A	114813S	BCK FRM 2" FIX 16WX11T ASM	2"deep towel bar-16w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,7,16,21A	114814S	BCK FRM 2" FIX 16WX12T ASM	2"deep towel bar-16w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,7,16,21A	114815S	BCK FRM 2" FIX 16WX13T ASM	2"deep towel bar-16w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,7,16,21A	114816S	BCK FRM 2" FIX 16WX14T ASM	2"deep towel bar-16w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,7,16,21A	114817S	BCK FRM 2" FIX 16WX15T ASM	2"deep towel bar-16w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,7,16,21A	114818S	BCK FRM 2" FIX 16WX16T ASM	2"deep towel bar-16w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,7,16,21A	114819S	BCK FRM 2" FIX 17WX8T ASM	2"deep towel bar-17w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,7,16,21A	114820S	BCK FRM 2" FIX 17WX9T ASM	2"deep towel bar-17w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,7,16,21A	114821S	BCK FRM 2" FIX 17WX10T ASM	2"deep towel bar-17w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,7,16,21A	114822S	BCK FRM 2" FIX 17WX11T ASM	2"deep towel bar-17w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,7,16,21A	114823S	BCK FRM 2" FIX 17WX12T ASM	2"deep towel bar-17w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1B,7,16,21A	114824S	BCK FRM 2" FIX 17WX13T ASM	2"deep towel bar-17w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,7,16,21A	114825S	BCK FRM 2" FIX 17WX14T ASM	2"deep towel bar-17w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,7,16,21A	114826S	BCK FRM 2" FIX 17WX15T ASM	2"deep towel bar-17w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,7,16,21A	114827S	BCK FRM 2" FIX 17WX16T ASM	2"deep towel bar-17w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,7,16,21A	114828S	BCK FRM 2" FIX 18WX8T ASM	2"deep towel bar-18w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,7,16,21A	114829S	BCK FRM 2" FIX 18WX9T ASM	2"deep towel bar-18w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,7,16,21A	114830S	BCK FRM 2" FIX 18WX10T ASM	2"deep towel bar-18w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,7,16,21A	114831S	BCK FRM 2" FIX 18WX11T ASM	2"deep towel bar-18w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,7,16,21A	114832S	BCK FRM 2" FIX 18WX12T ASM	2"deep towel bar-18w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,7,16,21A	114833S	BCK FRM 2" FIX 18WX13T ASM	2"deep towel bar-18w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,7,16,21A	114834S	BCK FRM 2" FIX 18WX14T ASM	2"deep towel bar-18w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,7,16,21A	114835S	BCK FRM 2" FIX 18WX15T ASM	2"deep towel bar-18w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,7,16,21A	114836S	BCK FRM 2" FIX 18WX16T ASM	2"deep towel bar-18w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,7,16,21A	114837S	BCK FRM 2" FIX 19WX8T ASM	2"deep towel bar-19w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,7,16,21A	114838S	BCK FRM 2" FIX 19WX9T ASM	2"deep towel bar-19w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,7,16,21A	114839S	BCK FRM 2" FIX 19WX10T ASM	2"deep towel bar-19w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,7,16,21A	114840S	BCK FRM 2" FIX 19WX11T ASM	2"deep towel bar-19w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,7,16,21A	114841S	BCK FRM 2" FIX 19WX12T ASM	2"deep towel bar-19w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,7,16,21A	114842S	BCK FRM 2" FIX 19WX13T ASM	2"deep towel bar-19w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,7,16,21A	114843S	BCK FRM 2" FIX 19WX14T ASM	2"deep towel bar-19w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,7,16,21A	114844S	BCK FRM 2" FIX 19WX15T ASM	2"deep towel bar-19w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,7,16,21A	114845S	BCK FRM 2" FIX 19WX16T ASM	2"deep towel bar-19w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,7,16,21A	114846S	BCK FRM 2" FIX 20WX8T ASM	2"deep towel bar-20w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,7,16,21A	114847S	BCK FRM 2" FIX 20WX9T ASM	2"deep towel bar-20w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1B,7,16,21A	114848S	BCK FRM 2" FIX 20WX10T ASM	2"deep towel bar-20w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,7,16,21A	114849S	BCK FRM 2" FIX 20WX11T ASM	2"deep towel bar-20w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,7,16,21A	114850S	BCK FRM 2" FIX 20WX12T ASM	2"deep towel bar-20w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,7,16,21A	114851S	BCK FRM 2" FIX 20WX13T ASM	2"deep towel bar-20w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,7,16,21A	114852S	BCK FRM 2" FIX 20WX14T ASM	2"deep towel bar-20w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,7,16,21A	114853S	BCK FRM 2" FIX 20WX15T ASM	2"deep towel bar-20w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,7,16,21A	114854S	BCK FRM 2" FIX 20WX16T ASM	2"deep towel bar-20w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH


Note: the Q7 has 8 degrees rearward to 27 degrees forward adjustment in 5 degree increments.

Note: Back frames on Inset Back measure 1" less than the chair width; example: for a 16" wide chair, a 15" wide back frame would be utilized; in addition, 12" wide chairs do not utilize a Inset Back.

Note: when moving backrest rearward with a Depth Adjustable Back item 25 will be used as a bushing for item 14.

Note: Lanyard not included in kits; order separately.

Note: if replacing Backrest Bracket parts on chairs built before S/N R4-018772, complete Backrest Bracket Kit is required; order 160933S BACK BRACKET KIT INSET ANO (new Back Plate Kit is required as Back Plate and mounting hardware have changed).

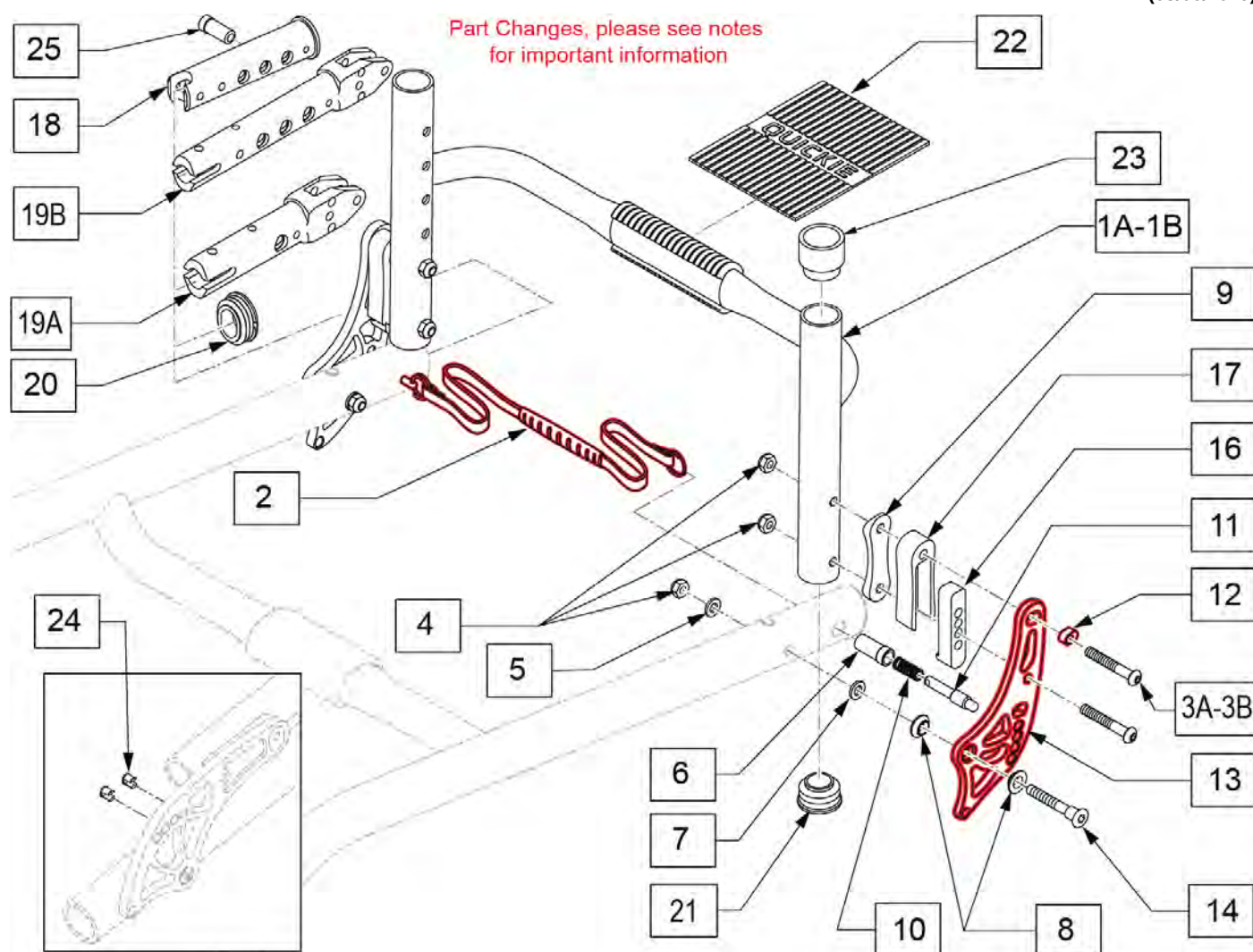

Pos.	Item Number	Description	Remarks
2	109765	LANYARD THERMO-PRESS 12W	Fits 13w chair w/ back inset
2	109766	LANYARD THERMO-PRESS 13W	Fits 14w chair w/ back inset
2	109767	LANYARD THERMO-PRESS 14W	Fits 15w chair w/ back inset
2	109768	LANYARD THERMO-PRESS 15W	Fits 16w chair w/ back inset
2	109769	LANYARD THERMO-PRESS 16W	Fits 17w chair w/ back inset
2	109770	LANYARD THERMO-PRESS 17W	Fits 18w chair w/ back inset
2	109771	LANYARD THERMO-PRESS 18W	Fits 19w chair w/ back inset
2	109772	LANYARD THERMO-PRESS 19W	Fits 20w chair w/ back inset
2	109773	LANYARD THERMO PRESS 20"W	Fits 21w chair w/ back inset
3A	058901-003	BHCS,M6X1X50,PART,PL	Used w/o Ht Adj Push Handles
3B	058901-004	BHCS,M6X1X55,PART,PL	Used w/ Ht Adj Push Handles
4	104381	NUT NYLOCK M6 X 1	
5	250104	WASHER FLAT M6	
7	010963	SPRING BELLEVILLE 8MM	Special concave washer
8	109198	BACK BRACKET BUSHING	
9	110160	BACKREST BRACKET SADDLE	
12	109197	ADJUSTER BACKREST INSERT	
14	109204	BACKREST BOLT	
16	112321	BACKREST INSET SPACER BOTTOM	
17	112320	BACKREST INSET SPACER TOP	

Pos.	Item Number	Description	Remarks
18	126833	FRAME INSERT OVAL DAB	Used w/ Depth Adj Back
19A	126832	ARM REST, PADDED TUBULAR MT	Used w/o Depth Adj Back
19B	126831	ARMREST, PADDED TUBULAR MT DAB	Used w/ Depth Adj Back
20	110161	FRAME INSERT, REAR SEAT TUB	
21	560735	7/8" ROUND TUBE PLUG	
22	112311	GRIP TOWEL BAR QUICKIE	
23	561082	1" TUBE END BUSHING THIN	
24	109822	BACKREST ADJ DELETE PLUG	PKG of 8
25	109730	PIN GUIDE BACKREST FIXED	See Note
6,10,11	114875	FLD BCK PIN REPLC KIT	
3A,3B,4,5,7-9,12-14	160933S	BACK BRACKET KIT INSET ANO	See note; Pair Brackets&Hdwr
1A,21,22,23	114615S	BCK FRM 4" ADJ STD 12W ASM	4"deep towel bar-12w-fits 10"-14",12"-16"&14"-18" back hts
1A,21,22,23	114616S	BCK FRM 4" ADJ STD 13W ASM	4"deep towel bar-13w-fits 10"-14",12"-16"&14"-18" back hts
1A,21,22,23	114617S	BCK FRM 4" ADJ STD 14W ASM	4"deep towel bar-14w-fits 10"-14",12"-16"&14"-18" back hts
1A,21,22,23	114618S	BCK FRM 4" ADJ STD 15W ASM	4"deep towel bar-15w-fits 10"-14",12"-16"&14"-18" back hts
1A,21,22,23	114619S	BCK FRM 4" ADJ STD 16W ASM	4"deep towel bar-16w-fits 10"-14",12"-16"&14"-18" back hts
1A,21,22,23	114620S	BCK FRM 4" ADJ STD 17W ASM	4"deep towel bar-17w-fits 10"-14",12"-16"&14"-18" back hts
1A,21,22,23	114621S	BCK FRM 4" ADJ STD 18W ASM	4"deep towel bar-18w-fits 10"-14",12"-16"&14"-18" back hts
1A,21,22,23	114622S	BCK FRM 4" ADJ STD 19W ASM	4"deep towel bar-19w-fits 10"-14",12"-16"&14"-18" back hts
1A,21,22,23	114626S	BCK FRM 4" ADJ TAL 12W ASM	4"deep towel bar-12w-fits 16"-20" back hts
1A,21,22,23	114627S	BCK FRM 4" ADJ TAL 13W ASM	4"deep towel bar-13w-fits 16"-20" back hts
1A,21,22,23	114628S	BCK FRM 4" ADJ TAL 14W ASM	4"deep towel bar-14w-fits 16"-20" back hts
1A,21,22,23	114629S	BCK FRM 4" ADJ TAL 15W ASM	4"deep towel bar-15w-fits 16"-20" back hts
1A,21,22,23	114630S	BCK FRM 4" ADJ TAL 16W ASM	4"deep towel bar-16w-fits 16"-20" back hts
1A,21,22,23	114631S	BCK FRM 4" ADJ TAL 17W ASM	4"deep towel bar-17w-fits 16"-20" back hts
1A,21,22,23	114632S	BCK FRM 4" ADJ TAL 18W ASM	4"deep towel bar-18w-fits 16"-20" back hts
1A,21,22,23	114633S	BCK FRM 4" ADJ TAL 19W ASM	4"deep towel bar-19w-fits 16"-20" back hts
1B,21,22,23	114637S	BCK FRM 2" ADJ STD 12W ASM	2"deep towel bar-13w-fits 10"-14",12"-16"&14"-18" back hts
1B,21,22,23	114638S	BCK FRM 2" ADJ STD 13W ASM	2"deep towel bar-14w-fits 10"-14",12"-16"&14"-18" back hts
1B,21,22,23	114639S	BCK FRM 2" ADJ STD 14W ASM	2"deep towel bar-15w-fits 10"-14",12"-16"&14"-18" back hts
1B,21,22,23	114640S	BCK FRM 2" ADJ STD 15W ASM	2"deep towel bar-16w-fits 10"-14",12"-16"&14"-18" back hts
1B,21,22,23	114641S	BCK FRM 2" ADJ STD 16W ASM	2"deep towel bar-17w-fits 10"-14",12"-16"&14"-18" back hts
1B,21,22,23	114642S	BCK FRM 2" ADJ STD 17W ASM	2"deep towel bar-18w-fits 10"-14",12"-16"&14"-18" back hts
1B,21,22,23	114643S	BCK FRM 2" ADJ STD 18W ASM	2"deep towel bar-19w-fits 10"-14",12"-16"&14"-18" back hts
1B,21,22,23	114644S	BCK FRM 2" ADJ STD 19W ASM	2"deep towel bar-19w-fits 10"-14",12"-16"&14"-18" back hts
1B,21,22,23	114648S	BCK FRM 2" ADJ TAL 12W ASM	2"deep towel bar-13w-fits 16"-20" back hts
1B,21,22,23	114649S	BCK FRM 2" ADJ TAL 13W ASM	2"deep towel bar-14w-fits 16"-20" back hts
1B,21,22,23	114650S	BCK FRM 2" ADJ TAL 14W ASM	2"deep towel bar-15w-fits 16"-20" back hts
1B,21,22,23	114651S	BCK FRM 2" ADJ TAL 15W ASM	2"deep towel bar-16w-fits 16"-20" back hts
1B,21,22,23	114652S	BCK FRM 2" ADJ TAL 16W ASM	2"deep towel bar-17w-fits 16"-20" back hts
1B,21,22,23	114653S	BCK FRM 2" ADJ TAL 17W ASM	2"deep towel bar-18w-fits 16"-20" back hts
1B,21,22,23	114654S	BCK FRM 2" ADJ TAL 18W ASM	2"deep towel bar-19w-fits 16"-20" back hts
1B,21,22,23	114655S	BCK FRM 2" ADJ TAL 19W ASM	2"deep towel bar-19w-fits 16"-20" back hts


Note: the Q7 has 8 degrees rearward to 27 degrees forward adjustment in 5 degree increments.

Note: Back frames on Inset Back measure 1" less than the chair width; example: for a 16" wide chair, a 15" wide back frame would be utilized; in addition, 12" wide chairs do not utilize a Inset Back.

Note: when moving backrest rearward with a Depth Adjustable Back item 25 will be used as a bushing for item 14.

Note: Lanyard not included in kits; order separately.

Note: if replacing Backrest Bracket parts on chairs built before S/N R4-018772, complete Backrest Bracket Kit is required; order 160933S BACK BRACKET KIT INSET ANO (new Back Plate Kit is required as Back Plate and mounting hardware have changed).

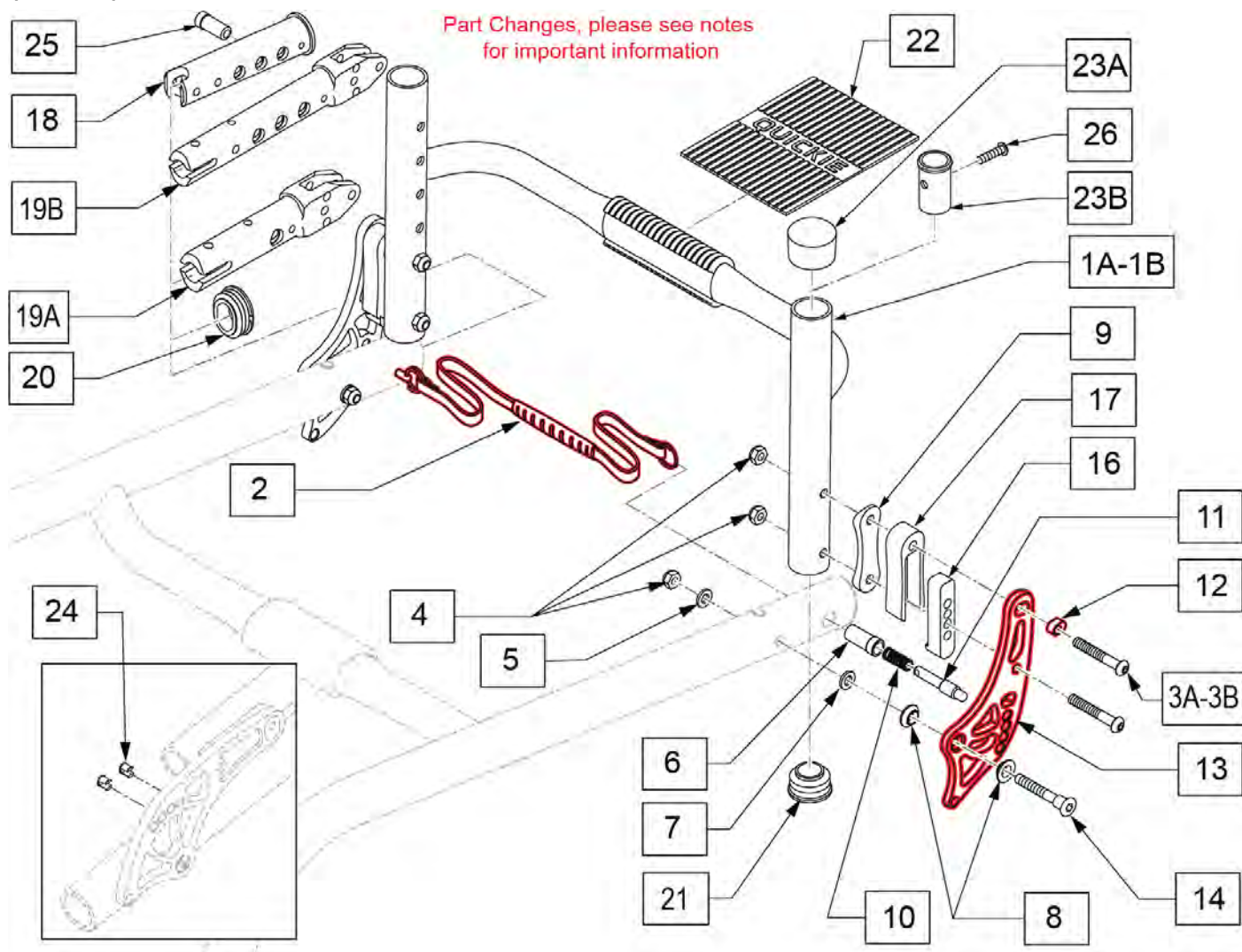

Pos.	Item Number	Description	Remarks
2	109765	LANYARD THERMO-PRESS 12W	Fits 13w chair w/ back inset
2	109766	LANYARD THERMO-PRESS 13W	Fits 14w chair w/ back inset
2	109767	LANYARD THERMO-PRESS 14W	Fits 15w chair w/ back inset
2	109768	LANYARD THERMO-PRESS 15W	Fits 16w chair w/ back inset
2	109769	LANYARD THERMO-PRESS 16W	Fits 17w chair w/ back inset
2	109770	LANYARD THERMO-PRESS 17W	Fits 18w chair w/ back inset
2	109771	LANYARD THERMO-PRESS 18W	Fits 19w chair w/ back inset
2	109772	LANYARD THERMO-PRESS 19W	Fits 20w chair w/ back inset
2	109773	LANYARD THERMO-PRESS 20"W	Fits 21w chair w/ back inset
3A	058901-003	BHCS,M6X1X50,PART,PL	Used w/o Ht Adj Push Handles
3B	058901-004	BHCS,M6X1X55,PART,PL	Used w/ Ht Adj Push Handles
4	104381	NUT NYLOCK M6 X 1	
5	250104	WASHER FLAT M6	
7	010963	SPRING BELLEVILLE 8MM	Special concave washer
8	109198	BACK BRACKET BUSHING	
9	110160	BACKREST BRACKET SADDLE	
12	109197	ADJUSTER BACKREST INSERT	
14	109204	BACKREST BOLT	
16	112321	BACKREST INSET SPACER BOTTOM	
17	112320	BACKREST INSET SPACER TOP	

Pos.	Item Number	Description	Remarks
18	126833	FRAME INSERT OVAL DAB	Used w/ Depth Adj Back
19A	126832	ARM REST, PADDED TUBULAR MT	Used w/o Depth Adj Back
19B	126831	ARMREST, PADDED TUBULAR MT DAB	Used w/ Depth Adj Back
20	110161	FRAME INSERT, REAR SEAT TUB	
21	561062	1" INSERT PLUG .065 RIBS	Used w/ Fixed Back
22	112311	GRIP TOWEL BAR QUICKIE	
23A	561014	1" MED CAP	Used for Omit Push Handle
23B	520148	ADAPTER, FLDG PUSH HANDLE	Used w/Fold Down Push Handle option
24	109822	BACKREST ADJ DELETE PLUG	PKG of 8
25	109730	PIN GUIDE BACKREST FIXED	See Note
26	102262-004	BHCS, M6 X 1 X 25, FULL, PL	Used w/Fold Down Push Handle option
6,10,11	114875	FLD BCK PIN REPLC KIT	
3A,3B,4,5,7-9,12-14	160933S	BACK BRACKET KIT INSET ANO	See note; Pair Brackets&Hdwr
1A,21,22,23A	114675S	BCK FRM 4" FIX 12WX8T ASM	4"deep towel bar-13w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,21,22,23A	114676S	BCK FRM 4" FIX 12WX9T ASM	4"deep towel bar-13w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,21,22,23A	114677S	BCK FRM 4" FIX 12WX10T ASM	4"deep towel bar-13w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,21,22,23A	114678S	BCK FRM 4" FIX 12WX11T ASM	4"deep towel bar-13w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,21,22,23A	114679S	BCK FRM 4" FIX 12WX12T ASM	4"deep towel bar-13w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,21,22,23A	114680S	BCK FRM 4" FIX 12WX13T ASM	4"deep towel bar-13w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,21,22,23A	114681S	BCK FRM 4" FIX 12WX14T ASM	4"deep towel bar-13w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,21,22,23A	114682S	BCK FRM 4" FIX 12WX15T ASM	4"deep towel bar-13w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,21,22,23A	114683S	BCK FRM 4" FIX 12WX16T ASM	4"deep towel bar-13w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,21,22,23A	114684S	BCK FRM 4" FIX 13WX8T ASM	4"deep towel bar-14w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,21,22,23A	114685S	BCK FRM 4" FIX 13WX9T ASM	4"deep towel bar-14w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,21,22,23A	114686S	BCK FRM 4" FIX 13WX10T ASM	4"deep towel bar-14w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,21,22,23A	114687S	BCK FRM 4" FIX 13WX11T ASM	4"deep towel bar-14w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,21,22,23A	114688S	BCK FRM 4" FIX 13WX12T ASM	4"deep towel bar-14w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,21,22,23A	114689S	BCK FRM 4" FIX 13WX13T ASM	4"deep towel bar-14w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,21,22,23A	114690S	BCK FRM 4" FIX 13WX14T ASM	4"deep towel bar-14w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,21,22,23A	114691S	BCK FRM 4" FIX 13WX15T ASM	4"deep towel bar-14w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,21,22,23A	114692S	BCK FRM 4" FIX 13WX16T ASM	4"deep towel bar-14w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1A,21,22,23A	114693S	BCK FRM 4" FIX 14WX8T ASM	4"deep towel bar-15w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,21,22,23A	114694S	BCK FRM 4" FIX 14WX9T ASM	4"deep towel bar-15w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,21,22,23A	114695S	BCK FRM 4" FIX 14WX10T ASM	4"deep towel bar-15w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,21,22,23A	114696S	BCK FRM 4" FIX 14WX11T ASM	4"deep towel bar-15w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,21,22,23A	114697S	BCK FRM 4" FIX 14WX12T ASM	4"deep towel bar-15w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,21,22,23A	114698S	BCK FRM 4" FIX 14WX13T ASM	4"deep towel bar-15w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,21,22,23A	114699S	BCK FRM 4" FIX 14WX14T ASM	4"deep towel bar-15w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,21,22,23A	114700S	BCK FRM 4" FIX 14WX15T ASM	4"deep towel bar-15w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,21,22,23A	114701S	BCK FRM 4" FIX 14WX16T ASM	4"deep towel bar-15w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,21,22,23A	114702S	BCK FRM 4" FIX 15WX8T ASM	4"deep towel bar-16w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,21,22,23A	114703S	BCK FRM 4" FIX 15WX9T ASM	4"deep towel bar-16w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,21,22,23A	114704S	BCK FRM 4" FIX 15WX10T ASM	4"deep towel bar-16w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,21,22,23A	114705S	BCK FRM 4" FIX 15WX11T ASM	4"deep towel bar-16w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,21,22,23A	114706S	BCK FRM 4" FIX 15WX12T ASM	4"deep towel bar-16w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,21,22,23A	114707S	BCK FRM 4" FIX 15WX13T ASM	4"deep towel bar-16w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,21,22,23A	114708S	BCK FRM 4" FIX 15WX14T ASM	4"deep towel bar-16w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,21,22,23A	114709S	BCK FRM 4" FIX 15WX15T ASM	4"deep towel bar-16w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,21,22,23A	114710S	BCK FRM 4" FIX 15WX16T ASM	4"deep towel bar-16w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,21,22,23A	114711S	BCK FRM 4" FIX 16WX8T ASM	4"deep towel bar-17w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,21,22,23A	114712S	BCK FRM 4" FIX 16WX9T ASM	4"deep towel bar-17w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,21,22,23A	114713S	BCK FRM 4" FIX 16WX10T ASM	4"deep towel bar-17w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,21,22,23A	114714S	BCK FRM 4" FIX 16WX11T ASM	4"deep towel bar-17w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,21,22,23A	114715S	BCK FRM 4" FIX 16WX12T ASM	4"deep towel bar-17w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,21,22,23A	114716S	BCK FRM 4" FIX 16WX13T ASM	4"deep towel bar-17w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1A,21,22,23A	114717S	BCK FRM 4" FIX 16WX14T ASM	4"deep towel bar-17w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,21,22,23A	114718S	BCK FRM 4" FIX 16WX15T ASM	4"deep towel bar-17w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,21,22,23A	114719S	BCK FRM 4" FIX 16WX16T ASM	4"deep towel bar-17w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,21,22,23A	114720S	BCK FRM 4" FIX 17WX8T ASM	4"deep towel bar-18w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,21,22,23A	114721S	BCK FRM 4" FIX 17WX9T ASM	4"deep towel bar-18w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,21,22,23A	114722S	BCK FRM 4" FIX 17WX10T ASM	4"deep towel bar-18w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,21,22,23A	114723S	BCK FRM 4" FIX 17WX11T ASM	4"deep towel bar-18w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,21,22,23A	114724S	BCK FRM 4" FIX 17WX12T ASM	4"deep towel bar-18w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,21,22,23A	114725S	BCK FRM 4" FIX 17WX13T ASM	4"deep towel bar-18w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,21,22,23A	114726S	BCK FRM 4" FIX 17WX14T ASM	4"deep towel bar-18w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,21,22,23A	114727S	BCK FRM 4" FIX 17WX15T ASM	4"deep towel bar-18w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,21,22,23A	114728S	BCK FRM 4" FIX 17WX16T ASM	4"deep towel bar-18w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,21,22,23A	114729S	BCK FRM 4" FIX 18WX8T ASM	4"deep towel bar-19w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,21,22,23A	114730S	BCK FRM 4" FIX 18WX9T ASM	4"deep towel bar-19w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,21,22,23A	114731S	BCK FRM 4" FIX 18WX10T ASM	4"deep towel bar-19w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1A,21,22,23A	114732S	BCK FRM 4" FIX 18WX11T ASM	4"deep towel bar-19w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,21,22,23A	114733S	BCK FRM 4" FIX 18WX12T ASM	4"deep towel bar-19w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,21,22,23A	114734S	BCK FRM 4" FIX 18WX13T ASM	4"deep towel bar-19w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,21,22,23A	114735S	BCK FRM 4" FIX 18WX14T ASM	4"deep towel bar-19w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,21,22,23A	114736S	BCK FRM 4" FIX 18WX15T ASM	4"deep towel bar-19w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,21,22,23A	114737S	BCK FRM 4" FIX 18WX16T ASM	4"deep towel bar-19w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,21,22,23A	114738S	BCK FRM 4" FIX 19WX8T ASM	4"deep towel bar-20w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1A,21,22,23A	114739S	BCK FRM 4" FIX 19WX9T ASM	4"deep towel bar-20w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1A,21,22,23A	114740S	BCK FRM 4" FIX 19WX10T ASM	4"deep towel bar-20w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1A,21,22,23A	114741S	BCK FRM 4" FIX 19WX11T ASM	4"deep towel bar-20w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1A,21,22,23A	114742S	BCK FRM 4" FIX 19WX12T ASM	4"deep towel bar-20w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1A,21,22,23A	114743S	BCK FRM 4" FIX 19WX13T ASM	4"deep towel bar-20w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1A,21,22,23A	114744S	BCK FRM 4" FIX 19WX14T ASM	4"deep towel bar-20w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1A,21,22,23A	114745S	BCK FRM 4" FIX 19WX15T ASM	4"deep towel bar-20w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,21,22,23A	114746S	BCK FRM 4" FIX 19WX16T ASM	4"deep towel bar-20w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1A,21,22,23A	114745S	BCK FRM 4" FIX 19WX15T ASM	4"deep towel bar-20w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1A,21,22,23A	114746S	BCK FRM 4" FIX 19WX16T ASM	4"deep towel bar-20w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,21,22,23A	114774S	BCK FRM 2" FIX 12WX8T ASM	2"deep towel bar-13w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,21,22,23A	114775S	BCK FRM 2" FIX 12WX9T ASM	2"deep towel bar-13w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,21,22,23A	114776S	BCK FRM 2" FIX 12WX10T ASM	2"deep towel bar-13w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,21,22,23A	114777S	BCK FRM 2" FIX 12WX11T ASM	2"deep towel bar-13w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,21,22,23A	114778S	BCK FRM 2" FIX 12WX12T ASM	2"deep towel bar-13w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,21,22,23A	114779S	BCK FRM 2" FIX 12WX13T ASM	2"deep towel bar-13w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,21,22,23A	114780S	BCK FRM 2" FIX 12WX14T ASM	2"deep towel bar-13w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,21,22,23A	114781S	BCK FRM 2" FIX 12WX15T ASM	2"deep towel bar-13w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,21,22,23A	114782S	BCK FRM 2" FIX 12WX16T ASM	2"deep towel bar-13w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,21,22,23A	114783S	BCK FRM 2" FIX 13WX8T ASM	2"deep towel bar-14w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,21,22,23A	114784S	BCK FRM 2" FIX 13WX9T ASM	2"deep towel bar-14w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,21,22,23A	114785S	BCK FRM 2" FIX 13WX10T ASM	2"deep towel bar-14w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,21,22,23A	114786S	BCK FRM 2" FIX 13WX11T ASM	2"deep towel bar-14w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,21,22,23A	114787S	BCK FRM 2" FIX 13WX12T ASM	2"deep towel bar-14w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,21,22,23A	114788S	BCK FRM 2" FIX 13WX13T ASM	2"deep towel bar-14w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,21,22,23A	114789S	BCK FRM 2" FIX 13WX14T ASM	2"deep towel bar-14w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1B,21,22,23A	114790S	BCK FRM 2" FIX 13WX15T ASM	2"deep towel bar-14w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,21,22,23A	114791S	BCK FRM 2" FIX 13WX16T ASM	2"deep towel bar-14w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,21,22,23A	114792S	BCK FRM 2" FIX 14WX8T ASM	2"deep towel bar-15w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,21,22,23A	114793S	BCK FRM 2" FIX 14WX9T ASM	2"deep towel bar-15w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,21,22,23A	114794S	BCK FRM 2" FIX 14WX10T ASM	2"deep towel bar-15w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,21,22,23A	114795S	BCK FRM 2" FIX 14WX11T ASM	2"deep towel bar-15w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,21,22,23A	114796S	BCK FRM 2" FIX 14WX12T ASM	2"deep towel bar-15w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,21,22,23A	114797S	BCK FRM 2" FIX 14WX13T ASM	2"deep towel bar-15w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,21,22,23A	114798S	BCK FRM 2" FIX 14WX14T ASM	2"deep towel bar-15w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,21,22,23A	114799S	BCK FRM 2" FIX 14WX15T ASM	2"deep towel bar-15w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,21,22,23A	114800S	BCK FRM 2" FIX 14WX16T ASM	2"deep towel bar-15w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,21,22,23A	114801S	BCK FRM 2" FIX 15WX8T ASM	2"deep towel bar-16w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,21,22,23A	114802S	BCK FRM 2" FIX 15WX9T ASM	2"deep towel bar-16w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,21,22,23A	114803S	BCK FRM 2" FIX 15WX10T ASM	2"deep towel bar-16w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,21,22,23A	114804S	BCK FRM 2" FIX 15WX11T ASM	2"deep towel bar-16w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,21,22,23A	114805S	BCK FRM 2" FIX 15WX12T ASM	2"deep towel bar-16w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,21,22,23A	114806S	BCK FRM 2" FIX 15WX13T ASM	2"deep towel bar-16w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,21,22,23A	114807S	BCK FRM 2" FIX 15WX14T ASM	2"deep towel bar-16w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,21,22,23A	114808S	BCK FRM 2" FIX 15WX15T ASM	2"deep towel bar-16w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,21,22,23A	114809S	BCK FRM 2" FIX 15WX16T ASM	2"deep towel bar-16w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,21,22,23A	114810S	BCK FRM 2" FIX 16WX8T ASM	2"deep towel bar-17w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,21,22,23A	114811S	BCK FRM 2" FIX 16WX9T ASM	2"deep towel bar-17w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,21,22,23A	114812S	BCK FRM 2" FIX 16WX10T ASM	2"deep towel bar-17w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,21,22,23A	114813S	BCK FRM 2" FIX 16WX11T ASM	2"deep towel bar-17w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH


Pos.	Item Number	Description	Remarks
1B,21,22,23A	114814S	BCK FRM 2" FIX 16WX12T ASM	2"deep towel bar-17w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,21,22,23A	114815S	BCK FRM 2" FIX 16WX13T ASM	2"deep towel bar-17w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,21,22,23A	114816S	BCK FRM 2" FIX 16WX14T ASM	2"deep towel bar-17w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,21,22,23A	114817S	BCK FRM 2" FIX 16WX15T ASM	2"deep towel bar-17w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,21,22,23A	114818S	BCK FRM 2" FIX 16WX16T ASM	2"deep towel bar-17w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,21,22,23A	114819S	BCK FRM 2" FIX 17WX8T ASM	2"deep towel bar-18w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,21,22,23A	114820S	BCK FRM 2" FIX 17WX9T ASM	2"deep towel bar-18w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,21,22,23A	114821S	BCK FRM 2" FIX 17WX10T ASM	2"deep towel bar-18w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,21,22,23A	114822S	BCK FRM 2" FIX 17WX11T ASM	2"deep towel bar-18w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,21,22,23A	114823S	BCK FRM 2" FIX 17WX12T ASM	2"deep towel bar-18w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,21,22,23A	114824S	BCK FRM 2" FIX 17WX13T ASM	2"deep towel bar-18w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,21,22,23A	114825S	BCK FRM 2" FIX 17WX14T ASM	2"deep towel bar-18w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,21,22,23A	114826S	BCK FRM 2" FIX 17WX15T ASM	2"deep towel bar-18w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,21,22,23A	114827S	BCK FRM 2" FIX 17WX16T ASM	2"deep towel bar-18w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,21,22,23A	114828S	BCK FRM 2" FIX 18WX8T ASM	2"deep towel bar-19w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1B,21,22,23A	114829S	BCK FRM 2" FIX 18WX9T ASM	2"deep towel bar-19w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,21,22,23A	114830S	BCK FRM 2" FIX 18WX10T ASM	2"deep towel bar-19w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,21,22,23A	114831S	BCK FRM 2" FIX 18WX11T ASM	2"deep towel bar-19w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,21,22,23A	114832S	BCK FRM 2" FIX 18WX12T ASM	2"deep towel bar-19w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,21,22,23A	114833S	BCK FRM 2" FIX 18WX13T ASM	2"deep towel bar-19w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1B,21,22,23A	114834S	BCK FRM 2" FIX 18WX14T ASM	2"deep towel bar-19w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1B,21,22,23A	114835S	BCK FRM 2" FIX 18WX15T ASM	2"deep towel bar-19w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1B,21,22,23A	114836S	BCK FRM 2" FIX 18WX16T ASM	2"deep towel bar-19w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1B,21,22,23A	114837S	BCK FRM 2" FIX 19WX8T ASM	2"deep towel bar-20w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1B,21,22,23A	114838S	BCK FRM 2" FIX 19WX9T ASM	2"deep towel bar-20w- fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1B,21,22,23A	114839S	BCK FRM 2" FIX 19WX10T ASM	2"deep towel bar-20w- fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1B,21,22,23A	114840S	BCK FRM 2" FIX 19WX11T ASM	2"deep towel bar-20w- fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1B,21,22,23A	114841S	BCK FRM 2" FIX 19WX12T ASM	2"deep towel bar-20w- fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1B,21,22,23A	114842S	BCK FRM 2" FIX 19WX13T ASM	2"deep towel bar-20w- fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH

Note: if replacing Backrest Bracket parts on chairs built before S/N R4-018772, complete Backrest Bracket Kit is required; order qty 1 of p/n 125781S or 160932S (new Back Plate Kit is required as Back Plate and mounting hardware have changed).

Note: the Q7 has 8 degrees rearward to 27 degrees forward adjustment in 5 degree increments.

Note: when moving backrest rearward with a Depth Adjustable Back item 20 will be used as a bushing for item 12.

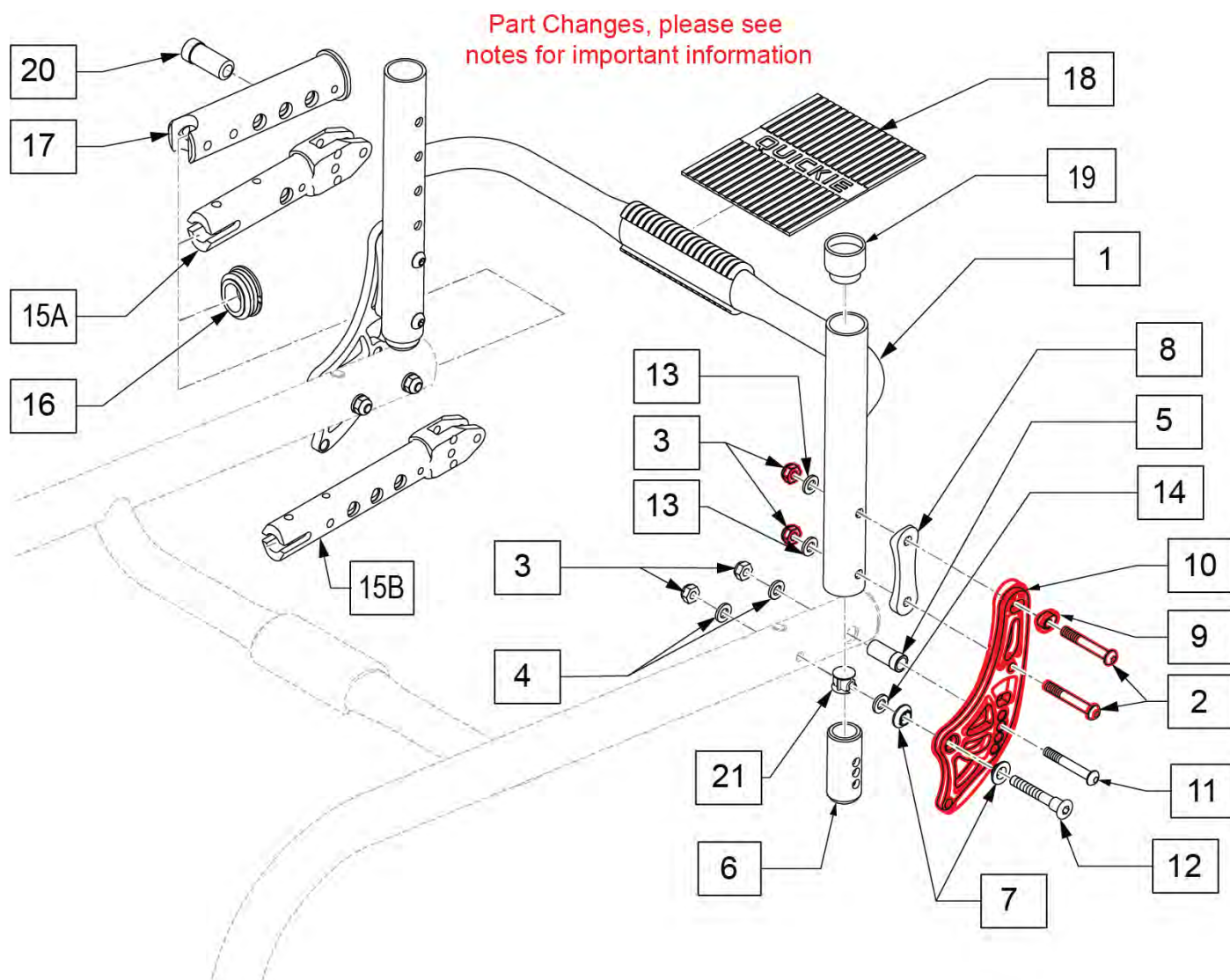


Pos.	Item Number	Description	Remarks
2	102262-007	BHCS, M6 X 1 X4 5, PART, N	
3	104381	NUT NYLOCK M6 X 1	
4	250104	WASHER FLAT M6	
5	109730	PIN GUIDE BACKREST FIXED	Fixed back only, not compatible with Folding Back
6	112317	BACKREST ADJUSTER STOP 2MM	

Pos.	Item Number	Description	Remarks
7	109198	BACK BRACKET BUSHING	
8	110160	BACKREST BRACKET SADDLE	
9	109197	ADJUSTER BACKREST INSERT	
11	102262-007	BHCS M6X1X45,FULL,PL	
12	109204	BACKREST BOLT	
13	104414	WAVE WASHER 6MM	
14	010963	SPRING BELLEVILLE 8MM	
15A	126832	ARM REST, PADDED TUBULAR MT	Used w/o Depth Adj Back
15B	126831	ARMREST, PADDED TUBULAR MT DAB	Used w/ Depth Adj Back
16	110161	FRAME INSERT, REAR SEAT TUBE	
17	126833	FRAME INSERT OVAL DAB	Used w/ Depth Adj Back
18	112311	GRIP TOWEL BAR QUICKIE	
19	561082	1" TUBE END BUSHING THIN	
20	109730	PIN GUIDE BACKREST FIXED	See Note
21	340038	PLUG TUBE 1.00 OD X .035 THK	
2-14	114873	N-FOLD ANGL ADJ BACK KIT	
2-5,7-10,12	125781S	ANO NON FLD BK BRKTS Q7 PR	See note; Anodized colors; Pair Brackets&Hdwr
2-4,7,9-12,14	160932S	BACK BRACKET KIT STD ANO	See note; Anodized colors; Pair Brackets&Hdwr
3,4,7-14	159634S	BRKT W/HDWR KIT FLDNG ANO	Pair: Brackets&Hdwr
1,6,18,19	114615S	BCK FRM 4" ADJ STD 12W ASM	4"deep towel bar-12w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114616S	BCK FRM 4" ADJ STD 13W ASM	4"deep towel bar-13w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114617S	BCK FRM 4" ADJ STD 14W ASM	4"deep towel bar-14w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114618S	BCK FRM 4" ADJ STD 15W ASM	4"deep towel bar-15w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114619S	BCK FRM 4" ADJ STD 16W ASM	4"deep towel bar-16w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114620S	BCK FRM 4" ADJ STD 17W ASM	4"deep towel bar-17w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114621S	BCK FRM 4" ADJ STD 18W ASM	4"deep towel bar-18w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114622S	BCK FRM 4" ADJ STD 19W ASM	4"deep towel bar-19w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114623S	BCK FRM 4" ADJ STD 20W ASM	4"deep towel bar-20w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114626S	BCK FRM 4" ADJ TAL 12W ASM	4"deep towel bar-12w-fits 16"-20" back hts
1,6,18,19	114627S	BCK FRM 4" ADJ TAL 13W ASM	4"deep towel bar-13w-fits 16"-20" back hts
1,6,18,19	114628S	BCK FRM 4" ADJ TAL 14W ASM	4"deep towel bar-14w-fits 16"-20" back hts
1,6,18,19	114629S	BCK FRM 4" ADJ TAL 15W ASM	4"deep towel bar-15w-fits 16"-20" back hts
1,6,18,19	114630S	BCK FRM 4" ADJ TAL 16W ASM	4"deep towel bar-16w-fits 16"-20" back hts
1,6,18,19	114631S	BCK FRM 4" ADJ TAL 17W ASM	4"deep towel bar-17w-fits 16"-20" back hts
1,6,18,19	114632S	BCK FRM 4" ADJ TAL 18W ASM	4"deep towel bar-18w-fits 16"-20" back hts
1,6,18,19	114633S	BCK FRM 4" ADJ TAL 19W ASM	4"deep towel bar-19w-fits 16"-20" back hts
1,6,18,19	114634S	BCK FRM 4" ADJ TAL 20W ASM	4"deep towel bar-20w-fits 16"-20" back hts
1,6,18,19	114637S	BCK FRM 2" ADJ STD 12W ASM	2"deep towel bar-12w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114638S	BCK FRM 2" ADJ STD 13W ASM	2"deep towel bar-13w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114639S	BCK FRM 2" ADJ STD 14W ASM	2"deep towel bar-14w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114640S	BCK FRM 2" ADJ STD 15W ASM	2"deep towel bar-15w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114641S	BCK FRM 2" ADJ STD 16W ASM	2"deep towel bar-16w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114642S	BCK FRM 2" ADJ STD 17W ASM	2"deep towel bar-17w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114643S	BCK FRM 2" ADJ STD 18W ASM	2"deep towel bar-18w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114644S	BCK FRM 2" ADJ STD 19W ASM	2"deep towel bar-19w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114645S	BCK FRM 2" ADJ STD 20W ASM	2"deep towel bar-20w-fits 10"-14",12"-16"&14"-18" back hts
1,6,18,19	114648S	BCK FRM 2" ADJ TAL 12W ASM	2"deep towel bar-12w-fits 16"-20" back hts
1,6,18,19	114649S	BCK FRM 2" ADJ TAL 13W ASM	2"deep towel bar-13w-fits 16"-20" back hts
1,6,18,19	114650S	BCK FRM 2" ADJ TAL 14W ASM	2"deep towel bar-14w-fits 16"-20" back hts
1,6,18,19	114651S	BCK FRM 2" ADJ TAL 15W ASM	2"deep towel bar-15w-fits 16"-20" back hts
1,6,18,19	114652S	BCK FRM 2" ADJ TAL 16W ASM	2"deep towel bar-16w-fits 16"-20" back hts
1,6,18,19	114653S	BCK FRM 2" ADJ TAL 17W ASM	2"deep towel bar-17w-fits 16"-20" back hts
1,6,18,19	114654S	BCK FRM 2" ADJ TAL 18W ASM	2"deep towel bar-18w-fits 16"-20" back hts
1,6,18,19	114655S	BCK FRM 2" ADJ TAL 19W ASM	2"deep towel bar-19w-fits 16"-20" back hts
1,6,18,19	114656S	BCK FRM 2" ADJ TAL 20W ASM	2"deep towel bar-20w-fits 16"-20" back hts

Note: the Q7 has 8 degrees rearward to 27 degrees forward adjustment in 5 degree increments.

Note: when moving backrest rearward with a Depth Adjustable Back item 20 will be used as a bushing for item 12.

Note: Backrest Handle (Ht Adj) not available on Fixed Height Backrest style.

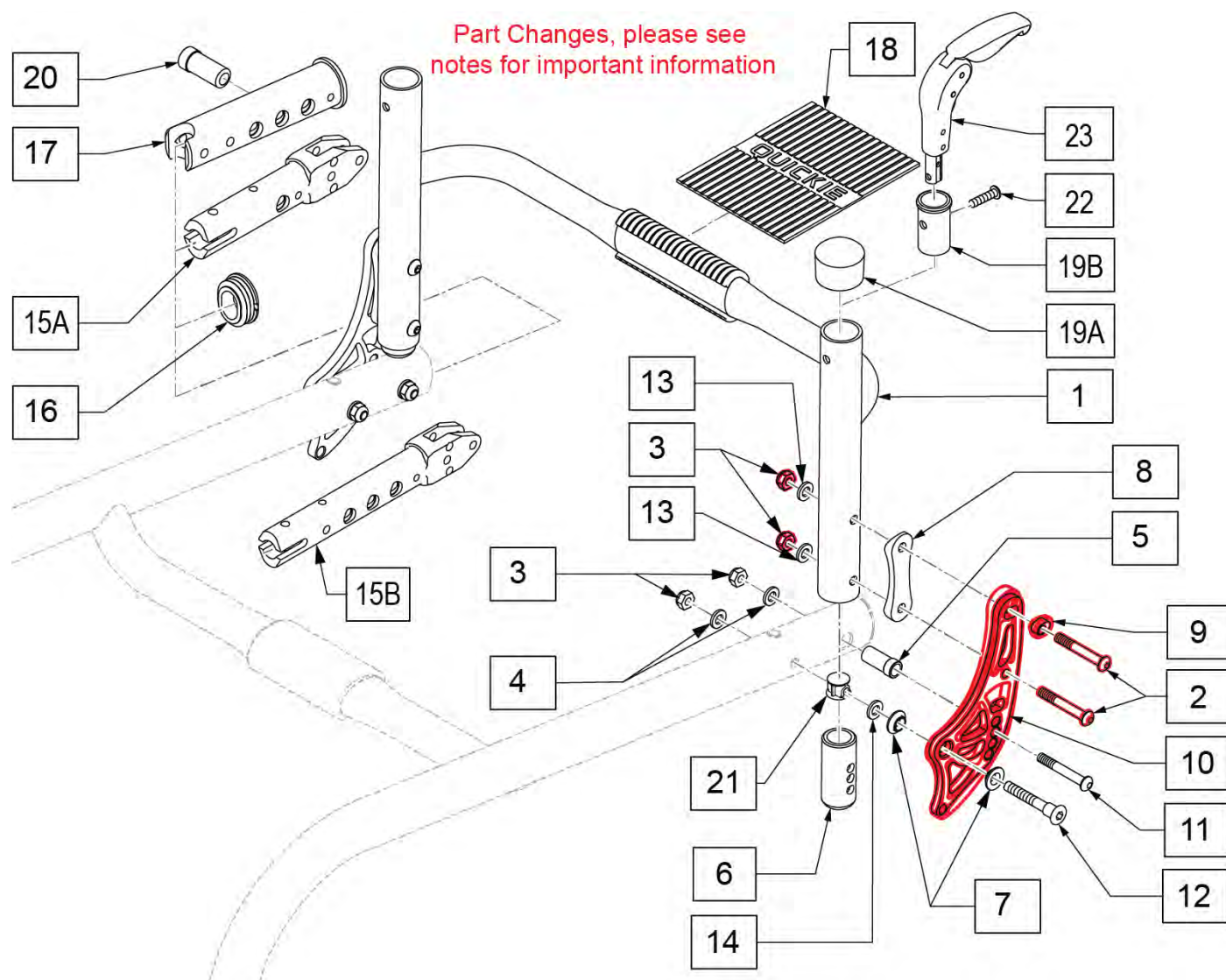

Pos.	Item Number	Description	Remarks
2	102262-006	BHCS, M6 X 1 X 40, PART,NL	
3	104381	NUT NYLOCK M6 X 1	
4	250104	WASHER FLAT M6	
5	109730	PIN GUIDE BACKREST FIXED	Fixed back only, not compatible with Folding Back
6	112318	BACKREST ADJUSTER STOP 1MM	
7	109198	BACK BRACKET BUSHING	
8	110160	BACKREST BRACKET SADDLE	
9	109197	ADJUSTER BACKREST INSERT	
11	102262-007	BHCS M6X1X45,FULL,PL	
12	109204	BACKREST BOLT	
13	104414	WAVE WASHER 6MM	
14	010963	SPRING BELLEVILLE 8MM	
15A	126832	ARM REST, PADDED TUBULAR MT	Used w/o Depth Adj Back
15B	126831	ARMREST, PADDED TUBULAR MT DAB	Used w/ Depth Adj Back
16	110161	FRAME INSERT, REAR SEAT TUBE	
17	126833	FRAME INSERT OVAL DAB	Used w/ Depth Adj Back
18	112311	GRIP TOWEL BAR QUICKIE	
19A	561014	1" MED CAP	Used w/ Omit Push Handle
19B	520148	ADAPTER, FLDG PUSH HANDLE	Used w/ Fold Down Push Handle option
20	109730	PIN GUIDE BACKREST FIXED	See Note
21	340038	PLUG TUBE 1.00 OD X .035 THK	
22	102262-004	BHCS, M6 X 1 X 25, FULL PL	Used w/ Fold Down Push Handle option
23	882604	FLDG PUSH HNDL W/INSRT FXD HGT	Each
19B,22,23	900164	FLDG PUSH HNDL ASSM FXD HGT	Each
2-14	114873	N-FOLD ANGL ADJ BACK KIT	
2A,2B,3-5,7-10,12	125781S	ANO NON FLD BK BRKTS Q7 PR	Pair, Anodized colors
2A,2B,4,7,9-12,14	160932S	BACK BRACKET KIT STD ANO	
3,4,7-14	159634S	BRKT W/HDWR KIT FLDNG ANO	See note; Pair Brackets&Hdwr
1,6,18,19A	114675S	BCK FRM 4" FIX 12WX8T ASM	4"deep towel bar-12w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114676S	BCK FRM 4" FIX 12WX9T ASM	4"deep towel bar-12w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114677S	BCK FRM 4" FIX 12WX10T ASM	4"deep towel bar-12w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114678S	BCK FRM 4" FIX 12WX11T ASM	4"deep towel bar-12w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114679S	BCK FRM 4" FIX 12WX12T ASM	4"deep towel bar-12w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114680S	BCK FRM 4" FIX 12WX13T ASM	4"deep towel bar-12w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114681S	BCK FRM 4" FIX 12WX14T ASM	4"deep towel bar-12w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114682S	BCK FRM 4" FIX 12WX15T ASM	4"deep towel bar-12w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114683S	BCK FRM 4" FIX 12WX16T ASM	4"deep towel bar-12w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114684S	BCK FRM 4" FIX 13WX8T ASM	4"deep towel bar-13w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114685S	BCK FRM 4" FIX 13WX9T ASM	4"deep towel bar-13w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114686S	BCK FRM 4" FIX 13WX10T ASM	4"deep towel bar-13w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1,6,18,19A	114687S	BCK FRM 4" FIX 13WX11T ASM	4"deep towel bar-13w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114688S	BCK FRM 4" FIX 13WX12T ASM	4"deep towel bar-13w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114689S	BCK FRM 4" FIX 13WX13T ASM	4"deep towel bar-13w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114690S	BCK FRM 4" FIX 13WX14T ASM	4"deep towel bar-13w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114691S	BCK FRM 4" FIX 13WX15T ASM	4"deep towel bar-13w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114692S	BCK FRM 4" FIX 13WX16T ASM	4"deep towel bar-13w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114693S	BCK FRM 4" FIX 14WX8T ASM	4"deep towel bar-14w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114694S	BCK FRM 4" FIX 14WX9T ASM	4"deep towel bar-14w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114695S	BCK FRM 4" FIX 14WX10T ASM	4"deep towel bar-14w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114696S	BCK FRM 4" FIX 14WX11T ASM	4"deep towel bar-14w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114697S	BCK FRM 4" FIX 14WX12T ASM	4"deep towel bar-14w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114698S	BCK FRM 4" FIX 14WX13T ASM	4"deep towel bar-14w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114699S	BCK FRM 4" FIX 14WX14T ASM	4"deep towel bar-14w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114700S	BCK FRM 4" FIX 14WX15T ASM	4"deep towel bar-14w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114701S	BCK FRM 4" FIX 14WX16T ASM	4"deep towel bar-14w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114702S	BCK FRM 4" FIX 15WX8T ASM	4"deep towel bar-15w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114703S	BCK FRM 4" FIX 15WX9T ASM	4"deep towel bar-15w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114704S	BCK FRM 4" FIX 15WX10T ASM	4"deep towel bar-15w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114705S	BCK FRM 4" FIX 15WX11T ASM	4"deep towel bar-15w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114706S	BCK FRM 4" FIX 15WX12T ASM	4"deep towel bar-15w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114707S	BCK FRM 4" FIX 15WX13T ASM	4"deep towel bar-15w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114708S	BCK FRM 4" FIX 15WX14T ASM	4"deep towel bar-15w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114709S	BCK FRM 4" FIX 15WX15T ASM	4"deep towel bar-15w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114710S	BCK FRM 4" FIX 15WX16T ASM	4"deep towel bar-15w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1,6,18,19A	114711S	BCK FRM 4" FIX 16WX8T ASM	4"deep towel bar-16w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114712S	BCK FRM 4" FIX 16WX9T ASM	4"deep towel bar-16w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114713S	BCK FRM 4" FIX 16WX10T ASM	4"deep towel bar-16w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114714S	BCK FRM 4" FIX 16WX11T ASM	4"deep towel bar-16w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114715S	BCK FRM 4" FIX 16WX12T ASM	4"deep towel bar-16w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114716S	BCK FRM 4" FIX 16WX13T ASM	4"deep towel bar-16w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114717S	BCK FRM 4" FIX 16WX14T ASM	4"deep towel bar-16w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114718S	BCK FRM 4" FIX 16WX15T ASM	4"deep towel bar-16w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114719S	BCK FRM 4" FIX 16WX16T ASM	4"deep towel bar-16w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114720S	BCK FRM 4" FIX 17WX8T ASM	4"deep towel bar-17w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114721S	BCK FRM 4" FIX 17WX9T ASM	4"deep towel bar-17w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114722S	BCK FRM 4" FIX 17WX10T ASM	4"deep towel bar-17w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114723S	BCK FRM 4" FIX 17WX11T ASM	4"deep towel bar-17w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114724S	BCK FRM 4" FIX 17WX12T ASM	4"deep towel bar-17w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114725S	BCK FRM 4" FIX 17WX13T ASM	4"deep towel bar-17w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114726S	BCK FRM 4" FIX 17WX14T ASM	4"deep towel bar-17w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114727S	BCK FRM 4" FIX 17WX15T ASM	4"deep towel bar-17w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114728S	BCK FRM 4" FIX 17WX16T ASM	4"deep towel bar-17w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114729S	BCK FRM 4" FIX 18WX8T ASM	4"deep towel bar-18w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114730S	BCK FRM 4" FIX 18WX9T ASM	4"deep towel bar-18w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114731S	BCK FRM 4" FIX 18WX10T ASM	4"deep towel bar-18w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114732S	BCK FRM 4" FIX 18WX11T ASM	4"deep towel bar-18w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114733S	BCK FRM 4" FIX 18WX12T ASM	4"deep towel bar-18w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114734S	BCK FRM 4" FIX 18WX13T ASM	4"deep towel bar-18w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1,6,18,19A	114735S	BCK FRM 4" FIX 18WX14T ASM	4"deep towel bar-18w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114736S	BCK FRM 4" FIX 18WX15T ASM	4"deep towel bar-18w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114737S	BCK FRM 4" FIX 18WX16T ASM	4"deep towel bar-18w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114738S	BCK FRM 4" FIX 19WX8T ASM	4"deep towel bar-19w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114739S	BCK FRM 4" FIX 19WX9T ASM	4"deep towel bar-19w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114740S	BCK FRM 4" FIX 19WX10T ASM	4"deep towel bar-19w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114741S	BCK FRM 4" FIX 19WX11T ASM	4"deep towel bar-19w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114742S	BCK FRM 4" FIX 19WX12T ASM	4"deep towel bar-19w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114743S	BCK FRM 4" FIX 19WX13T ASM	4"deep towel bar-19w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114744S	BCK FRM 4" FIX 19WX14T ASM	4"deep towel bar-19w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114745S	BCK FRM 4" FIX 19WX15T ASM	4"deep towel bar-19w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114746S	BCK FRM 4" FIX 19WX16T ASM	4"deep towel bar-19w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114747S	BCK FRM 4" FIX 20WX8T ASM	4"deep towel bar-20w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114748S	BCK FRM 4" FIX 20WX9T ASM	4"deep towel bar-20w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114749S	BCK FRM 4" FIX 20WX10T ASM	4"deep towel bar-20w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114750S	BCK FRM 4" FIX 20WX11T ASM	4"deep towel bar-20w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114751S	BCK FRM 4" FIX 20WX12T ASM	4"deep towel bar-20w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114752S	BCK FRM 4" FIX 20WX13T ASM	4"deep towel bar-20w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114753S	BCK FRM 4" FIX 20WX14T ASM	4"deep towel bar-20w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114754S	BCK FRM 4" FIX 20WX15T ASM	4"deep towel bar-20w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114755S	BCK FRM 4" FIX 20WX16T ASM	4"deep towel bar-20w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114774S	BCK FRM 2" FIX 12WX8T ASM	2"deep towel bar-12w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114775S	BCK FRM 2" FIX 12WX9T ASM	2"deep towel bar-12w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114776S	BCK FRM 2" FIX 12WX10T ASM	2"deep towel bar-12w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1,6,18,19A	114777S	BCK FRM 2" FIX 12WX11T ASM	2"deep towel bar-12w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114778S	BCK FRM 2" FIX 12WX12T ASM	2"deep towel bar-12w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114779S	BCK FRM 2" FIX 12WX13T ASM	2"deep towel bar-12w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114780S	BCK FRM 2" FIX 12WX14T ASM	2"deep towel bar-12w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114781S	BCK FRM 2" FIX 12WX15T ASM	2"deep towel bar-12w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114782S	BCK FRM 2" FIX 12WX16T ASM	2"deep towel bar-12w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114783S	BCK FRM 2" FIX 13WX8T ASM	2"deep towel bar-13w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114784S	BCK FRM 2" FIX 13WX9T ASM	2"deep towel bar-13w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114785S	BCK FRM 2" FIX 13WX10T ASM	2"deep towel bar-13w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114786S	BCK FRM 2" FIX 13WX11T ASM	2"deep towel bar-13w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114787S	BCK FRM 2" FIX 13WX12T ASM	2"deep towel bar-13w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114788S	BCK FRM 2" FIX 13WX13T ASM	2"deep towel bar-13w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114789S	BCK FRM 2" FIX 13WX14T ASM	2"deep towel bar-13w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114790S	BCK FRM 2" FIX 13WX15T ASM	2"deep towel bar-13w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114791S	BCK FRM 2" FIX 13WX16T ASM	2"deep towel bar-13w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114792S	BCK FRM 2" FIX 14WX8T ASM	2"deep towel bar-14w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114793S	BCK FRM 2" FIX 14WX9T ASM	2"deep towel bar-14w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114794S	BCK FRM 2" FIX 14WX10T ASM	2"deep towel bar-14w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114795S	BCK FRM 2" FIX 14WX11T ASM	2"deep towel bar-14w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114796S	BCK FRM 2" FIX 14WX12T ASM	2"deep towel bar-14w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114797S	BCK FRM 2" FIX 14WX13T ASM	2"deep towel bar-14w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114798S	BCK FRM 2" FIX 14WX14T ASM	2"deep towel bar-14w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114799S	BCK FRM 2" FIX 14WX15T ASM	2"deep towel bar-14w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114800S	BCK FRM 2" FIX 14WX16T ASM	2"deep towel bar-14w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1,6,18,19A	114801S	BCK FRM 2" FIX 15WX8T ASM	2"deep towel bar-15w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114802S	BCK FRM 2" FIX 15WX9T ASM	2"deep towel bar-15w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114803S	BCK FRM 2" FIX 15WX10T ASM	2"deep towel bar-15w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114804S	BCK FRM 2" FIX 15WX11T ASM	2"deep towel bar-15w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114805S	BCK FRM 2" FIX 15WX12T ASM	2"deep towel bar-15w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114806S	BCK FRM 2" FIX 15WX13T ASM	2"deep towel bar-15w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114807S	BCK FRM 2" FIX 15WX14T ASM	2"deep towel bar-15w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114808S	BCK FRM 2" FIX 15WX15T ASM	2"deep towel bar-15w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114809S	BCK FRM 2" FIX 15WX16T ASM	2"deep towel bar-15w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114810S	BCK FRM 2" FIX 16WX8T ASM	2"deep towel bar-16w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114811S	BCK FRM 2" FIX 16WX9T ASM	2"deep towel bar-16w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114812S	BCK FRM 2" FIX 16WX10T ASM	2"deep towel bar-16w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114813S	BCK FRM 2" FIX 16WX11T ASM	2"deep towel bar-16w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114814S	BCK FRM 2" FIX 16WX12T ASM	2"deep towel bar-16w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114815S	BCK FRM 2" FIX 16WX13T ASM	2"deep towel bar-16w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114816S	BCK FRM 2" FIX 16WX14T ASM	2"deep towel bar-16w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114817S	BCK FRM 2" FIX 16WX15T ASM	2"deep towel bar-16w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114818S	BCK FRM 2" FIX 16WX16T ASM	2"deep towel bar-16w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114819S	BCK FRM 2" FIX 17WX8T ASM	2"deep towel bar-17w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114820S	BCK FRM 2" FIX 17WX9T ASM	2"deep towel bar-17w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114821S	BCK FRM 2" FIX 17WX10T ASM	2"deep towel bar-17w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114822S	BCK FRM 2" FIX 17WX11T ASM	2"deep towel bar-17w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114823S	BCK FRM 2" FIX 17WX12T ASM	2"deep towel bar-17w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114824S	BCK FRM 2" FIX 17WX13T ASM	2"deep towel bar-17w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH


Pos.	Item Number	Description	Remarks
1,6,18,19A	114825S	BCK FRM 2" FIX 17WX14T ASM	2"deep towel bar-17w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114826S	BCK FRM 2" FIX 17WX15T ASM	2"deep towel bar-17w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114827S	BCK FRM 2" FIX 17WX16T ASM	2"deep towel bar-17w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114828S	BCK FRM 2" FIX 18WX8T ASM	2"deep towel bar-18w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114829S	BCK FRM 2" FIX 18WX9T ASM	2"deep towel bar-18w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114830S	BCK FRM 2" FIX 18WX10T ASM	2"deep towel bar-18w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114831S	BCK FRM 2" FIX 18WX11T ASM	2"deep towel bar-18w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114832S	BCK FRM 2" FIX 18WX12T ASM	2"deep towel bar-18w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114833S	BCK FRM 2" FIX 18WX13T ASM	2"deep towel bar-18w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114834S	BCK FRM 2" FIX 18WX14T ASM	2"deep towel bar-18w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114835S	BCK FRM 2" FIX 18WX15T ASM	2"deep towel bar-18w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114836S	BCK FRM 2" FIX 18WX16T ASM	2"deep towel bar-18w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114837S	BCK FRM 2" FIX 19WX8T ASM	2"deep towel bar-19w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114838S	BCK FRM 2" FIX 19WX9T ASM	2"deep towel bar-19w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114839S	BCK FRM 2" FIX 19WX10T ASM	2"deep towel bar-19w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,6,18,19A	114840S	BCK FRM 2" FIX 19WX11T ASM	2"deep towel bar-19w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114841S	BCK FRM 2" FIX 19WX12T ASM	2"deep towel bar-19w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114842S	BCK FRM 2" FIX 19WX13T ASM	2"deep towel bar-19w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114843S	BCK FRM 2" FIX 19WX14T ASM	2"deep towel bar-19w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114844S	BCK FRM 2" FIX 19WX15T ASM	2"deep towel bar-19w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114845S	BCK FRM 2" FIX 19WX16T ASM	2"deep towel bar-19w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,6,18,19A	114846S	BCK FRM 2" FIX 20WX8T ASM	2"deep towel bar-20w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,6,18,19A	114847S	BCK FRM 2" FIX 20WX9T ASM	2"deep towel bar-20w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,6,18,19A	114848S	BCK FRM 2" FIX 20WX10T ASM	2"deep towel bar-20w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH

Pos.	Item Number	Description	Remarks
1,6,18,19A	114849S	BCK FRM 2" FIX 20WX11T ASM	2"deep towel bar-20w- fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,6,18,19A	114850S	BCK FRM 2" FIX 20WX12T ASM	2"deep towel bar-20w- fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,6,18,19A	114851S	BCK FRM 2" FIX 20WX13T ASM	2"deep towel bar-20w- fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,6,18,19A	114852S	BCK FRM 2" FIX 20WX14T ASM	2"deep towel bar-20w- fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,6,18,19A	114853S	BCK FRM 2" FIX 20WX15T ASM	2"deep towel bar-20w- fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,6,18,19A	114854S	BCK FRM 2" FIX 20WX16T ASM	2"deep towel bar-20w- fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH

Note: the Q7 has 8 degrees rearward to 27 degrees forward adjustment in 5 degree increments.

Note: when moving backrest rearward with a Depth Adjustable Back item 21 will be used as a bushing for item 12.

Note: if replacing Backrest Bracket parts on chairs built before S/N R4-018772, complete Backrest Bracket Kit is required; order p/n 114874 N-FOLD ANGL ADJ NST BACK KIT (new Back Plate Kit is required as Back Plate and mounting hardware have changed).

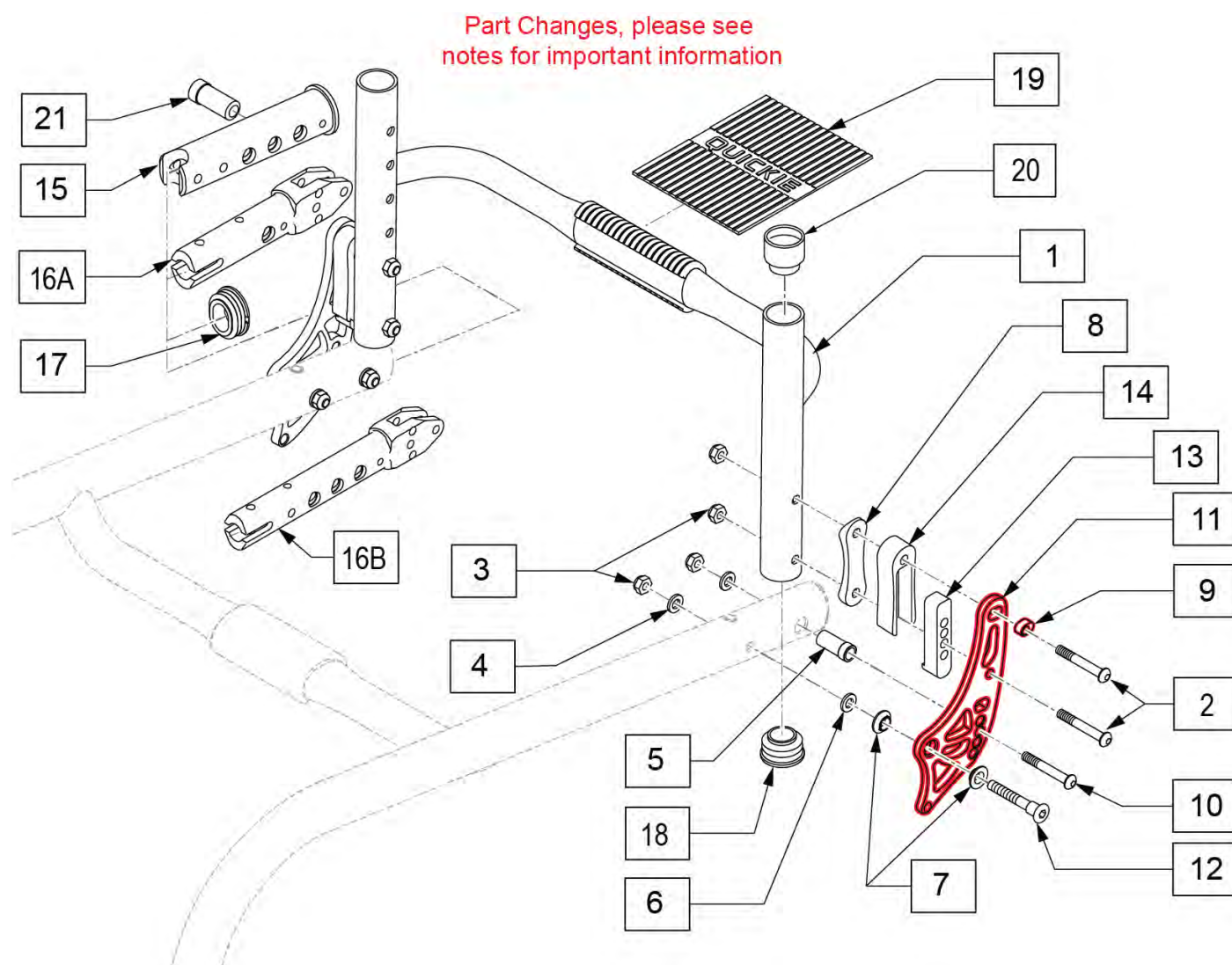


Pos.	Item Number	Description	Remarks
2	058901-004	BHCS,M6X1X55,PART,PL	
3	104381	NUT NYLOCK M6 X 1	
4	250104	WASHER FLAT M6	
5	109730	PIN GUIDE BACKREST FIXED	Fixed back only, not compatible with Folding Back
6	010963	SPRING BELLEVILLE 8MM	special concave washer

Pos.	Item Number	Description	Remarks
7	109198	BACK BRACKET BUSHING	
8	110160	BACKREST BRACKET SADDLE	
9	109197	ADJUSTER BACKREST INSERT	
10	102262-007	BHCS M6X1X45,FULL,PL	
11	159634S	BRKT W/HDWR KIT FLDNG ANO	Pair: Brackets&Hdwr
12	109204	BACKREST BOLT	
13	112321	BACKREST INSET SPACER BOTTOM	
14	112320	BACKREST INSET SPACER TOP	
15	126833	FRAME INSERT OVAL DAB	
16A	126832	ARM REST, PADDED TUBULAR MT	Used w/o Depth Adj Back
16B	126831	ARMREST, PADDED TUBULAR MT DAB	Used w/ Depth Adj Back
17	110161	FRAME INSERT, REAR SEAT TUB	
18	560735	7/8" ROUND TUBE PLUG	
19	112311	GRIP TOWEL BAR QUICKIE	
20	561082	1" TUBE END BUSHING THIN	
21	109730	PIN GUIDE BACKREST FIXED	See Note
2-14	114874	N-FOLD ANGL ADJ NST BACK KIT	See note; Pair Brackets&Hdwr
2-14,18A,18B	125783S	ANO INSET NON FLD BRKTS Q7 PR	Pair, Anodized colors
2-4,6-12	160933S	BACK BRACKET KIT INSET ANO	
1,18-20	114615S	BCK FRM 4" ADJ STD 12W ASM	4"deep towel bar-12w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114616S	BCK FRM 4" ADJ STD 13W ASM	4"deep towel bar-13w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114617S	BCK FRM 4" ADJ STD 14W ASM	4"deep towel bar-14w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114618S	BCK FRM 4" ADJ STD 15W ASM	4"deep towel bar-15w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114619S	BCK FRM 4" ADJ STD 16W ASM	4"deep towel bar-16w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114620S	BCK FRM 4" ADJ STD 17W ASM	4"deep towel bar-17w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114621S	BCK FRM 4" ADJ STD 18W ASM	4"deep towel bar-18w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114622S	BCK FRM 4" ADJ STD 19W ASM	4"deep towel bar-19w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114626S	BCK FRM 4" ADJ TAL 12W ASM	4"deep towel bar-12w-fits 16"-20" back hts
1,18-20	114627S	BCK FRM 4" ADJ TAL 13W ASM	4"deep towel bar-13w-fits 16"-20" back hts
1,18-20	114628S	BCK FRM 4" ADJ TAL 14W ASM	4"deep towel bar-14w-fits 16"-20" back hts
1,18-20	114629S	BCK FRM 4" ADJ TAL 15W ASM	4"deep towel bar-15w-fits 16"-20" back hts
1,18-20	114630S	BCK FRM 4" ADJ TAL 16W ASM	4"deep towel bar-16w-fits 16"-20" back hts
1,18-20	114631S	BCK FRM 4" ADJ TAL 17W ASM	4"deep towel bar-17w-fits 16"-20" back hts
1,18-20	114632S	BCK FRM 4" ADJ TAL 18W ASM	4"deep towel bar-18w-fits 16"-20" back hts
1,18-20	114633S	BCK FRM 4" ADJ TAL 19W ASM	4"deep towel bar-19w-fits 16"-20" back hts
1,18-20	114637S	BCK FRM 2" ADJ STD 12W ASM	2"deep towel bar-13w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114638S	BCK FRM 2" ADJ STD 13W ASM	2"deep towel bar-14w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114639S	BCK FRM 2" ADJ STD 14W ASM	2"deep towel bar-15w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114640S	BCK FRM 2" ADJ STD 15W ASM	2"deep towel bar-16w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114641S	BCK FRM 2" ADJ STD 16W ASM	2"deep towel bar-17w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114642S	BCK FRM 2" ADJ STD 17W ASM	2"deep towel bar-18w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114643S	BCK FRM 2" ADJ STD 18W ASM	2"deep towel bar-19w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114644S	BCK FRM 2" ADJ STD 19W ASM	2"deep towel bar-19w-fits 10"-14",12"-16"&14"-18" back hts
1,18-20	114648S	BCK FRM 2" ADJ TAL 12W ASM	2"deep towel bar-13w-fits 16"-20" back hts
1,18-20	114649S	BCK FRM 2" ADJ TAL 13W ASM	2"deep towel bar-14w-fits 16"-20" back hts
1,18-20	114650S	BCK FRM 2" ADJ TAL 14W ASM	2"deep towel bar-15w-fits 16"-20" back hts
1,18-20	114651S	BCK FRM 2" ADJ TAL 15W ASM	2"deep towel bar-16w-fits 16"-20" back hts
1,18-20	114652S	BCK FRM 2" ADJ TAL 16W ASM	2"deep towel bar-17w-fits 16"-20" back hts
1,18-20	114653S	BCK FRM 2" ADJ TAL 17W ASM	2"deep towel bar-18w-fits 16"-20" back hts
1,18-20	114654S	BCK FRM 2" ADJ TAL 18W ASM	2"deep towel bar-19w-fits 16"-20" back hts
1,18-20	114655S	BCK FRM 2" ADJ TAL 19W ASM	2"deep towel bar-19w-fits 16"-20" back hts

Note: when moving backrest rearward with a Depth Adjustable Back item 21 will be used as a bushing for item 12.

Note: Backrest Handle (Ht Adj) not available on Fixed Height Backrest style.

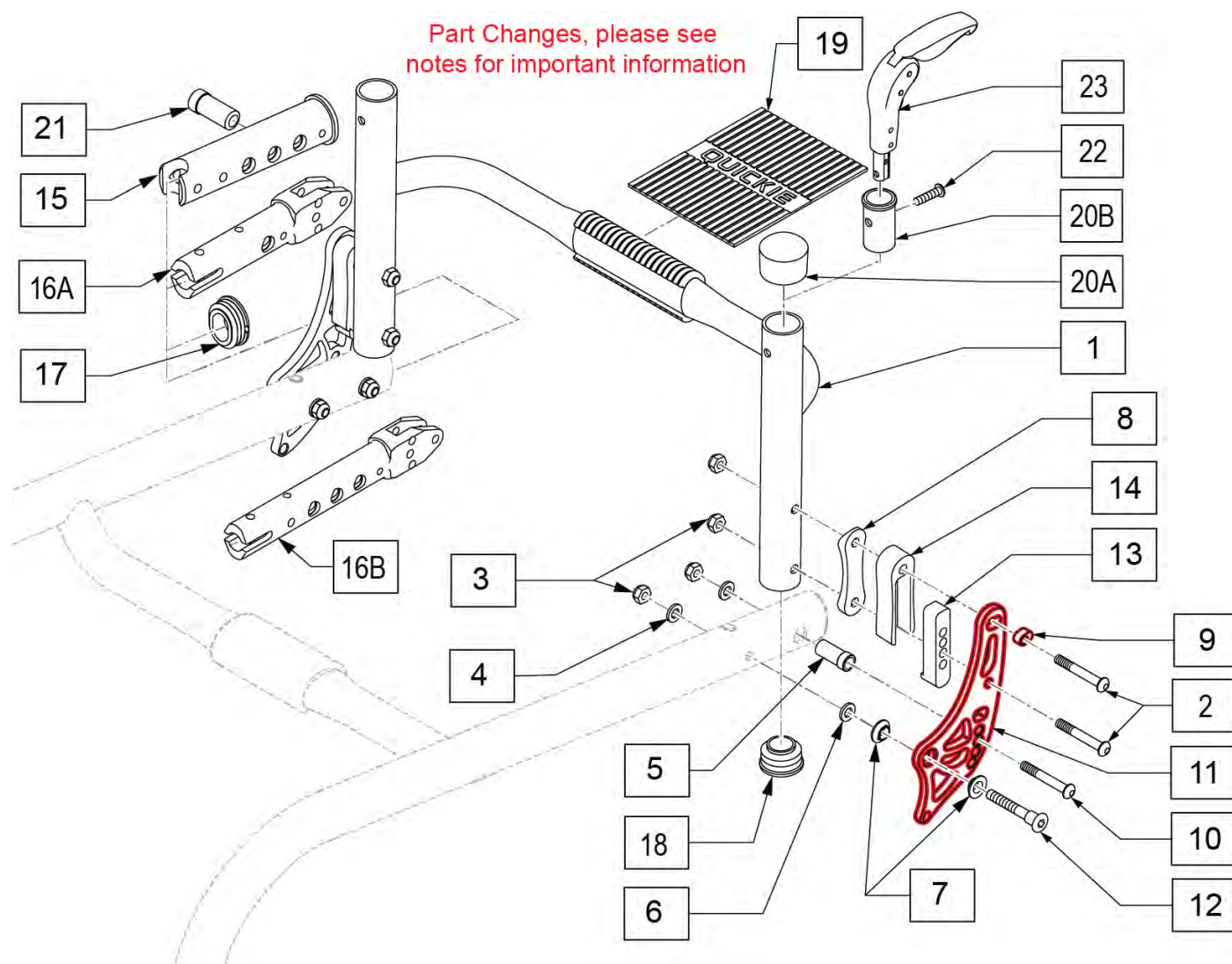


Pos.	Item Number	Description	Remarks
2	058901-003	BHCS,M6X1X50,PART,PL	


Pos.	Item Number	Description	Remarks
3	104381	NUT NYLOCK M6 X 1	
4	250104	WASHER FLAT M6	
5	109730	PIN GUIDE BACKREST FIXED	Fixed back only, not compatible with Folding Back
6	010963	SPRING BELLEVILLE 8MM	special concave washer
7	109198	BACK BRACKET BUSHING	
8	110160	BACKREST BRACKET SADDLE	
9	109197	ADJUSTER BACKREST INSERT	
10	102262-007	BHCS M6X1X45,FULL,PL	
11	159634S	BRKT W/HDWR KIT FLDNG ANO	Pair: Brackets&Hdwr
12	109204	BACKREST BOLT	
13	112321	BACKREST INSET SPACER BOTTOM	
14	112320	BACKREST INSET SPACER TOP	
15	126833	FRAME INSERT OVAL DAB	
16A	126832	ARM REST, PADDED TUBULAR MT	Used w/o Depth Adj Back
16B	126831	ARMREST, PADDED TUBULAR MT DAB	Used w/ Depth Adj Back
17	110161	FRAME INSERT, REAR SEAT TUB	
18	561062	1" INSERT PLUG .065 RIBS	
19	112311	GRIP TOWEL BAR QUICKIE	
20A	561014	1" MED CAP	Used w/ Omit Push Handle
20B	520148	ADAPTER, FLDG PUSH HANDLE	Used w/ Fold Down Push Handle option
21	109730	PIN GUIDE BACKREST FIXED	See Note
22	102262-004	BHCS, M6 X 1 X 25, FULL PL	Used w/ Fold Down Push Handle option
23	882604	FLDG PUSH HNDL W/INSRT FXD HGT	Each
20B,22,23	900164	FLDG PUSH HNDL ASSM FXD HGT	Each
2-14	114874	N-FOLD ANGL ADJ NST BACK KIT	See note; Pair Brackets&Hdwr
2-14,18	125783S	ANO INSET NON FLD BRKTS Q7 PR	Pair, Anodized colors
2-4,6-12	160933S	BACK BRACKET KIT INSET ANO	
1,18,19,20A	114675S	BCK FRM 4" FIX 12WX8T ASM	4"deep towel bar-13w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,18,19,20A	114676S	BCK FRM 4" FIX 12WX9T ASM	4"deep towel bar-13w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,18,19,20A	114677S	BCK FRM 4" FIX 12WX10T ASM	4"deep towel bar-13w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,18,19,20A	114678S	BCK FRM 4" FIX 12WX11T ASM	4"deep towel bar-13w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,18,19,20A	114679S	BCK FRM 4" FIX 12WX12T ASM	4"deep towel bar-13w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,18,19,20A	114680S	BCK FRM 4" FIX 12WX13T ASM	4"deep towel bar-13w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,18,19,20A	114681S	BCK FRM 4" FIX 12WX14T ASM	4"deep towel bar-13w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,18,19,20A	114682S	BCK FRM 4" FIX 12WX15T ASM	4"deep towel bar-13w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,18,19,20A	114683S	BCK FRM 4" FIX 12WX16T ASM	4"deep towel bar-13w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,18,19,20A	114684S	BCK FRM 4" FIX 13WX8T ASM	4"deep towel bar-14w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,18,19,20A	114685S	BCK FRM 4" FIX 13WX9T ASM	4"deep towel bar-14w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,18,19,20A	114686S	BCK FRM 4" FIX 13WX10T ASM	4"deep towel bar-14w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,18,19,20A	114687S	BCK FRM 4" FIX 13WX11T ASM	4"deep towel bar-14w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,18,19,20A	114688S	BCK FRM 4" FIX 13WX12T ASM	4"deep towel bar-14w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,18,19,20A	114689S	BCK FRM 4" FIX 13WX13T ASM	4"deep towel bar-14w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,18,19,20A	114690S	BCK FRM 4" FIX 13WX14T ASM	4"deep towel bar-14w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,18,19,20A	114691S	BCK FRM 4" FIX 13WX15T ASM	4"deep towel bar-14w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,18,19,20A	114692S	BCK FRM 4" FIX 13WX16T ASM	4"deep towel bar-14w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,18,19,20A	114693S	BCK FRM 4" FIX 14WX8T ASM	4"deep towel bar-15w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,18,19,20A	114694S	BCK FRM 4" FIX 14WX9T ASM	4"deep towel bar-15w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,18,19,20A	114695S	BCK FRM 4" FIX 14WX10T ASM	4"deep towel bar-15w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH
1,18,19,20A	114696S	BCK FRM 4" FIX 14WX11T ASM	4"deep towel bar-15w-fits 11" fix ht Omit PH & 14" fix ht w/ Fold Down PH
1,18,19,20A	114697S	BCK FRM 4" FIX 14WX12T ASM	4"deep towel bar-15w-fits 12" fix ht Omit PH & 15" fix ht w/ Fold Down PH
1,18,19,20A	114698S	BCK FRM 4" FIX 14WX13T ASM	4"deep towel bar-15w-fits 13" fix ht Omit PH & 16" fix ht w/ Fold Down PH
1,18,19,20A	114699S	BCK FRM 4" FIX 14WX14T ASM	4"deep towel bar-15w-fits 14" fix ht Omit PH & 17" fix ht w/ Fold Down PH
1,18,19,20A	114700S	BCK FRM 4" FIX 14WX15T ASM	4"deep towel bar-15w-fits 15" fix ht Omit PH & 18" fix ht w/ Fold Down PH
1,18,19,20A	114701S	BCK FRM 4" FIX 14WX16T ASM	4"deep towel bar-15w-fits 16" fix ht Omit PH & 19" fix ht w/ Fold Down PH
1,18,19,20A	114702S	BCK FRM 4" FIX 15WX8T ASM	4"deep towel bar-16w-fits 8" fix ht Omit PH & 11" fix ht w/ Fold Down PH
1,18,19,20A	114703S	BCK FRM 4" FIX 15WX9T ASM	4"deep towel bar-16w-fits 9" fix ht Omit PH & 12" fix ht w/ Fold Down PH
1,18,19,20A	114704S	BCK FRM 4" FIX 15WX10T ASM	4"deep towel bar-16w-fits 10" fix ht Omit PH & 13" fix ht w/ Fold Down PH

QUICKIE Q7

QUICKIE Q7

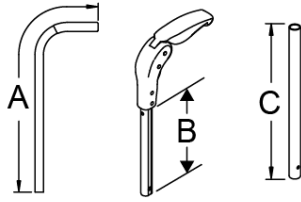MEASURING GUIDE

BACK TUBE	"A"	"B"	"C"
LOW	9"	—	7 1/4"
MED	11"	6 1/2"	9 1/4"
TALL	13"	8 1/2"	11 1/4"

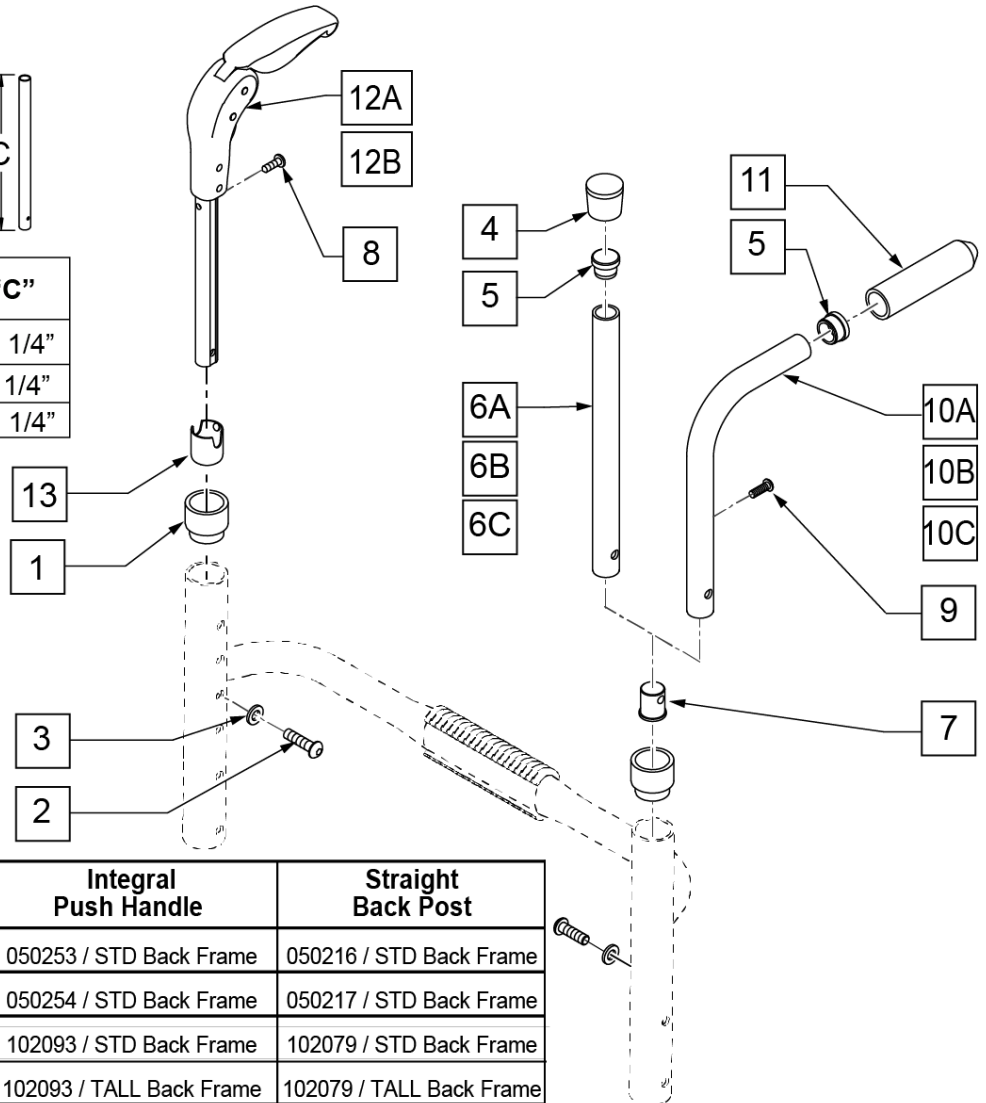

		Folding Push Handle	Integral Push Handle	Straight Back Post
Back Height	10"-14"	N/A	050253 / STD Back Frame	050216 / STD Back Frame
	12"-16"	882605 / STD Back Frame	050254 / STD Back Frame	050217 / STD Back Frame
	14"-18"	882606 / STD Back Frame	102093 / STD Back Frame	102079 / STD Back Frame
	16"-20"	882606 / TALL Back Frame	102093 / TALL Back Frame	102079 / TALL Back Frame


Pos.	Item Number	Description	Remarks
1	561082	1" TUBE END BUSHING THIN	
2	102262-003	BHCS M6 X 1 X 20 INORGANIC	
3	262130	M6 WASHER WAVE	
4	561010	3/4" CAP	
5	561000-QU	3/4" INSERT PLUG .083	
7	050019	PLUG BACKREST M6	
8	281127	#12 x 3/4" PAN HD PHIL S/ TAP	
9	107384	PAN HD PHIL 10/32 x 5/8 ROHS	
11	920026	QUIKIE GRIP 3/4" W/ INSTR SH	
12A	882605	FOLDING PUSH HANDLE W/ INSERT 11 -14"	See Table
12B	882606	FOLDING PUSH HANDLE W/ INSERT 12 -16"	See Table
13	104369	BUSHING BACKREST FOLDING HANDLE	
2-5,6A,7	050216	BCK ADJ TUBE RIGID STD 10"-14"	See Table
2-5,6B,7	050217	BCK ADJ TB STD 12-16/TL 14-18	See Table
2-5,6C,7	102079	BCK ADJ TUBE RIGID TL 16	See Table -16"-20"
2,3,5,7,10A,11	050253	INTEGRAL PUSH HANDLE STANDARD	See Table
2,3,5,7,10B,11	050254	INTEGRAL PUSH HANDLE MEDIUM	See Table
2,3,5,7,10C,11	102093	INTEGRAL PUSH HANDLE TALL	See Table

Pos.	Item Number	Description	Remarks
1-3,7	159564	INTGRL PH HDWR LOWER	Integral & Omit Push Handle - 1 side
1-3,13	159563	FOLDING PH HDWR LOWER	Ht Adj Push Handle only - 1 side


Note: on 8-10" wide chairs, only one Push Handle is used.

Note: After S/N RZ-010376 (Zone) and R4-025925 (Q7) the Push Handle changed from tubular to double D extrusion. For Telescoping Push Handles prior to these S/Ns replace with proper DD PH RPLCMENT KIT.

Part Changes, please see notes for important information

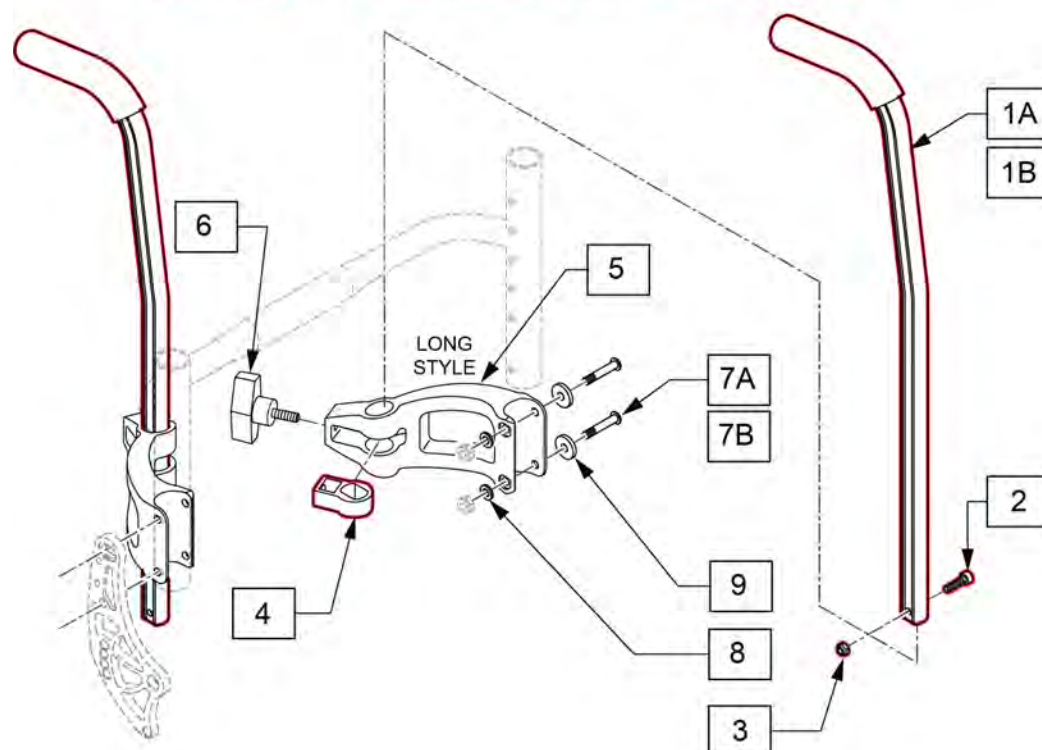

MEASURING GUIDE

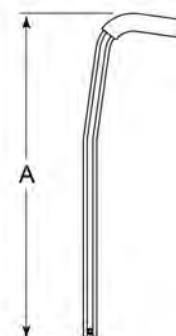


Description	"A"
STD	18"
TALL	24"

Pos.	Item Number	Description	Remarks
1-3	129370	ASSY HT ADJ PH DD TALL	For Standard PH Height
1-3	129371	ASST HT ADJ PH DD X-TALL	For Tall PH Height
1-4	145122	DD PH RPLCMENT KIT STD PAIR	Std Double D upgrade for existing Bolt On PH
1-4	145123	DD PH RPLCMENT KIT TALL PAIR	Tall Double D upgrade for existing Bolt On PH
1-4	145125	DD PH RPLCMENT KIT STD SINGLE	Std Double D upgrade for existing Bolt On PH, 8-11 WD
1-4	145124	DD PH RPLCMENT KIT TALL SINGLE	Tall Double D upgrade for existing Bolt On PH, 8-11 WD
2	102260-001	SHCS M5X0.8X18 FULL NP	
3	220121	NUT HEX NYLOCK M5 BLACK	
4	129349	INSERT FOR DD TUBE	
5	117074	HGT ADJ PH REC	
6	113245	1/4"-20 T-HANDLE	
7A	113320	BHCS,M6X1X40,PART,PL	Used w/ Std Back Rests
7B	058901-004	BHCS,M6X1X55,PART,PL	Used w/ Inset backrests
8	262130	6mm WAVE WASHER	
9	561026	3/4" WASHER 1/4" HOLE	Zippie Zone only
		Complete Kits	
1A,2-8	145126	BOLT ON PUSH HNDL DD STD PR	Pair; 2 Push Handles-Double D-Std Q7
1B,2-8	145127	BOLT ON PUSH HNDL DD TALL PR	Pair; 2 Push Handles-Double D-Tall Q7

Pos.	Item Number	Description	Remarks
1A,2-9	145128	BLT ON PSH DD PED STD 12-16W PR	Pair; 2 Push Handles-Double D-Std Zone 12-16 wide
1B,2-9	145129	BLT ON PSH DD PED TAL 12-16W PR	Pair; 2 Push Handles-Double D-Tall Zone 12-16 wide
1A,2-9	145130	BOLT ON PUSH DD PED STD 8-11	1 Push Handle-Double D Std Zone 8-11 wide
1B,2-9	145131	BOLT ON PUSH DD PED TALL 8-11	1 Push Handle-Double D-Tall Zone 8-11 wide

NOTE: FOR CHAIRS BUILT PRIOR TO JANUARY 2005 BOTH PUSH HANDLES NEED TO BE REPLACED DUE TO DESIGN CHANGE.

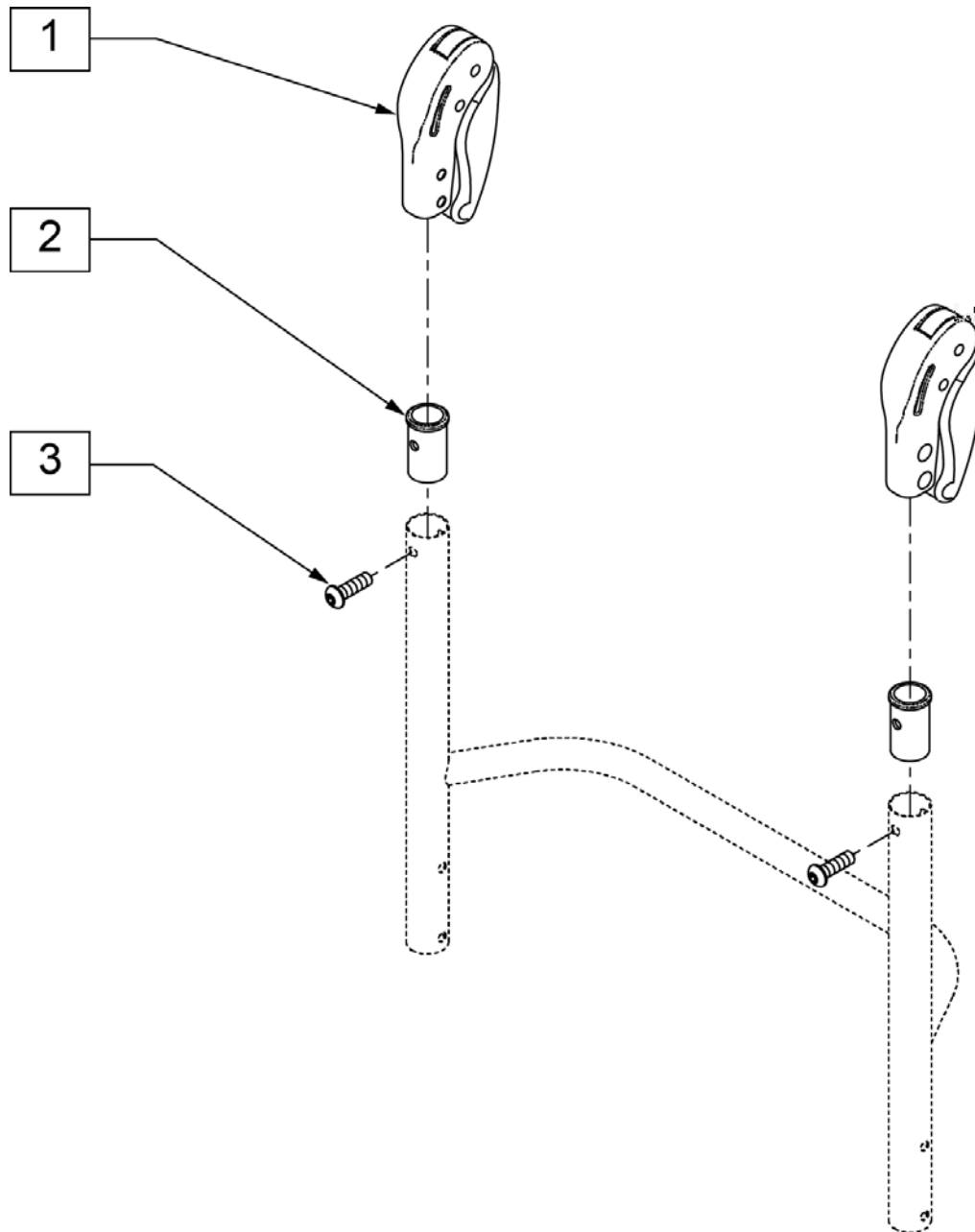


Pos.	Item Number	Description
1	882604	FOLDING PUSH HANDLE W/ INSERT FIXED HEIGHT
2	520148	ADAPTER FOLDING PUSH HANDLE
3	230116	BHCS M6 x 1 x 30mm W/ PL
1-3	900164	FOLDING PUSH HANDLE W/ INSERT FIXED HEIGHT


NOTE: the Q7 Active Rigid Backframe and the 7RS Welded Backframe are not available as replacement parts because these Back Frames are welded to the Frame. Please contact Customer Service for replacement information.

Note: on the Omit Push Handle Option, the Back Frame tube is cut to the fixed height chosen on the chair and then the 1" cap is installed on the top of the frame tubes.


FOLDING PUSH HANDLE OPTION

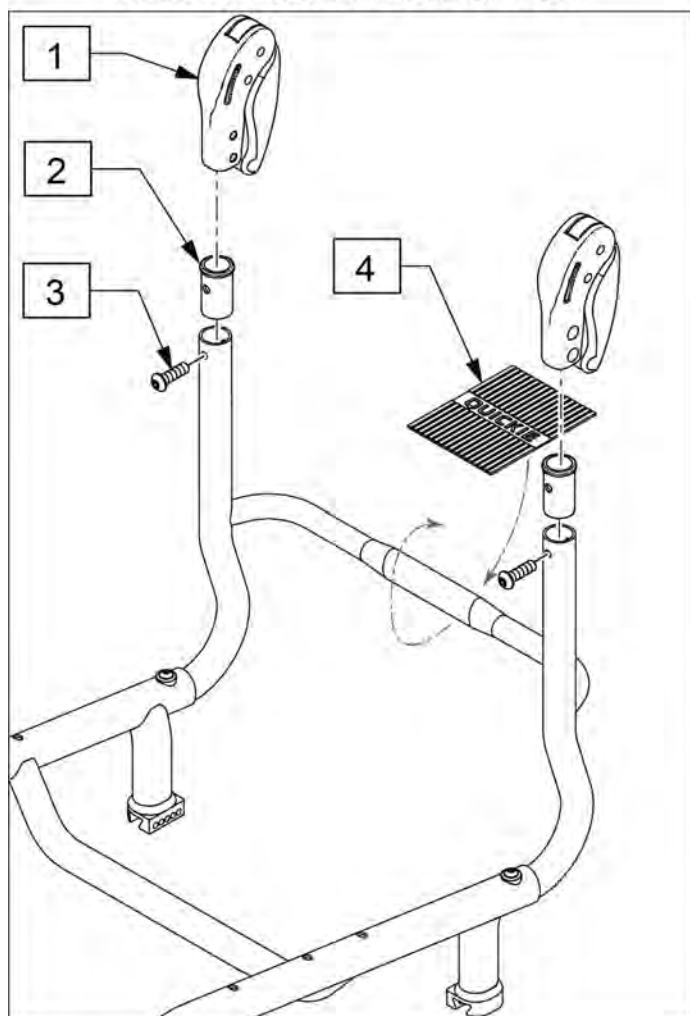

OMIT PUSH HANDLE OPTION

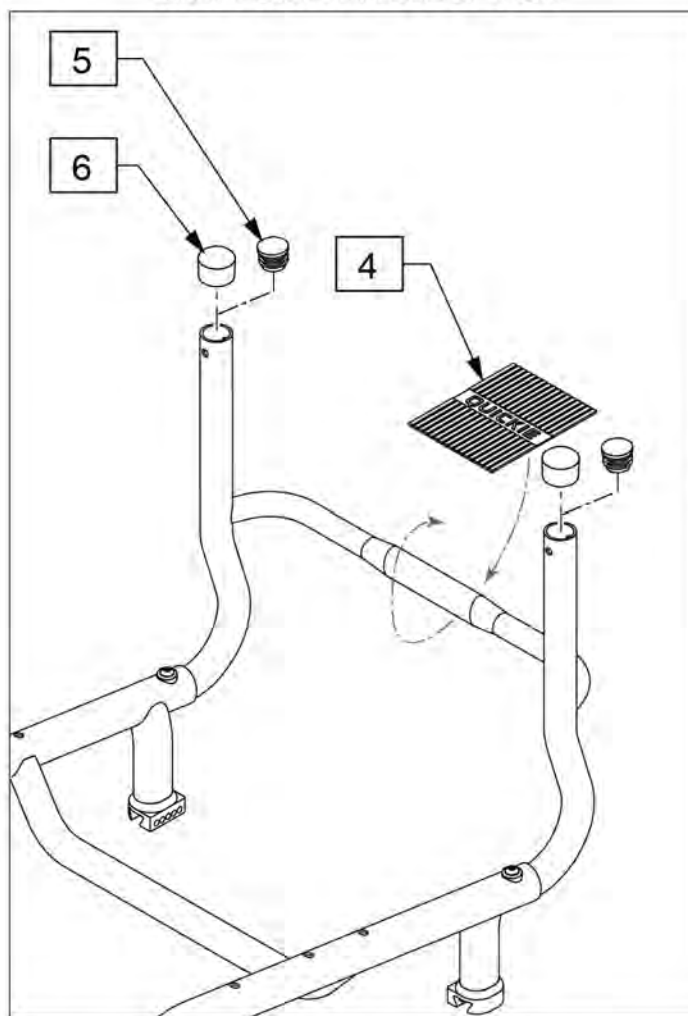


Pos.	Item Number	Description	Remarks
1	882604	FOLDING PUSH HANDLE W/ INSERT FIXED HEIGHT	
2	520148	ADAPTER FOLDING PUSH HANDLE	
3	230116	BHCS M6 x 1 x 30mm W/ PL	
4	112311	GRIP TOWEL BAR QUICKIE	
5	561002	1" INSERT PLUG .083	
6	561014	1" MED CAP	
1-3	900164	FOLDING PUSH HANDLE W/ INSERT FIXED HEIGHT	includes one Push Handle only

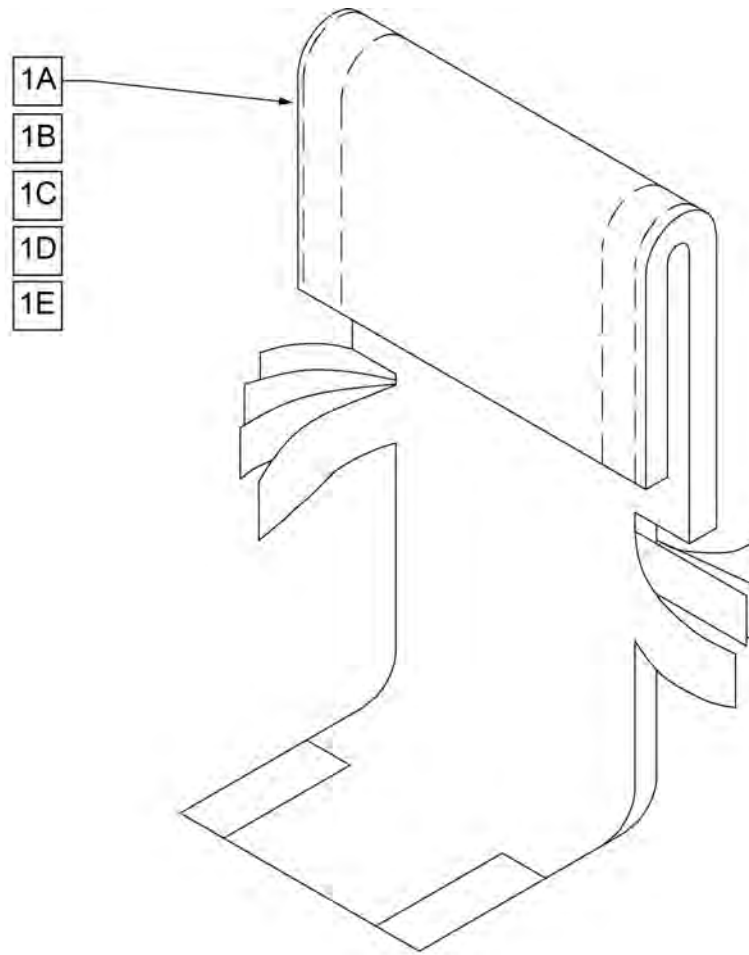

Pos.	Item Number	Description	Remarks
1A	680212	BACK UPH 12" 8-12" W/TAIL	(12" WIDE) 8-12" for Ht Adj Back & 8,9,10" for Fixed Back Hts
1A	680213	BACK UPH 13" 8-12" W/TAIL	(13" WIDE) 8-12" for Ht Adj Back & 8,9,10" for Fixed Back Hts
1A	680308	BACK UPH 14" 8-12" W/TAIL	(14" WIDE) 8-12" for Ht Adj Back & 8,9,10" for Fixed Back Hts
1A	680408	BACK UPH 15" 8-12" W/TAIL	(15" WIDE) 8-12" for Ht Adj Back & 8,9,10" for Fixed Back Hts
1A	680508	BACK UPH 16" 8-12" W/TAIL	(16" WIDE) 8-12" for Ht Adj Back & 8,9,10" for Fixed Back Hts
1A	680608	BACK UPH 17" 8-12" W/TAIL	(17" WIDE) 8-12" for Ht Adj Back & 8,9,10" for Fixed Back Hts
1A	680708	BACK UPH 18" 8-12" W/TAIL	(18" WIDE) 8-12" for Ht Adj Back & 8,9,10" for Fixed Back Hts
1A	680808	BACK UPH 19" 8-12" W/TAIL	(19" WIDE) 8-12" for Ht Adj Back & 8,9,10" for Fixed Back Hts
1B	680208	BACK UPH 12" 11-15" W/ TAIL	(12" WIDE) 10-14" for Ht Adj Back & 11" for Fixed Back Hts
1B	680201	BACK UPH 13" 11-15" W/ TAIL	(13" WIDE) 10-14" for Ht Adj Back & 11" for Fixed Back Hts
1B	680301	BACK UPH 14" 11-15" W/ TAIL	(14" WIDE) 10-14" for Ht Adj Back & 11" for Fixed Back Hts
1B	680401	BACK UPH 15" 11-15" W/ TAIL	(15" WIDE) 10-14" for Ht Adj Back & 11" for Fixed Back Hts
1B	680501	BACK UPH 16" 11-15" W/ TAIL	(16" WIDE) 10-14" for Ht Adj Back & 11" for Fixed Back Hts
1B	680601	BACK UPH 17" 11-15" W/ TAIL	(17" WIDE) 10-14" for Ht Adj Back & 11" for Fixed Back Hts
1B	680701	BACK UPH 18" 11-15" W/ TAIL	(18" WIDE) 10-14" for Ht Adj Back & 11" for Fixed Back Hts
1B	680801	BACK UPH 19" 11-15" W/ TAIL	(19" WIDE) 10-14" for Ht Adj Back & 11" for Fixed Back Hts
1B	680901	BACK UPH 20" 11-15" W/ TAIL	(20" WIDE) 10-14" for Ht Adj Back & 11" for Fixed Back Hts
1C	680209	BACK UPH 12" 12-16" W/ TAIL	(12" WIDE) 12 -16" for Ht Adj Back & 12"&13" for Fixed Back Hts
1C	680203	BACK UPH 13" 12-16" W/ TAIL	(13" WIDE) 12 -16" for Ht Adj Back & 12"&13" for Fixed Back Hts
1C	680303	BACK UPH 14" 12-16" W/ TAIL	(14" WIDE) 12 -16" for Ht Adj Back & 12"&13" for Fixed Back Hts
1C	680403	BACK UPH 15" 12-16" W/ TAIL	(15" WIDE) 12 -16" for Ht Adj Back & 12"&13" for Fixed Back Hts


Pos.	Item Number	Description	Remarks
1C	680503	BACK UPH 16" 12-16" W/ TAIL	(16" WIDE) 12 -16" for Ht Adj Back & 12"&13" for Fixed Back Hts
1C	680603	BACK UPH 17" 12-16" W/ TAIL	(17" WIDE) 12 -16" for Ht Adj Back & 12"&13" for Fixed Back Hts
1C	680703	BACK UPH 18" 12-16" W/ TAIL	(18" WIDE) 12 -16" for Ht Adj Back & 12"&13" for Fixed Back Hts
1C	680803	BACK UPH 19" 12-16" W/ TAIL	(19" WIDE) 12 -16" for Ht Adj Back & 12"&13" for Fixed Back Hts
1C	680903	BACK UPH 20" 12-16" W/ TAIL	(20" WIDE) 12 -16" for Ht Adj Back & 12"&13" for Fixed Back Hts
1D	680210	BACK UPH 12" 14-18" W/ TAIL	(12" WIDE) 14 -18" for Ht Adj Back & 14,15,16" for Fixed Back Hts
1D	680205	BACK UPH 13" 14-18" W/ TAIL	(13" WIDE) 14 -18" for Ht Adj Back & 14,15,16" for Fixed Back Hts
1D	680305	BACK UPH 14" 14-18" W/ TAIL	(14" WIDE) 14 -18" for Ht Adj Back & 14,15,16" for Fixed Back Hts
1D	680405	BACK UPH 15" 14-18" W/ TAIL	(15" WIDE) 14 -18" for Ht Adj Back & 14,15,16" for Fixed Back Hts
1D	680505	BACK UPH 16" 14-18" W/ TAIL	(16" WIDE) 14 -18" for Ht Adj Back & 14,15,16" for Fixed Back Hts
1D	680605	BACK UPH 17" 14-18" W/ TAIL	(17" WIDE) 14 -18" for Ht Adj Back & 14,15,16" for Fixed Back Hts
1D	680705	BACK UPH 18" 14-18" W/ TAIL	(18" WIDE) 14 -18" for Ht Adj Back & 14,15,16" for Fixed Back Hts
1D	680805	BACK UPH 19" 14-18" W/ TAIL	(19" WIDE) 14 -18" for Ht Adj Back & 14,15,16" for Fixed Back Hts
1D	680905	BACK UPH 20" 14-18" W/ TAIL	(20" WIDE) 14 -18" for Ht Adj Back & 14,15,16" for Fixed Back Hts
1E	680211	BACK UPH 12" 16-20" W/ TAIL	(12" WIDE) 16 -20" for Ht Adj Back & 17,18,19" for Fixed Back Hts
1E	680207	BACK UPH 13" 16-20" W/ TAIL	(13" WIDE) 16 -20" for Ht Adj Back & 17,18,19" for Fixed Back Hts
1E	680307	BACK UPH 14" 16-20" W/ TAIL	(14" WIDE) 16 -20" for Ht Adj Back & 17,18,19" for Fixed Back Hts
1E	680407	BACK UPH 15" 16-20" W/ TAIL	(15" WIDE) 16 -20" for Ht Adj Back & 17,18,19" for Fixed Back Hts
1E	680507	BACK UPH 16" 16-20" W/ TAIL	(16" WIDE) 16 -20" for Ht Adj Back & 17,18,19" for Fixed Back Hts
1E	680607	BACK UPH 17" 16-20" W/ TAIL	(17" WIDE) 16 -20" for Ht Adj Back & 17,18,19" for Fixed Back Hts
1E	680707	BACK UPH 18" 16-20" W/ TAIL	(18" WIDE) 16 -20" for Ht Adj Back & 17,18,19" for Fixed Back Hts
1E	680807	BACK UPH 19" 16-20" W/ TAIL	(19" WIDE) 16 -20" for Ht Adj Back & 17,18,19" for Fixed Back Hts
1E	680907S	BACK UPH CUSTOM Q1/GP F BCK	(20" WIDE) 16 -20" for Ht Adj Back & 17,18,19" for Fixed Back Hts

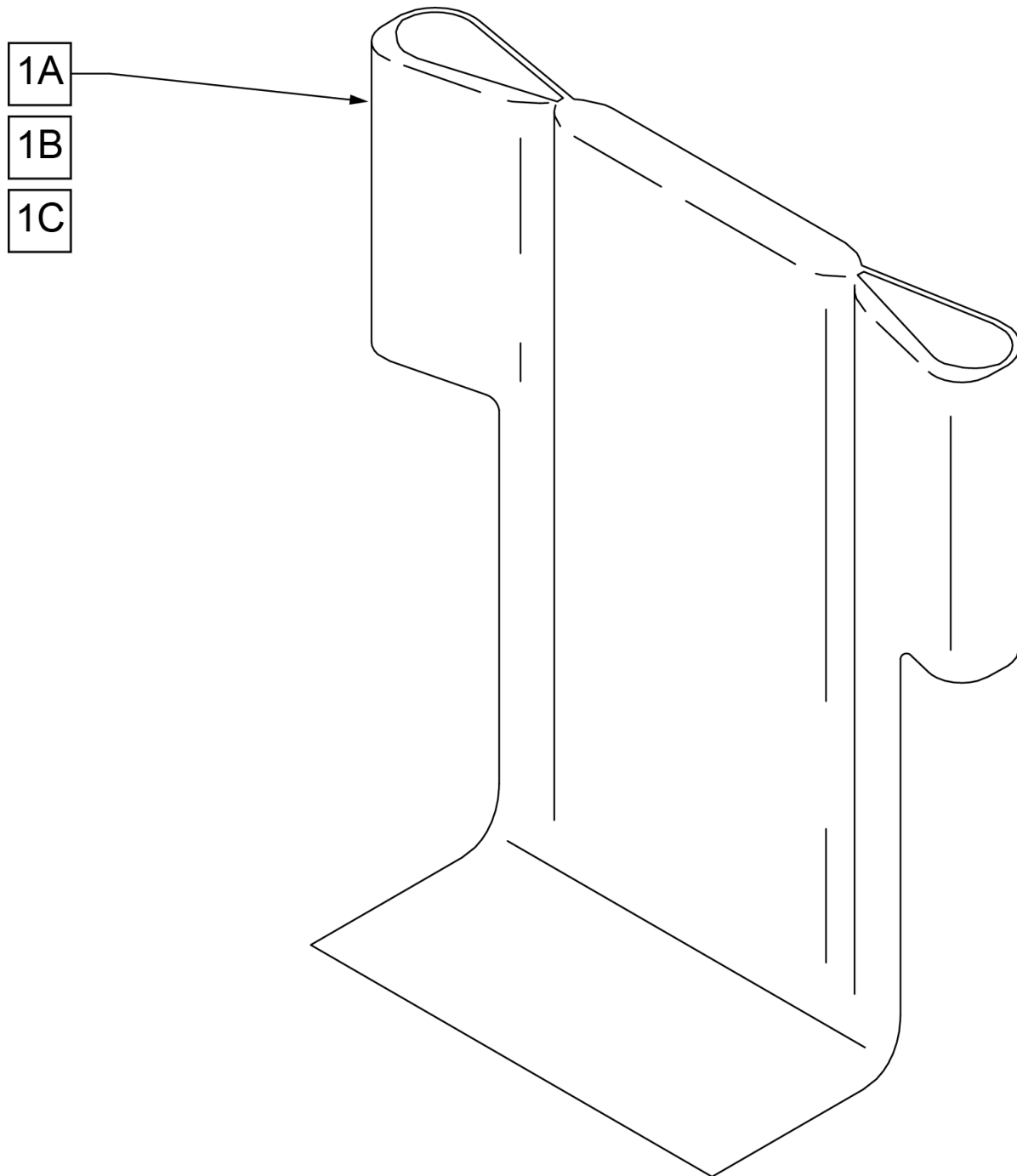

Pos.	Item Number	Description	Remarks
1A	680442	BACK UPH 12" 11-14/12-16 RDG	10 -14" & 12 -16" BACK HEIGHT
1A	680443	BACK UPH 13" 11-14/12-16 RDG	10 -14" & 12 -16" BACK HEIGHT
1A	680454	BACK UPH 14" 11-14/12-16 RDG	10 -14" & 12 -16" BACK HEIGHT
1A	680455	BACK UPH 15" 11-14/12-16 RDG	10 -14" & 12 -16" BACK HEIGHT
1A	680456	BACK UPH 16" 11-14/12-16 RDG	10 -14" & 12 -16" BACK HEIGHT
1A	680457	BACK UPH 17" 11-14/12-16 RDG	10 -14" & 12 -16" BACK HEIGHT
1A	680458	BACK UPH 18" 11-14/12-16 RDG	10 -14" & 12 -16" BACK HEIGHT
1A	680459	BACK UPH 19" 11-14/12-16 RDG	10 -14" & 12 -16" BACK HEIGHT
1A	680460	BACK UPH 20" 11-14/12-16 RDG	10 -14" & 12 -16" BACK HEIGHT

Pos.	Item Number	Description	Remarks
1B	680462	BACK UPH 12" 14-18 RDG	14 -18" BACK HEIGHT
1B	680463	BACK UPH 13" 14-18 RDG	14 -18" BACK HEIGHT
1B	680464	BACK UPH 14" 14-18 RDG	14 -18" BACK HEIGHT
1B	680465	BACK UPH 15" 14-18 RDG	14 -18" BACK HEIGHT
1B	680466	BACK UPH 16" 14-18 RDG	14 -18" BACK HEIGHT
1B	680467	BACK UPH 17" 14-18 RDG	14 -18" BACK HEIGHT
1B	680468	BACK UPH 18" 14-18 RDG	14 -18" BACK HEIGHT
1B	680469	BACK UPH 19" 14-18 RDG	14 -18" BACK HEIGHT
1B	680470	BACK UPH 20" 14-18 RDG	14 -18" BACK HEIGHT
1C	680472	BACK UPH 12" 16-20 RDG	16 -20" BACK HEIGHT
1C	680473	BACK UPH 13" 16-20 RDG	16 -20" BACK HEIGHT
1C	680474	BACK UPH 14" 16-20 RDG	16 -20" BACK HEIGHT
1C	680475	BACK UPH 15" 16-20 RDG	16 -20" BACK HEIGHT
1C	680476	BACK UPH 16" 16-20 RDG	16 -20" BACK HEIGHT
1C	680477	BACK UPH 17" 16-20 RDG	16 -20" BACK HEIGHT
1C	680478	BACK UPH 18" 16-20 RDG	16 -20" BACK HEIGHT
1C	680479	BACK UPH 19" 16-20 RDG	16 -20" BACK HEIGHT
1C	680480	BACK UPH 20" 16-20 RDG	16 -20" BACK HEIGHT

Note: Each back height uses a different variation of back straps; replacement strap kits contain more straps than needed to support all back heights.

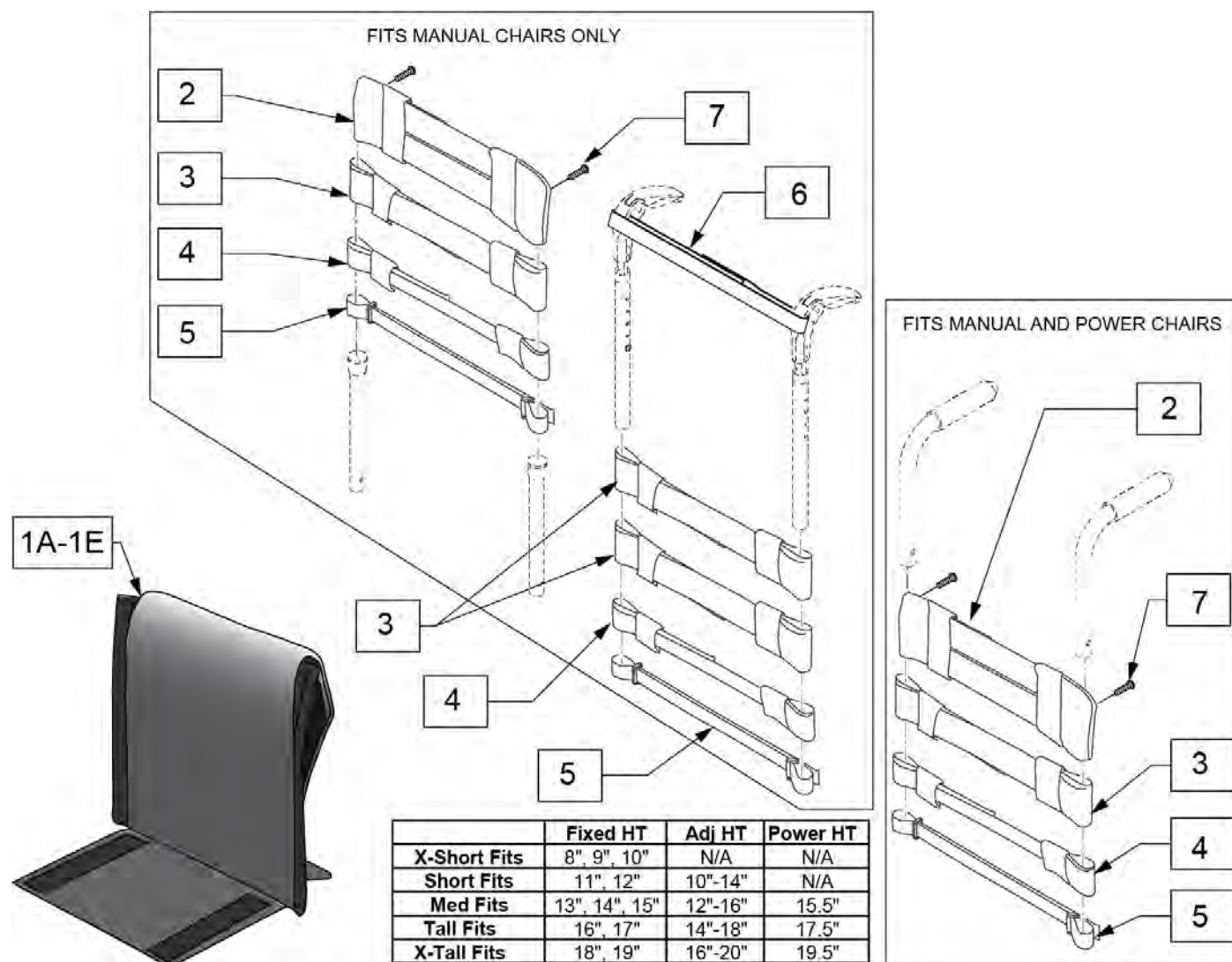


Pos.	Item Number	Description	Remarks
2	113712	TOP CAP, STD, 11 -12"	Used for Omit Push Handle & Integral Push Handle only
2	113713	TOP CAP, STD, 13 -14"	Used for Omit Push Handle & Integral Push Handle only
2	113714	TOP CAP, STD, 15 -16"	Used for Omit Push Handle & Integral Push Handle only
2	113715	TOP CAP, STD, 17 -18"	Used for Omit Push Handle & Integral Push Handle only
2	113716	TOP CAP, STD, 19 -20"	Used for Omit Push Handle & Integral Push Handle only
3	113727	STRAP PADDED 2", 11 -12"	
3	113728	STRAP PADDED 2", 13 -14"	
3	113729	STRAP PADDED 2", 15 -16"	
3	113730	STRAP PADDED 2", 17 -18"	
3	113731	STRAP PADDED 2", 19 -20"	
4	113732	STRAP PADDED 1", 11 -12"	
4	113733	STRAP PADDED 1", 13 -14"	
4	113734	STRAP PADDED 1", 15 -16"	
4	113735	STRAP PADDED 1", 16 -17"	

Pos.	Item Number	Description	Remarks
4	113736	STRAP PADDED 1", 18 -19"	
5	113717	POSITION STRAP 1", 11 -12	
5	113718	POSITION STRAP 1", 13 -14	
5	113719	POSITION STRAP 1", 15 -16	
5	113720	POSITION STRAP 1", 17 -18	
5	113721	POSITION STRAP 1", 19 -20	
6	052435-001	UPH,VNTD ADJ.,FPH STRP,12-13"W	Fold Down Push Handles only
6	052435-002	UPH,VNTD ADJ.,FPH STRP,14-15"W	Fold Down Push Handles only
6	052435-003	UPH,VNTD ADJ.,FPH STRP,16-17"W	Fold Down Push Handles only
6	052435-004	UPH,VNTD ADJ.,FPH STRP,18-20"W	Fold Down Push Handles only
7	107384	PAN HD PHIL 10/32 X 5/8 ROHS	
2-7	114879	ADJ UPH 11-12W STRAP KIT	
2-7	114880	ADJ UPH 13-14W STRAP KIT	
2-7	114881	ADJ UPH 15-16W STRAP KIT	
2-7	114882	ADJ UPH 17-18W STRAP KIT	
2-7	114883	ADJ UPH 19-20W STRAP KIT	
1A,2-7	113321-001	ADJ UP 12W X-SHT ASM	
1B,2-7	113321-002	ADJ UP 12W SHT ASM	
1C,2-7	113321-003	ADJ UP 12W MED ASM	
1D,2-7	113321-004	ADJ UP 12W TAL ASM	
1E,2-7	113321-005	ADJ UP 12W X-TAL ASM	
1A,2-7	113321-006	ADJ UP 13W X-SHT ASM	
1B,2-7	113321-007	ADJ UP 13W SHT ASM	
1C,2-7	113321-008	ADJ UP 13W MED ASM	
1D,2-7	113321-009	ADJ UP 13W TAL ASM	
1E,2-7	113321-010	ADJ UP 13W X-TAL ASM	
1A,2-7	113321-011	ADJ UP 14W X-SHT ASM	
1B,2-7	113321-012	ADJ UP 14W SHT ASM	
1C,2-7	113321-013	ADJ UP 14W MED ASM	
1D,2-7	113321-014	ADJ UP 14W TAL ASM	
1E,2-7	113321-015	ADJ UP 14W X-TAL ASM	
1A,2-7	113321-016	ADJ UP 15W X-SHT ASM	
1B,2-7	113321-017	ADJ UP 15W SHT ASM	
1C,2-7	113321-018	ADJ UP 15W MED ASM	
1D,2-7	113321-019	ADJ UP 15W TAL ASM	
1E,2-7	113321-020	ADJ UP 15W X-TAL ASM	
1A,2-7	113321-021	ADJ UP 16W X-SHT ASM	
1B,2-7	113321-022	ADJ UP 16W SHT ASM	
1C,2-7	113321-023	ADJ UP 16W MED ASM	
1D,2-7	113321-024	ADJ UP 16W TAL ASM	
1E,2-7	113321-025	ADJ UP 16W X-TAL ASM	
1A,2-7	113321-026	ADJ UP 17W X-SHT ASM	
1B,2-7	113321-027	ADJ UP 17W SHT ASM	
1C,2-7	113321-028	ADJ UP 17W MED ASM	
1D,2-7	113321-029	ADJ UP 17W TAL ASM	
1E,2-7	113321-030	ADJ UP 17W X-TAL ASM	
1A,2-7	113321-031	ADJ UP 18W X-SHT ASM	
1B,2-7	113321-032	ADJ UP 18W SHT ASM	
1C,2-7	113321-033	ADJ UP 18W MED ASM	
1D,2-7	113321-034	ADJ UP 18W TAL ASM	
1E,2-7	113321-035	ADJ UP 18W X-TAL ASM	
1A,2-7	113321-036	ADJ UP 19W X-SHT ASM	
1B,2-7	113321-037	ADJ UP 19W SHT ASM	
1C,2-7	113321-038	ADJ UP 19W MED ASM	
1D,2-7	113321-039	ADJ UP 19W TAL ASM	
1E,2-7	113321-040	ADJ UP 19W X-TAL ASM	
1A,2-7	113321-041	ADJ UP 20W X-SHT ASM	

Pos.	Item Number	Description	Remarks
1B,2-7	113321-042	ADJ UP 20W SHT ASM	
1C,2-7	113321-043	ADJ UP 20W MED ASM	
1D,2-7	113321-044	ADJ UP 20W TAL ASM	
1E,2-7	113321-045	ADJ UP 20W X-TAL ASM	

Note: Each back height uses a different variation of back straps; replacement strap kits contain more straps than needed to support all back heights.

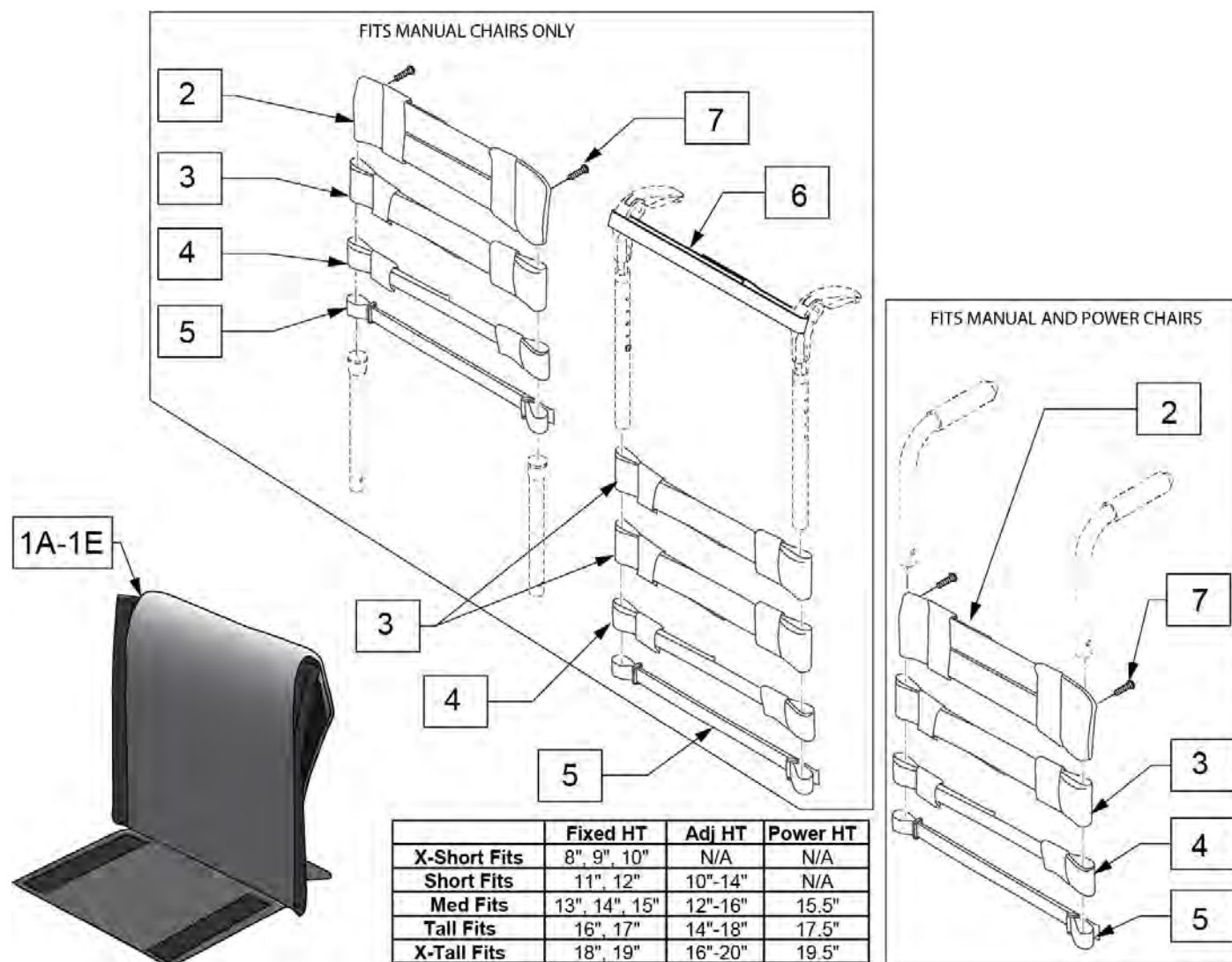


Pos.	Item Number	Description	Remarks
2	113712	TOP CAP, STD, 11 -12"	Used for Omit Push Handle & Integral Push Handle only
2	113713	TOP CAP, STD, 13 -14"	Used for Omit Push Handle & Integral Push Handle only
2	113714	TOP CAP, STD, 15 -16"	Used for Omit Push Handle & Integral Push Handle only
2	113715	TOP CAP, STD, 17 -18"	Used for Omit Push Handle & Integral Push Handle only
2	113716	TOP CAP, STD, 19 -20"	Used for Omit Push Handle & Integral Push Handle only
3	113727	STRAP PADDED 2", 11 -12"	
3	113728	STRAP PADDED 2", 13 -14"	
3	113729	STRAP PADDED 2", 15 -16"	
3	113730	STRAP PADDED 2", 17 -18"	
3	113731	STRAP PADDED 2", 19 -20"	
4	113732	STRAP PADDED 1", 11 -12"	
4	113733	STRAP PADDED 1", 13 -14"	
4	113734	STRAP PADDED 1", 15 -16"	
4	113735	STRAP PADDED 1", 16 -17"	

Pos.	Item Number	Description	Remarks
4	113736	STRAP PADDED 1", 18 -19"	
5	113717	POSITION STRAP 1", 11 -12	
5	113718	POSITION STRAP 1", 13 -14	
5	113719	POSITION STRAP 1", 15 -16	
5	113720	POSITION STRAP 1", 17 -18	
5	113721	POSITION STRAP 1", 19 -20	
6	052435-001	UPH,VNTD ADJ.,FPH STRP,12-13"W	Fold Down Push Handles only
6	052435-002	UPH,VNTD ADJ.,FPH STRP,14-15"W	Fold Down Push Handles only
6	052435-003	UPH,VNTD ADJ.,FPH STRP,16-17"W	Fold Down Push Handles only
6	052435-004	UPH,VNTD ADJ.,FPH STRP,18-20"W	Fold Down Push Handles only
7	107384	PAN HD PHIL 10/32 X 5/8 ROHS	
2-7	114879	ADJ UPH 11-12W STRAP KIT	
2-7	114880	ADJ UPH 13-14W STRAP KIT	
2-7	114881	ADJ UPH 15-16W STRAP KIT	
2-7	114882	ADJ UPH 17-18W STRAP KIT	
2-7	114883	ADJ UPH 19-20W STRAP KIT	
1A,2-7	113322-001	VNT ADJ UP 12W X-SHT ASM	
1B,2-7	113322-002	VNT ADJ UP 12W SHT ASM	
1C,2-7	113322-003	VNT ADJ UP 12W MED ASM	
1D,2-7	113322-004	VNT ADJ UP 12W TAL ASM	
1E,2-7	113322-005	VNT ADJ UP 12W X-TAL ASM	
1A,2-7	113322-006	VNT ADJ UP 13W X-SHT ASM	
1B,2-7	113322-007	VNT ADJ UP 13-W SHT ASM	
1C,2-7	113322-008	VNT ADJ UP 13W MED ASM	
1D,2-7	113322-009	VNT ADJ UP 13W TAL ASM	
1E,2-7	113322-010	VNT ADJ UP 13W X-TAL ASM	
1A,2-7	113322-011	VNT ADJ UP 14W X-SHT ASM	
1B,2-7	113322-012	VNT ADJ UP 14W SHT ASM	
1C,2-7	113322-013	VNT ADJ UP 14W MED ASM	
1D,2-7	113322-014	VNT ADJ UP 14W TAL ASM	
1E,2-7	113322-015	VNT ADJ UP 14W X-TAL ASM	
1A,2-7	113322-016	VNT ADJ UP 15W X-SHT ASM	
1B,2-7	113322-017	VNT ADJ UP 15W SHT ASM	
1C,2-7	113322-018	VNT ADJ UP 15W MED ASM	
1D,2-7	113322-019	VNT ADJ UP 15W TAL ASM	
1E,2-7	113322-020	VNT ADJ UP 15W X-TAL ASM	
1A,2-7	113322-021	VNT ADJ UP 16W X-SHT ASM	
1B,2-7	113322-022	VNT ADJ UP 16W SHT ASM	
1C,2-7	113322-023	VNT ADJ UP 16W MED ASM	
1D,2-7	113322-024	VNT ADJ UP 16W TAL ASM	
1E,2-7	113322-025	VNT ADJ UP 16W X-TAL ASM	
1A,2-7	113322-026	VNT ADJ UP 17W X-SHT ASM	
1B,2-7	113322-027	VNT ADJ UP 17W SHT ASM	
1C,2-7	113322-028	VNT ADJ UP 17W MED ASM	
1D,2-7	113322-029	VNT ADJ UP 17W TAL ASM	
1E,2-7	113322-030	VNT ADJ UP 17W X-TAL ASM	
1A,2-7	113322-031	VNT ADJ UP 18W X-SHT ASM	
1B,2-7	113322-032	VNT ADJ UP 18W SHT ASM	
1C,2-7	113322-033	VNT ADJ UP 18W MED ASM	
1D,2-7	113322-034	VNT ADJ UP 18W TAL ASM	
1E,2-7	113322-035	VNT ADJ UP 18W X-TAL ASM	
1A,2-7	113322-036	VNT ADJ UP 19W X-SHT ASM	
1B,2-7	113322-037	VNT ADJ UP 19W SHT ASM	
1C,2-7	113322-038	VNT ADJ UP 19W MED ASM	
1D,2-7	113322-039	VNT ADJ UP 19W TAL ASM	
1E,2-7	113322-040	VNT ADJ UP 19W X-TAL ASM	
1A,2-7	113322-041	VNT ADJ UP 20W X-SHT ASM	

Pos.	Item Number	Description	Remarks
1B,2-7	113322-042	VNT ADJ UP 20W SHT ASM	
1C,2-7	113322-043	VNT ADJ UP 20W MED ASM	
1D,2-7	113322-044	VNT ADJ UP 20W TAL ASM	
1E,2-7	113322-045	VNT ADJ UP 20W X-TAL ASM	

Note: in order to prevent deformation of the Footrest Tube do not over tighten Item (1). Max torque is 62 inch pounds.

Note: For High Mount Footrest Clamp part numbers please refer to the High Mount Footrest Clamp parts manual page.

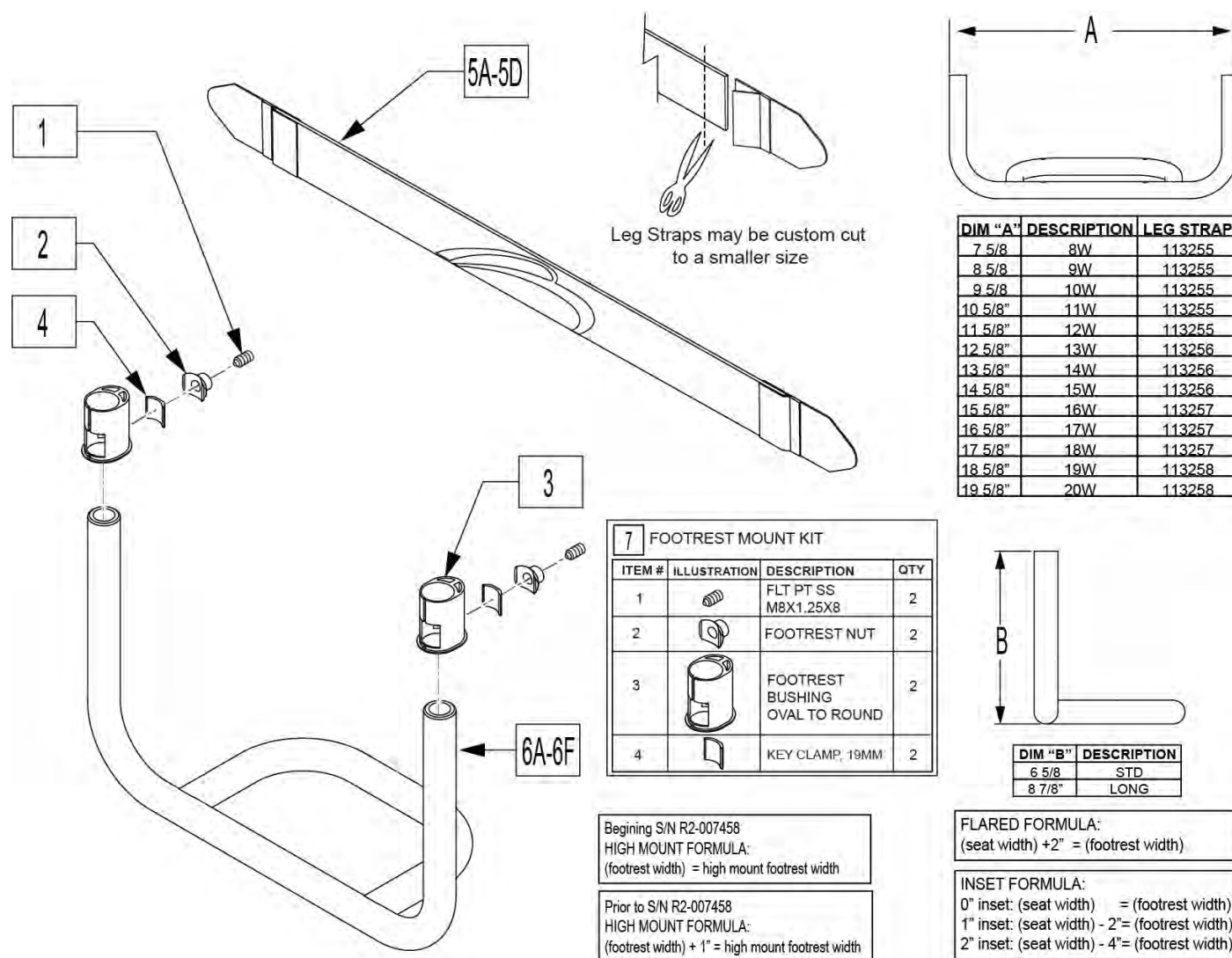


Pos.	Item Number	Description	Remarks
1	112339	FLT PT SS M8X1.25X8	
2	112648	FOOTREST NUT	
3	112348	FOOTREST BUSHING OVAL TO ROUND	
4	112651	KEY CLAMP, 19MM	
5A	113255	LEG STRAP 11"-12" BLK	
5B	113256	LEG STRAP 13"-15" BLK	
5C	113257	LEG STRAP 16"-18" BLK	
5D	113258	LEG STRAP 19"-20" BLK	
7	114934	FTRST MOUNT KIT	
1-4,5A,6D	114935	TBLAR FTRST 75FA 10W LNG KIT	Q7 only-Frame angle A

Pos.	Item Number	Description	Remarks
1-4,5A,6A	114936	TBLAR FTRST 75FA 10W STD KIT	Q7 only-Frame angle A
1-4,5A,6D	114937	TBLAR FTRST 75FA 11W LNG KIT	Q7 only-Frame angle A
1-4,5A,6A	114938	TBLAR FTRST 75FA 11W STD KIT	Q7 only-Frame angle A
1-4,5A,6D	114939	TBLAR FTRST 75FA 12W LNG KIT	Q7 only-Frame angle A
1-4,5A,6A	114940	TBLAR FTRST 75FA 12W STD KIT	Q7 only-Frame angle A
1-4,5B,6D	114941	TBLAR FTRST 75FA 13W LNG KIT	Q7 only-Frame angle A
1-4,5B,6A	114942	TBLAR FTRST 75FA 13W STD KIT	Q7 only-Frame angle A
1-4,5B,6D	114943	TBLAR FTRST 75FA 14W LNG KIT	Q7 only-Frame angle A
1-4,5B,6A	114944	TBLAR FTRST 75FA 14W STD KIT	Q7 only-Frame angle A
1-4,5B,6D	114945	TBLAR FTRST 75FA 15W LNG KIT	Q7 only-Frame angle A
1-4,5B,6A	114946	TBLAR FTRST 75FA 15W STD KIT	Q7 only-Frame angle A
1-4,5C,6D	114947	TBLAR FTRST 75FA 16W LNG KIT	Q7 only-Frame angle A
1-4,5C,6A	114948	TBLAR FTRST 75FA 16W STD KIT	Q7 only-Frame angle A
1-4,5C,6D	114949	TBLAR FTRST 75FA 17W LNG KIT	Q7 only-Frame angle A
1-4,5C,6A	114950	TBLAR FTRST 75FA 17W STD KIT	Q7 only-Frame angle A
1-4,5C,6D	114951	TBLAR FTRST 75FA 18W LNG KIT	Q7 only-Frame angle A
1-4,5C,6A	114952	TBLAR FTRST 75FA 18W STD KIT	Q7 only-Frame angle A
1-4,5D,6D	114953	TBLAR FTRST 75FA 19W LNG KIT	Q7 only-Frame angle A
1-4,5D,6A	114954	TBLAR FTRST 75FA 19W STD KIT	Q7 only-Frame angle A
1-4,5D,6D	114955	TBLAR FTRST 75FA 20W LNG KIT	Q7 only-Frame angle A
1-4,5D,6A	114956	TBLAR FTRST 75FA 20W STD KIT	Q7 only-Frame angle A
1-4,5A,6G	116845	TBLAR FTRST 80FA 8W STD KIT	Zippie Zone only
1-4,5A,6G	116844	TBLAR FTRST 80FA 8W LNG KIT	Zippie Zone only
1-4,5A,6G	116847	TBLAR FTRST 80FA 9W STD KIT	Zippie Zone only
1-4,5A,6G	116846	TBLAR FTRST 80FA 9W LNG KIT	Zippie Zone only
1-4,5A,6E	114957	TBLAR FTRST 80FA 10W LNG KIT	Frame angle B
1-4,5A,6B	114958	TBLAR FTRST 80FA 10W STD KIT	Frame angle B
1-4,5A,6E	114959	TBLAR FTRST 80FA 11W LNG KIT	Frame angle B
1-4,5A,6B	114960	TBLAR FTRST 80FA 11W STD KIT	Frame angle B
1-4,5A,6E	114961	TBLAR FTRST 80FA 12W LNG KIT	Frame angle B
1-4,5A,6B	114962	TBLAR FTRST 80FA 12W STD KIT	Frame angle B
1-4,5B,6E	114963	TBLAR FTRST 80FA 13W LNG KIT	Frame angle B
1-4,5B,6B	114964	TBLAR FTRST 80FA 13W STD KIT	Frame angle B
1-4,5B,6E	114965	TBLAR FTRST 80FA 14W LNG KIT	Frame angle B
1-4,5B,6B	114966	TBLAR FTRST 80FA 14W STD KIT	Frame angle B
1-4,5B,6E	114967	TBLAR FTRST 80FA 15W LNG KIT	Frame angle B
1-4,5B,6B	114968	TBLAR FTRST 80FA 15W STD KIT	Frame angle B
1-4,5C,6E	114969	TBLAR FTRST 80FA 16W LNG KIT	Frame angle B
1-4,5C,6B	114970	TBLAR FTRST 80FA 16W STD KIT	Frame angle B
1-4,5C,6E	114971	TBLAR FTRST 80FA 17W LNG KIT	Frame angle B
1-4,5C,6B	114972	TBLAR FTRST 80FA 17W STD KIT	Frame angle B
1-4,5C,6E	114973	TBLAR FTRST 80FA 18W LNG KIT	Frame angle B
1-4,5C,6B	114974	TBLAR FTRST 80FA 18W STD KIT	Frame angle B
1-4,5D,6E	114975	TBLAR FTRST 80FA 19W LNG KIT	Frame angle B
1-4,5D,6B	114976	TBLAR FTRST 80FA 19W STD KIT	Frame angle B
1-4,5D,6E	114977	TBLAR FTRST 80FA 20W LNG KIT	Frame angle B
1-4,5D,6B	114978	TBLAR FTRST 80FA 20W STD KIT	Frame angle B
1-4,5A,6F	114979	TBLAR FTRST 85FA 10W LNG KIT	Q7 only-Frame angle C
1-4,5A,6C	114980	TBLAR FTRST 85FA 10W STD KIT	Q7 only-Frame angle C
1-4,5A,6F	114981	TBLAR FTRST 85FA 11W LNG KIT	Q7 only-Frame angle C
1-4,5A,6C	114982	TBLAR FTRST 85FA 11W STD KIT	Q7 only-Frame angle C
1-4,5A,6F	114983	TBLAR FTRST 85FA 12W LNG KIT	Q7 only-Frame angle C
1-4,5A,6C	114984	TBLAR FTRST 85FA 12W STD KIT	Q7 only-Frame angle C
1-4,5B,6F	114985	TBLAR FTRST 85FA 13W LNG KIT	Q7 only-Frame angle C
1-4,5B,6C	114986	TBLAR FTRST 85FA 13W STD KIT	Q7 only-Frame angle C
1-4,5B,6F	114987	TBLAR FTRST 85FA 14W LNG KIT	Q7 only-Frame angle C
1-4,5B,6C	114988	TBLAR FTRST 85FA 14W STD KIT	Q7 only-Frame angle C

Pos.	Item Number	Description	Remarks
1-4,5B,6F	114989	TBLAR FTRST 85FA 15W LNG KIT	Q7 only-Frame angle C
1-4,5B,6C	114990	TBLAR FTRST 85FA 15W STD KIT	Q7 only-Frame angle C
1-4,5C,6F	114991	TBLAR FTRST 85FA 16W LNG KIT	Q7 only-Frame angle C
1-4,5C,6C	114992	TBLAR FTRST 85FA 16W STD KIT	Q7 only-Frame angle C
1-4,5C,6F	114993	TBLAR FTRST 85FA 17W LNG KIT	Q7 only-Frame angle C
1-4,5C,6C	114994	TBLAR FTRST 85FA 17W STD KIT	Q7 only-Frame angle C
1-4,5C,6F	114995	TBLAR FTRST 85FA 18W LNG KIT	Q7 only-Frame angle C
1-4,5C,6C	114996	TBLAR FTRST 85FA 18W STD KIT	Q7 only-Frame angle C
1-4,5D,6F	114997	TBLAR FTRST 85FA 19W LNG KIT	Q7 only-Frame angle C
1-4,5D,6C	114998	TBLAR FTRST 85FA 19W STD KIT	Q7 only-Frame angle C
1-4,5D,6F	114999	TBLAR FTRST 85FA 20W LNG KIT	Q7 only-Frame angle C
1-4,5D,6C	115000	TBLAR FTRST 85FA 20W STD KIT	Q7 only-Frame angle C

Note: in order to prevent deformation of the Footrest Tube do not over tighten Item (1). Max torque is 62 inch pounds.

Note: For High Mount Footrest Clamp part numbers please refer to the High Mount Footrest Clamp parts manual page.


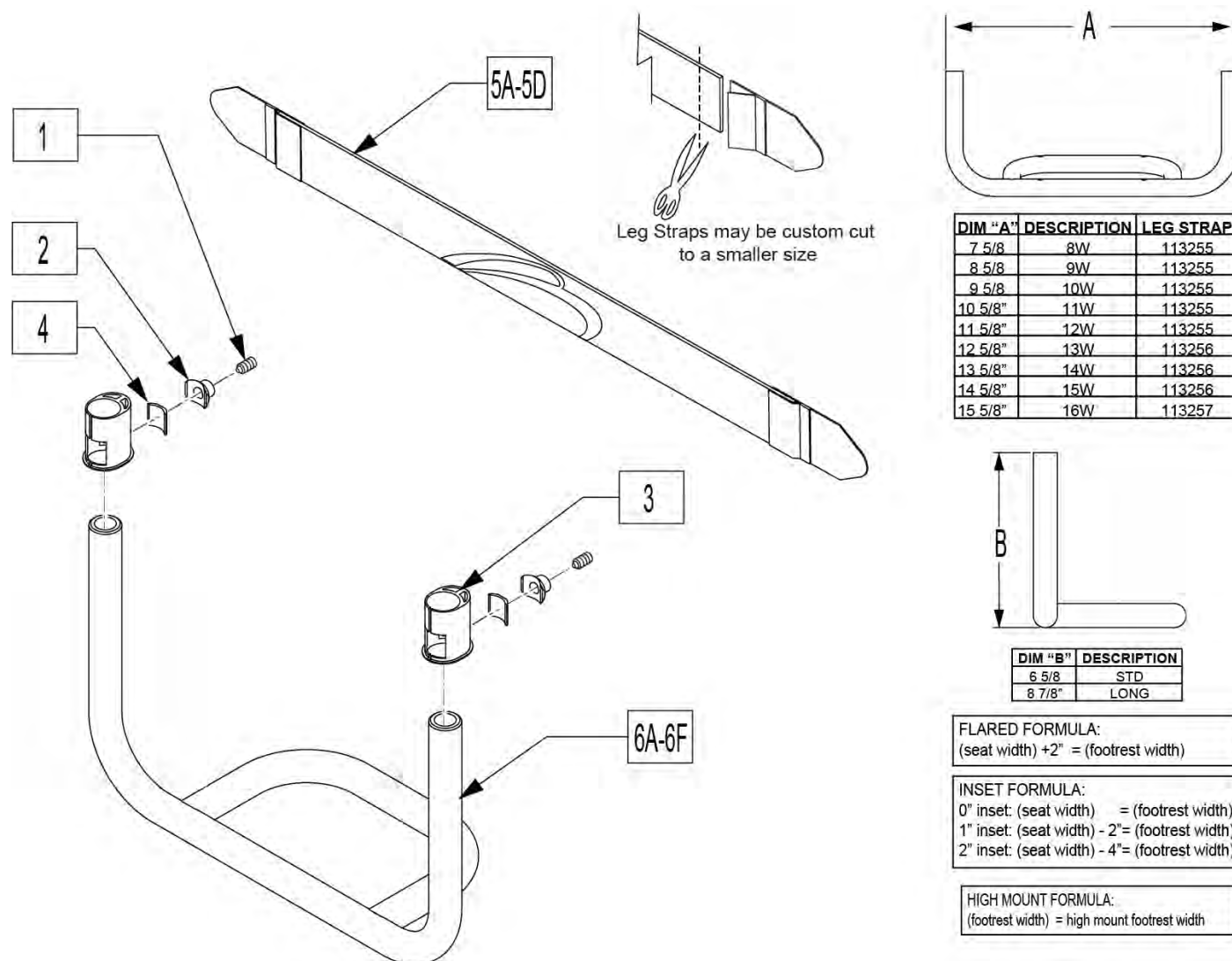


Pos.	Item Number	Description	Remarks
1	112339	FLT PT SS M8X1.25X8	
2	112648	FOOTREST NUT	
3	112651	KEY CLAMP, 19MM	
4	112348	FOOTREST BUSHING OVAL TO ROUND	
1-4	114934	FTRST MOUNT KIT	
5A	113255	LEG STRAP 11"-12" BLK	
5B	113256	LEG STRAP 13"-15" BLK	
5C	113257	LEG STRAP 16"-18" BLK	
5D	113258	LEG STRAP 19"-20" BLK	
1-4,5A,6D	114935	TBLAR FTRST 75FA 10W LNG KIT	Q7 only-Frame angle A


Pos.	Item Number	Description	Remarks
1-4,5A,6A	114936	TBLAR FTRST 75FA 10W STD KIT	Q7 only-Frame angle A
1-4,5A,6D	114937	TBLAR FTRST 75FA 11W LNG KIT	Q7 only-Frame angle A
1-4,5A,6A	114938	TBLAR FTRST 75FA 11W STD KIT	Q7 only-Frame angle A
1-4,5A,6D	114939	TBLAR FTRST 75FA 12W LNG KIT	Q7 only-Frame angle A
1-4,5A,6A	114940	TBLAR FTRST 75FA 12W STD KIT	Q7 only-Frame angle A
1-4,5B,6D	114941	TBLAR FTRST 75FA 13W LNG KIT	Q7 only-Frame angle A
1-4,5B,6A	114942	TBLAR FTRST 75FA 13W STD KIT	Q7 only-Frame angle A
1-4,5B,6D	114943	TBLAR FTRST 75FA 14W LNG KIT	Q7 only-Frame angle A
1-4,5B,6A	114944	TBLAR FTRST 75FA 14W STD KIT	Q7 only-Frame angle A
1-4,5B,6D	114945	TBLAR FTRST 75FA 15W LNG KIT	Q7 only-Frame angle A
1-4,5B,6A	114946	TBLAR FTRST 75FA 15W STD KIT	Q7 only-Frame angle A
1-4,5C,6D	114947	TBLAR FTRST 75FA 16W LNG KIT	Q7 only-Frame angle A
1-4,5C,6A	114948	TBLAR FTRST 75FA 16W STD KIT	Q7 only-Frame angle A
1-4,5A,6E	114957	TBLAR FTRST 80FA 10W LNG KIT	Frame angle B
1-4,5A,6B	114958	TBLAR FTRST 80FA 10W STD KIT	Frame angle B
1-4,5A,6E	114959	TBLAR FTRST 80FA 11W LNG KIT	Frame angle B
1-4,5A,6B	114960	TBLAR FTRST 80FA 11W STD KIT	Frame angle B
1-4,5A,6E	114961	TBLAR FTRST 80FA 12W LNG KIT	Frame angle B
1-4,5A,6B	114962	TBLAR FTRST 80FA 12W STD KIT	Frame angle B
1-4,5B,6E	114963	TBLAR FTRST 80FA 13W LNG KIT	Frame angle B
1-4,5B,6B	114964	TBLAR FTRST 80FA 13W STD KIT	Frame angle B
1-4,5B,6E	114965	TBLAR FTRST 80FA 14W LNG KIT	Frame angle B
1-4,5B,6B	114966	TBLAR FTRST 80FA 14W STD KIT	Frame angle B
1-4,5B,6E	114967	TBLAR FTRST 80FA 15W LNG KIT	Frame angle B
1-4,5B,6B	114968	TBLAR FTRST 80FA 15W STD KIT	Frame angle B
1-4,5C,6E	114969	TBLAR FTRST 80FA 16W LNG KIT	Frame angle B
1-4,5C,6B	114970	TBLAR FTRST 80FA 16W STD KIT	Frame angle B
1-4,5A,6F	114979	TBLAR FTRST 85FA 10W LNG KIT	Q7 only-Frame angle C
1-4,5A,6C	114980	TBLAR FTRST 85FA 10W STD KIT	Q7 only-Frame angle C
1-4,5A,6F	114981	TBLAR FTRST 85FA 11W LNG KIT	Q7 only-Frame angle C
1-4,5A,6C	114982	TBLAR FTRST 85FA 11W STD KIT	Q7 only-Frame angle C
1-4,5A,6F	114983	TBLAR FTRST 85FA 12W LNG KIT	Q7 only-Frame angle C
1-4,5A,6C	114984	TBLAR FTRST 85FA 12W STD KIT	Q7 only-Frame angle C
1-4,5B,6F	114985	TBLAR FTRST 85FA 13W LNG KIT	Q7 only-Frame angle C
1-4,5B,6C	114986	TBLAR FTRST 85FA 13W STD KIT	Q7 only-Frame angle C
1-4,5B,6F	114987	TBLAR FTRST 85FA 14W LNG KIT	Q7 only-Frame angle C
1-4,5B,6C	114988	TBLAR FTRST 85FA 14W STD KIT	Q7 only-Frame angle C
1-4,5B,6F	114989	TBLAR FTRST 85FA 15W LNG KIT	Q7 only-Frame angle C
1-4,5B,6C	114990	TBLAR FTRST 85FA 15W STD KIT	Q7 only-Frame angle C
1-4,5C,6F	114991	TBLAR FTRST 85FA 16W LNG KIT	Q7 only-Frame angle C
1-4,5C,6C	114992	TBLAR FTRST 85FA 16W STD KIT	Q7 only-Frame angle C

Note: in order to prevent deformation of the Footrest Tube do not over tighten Item (1). Max torque is 62 inch pounds.

Note: For High Mount Footrest Clamp part numbers please refer to the High Mount Footrest Clamp parts manual page.

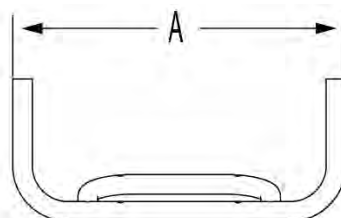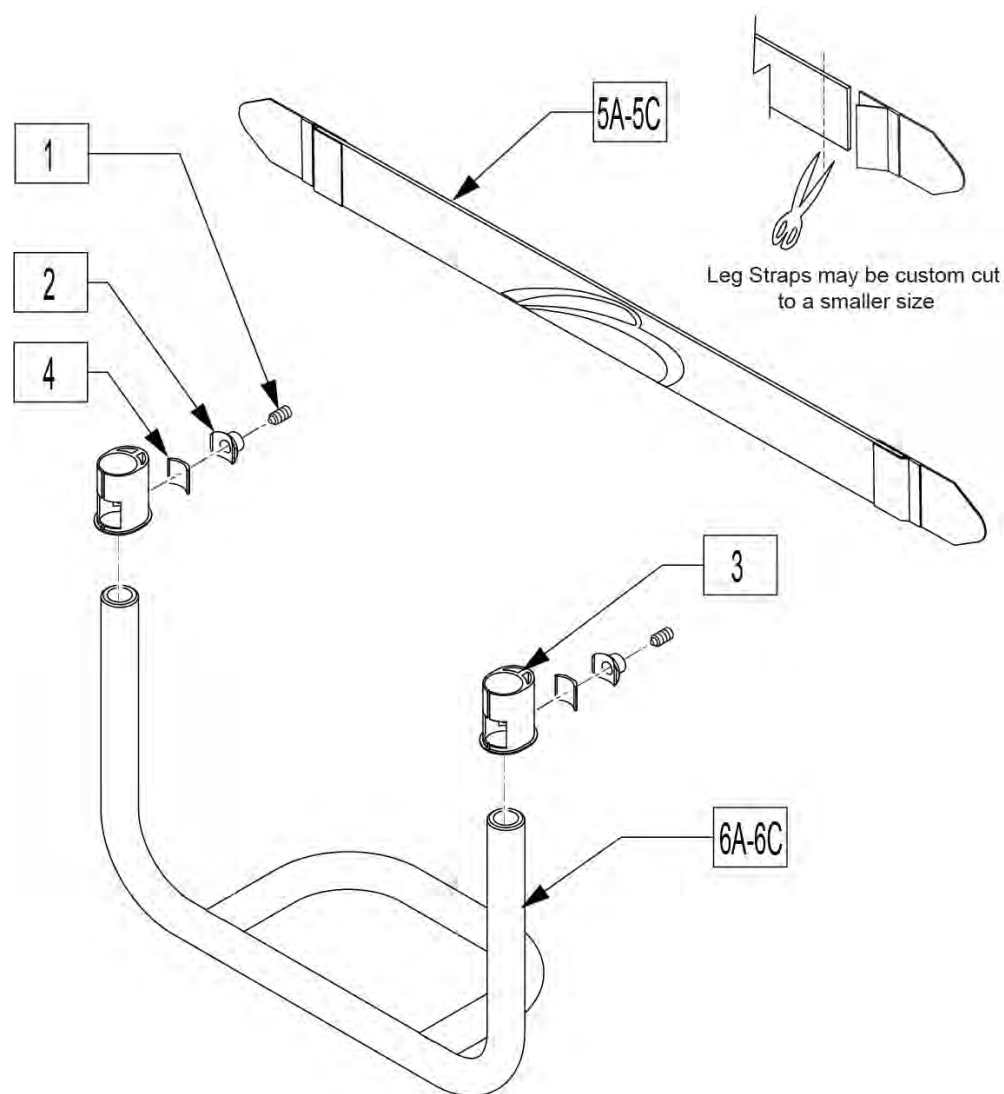

DIM "A"	DESCRIPTION	LEG STRAP
12 5/8"	13W	113256
13 5/8"	14W	113256
14 5/8"	15W	113256
15 5/8"	16W	113257
16 5/8"	17W	113257
17 5/8"	18W	113257
18 5/8"	19W	113258
19 5/8"	20W	113258

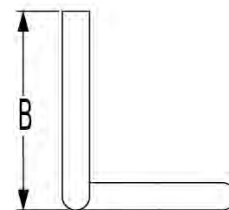

DIM "B"	DESCRIPTION
6 5/8"	STD
8 7/8"	LONG

FLARED FORMULA:
(seat width) + 2" = (footrest width)


INSET FORMULA:
 0" inset: (seat width) = (footrest width)
 1" inset: (seat width) - 2" = (footrest width)
 2" inset: (seat width) - 4" = (footrest width)

HIGH MOUNT FORMULA:
(footrest width) = high mount footrest width

Pos.	Item Number	Description	Remarks
1	112339	FLT PT SS M8X1.25X8	
2	112648	FOOTREST NUT	
3	112651	KEY CLAMP, 19MM	
4	112348	FOOTREST BUSHING OVAL TO ROUND	
1-4	114934	FTRST MOUNT KIT	
5A	113256	LEG STRAP 13"-15" BLK	
5B	113257	LEG STRAP 16"-18" BLK	
5C	113258	LEG STRAP 19"-20" BLK	
1-4,6A	128279	FOOTREST 75FA 13W STD KIT	Frame angle A
1-4,6A	128280	FOOTREST 75FA 13W LONG KIT	Frame angle A

Pos.	Item Number	Description	Remarks
1-4,6A	128281	FOOTREST 75AF 14W STD KIT	Frame angle A
1-4,6A	128282	FOOTREST 75FA 14W LONG KIT	Frame angle A
1-4,6A	128283	FOOTREST 75FA 15W STD KIT	Frame angle A
1-4,6A	128284	FOOTREST 75FA 15W LONG KIT	Frame angle A
1-4,6A	128285	FOOTREST 75FA 16W STD KIT	Frame angle A
1-4,6A	128286	FOOTREST 75FA 16W LONG KIT	Frame angle A
1-4,6A	128287	FOOTREST 75FA 17W STD KIT	Frame angle A
1-4,6A	128288	FOOTREST 75FA 17W LONG KIT	Frame angle A
1-4,6A	128289	FOOTREST 75FA 18W STD KIT	Frame angle A
1-4,6A	128290	FOOTREST 75FA 18W LONG KIT	Frame angle A
1-4,6A	128291	FOOTREST 75FA 19W STD KIT	Frame angle A
1-4,6A	128292	FOOTREST 75FA 19W LONG KIT	Frame angle A
1-4,6A	128293	FOOTREST 75FA 20W STD KIT	Frame angle A
1-4,6A	128294	FOOTREST 75FA 20W LONG KIT	Frame angle A
1-4,6B	128295	FOOTREST 80FA, 13W STD KIT	Frame angle B
1-4,6B	128296	FOOTREST 80FA, 13W LONG KIT	Frame angle B
1-4,6B	128297	FOOTREST 80FA, 14W STD KIT	Frame angle B
1-4,6B	128298	FOOTREST 80FA, 14W LONG KIT	Frame angle B
1-4,6B	128299	FOOTREST 80FA, 15W STD KIT	Frame angle B
1-4,6B	128300	FOOTREST 80FA, 15W LONG KIT	Frame angle B
1-4,6B	128301	FOOTREST 80FA, 16W STD KIT	Frame angle B
1-4,6B	128302	FOOTREST 80FA, 16W LONG KIT	Frame angle B
1-4,6B	128303	FOOTREST 80FA, 17W STD KIT	Frame angle B
1-4,6B	128304	FOOTREST 80FA, 17W LONG KIT	Frame angle B
1-4,6B	128305	FOOTREST 80FA, 18W STD KIT	Frame angle B
1-4,6B	128306	FOOTREST 80FA, 18W LONG KIT	Frame angle B
1-4,6B	128307	FOOTREST 80FA, 19W STD KIT	Frame angle B
1-4,6B	128308	FOOTREST 80FA, 19W LONG KIT	Frame angle B
1-4,6B	128309	FOOTREST 80FA, 20W STD KIT	Frame angle B
1-4,6B	128310	FOOTREST 80FA, 20W LONG KIT	Frame angle B
1-4,6C	128311	FOOTREST 85FA 13W STD KIT	Frame angle C
1-4,6C	128312	FOOTREST 85FA 13W LONG KIT	Frame angle C
1-4,6C	128313	FOOTREST 85FA 14W STD KIT	Frame angle C
1-4,6C	128314	FOOTREST 85FA 14W LONG KIT	Frame angle C
1-4,6C	128315	FOOTREST 85FA 15W STD KIT	Frame angle C
1-4,6C	128316	FOOTREST 85FA 15W LONG KIT	Frame angle C
1-4,6C	128317	FOOTREST 85FA 16W STD KIT	Frame angle C
1-4,6C	128318	FOOTREST 85FA 16W LONG KIT	Frame angle C
1-4,6C	128319	FOOTREST 85FA 17W STD KIT	Frame angle C
1-4,6C	128320	FOOTREST 85FA 17W LONG KIT	Frame angle C
1-4,6C	128321	FOOTREST 85FA 18W STD KIT	Frame angle C
1-4,6C	128322	FOOTREST 85FA 18W LONG KIT	Frame angle C
1-4,6C	128323	FOOTREST 85FA 19W STD KIT	Frame angle C
1-4,6C	128324	FOOTREST 85FA 19W LONG KIT	Frame angle C
1-4,6C	128325	FOOTREST 85FA 20W STD KIT	Frame angle C
1-4,6C	128326	FOOTREST 85FA 20W LONG KIT	Frame angle C


Note: in order for wear pads to properly bond to the footrest tubes, secure with cable ties for 24 hours after adhering.

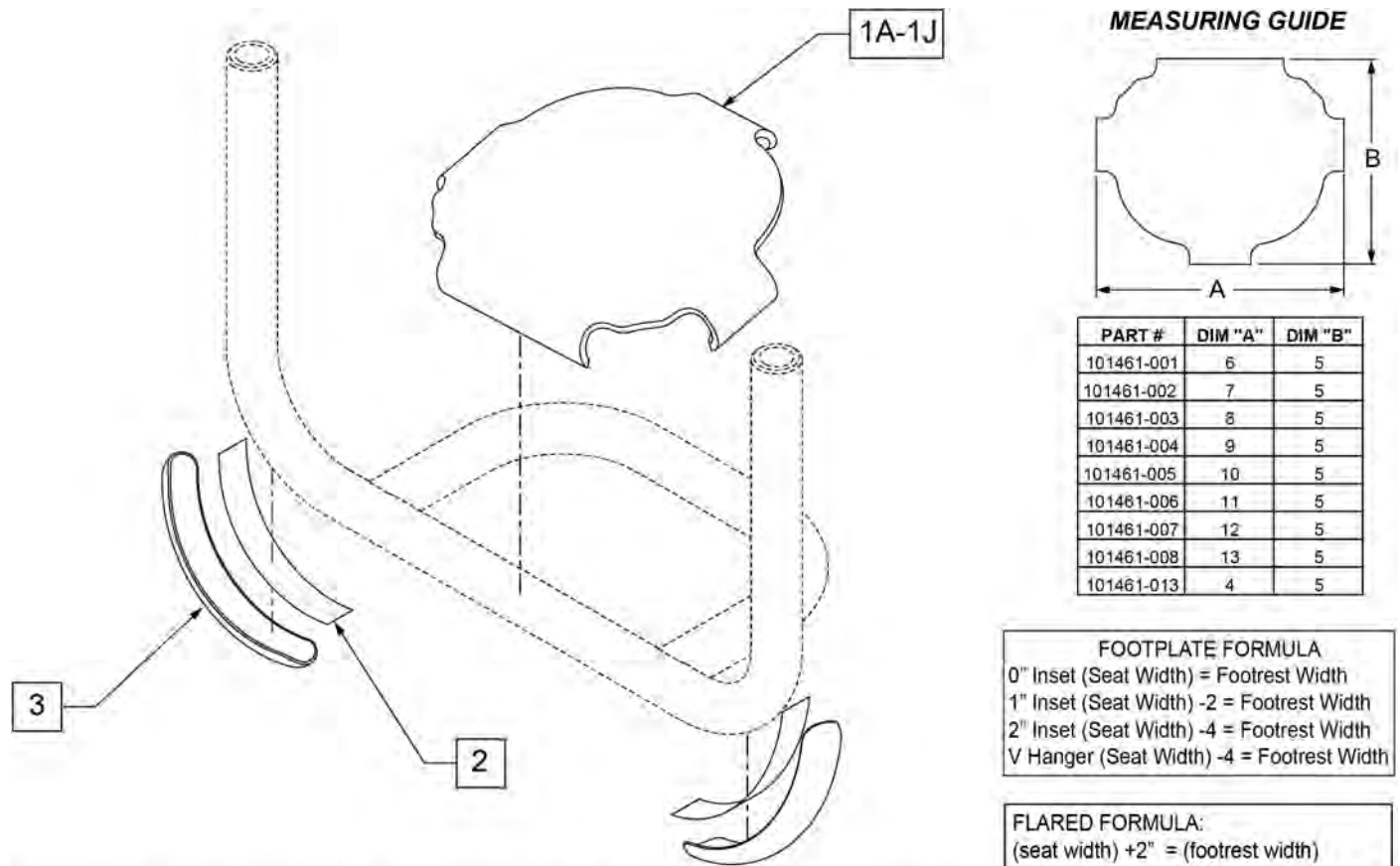

	DIM A	8 5/8	10 5/8	11 5/8	12 5/8	13 5/8	14 5/8	15 5/8	16 5/8	17 5/8	18 5/8	19 5/8
CHAIR SIZE		8w & 9w	10 & 11 w	12 w	13 w	14 w	15 w	16 w	17 w	18 w	19 w	20 w
GT		N/A	101461-002	101461-002	101461-002	101461-004	101461-004	101461-006	101461-006	101461-008	101461-008	101461-008
GTX		N/A	N/A	101461-002	101461-002	101461-002	101461-004	101461-004	101461-006	101461-006	101461-008	101461-008
GTI		N/A	101461-001	101461-002	101461-003	101461-004	101461-005	101461-006	101461-007	101461-008	101461-008	101461-008
Q7		N/A	101461-001	101461-002	101461-003	101461-004	101461-005	101461-006	101461-007	101461-008	101461-008	101461-008
ZIPPIE ZONE		101461-013	101461-001	101461-002	101461-003	101461-004	101461-005	101461-006	101461-007	101461-008	N/A	N/A

Pos.	Item Number	Description	Remarks
1A	101461-013	COVER FOOTREST SNAP ON 9W	
1B	101461-001	COVER FOOTREST SNAP-ON 11" W	
1C	101461-002	COVER FOOTREST SNAP-ON 12" W	
1D	101461-003	COVER FOOTREST SNAP-ON 13" W	
1E	101461-004	COVER FOOTREST SNAP-ON 14" W	
1F	101461-005	COVER FOOTREST SNAP-ON 15" W	
1G	101461-006	COVER FOOTREST SNAP-ON 16" W	
1H	101461-007	COVER FOOTREST SNAP-ON 17" W	
1J	101461-008	COVER FOOTREST SNAP-ON 18" W	
2,3	103107	WEAR PAD W/ ADHESIVE	Includes cable ties

Note: in order to prevent deformation of the Footrest Tube do not over tighten Item (1). Max torque is 62 inch pounds.

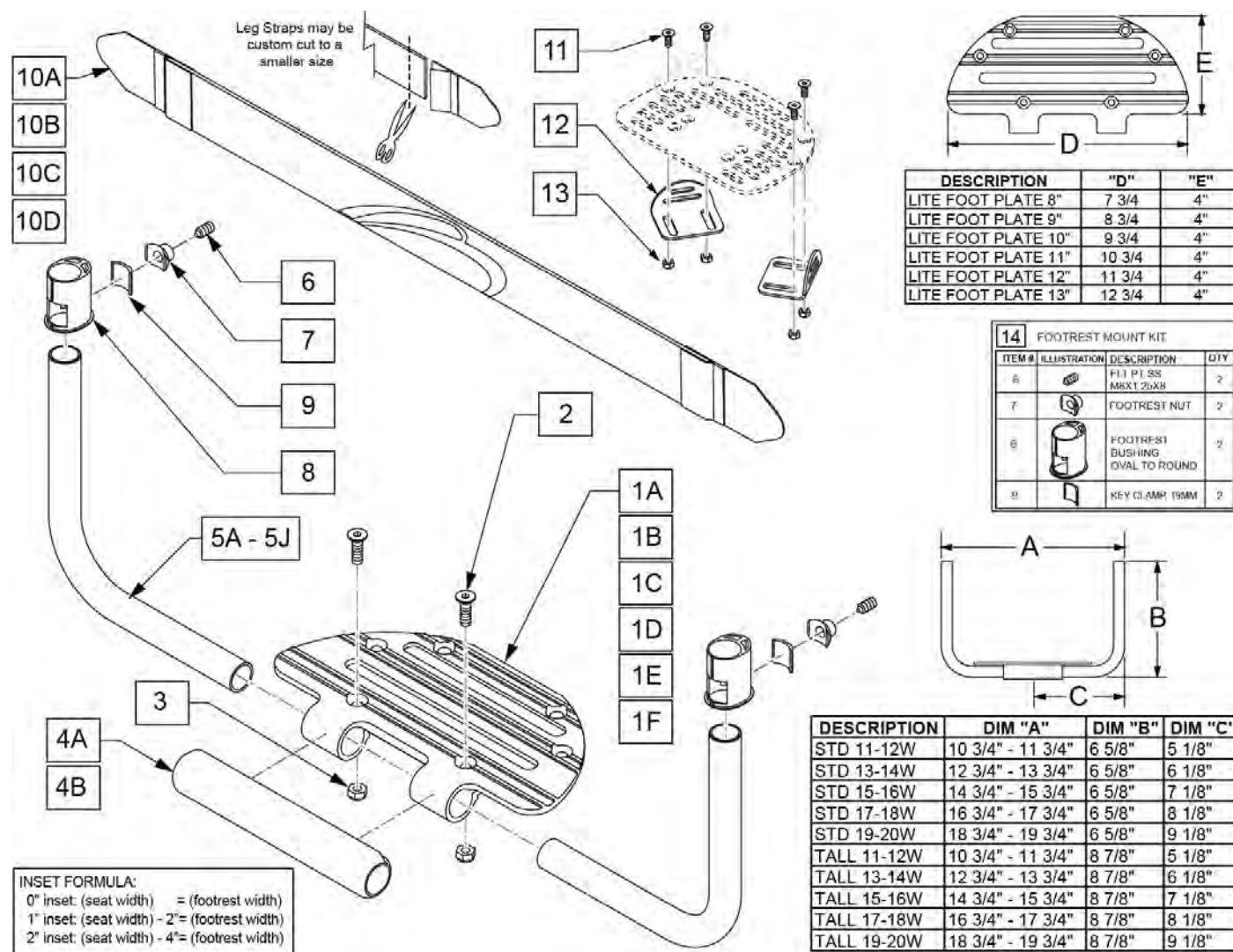

Pos.	Item Number	Description	Remarks
1A	111668	LITE FOOT PLATE 8"	Used for 11-13 Footrest Width
1B	111669	LITE FOOT PLATE 9"	Used for 14 Footrest Width
1C	111670	LITE FOOT PLATE 10"	Used for 15 Footrest Width
1D	111671	LITE FOOT PLATE 11"	Used for 16 Footrest Width
1E	111672	LITE FOOT PLATE 12"	Used for 17-18 Footrest Width
1F	111673	LITE FOOT PLATE 13"	Used for 19-20 Footrest Width
2,3	159565	LITE FOOTPLATE FASTENERS	2 of each
4A	066203-001	TUBE FRONT COVER, SMALL	Used for 11-15 Footrest Width
4B	066203-002	TUBE FRONT COVER, LARGE	Used for 16-20 Footrest Width
5A	111677	PLTFRM FTRST TUBE STD 11-12W	Used for 11-12 Footrest Width
5B	111678	PLTFRM FTRST TUBE TALL 11-12W	Used for 11-12 Footrest Width
5C	111679	PLTFRM FTRST TUBE STD 13-14W	Used for 13-14 Footrest Width
5D	111680	PLTFRM FTRST TUBE TALL 13-14W	Used for 13-14 Footrest Width
5E	111681	PLTFRM FTRST TUBE STD 15-16W	Used for 15-16 Footrest Width

Pos.	Item Number	Description	Remarks
5F	111682	PLTFRM FTRST TUBE TALL 15-16W	Used for 15-16 Footrest Width
5G	111683	PLTFRM FTRST TUBE STD 17-18W	Used for 17-18 Footrest Width
5H	111684	PLTFRM FTRST TUBE TALL 17-18W	Used for 17-18 Footrest Width
5J	111685	PLTFRM FTRST TUBE STD 19-20W	Used for 19-20 Footrest Width
5K	111686	PLTFRM FTRST TUBE TALL 19-20W	Used for 19-20 Footrest Width
10A	113255	LEG STRAP 11"-12" BLK	Used for 11-12 Footrest Width
10B	113256	LEG STRAP 13"-15" BLK	Used for 13-15 Footrest Width
10C	113257	LEG STRAP 16"-18" BLK	Used for 16-18 Footrest Width
10D	113258	LEG STRAP 19"-20" BLK	Used for 19-20 Footrest Width
14	114934	FTRST MOUNT KIT	Pair
11,13	159566	POS PLATE FASTENERS	4 each
11-13	050228	PLATE POSITIONING FOOT W/HDWR	
1A,2,3,4A,5B, 6-9	115001	ALUM ULITE 11-12W LNG KIT	
1A,2,3,4A,5A, 6-9	115002	ALUM ULITE 11-12W STD KIT	
1A,2,3,4A,5D, 6-9	115003	ALUM ULITE 13W LNG KIT	
1A,2,3,4A,5C, 6-9	115004	ALUM ULITE 13W STD KIT	
1B,2,3,4A,5D, 6-9	115005	ALUM ULITE 14W LNG KIT	
1B,2,3,4A,5C, 6-9	115006	ALUM ULITE 14W STD KIT	
1C,2,3,4A,5F, 6-9	115007	ALUM ULITE 15W LNG KIT	
1C,2,3,4A,5E, 6-9	115008	ALUM ULITE 15W STD KIT	
1D,2,3,4A,5F, 6-9	115009	ALUM ULITE 16W LNG KIT	
1D,2,3,4A,5E, 6-9	115010	ALUM ULITE 16W STD KIT	
1E,2,3,4A,5H, 6-9	115011	ALUM ULITE 17-18W LNG KIT	
1E,2,3,4A,5G, 6-9	115012	ALUM ULITE 17-18W STD KIT	
1E,2,3,4A,5K, 6-9	115013	ALUM ULITE 19-20W LNG KIT	
1F,2,3,4A,5J, 6-9	115014	ALUM ULITE 19-20W STD KIT	

Note: In order to prevent deformation of the Footrest Tube do not over tighten Item (7). Max torque is 62 inch pounds.

Note: For High Mount Footrest Clamp please refer to the High Mount Footrest Clamp parts manual page.

Note: Prior to 3/16/11 footplates were used w/o item 13D (561112) spacer and utilized smaller footplates. If replacing a footplate on a chair built after 3/16/11 items 1B and 13D should be ordered.

INSET FORMULA:
 0" inset: (seat width) = (footrest width)
 1" inset: (seat width) - 2" = (footrest width)
 2" inset: (seat width) - 4" = (footrest width)

HIGH MOUNT FORMULA:
 (footrest width) + 1" = high mount footrest width

FLARED FORMULA:
 (seat width) + 2" = (footrest width)

FOOTREST MEASURING GUIDE

FOOTPLATE P/N	"D"	"E"
500690	6"	5 1/2"
500691	8"	5 1/2"
500692	10"	5 1/2"
500693	12"	5 1/2"

15 FOOTREST MOUNT KIT

ITEM #	ILLUSTRATION	DESCRIPTION	QTY
7		FLT PT SS M8X1.25X8	2
8		FOOTREST NUT	2
9		FOOTREST BUSHING OVAL TO ROUND	2
10		KEY CLAMP 19MM	2

P/N	DESCRIPTION	DIM "A"	DIM "B"	DIM "C"
116228	STD 9-10W	8 3/4" - 9 3/4"	6 5/8"	4 1/8"
111677	STD 11-12W	10 3/4" - 11 3/4"	6 5/8"	5 1/8"
111679	STD 13-14W	12 3/4" - 13 3/4"	6 5/8"	6 1/8"
111681	STD 15-16W	14 3/4" - 15 3/4"	6 5/8"	7 1/8"
111683	STD 17-18W	16 3/4" - 17 3/4"	6 5/8"	8 1/8"
111685	STD 19-20W	18 3/4" - 19 3/4"	6 5/8"	9 1/8"
116229	TALL 9-10W	8 3/4" - 9 3/4"	8 7/8"	4 1/8"
111678	TALL 11-12W	10 3/4" - 11 3/4"	8 7/8"	5 1/8"
111680	TALL 13-14W	12 3/4" - 13 3/4"	8 7/8"	6 1/8"
111682	TALL 15-16W	14 3/4" - 15 3/4"	8 7/8"	7 1/8"
111684	TALL 17-18W	16 3/4" - 17 3/4"	8 7/8"	8 1/8"
111686	TALL 19-20W	18 3/4" - 19 3/4"	8 7/8"	9 1/8"

Pos.	Item Number	Description	Remarks
1A	102265-003	FHCS M6X1X25 FULL NP	Used w/o Adapter Spacer

Pos.	Item Number	Description	Remarks
1B	103500-009	FHCSM6X1X30FULLNP	Used w/Adapter Spacer; used for chairs after 3/16/11
2A	116228	PLT FRM FTRST TUBE STD 9-10W	
2A	111677	PLTFRM FTRST TUBE STD 11-12W	
2A	111679	PLTFRM FTRST TUBE STD 13-14W	
2A	111681	PLTFRM FTRST TUBE STD 15-16W	
2A	111683	PLTFRM FTRST TUBE STD 17-18W	
2A	111685	PLTFRM FTRST TUBE STD 19-20W	
2B	116229	PLT FRM FTRST TUBE TALL 9-10W	
2B	111678	PLTFRM FTRST TUBE TALL 11-12W	
2B	111680	PLTFRM FTRST TUBE TALL 13-14W	
2B	111682	PLTFRM FTRST TUBE TALL 15-16W	
2B	111684	PLTFRM FTRST TUBE TALL 17-18W	
2B	111686	PLTFRM FTRST TUBE TALL 19-20W	
3	550351	PLATFORM FOOTPLATE CLAMP	
4	102268-001	HHCS M6 x 1 x 25	Used w/Adapter Spacer
5	104381	NUT HEX NYLOCK M6X1.0	Used w/o Adapter Spacer
6A	116225	NEW PATFRORM F/PT 4 X 5 1/2	Used for 9-10 Footrest Width prior to 3/16/11
6B	500690	NEW PLTFORM F/PLT 6X5.5	Used for 10" Footrest Width
6C	500691	NEW PLTFORM F/PLT 8X5.5	Used for 11"-12" Footrest Width
6D	500692	NEW PLTFORM F/PLT 10X5.5	Used for 13"-14" Footrest Width
6E	500693	NEW PLTFORM F/PLT 12X5.5	Used for 15"-20" Footrest Width
7	112339	FLT PT SS M8X1.25X8	
8	112648	FOOTREST NUT	
9	112348	FOOTREST BUSHING OVAL TO ROUND	
10	112651	KEY CLAMP, 19MM	
11A	113255	LEG STRAP 11"-12" BLK	
11B	113256	LEG STRAP 13"-15" BLK	
11C	113257	LEG STRAP 16"-18" BLK	
11D	113258	LEG STRAP 19"-20" BLK	
12	050073	PLATE POSITIONING FOOT	
13A	050075-001	ADAPTIVE FT/RST SPCR 2" P/FRM	
13B	050075-002	ADAPTIVE FT/RST SPCR 4" P/FRM	

Pos.	Item Number	Description	Remarks
13C	050075-003	ADAPTIVE FT/RST SPCR 6" P/FRM	
13D	561112	3/4 OD X .318 ID X .25 NYLON	Used for chairs after 3/16/11
14	561010	3/4" CAP	For use with High Mount Footrest only
15	114934	FTRST MOUNT KIT	
1B,5,12	050228	PLATE POSITIONING FOOT W/HDWR	
1AB,2B,3-5,6B,7-10,11A,13A,13B,13C	116848	ALUM FTRST 9-10W LNG KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2A,3-5,6B,7-10,11A,13A,13B,13C	116849	ALUM FTRST 9-10W STD KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2B,3-5,6C,7-10,11A,13A,13B,13C	115015	ALUM FTRST 11-12W LNG KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2A,3-5,6C,7-10,11A,13A,13B,13C	115016	ALUM FTRST 11-12W STD KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2B,3-5,6D,7-10,11B,13A,13B,13C	115017	ALUM FTRST 13W LNG KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2A,3-5,6D,7-10,11B,13A,13B,13C	115018	ALUM FTRST 13W STD KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2B,3-5,6D,7-10,11B,13A,13B,13C	115019	ALUM FTRST 14W LNG KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap


Pos.	Item Number	Description	Remarks
1AB,2A,3-5,6D,7-10,11B,13A,13B,13C	115020	ALUM FTRST 14W STD KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2B,3-5,6E,7-10,11B,13A,13B,13C	115021	ALUM FTRST 15W LNG KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2A,3-5,6E,7-10,11B,13A,13B,13C	115022	ALUM FTRST 15W STD KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2B,3-5,6E,7-10,11C,13A,13B,13C	115023	ALUM FTRST 16W LNG KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2A,3-5,6E,7-10,11C,13A,13B,13C	115024	ALUM FTRST 16W STD KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2B,3-5,6E,7-10,11C,13A,13B,13C	115025	ALUM FTRST 17W LNG KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2A,3-5,6E,7-10,11C,13A,13B,13C	115026	ALUM FTRST 17W STD KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2B,3-5,6E,7-10,11C,13A,13B,13C	115027	ALUM FTRST 18W LNG KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2A,3-5,6E,7-10,11C,13A,13B,13C	115028	ALUM FTRST 18W STD KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap

Pos.	Item Number	Description	Remarks
1AB,2B,3-5,6E,7-10,11D,13A,13B,13C	115029	ALUM FTRST 19-20W LNG KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap
1AB,2A,3-5,6E,7-10,11D,13A,13B,13C	115030	ALUM FTRST 19-20W STD KIT	Includes 2", 4", 6" Adaptive Footrest Spacers & Legrest Strap

Note: For High Mount Footrest Clamp part numbers please refer to the High Mount Footrest Clamp parts manual page.

Note: In order to prevent deformation of the Footrest Tube do not over tighten Item 11. Max torque is 62 inch pounds.

Note: For Q7 this option is not offered on 19" & 20" footrest widths.

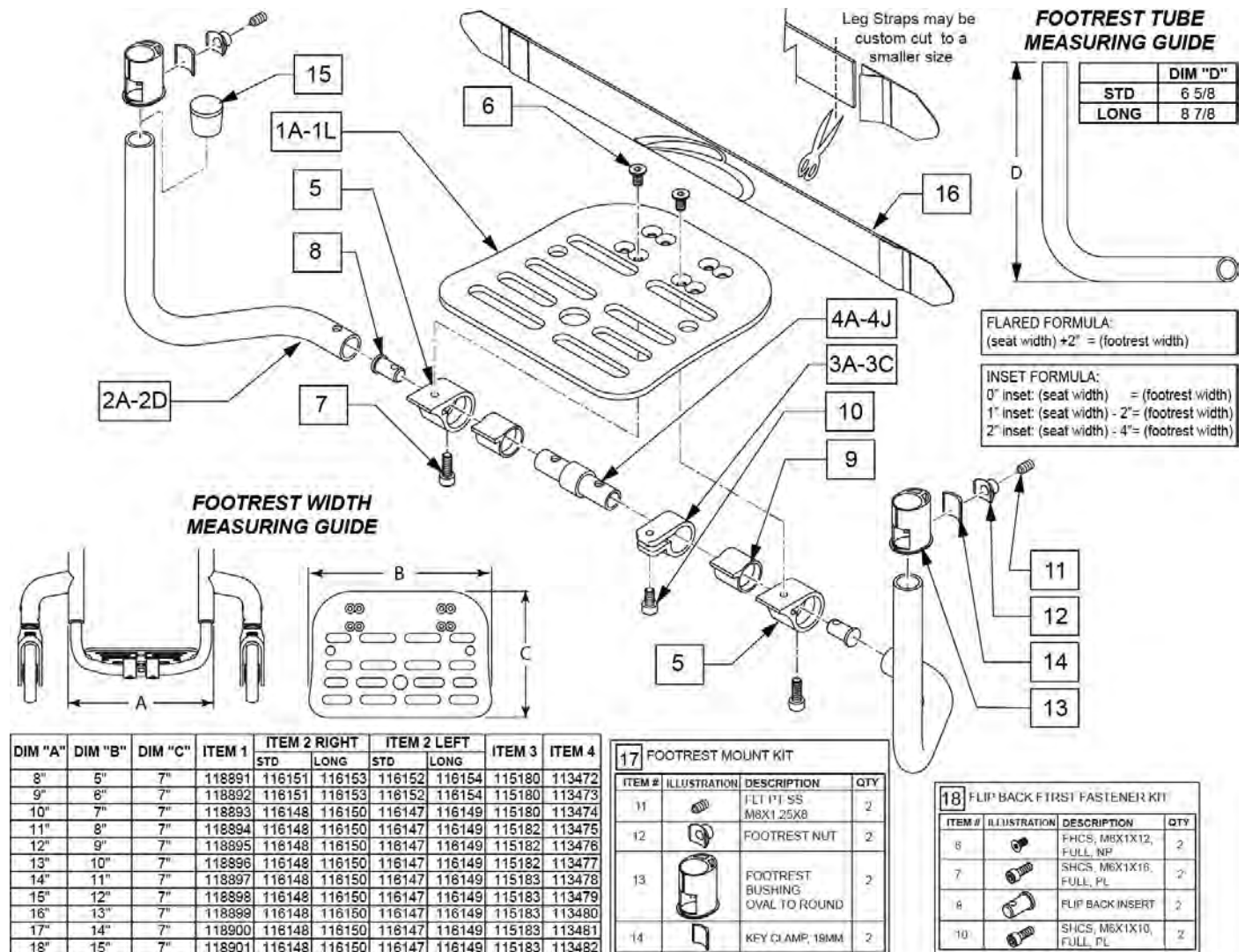


Pos.	Item Number	Description	Remarks
1A	118891	FOOTPLATE 8-9 PED	
1B	118892	FOOTPLATE 9-10 PED	
1C	118893	FOOTPLATE 10-11 PED	
1D	118894	FOOTPLATE 11-12 PED	
1E	118895	FOOTPLATE 12-13 PED	
1F	118896	FOOTPLATE 13-14 PED	

Pos.	Item Number	Description	Remarks
1G	118897	FOOTPLATE 14-15 PED	
1H	118898	FOOTPLATE 15-16 PED	
1J	118899	FOOTPLATE 16-17 PED	
1K	118900	FOOTPLATE 17-18 PED	
1L	118901	FOOTPLATE 18-19 PED	
2A	116147	FB TUBE LH STD	Left-Used w/ 10"-18" wide Footrest
2A	116148	FB TUBE RH STD	Right-Used w/ 10"-18" wide Footrest
2B	116149	FB TUBE LH LNG	Left-Used w/ 10"-18" wide Footrest
2B	116150	FB TUBE RH LNG	Right-Used w/ 10"-18" wide Footrest
2C	116151	FB TUBE RH STD 8-9	Right-Used w/ 8"-9" wide Footrest
2C	116152	FB TUBE LH STD 8-9	Left-Used w/ 8"-9" wide Footrest
2D	116153	FB TUBE RH LNG 8-9	Right-Used w/ 8"-9" wide Footrest
2D	116154	FB TUBE LH LNG 8-9	Left-Used w/ 8"-9" wide Footrest
3A	115180	FB FOOT ANGLE ADJ 8-9	
3B	115182	FB FOOT ANGLE ADJ 10-14	
3C	115183	FB FOOT ANGLE ADJ 15-18	
15	561010	3/4" CAP	Use on High Mount Footrest only
16	113255	LEG STRAP 11"-12" BLK	
16	113256	LEG STRAP 13"-15" BLK	
16	113257	LEG STRAP 16"-18" BLK	
16	113258	LEG STRAP 19"-20" BLK	
17	114934	FTRST MOUNT KIT	
18	159571	FLP BACK FTRST FASTENER KIT	2 of each
1A,2C,3A,4A,5-16	120238	F/BCK F/RST STD ASSM 8	Fits: 8"
1B,2C,3A,4B,5-16	120239	F/BCK F/RST STD ASSM 9	Fits: 9"
1C,2A,3B,4A,5-16	120240	F/BCK F/RST STD ASSM 10	Fits:10"
1D,2A,3B,4B,5-16	120241	F/BCK F/RST STD ASSM 11	Fits:11"
1E,2A,3B,4C,5-16	120242	F/BCK F/RST STD ASSM 12	Fits: 12"
1F,2A,3B,4D,5-16	120243	F/BCK F/RST STD ASSM 13	Fits: 13"
1G,2A,3B,4E,5-16	120244	F/BCK F/RST STD ASSM 14	Fits: 14"
1H,2A,3C,4F,5-16	120245	F/BCK F/RST STD ASSM 15	Fits: 15"
1J,2A,3C,4G,5-16	120246	F/BCK F/RST STD ASSM 16	Fits: 16"
1K,2A,3C,4H,5-16	120247	F/BCK F/RST STD ASSM 17	Fits: 17"
1L,2A,3C,4J,5-16	120248	F/BCK F/RST STD ASSM 18	Fits: 18"
1A,2D,3A,4A,5-16	120249	F/BCK F/RST LNG ASSM 8	Fits: 8"
1B,2D,3A,4B,5-16	120250	F/BCK F/RST LNG ASSM 9	Fits:9"
1C,2B,3B,4A,5-16	120251	F/BCK F/RST LNG ASSM 10	Fits: 10"
1D,2B,3B,4B,5-16	120252	F/BCK F/RST LNG ASSM 11	Fits 11"
1A,2B,3B,4C,5-16	120253	F/BCK F/RST LNG ASSM 12	Fits: 12"
1B,2B,3B,4D,5-16	120254	F/BCK F/RST LNG ASSM 13	Fits: 13"
1C,2B,3B,4E,5-16	120255	F/BCK F/RST LNG ASSM 14	Fits: 14"
1D,2B,3C,4F,5-16	120256	F/BCK F/RST LNG ASSM 15	Fits: 15"
1E,2B,3C,4G,5-16	120257	F/BCK F/RST LNG ASSM 16	Fits: 16"
1F,2B,3C,4H,5-16	120258	F/BCK F/RST LNG ASSM 17	Fits: 17"
1G,2B,3C,4J,5-16	120259	F/BCK F/RST LNG ASSM 18	Fits: 18"
3,4A,5-10	159572	FLIP BACK CENTER KIT 8 & 10	Fits: 8"&10"
3,4B,5-10	159573	FLIP BACK CENTER KIT 9 & 11	Fits: 9"&11"
3,4C,5-10	159574	FLIP BACK CENTER KIT 12	Fits: 12"
3,4D,5-10	159575	FLIP BACK CENTER KIT 13	Fits: 13"
3,4E,5-10	159576	FLIP BACK CENTER KIT 14	Fits: 14"
3,4F,5-10	159577	FLIP BACK CENTER KIT 15	Fits: 15"
3,4G,5-10	159578	FLIP BACK CENTER KIT 16	Fits: 16"
3,4H,5-10	159579	FLIP BACK CENTER KIT 17	Fits: 17"
3,4J,5-10	159580	FLIP BACK CENTER KIT 18	Fits: 18"
1A,8	159954	ALU FB PLT FRM REPLACEMENT 12	Fits: 12"
1B,8	159955	ALU FB PLT FRM REPLACEMENT 13	Fits: 13"
1C,8	159956	ALU FB PLT FRM REPLACEMENT 14	Fits: 14"

Pos.	Item Number	Description	Remarks
1D,8	159957	ALU FB PLT FRM REPLACEMENT 15	Fits: 15"
1E,8	159958	ALU FB PLT FRM REPLACEMENT 16	Fits: 16"
1F,8	159959	ALU FB PLT FRM REPLACEMENT 17	Fits: 17"
1G,8	159960	ALU FB PLT FRM REPLACEMENT 18	Fits: 18"


Note: For footrest part numbers please refer to the Flip-Back Footrest parts manual page.

Note: See information panel for Instruction Sheet.

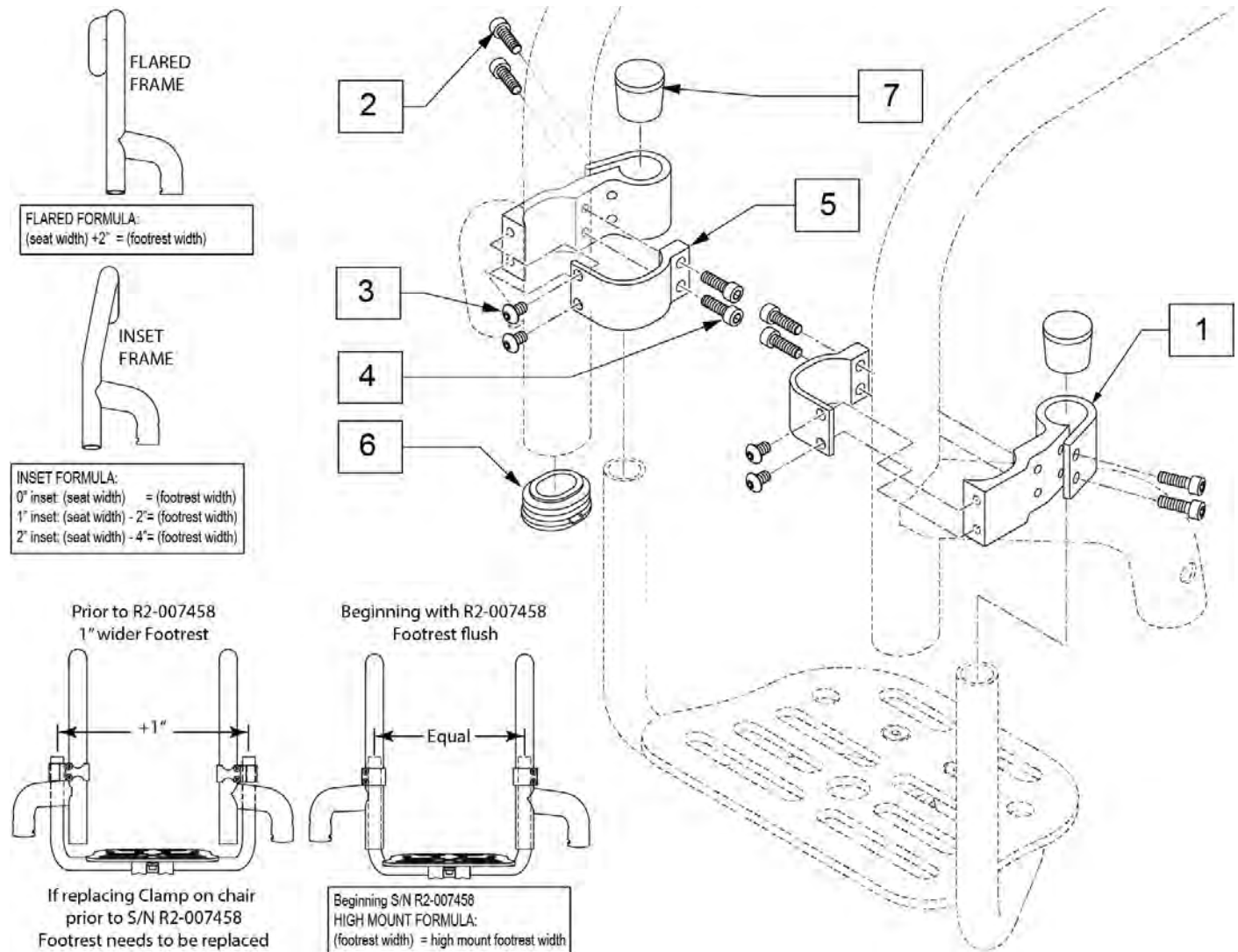

Pos.	Item Number	Description	Remarks
2	103181	SHCS M5 X .8 X 14 W/PL	
3	103184-002	BHCS, M5X0.8X8, FULL, PL	
4	113317	SHCS M5X0.8X25 FULL PL	
6	110161	FRAME INSERT, REAR SEAT TUBE	
7	561010	3/4" CAP	
1-7	123463	HIGH MOUNT FOOTREST KIT	Pair


Note: In order to prevent deformation of the Footrest Tube do not over tighten Item 21. Max torque is 62 inch pounds.

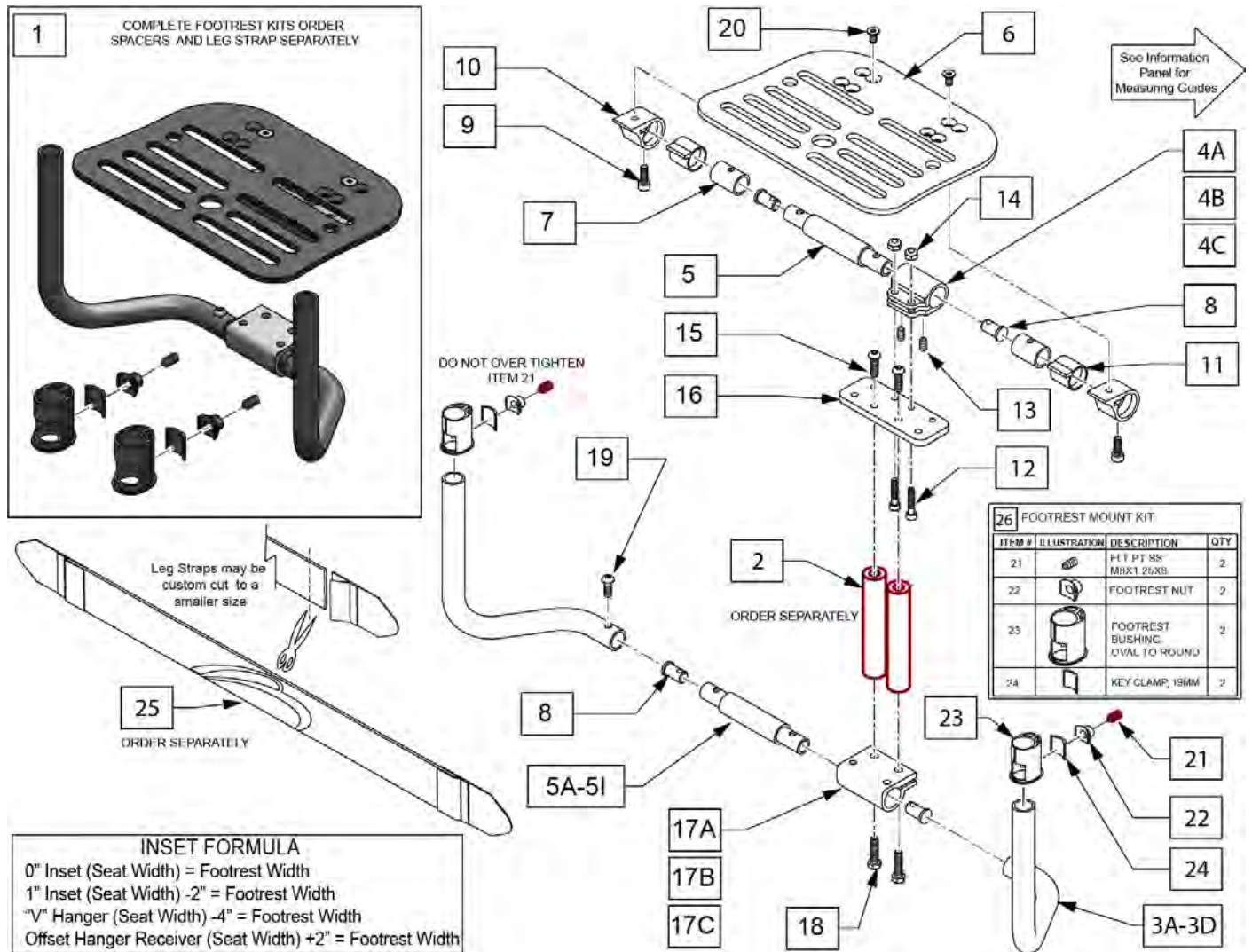


Pos.	Item Number	Description	Remarks
1	145100	F/BCK FTRST ADP STD ASSM 8	Fits: 8"
1	145101	F/BCK FTRST ADP STD ASSM 9	Fits: 9"
1	145102	F/BCK FTRST ADP STD ASSM 10	Fits: 10"
1	145103	F/BCK FTRST ADP STD ASSM 11	Fits: 11"
1	145104	F/BCK FTRST ADP STD ASSM 12	Fits: 12"
1	145105	F/BCK FTRST ADP STD ASSM 13	Fits: 13"
1	145106	F/BCK FTRST ADP STD ASSM 14	Fits: 14"
1	145107	F/BCK FTRST ADP STD ASSM 15	Fits: 15"
1	145108	F/BCK FTRST ADP STD ASSM 16	Fits: 16"
1	145109	F/BCK FTRST ADP STD ASSM 17	Fits: 17"
1	145110	F/BCK FTRST ADP STD ASSM 18	Fits: 18"
1	145115	F/BCK FTRST ADP LNG ASSM 12	Fits: 12"
1	145116	F/BCK FTRST ADP LNG ASSM 13	Fits: 13"
1	145117	F/BCK FTRST ADP LNG ASSM 14	Fits: 14"
1	145118	F/BCK FTRST ADP LNG ASSM 15	Fits: 15"

Pos.	Item Number	Description	Remarks
1	145119	F/BCK FTRST ADP LNG ASSM 16	Fits: 16"
1	145120	F/BCK FTRST ADP LNG ASSM 17	Fits: 17"
1	145121	F/BCK FTRST ADP LNG ASSM 18	Fits: 18"
2	050075-001	ADAPTIVE FT/RST SPCR 2" P/FRM	
2	050075-002	ADAPTIVE FT/RST SPCR 4" P/FRM	
2	050075-003	ADAPTIVE FT/RST SPCR 6" P/FRM	
3A	116147	FB TUBE LH STD	Left-Used w/ 10"-18" wide Footrest
3A	116148	FB TUBE RH STD	Right-Used w/ 10"-18" wide Footrest
3B	116149	FB TUBE LH LNG	Left-Used w/ 10"-18" wide Footrest
3B	116150	FB TUBE RH LNG	Right-Used w/ 10"-18" wide Footrest
3C	116151	FB TUBE RH STD 8-9	Right-Used w/ 8"-9" wide Footrest
3C	116152	FB TUBE LH STD 8-9	Left-Used w/ 8"-9" wide Footrest
3D	116153	FB TUBE RH LNG 8-9	Right-Used w/ 8"-9" wide Footrest
3D	116154	FB TUBE LH LNG 8-9	Left-Used w/ 8"-9" wide Footrest
4A	115180	FB FOOT ANGLE ADJ 8-9	
4B	115182	FB FOOT ANGLE ADJ 10-14	
4C	115183	FB FOOT ANGLE ADJ 15-18	
5A	113472	FLIP BACK CENTER TUBE 10"	Fits: 8"&10"
5B	113473	FLIP BACK CENTER TUBE 11"	Fits: 9"&11"
5C	113474	FLIP BACK CENTER TUBE 12"	Fits: 12"
5D	113475	FLIP BACK CENTER TUBE 13"	Fits: 13"
5E	113476	FLIP BACK CENTER TUBE 14"	Fits: 14"
5F	113477	FLIP BACK CENTER TUBE 15"	Fits: 15"
5G	113478	FLIP BACK CENTER TUBE 16"	Fits: 16"
5H	113479	FLIP BACK CENTER TUBE 17"	Fits: 17"
5I	113480	FLIP BACK CENTER TUBE 18"	Fits: 18"
6	118891	FOOTPLATE 8-9 PED	
6	118892	FOOTPLATE 9-10 PED	
6	118893	FOOTPLATE 10-11 PED	
6	118894	FOOTPLATE 11-12 PED	
6	118895	FOOTPLATE 12-13 PED	
6	118896	FOOTPLATE 13-14 PED	
6	118897	FOOTPLATE 14-15 PED	
6	118898	FOOTPLATE 15-16 PED	
6	118899	FOOTPLATE 16-17 PED	
6	118900	FOOTPLATE 17-18 PED	
6	118901	FOOTPLATE 18-19 PED	
7	128884	FB PIVOT TUBE	
8	113449	FLIP BACK INSERT	
9	104522	SHCS M6X1X16 FULL PL	
10	115181	FB FOOTPLATE RECEIVER	
11	115774	FLIP BACK FOOTREST BUSHING	
12	102260-002	SHCS,M5 X .8 X 25,FULL,NP	
13	061615	SET SCREW M5 X 6	
14	220121	NUT HEX NYLOCK M5 BLACK	
15	058901-001	BHCS,M6X1X20,FULL,PL	
16	128883	FB FOOTPLATE TO SPACER PLATE	
17A	128882	PLATFORM FOOTPLATE	
17B	550351	CLAMP, FOOTPLATE	
18	102268-001	HHCS M6 X 1 X 25 BLK	
19	112346	BHCS,M6X1X16,FULL,PL	
20	103500-004	FHCSM6X1X12FULLNP	
21	112339	FLT PT SS M8X1.25X8	
22	112648	FOOTREST NUT	
23	112651	KEY CLAMP, 19MM	
24	112348	FOOTREST BUSHING OVAL TO ROUND	
25	113255	LEG STRAP 11"-12" BLK	

Pos.	Item Number	Description	Remarks
25	113256	LEG STRAP 13"-15" BLK	
25	113257	LEG STRAP 16"-18" BLK	
25	113258	LEG STRAP 19"-20" BLK	
26	114934	FTRST MOUNT KIT	
5A,8,17A,19	159582	ADAPTIVE MNT CENTER KIT 8 &10	
5B,8,17A,19	159583	ADAPTIVE MNT CENTER KIT 9 & 11	
5C,8,17A,19	159584	ADAPTIVE MNT CENTER KIT 12	
5D,8,17B,19	159585	ADAPTIVE MNT CENTER KIT 13	
5E,8,17B,19	159586	ADAPTIVE MNT CENTER KIT 14	
5F,8,17B,19	159587	ADAPTIVE MNT CENTER KIT 15	
5G,8,17B,19	159588	ADAPTIVE MNT CENTER KIT 16	
5H,8,17B,19	159589	ADAPTIVE MNT CENTER KIT 17	
5I,8,17B,19	159590	ADAPTIVE MNT CENTER KIT 18	
4,5A,7-11	159572	FLIP BACK CENTER KIT 8 & 10	
4,5B,7-11	159573	FLIP BACK CENTER KIT 9 & 11	
4,5C,7-11	159574	FLIP BACK CENTER KIT 12	
4,5D,7-11	159575	FLIP BACK CENTER KIT 13	
4,5E,7-11	159576	FLIP BACK CENTER KIT 14	
4,5F,7-11	159577	FLIP BACK CENTER KIT 15	
4,5G,7-11	159578	FLIP BACK CENTER KIT 16	
4,5H,7-11	159579	FLIP BACK CENTER KIT 17	
4,5I,7-11	159580	FLIP BACK CENTER KIT 18	
2,15,18	159951	ADAPTIVE SPACER SET 2"	Multiple fasteners included; refer to parts page
2,15,18	159952	ADAPTIVE SPACER SET 4"	Multiple fasteners included; refer to parts page
2,15,18	159953	ADAPTIVE SPACER SET 6"	Multiple fasteners included; refer to parts page

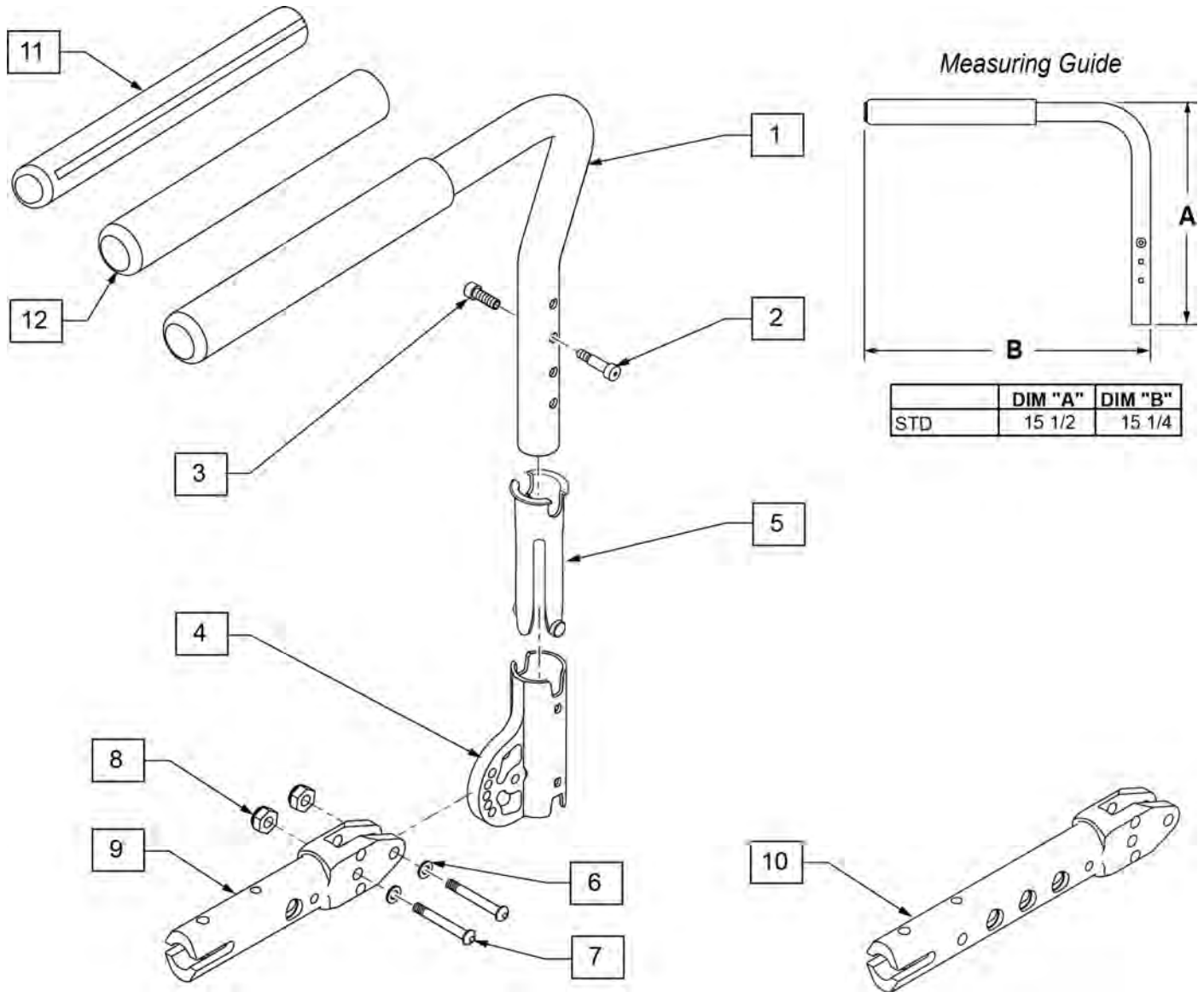

Pos.	Item Number	Description	Remarks
1-3	115040	PADDED S/A WITH BOLT ASSM L	LEFT - Std w/grip
1-3	115041	PADDED S/A WITH BOLT ASSM R	RIGHT - Std w/grip
2,3	108118	SEXBOLTS ARMREST KIT	PAIR
4	121916	RECEIVER MOUNT HD	
5	129069	SLEEVE, SWING AWAY ARMREST	
6-8	159591	PADDED SA ARM FASTENER SET	2 Each for 1 side
9	126832	ARMREST PADDED TUBULAR MT	Standard
10	134044	INSERT PADDED ARMREST DAB	For DAB
11	108315	GEL PAD S/A ARM	PAIR
12	920070	9X.940X.200 FOAM GRIP	1 grip for replacement only

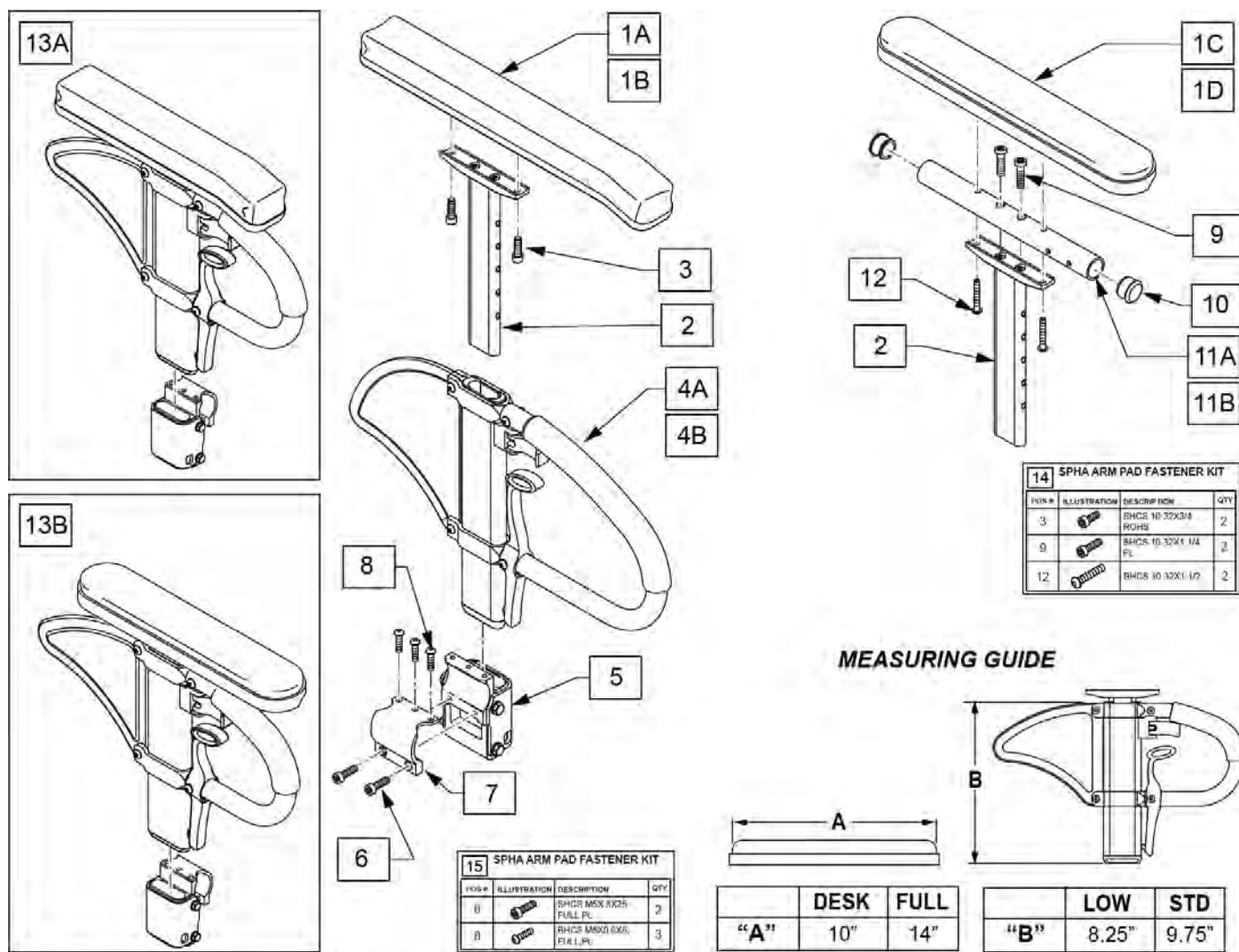


Pos.	Item Number	Description	Remarks
1A	104568-001	SPHA PAD DESK	CONTOUR DESK
1B	104568-002	SPHA PAD FULL	CONTOUR FULL
1C	561024	ARMREST PAD 10" DESK	CLASSIC DESK
1D	561031	ARMREST PAD 14" FULL	CLASSIC FULL
2	116312	SNGL POST UPPR ASSM LOW	LOW/SMALL
2	116313	SNGL POST UPPR ASSM STD	STANDARD
2,4A	104509-001	SPHA ARMREST ASSY STD	STANDARD
2,4B	104509-002	SPHA ARMREST ASSY LOW	LOW/SMALL
10	560735	7/8" ROUND TUBE PLUG	
11A	521812	HT ADJ PD TUBE 10IN DESK POWE	Desk
11B	521813	HT ADJ PD TUBE 14IN FULL POWE	Full
		Upper armrest	
1A-3	159596	SPHA PAD DESK CONT UPPER STD	1 side, lft or rt, standard
1B-3	159598	SPHA PAD FULL CONT UPPER STD	1 side, lft or rt, standard
1A-3	159597	SPHA PAD DESK CONT UPPER LOW	1 side, lft or rt, low
1B-3	159599	SPHA PAD FULL CONT UPPER LOW	1 side, lft or rt, low
1C,2,10,11A,12	159600	SPHA STD CLAS DESK UPPR R	
1C,2,10,11A,12	159601	SPHA STD CLAS DESK UPPR L	

Pos.	Item Number	Description	Remarks
1D,2,10,11A,12	159602	SPHA STD CLAS FULL UPPR R	
1D,2,10,11A,12	159603	SPHA STD CLAS FULL UPPR L	
1C,2,10,11B,12	159604	SPHA LOW CLAS DESK UPPR R	
1C,2,10,11B,12	159605	SPHA LOW CLAS DESK UPPR L	
1D,2,10,11B,12	159606	SPHA LOW CLAS FULL UPPR R	
1D,2,10,11B,12	159607	SPHA LOW CLAS FULL UPPR L	
		Complete Armrest, no receiver	
1A,2,3,4A	108595	SPHA STD CNTUR DESK PAD	1 side, left or right
1B,2,3,4A	108597	SPHA STD CNTUR FULL PAD	1 side, left or right
1A,2,3,4B	108596	SPHA LOW CNTUR DESK PAD	1 side, left or right
1B,2,3,4B	108598	SPHA LOW CNTUR FULL PAD	1 side, left or right
1C,2,3,4A,8-11A	108599	SPHA STD CLASIC DESK PAD R	Right-no receiver
1C,2,3,4A,8-11A	108603	SPHA STD CLASIC DESK PAD L	Left-no receiver
1D,2,3,4A,8-11B	108601	SPHA STD CLASIC FULL PAD R	Right-no receiver
1D,2,3,4A,8-11B	108605	SPHA STD CLASIC FULL PAD L	Left-no receiver
1C,2,3,4B,8-11A	108600	SPHA LOW CLASIC DESK PAD R	Right-no receiver
1C,2,3,4B,8-11A	108604	SPHA LOW CLASIC DESK PAD L	Left-no receiver
1D,2,3,4B,8-11B	108602	SPHA LOW CLASIC FULL PAD R	Right-no receiver
1D,2,3,4B,8-11B	108606	SPHA LOW CLASIC FULL PAD L	Left-no receiver
		Complete armrest with receiver	
13A	159608	SPHA STD CNTUR DESK ASSM	
13A	159609	SPHA STD CNTUR FULL ASSM	
13A	159610	SPHA LOW CNTUR DESK ASSM	
13A	159611	SPHA LOW CNTUR FULL ASSM	
13B	159612	SPHA STD CLASSIC DESK ASSM R	
13B	159613	SPHA STD CLASSIC DESK ASSM L	
13B	159614	SPHA STD CLASSIC FULL ASSM R	
13B	159615	SPHA STD CLASSIC FULL ASSM L	
13B	159616	SPHA LOW CLASSIC DESK ASSM R	
13B	159617	SPHA LOW CLASSIC DESK ASSM L	
13B	159618	SPHA LOW CLASSIC FULL ASSM R	
13B	159619	SPHA LOW CLASSIC FULL ASSM L	
14	159620	SPHA ARM PAD FASTENER KIT	
15	159621	SPHA 7S RECEIVER FASTENER KIT	
5	111691	SPHA RECEIVER ASSY	
5-8	115031	SPHA RECEIVER ASSM	

Note: The Yamaha Xtender is not transit approved and can not be added to a chair that is transit approved.

Note: retro kit hardware 158647 is used for chairs with s/n after R4-023230.


Note: Xtender Retro Fit Parts (7 Series) is discontinued; for complete assembly, see GEN 2 XTENDER RETRO FIT PARTS (7 SERIES) parts page.

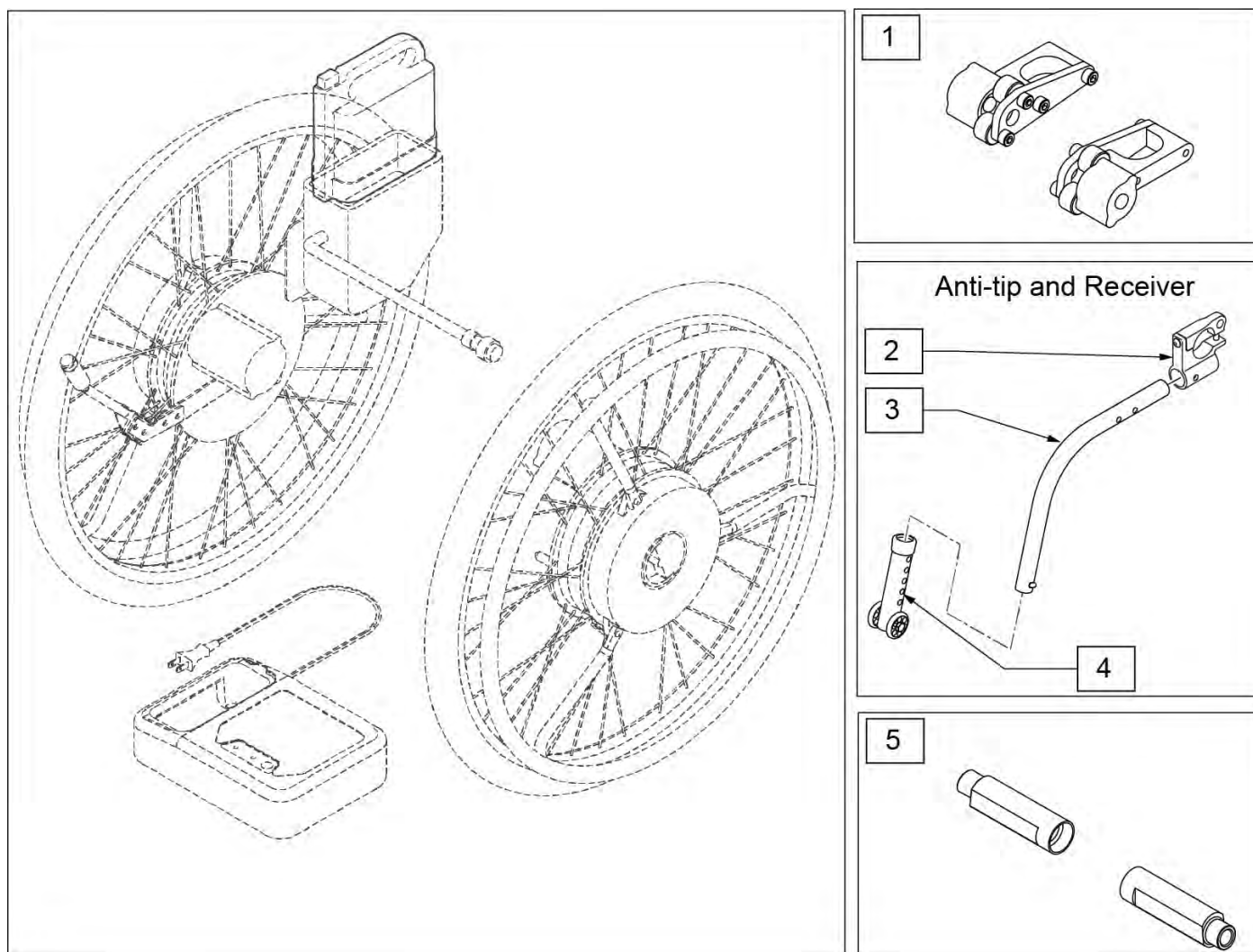

Pos.	Item Number	Description	Remarks
1	158647	4X XTENDER RETRO FIT HDW Q7	Required to retro fit
2	114440	ANTI TIP REC Q7 KIT	2 required if no anti-tips on chair
3	104221-004	ANTI TIP ARM MED RGT	Right; required to retro fit
3	104221-003	ANTI TIP ARM MED LFT	Left; required to retro fit
4	104640	ANTI TIP ARM ADJ. LONG	Required to retro fit
5	128077	CAMBER PLUG 0 DEG XTENDER	2 required if replacing on chair with 3 or 6 degree camber

Note: Fits Q7 chairs built after S/N R4-023230 only.


Note: If replacing or retrofitting an Xtender, please contact Customer Service to ensure compatibility.

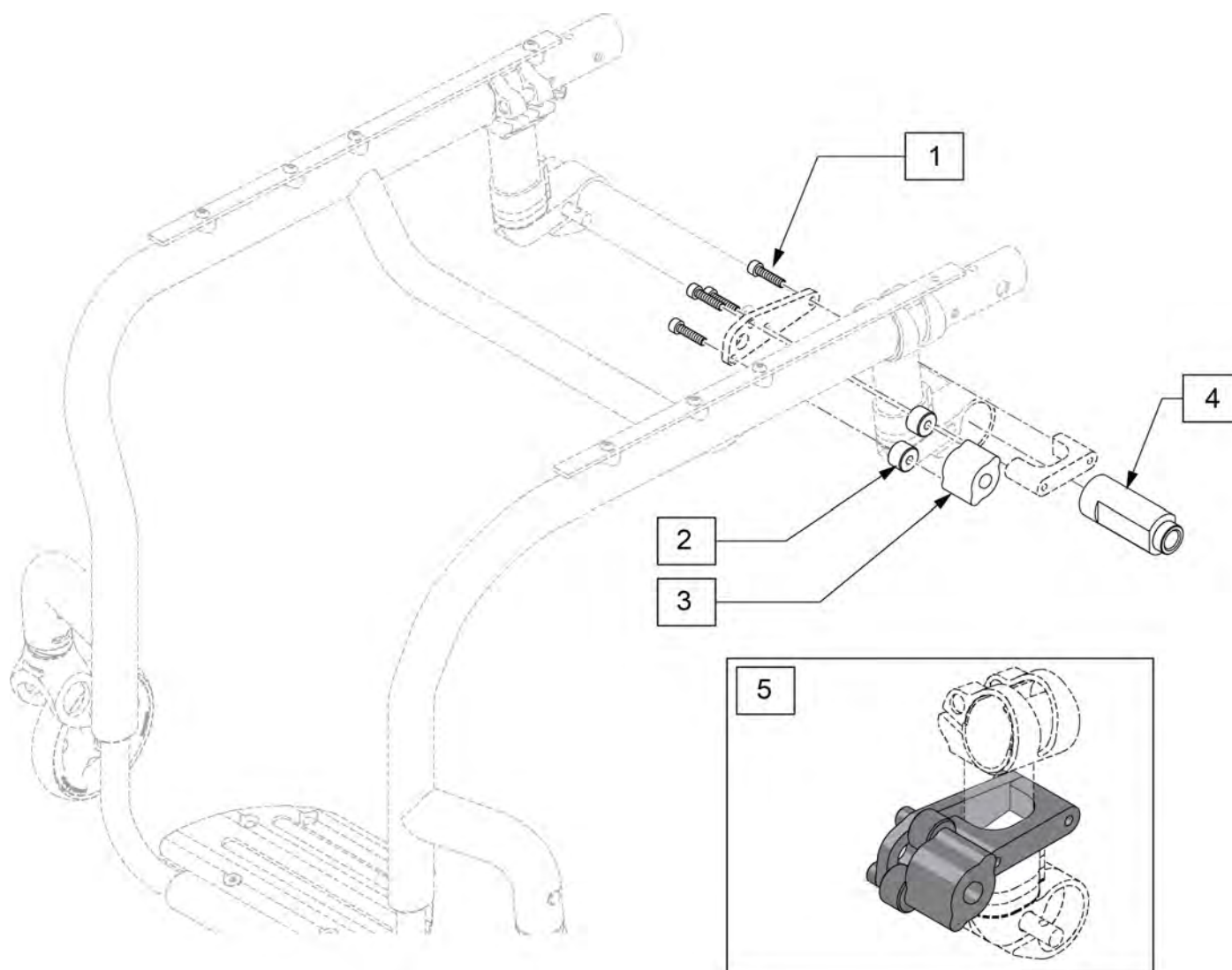


Pos.	Item Number	Description
1	102263-003	SHCS M6 X 1 X 25 W/PL INORG
2	330031	SUSPENSION SPACER 1/2"
3	541011	BRACKET ENGINE 1
4	128077	CAMBER PLUG 0 DEG XTENDER
5	128122	PIN HOLDER KIT

Note: If replacing or retrofitting an Xtender, please contact Customer Service to ensure compatibility.

Note: Battery Type :NiMH- Nickel Metal Hydride, Capacity: 24V x 6.7Ah , Weight 2.9 [kg], Fuse: 20A.

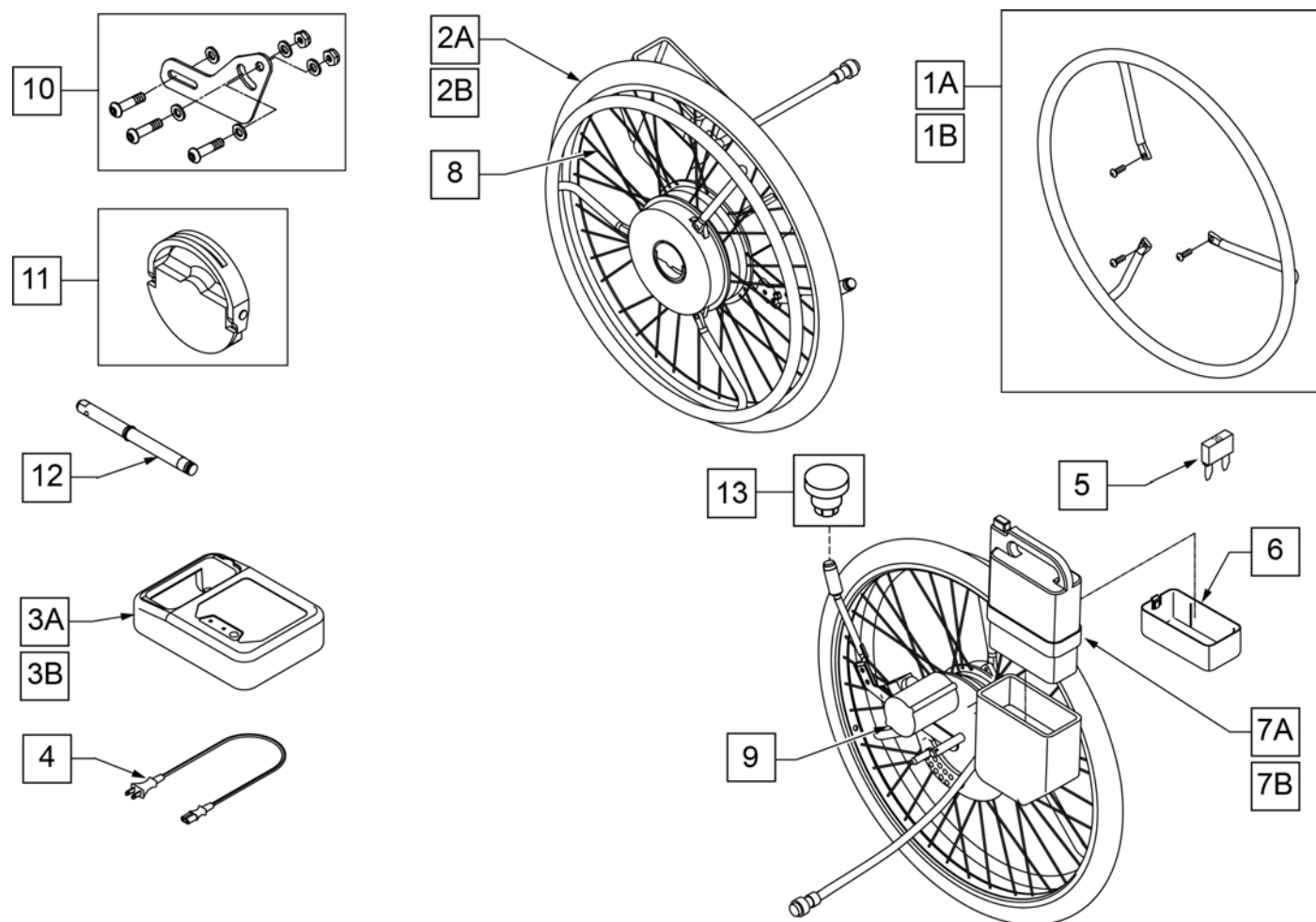


Pos.	Item Number	Description	Remarks
1A	242232	HNDRM W/O COATING 24", 21" DIA	24" Stainless Steel Handrim w/3 fixing bolts
1B	242234	HNDRM W/COATING 24", 21" DIA	24" Plastic Coated Handrim w/3 fixing bolts
2A	385300	24" PNEUMATIC TIRE	
2B	382325	24" TUBE	
3A,4	114474	CHARGER YAMAHA W/ ADDRESS LBL	Nickel Battery Charger
3B,4	120710	BATTERY CHARGER YAMAHA	Lithium Battery Charger
4	134121	PWR CORD BATT CHARGER YAMAHA	Used on Nickel Battery Charger only
5	138219	FUSE 20A BLADE	
5,6,7A	134000	JWX-2 NICKEL BATTERY BLACK	Nickel Battery
5,6,7B	134002	JWX-2 LI-ION BATTERY BLACK	Lithium Battery
8	133108	SPOKE SET 24" YAMAHA	Includes 36 spokes
9	490013	MOTOR COVER YAMAHA	
10	921763	XTNDR ADJ BRCKT KIT (W/HDWR)	
11	107011	YAMAHA PLATE ASSM KIT	
12	541155	XTNDR QUICK RELEASE AXLE	
13	108273	PWR BUTTON SWITCH	

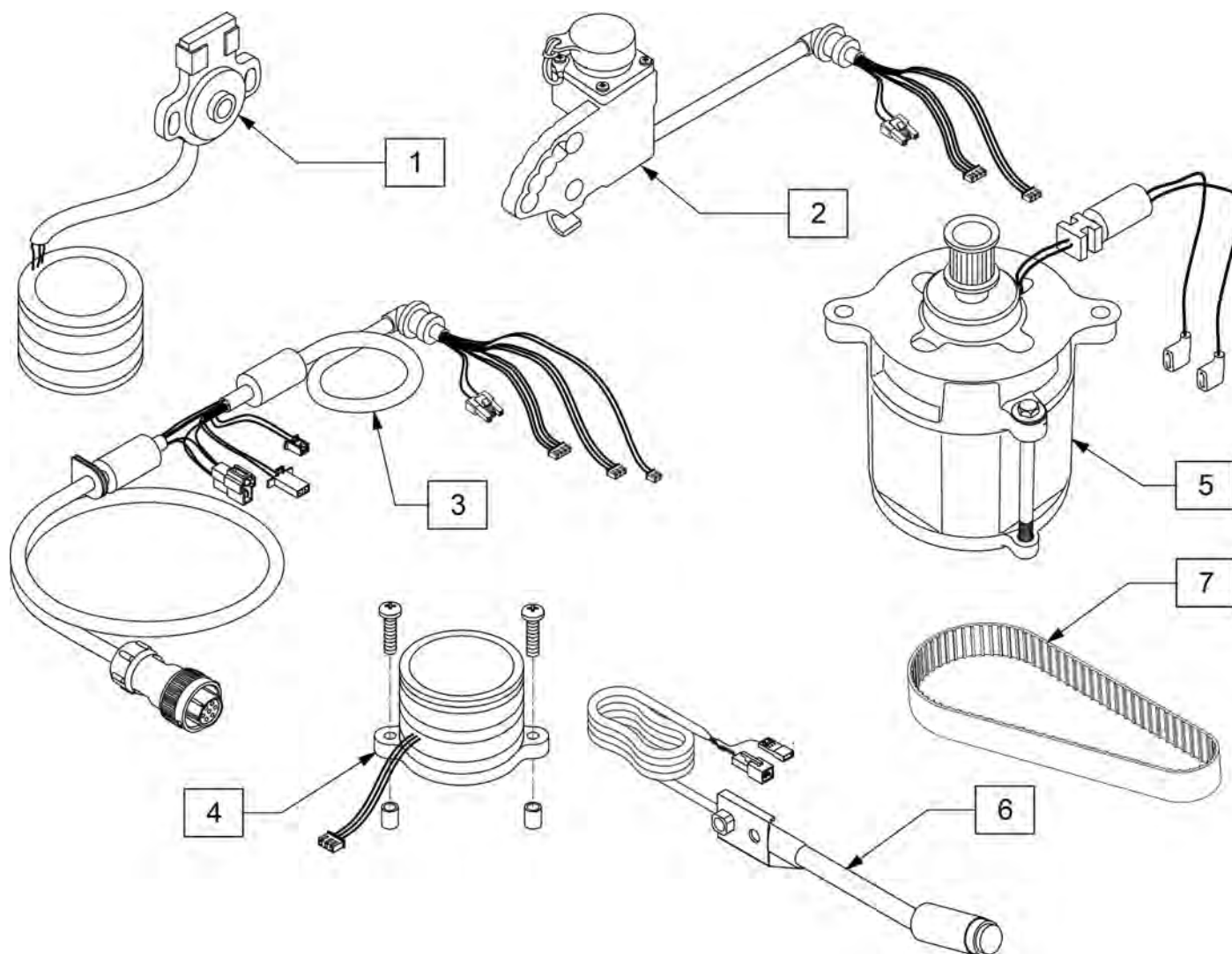


Pos.	Item Number	Description
1	107008	YAMAHA SENSOR KIT
2	107005	YAMAHA RECEPTICAL KIT
3	107010	YAMAHA COUPLER HRNS
4	107007	YAMAHA ROTARY TRANS KIT
5	107009	YAMAHA MOTOR REPLACEMENT KIT
6	107012	YAMAHA SWITCH KIT
7	107006	YAMAHA DRIVE BELT KIT

Note: The Yamaha Xtender is not transit approved and can not be added to a chair that is transit approved.

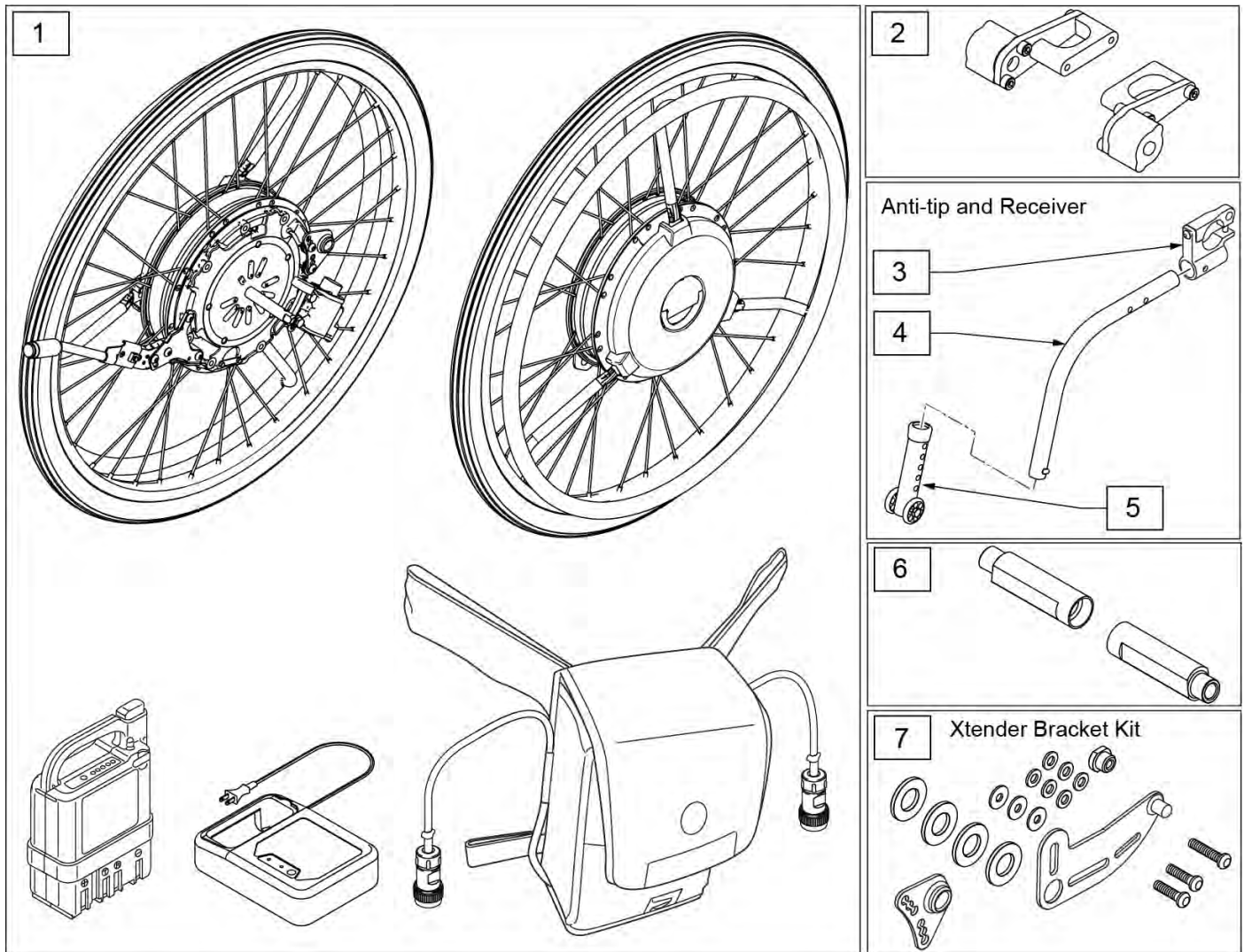


Pos.	Item Number	Description	Remarks
1,5	133982	JWX-2 XTENDER KIT	24", Nickle Batt,SS Handrim
1,5	133983	JWX-2 24IN LI SS	24", Lith Ion Batt,SS Handrim
1,5	133984	JWX-2 24IN NI PLS	24", Nickle Batt, Plastic Handrim
1,5	133985	JWX-2 24IN LI PLS	24", Lith Ion Batt,Plastic Handrim
1,5	133986	JWX-2 22IN NI SS	22", Nickle Batt,SS Handrim
1,5	133987	JWX-2 22IN LI SS	22", Lith Ion Batt,SS Handrim
1,5	133988	JWX-2 22IN NI PLS	22", Nickle Batt, Plastic Handrim
1,5	133989	JWX-2 22IN LI PLS	22", Lith Ion Batt,Plastic Handrim
2	158647	4X XTENDER RETRO FIT HDW Q7	Required to retro fit
3	114440	ANTI TIP REC Q7 KIT	2 required if no anti-tips on chair
4	104221-004	ANTI TIP ARM MED RGT	Right
4	104221-003	ANTI TIP ARM MED LFT	Left
5	104640	ANTI TIP ARM ADJ. LONG	
6	128077	CAMBER PLUG 0 DEG XTENDER	2 required if replacing on chair with 3 or 6 degree camber


Pos.	Item Number	Description	Remarks
7	241917	JWX-2 BRACKET KIT RH	Right
7	241916	JWX-2 BRACKET KIT LH	Left
1,2,7	161417	24" JWX-2 PNEU HR SS NI	24", Nickle Batt,SS Handrim
1,2,7	161418	24" JWX-2 PNEU HR SS LI	24", Lith Ion Batt,SS Handrim
1,2,7	161419	24" JWX-2 PNEU HR PLS NI	24", Nickle Batt, Plastic Handrim
1,2,7	161420	24" JWX-2 PNEU HR PLS LI	24", Lith Ion Batt,Plastic Handrim
1,2,7	161421	22" JWX-2 PNEU HR SS NI	22", Nickle Batt,SS Handrim
1,2,7	161422	22" JWX-2 PNEU HR SS LI	22", Lith Ion Batt,SS Handrim
1,2,7	161423	22" JWX-2 PNEU HR PLS NI	22", Nickle Batt, Plastic Handrim
1,2,7	161424	22" JWX-2 PNEU HR PLS LI	22", Lith Ion Batt,Plastic Handrim

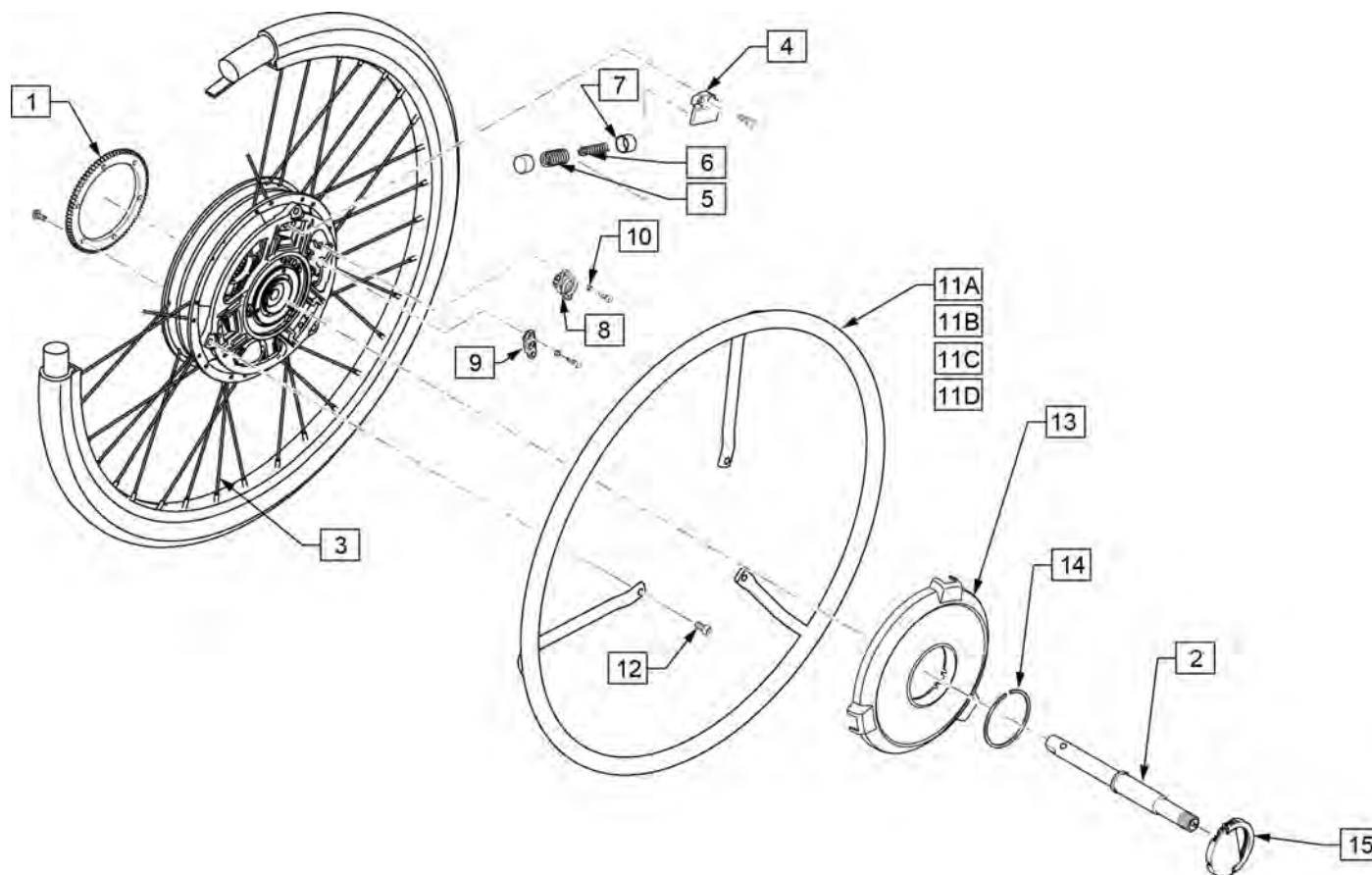


Pos.	Item Number	Description	Remarks
1	242261	GEAR, WHEEL 3	
2	242237	RR AXLE ASSY	
3	242235	RR SPOKE SET 22"	for 22" Wheel; 36 spokes & 36 nipples
3	242236	RR SPOKE SET 24"	for 24" Wheel; 36 spokes & 36 nipples
4	242255	RETAINER	
8	242252	DAMPER	
9	242270	ARM, MOVING	
10	242272	COLLAR	To fix Damper (item 8) & Arm (item 9)
11A,12	242231	HNDRM W/O COATING 22", 19" DIA	22" Stainless Steel Handrim w/3 fixing bolts
11B,12	242233	HNDRM W/ COATING 22", 19" DIA	22" Plastic Coated Handrim w/3 fixing bolts
11C,12	242232	HNDRM W/O COATING 24", 21" DIA	24" Stainless Steel Handrim w/3 fixing bolts
11D,12	242234	HNDRM W/ COATING 24", 21" DIA	24" Plastic Coated Handrim w/3 fixing bolts
13	134003	JWX-2 WHEEL CAP ASSEMBLY	Black, Hub cover w/decal
14	242256	SPRING CLIP 42MM	
15	242238	YAMAHA PLATE ASSY	Handle Plate for the unit detaching
5-9	242244	SPRING, LEVER RETURN E	

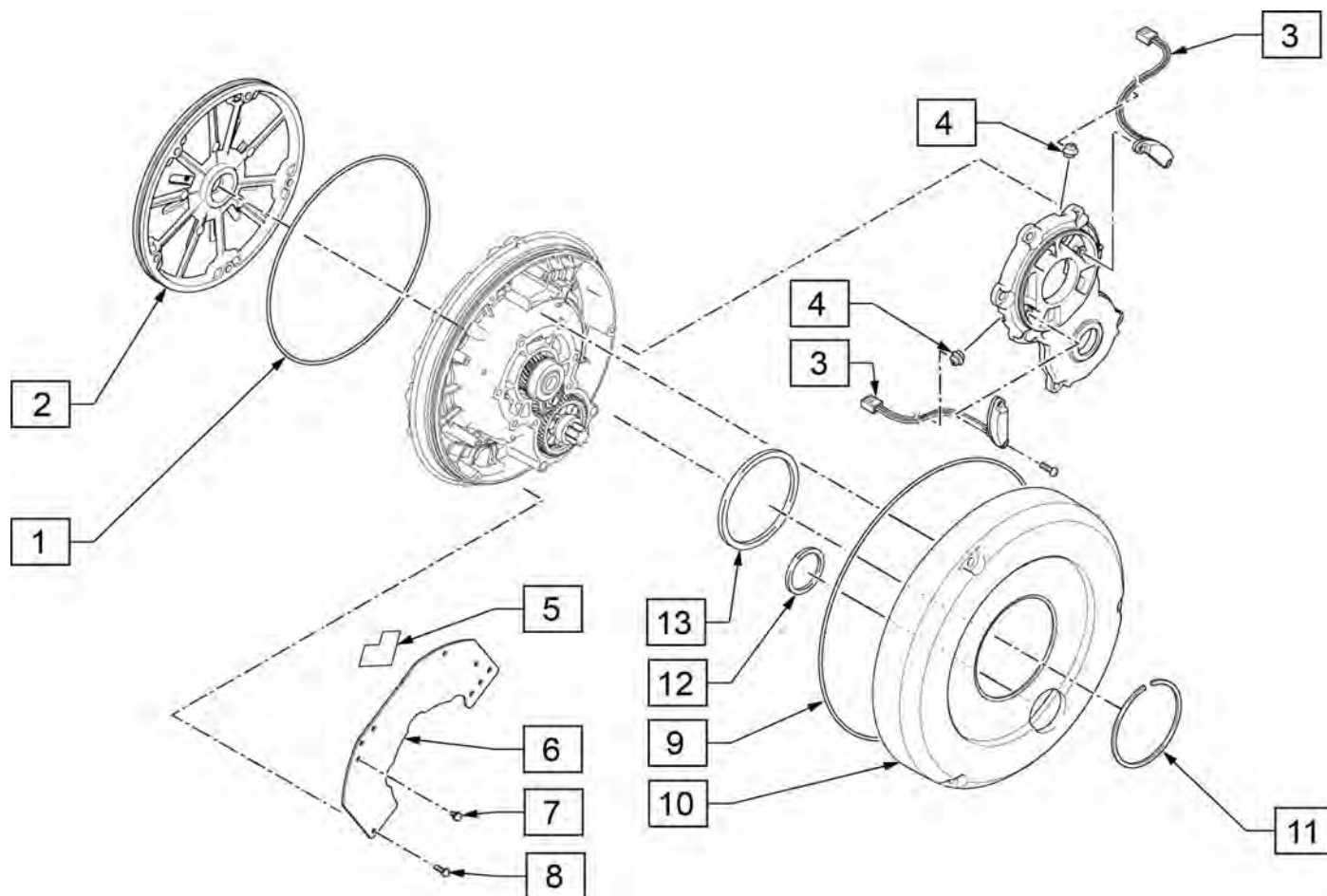


Pos.	Item Number	Description	Remarks
1	242250	JWX-2 O-RING 1	
2	242230	YAMAHA HSG ASSY	Unit Back Plate;
3,4	242242	SENSOR ASSY	
5-8	242273	PRINT BOARD ASSY 1	Pair, Main Electric Board
9	241969	JWX-2 SEAL 1	O-ring for Transparent Cover
10	242240	YAMAHA COVER, TRANSMISSION	Transparent Plastic Cover
11	242260	CLIP	C clip spring to fix Transparent Cover
12	242258	GASKET 1	urethane flat O-ring, small
13	242259	GASKET 2	urethane flat O-ring, large

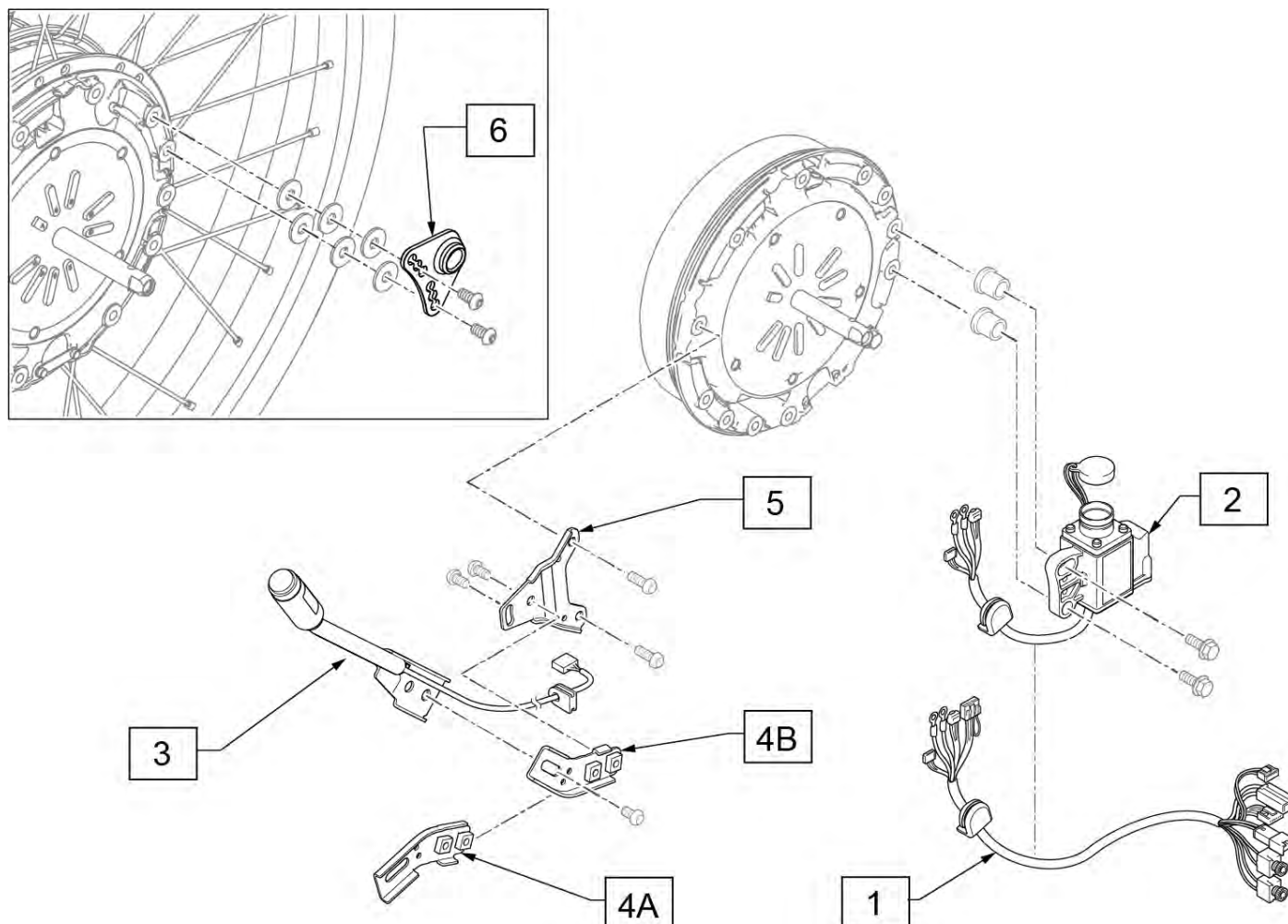


Pos.	Item Number	Description	Remarks
1	242271	WIRE 6	Connecting Wire from Battery Seat to Drive Unit
2	133999	JWX-2 WIRE HARNESS ASSY RH	Right Side Wheel
2	242245	JWX-2 WIRE HARNESS ASSY LH	Left Side Wheel
3	133995	JWX-2 MAIN SWITCH SUB ASSY	Power Switch Main Unit
4A	133993	JWX-2 POWER SWITCH ANGLE A	Angle Bracket (A)
4B	133994	JWX-2 POWER SWITCH ANGLE B	Angle Bracket (B)
5	133992	JWX-2 XTENDER "A" BRACKET	
6	241957	JWX-2 XTENDER PLATE LOCK	

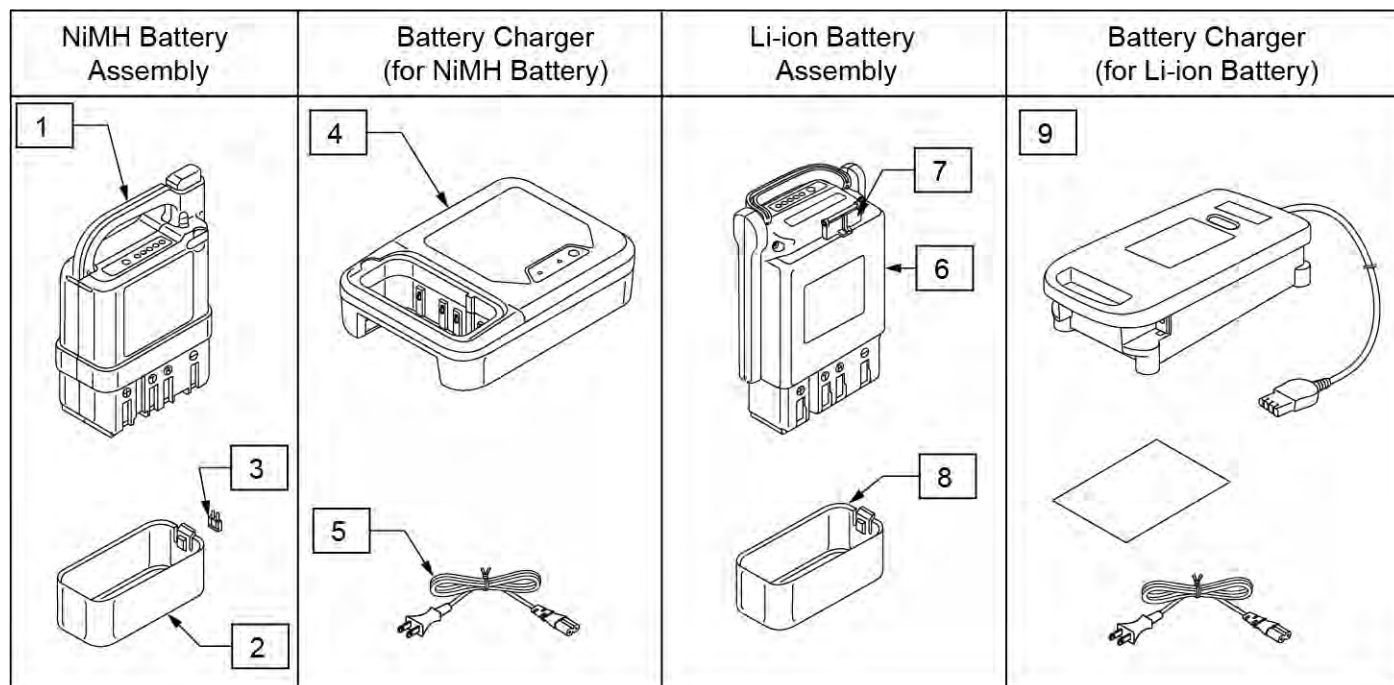

Pos.	Item Number	Description
1-3	134000	JWX-2 NICKEL BATTERY BLACK
4	134001	JWX-2 BATTERY CHARGER NI
5	134121	POWER CORD BATT CHARGER YAMAHA
6-8	134002	JWX-2 LI-ION BATTERY BLACK
9	120710	BATTERY CHARGER YAMAHA

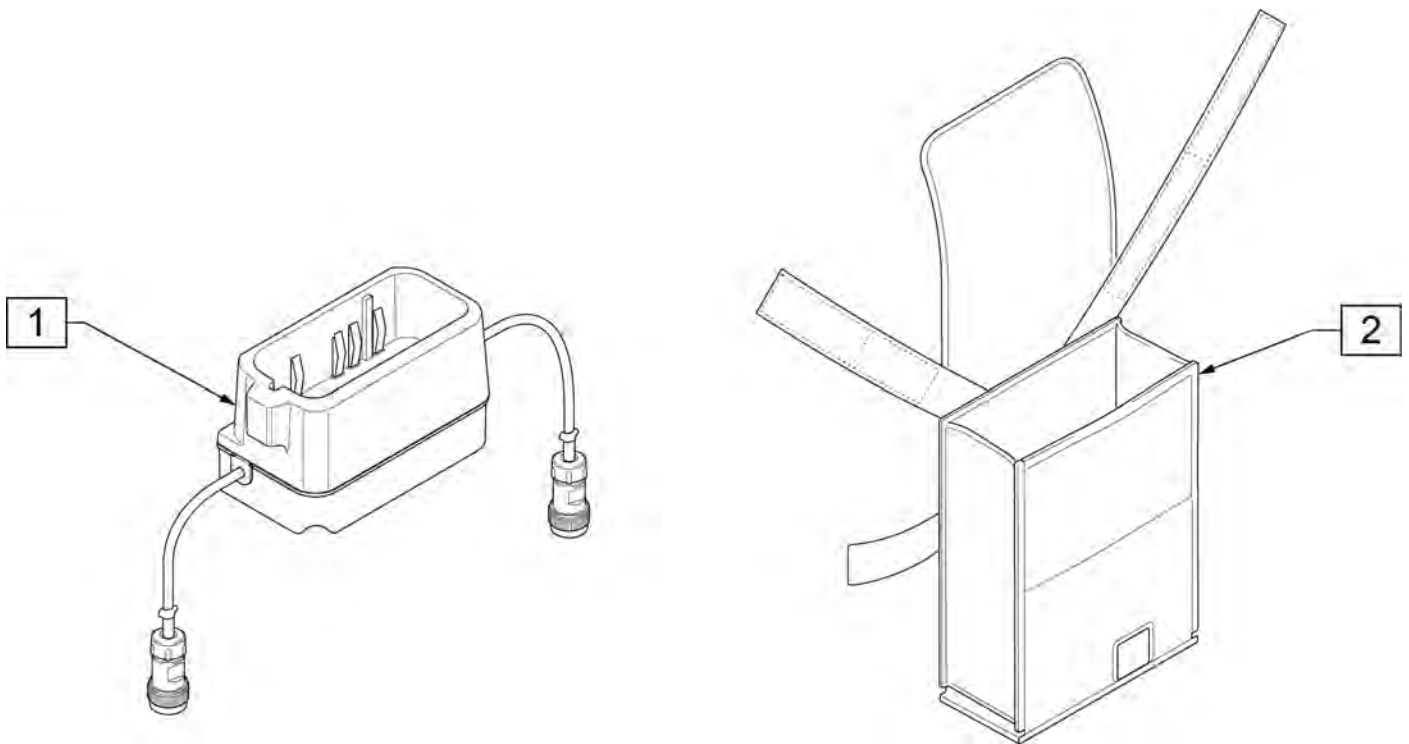


Pos.	Item Number	Description	Remarks
1	241919	JWX-2 BATTERY BOX ASSEMBLY	
2	241918	JWX-2 HANGER BAG	Battery Bag

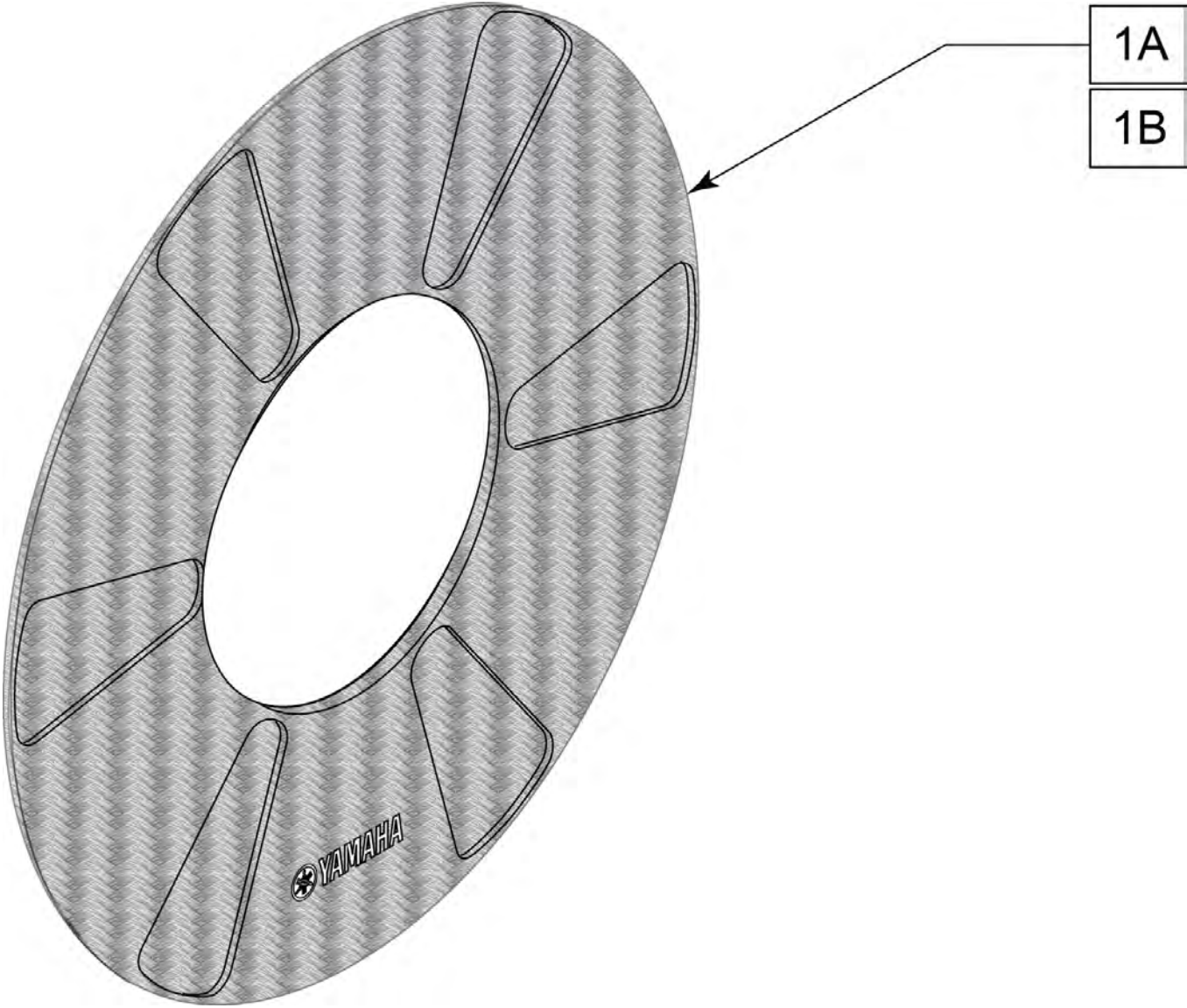


Pos.	Item Number	Description	Remarks
1A	243342	JWX-2 SPOKE GUARD 22 IN	Each; actual Outside Diameter is 18"
1B	243343	JWX-2 SPOKE GUARD 24 IN	Each; actual Outside Diameter is 20"

NOTE: Right hand side shown.

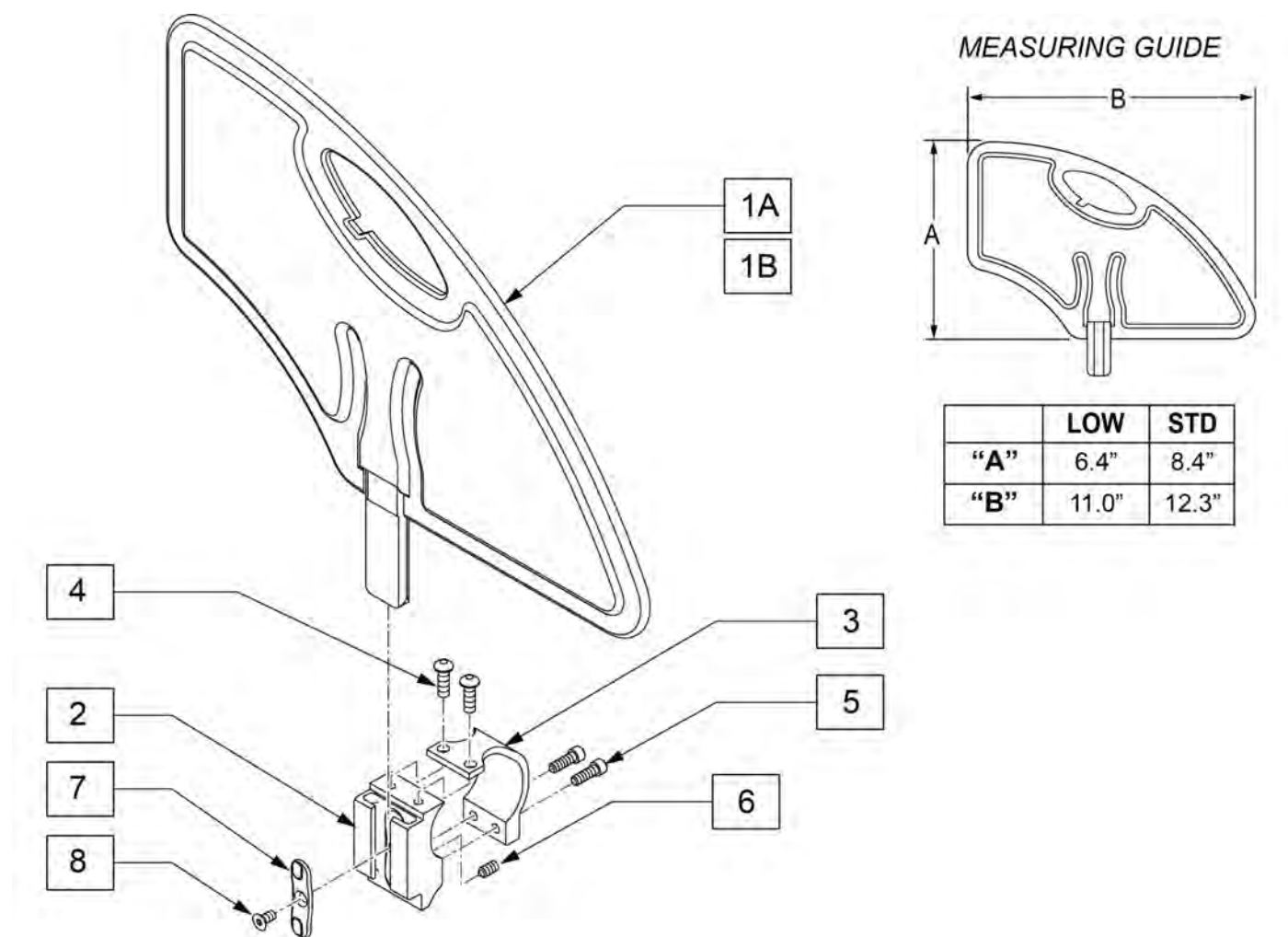


Pos.	Item Number	Description	Remarks
1A	104164-001	SIDE GUARD ASSY STD	Standard
1B	104164-002	SIDE GUARD ASSY SMALL	Low
3	112695	PLASTIC SG INNER	
4	103184-002	BHCS,M5X0.8X8,FULL,PL	
5	112697	SHCS,M5X0.8X20,FULL,PL	
6	112698	SHSS M5X0.8X8 NP	
7	108780	RECEIVER SG PLASTIC INSERT	
8	107135	FHCS #6-32 X 1/4	
2-8	115046	PLST SDE GRD REC ASSM	
2,6-8	111695	ASSY PLASTIC SG RECEIVER	
1A-8	115047	PLST SDE GRD REG W/ REC KIT (PR)	Pair-Standard
1B-8	115048	PLST SDE GRD LOW W/ REC KIT (PR)	Pair- Low

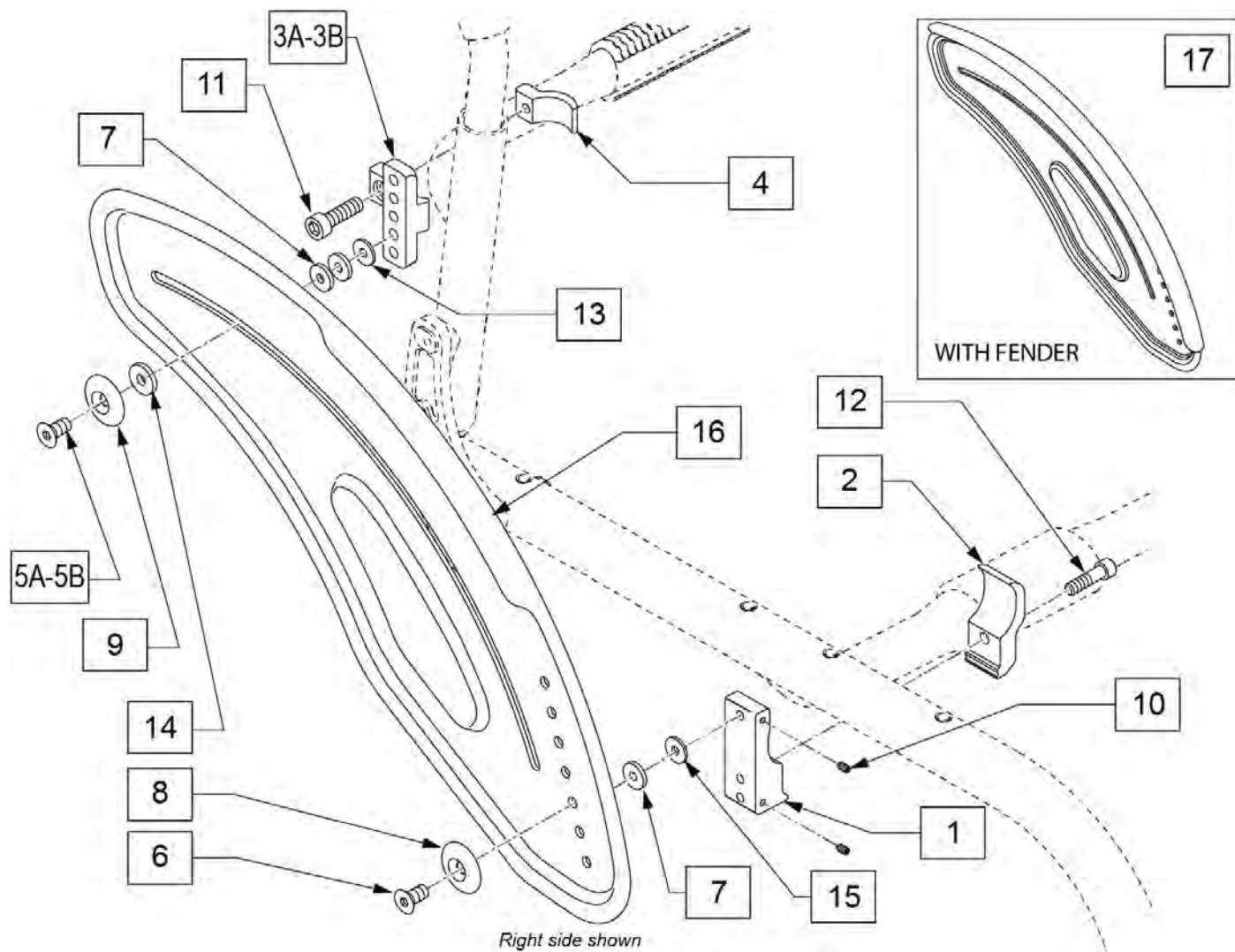

Pos.	Item Number	Description	Remarks
1	111844	MOUNT SIDE GUARD OVAL FRM	
2	111693	SPHA RECEIVER CLAMP	
3A	113299	AL SIDE GUARD REAR REC	prior to s/n RZ-005898
3B	118200	AL S/G REAR REC M5	after s/n RZ-005898
4	113301	AL SIDE GUARD CLAMP	
5A	113360	FH C/S M6 x 1 x 25 FULL PL	prior to s/n RZ-005898
5B	118450	FHCS, M5X0.8X25, FULL, PL	after s/n RZ-005898
6	102265-002	M6 x 1 x 20 FH C/S FULL PL	
7	561026	3/4" WASHER 1/4" HOLE	
8	000 138 017	ALUMINUM SPACER	
9	000 138 014	PLASTIC SPACER	
10	113319	SHSS M4 x 0.7 x 6 FULL NP	
11	102263-005	SHCS, M6 x 1 x 20, FULL, PL	
12	102263-003	SHCS, M6 x 1 x 25, FULL, PL	
13	011111	WASHER M6 X 18 REGULAR BLK	
14	118202	SIDE GUARD SLIDE BSHNG M5	
15	561112	3/4 OD X .318 ID X .25 NYLON	
16	115052S	SIDE GUARD AL W/O FENDER LH	Left
16	115053S	SIDE GUARD AL W/O FENDER RH	Right


Pos.	Item Number	Description	Remarks
17	115050S	SIDE GUARD AL W/ FENDER LH	Left
17	115051S	SIDE GUARD AL W/ FENDER RH	Right
1-12	115049	SDE GRD REC ASSM	
1-15,17	115054S	SDE GRD AL W/ FNDR KIT (PR)	Pair
1-16	115055S	SDE GRD AL W/O FNDR KIT (PR)	Pair

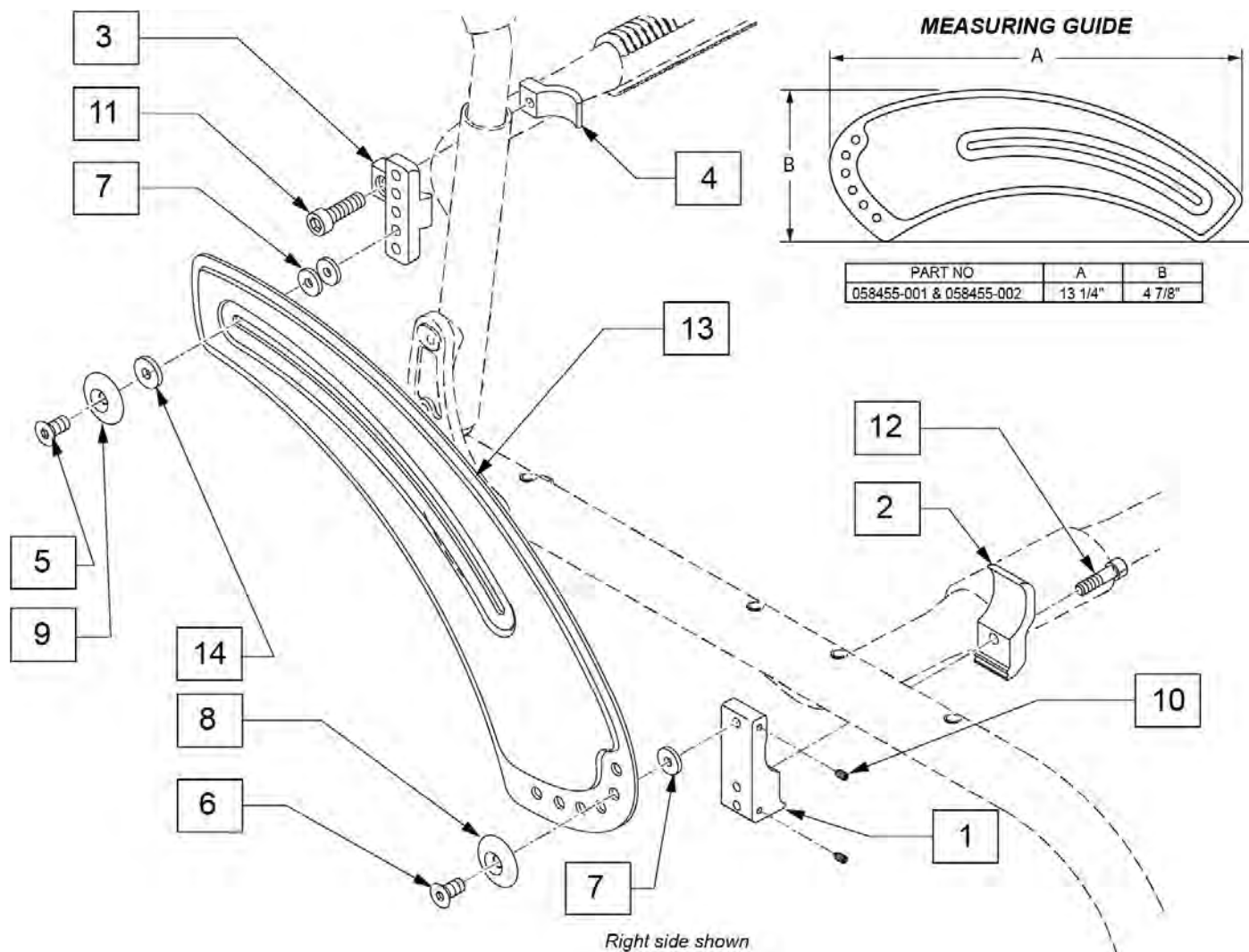

Pos.	Item Number	Description	Remarks
1	111844	MOUNT SIDE GUARD OVAL FRM	
2	111693	SPHA RECEIVER CLAMP	
3	113299	AL SIDE GUARD REAR REC	
4	113301	AL SIDE GUARD CLAMP	
5	113360	FHCS, M6 x 1 x 25 FULL, PL	
6	102265-002	FHCS, M6 x 1 x 20 FULL PL	
7	561026	3/4" WASHER 1/4" HOLE	
8	000 138 017	ALUMINUM SPACER	
9	000 138 014	PLASTIC SPACER	
10	113319	SHSS M4 x 0.7 x 6 FULL NP	
11	102263-005	SHCS, M6 x 1 x 20, FULL, PL	
12	102263-003	SHCS, M6 x 1 x 25, FULL, PL	
13	058455-001	SIDEGUARD CARBON	LEFT
13	058455-002	SIDEGUARD CARBON	RIGHT
14	115220	SIDE GUARD SLIDE BUSHING	
1-12	115049	SDE GRD REC ASSM	
1-13	115056	SIDE GUARD CARBON KIT (PR)	Pair

Note: some items listed may not be available with every chair model or in conjunction with another chair feature.

NOTE: when mounting Positioning Belts to the 5R, GT or GTI, it is recommended to attach the belt by using the Backrest Support Bracket Hardware.


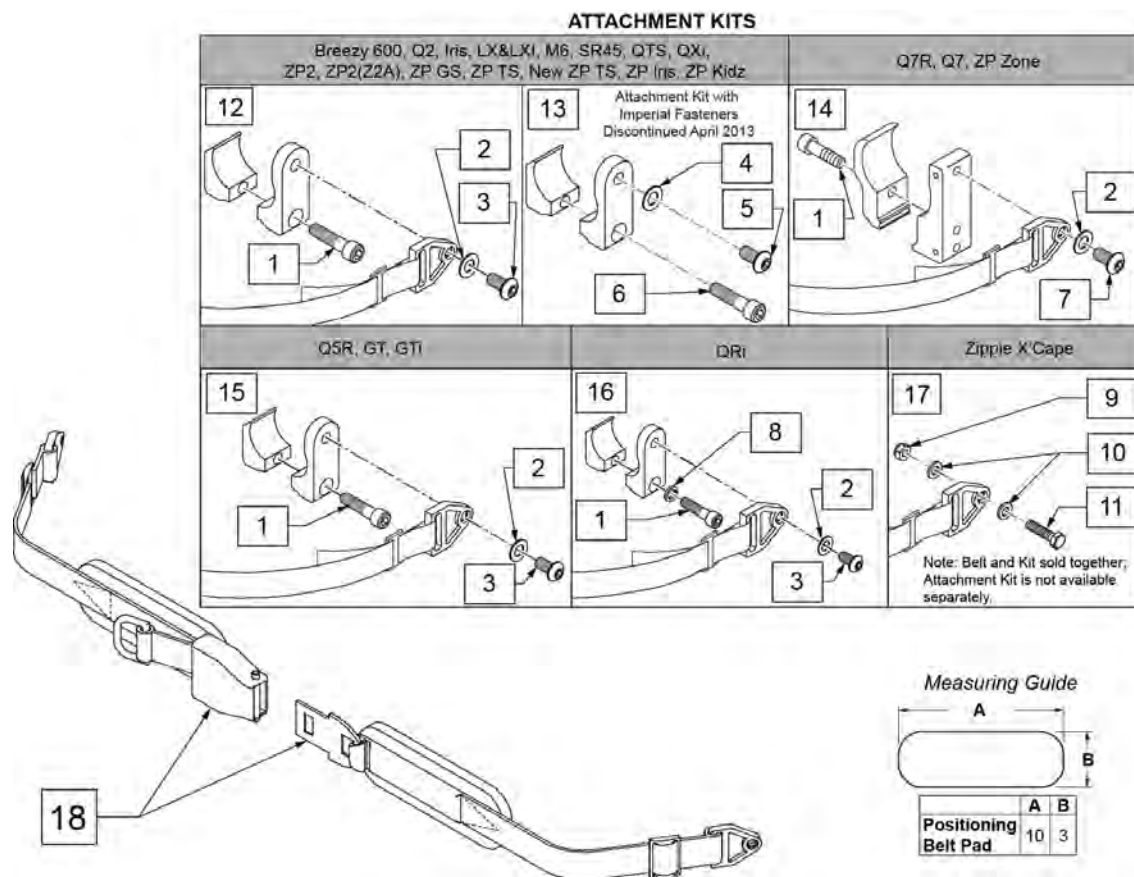


Pos.	Item Number	Description	Remarks
1	102263-003	SHCS M6 X 1 X 25 W/PL INORG	Metric
2	250104	WASHER FLAT M6	Metric
3	103185-005	BHCS M6 X 1 X 12 FULL PL	Metric
4	106943	WASHER FLAT AN 1/4 ROHS	Imperial (standard)
5	226154	1/4-20 X 1/2 BH C/S W/PATCH	Imperial (standard)
6	221116	1/4-20 X 1 SOC HD BLK	Imperial (standard)
7	230113	BHCS M6 X 1 X 12	
8	050129-001	LOCKWASHER HIGH COLLAR M6	
9	104381	NUT HEX NYLOCK M6 X 1.0	
10	107383	WASHER FLAT M6	
11	102268-002	HHCS M6 X 1 X 35 BLK	
12	120750	PSTN BELT CLAMP ASSM PR	Metric; Pair
13	900116	POSITION BELT ATTACHMENT KIT	Set of 2; Imperial (standard)
14	116268	OVAL CLAMP ASSM KIT	Kit contains 1 Clamp
15	159122	POSITIONING BELT CLAMPS PR	Pair
16	123534	POS BELT CLAMP KIT RPO	Set of 2

Pos.	Item Number	Description	Remarks
18	450418-QU	BELT 2" AIRCRAFT BUCKLE	Max length of belt 64"
18	450423-QU	BELT 2" AIRCRAFT BUCKLE W/PAD	Max length of belt 64"
18	450424	BELT 2" AIRCRAFT BKL LONG W/PAD	Max length of belt 72"
18	450425	BELT 2" AIRCRAFT BKL LONG	Max length of belt 72"
18	450419-QU	BELT 2" AIRCRAFT BKL W/QUAD LP	Max length of belt 64"
18	468419	BELT 2" AIRCRAFT BKL W/QD LPCZ	Max length of belt 64"
17,18	136464	AIRCRAFT HIP BELT 2" KIT	ZP Xcape: includes Belt & Attachment Kit
17,18	136465	AIRCRAFT HIP BELT 2" PAD KIT	ZP Xcape: includes Belt & Attachment Kit


NOTE: when mounting Positioning Belts to the 5R, GT or GTi, it is recommended to attach the belt by using the Backrest Support Bracket Hardware.

ATTACHMENT KITS

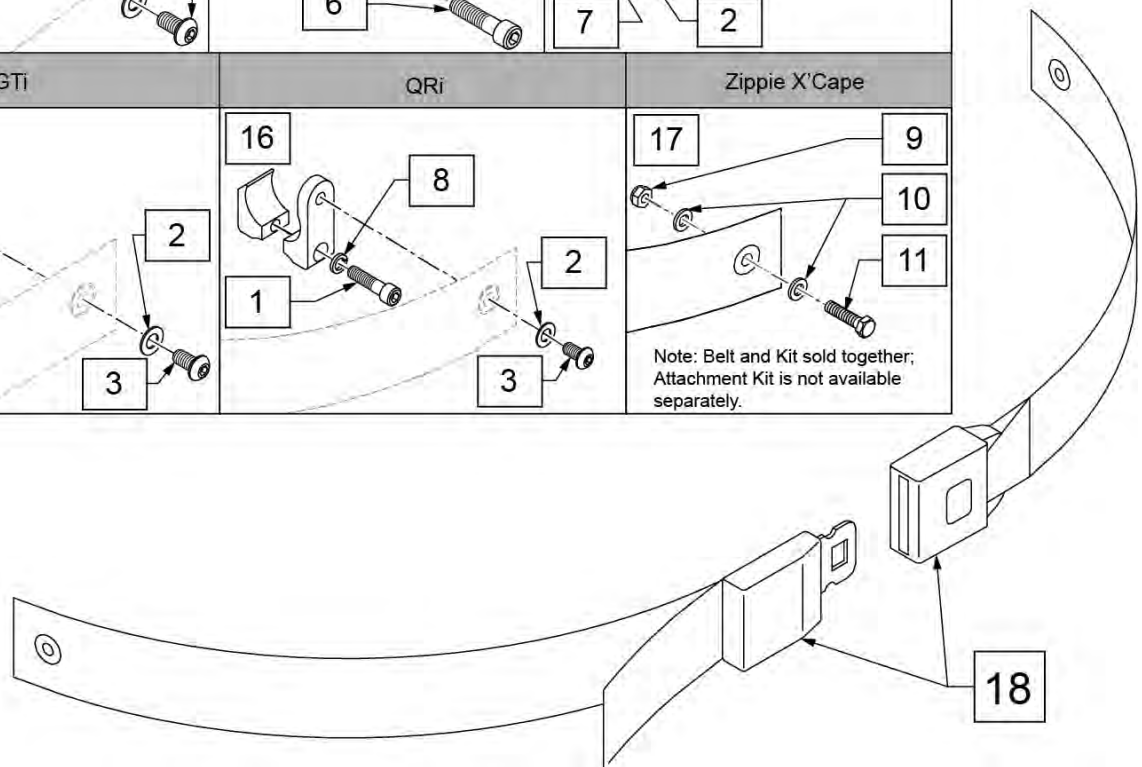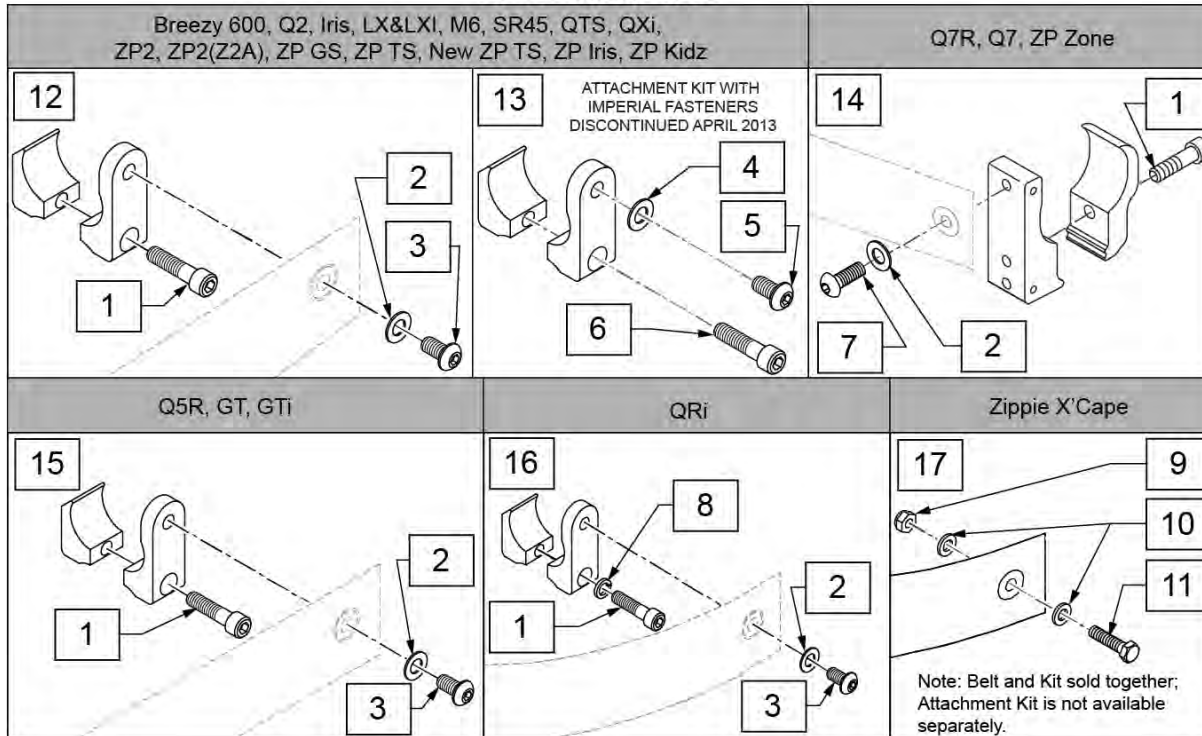


Pos.	Item Number	Description	Remarks
1	102263-003	SHCS M6 X 1 X 25 W/PL INORG	Metric
2	250104	WASHER FLAT M6	Metric
3	103185-005	BHCS M6 X 1 X 12 FULL PL	Metric
4	106943	WASHER FLAT AN 1/4 ROHS	Imperial (standard)
5	226154	1/4-20 X 1/2 BH C/S W/PATCH	Imperial (standard)
6	221116	1/4-20 X 1 SOC HD BLK	Imperial (standard)
7	230113	BHCS M6 X 1 X 12	
8	050129-001	LOCKWASHER HIGH COLLAR M6	
9	104381	NUT HEX NYLOCK M6 X 1.0	
10	107383	WASHER FLAT M6	
11	102268-002	HHCS M6 X 1 X 35 BLK	
12	120750	PSTN BELT CLAMP ASSM PR	Metric; Pair

Pos.	Item Number	Description	Remarks
13	900116	POSITION BELT ATTACHMENT KIT	Set of 2; Imperial (standard)
14	116268	OVAL CLAMP ASSM KIT	Kit contains 1 Clamp
15	159122	POSITIONING BELT CLAMPS PR	
16	123534	POS BELT CLAMP KIT RPO	Set of 2
18	750276	1 1/2" POSITION BELT	AUTOSTYLE-64" Max length of belt.
16,18	124269	AUTO POS BELT W/CLAMPS RPO	QRI: includes Belt & Attachment Kit
17,18	136463	AUTO HIP BELT 1.5" KIT	ZP Xcape: includes Belt & Attachment Kit

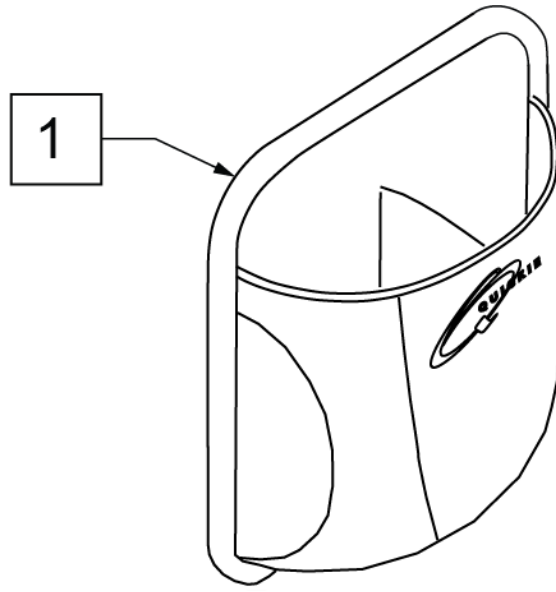


Pos.	Item Number	Description	Remarks
1	013475	QUICKIE SIDE POCKET	Size: 6.5"x 6.5"x 2"

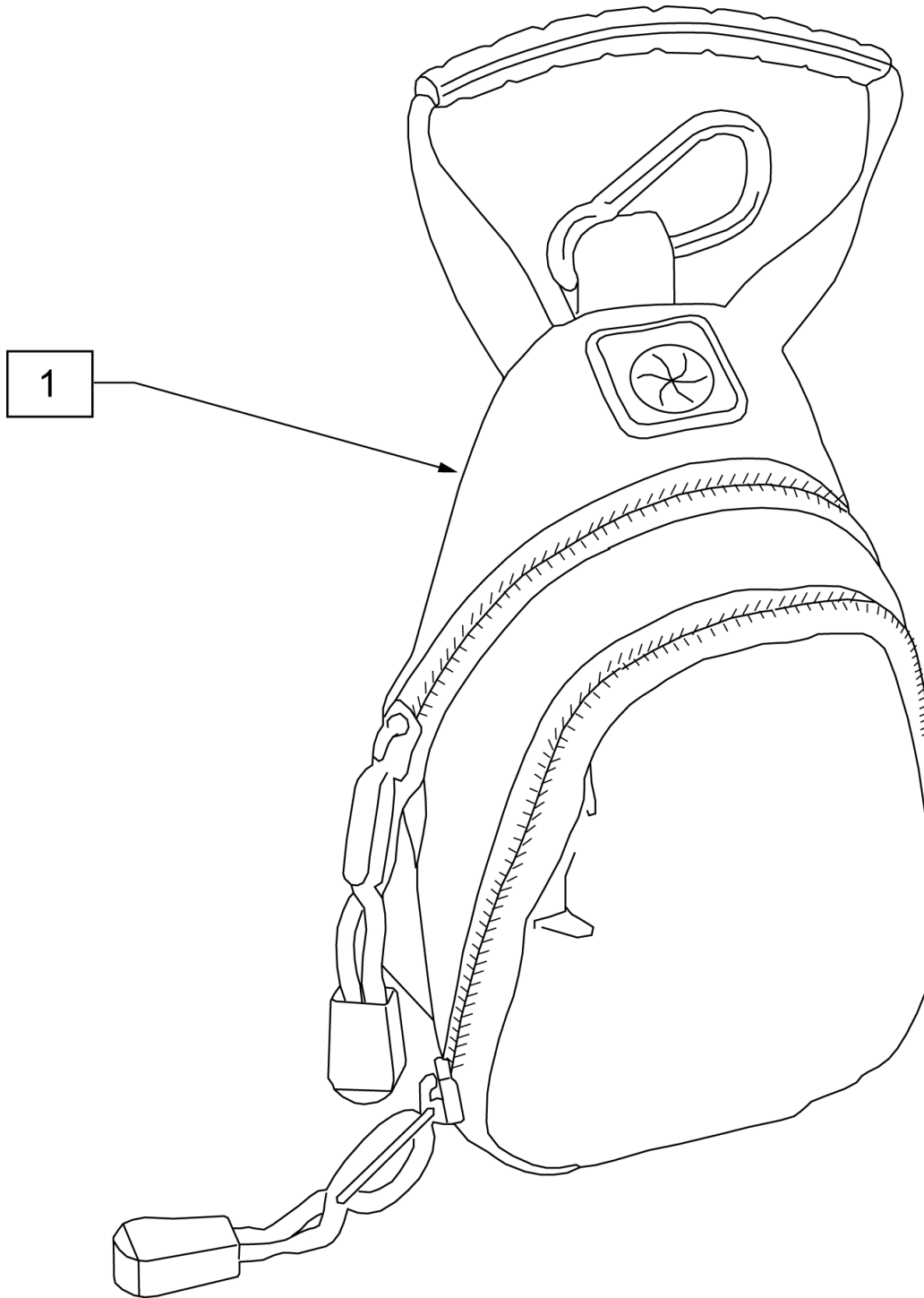


Pos.	Item Number	Description
1	109229	CELL PHONE PACK (ORANGE)

NOTE: KIDS BACKPACKS DO NOT HAVE ADJUSTABLE STRAPS W/ BUCKLES.

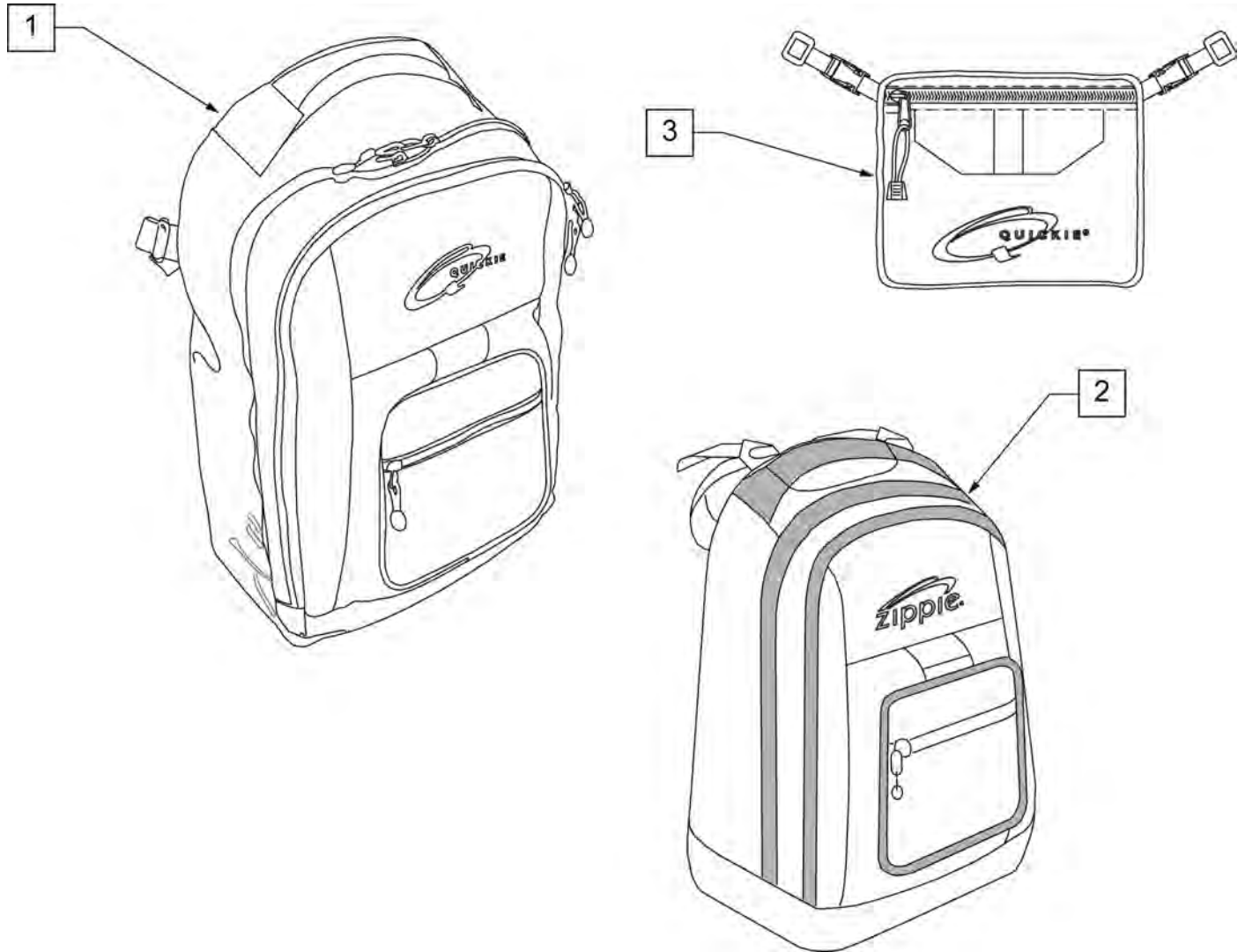

Pos.	Item Number	Description	Remarks
1	750115	BACK PACK QUICKIE	Overall dimensions:width 12.5", height 18.5", depth 6.5"
2	104572	BACKPACK ZIPPIE	Overall dimensions: width 12", height 16.5", depth 4.5"
3	750307	SEAT POUCH QUICKIE	Dimensions: width 8.5", height 6", depth top 2.5", depth bottom 3.5"


Note: some wheel sizes listed may not be available on all chairs.

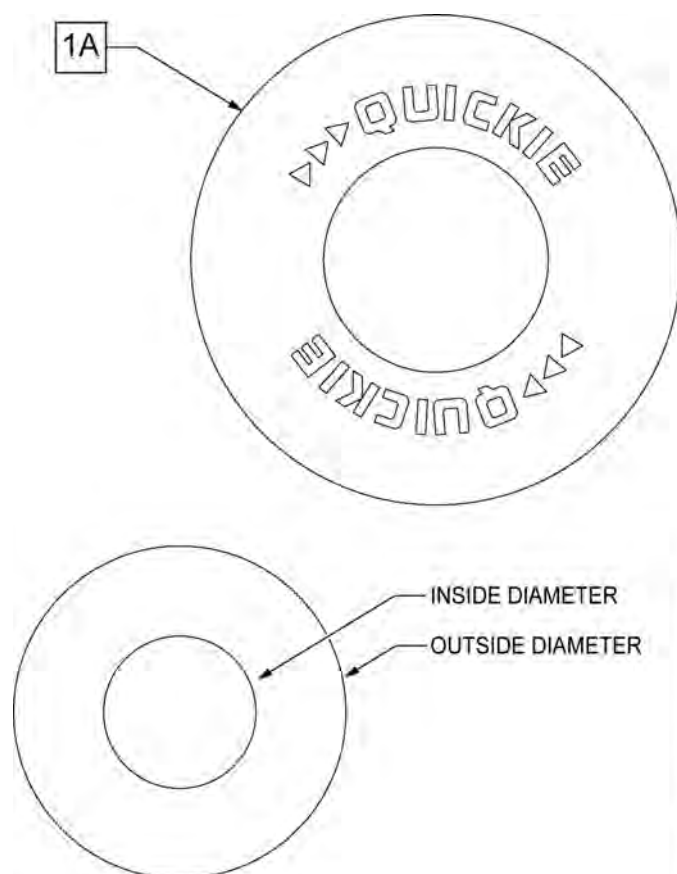


SPOKE GUARD MEASURING GUIDE

PART NUMBER	INSIDE DIAMETER	OUTSIDE DIAMETER
902001	8 1/4	15 7/8
902011	8 1/4	18 1/8
902021	10 1/4	19 5/8
902025	10 1/4	20 3/8
902037	10 1/4	21 7/8
902038	10 1/4	23 3/8
103660-001	8 1/4	16
103660-002	8 1/4	18
103663-001	10 1/4	20 1/2
103663-002	10 1/4	23 1/2
103734-001	10 1/4	19 1/4
103734-002	10 1/4	22
119336	10 1/4	19 1/4


Pos.	Item Number	Description	Remarks
>>>>		BLACK SPOKE GUARDS	
1A	902001	SPK GRDS 20" QUICKIE BLK	PR-Black-fits 20" Whls
1A	902011	SPK GRDS 22" QUICKIE BLK	PR-Black-fits 22" Whls, 24" (540) Whls w/ Proj HR, 24" Mountain Spoke Whls, M6 24" (540) Spoke Whls
1A	902021	SPK GRD 24" QUICKIE SCR MT BLK	PR-Black-fits 24" (540) Mag & Spoke Whls
1A	902025	SPK GRD 24" QUICKIE TAB MT BLK	PR-Black-fits 25" (559) Spk Whls & 26" Mountain Spoke Whls
1A	902037	SPK GRD 26" QUICKIE TAB MT BLK	PR-Black-fits 26" (590) Whls
1A	902038	SPK GRD 26" QUICKIE HT BLK	PR-Black-fits 27" (700) Whls
>>>>		CLEAR SPOKE GUARDS	
1B	103660-001	CLEAR SPOKE GUARD 20"	PR-Clear-fits 20" Whls
1B	103660-002	CLEAR SPOKE GUARD 22"	PR-Clear-fits 22" Whls, 24" (540) Whls w/ Proj HR, 24" Mountain Spoke Whls, M6 24" (540) Spoke Whls

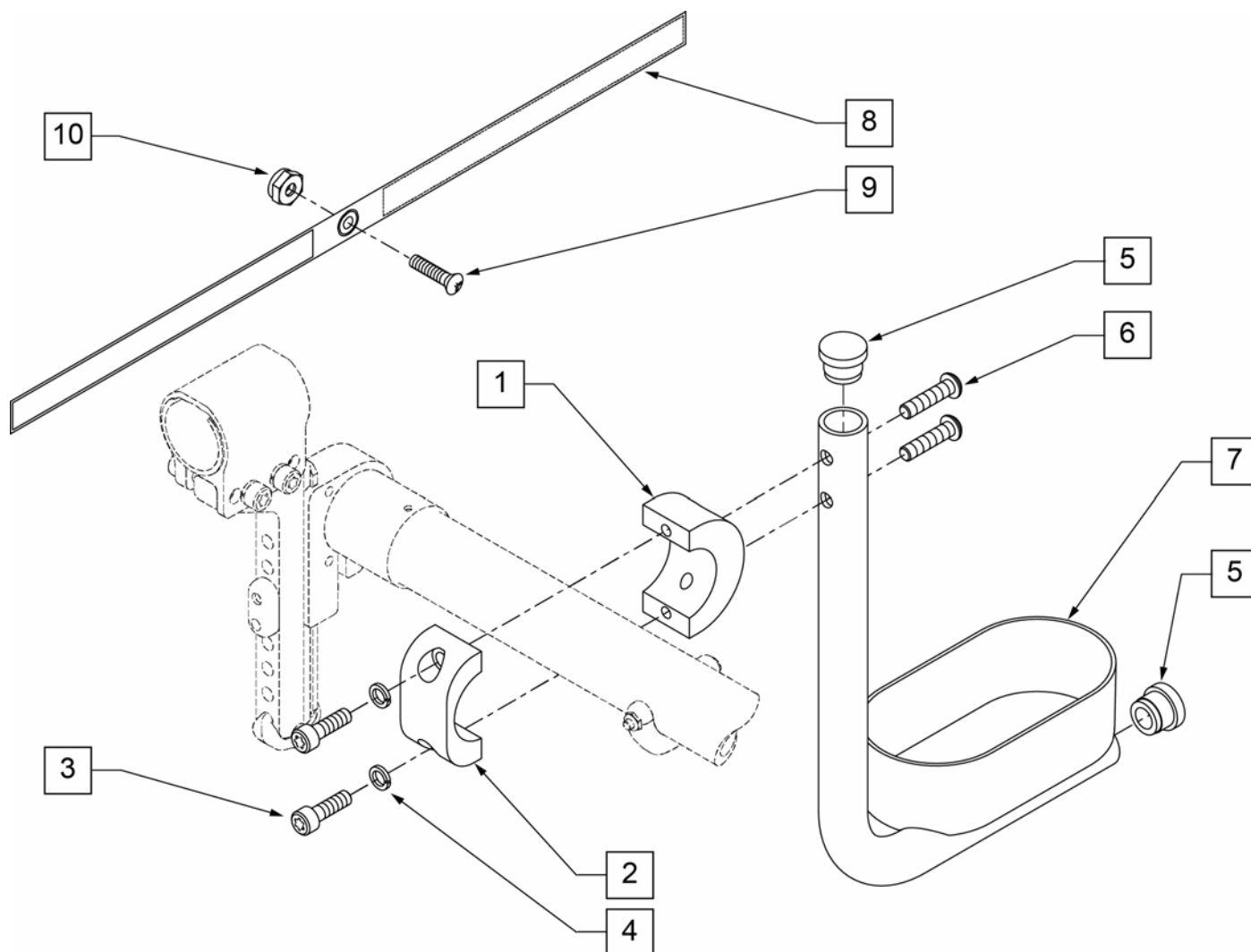


Pos.	Item Number	Description
1	050076	CLAMP SADDLE CRUTCH HOLDER
2	050077	CLAMP CRUTCH HOLDER
3	102263-001	SHCS M6 x 1 x 16 FULL NP
4	050129-001	LOCKWASHER HIGH COLLAR M6
5	561000-QU	3/4" INSERT PLUG .083
6	102262-004	BHCS M6 X 1 X 25 W/PL INORG
8	750495	CRUTCH HOLDER STRAP F
9	281110	10-32 X 1/2 PAN PHIL
10	200100-QU	10-32 NYLOCK NUT
1-10	050169	ASSM CRUTCH HOLDER GT W/HDWR

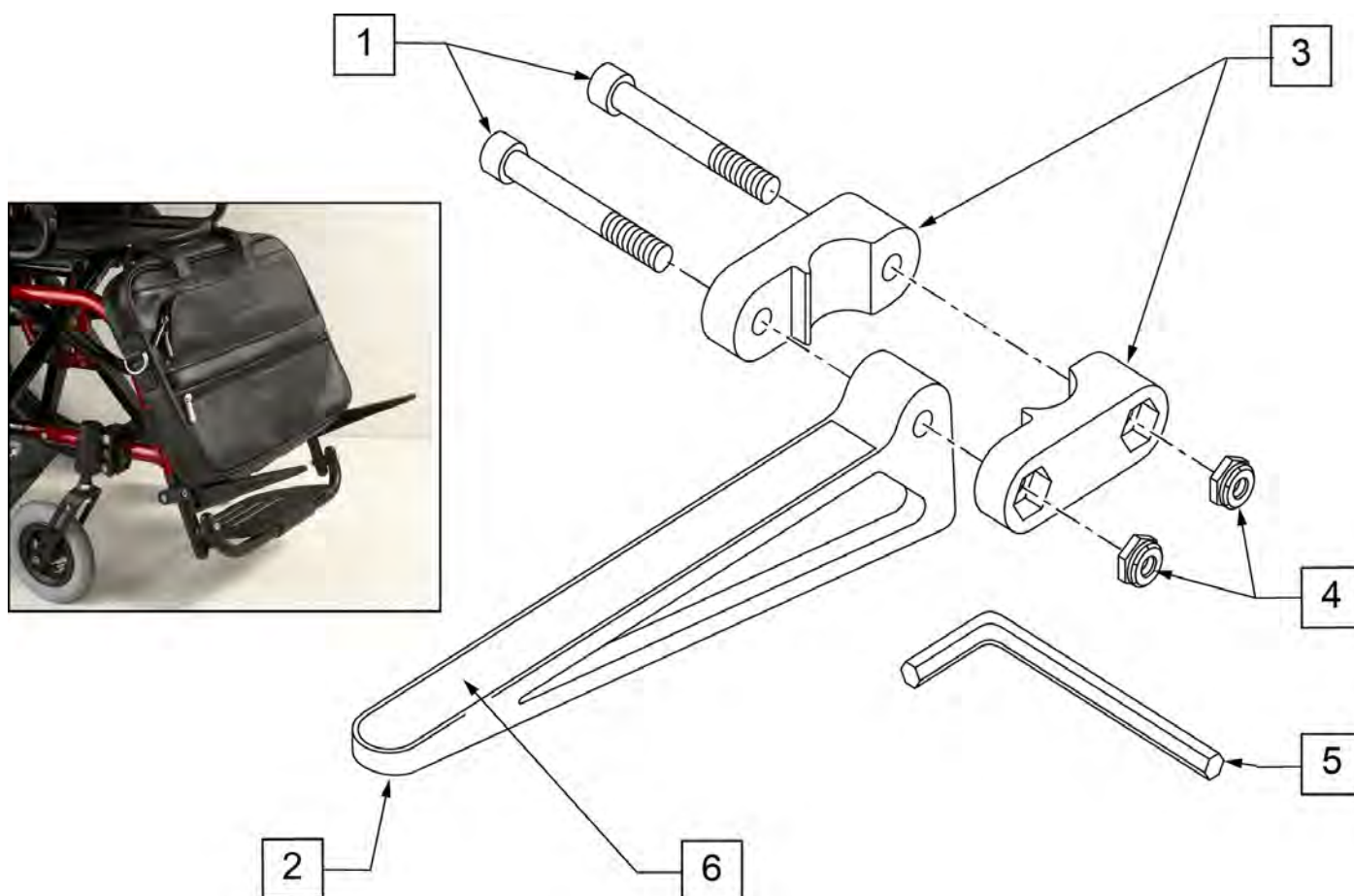


Pos.	Item Number	Description	Remarks
1	106975	1/4-28 x 1 1/4 SOC C/S	
2	561105	CADDY ARM	Clamps on to Extension Tube (3/4" round and D2); Caddy Arm length is 7 1/2"
3	561104	CADDY CLAMP	Clamps on to Extension Tube (3/4" round and D2)
4	106935	1/4-28 NYLOCK NUT THIN	
5	349998	3/16" ALLEN WRENCH	
6	580050	CADDY TAPE	
1-6	902350	QUICKIE CADDY (PR)	

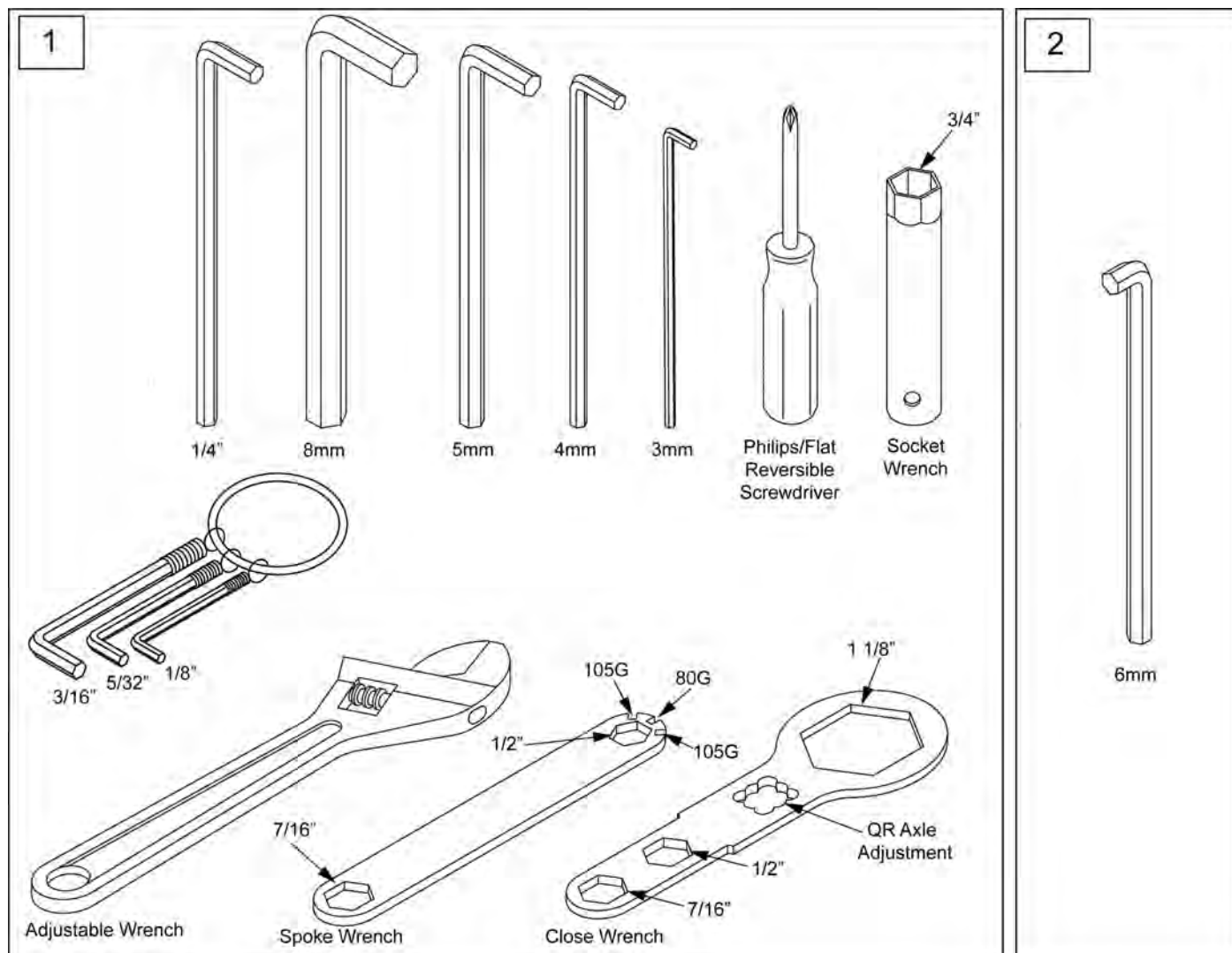

Pos.	Item Number	Description	Remarks
1	108314	QUICKIE COMBO TOOL KIT	Includes pouch
2	113403	6MM ALLEN WRENCH L, 7MM SHORT	caster spline chairs only

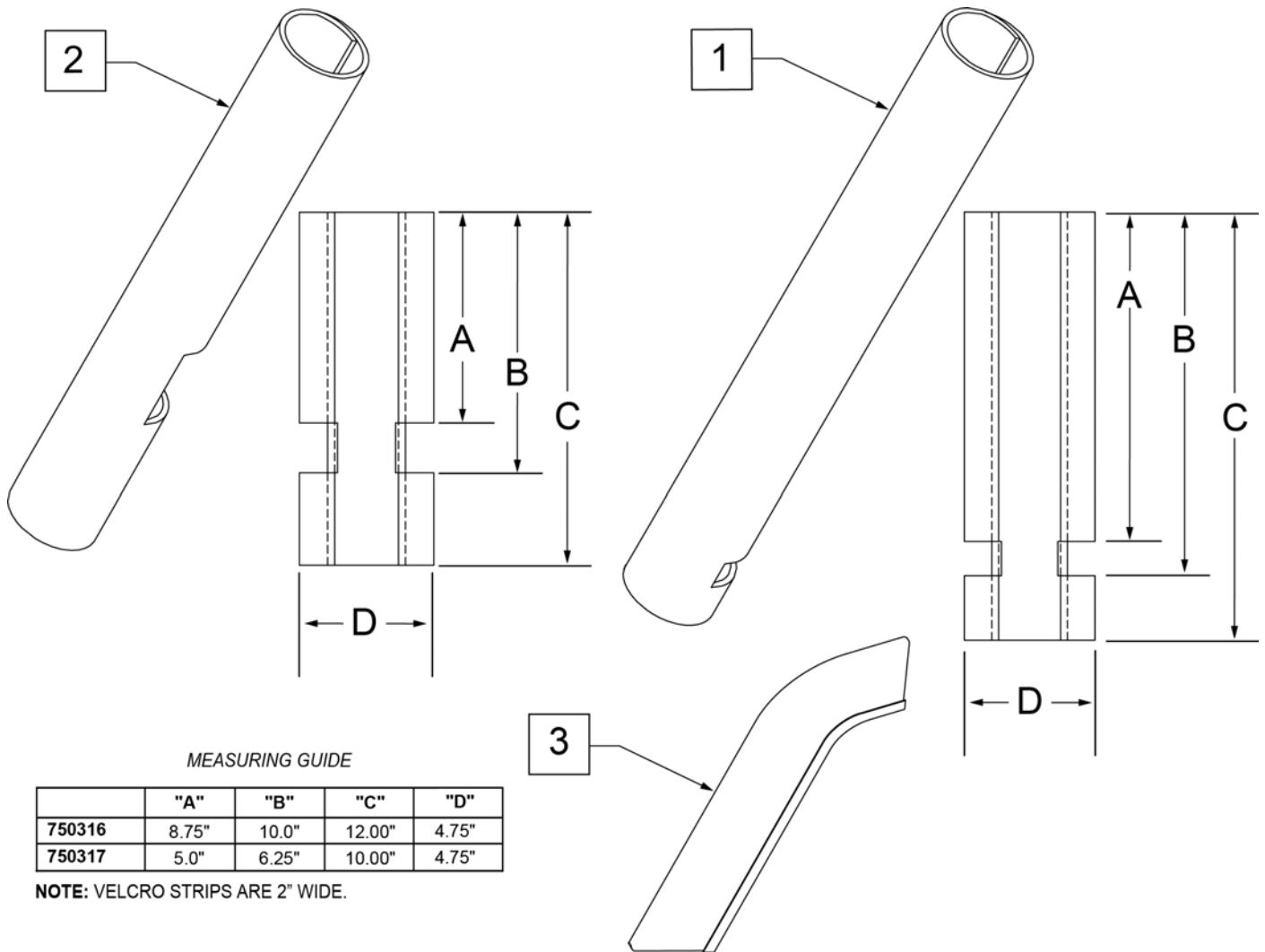

Pos.	Item Number	Description	Remarks
1	750316	IMPACT GUARDS NEOPRENE FIXED (PR)	PAIR-Fixed Front Frame
2	750317	IMPACT GUARDS NEOPRENE S/A	PAIR-S/A Footrest & Q7
3	109231	GEL VELCRO PAD HANGER/FRAME	PAIR- fits S/A & Fixed Front Frame
4	108315	GEL PAD SA ARM	Single, fits Padded SA Armrest

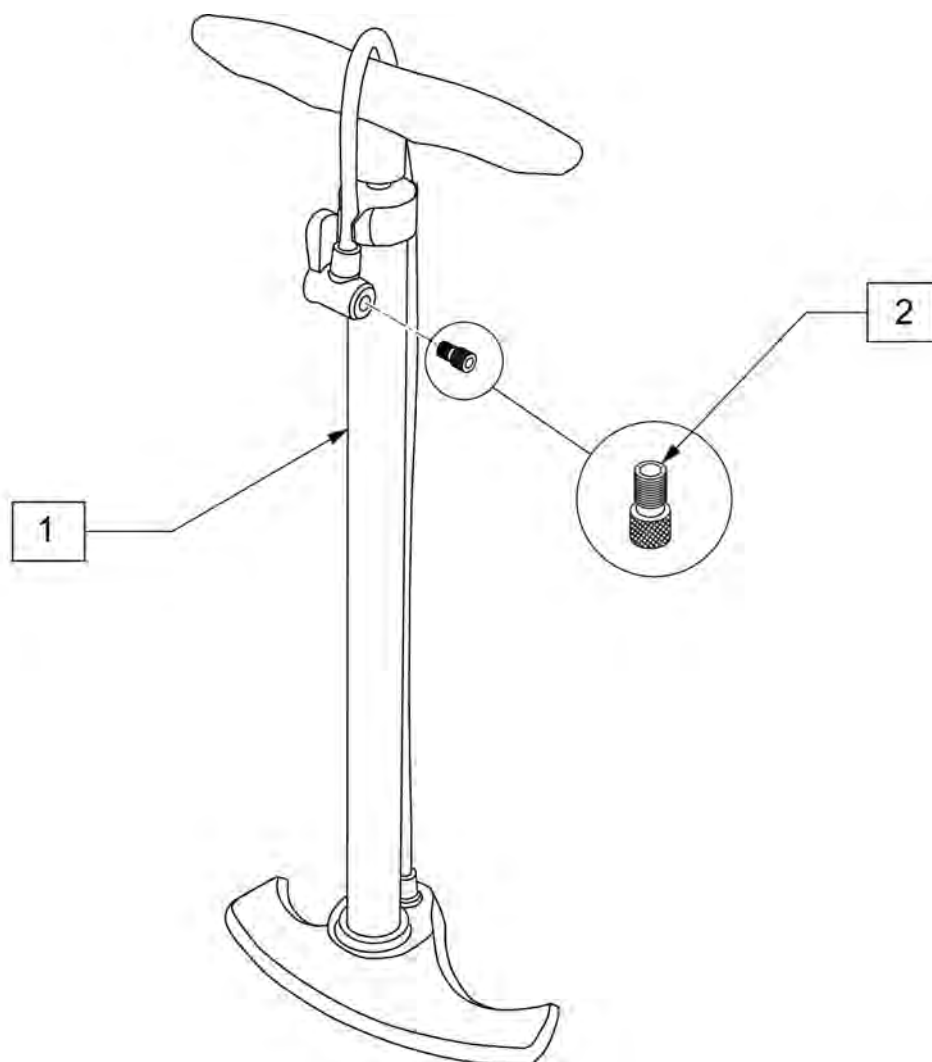

Pos.	Item Number	Description
1	389050	TIRE PUMP
2	381230	VALVE ADAPTER PRESTA

