


(Discontinued) Rigid Height Adjustable T-Arm

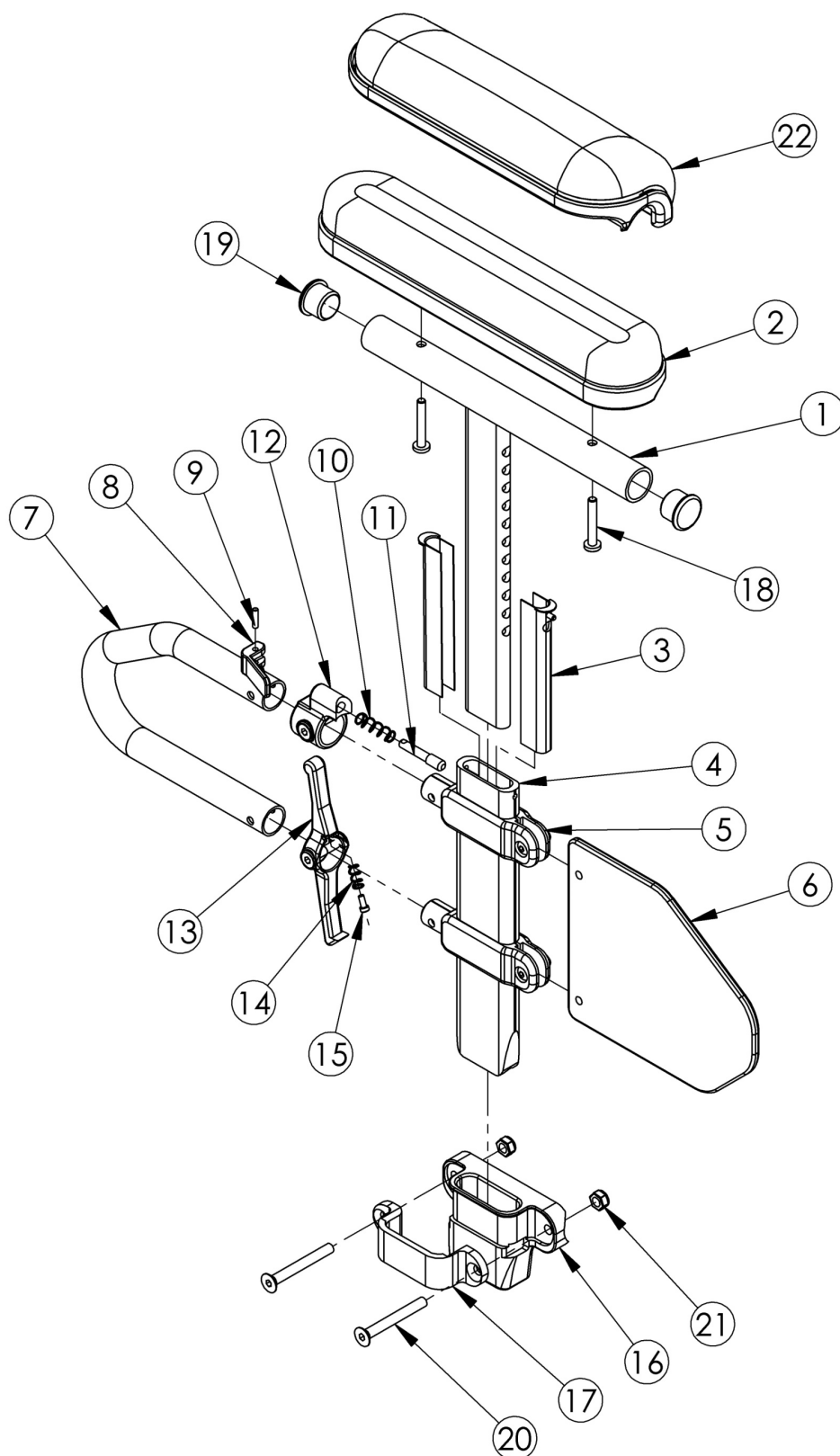

Height Adjustable T-Arms cannot be retrofitted

Item Number	Part Number	Description	Quantity
1a, 3-5, 6, 7-15, 19	100238	Height Adj T Armrest Short Desk Right <i>Replaced with 003463</i>	1
1a, 3-5, 6, 7-15, 19	100242	Height Adj T Armrest Short Desk Left <i>Replaced with 003463</i>	1
1b, 3-5, 6, 7-15, 19	100239	Height Adj T Armrest Short Full Right <i>Replaced with 003464</i>	1
1b, 3-5, 6, 7-15, 19	100243	Height Adj T Armrest Short Full Left <i>Replaced with 003464</i>	1
1a, 2b, 3-5, 6, 7-15, 18, 19	501682	Height Adj T-Arm Assm Short Desk w/ Std Pad Left <i>Replaced with 505646</i>	1
1a, 2b, 3-5, 6, 7-15, 18, 19	501683	Height Adj T-Arm Assm Short Desk w/ Std Pad Right <i>Replaced with 505646</i>	1
1b, 2a, 3-5, 6, 7-15, 18, 19	501684	Height Adj T-Arm Assm Short Full w/ Std Pad Left <i>Replaced with 505644</i>	1
1b, 2a, 3-5, 6, 7-15, 18, 19	501685	Height Adj T-Arm Assm Short Full w/ Std Pad Right <i>Replaced with 505644</i>	1
1a, 2d, 3-5, 6, 7-15, 18, 19	501690	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad Left	1
1a, 2d, 3-5, 6, 7-15, 18, 19	501691	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad Right	1
1b, 2c, 3-5, 6, 7-15, 18, 19	501692	Height Adj T-Arm Assm Short Full w/ Waterfall Pad Left	1
1b, 2c, 3-5, 6, 7-15, 18, 19	501693	Height Adj T-Arm Assm Short Full w/ Waterfall Pad Right	1
16b, 17, 20, 21	501731	Height Adj T-Arm Receiver Assembly 1-3/8", RG ALX/LW XP	1
16a, 17, 20, 21	501732	Height Adj T-Arm Receiver Assembly 1-1/8" TSU TI/CLIK	1
1a, 2b, 3-5, 6, 7-15, 16b, 17-21	501714	Height Adj T-Arm Assm Short Desk w/ Std Pad and 1-3/8" Receiver Left TSU/RG <i>Replaced with 506081</i>	1
1a, 2b, 3-5, 6, 7-15, 16b, 17-21	501715	Height Adj T-Arm Assm Short Desk w/ Std Pad and 1-3/8" Receiver Right TSU/RG <i>Replaced with 506081</i>	1
1b, 2a, 3-5, 6, 7-15, 16b, 17-21	501716	Height Adj T-Arm Assm Short Full w/ Std Pad and 1-3/8" Receiver Left TSU/RG <i>Replaced with 506093</i>	1
1b, 2a, 3-5, 6, 7-15, 16b, 17-21	501717	Height Adj T-Arm Assm Short Full w/ Std Pad and 1-3/8" Receiver Right TSU/RG <i>Replaced with 506093</i>	1

Item Number	Part Number	Description	Quantity
1a, 2b, 3-5, 6, 7-15, 16a, 17-21	501722	Height Adj T-Arm Assm Short Desk w/ Std Pad and 1-1/8" Receiver Left TSU TI/CLIK	1
1a, 2b, 3-5, 6, 7-15, 16a, 17-21	501723	Height Adj T-Arm Assm Short Desk w/ Std Pad and 1-1/8" Receiver Right TSU TI/CLIK	1
1b, 2a, 3-5, 6, 7-15, 16a, 17-21	501724	Height Adj T-Arm Assm Short Full w/ Std Pad and 1-1/8" Receiver Left TSU TI/CLIK	1
1b, 2a, 3-5, 6, 7-15, 16a, 17-21	501725	Height Adj T-Arm Assm Short Full w/ Std Pad and 1-1/8" Receiver Right TSU TI/CLIK	1
1a, 2d, 3-5, 6, 7-15, 16b, 17-21	502377	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad and 1-1/8" Receiver Left TSU TI/CLIK	1
1a, 2d, 3-5, 6, 7-15, 16b, 17-21	502378	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad and 1-1/8" Receiver Right TSU TI/CLIK	1
1b, 2c, 3-5, 6, 7-15, 16b, 17-21	502379	Height Adj T-Arm Assm Short Full w/ Waterfall Pad and 1-1/8" Receiver Left TSU TI/CLIK	1
1b, 2c, 3-5, 6, 7-15, 16b, 17-21	502380	Height Adj T-Arm Assm Short Full w/ Waterfall Pad and 1-1/8" Receiver Right TSU TI/CLIK	1
8-12	500541	Arm Height Adjustable T Upper Lever Assembly	1
1a, 19	000091	Height Adj T Armrest Upper Desk	1
1b, 19	000101	Height Adj T Armrest Upper Full	1
2a	100500	Arm Pad Full	1
2b	100501	Arm Pad Desk	1
3	000090	Height Adjustable T Armrest Shim	2
4	000089	Height Adjustable T Armrest Post	1
6	000097	Height Adj T Armrest Side Guard Short	1
7	000095	Height Adjustable T Armrest Transfer Tube	1
9	200257	Dowel Pin 1/8 x 1/2	1
10	000134	Height Adjustable Armrest Lever Spring	1
11	000099	Height Adjustable Armrest Lever Pin	1
12	000096	Height Adjustable T Armrest Pin Lock Housing	1
14	000135	Height Adjustable T Armrest Release Lever Spring	1
15	000152	Height Adjustable T Armrest Release Spring Button	1
16a	000387	Height Adj T Armrest Receiver 1 1/8" Tsunami Titanium	1

Item Number	Part Number	Description	Quantity
16b	004578	Height Adjustable T-Arm Receiver 1.375"	1
17	000088	Height Adjustable T Armrest Bracket	1
18a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2
18b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2
19	100803	Tube End Plug 7/8"	2
20	101130	M6 x 55 FSHCS P/THD BLKZC	2
21	100658	M6 Nylock Nut Blk Zn	2
22a	102452	Waterfall Arm Pad Full	1
22b	102521	Waterfall Arm Pad Desk	1