


Spark Cross Braces

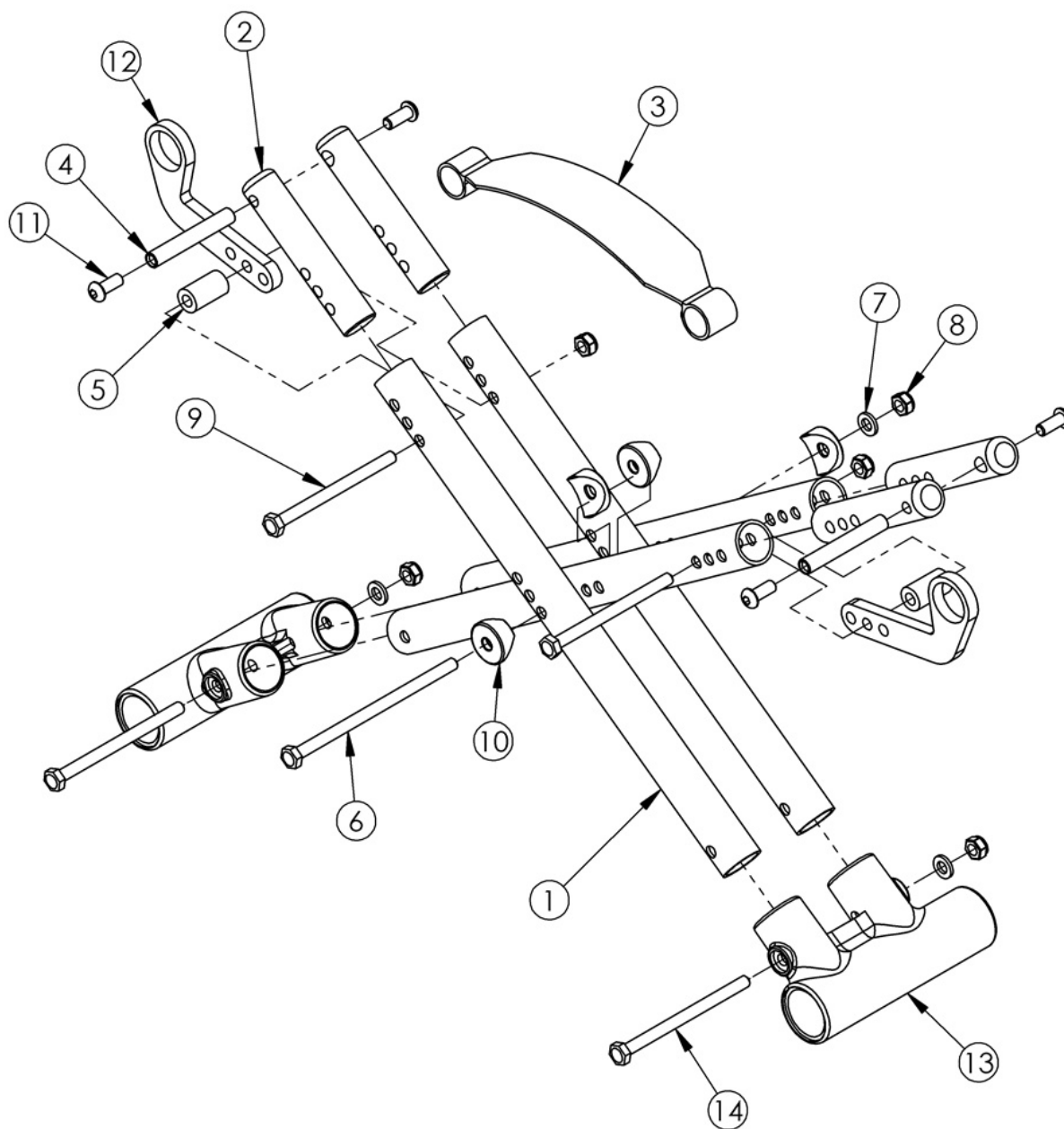

Item Number	Part Number	Description	Quantity
4-11, 14	501512	Cross Brace Hardware Kit SPARK	1
1a	000636	Cross Tube 10-11" Wide SPARK	4
1b	000637	Cross Tube 11-13" Wide SPARK	4
1c	000638	Cross Tube 12-14" Wide SPARK	4

Item Number	Part Number	Description	Quantity
1d	000639	Cross Tube 13-15" Wide SPARK	4
1e	000640	Cross Tube 14-16" Wide SPARK	4
2a	000630	Cross Tube Extension 10-11" Wide SPARK	4
2b	000631	Cross Tube Extension 11-13" Wide SPARK	4
2c	000632	Cross Tube Extension 12-14" Wide SPARK	4
2d	000633	Cross Tube Extension 13-15" Wide SPARK	4
2e	000634	Cross Tube Extension 14-16" Wide SPARK	4
3	000673	Cross Tube Lift Strap, Spark/Arc	1
4	000422	M6 x 56 Threaded Barrel	2
5	101099	Cross Tube Spacer SPARK	2
6	100678	M6 x 110 CLS8.8 DIN931 HHCS ZC	1
7	100746	M6 Flat Washer Blk Zn	5
8	100658	M6 Nylock Nut Blk Zn	5
9	100660	M6 x 80mm CLS8.8 DIN931 HHCS ZC	2
10	000054	Saddle 7/8"	4
11	100668	M6 x 20 CLS10.9 BTN SHCS ZC w/Patch	4
12a	000641	Cross Tube Hinge 10-11" Wide SPARK	2
12b	000642	Cross Tube Hinge 11-13" Wide SPARK	2
12c	000643	Cross Tube Hinge 12-14" Wide SPARK	2
12d	000644	Cross Tube Hinge 13-15" Wide SPARK	2
12e	000645	Cross Tube Hinge 14-16" Wide SPARK	2
13	000001	Lower Frame Pivot CAT	2
14	100704	M6 x 90 CLS8.8 DIN931 HHCS ZC	2