


Flip / Arc Adjustable Height Rigidizer Bar

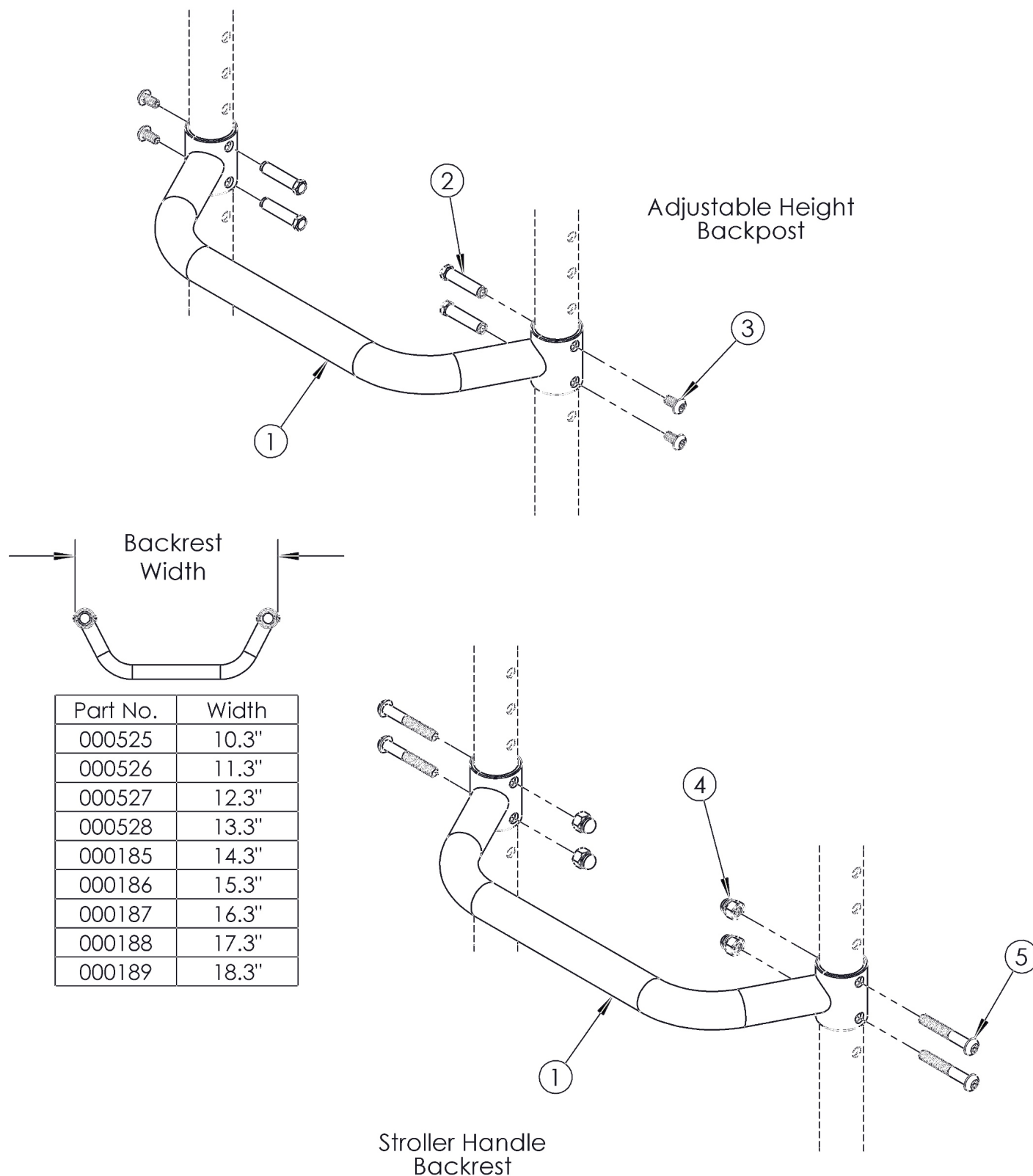

Adjustable Height Rigidizer Bar based on seat width

Item Number	Part Number	Description	Quantity
2, 3	503534	Rigidizer Bar Hardware Kit for Height Adjustable Back	1
4, 5	503535	Rigidizer Bar Hardware Kit for Stroller Handle Back	1
1a	000525	Adjustable Height Rigidizer Bar 10"	1
1b	000526	Adjustable Height Rigidizer Bar 11"	1
1c	000527	Adjustable Height Rigidizer Bar 12" Wide	1
1d	000528	Adjustable Height Rigidizer Bar 13" Wide	1
1e	000185	Adjustable Height Rigidizer Bar 14" Wide	1
1f	000186	Adjustable Height Rigidizer Bar 15" Wide	1
1g	000187	Adjustable Height Rigidizer Bar 16" Wide	1
1h	000188	Adjustable Height Rigidizer Bar 17" Wide	1
1i	000189	Adjustable Height Rigidizer Bar 18" Wide	1
2	000987	Threaded Fastener	4
3	100669	M5 x 10 BTN SHCS BLKZC w/Patch	4
4	100558	M6 Nyl Ins Domed Cap L/N Zc	4
5	100797	M6 x 40 BHCS BLK ZC	4