


Rogue ALX Backrest Mount and Hardware - Growth

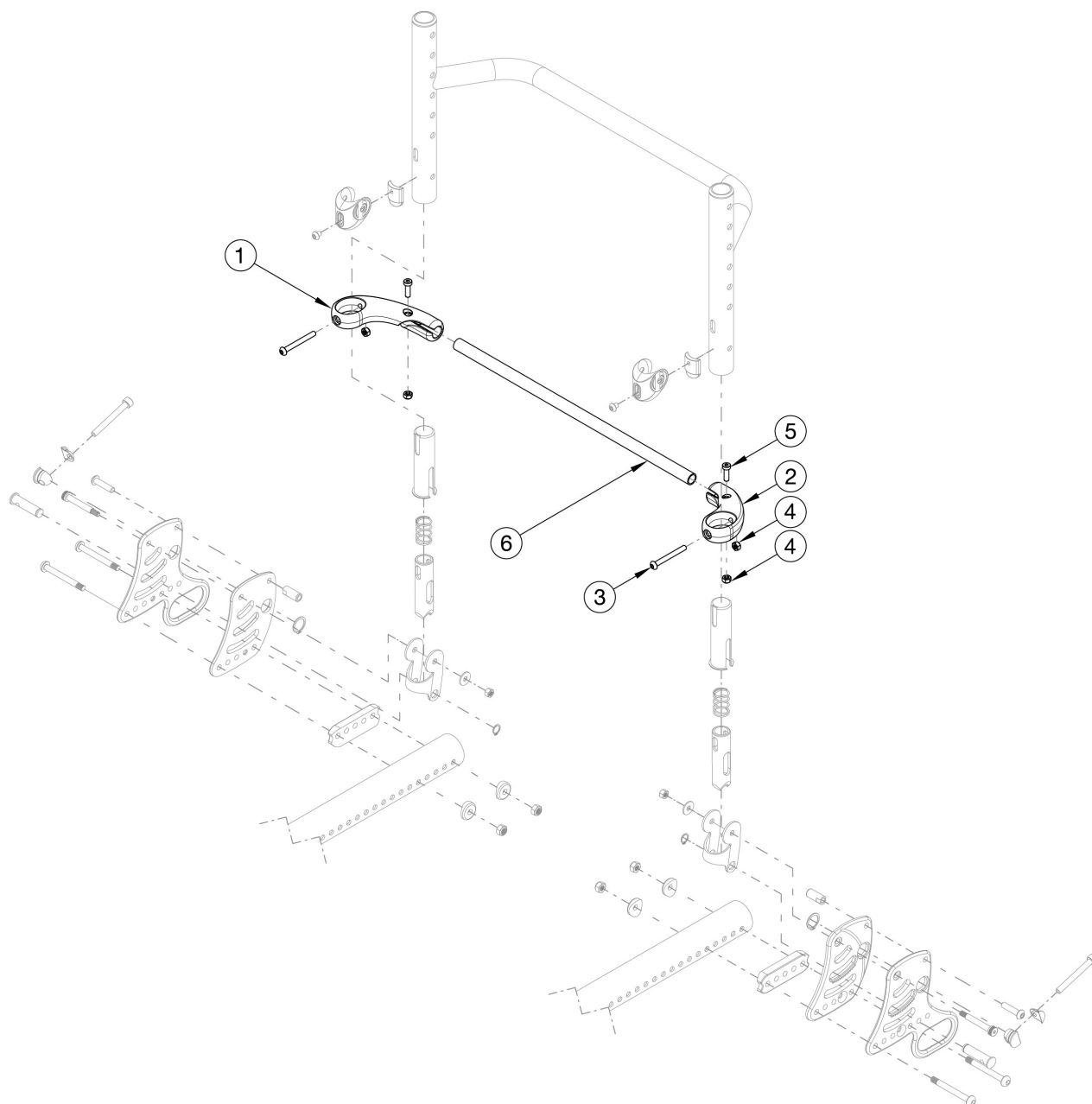

Utilize this page if chair has Rigid Locking Fold Down Back.

Item Number	Part Number	Description	Quantity
1 - 5, 6a	116879	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 12" Wide, Black	1
1 - 5, 6b	116880	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 13" Wide, Black	1
1 - 5, 6c	116881	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 14" Wide, Black	1

Item Number	Part Number	Description	Quantity
1 - 5, 6d	116882	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 15" Wide, Black	1
1 - 5, 6e	116883	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 16" Wide, Black	1
1 - 5, 6f	116884	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 17" Wide, Black	1
1 - 5, 6g	116885	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 18" Wide, Black	1
1 - 5, 6h	116886	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 19" Wide, Black	1
1 - 5, 6i	116887	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 20" Wide, Black	1
3	000142	M5 x 45 BHCS, P/THD, BLZ	2
4	100657	M5 Nylock Nut Blk Zn	4
5	102801	M5 x 16, Low SHCS, Blk Zn	2
6a	115554	Backrest Actuation Bar, 12" Wide, Black	1
6b	115555	Backrest Actuation Bar, 13" Wide, Black	1
6c	115556	Backrest Actuation Bar, 14" Wide, Black	1
6d	115557	Backrest Actuation Bar, 15" Wide, Black	1
6e	115558	Backrest Actuation Bar, 16" Wide, Black	1
6f	115559	Backrest Actuation Bar, 17" Wide, Black	1
6g	115560	Backrest Actuation Bar, 18" Wide, Black	1
6h	115561	Backrest Actuation Bar, 19" Wide, Black	1
6i	115562	Backrest Actuation Bar, 20" Wide, Black	1