



BATTERY REPLACEMENT INSTRUCTION SHEET




Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at +1 (800) 333-4000.

Parts included:

- 2 ea 50Ah Battery

Required tools:





- | | |
|------------------------------------|-----------------------|
| 1. 5mm hex key | 3. 4mm hex key |
| 2. 2x 13mm socket wrench or wrench | 4. 10mm socket wrench |
| | 5. Torque wrench |

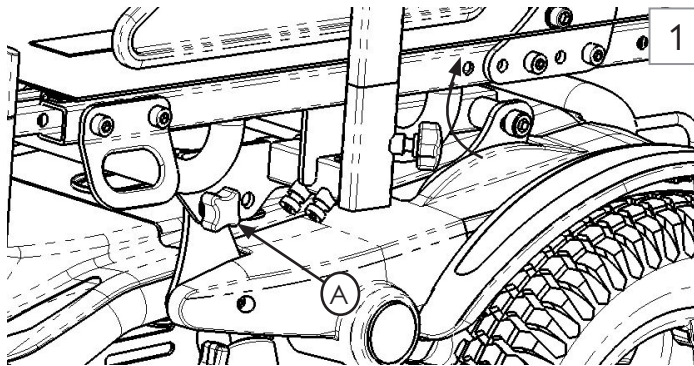

Note: place a rag between the mudguard and the seat rails

A. Replacing the Batteries on a Q200R without Tilt

- Remove thumb screw (A) on both the left and right sides. Manually tilt the seat frame back. Place a cloth on top of the mudguard to protect the shroud and decal. (Fig 1)
- Remove front battery cover (B) by lifting up and forward on the front hand grip. Remove the back battery cover (C) by removing the 4 screws (D). (Fig 2)
- Disconnect the battery harnesses from the front and rear battery. Use the handles on the front battery (E) to lift and remove it from the battery box. (Fig 3)
- Remove the foam battery spacer (G) (Fig 5) and slide the rear battery (F) forward to remove. (Fig 4)
- Install the new rear battery (F). Refer to figure 5 for the correct battery post position. Re-install the foam battery spacer (G) as shown in figure 5. Install the new front battery (E). Refer to figure 5 for the correct battery post position.
- Re-install the battery harnesses. Refer to the battery schematic located inside of the battery shroud (C) removed in step 2.
- Re-install the battery cover (C) using the previous 4 screws (D). Torque to 4-6 Nm.
- Lower the seat and re-install the thumb screw (A) on both the left and right sides. (Fig 1)

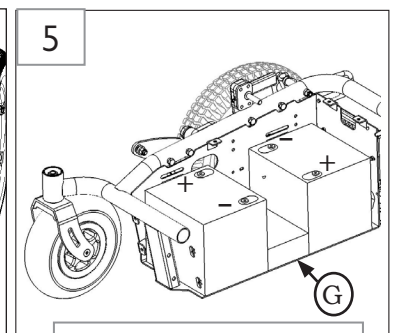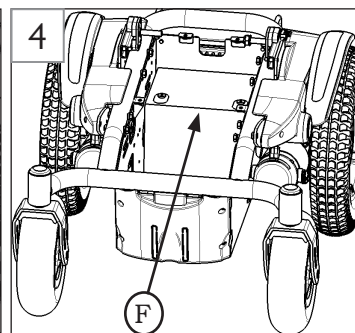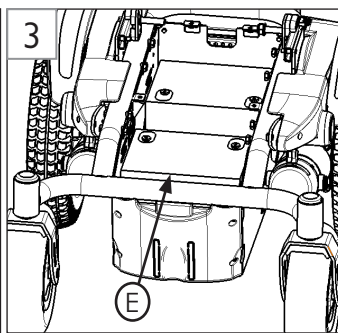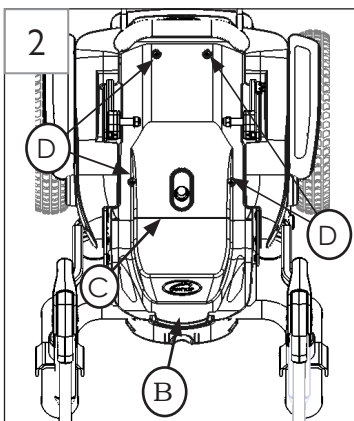

Note: frame cut away for clarity

⚠ DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this wheelchair without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Sunrise Medical Technical Service Department before operating and/or servicing the Zippie wheelchair. Failure to do so may result in damage and/or injury.

Find more information and important warnings see the wheelchair owner's manual or at: www.SunriseMedical.com.

Sunrise Medical 2842 N. Business Park Ave Fresno, CA 93727 USA

Customer service: +1(800) 333-4000
or visit www.SunriseMedical.com



© 2021 Sunrise Medical (US) LLC
02/2022 255776 Rev.A Page 1 of 1