


Spark Side Frame Assembly

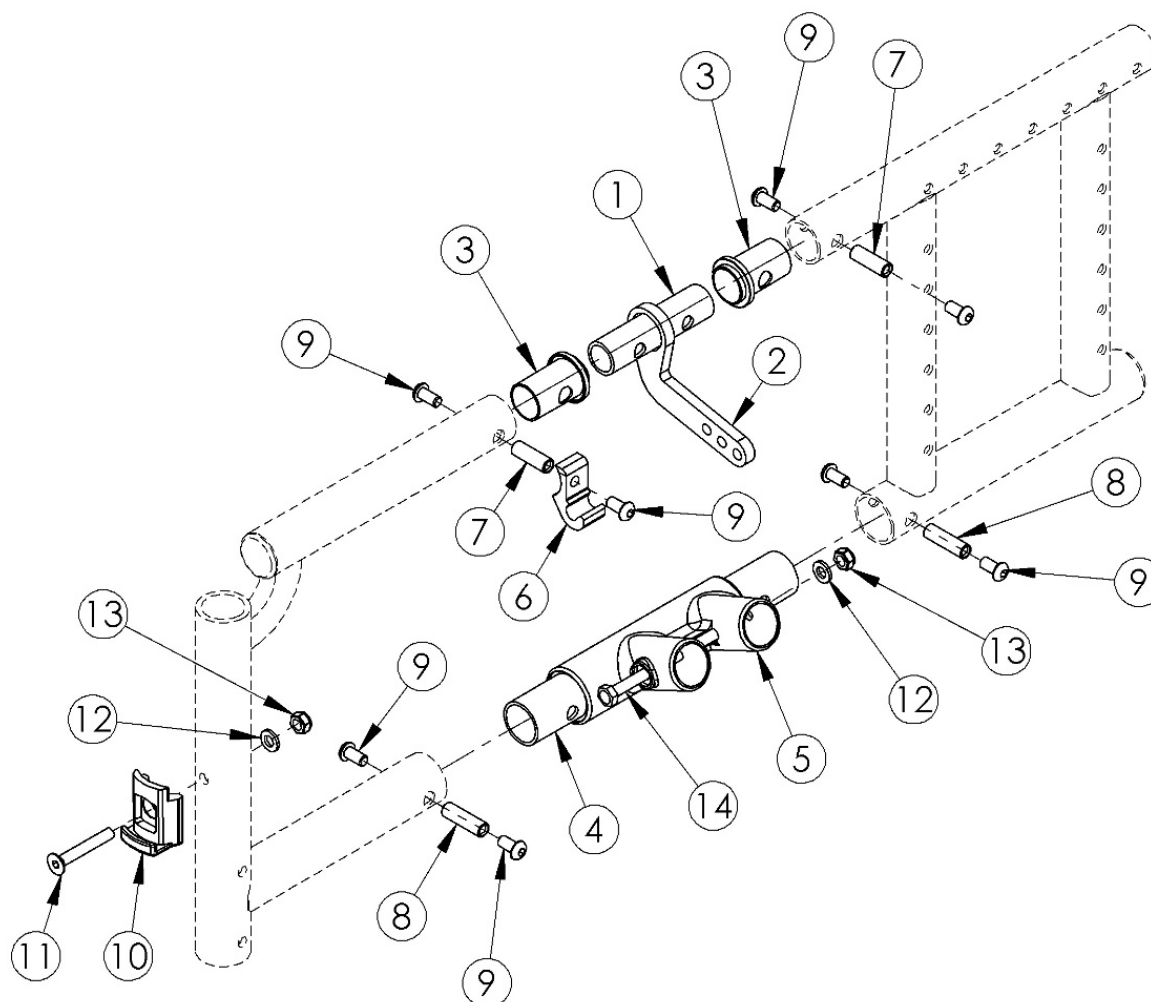

Item Number	Part Number	Description	Quantity
1	000628	Upper Pivot Tube SPARK	1
2a	000641	Cross Tube Hinge 10-11" Wide SPARK	1
2b	000642	Cross Tube Hinge 11-13" Wide SPARK	1
2c	000643	Cross Tube Hinge 12-14" Wide SPARK	1
2d	000644	Cross Tube Hinge 13-15" Wide SPARK	1
2e	000645	Cross Tube Hinge 14-16" Wide SPARK	1
3	000627	Cross Tube Hinge Sleeve SPARK	2
4	000009	Frame Connector Lower CAT	1

Item Number	Part Number	Description	Quantity
5	000001	Lower Frame Pivot CAT	1
6	000626	Spark X-Tube Hook	1
7	000065	M6 x 23.68 Threaded Barrel	2
8	000066	M6 x 27 Threaded Barrel	2
9	100667	M6 x 12 BTN SHCS BLKZC w/Patch	8
10	000336	Latch Block	1
11	101819	M6 x 40 FHCS Blk Zn	1
12	100746	M6 Flat Washer Blk Zn	2
13	100658	M6 Nylock Nut Blk Zn	2
14	100704	M6 x 90 CLS8.8 DIN931 HHCS ZC	1