


# Arc Armrests - Height Adjustable Tall T-Arm

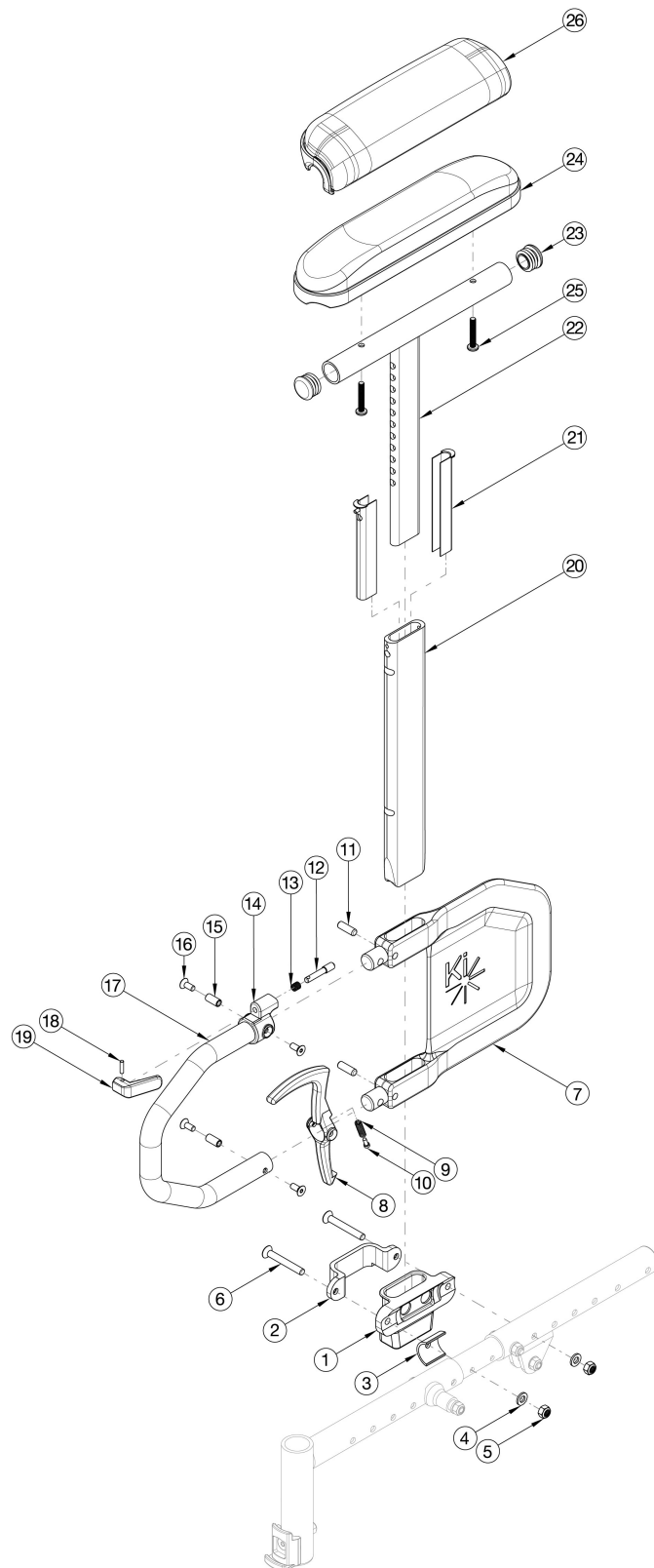

If retrofitting, Receivers are needed.

Item Number	Part Number	Description	Quantity
1-21, 22a, 23, 24b, 25a	116274	Height Adj Tall T-Arm Assm w/ Short Side Shield and Standard Pad Desk and Receiver, Arc	1
1-21, 22b, 23, 24a, 25a	116275	Height Adj Tall T-Arm Assm w/ Short Side Shield and Standard Pad Full and Receiver, Arc	1
1-21, 22a, 23, 25b, 26b	116276	Height Adj Tall T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk and Receiver, Arc	1
1-21, 22b, 23, 25b, 26a	116277	Height Adj Tall T-Arm Assm w/ Short Side Shield and Waterfall Pad Full and Receiver, Arc	1
1-21, 22a, 23, 24b, 25a	116278	Height Adj Tall T-Arm Assm w/ Short Side Shield and Standard Pad Desk and Receiver Pair, Arc	1
1-21, 22b, 23, 24a, 25a	116279	Height Adj Tall T-Arm Assm w/ Short Side Shield and Standard Pad Full and Receiver Pair, Arc	1
1-21, 22a, 23, 25b, 26b	116280	Height Adj Tall T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk and Receiver Pair, Arc	1
1-21, 22b, 23, 25b, 26a	116281	Height Adj Tall T-Arm Assm w/ Short Side Shield and Waterfall Pad Full and Receiver Pair, Arc	1
7-21, 22a, 23, 24b, 25a	116163	Height Adjustable Tall T-Arm Assembly w/Short Side Shield and Std Pad Desk, Pair	1
7-21, 22b, 23, 24a, 25a	116165	Height Adjustable Tall T-Arm Assembly w/Short Side Shield and Std Pad Full, Pair	1
7-21, 22a, 23, 25b, 26b	116167	Height Adjustable Tall T-Arm Assembly w/Short Side Shield and Waterfall Pad Desk, Pair	1
7-21, 22b, 23, 25b, 26a	116169	Height Adjustable Tall T-Arm Assembly w/Short Side Shield and Waterfall Pad Full, Pair	1
7-21, 22a, 23, 24b, 25a	116162	Height Adjustable Tall T-Arm Assembly w/Short Side Shield and Std Pad Desk	1
7-21, 22b, 23, 24a, 25a	116164	Height Adjustable Tall T-Arm Assembly w/Short Side Shield and Std Pad Full	1
7-21, 22a, 23, 25b, 26b	116166	Height Adjustable Tall T-Arm Assembly w/Short Side Shield and Waterfall Pad Desk	1
7-21, 22b, 23, 25b, 26a	116168	Height Adjustable Tall T-Arm Assembly w/Short Side Shield and Waterfall Pad Full	1
12-14, 16, 18, 19	505490	Height Adjustable T-Arm Upper Latch Assembly	1
8-10, 15	116375	Height Adjustable Tall T-Arm Release Lever Assembly	1
7, 11, 15	116377	Height Adjustable Tall T-Arm Short Shield Assembly	1
1-6	113772	Height Adj T-Arm Receiver Assembly, Arc	1
3-6	113771	Height Adj T-Arm Receiver Hardware Kit, Arc	1
3, 6a, 6b, 6c	113793	Height Adj T-Arm Receiver Screw Kit, Arc <i>Includes all three receiver screw sizes and spacer. Accommodates one side.</i>	1
7-21	114150	Height Adjustable T-Arm Short Lower Assembly, Tall <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1
7-21, 22a, 23	114432	Height Adjustable T-Arm Short Side Shield, Desk, Tall <i>Does not include arm pad or arm pad screws</i>	1

Item Number	Part Number	Description	Quantity
7-21, 22b, 23	114433	Height Adjustable T-Arm Short Side Shield, Full, Tall <i>Does not include arm pad or arm pad screws</i>	1
1	110240	Receiver, T-Am	1
2	000088	Height Adjustable T Armrest Bracket	1
3	114443	Tube Spacer, Black, 1" OD x .083" x .80 LG	1
4	100746	M6 Flat Washer Blk Zn	2
5	100658	M6 Nylock Nut Blk Zn	2
6a	101592	M6 x 45 CLS10.9 DIN7991 FHCS BLKZC	2
6b	110286	FHCS, M6 x 1.0 x 70 CLS 12.9 BLZ, P/T, DIN7991	2
6c	100662	M6 x 25 FSHCS BLK ZC	2
9	000135	Height Adjustable T Armrest Release Lever Spring	1
10	000152	Height Adjustable T Armrest Release Spring Button	1
11	003641	.25" x .75" Dowel Pin	1
15	003457	M5 x 14mm Barrel Nut	1
16	103522	M5 x 0.8 x 12 FHCS CLS 10.9 Nyl Blk Zn DIN7991	1
17	113138	Height Adjustable T-Arm Transfer Tube, Tall	1
20	113137	Height Adjustable T-Arm Post, Tall	1
21	000090	Height Adjustable T Armrest Shim	2
22a, 23	116379	Height Adjustable Tall T Armrest Upper Desk w/ Tube End Plug	1
22b, 23	116380	Height Adjustable Tall T Armrest Upper Full w/ Tube End Plug	1
23	100803	Tube End Plug 7/8"	2
24a	100500	Arm Pad Full	1
24b	100501	Arm Pad Desk	1
25a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2
25b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2
26a	102452	Waterfall Arm Pad Full	1
26b	102521	Waterfall Arm Pad Desk	1