


Axiom Swing Away Laterals for AXPB

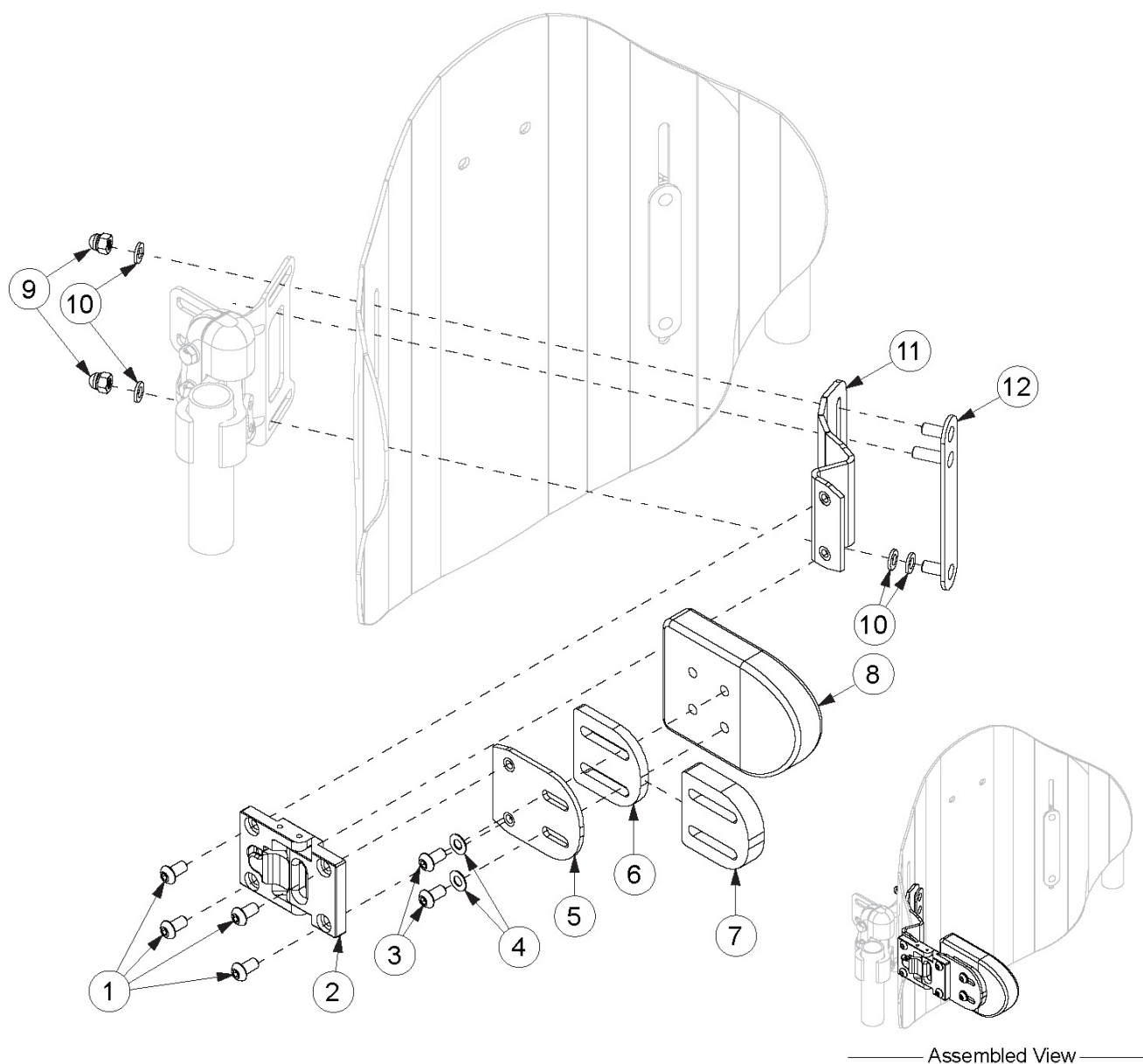

Item Number	Part Number	Description	Quantity	Retail
1	104031	1/4-20 X 3/8 Stainless Button Head Cap Screw	4	\$3.21
2	104463	Lateral, Swingaway Modular Quick Release Mount	1	\$168.68
3a	104017	1/4-20 X 1/2 Stainless Button Head Cap Screw <i>Used when no spacers are used.</i>	2	\$3.21
3b	104028	1/4-20 X 3/4 Stainless Button Head Cap Screw	2	\$5.36

Item Number	Part Number	Description	Quantity	Retail
		<i>Used when the 1/4" spacer is used alone.</i>		
3c	104014	BHCS, 1/4"-20 X 1.00, 18-8, BLK OX, ASME B18.3 <i>Used when the 1/2" spacer is used alone.</i>	2	\$5.36
3d	104024	1/4-20 X 1-1/4 Stainless Button Head Cap Screw <i>Used when the 1/4" and 1/2" spacers are used together.</i>	2	\$7.50
4	102803	Washer Flat .250 x .500 x .027 HX BLZ	2	\$1.66
5	004888	Axiom Back Lateral Adapter, Swing Away	1	\$26.78
6	107555	Axiom Back Lateral Spacer 1/4"	1	\$32.13
7	107556	Axiom Back Lateral Spacer 1/2"	1	\$32.13
6 - 7, 8a	108261	3x4 Lateral Pads (Pair)	1	\$192.78
6 - 7, 8b	108259	4x5 Lateral Pads (Pair)	1	\$192.78
6 - 7, 8c	108262	3x4 Lateral Pad (Each)	1	\$96.39
6 - 7, 8d	108260	4x5 Lateral Pad (Each)	1	\$96.39
9	100558	M6 NYL INS Domed Cap L/N ZC	3	\$1.10
10	100746	M6 Flat Washer Blk Zn	5	\$1.66
11a	004884	Axiom PB Back Lateral Mount, Swing Away, Right	1	\$37.49
11b	004885	Axiom PB Back Lateral Mount, Swing Away, Left	1	\$37.49
12	004889	Axiom Back Lateral Stud Plate	1	\$16.07
1-2, 3a-3d, 4-7, 8a - 10, 11a, 11b, 12	LS3X4P	3 x 4 Lateral Pads with Swing Away Mounting Hardware for Axiom Posterior Back (Pair)	1	\$667.00
1-2, 3a-3d, 4-7, 8b - 10, 11a, 11b, 12	LS4X5P	4 x 5 Lateral Pads with Swing Away Mounting Hardware for Axiom Posterior Back (Pair)	1	\$667.00
1-2, 3a-3d, 4-7, 9 - 10, 11a, 12	108255	Posterior - Swing Away Laterals Right <i>Pad not included.</i>	1	\$254.36
1-2, 3a-3d, 4-7, 9 - 10, 11b, 12	108254	Posterior - Swing Away Lateral Left <i>Pad not included.</i>	1	\$254.36