


Rogue ALX Caster Pin Locks (Formerly Tsunami)

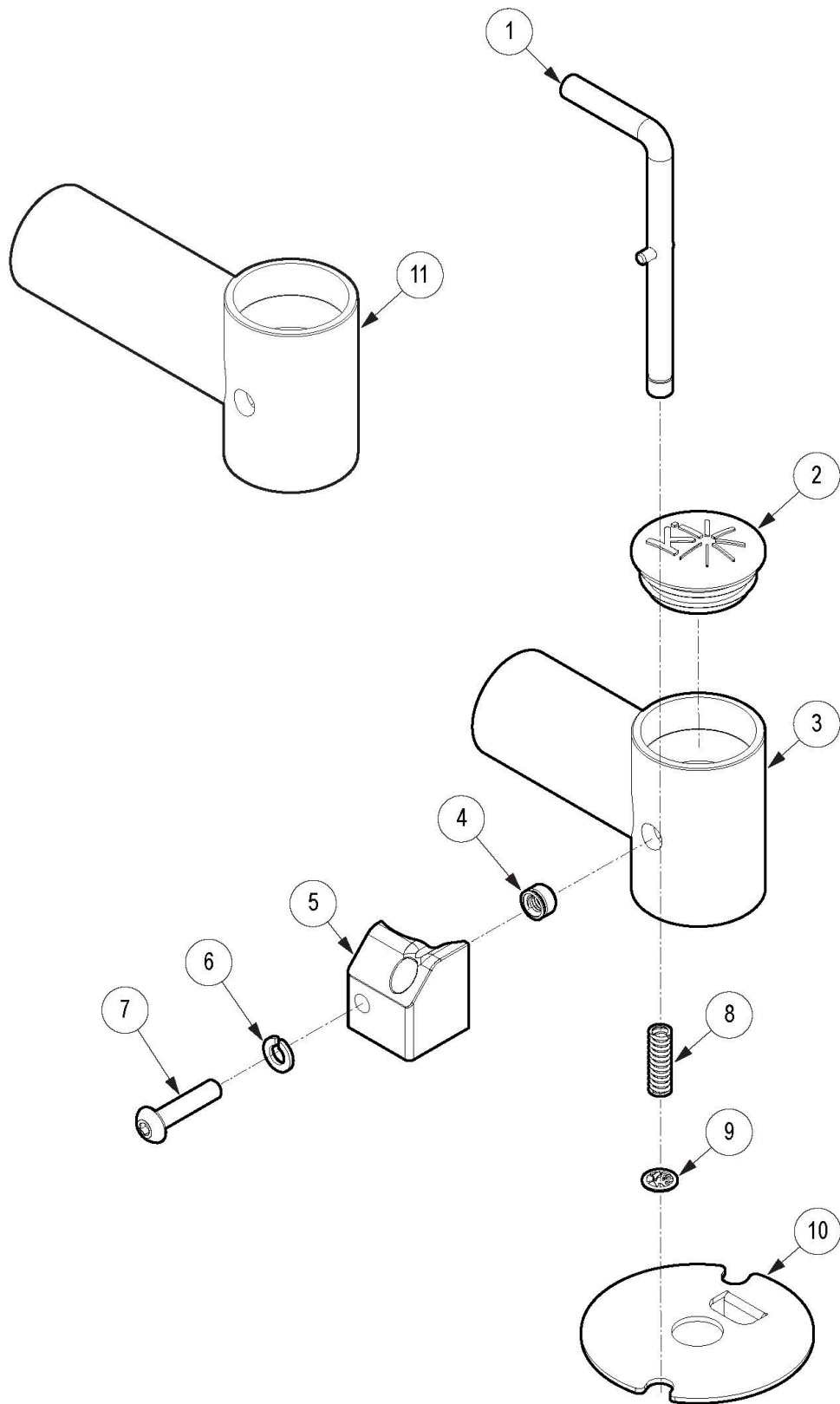

This design effective starting with serial numbers TS010961, TSLW003890, TSHD003974, TSTR004740 and RG0018469 To retrofit from previous Caster Pin Lock design, you will need new caster housings, caster lock mounting blocks and M5 split lock washers. If retrofitting to add Caster Pin Locks, you must order an assembly that contains caster housings.

Item Number	Part Number	Description	Quantity
1a, 5a, 6 - 9, 10a	109980	Caster Pin Lock Assembly, Left, Standard, RG ALX <i>This assembly does not contain a caster housing or caster housing cap</i>	1
1a, 5b, 6 - 9, 10b	109981	Caster Pin Lock Assembly, Right, Standard, RG ALX <i>This assembly does not contain a caster housing or caster housing cap</i>	1
1a, 5a, 5b, 6 - 9, 10a, 10b	109982	Caster Pin Lock Assembly, Standard, PAIR, RG ALX <i>This assembly does not contain caster housings or caster housing caps</i>	1
1b, 5a, 6 - 9, 10a	109983	Caster Pin Lock Assembly, Left, 3/4, RG ALX <i>This assembly does not contain a caster housing or caster housing cap</i>	1
1b, 5b, 6 - 9, 10b	109984	Caster Pin Lock Assembly, Right, 3/4, RG ALX <i>This assembly does not contain a caster housing or caster housing cap</i>	1
1b, 5a, 5b, 6 - 9, 10a, 10b	109985	Caster Pin Lock Assembly, 3/4, PAIR, RG ALX <i>This assembly does not contain caster housings or caster housing caps</i>	1
1a, 2, 3a, 4, 5a, 6 - 9, 10a	109986	Caster Pin Lock Assembly with Caster Housing with Rivnut, Left, Standard, RG ALX	1
1a, 2, 3b, 4, 5b, 6 - 9, 10b	109987	Caster Pin Lock Assembly with Caster Housing with Rivnut, Right, Standard, RG ALX	1
1a, 2, 3a, 3b, 4, 5a, 5b, 6 - 9, 10a, 10b	109988	Caster Pin Lock Assembly with Caster Housing with Rivnut, PAIR, Standard, RG ALX	1
1a, 2, 11a, 4, 5a, 6 - 9, 10a	109989	Caster Pin Lock Assembly with Extended Caster Housing with Rivnut, Left, Standard, RG ALX	1
1a, 2, 11b, 4, 5b, 6 - 9, 10b	109990	Caster Pin Lock Assembly with Extended Caster Housing with Rivnut, Right, Standard, RG ALX	1
1a, 2, 4, 5a, 5b, 6 - 9, 10a, 10b, 11a, 11b	109991	Caster Pin Lock Assembly with Extended Caster Housing with Rivnut, PAIR, Standard, RG ALX	1
1b, 2, 3a, 4, 5a, 6 - 9, 10a	109992	Caster Pin Lock Assembly with Caster Housing with Rivnut, Left, 3/4, RG ALX	1
1b, 2, 3b, 4, 5b, 6 - 9, 10b	109993	Caster Pin Lock Assembly with Caster Housing with Rivnut, Right, 3/4, RG ALX	1
1b, 2, 3a, 3b, 4, 5a, 5b, 6 - 9, 10a, 10b	109994	Caster Pin Lock Assembly with Caster Housing with Rivnut, PAIR, 3/4, RG ALX	1
1b, 2, 11b, 4, 5a, 6 - 9, 10a	109995	Caster Pin Lock Assembly with Extended Caster Housing with Rivnut, Left, 3/4, RG ALX	1
1b, 2, 11b, 4, 5b, 6 - 9, 10b	109996	Caster Pin Lock Assembly with Extended Caster Housing with Rivnut, Right, 3/4, RG ALX	1
1b, 2, 4, 5a, 5b, 6 - 9, 10a, 10b, 11b, 11b	109997	Caster Pin Lock Assembly with Extended Caster Housing with Rivnut, PAIR, 3/4, RG ALX	1

Item Number	Part Number	Description	Quantity
1a	000444	Caster Lock Pin Std, RG ALX <i>Used with Standard Stem</i>	1
1b	000443	Caster Lock Pin 3/4, RG ALX <i>Used with + 3/4" Stem</i>	1
2	102005	Ki Caster Housing Cap	1
3a, 4	108108	Caster Housing with Rivnut for Caster Pin Locks, Left, RG ALX	1
3b, 4	108109	Caster Housing with Rivnut for Caster Pin Locks, Right, RG ALX	1
5a	200419	Caster Lock Mounting Block Left, RG ALX	1
5b	200420	Caster Lock Mounting Block Right, RG ALX	1
6	102614	M5 Split Lock Washer BLK ZC	1
7	101117	M5 x 22 BTN SHCS BLK ZC	1
8	200418	Caster Pin Lock Spring	1
9	102043	1/4" Pushnut Plated	1
10a	000397L	Caster Pin Lock Plate Left	1
10b	000397R	Caster Pin Lock Plate Right	1
11a, 4	108110	Extended Caster Housing with Rivnut for Caster Pin Locks, Left, RG ALX	1
11b, 4	108111	Extended Caster Housing with Rivnut for Caster Pin Locks, Right, RG ALX	1