


Catalyst 4 Height Adjustable Flip Back T-Arm

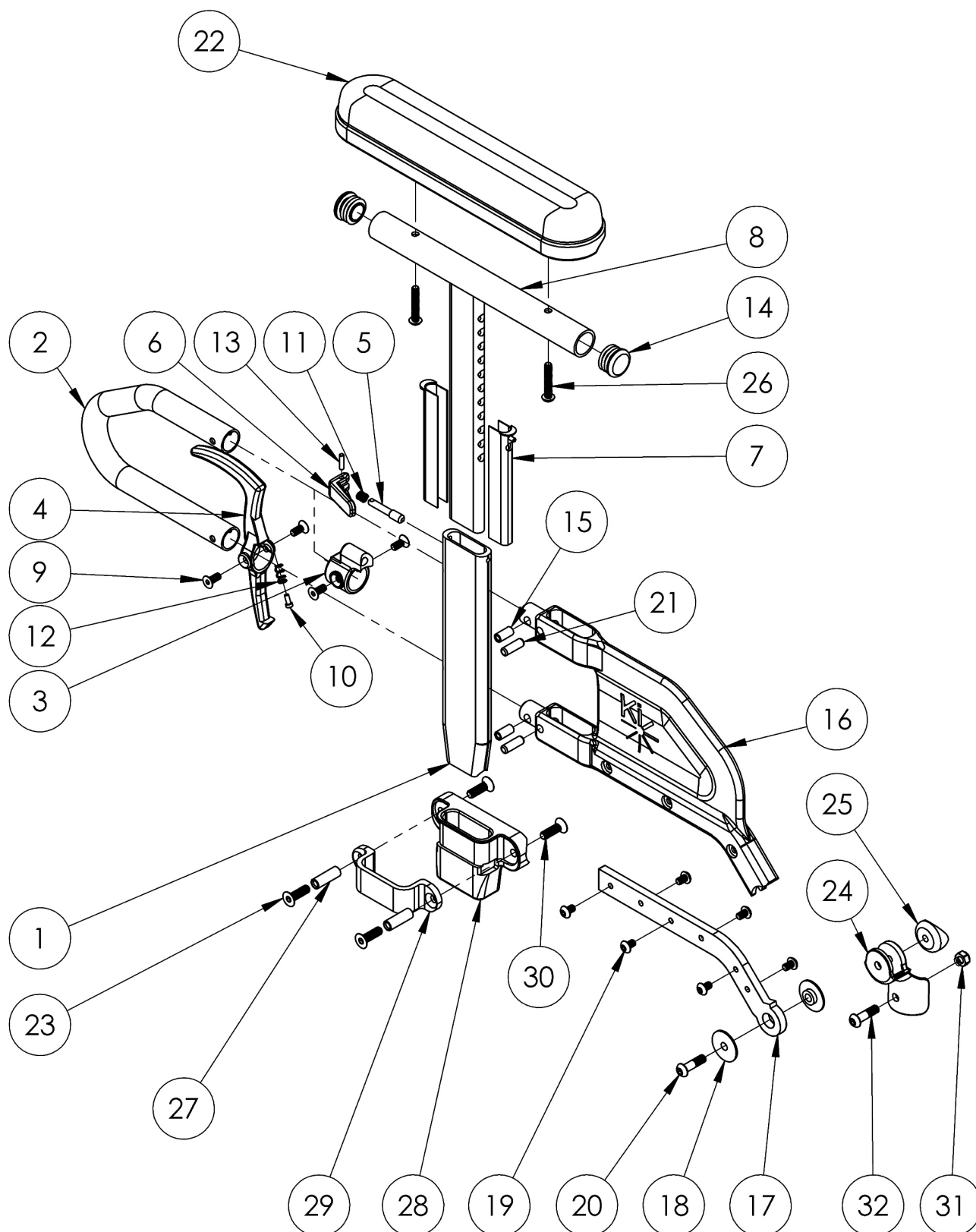

Short Side Shield used with 14"-17" Seat Depths. Long Side Shield used with 18"-20" Seat Depths.

Item Number	Part Number	Description	Quantity	Retail
1-7, 9-13, 15, 16a, 17a, 19, 21	004158	Height Adjustable Flip Back T-Arm Lower Assembly Short <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1	\$56.28
1-7, 9-13, 15, 16b, 17b, 19, 21	004159	Height Adjustable Flip Back T-Arm Lower Assembly Long <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1	\$56.28
1-7, 8a, 9-15, 16a, 17a, 19, 21	003459	Height Adjustable Flip Back T-Arm Short Desk <i>Does not include arm pad or arm pad screws</i>	1	\$131.33
1-7, 8b, 9-15, 16a, 17a, 19, 21	003460	Height Adjustable Flip Back T-Arm Short Full <i>Does not include arm pad or arm pad screws</i>	1	\$131.33
1-7, 8a, 9-15, 16b, 17b, 19, 21	003461	Height Adjustable Flip Back T-Arm Long Desk <i>Does not include arm pad or arm pad screws</i>	1	\$131.33
1-7, 8b, 9-15, 16b, 17b, 19, 21	003462	Height Adjustable Flip Back T-Arm Long Full <i>Does not include arm pad or arm pad screws</i>	1	\$131.33
1-7, 9-13, 15, 16a, 17a, 18-21, 24, 25	505617	Height Adjustable Flip Back T-Arm Short Retrofit Assembly, Catalyst 4 <i>This kit is used to retrofit from a Height Adjustable T-Arm to a Height Adjustable Flip Back T-Arm. This kit is to retrofit one side.</i>	1	\$193.23
1-7, 9-13, 15, 16b, 17b, 18-21, 24, 25	505618	Height Adjustable Flip Back T-Arm Long Retrofit Assembly, Catalyst 4 <i>This kit is used to retrofit from a Height Adjustable T-Arm to a Height Adjustable Flip Back T-Arm. This kit is to retrofit one side.</i>	1	\$195.33
1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 24-26, 31, 32	505266	Height Adjustable Flip Back T-Arm Short Desk with Standard Pad Assembly, Catalyst 4	1	\$175.98
1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 24-26, 31, 32	505267	Height Adjustable Flip Back T-Arm Short Full with Standard Pad Assembly, Catalyst 4	1	\$175.98
1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 24-26, 31, 32	505268	Height Adjustable Flip Back T-Arm Long Desk with Standard Pad Assembly, Catalyst 4	1	\$175.98
1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 24-26, 31, 32	505269	Height Adjustable Flip Back T-Arm Long Full with Standard Pad Assembly, Catalyst 4	1	\$175.98
1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 23-32	505270	Height Adjustable Flip Back T-Arm Short Desk with Standard Pad and Receiver Assembly, Catalyst 4	1	\$235.30

Item Number	Part Number	Description	Quantity	Retail
1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 23-32	505271	Height Adjustable Flip Back T-Arm Short Full with Standard Pad and Receiver Assembly, Catalyst 4	1	\$235.30
1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 23-32	505272	Height Adjustable Flip Back T-Arm Long Desk with Standard Pad and Receiver Assembly, Catalyst 4	1	\$235.30
1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 23-32	505273	Height Adjustable Flip Back T-Arm Long Full with Standard Pad and Receiver Assembly, Catalyst 4	1	\$235.30
18, 20, 24, 25	112957	Armrest Flip Back Height Adj T-Arm Hinge Assembly, PAIR, Catalyst 4 <i>This kit is for a pair of hinge assemblies. A pair is required as a replacement as of May 19th, 2021</i>	1	\$41.20
3, 5, 6, 9, 11, 13	504560	Height Adjustable Flip Back T-Arm Latch Assembly	1	\$45.45
4, 9, 10, 12	504561	Height Adjustable Flip Back T-Arm Release Lever Assembly	1	\$11.64
23, 27-30	501730	Height Adj T-Arm Receiver Assm 1" CAT	1	\$55.66
31, 32	504577	Height Adjustable Flip Back T-Arm Mounting Hardware Assembly <i>This kit contains various mounting hardware for multiple configurations</i>	1	\$13.66
1	003420	T-Arm Post	1	\$23.11
2	000095	Height Adj T Armrest Transfer Tube	1	\$28.07
7	000090	Height Adj T Armrest Shim	2	\$2.55
8a, 14	112906	Height Adj T Armrest Upper Desk w/ Tube End Plug	1	\$76.67
8b, 14	112907	Height Adj T Armrest Upper Full w/ Tube End Plug	1	\$76.67
14	100803	Tube End Plug 7/8"	2	\$1.58
15, 16a, 21	504562	Height Adjustable Flip Back T-Arm Side Shield Short	1	\$29.42
15, 16b, 21	504563	Height Adjustable Flip Back T-Arm Side Shield Long	1	\$29.42
17a, 19	504564	Height Adjustable Flip Back T-Arm Metal Insert Short	1	\$33.62
17b, 19	504565	Height Adjustable Flip Back T-Arm Metal Insert Long	1	\$35.72
22a	100500	Arm Pad Full	1	\$23.64
22b	100501	Arm Pad Desk	1	\$23.64
23	100866	M6 x 20 CLS12.9 DIN7991 FSHCS Blk Zc w/Patch	2	\$2.63
26	100696	M5 x 30mm PPH Capscrew	2	\$1.58
27	000065	M6 x 23.68 Threaded Barrel	2	\$4.13
28	000087	Height Adjustable T-Arm Receiver 1" CAT	1	\$15.25

Item Number	Part Number	Description	Quantity	Retail
29	000088	Height Adj T Armrest Bracket	1	\$27.42
30	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.58