

INSTRUCTION SHEET FROG LEG REASSEMBLY



 Please read and understand these instructions before beginning the installation.

Tools Required:

Two 4mm Allen Wrenchs

Parts Required: Insert tool (fig 1) P/N 116351

- 1. Thread the insert tool onto the pivot pin as shown in fig 2.
- 2. With the elastomer and the lower fork inplace insert the pivot pin into the assembly as far as possible (fig 3)
- 3. Use the 4mm Allen Wrench to apply a downward force (fig 4) while turning the wrench clockwise.
- 4. When the pivot pin bottoms out (fig 5) remove the insert tool and install the button head cap screw (fig 6) and tighten.













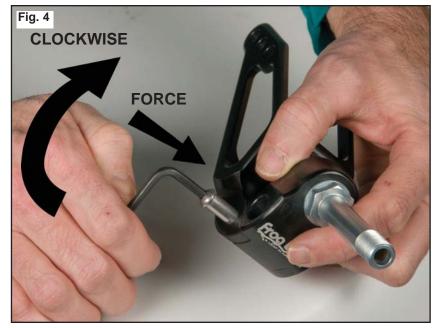

SUNRISE MEDICAL IS00104 REV. A JANUARY 2011 PAGE 1 OF 1

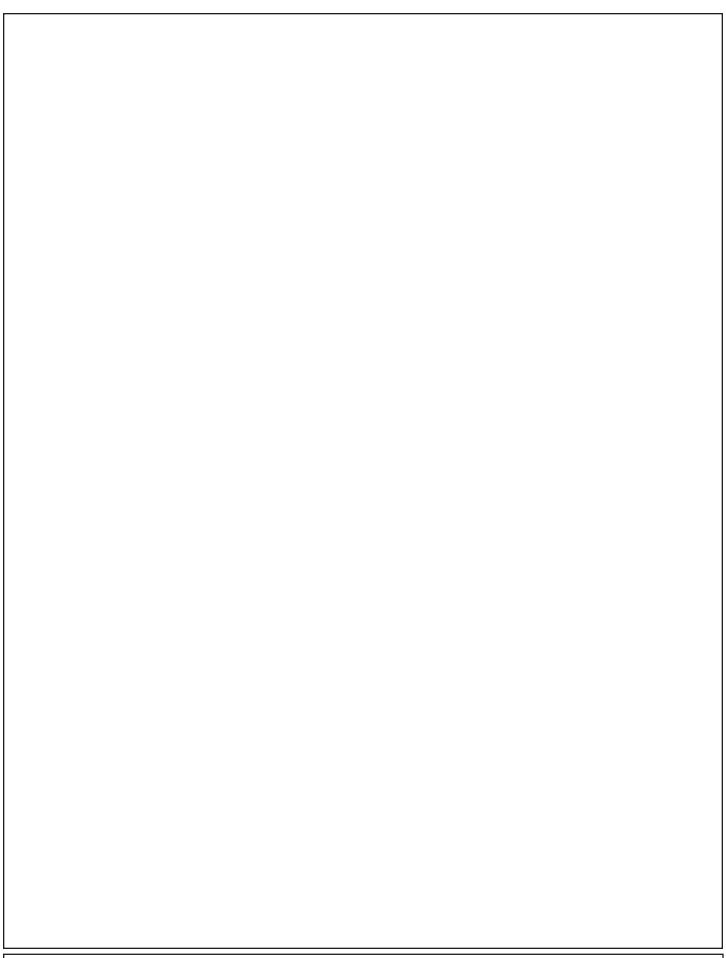

SUNRISE MEDICAL IS00104 REV. A JANUARY 2011 PAGE 2 OF 1