


Focus CR Seat Pan

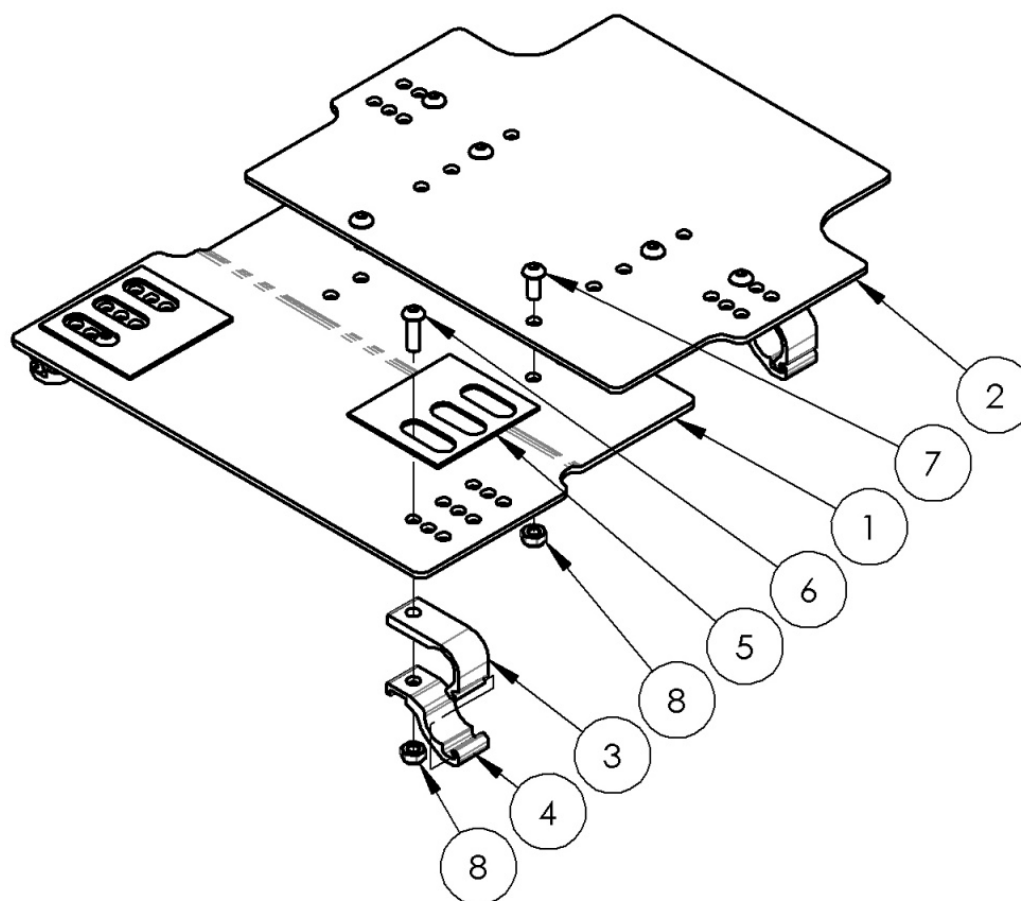

Grows two inches in width from even widths and one inch from odd widths. Max width is 24". Depth growth is matched to frame depth range and can be adjusted to any depth in the range.

Item Number	Part Number	Description	Quantity
1a, 2a, 3 - 8	501144	14-16" Wide, Short, Seat Pan Assembly, Tilt	1
1b, 2b, 3 - 8	501145	16-18" Wide, Short, Seat Pan Assembly Tilt	1
1c, 2c, 3 - 8	501146	18-20" Wide, Short, Seat Pan Assembly Tilt	1
1d, 2d, 3 - 8	501147	20-22" Wide, Short, Seat Pan Assembly Tilt	1
1e, 2e, 3 - 8	501218	Seat Pan Assembly Short 22" Wide TILT	1
1f, 2a, 3 - 8	501148	Seat Pan Assembly Medium 14-16" Wide TILT	1

Item Number	Part Number	Description	Quantity
1g, 2b, 3 - 8	501149	Seat Pan Assembly Medium 16-18" Wide TILT	1
1h, 2c, 3 - 8	501150	Seat Pan Assembly Medium 18-20" Wide TILT	1
1i, 2d, 3 - 8	501151	Seat Pan Assembly Medium 20-22" Wide TILT	1
1j, 2e, 3 - 8	501219	Seat Pan Assembly Medium 22" Wide TILT	1
1k, 2a, 3 - 8	501152	Seat Pan Assembly Long 14-16" Wide TILT	1
1l, 2b, 3 - 8	501153	Seat Pan Assembly Long 16-18" Wide TILT	1
1m, 2c, 3 - 8	501154	Seat Pan Assembly Long 18-20" Wide TILT	1
1n, 2d, 3 - 8	501155	Seat Pan Assembly Long 20-22" Wide TILT	1
1o, 2e, 3 - 8	501220	Seat Pan Assembly Long 22-24" Wide TILT	1
1a	001184	Seat Pan Front Short 14-16" Wide TILT	1
1b	001185	Seat Pan Front Short 16-18" Wide TILT	1
1c	001186	Seat Pan Front Short 18-20" Wide TILT	1
1d	001187	Seat Pan Front Short 20-22" Wide TILT	1
1e	001188	Seat Pan Front Short 22" Wide TILT	1
1f	001189	Seat Pan Front Medium 14-16" Wide TILT	1
1g	001190	Seat Pan Front Medium 16-18" Wide TILT	1
1h	001191	Seat Pan Front Medium 18-20" Wide TILT	1
1i	001192	Seat Pan Front Medium 20-22" Wide TILT	1
1j	001193	Seat Pan Front Medium 22" Wide TILT	1
1k	001194	Seat Pan Front Long 14-16" Wide TILT	1
1l	001195	Seat Pan Front Long 16-18" Wide TILT	1
1m	001196	Seat Pan Front Long 18-20" Wide TILT	1
1n	001197	Seat Pan Front Long 20-22" Wide TILT	1
1o	001198	Seat Pan Front Long 22-24" Wide TILT	1
2a	001199	Seat Pan Rear 14-16" Wide TILT	1
2b	001200	Seat Pan Rear 16-18" Wide TILT	1
2c	001201	Seat Pan Rear 18-20" Wide TILT	1
2d	001202	Seat Pan Rear 20-22" Wide TILT	1

Item Number	Part Number	Description	Quantity
2e	001203	Seat Pan Rear 22-24" Wide TILT	1
3	001204	Seat Pan/Base Bracket TILT	4
4	001205	Seat Pan/Base Clamp TILT	4
5	001206	AB Tape 3x3 Die Cut Tilt	2
6	101814	M8 x 25 BHCS Blk Zn	4
7	101813	M8 x 18 BHCS Blk Zn	4
8	100659	M8 Nylock Nut Blk Zn	8