


Arc Cross Braces - Rigid

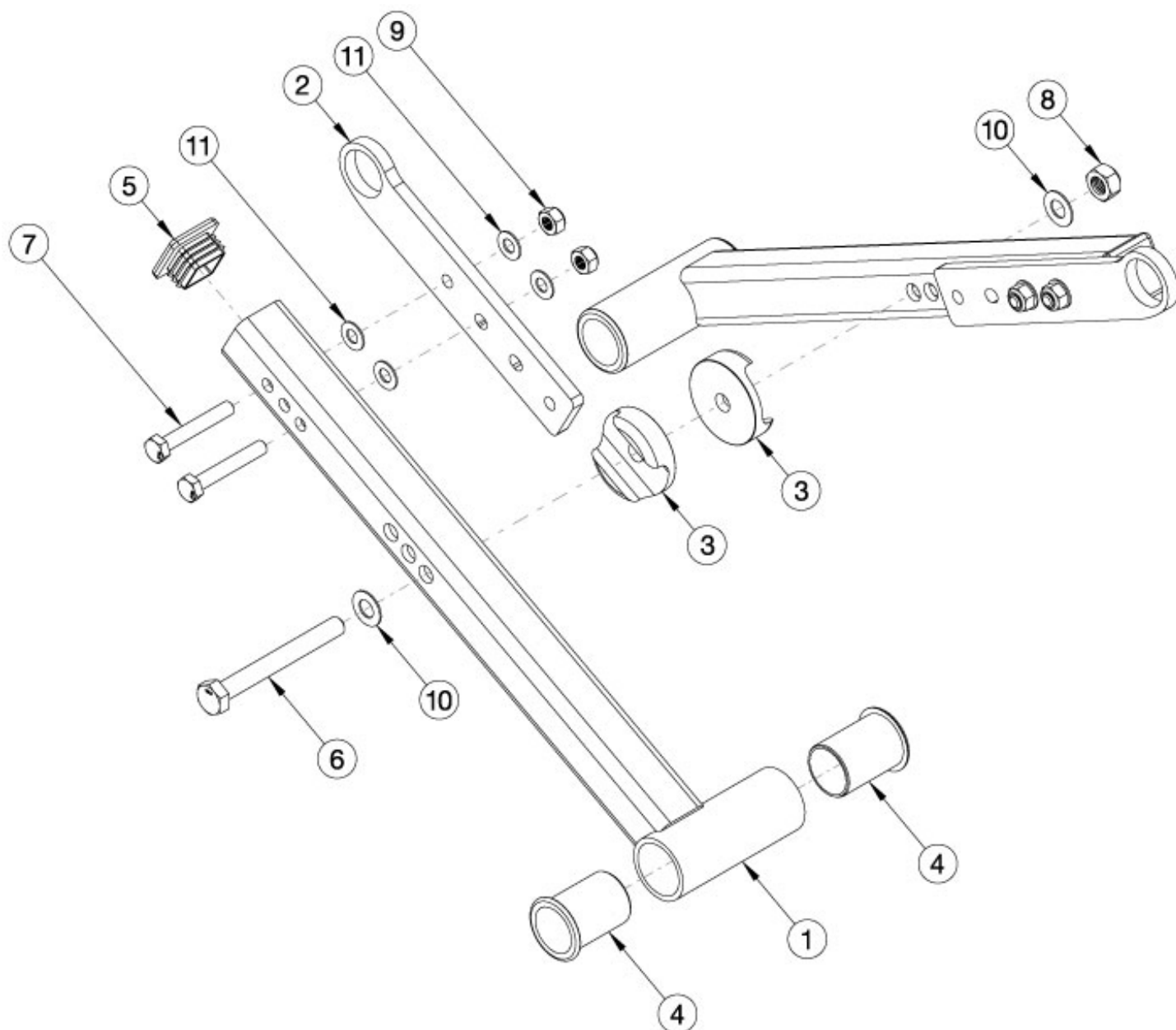

Configuration	Width Range	Item 1 Part #	Item 2 Part #
Small	10-12"	110170	111600
Medium	12-14"	110171	111601
Large	14-16"	110172	100602
Extra Large	16-18"	110173	111603

Item Number	Part Number	Description	Quantity
1a, 2a, 3-11	112948	Rigid Cross Brace Kit Small, Arc <i>Includes parts for both sides.</i>	1

Item Number	Part Number	Description	Quantity
1b, 2b, 3-11	112949	Rigid Cross Brace Kit Medium, Arc <i>Includes parts for both sides.</i>	1
1c, 2c, 3-11	112950	Rigid Cross Brace Kit Large, Arc <i>Includes parts for both sides.</i>	1
1d, 2d, 3-11	112951	Rigid Cross Brace Kit X-Large, Arc <i>Includes parts for both sides.</i>	1
1a	110170	X-Tube, Small, Arc	2
1b	110171	X-Tube, Medium, Arc	2
1c	110172	X-Tube, Large, Arc	2
1d	110173	X-Tube, Extra Large, Arc	2
2a	111600	Extension, X-Tube, Fixed, Small	2
2b	111601	Extension, X-Tube, Fixed, Medium	2
2c	111602	Extension, X-Tube, Fixed, Large	2
2d	111603	Extension, X-Tube, Fixed, X-Large	2
3	110183	Saddle, X-Tube	2
4	110184	Bushing, Pivot, X-Tube	4
5	111599	Plug, Square, 1" x .72"	2
6	101830	M8 x 70 HHCS Blk Zn	1
7	101825	M6 x 40 HHCS Blk Zn	4
8	110287	Nut, Toplock, M8x1.25, CLS10, BLZ, DIN980V	1
9	100658	M6 Nylock Nut Blk Zn	4
10	110295	Washer, Nylon, .323ID x .625OD x .031	2
11	100831	1/2" Washer OD .040" Thick	8