


PAL Spare Parts Catalogue



Contents


03	PAL Seat Product Overview
04-05	PAL Seat Assembly
06	PAL High Seat Assembly
07	PAL Tray Assembly
08	PAL Ski Assembly
09	PAL Footrest Assembly
10	PAL Two Person Table Assembly
11	PAL Four Person Table Assembly
12	PAL Rocker/Stabiliser Assembly

PAL Seat - Product Overview

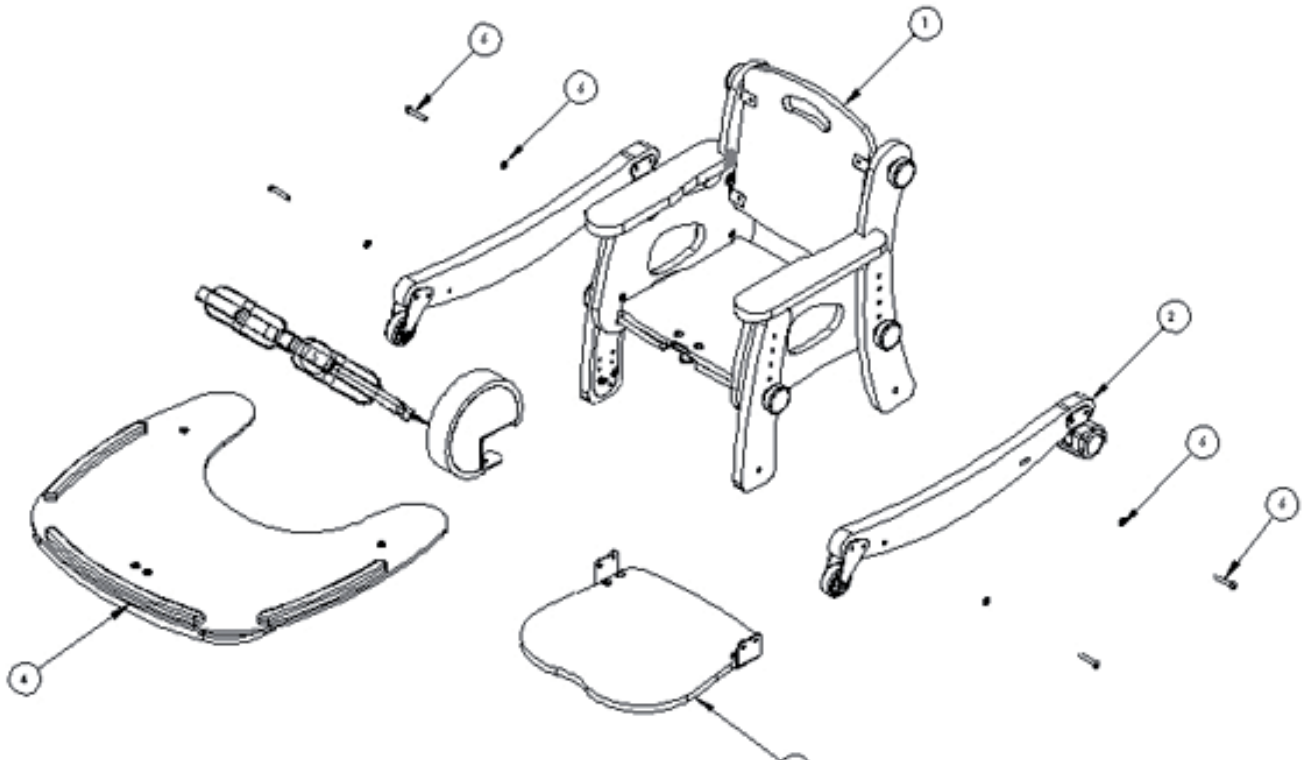

Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
1	135-1600-XX	Chair Size 1	1	✓		
1	135-2600-XX	Chair Size 2	1		✓	
1	135-3600-XX	Chair Size 3	1			✓
2	135-1610	Ski With Castors Size 1	2	✓		
2	135-2610	Ski With Castors Size 2	2		✓	
2	135-3610	Ski With Castors Size 3	2			✓
3	135-1603	Footrest Size 1	1	✓		
3	135-2603	Footrest Size 2	1		✓	
3	135-3603	Footrest Size 3	1			✓
4	135-1620	Activity Tray Size 1	1	✓		
4	135-2620	Activity Tray Size 2	1		✓	
4	135-3620	Activity Tray Size 3	1			✓
5	135-1602-XX	Pommel Size 1	1	✓		
5	135-3602-XX	Pommel Size 3	1		✓	✓
6	SP-1040	Ski Screw Kit	4	✓	✓	✓

PAL Seat Assembly

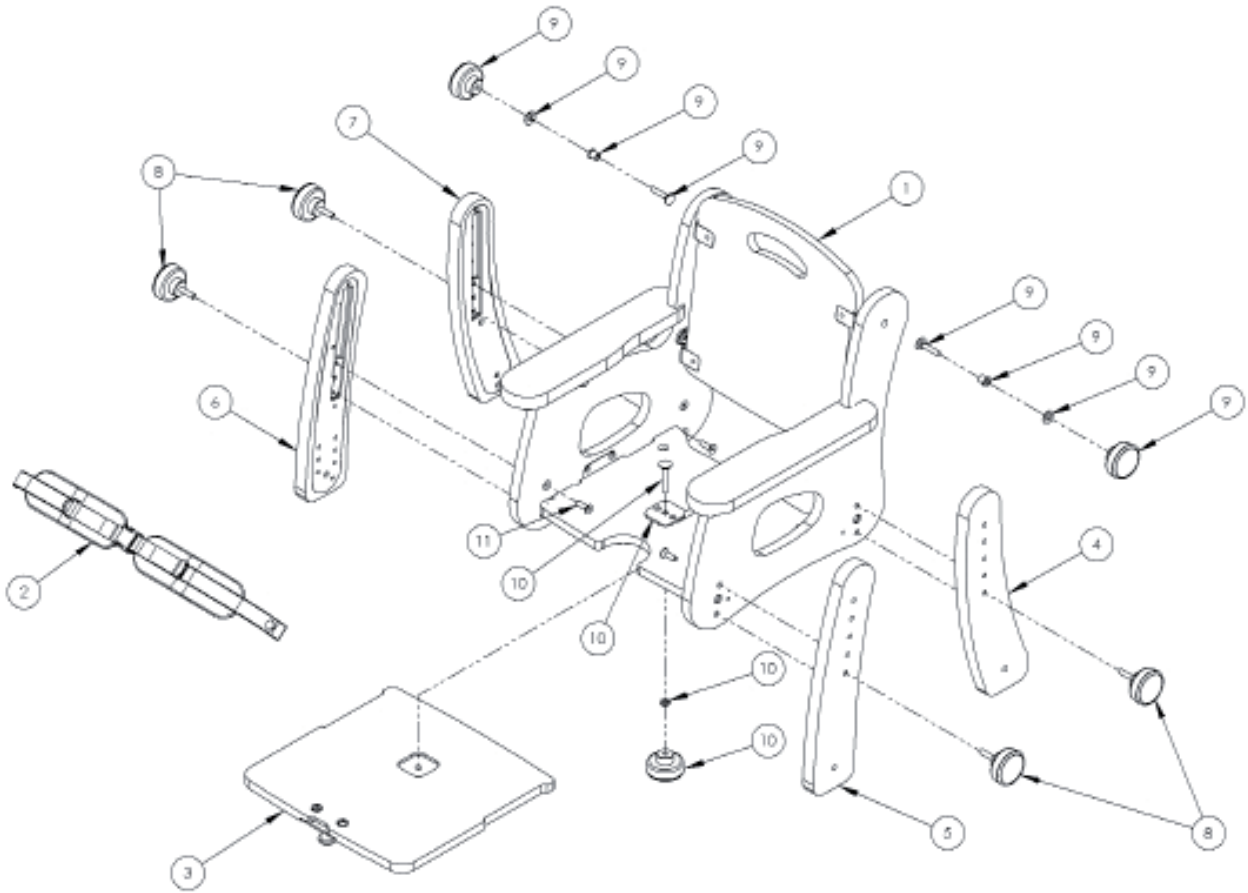


Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
1	135-9001-XX	Seat Size 1	1	✓		
1	135-9002-XX	Seat Size 2	1		✓	
1	135-9003-XX	Seat Size 3	1			✓
2	135-1657	Lap Belt Size 1	1	✓		
2	135-2657	Lap Belt Size 2	1		✓	
2	135-3657	Lap Belt Size 3	1			✓
3	135-1108	Sliding Seatbase Size 1	1	✓		
3	135-2108	Sliding Seatbase Size 2	1		✓	
3	135-3108	Sliding Seatbase Size 3	1			✓
4	135-1612	Leg Rear Left Size 1	1	✓		
4	135-2612	Leg Rear Left Size 2	1		✓	
4	135-3612	Leg Rear Left Size 3	1			✓

PAL Seat Assembly


Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
5	135-1618	Leg Front Left Size 1	1	✓		
5	135-2618	Leg Front Left Size 2	1		✓	
5	135-3618	Leg Front Left Size 3	1			✓
6	135-1611	Leg Front Right Size 1	1	✓		
6	135-2611	Leg Front Right Size 2	1		✓	
6	135-3611	Leg Front Right Size 3	1			✓
7	135-1613	Leg Rear Right Size 1	1	✓		
7	135-2613	Leg Rear Right Size 2	1		✓	
7	135-3613	Leg Rear Right Size 3	1			✓
8	SF90	M6 Knob Male X 25 Thread	4	✓	✓	✓
9	SP-1041	Backrest Knob & Screw Kit	4	✓	✓	✓
10	135-9000	Sliding Seatbase Knob & Screw Kit	1	✓	✓	✓
11	SP-1042	Leg Attachment Screw Kit	4	✓	✓	✓

PAL - High Seat Assembly

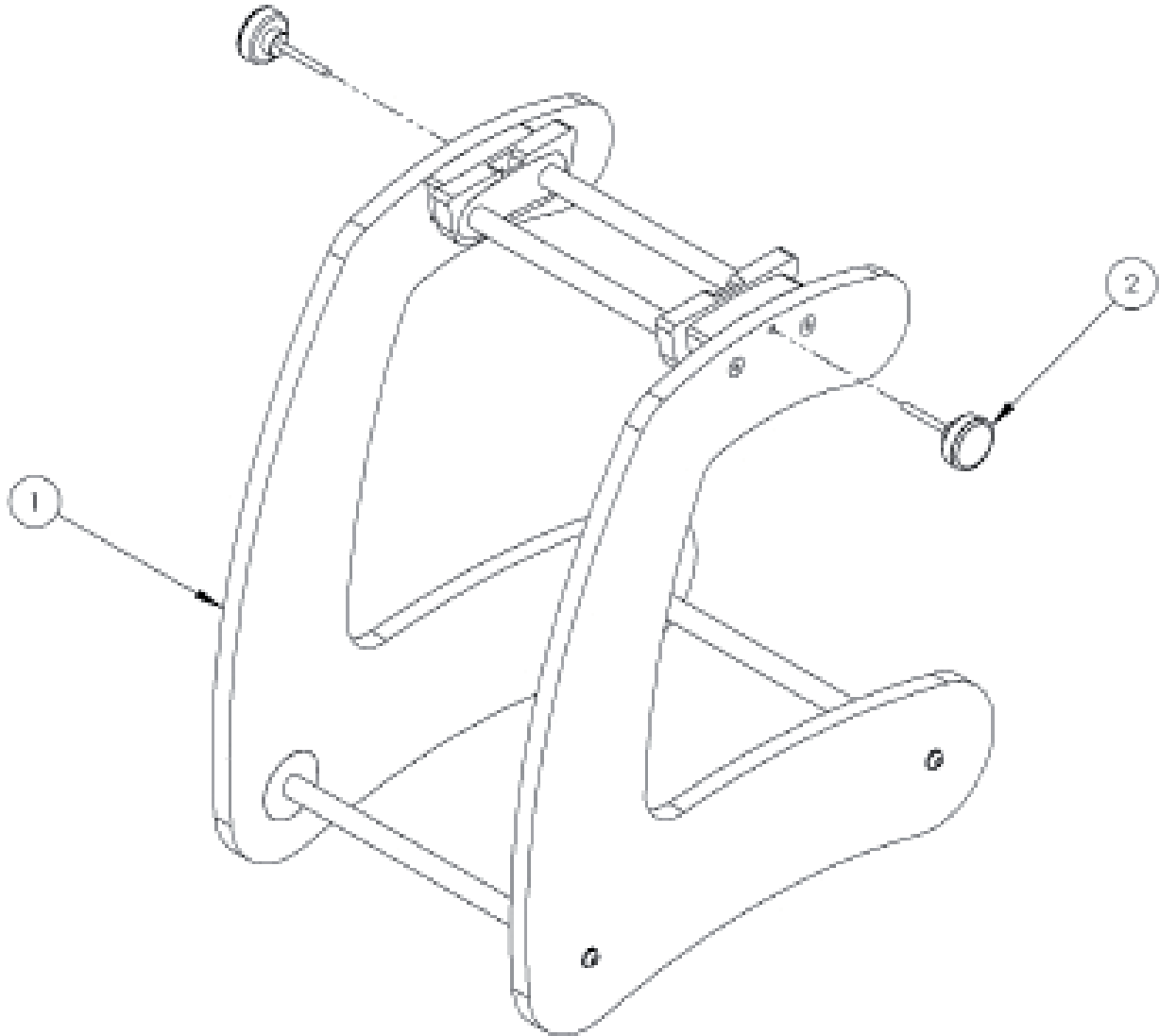


Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
1	135-1630	High Seat Frame (Size 1 Only)	1	✓		
2	SF90	Wixroyd Knob M6	2	✓		

PAL -Tray Assembly

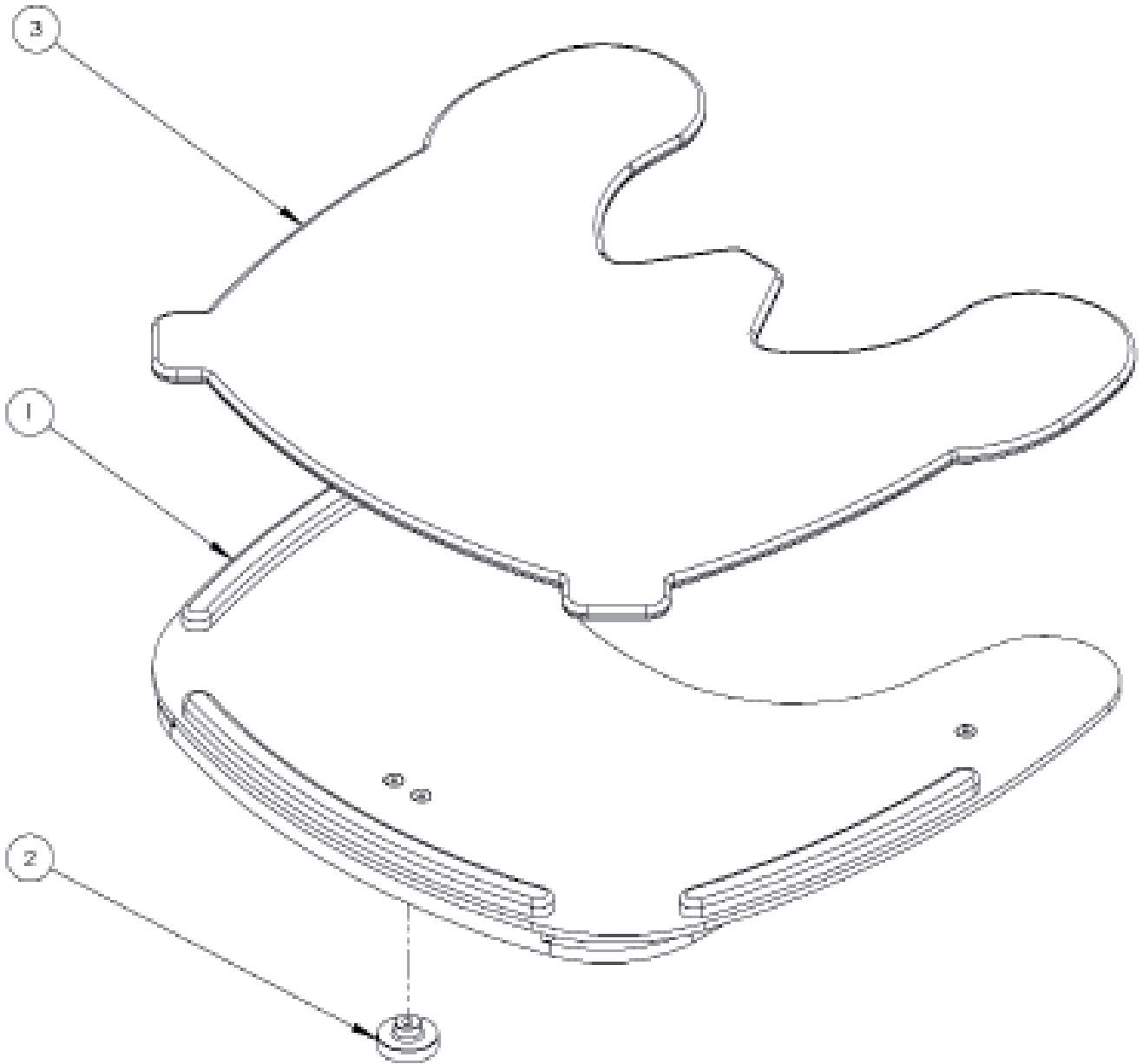


Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
1	135-1620	Activity Tray Size 1	1	✓		
1	135-2620	Activity Tray Size 2	1		✓	
1	135-3620	Activity Tray Size 3	1			✓
2	SF90	Wixroyd Knob M6	1	✓	✓	✓
3	135-1656	Tray Pad Size 1	1	✓		
3	135-2656	Tray Pad Size 2	1		✓	
3	135-3656	Tray Pad Size 3	1			✓

PAL -Ski Assembly

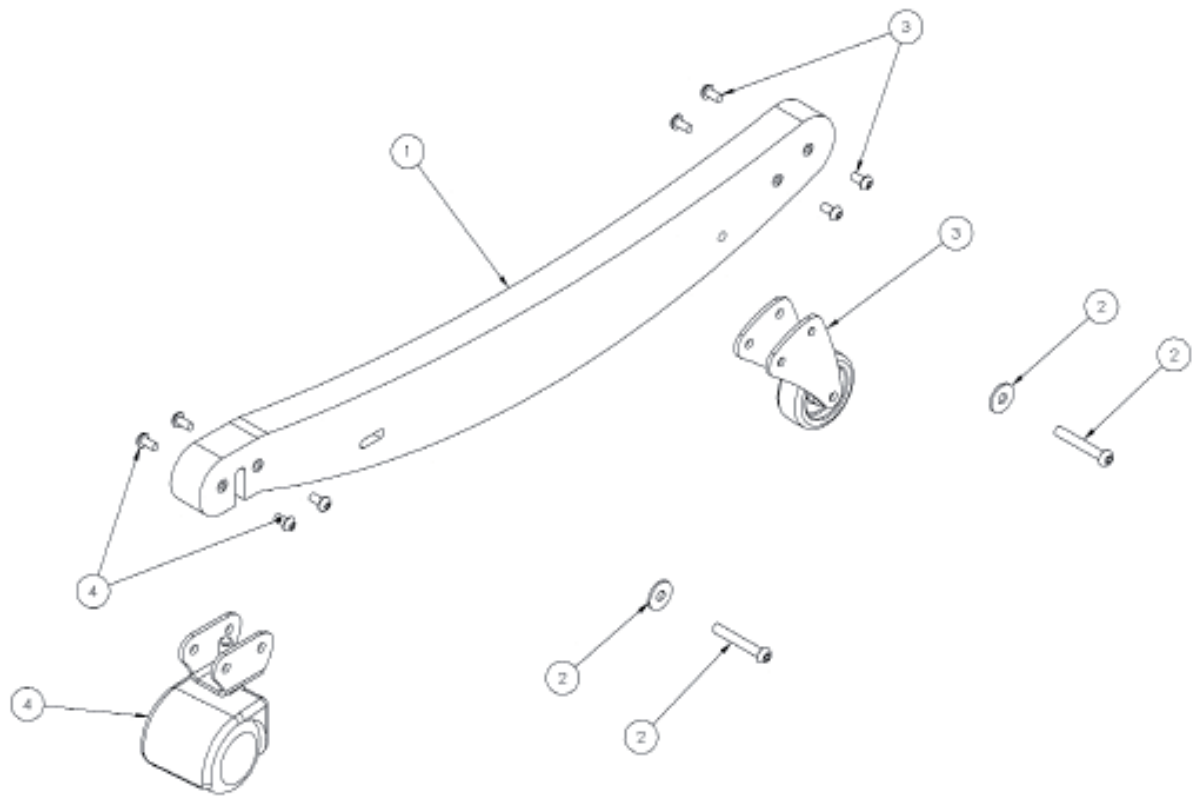


Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
1	135-1610	Ski With Castors Size 1	1	✓		
1	135-2610	Ski With Castors Size 2	1		✓	
1	135-3610	Ski With Castors Size 3	1			✓
2	SP-1043	Ski Attachment Screw Kit	1	✓	✓	✓
3	135-9004	Front Castor Assembly	1	✓	✓	✓
4	135-9005	Rear Castor Assembly	1	✓	✓	✓

PAL -Footrest Assembly

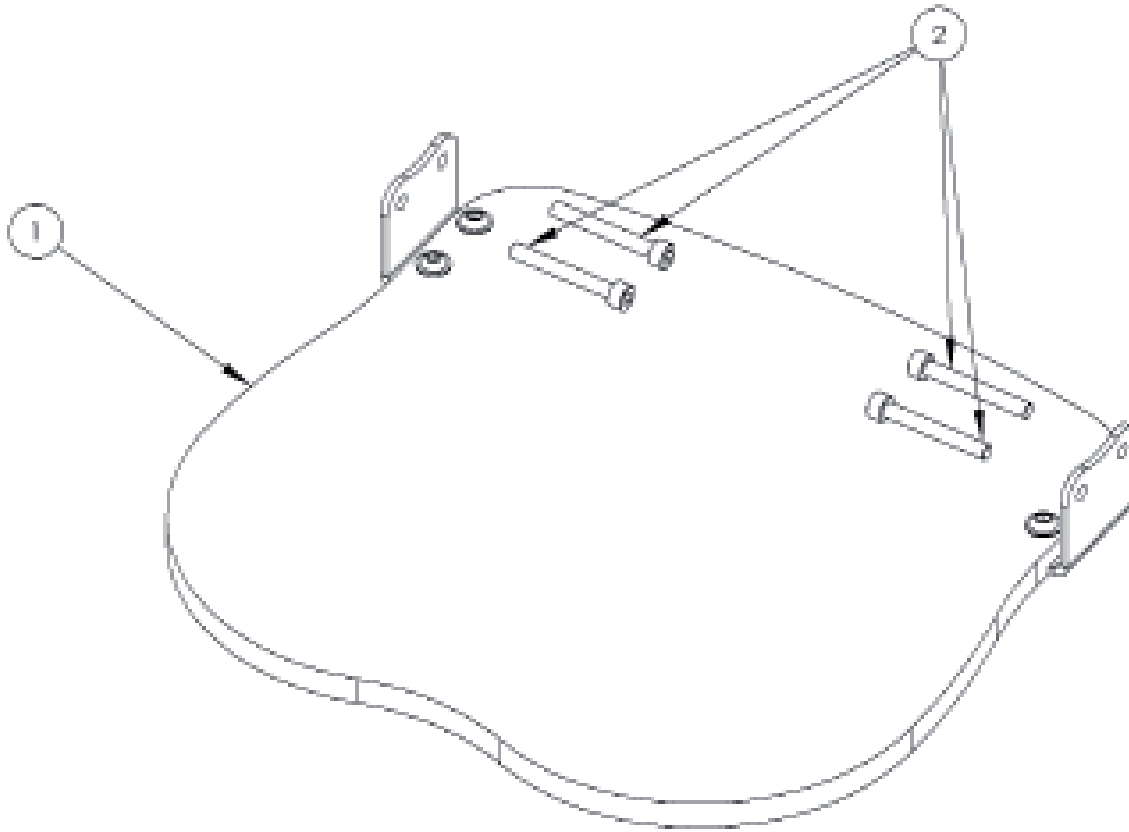


Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
1	135-1603	Footrest Size 1	1	✓		
1	135-2603	Footrest Size 2	1		✓	
1	135-3603	Footrest Size 3	1			✓
2	SP-1044	Footrest Attachment Screw Kit	1	✓	✓	✓

PAL - Two Person Table Assembly

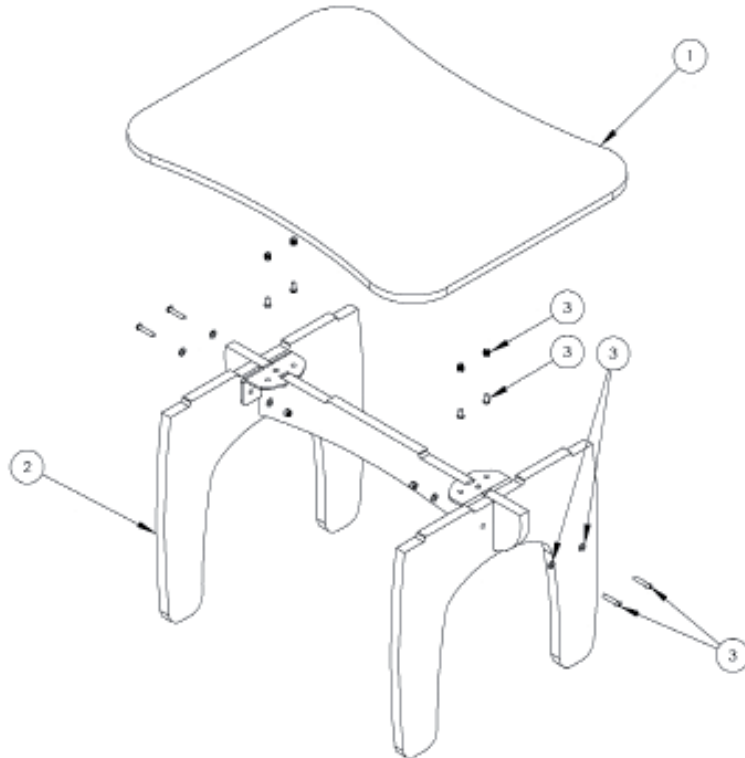


Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
1	135-1710-XX	Table Top	1	✓	✓	✓
2	135-1715	Table Leg Assembly Size 1	1	✓		
2	135-2715	Table Leg Assembly Size 2	1		✓	
2	135-3715	Table Leg Assembly Size 3	1			✓
3	SP-1045	Table Attachment Screw Kit	1	✓	✓	✓

PAL - Four Person Table Assembly

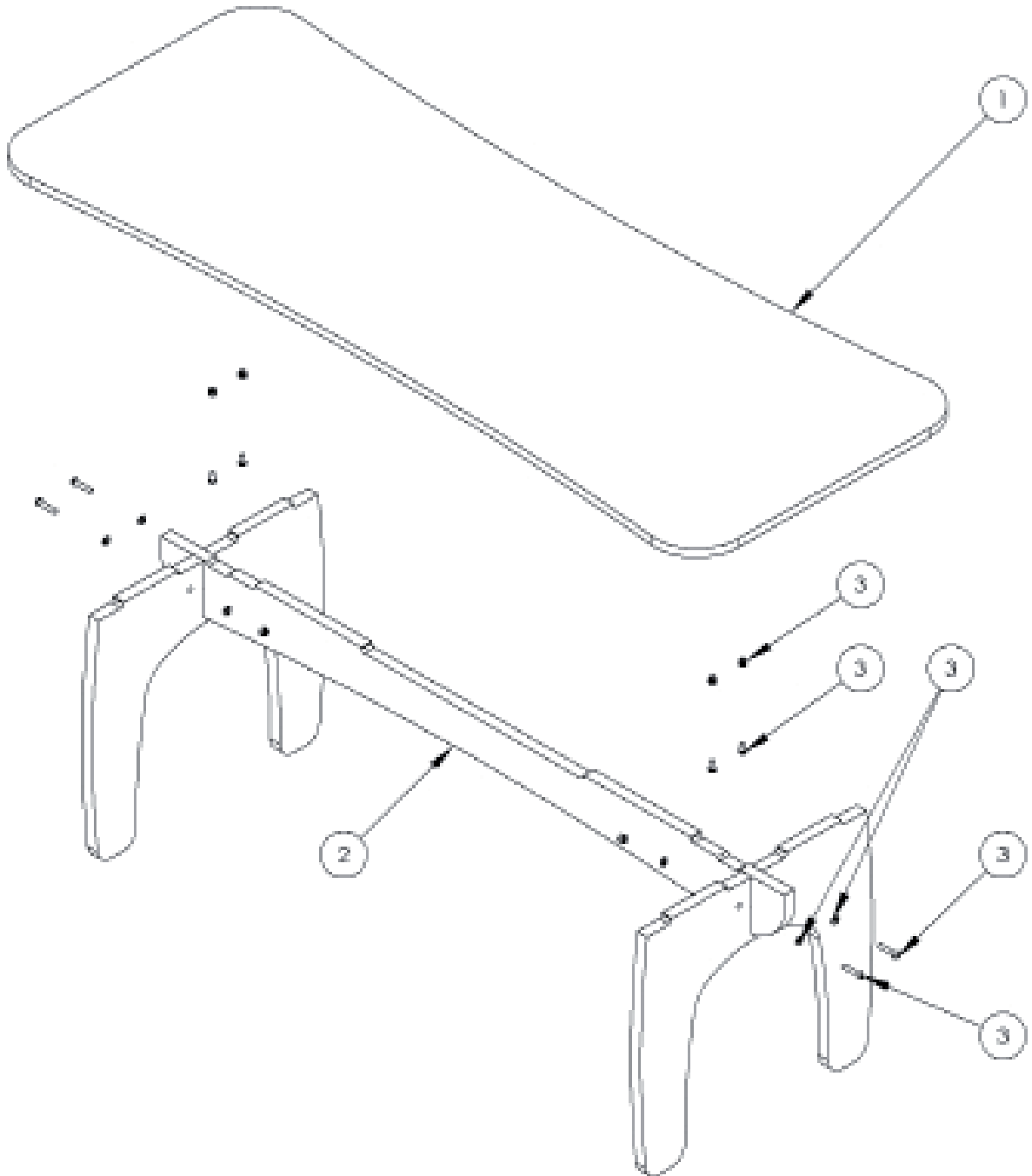

Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
1	135-1720-XX	Table Top	1	✓	✓	✓
2	135-1715	Table Leg Assembly Size 1	1	✓		
2	135-2715	Table Leg Assembly Size 2	1		✓	
2	135-3715	Table Leg Assembly Size 3	1			✓
3	SP-1045	Table Attachment Screw Kit	1	✓	✓	✓

PAL - Rocker/Stabiliser Assembly

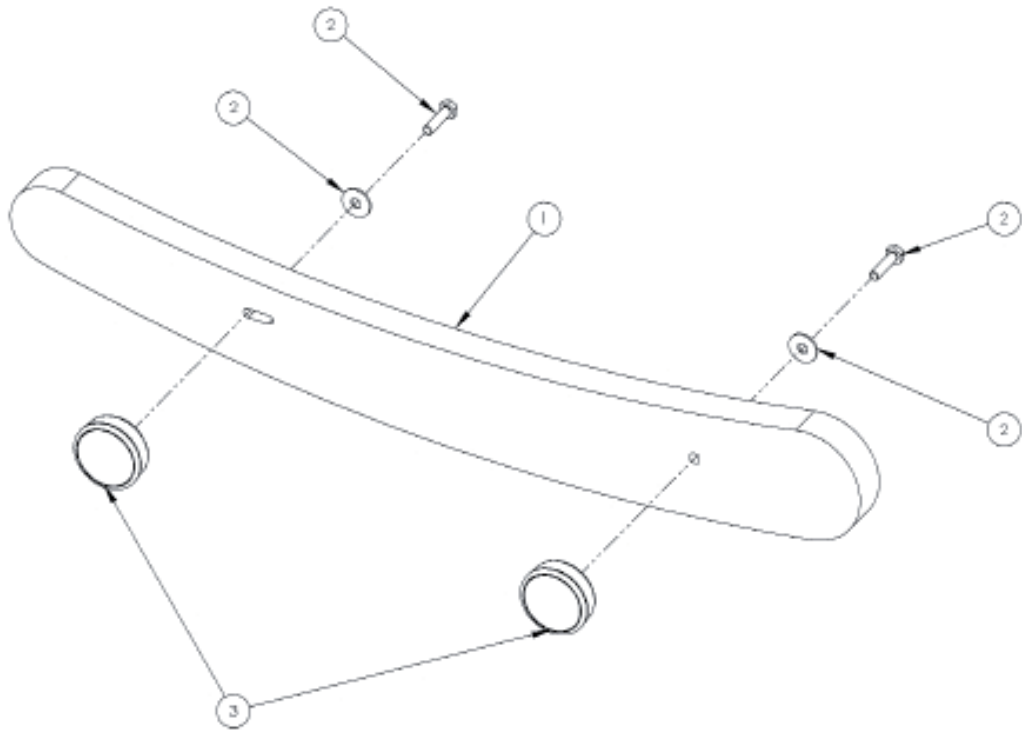

Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
1	135-1120	Rocker/Stabiliser Size 1	1	✓		
1	135-2120	Rocker/Stabiliser Size 2	1		✓	
1	135-3120	Rocker/Stabiliser Size 3	1			✓
2	SP-1046	Rocker/Stabiliser Screw Kit	1	✓	✓	✓
3	SF90	Wixroyd Knob M6	2	✓	✓	✓