
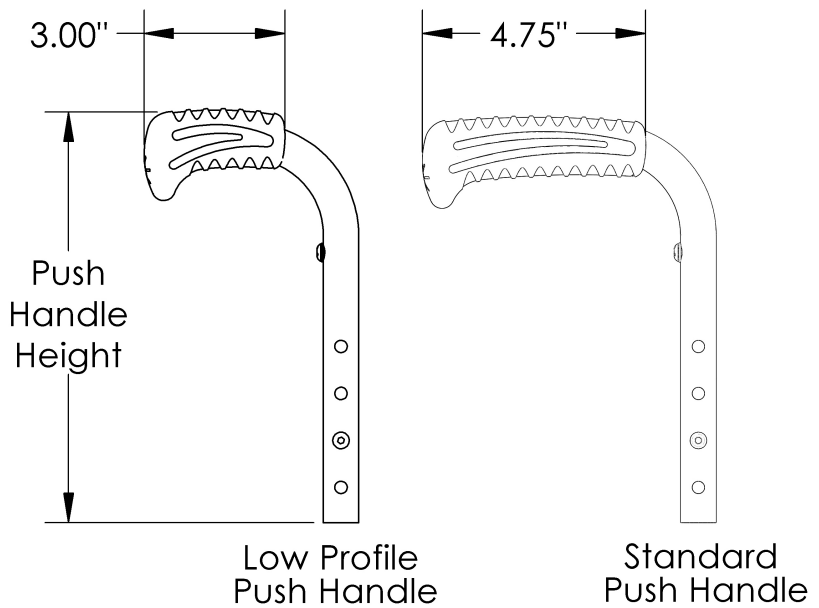


Rigid Push Handle

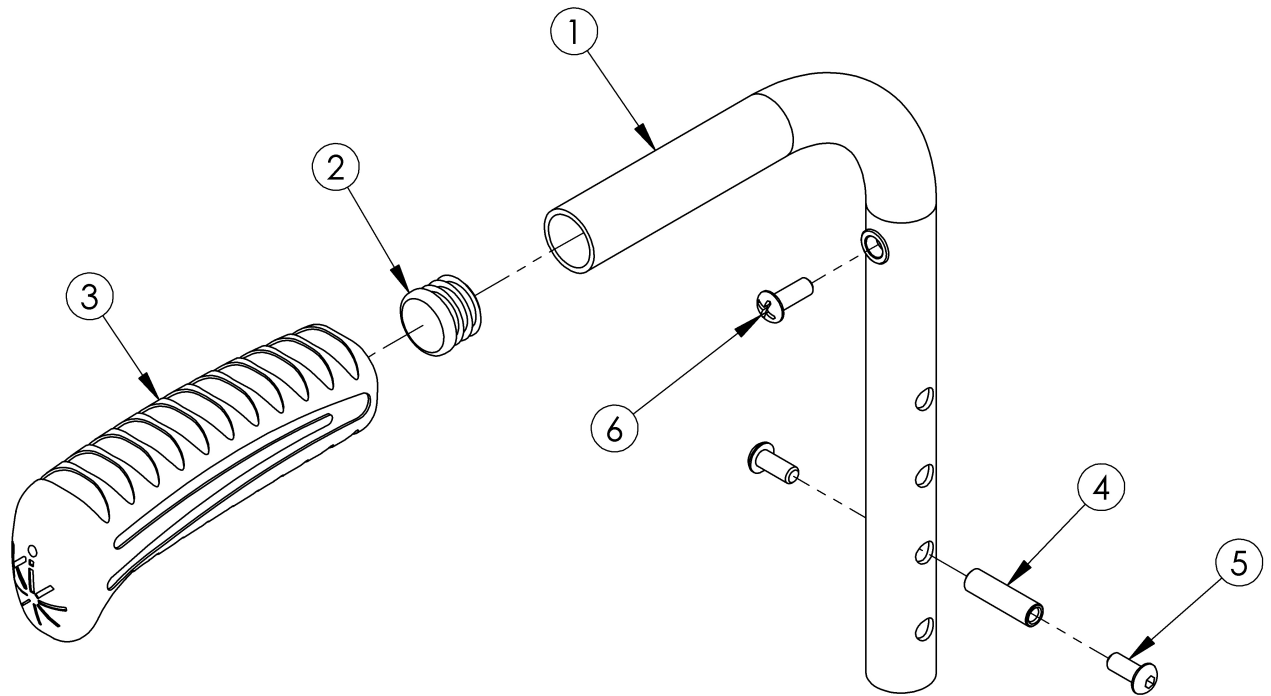

| Size | Height Range | Length |
|--------|--------------|--------|
| Short | 11"-14" | 8.5" |
| Medium | 13"-17" | 11.5" |
| Tall | 17"-20" | 15.5" |

The above chart is in reference to the height adjustable back post w/ push handle. If changing from another style backrest and the Drum Brake is present, a new Drum Brake Cable Kit may be required.

| Item Number | Part Number | Description | Quantity |
|------------------|-------------|---|----------|
| 1a, 2, 3a, 4 - 6 | 500779 | Standard Push Handle w/Hardware Assembly Short Rigid <i>Used with all fixed height backrests w/standard push handle or height adjustable backrests w/standard push handle with a short back height range and drum brake is not present</i> | 1 |

| Item Number | Part Number | Description | Quantity |
|------------------|-------------|---|----------|
| 1b, 2, 3a, 4 - 6 | 506730 | Standard Push Handle w/Hardware Assembly Medium Rigid <i>Used with height adjustable backrests w/standard push handle with a medium back height range and drum brake is not present</i> | 1 |
| 1c, 2, 3a, 4 - 6 | 506731 | Standard Push Handle w/Hardware Assembly Tall Rigid <i>Used with height adjustable backrests w/standard push handle with a tall back height range and drum brake is not present</i> | 1 |
| 1a, 2, 3b | 002758 | Low Profile Push Handle Short Rigid <i>Used with all fixed height backrests w/low profile push handle or height adjustable backrests w/low profile push handle with a short back height range</i> | 1 |
| 1b, 2, 3b | 506732 | Back Post Assembly Medium with Low Profile Push Handle Rigid <i>Used with height adjustable backrests w/low profile push handle with a medium back height range</i> | 1 |
| 1c, 2, 3b | 506733 | Back Post Assembly Tall with Low Profile Push Handle Rigid <i>Used with height adjustable backrests w/low profile push handle with a tall back height range</i> | 1 |
| 1d, 2, 3a | 113221 | Backtube w/ Push Handle Short with Backcane Extension, Drum Brake, Rigid <i>Used with all fixed height backrests w/standard push handle or height adjustable backrests w/standard push handle with short back height range and drum brake is present</i> | 1 |
| 1e, 2, 3a | 113222 | Backtube w/ Push Handle Tall with Backcane Extension, Drum Brake, Rigid <i>Used with height adjustable backrests w/standard push handle with a medium back height range and drum brake is present</i> | 1 |
| 1f, 2, 3a | 113223 | Backtube w/ Push Handle Tall with Backcane Extension, Drum Brake, Rigid <i>Used with height adjustable backrests w/standard push handle with a tall back height range and drum brake is present</i> | 1 |
| 2 | 100638 | Tube End Plug 3/4" | 1 |
| 3a | 109432 | Ergo Push Handle Grip 3/4" Standard with Instruction | 1 |
| 3b | 109430 | Ergo Push Handle Grip 3/4" Low Profile with Instruction | 1 |
| 4 | 000073 | M5 x 24 Threaded Barrel | 1 |
| 5 | 100669 | M5 x 10 BTN SHCS BLKZC w/Patch | 2 |
| 6 | 100734 | M5 x 16 PHMS ZC <i>Used to mount back upholstery</i> | 1 |