



# INSTALLATION GUIDE

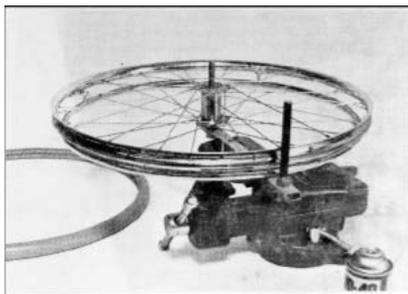


## STEP 1

Remove center rod from tool. Mount tool in vise. Place wheel on tool with hand rim down. Three different size bolts should be used to assure proper axel fit.

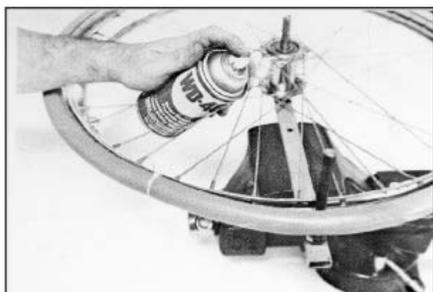

## STEP 2

Secure tire to wheel with tie. Place tire outside roller rod as shown and spray lubricant on rod.

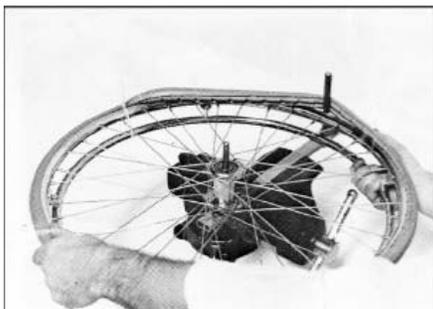

## STEP 3

Slowly rotate tire clockwise to stretch tire and guide it onto rim. Tire will ride up and be partially mounted as shown. Use both hands to employ several short turning movements.



## STEP 4

Continue turning clockwise and stop when you come to the tire installed in step 1.



## STEP 5

Slowly rotate wheel counter-clockwise one full revolution and tire will seat itself. Remove wheel from tool by pulling straight up. Remove tie and installation is complete.