


# Focus CR Dual Hand Tilt Mechanism Stroller Back

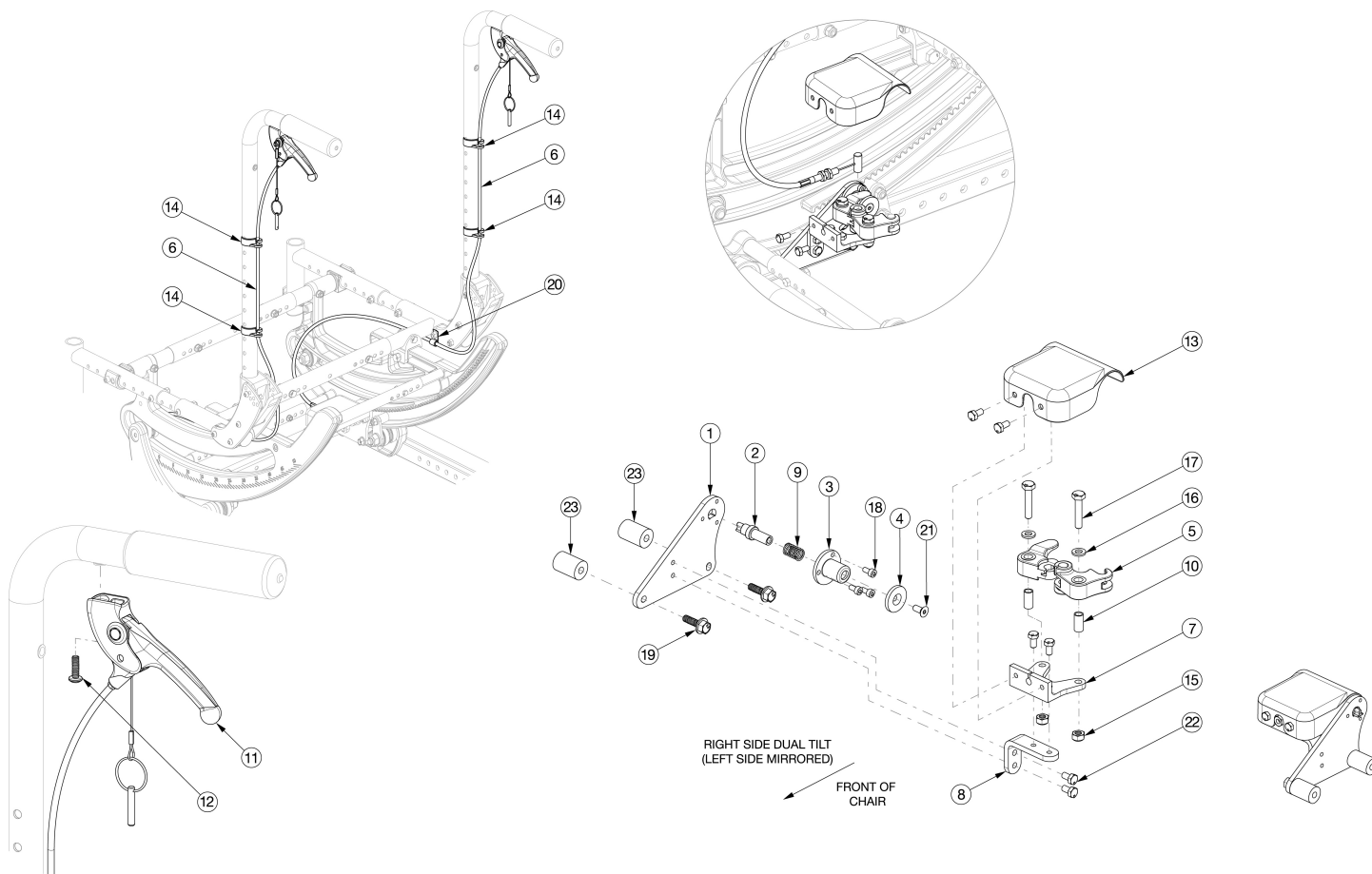

This design effective starting with serial numbers FCR0026599 and FCHD002333. <p>

If serial number is prior to FCR0026599 and FCHD002333 new back cane assemblies are required in addition to the hand tilt mechanism.

| Item Number                               | Part Number | Description                                                                                                                                                             | Quantity | Retail   |
|-------------------------------------------|-------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------|----------|
| 1-5, 7a, 7b, 8-10, 13, 15-18, 19a, 21, 22 | 506274      | Dual Tilt Lock Mechanism Kit, Focus<br><i>For complete dual hand tilt mechanism, select appropriate Hand Tilt Cable/Trigger Kit in addition to this kit</i>             | 1        | \$443.14 |
| 1-5, 7a, 7b, 8-10, 13, 15-18, 19b, 21-23  | 506275      | Dual Tilt Lock Mechanism Kit, Heavy Duty, Focus<br><i>For complete dual hand tilt mechanism, select appropriate Hand Tilt Cable/Trigger Kit in addition to this kit</i> | 1        | \$555.77 |
| 6a, 11, 12, 14, 20                        | 506784      | Hand Tilt Cable/Trigger Kit with Short Cable, Stroller Handle, Focus<br><i>Used with Short Stroller Back Post</i>                                                       | 2        | \$132.60 |
| 6b, 11, 12, 14, 20                        | 506785      | Hand Tilt Cable/Trigger Kit with Medium Cable, Stroller Handle, Focus<br><i>Used with Tall Stroller Back Post</i>                                                       | 2        | \$133.48 |
| 1                                         | 004280      | Mount Plate, Tilt Lock                                                                                                                                                  | 2        | \$14.49  |

| Item Number                                 | Part Number | Description                                                                                 | Quantity | Retail   |
|---------------------------------------------|-------------|---------------------------------------------------------------------------------------------|----------|----------|
| 2                                           | 004281      | Pin, Plunger, Tilt Lock                                                                     | 2        | \$61.67  |
| 3                                           | 004282      | Housing, Plunger, Tilt Lock                                                                 | 2        | \$10.51  |
| 4                                           | 004283      | Thrust Washer, Tilt Lock                                                                    | 2        | \$7.25   |
| 5                                           | 004284      | Cam Lever, Drive, Tilt Lock                                                                 | 2        | \$32.35  |
| 1-5, 7a, 8-10,<br>13, 15-18, 19a,<br>21, 22 | 506276      | Tilt Lock Mechanism Dual Hand, Right                                                        | 1        | \$221.57 |
| 1-5, 7b, 8-10,<br>13, 15-18, 19a,<br>21, 22 | 506277      | Tilt Lock Mechanism Dual Hand, Left                                                         | 1        | \$221.57 |
| 1-5, 7a, 8-10,<br>13, 15-18, 19b,<br>21-23  | 506278      | Tilt Lock Mechanism Dual Hand, Heavy Duty<br>Right                                          | 1        | \$277.89 |
| 1-5, 7b, 8-10,<br>13, 15-18, 19b,<br>21-23  | 506279      | Tilt Lock Mechanism Dual Hand, Heavy Duty Left                                              | 1        | \$277.89 |
| 6a                                          | 004301      | Cable, Tilt, Medium<br><i>Used with Short Stroller Back Post</i>                            | 2        | \$106.11 |
| 6b                                          | 004302      | Cable, Tilt, Long<br><i>Used with Tall Stroller Back Post</i>                               | 2        | \$108.21 |
| 7a                                          | 004287      | Linkage Mount, Right                                                                        | 1        | \$46.23  |
| 7b                                          | 004288      | Linkage Mount, Left                                                                         | 1        | \$46.23  |
| 8                                           | 004289      | Bracket, Mount, Tilt Lock                                                                   | 2        | \$15.54  |
| 9                                           | 004290      | Spring, Comp, .42OD x 1.0L x .038W                                                          | 2        | \$1.05   |
| 10                                          | 004291      | Bushing, Pivot, Tilt Lock                                                                   | 2        | \$6.83   |
| 11                                          | 003285      | Lever Assembly, Angle 135°                                                                  | 2        | \$24.95  |
| 12                                          | 100734      | M5 x 16 PHMS ZC                                                                             | 2        | \$1.58   |
| 13                                          | 004298      | Cover, Tilt Lock                                                                            | 2        | \$13.03  |
| 14                                          | 001117      | Cable Clip 1"                                                                               | 4        | \$3.83   |
| 15                                          | 100658      | M6 Nylock Nut Blk Zn                                                                        | 2        | \$1.05   |
| 16                                          | 100746      | M6 Flat Washer Blk Zn                                                                       | 2        | \$1.58   |
| 17                                          | 101824      | M6 x 30 HHCS Blk Zn                                                                         | 2        | \$2.10   |
| 18                                          | 102800      | M4 x 8mm SHCS Blk Zn                                                                        | 6        | \$0.53   |
| 19a                                         | 103510      | M6 x 1 x 20 HHCS SFLG CLS8.8 Blk Zn<br>DIN6921<br><i>Used with Standard Weight Capacity</i> | 4        | \$1.05   |

| Item Number | Part Number | Description                                                                          | Quantity | Retail  |
|-------------|-------------|--------------------------------------------------------------------------------------|----------|---------|
| 19b         | 103511      | M6 x 1 x 45 HHCS SFLG CLS8.8 Blk Zn<br>DIN6921<br><i>Used with Heavy Duty Option</i> | 4        | \$3.15  |
| 20          | 001239      | P-Clip 3/16"                                                                         | 2        | \$6.30  |
| 21          | 103522      | M5 x 0.8 x 12 FHCS CLS 10.9 Nyl Blk Zn<br>DIN7991                                    | 2        | \$1.05  |
| 22          | 200427      | M5 x 10 CLS8.8 HHCS ZC w/Patch DIN933                                                | 12       | \$0.53  |
| 23          | 001405      | Spacer, .69OD x .24ID x .1.01Lg<br><i>Used with Heavy Duty Option</i>                | 4        | \$26.05 |