


Catalyst 5 Standard Caster Housing for Swing Away Frame

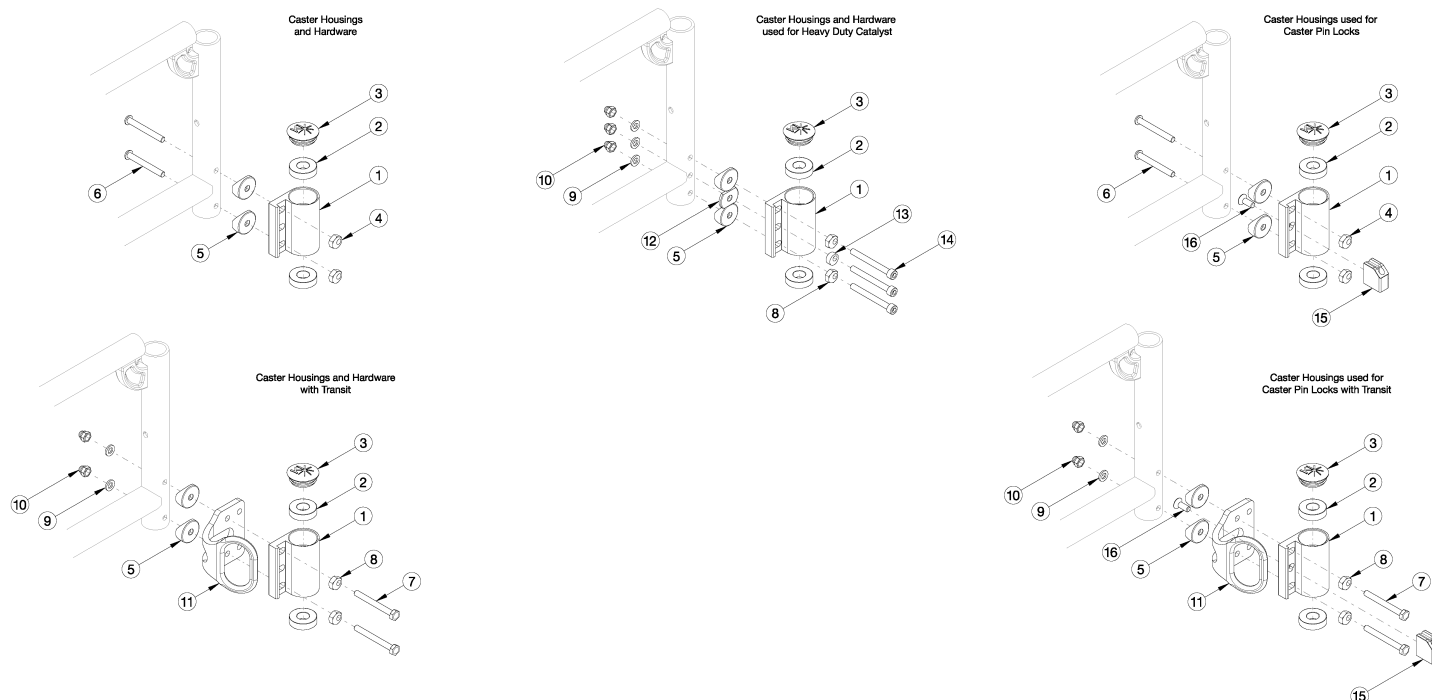

Item Number	Part Number	Description	Quantity	Retail
1a, 2-6	501172	Standard Caster Housing Assembly, Black, Left, No Transit, Standard Weight Capacity, Catalyst	1	\$46.38
1b, 2-6	501172_ _ _	Color Anodized Standard Caster Housing Assembly, Left, No Transit, Standard Weight Capacity, Catalyst <i>Not available with Heavy Duty on chairs starting with serial numbers C40043964, C50099247 or CVX0099946</i>	1	\$114.67
1a, 2, 3, 5, 8, 9, 10, 12, 13, 14	114594	Standard Caster Housing Assembly, Black, Left, No Transit, Heavy Duty Weight Capacity, Catalyst	1	\$79.27
1c, 2-6	501173	Standard Caster Housing Assembly, Black, Right, No Transit, Standard Weight Capacity, Catalyst	1	\$46.38
1d, 2-6	501173_ _ _	Color Anodized Standard Caster Housing Assembly, Right, No Transit, Standard Weight Capacity, Catalyst <i>Not available with Heavy Duty on chairs starting with serial numbers C40043964, C50099247 or CVX0099946</i>	1	\$114.67
1c, 2, 3, 5, 8, 9, 10, 12, 13, 14	114595	Standard Caster Housing Assembly, Black, Right, No Transit, Heavy Duty Weight Capacity, Catalyst	1	\$79.27
1a, 1c, 2-6	501174	Standard Caster Housing Assembly, Black, PAIR, No Transit, Standard Weight Capacity, Catalyst	1	\$92.78
1b, 1d, 2-6	501174_ _ _	Color Anodized Standard Caster Housing Assembly, PAIR, No Transit, Standard Weight Capacity, Catalyst <i>Not available with Heavy Duty on chairs starting with serial numbers C40043964, C50099247 or CVX0099946</i>	1	\$229.35
1a, 1c, 2, 3, 5, 8, 9, 10, 12, 13, 14	114596	Standard Caster Housing Assembly, Black, PAIR, No Transit, Heavy Duty Weight Capacity, Catalyst	1	\$158.54

Item Number	Part Number	Description	Quantity	Retail
1a, 2, 3, 5, 7-10	501175	Standard Caster Housing Assembly, Black, Left, Transit, Standard Weight Capacity, Catalyst <i>Transit Bracket not included in assembly</i>	1	\$50.87
1b, 2, 3, 5, 7-10	501175_ _ _	Color Anodized Standard Caster Housing Assembly, Left, Transit, Standard Weight Capacity, Catalyst <i>Transit Bracket not included in assembly. Not available with Heavy Duty on chairs starting with serial numbers C40043964, C50099247 or CVX0099946.</i>	1	\$119.16
1c, 2, 3, 5, 7-10	501176	Standard Caster Housing Assembly, Black, Right, Transit, Standard Weight Capacity, Catalyst <i>Transit Bracket not included in assembly</i>	1	\$50.87
1d, 2, 3, 5, 7-10	501176_ _ _	Color Anodized Standard Caster Housing Assembly, Right, Transit, Standard Weight Capacity, Catalyst <i>Transit Bracket not included in assembly. Not available with Heavy Duty on chairs starting with serial numbers C40043964, C50099247 or CVX0099946.</i>	1	\$119.16
1a, 1c, 2, 3, 5, 7-10	501177	Standard Caster Housing Assembly, Black, PAIR, Transit, Standard Weight Capacity, Catalyst <i>Transit Bracket not included in assembly</i>	1	\$101.75
1b, 1d, 2, 3, 5, 7-10	501177_ _ _	Color Anodized Standard Caster Housing Assembly, PAIR, Transit, Standard Weight Capacity, Catalyst <i>Transit Bracket not included in assembly. Not available with Heavy Duty on chairs starting with serial numbers C40043964, C50099247 or CVX0099946.</i>	1	\$238.32
1a, 2	100526	Caster Housing Left, Black, CAT/LFT	1	\$37.66
1b, 2	506333_ _ _	Color Anodized Standard Caster Housing Left Catalyst <i>Not available with Heavy Duty on chairs starting with serial numbers C40043964, C50099247 or CVX0099946</i>	1	\$105.95
1c, 2	100527	Caster Housing Right, Black, CAT/LFT	1	\$37.66
1d, 2	506334_ _ _	Color Anodized Standard Caster Housing Right Catalyst <i>Not available with Heavy Duty on chairs starting with serial numbers C40043964, C50099247 or CVX0099946</i>	1	\$105.95
1e, 2	001431	Standard Caster Housing for Caster Pin Lock, Left, Catalyst, Black <i>Not available with Heavy Duty option</i>	1	\$37.66
1f, 2	001431_ _ _	Color Anodized Standard Caster Housing for Caster Pin Lock, Left, Catalyst <i>Not available with Heavy Duty option</i>	1	\$105.95
1g, 2	001432	Standard Caster Housing for Caster Pin Lock, Right, Catalyst, Black <i>Not available with Heavy Duty option</i>	1	\$37.66
1h, 2	001432_ _ _	Color Anodized Standard Caster Housing for Caster Pin Lock, Right, Catalyst <i>Not available with Heavy Duty option</i>	1	\$105.95
2	100530	Bearing M28 x 1/2" 6001RS	2	\$12.02
3	102005	Ki Caster Housing Cap	1	\$1.17
4, 6	112171	Standard Caster Housing Attaching Hardware for Swing Away Frame, No Transit, Standard Weight Capacity, PAIR	1	\$12.93

Item Number	Part Number	Description	Quantity	Retail
4, 6, 15, 16	114599	Standard Caster Housing Attaching Hardware for Swing Away Frame with Caster Pin Locks, No Transit, Standard Weight Capacity, PAIR <i>Not all hardware used with Caster Pin Locks is shown in image</i>	1	\$59.77
4, 5, 6	114600	Standard Caster Housing Attaching Hardware Including Saddles for Swing Away Frame, No Transit, Standard Weight Capacity, PAIR	1	\$19.25
4, 5, 6, 15, 16	114601	Standard Caster Housing Attaching Hardware Including Saddles for Swing Away Frame with Caster Pin Locks, No Transit, Standard Weight Capacity, PAIR <i>Not all hardware used with Caster Pin Locks is shown in image</i>	1	\$62.93
5	000056	Saddle 1 1/8"	2	\$1.58
7-10	112956	Standard Caster Housing Attaching Hardware for Swing Away Frame, Transit, Standard Weight Capacity, PAIR <i>Transit Brackets not included in assembly</i>	1	\$23.00
7-10, 15, 16	114602	Standard Caster Housing Attaching Hardware for Swing Away Frame with Caster Pin Locks, Transit, Standard Weight Capacity, PAIR <i>Transit Brackets not included in assembly. Not all hardware used with Caster Pin Locks is shown in image</i>	1	\$69.84
5, 7-10	114603	Standard Caster Housing Attaching Hardware Including Saddles for Swing Away Frame, Transit, Standard Weight Capacity, PAIR <i>Transit Brackets not included in assembly</i>	1	\$29.32
5, 7-10, 15, 16	114604	Standard Caster Housing Attaching Hardware Including Saddles for Swing Away Frame with Caster Pin Locks, Transit, Standard Weight Capacity, PAIR <i>Transit Brackets not included in assembly. Not all hardware used with Caster Pin Locks is shown in image</i>	1	\$76.16
7	100677	M6 x 60 CLS10.9 DIN931 HHCS ZC	2	\$1.58
8	100510	Eccentric Washer	2	\$1.65
9	100746	M6 Flat Washer Blk Zn	3	\$1.58
10	100558	M6 NYL INS Domed Cap L/N ZC	3	\$1.05
12	111813	1 1/8" Half Saddle <i>Only used if Heavy Duty option is present on chairs starting with serial numbers C40043964, C50099247 or CVX0099946</i>	1	\$5.57
13	111544	Eccentric Washer, 0.50" Diameter <i>Only used if Heavy Duty option is present on chairs starting with serial numbers C40043964, C50099247 or CVX0099946</i>	1	\$5.13
14	111814	M6 x 55 SHCS, 45mm Shoulder, 10 mm Thread, CLS 12.9 <i>Only used if Heavy Duty option is present on chairs starting with serial numbers C40043964, C50099247 or CVX0099946</i>	3	\$5.13
15	200417	Caster Lock Mounting Block CAT	1	\$21.32

Item Number	Part Number	Description	Quantity	Retail
16	101721	M6 x 16 Btn SHCS w/ Patch	1	\$2.10
8, 9, 10, 13, 14	114597	Standard Caster Housing Attaching Hardware for Swing Away Frame, Heavy Duty Weight Capacity, PAIR	1	\$63.42
5, 8, 9, 10, 12, 13, 14	114598	Standard Caster Housing Attaching Hardware Including Saddles for Swing Away Frame, Heavy Duty Weight Capacity, PAIR	1	\$80.88

Color Anodizing Samples

