


Liberty Caster Forks and Stems

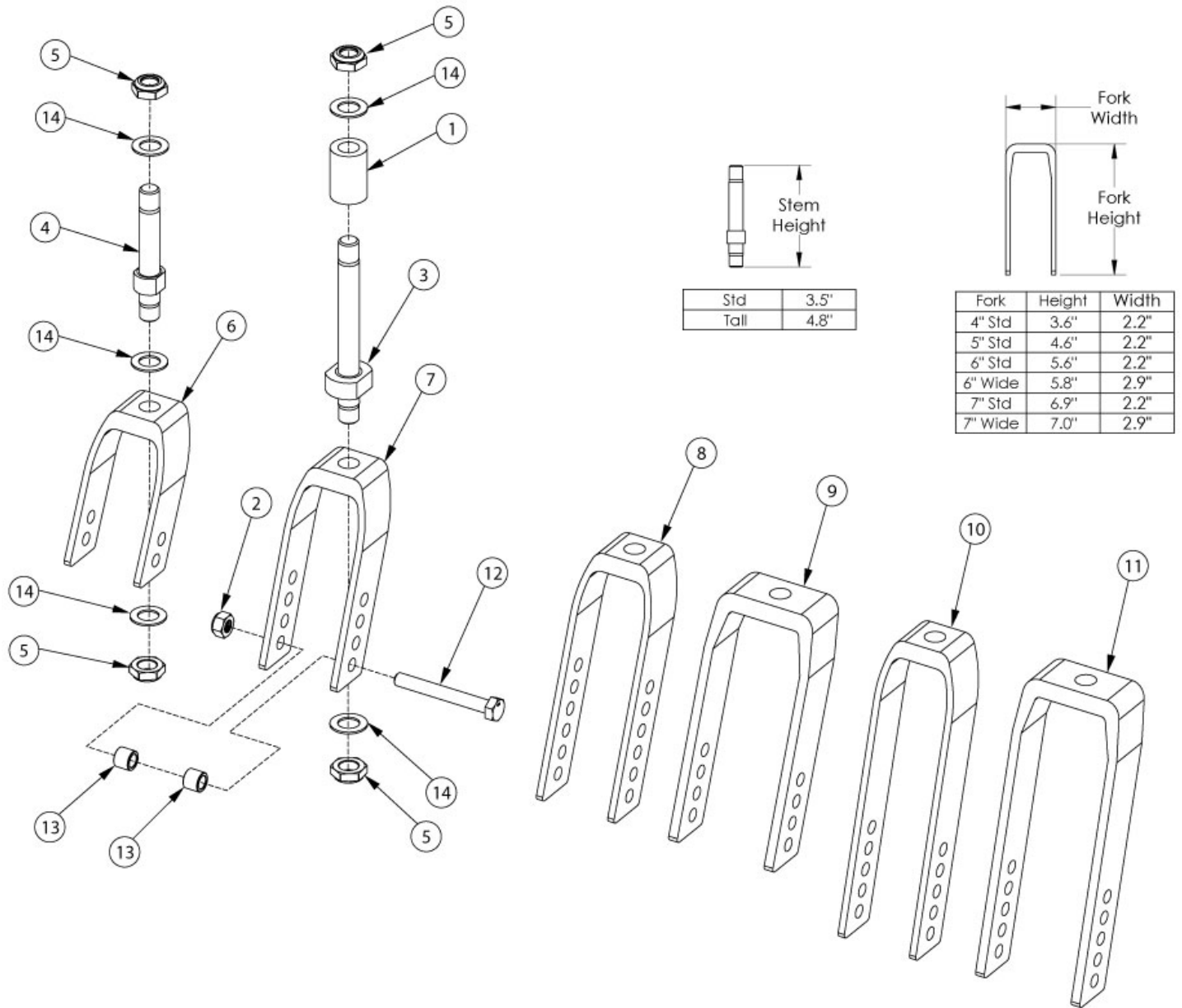

If changing to 2" casters, wide forks and hardware needed.

Item Number	Part Number	Description	Quantity
4, 5, 6, 14	112904	Fork Assembly, Black, 4" Std Stem, Tilt	1
4, 5, 7, 14	503816	Fork Assembly 5" Std Stem, Tilt	1
4, 5, 8, 14	503814	Fork Assembly 6" Std Stem, Tilt	1
4, 5, 9, 14	503817	Fork Assembly 6" Wide Std Stem, Tilt <i>Only used with 2" wide casters</i>	1

Item Number	Part Number	Description	Quantity
4, 5, 10, 14	503815	Fork Assembly 7" Std Stem, Tilt	1
4, 5, 11, 14	503818	Fork Assembly 7" Wide Std Stem, Tilt <i>Only used with 2" wide casters</i>	1
1, 3, 5, 6, 14	506153	Fork Assembly 4" Tall Stem, Tilt	1
1, 3, 5, 7, 14	506154	Fork Assembly 5" Tall Stem, Tilt	1
1, 3, 5, 8, 14	506155	Fork Assembly 6" Tall Stem, Tilt	1
1, 3, 5, 9, 14	506157	Fork Assembly 6" Wide Tall Stem, Tilt <i>Only used with 2" wide casters</i>	1
1, 3, 5, 10, 14	506156	Fork Assembly 7" Tall Stem, Tilt	1
1, 3, 5, 11, 14	506158	Fork Assembly 7" Wide Tall Stem, Tilt <i>Only used with 2" wide casters</i>	1
1	002476	Spacer, 3/4" OD x 1/2" ID x 1.23 <i>Only used with Tall Stems</i>	1
2	100659	M8 Nylock Nut Blk Zn	1
3, 5	506490	Caster Stem Tall, Liberty	1
4, 5	506491	Caster Stem Standard, Liberty	1
5	100529	M12 x 1.25mm Lock Nut Black	2
5, 6	506492	Caster Fork 4", Tilt	1
5, 7	506493	Caster Fork 5", Liberty	1
5, 8	506494	Caster Fork 6", Liberty	1
5, 9	506495	Caster Fork 6", Wide Liberty <i>Only used with 2" wide casters</i>	1
5, 10	506496	Caster Fork 7", Liberty	1
5, 11	506497	Caster Fork 7", Wide Liberty <i>Only used with 2" wide casters</i>	1
12a	101829	M8 x 65 HHCS Blk Zn	1
12b	101831	M8 x 80 HHCS Blk Zn <i>Only used with 2" wide casters</i>	1
13a	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer <i>Used with all casters except 6x2 and 8x2</i>	2
13b	001246	Caster Spacer Black 6 x 2 Poly .472ODX.319IDX.370LG <i>Only used with 6" x 2" casters</i>	2

Item Number	Part Number	Description	Quantity
14	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	3
2, 12a, 13a	111379	Caster Attaching Hardware Kit <i>Used will all casters except 6x2 and 8x2</i>	1
2, 12b, 13b	111380	Caster Attaching Hardware Kit for Wide Forks <i>Only used with 2" wide casters</i>	1