


Rogue2 Backrest Mount and Hardware

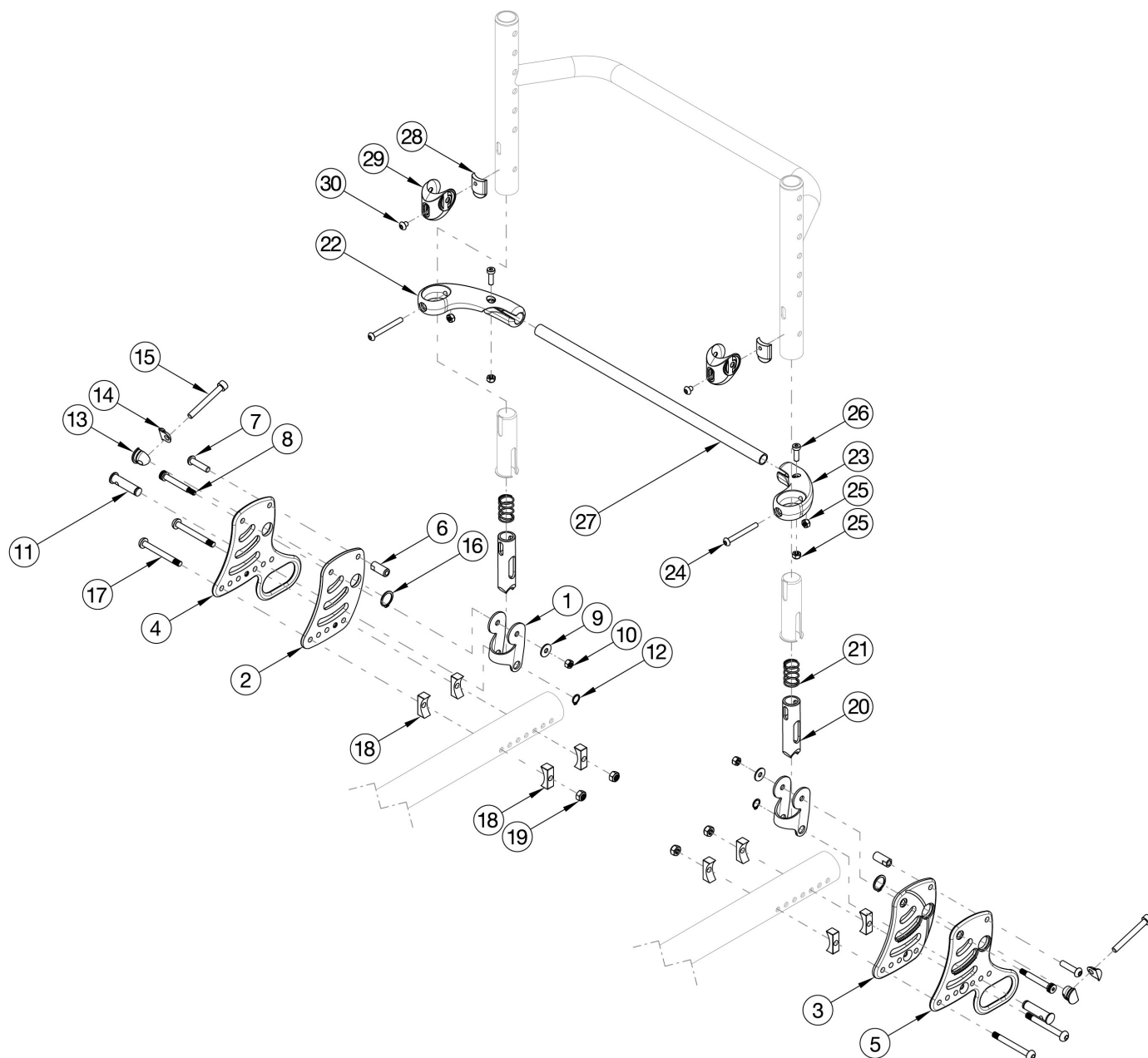

Item Number	Part Number	Description	Quantity
1, 2a, 3a, 6 - 21, 28 - 30	117288	Back Plate and Hardware, Non-Transit, Black, Rogue, PAIR	1
1, 2a, 6 - 19	116894	Back Plate and Hardware, Non-Transit, Black, Right, Rogue	1
1, 3a, 6 - 19	116895	Back Plate and Hardware, Non-Transit, Black, Left, Rogue	1
1, 2b, 3b, 6 - 21, 28 - 30	117290 _ _ _	Back Plate and Hardware, Non-Transit, Color Anodized/Color Accent, Rogue, PAIR	1

Item Number	Part Number	Description	Quantity
1, 2b, 6 - 19	116916_ _ _	Back Plate and Hardware, Non Transit, Color Anodized/Color Accent, Right, Rogue	1
1, 3b, 6 - 19	116917_ _ _	Back Plate and Hardware, Non Transit, Color Anodized/Color Accent, Left, Rogue	1
1, 4 - 21, 28 - 30	117289	Back Plate and Hardware, Transit, Black, Rogue, PAIR	1
1, 4, 6 - 19	116896	Back Plate and Hardware, Transit, Black, Right, Rogue	1
1, 5 - 19	116897	Back Plate and Hardware, Transit, Black, Left, Rogue	1
1, 4 - 21, 28 - 30	117289_ _ _	Back Plate and Hardware, Transit, Color Accent, Rogue, PAIR	1
1, 4, 6 - 19	116896_ _ _	Back Plate and Hardware, Transit, Color Accent, Rogue, Right	1
1, 5 - 19	116897_ _ _	Back Plate and Hardware, Transit, Color Accent, Rogue, Left	1
28 - 30	116876	Relax Subassembly	1
22, 24 - 26	116877	Release Lever with Fasteners, Right	1
23 - 26	116878	Release Lever with Fasteners, Left	1
22 - 26, 27a	116879	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 12" Wide, Black	1
22 - 26, 27b	116880	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 13" Wide, Black	1
22 - 26, 27c	116881	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 14" Wide, Black	1
22 - 26, 27d	116882	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 15" Wide, Black	1
22 - 26, 27e	116883	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 16" Wide, Black	1
22 - 26, 27f	116884	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 17" Wide, Black	1
22 - 26, 27g	116885	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 18" Wide, Black	1
22 - 26, 27h	116886	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 19" Wide, Black	1
22 - 26, 27i	116887	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 20" Wide, Black	1
22 - 26, 27j	117832_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 12" Wide, Painted	1
22 - 26, 27k	117833_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 13" Wide, Painted	1
22 - 26, 27l	117913_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 14" Wide, Painted	1
22 - 26, 27m	117914_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 15" Wide, Painted	1
22 - 26, 27n	117915_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 16" Wide, Painted	1
22 - 26, 27o	117916_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 17" Wide, Painted	1
22 - 26, 27p	117917_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 18" Wide, Painted	1

Item Number	Part Number	Description	Quantity
22 - 26, 27q	117918_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 19" Wide, Painted	1
22 - 26, 27r	117919_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 20" Wide, Painted	1
2a	115550	Back Plate, Right, Black	1
2b	115550_ _ _	Color Anodized/Color Accent Back Plate, Right	1
3a	115551	Back Plate, Left, Black	1
3b	115551_ _ _	Color Anodized/Color Accent Back Plate, Left	1
4	115552	Back Plate, Transit, Right, Black	1
4	115552_ _ _	Color Accent Back Plate, Transit, Right	1
5	115553	Back Plate, Transit, Left, Black	1
5	115553_ _ _	Color Accent Back Plate, Transit, Left	1
6	115546	Strike, Ø.364	2
7	115573	M6 x 25, BHCS, CLS 10.9, NYL, BLZ	2
8a	103284	M6 x 35 Shoulder Bolt BZ	2
8b	115574	M6 x 30 BHCS, CLS 10.9, NYL, BLZ <i>These are required if the chair has swing away armrests</i>	2
9	103022	M5 Washer, Blk Zn	2
10	103283	M5 Top Lock Nut BZ	2
12	115570	Eternal Retaining Ring, 3/8"	2
15	115545	M6 x 1.0 x 50 SHCS, FT, NYL, BLZ	2
16	115571	Eternal Retaining Ring, 5/8"	2
17a	003232	M6 x 55 BHCS Blk Zc	2
17b	003241	M6 x 60 BHCS P/THD Custom Blk Zc <i>These are required if the chair has swing away armrests</i>	2
18	116329	Rigid Backrest Saddle, Narrow, 1.375"	8
19	100658	M6 Nylock Nut Blk Zn	4
20	115539	Release Pin	2
21	115547	Compression Spring, .66" OD x 1" LG	2
24	000142	M5 x 45 BHCS, P/THD, BLZ	1

Item Number	Part Number	Description	Quantity
25	100657	M5 Nylock Nut Blk Zn	2
26	102801	M5 x 16, Low SHCS, Blk Zn	4
27a	115554	Backrest Actuation Bar, 12" Wide, Black	2
27b	115555	Backrest Actuation Bar, 13" Wide, Black	1
27c	115556	Backrest Actuation Bar, 14" Wide, Black	1
27d	115557	Backrest Actuation Bar, 15" Wide, Black	1
27e	115558	Backrest Actuation Bar, 16" Wide, Black	1
27f	115559	Backrest Actuation Bar, 17" Wide, Black	1
27g	115560	Backrest Actuation Bar, 18" Wide, Black	1
27h	115561	Backrest Actuation Bar, 19" Wide, Black	1
27i	115562	Backrest Actuation Bar, 20" Wide, Black	1
27j	117921_ _ _	Backrest Actuation Bar, 12" Wide, Painted	1
27k	117922_ _ _	Backrest Actuation Bar, 13" Wide, Painted	1
27l	117923_ _ _	Backrest Actuation Bar, 14" Wide, Painted	1
27m	117924_ _ _	Backrest Actuation Bar, 15" Wide, Painted	1
27n	117925_ _ _	Backrest Actuation Bar, 16" Wide, Painted	1
27o	117926_ _ _	Backrest Actuation Bar, 17" Wide, Painted	1
27p	117927_ _ _	Backrest Actuation Bar, 18" Wide, Painted	1
27q	117928_ _ _	Backrest Actuation Bar, 19" Wide, Painted	1
27r	117929_ _ _	Backrest Actuation Bar, 20" Wide, Painted	1
28	115549	Relax Bumper	2
29	115548	Relax Saddle	2
30	115572	M5 x 0.8 x 6 BHCS BLZ, CLS 10.9	2