


Catalyst 5 and Spark Transit

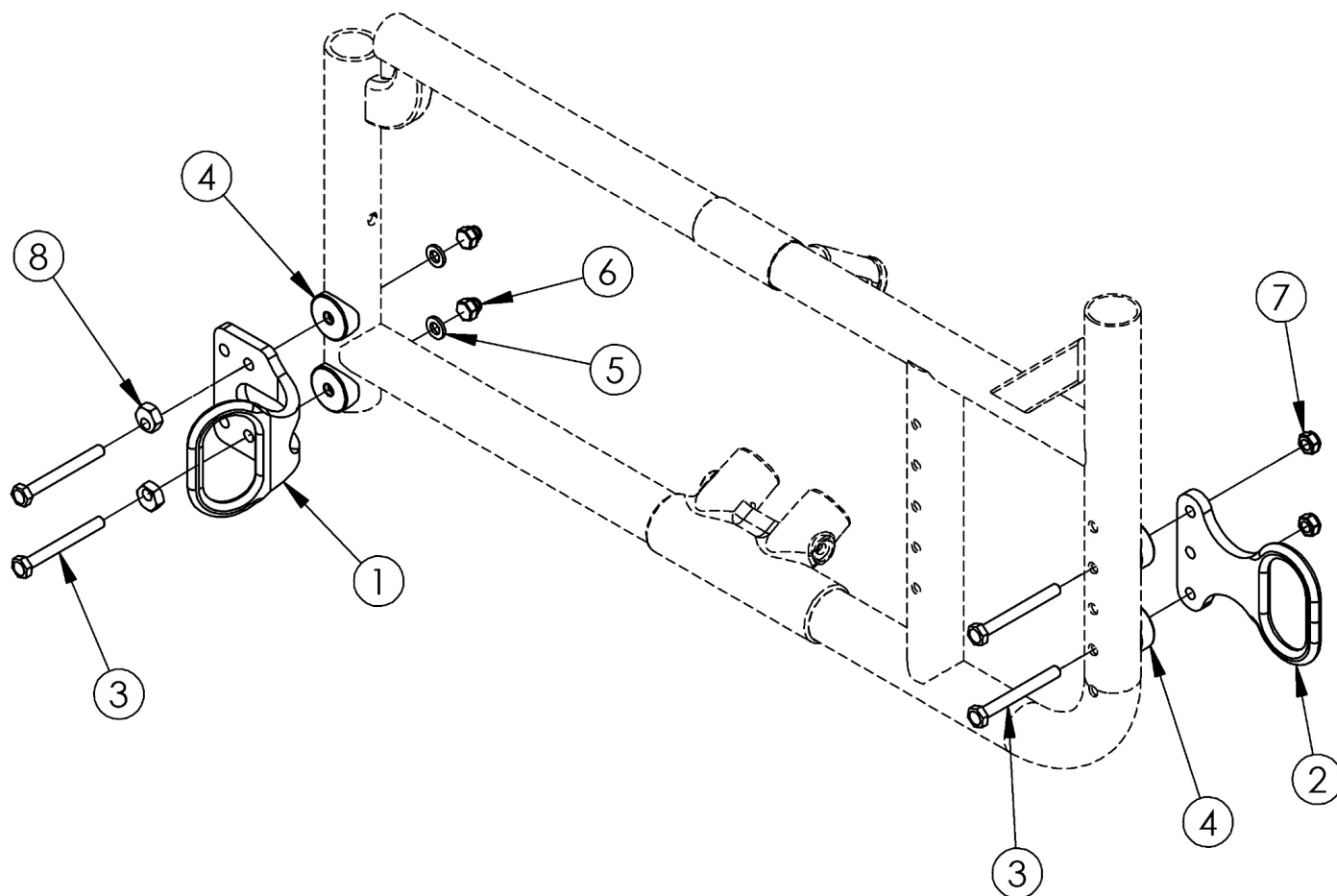

Notice: Chair configuration review required prior to parts being quoted or ordered. Not available with Heavy Duty Option, Reclining Backrest or Removable Non-Transit Seat Pan See Axle Plate pages for all mounting hardware If retrofitting and seat pan is present, cross braces will need to be changed to seating system and transit seat pan added.

Item Number	Part Number	Description	Quantity
1-2, 3a-3f, 4a-4b, 5-8	500548	Transit Bracket Retrofit Kit CAT5 / SPARK <i>Not available with Heavy Duty Option. This kit is a complete retrofit, containing a pair of front and rear transit brackets and hardware.</i>	1
3a	100674	M6 x 45 CLS10.9 DIN931 HHCS ZC	8
3b	101826	M6 x 50 HHCS Blk Zn	8
3c	100676	M6 x 55 CLS10.9 DIN931 HHCS ZC	4
3d	100677	M6 x 60 CLS10.9 DIN931 HHCS ZC	8
3e	100679	M6 x 65 CLS 8.8 DIN931 HHCS ZC	4
3f	100680	M6 x 70 CLS 8.8 DIN931 HHCS ZC	4

Item Number	Part Number	Description	Quantity
4a	000056	Saddle 1 1/8"	10
4b	000055	Saddle 1"	6
5	100746	M6 Flat Washer Blk Zn	4
6	100558	M6 Nyl Ins Domed Cap L/N Zc	4
7	100658	M6 Nylock Nut Blk Zn	8
8	100510	Eccentric Washer	4

CATALYST 5 CURVED REAR FRAME		DEPTH ADJUSTABLE REAR FRAME	
AXLE PLATE, FRAME	100674	AXLE PLATE, FRAME	100681
AXLE PLATE, FRAME, ANTI-TIP	101826	AXLE PLATE, FRAME, ANTI-TIP	100674
AXLE PLATE, FRAME, TRANSIT	101826	AXLE PLATE, FRAME, TRANSIT	101826
AXLE PLATE, FRAME, SWING AWAY	101826	AXLE PLATE, FRAME, SWING AWAY	101826
AXLE PLATE, FRAME, ANTI-TIP, TRANSIT	100677	AXLE PLATE, FRAME, ANTI-TIP, TRANSIT	100676
AXLE PLATE, FRAME, SWING AWAY, TRANSIT	100677	AXLE PLATE, FRAME, SWING AWAY, TRANSIT	100676
AXLE PLATE, FRAME, SWING AWAY, ANTI-TIP	100677	AXLE PLATE, FRAME, SWING AWAY, ANTI-TIP	100676
AXLE PLATE, FRAME, SWING AWAY, TRANSIT AND ANTI-TIP	100679	AXLE PLATE, FRAME, SWING AWAY, TRANSIT AND ANTI-TIP	100679