


CR45 Solid Seat Pan

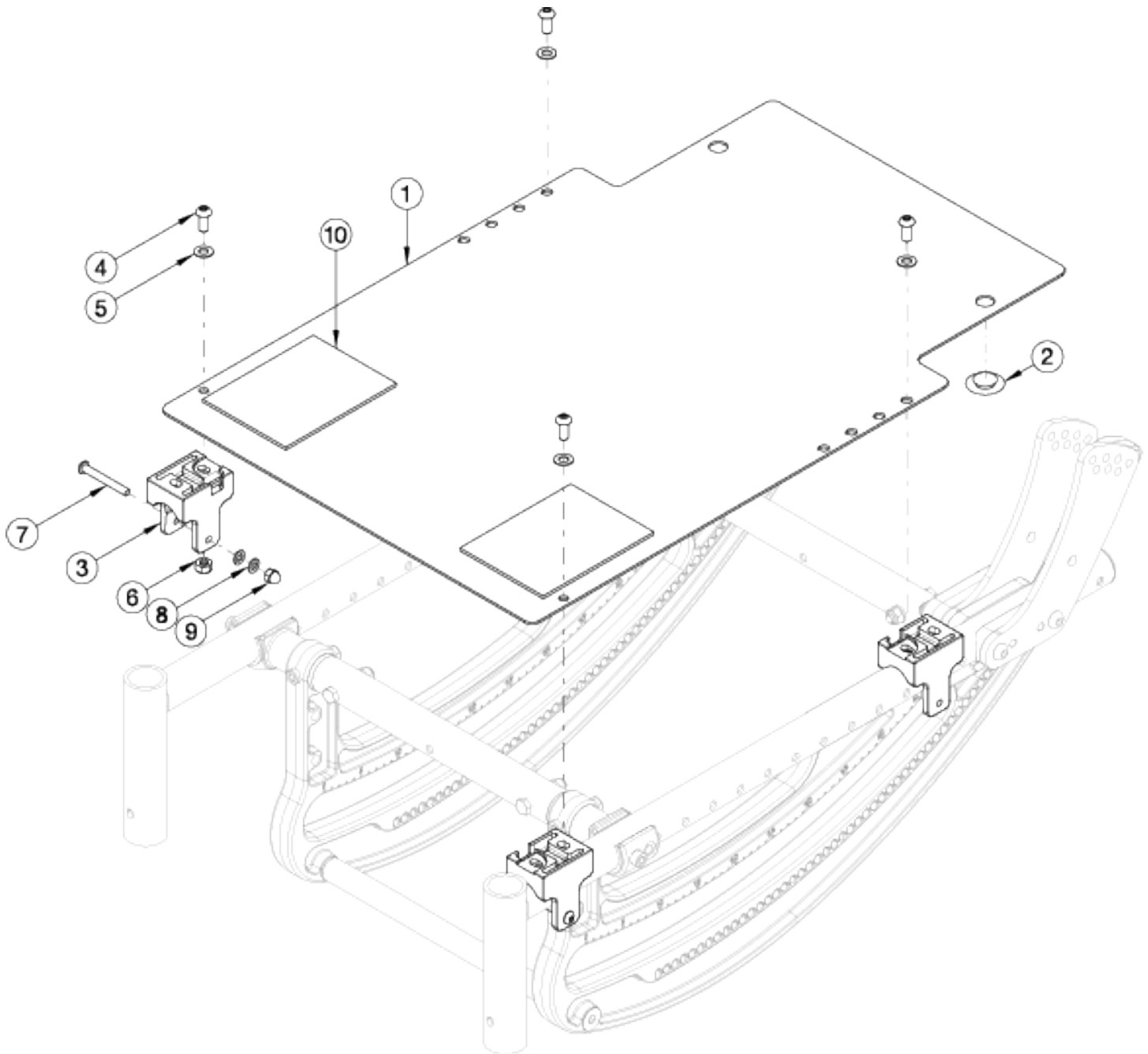

Item Number	Part Number	Description	Quantity
1a, 2-10	121255	14W, Short, Solid Seat Pan Assembly Tilt	1
1b, 2-10	121256	15W, Short, Solid Seat Pan Assembly Tilt	1
1c, 2-10	121257	16W, Short, Solid Seat Pan Assembly Tilt	1
1d, 2-10	121258	17W, Short, Solid Seat Pan Assembly Tilt	1
1e, 2-10	121259	18W, Short, Solid Seat Pan Assembly Tilt	1

Item Number	Part Number	Description	Quantity
1f, 2-10	121260	19W, Short, Solid Seat Pan Assembly Tilt	1
1g, 2-10	121261	20W, Short, Solid Seat Pan Assembly Tilt	1
1h, 2-10	121262	21W, Short, Solid Seat Pan Assembly Tilt	1
1i, 2-10	121263	22W, Short, Solid Seat Pan Assembly Tilt	1
1j, 2-10	121264	14W, Medium, Solid Seat Pan Assembly Tilt	1
1k, 2-10	121265	15W, Medium, Solid Seat Pan Assembly Tilt	1
1l, 2-10	121266	16W, Medium, Solid Seat Pan Assembly Tilt	1
1m, 2-10	121267	17W, Medium, Solid Seat Pan Assembly Tilt	1
1n, 2-10	121268	18W, Medium, Solid Seat Pan Assembly Tilt	1
1o, 2-10	121269	19W, Medium, Solid Seat Pan Assembly Tilt	1
1p, 2-10	121270	20W, Medium, Solid Seat Pan Assembly Tilt	1
1q, 2-10	121271	21W, Medium, Solid Seat Pan Assembly Tilt	1
1r, 2-10	121272	22W, Medium, Solid Seat Pan Assembly Tilt	1
1s, 2-10	121273	14W, Long, Solid Seat Pan Assembly Tilt	1
1t, 2-10	121274	15W, Long, Solid Seat Pan Assembly Tilt	1
1u, 2-10	121275	16W, Long, Solid Seat Pan Assembly Tilt	1
1v, 2-10	121276	17W, Long, Solid Seat Pan Assembly Tilt	1
1w, 2-10	121277	18W, Long, Solid Seat Pan Assembly Tilt	1
1x, 2-10	121278	19W, Long, Solid Seat Pan Assembly Tilt	1
1y, 2-10	121279	20W, Long, Solid Seat Pan Assembly Tilt	1
1z, 2-10	121280	21W, Long, Solid Seat Pan Assembly Tilt	1
1ab, 2-10	121281	22W, Long, Solid Seat Pan Assembly Tilt	1
3	116538	Universal Seat Pan Transit Bracket, Catalyst and CR45	4
4	101708	M6 x 14 BTN SHCS BLKZC w/Patch	4
5	102803	Washer Flat .250 X .500 X .027 Hx Blz	4
6	100658	M6 Nylock Nut Blk Zn	4
7	121064	BHCS, M5x0.8x40 CLS 10.9 P/THD BLKZ	2
8	101856	Flat Washer, M5, Blk Zn	4

Item Number	Part Number	Description	Quantity
9	103056	M5 Cap Nut Black Zinc	2
10	004458	AB Tape, 3" x 4"	2