


Spark Stroller Handle Back Post

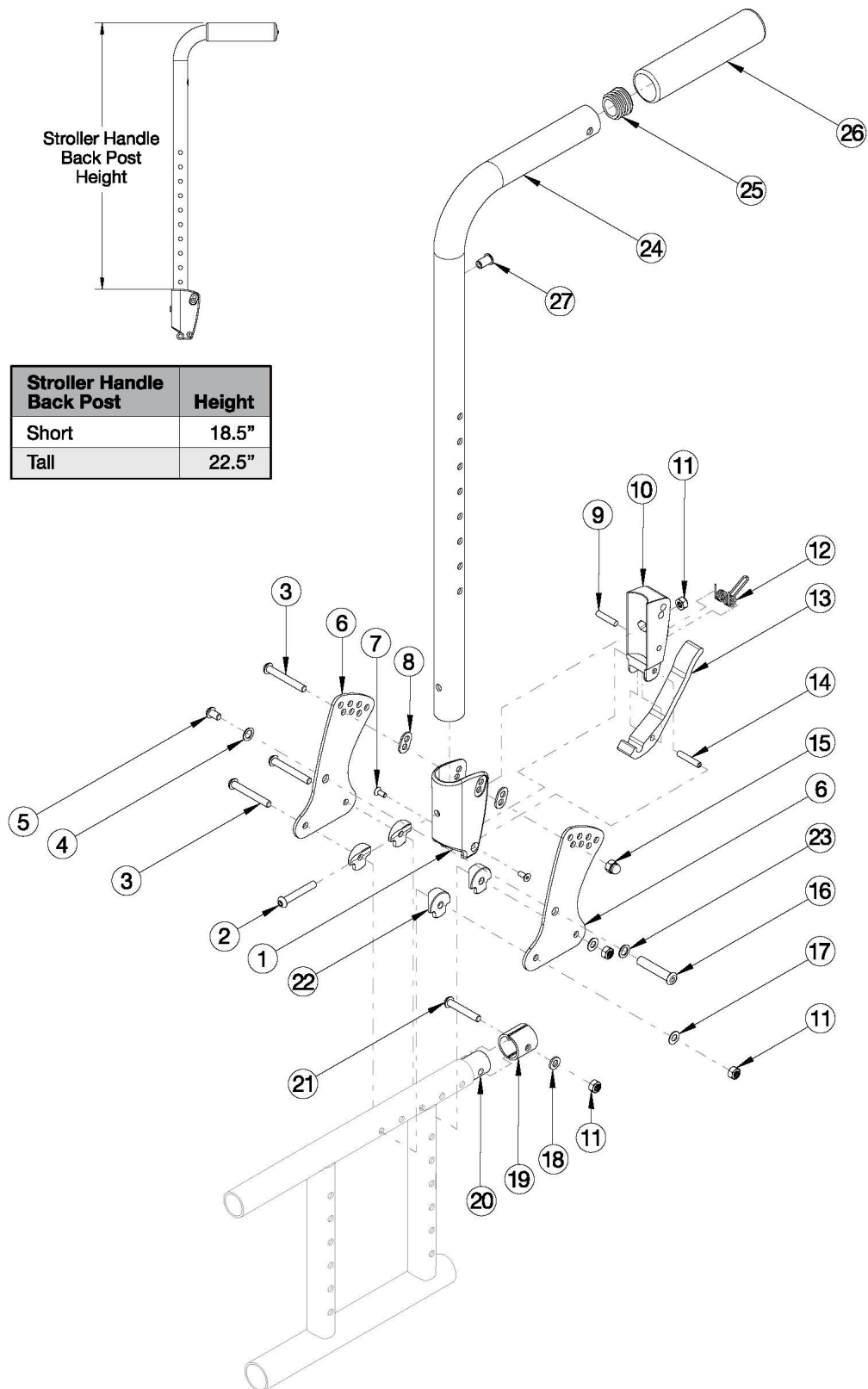

Stroller Handle Back Post	Height
Short	18.5"
Tall	22.5"

This backrest style was released February 8th, 2021

Item Number	Part Number	Description	Quantity	Retail
1-20, 22, 23, 24a, 25, 26, 27	112601	Stroller Handle Back Post with Grip Assembly with All Hardware, for Seating Cross Braces, Short	1	\$181.84
1-20, 22, 23, 24b, 25, 26, 27	112602	Stroller Handle Back Post with Grip Assembly with All Hardware, for Seating Cross Braces, Tall	1	\$181.84
24a, 25, 26, 27	506811	Stroller Handle Back Post with Grip Assembly, Short	1	\$49.67
24b, 25, 26, 27	506812	Stroller Handle Back Post with Grip Assembly, Tall	1	\$49.67
1-19, 21, 22, 23	112605	Depth Adjustable Backrest Hardware Assembly, for Seating Cross Braces, One Side	1	\$138.33
1-19, 21, 22, 23	112606	Depth Adjustable Backrest Hardware Assembly, for Seating Cross Braces, PAIR	1	\$276.66
1, 8	503834	Cover Plate, Latch Insert Kit	1	\$25.33
2	100797	M6 x 40 BHCS BLK ZC	1	\$1.05
3	100670	M6 x 45 BTN SHCS BLK ZC	3	\$1.58
4	101701	Washer 3.315/.350 x .425/.525 x .059/.067 18-8 F/W	1	\$2.63
5	101184	M6 x 10 BTN SHCS BLKZC w/Patch	1	\$1.58
6	111833	Depth Adjustable Back Plate, Folding	2	\$23.21
7	101818	M4 x 10 FHCS BLKZC w/Patch	2	\$1.05
8	001897	Washer, Dual, PSA	2	\$2.09
9	102694	Dowel Pin, M5 X 24, Blk Zn	1	\$1.58
10	001895	Latch Insert, Folding Backrest	1	\$6.60
11	100658	M6 Nylock Nut Blk Zn	4	\$1.05
12	001896	Spring, Backrest Latch	1	\$0.53
13	001893	Latch Lever, Backrest	1	\$13.34
14	002513	Threaded Barrel, 5mm OD x M4	1	\$1.94
15	100558	M6 NYL INS Domed Cap L/N ZC	1	\$1.05
16	003284	Binding Post, 8mm x 40mm x M6	1	\$1.46
17	102803	Washer Flat .250 x .500 x .027 HX BLZ	2	\$1.58
18	100746	M6 Flat Washer Blk Zn	1	\$1.58
19	113140	Tube Spacer .88 x .083 <i>Replacement for part number 001230</i>	2	\$3.37

Item Number	Part Number	Description	Quantity	Retail
20	000605	Depth Adjustable Frame Extension Tube, Catalyst/Spark	1	\$37.14
21	101811	M6 x 35 BHCS Blk Zn	1	\$1.05
22	000612	1" Half Saddle Washer	4	\$1.92
23	101861	Nylon Washer 5/16 x 1/2 x .03	1	\$0.53
25	100804	Tube End Plug 1"	1	\$1.58