


Focus CR Transit Forks and Stems

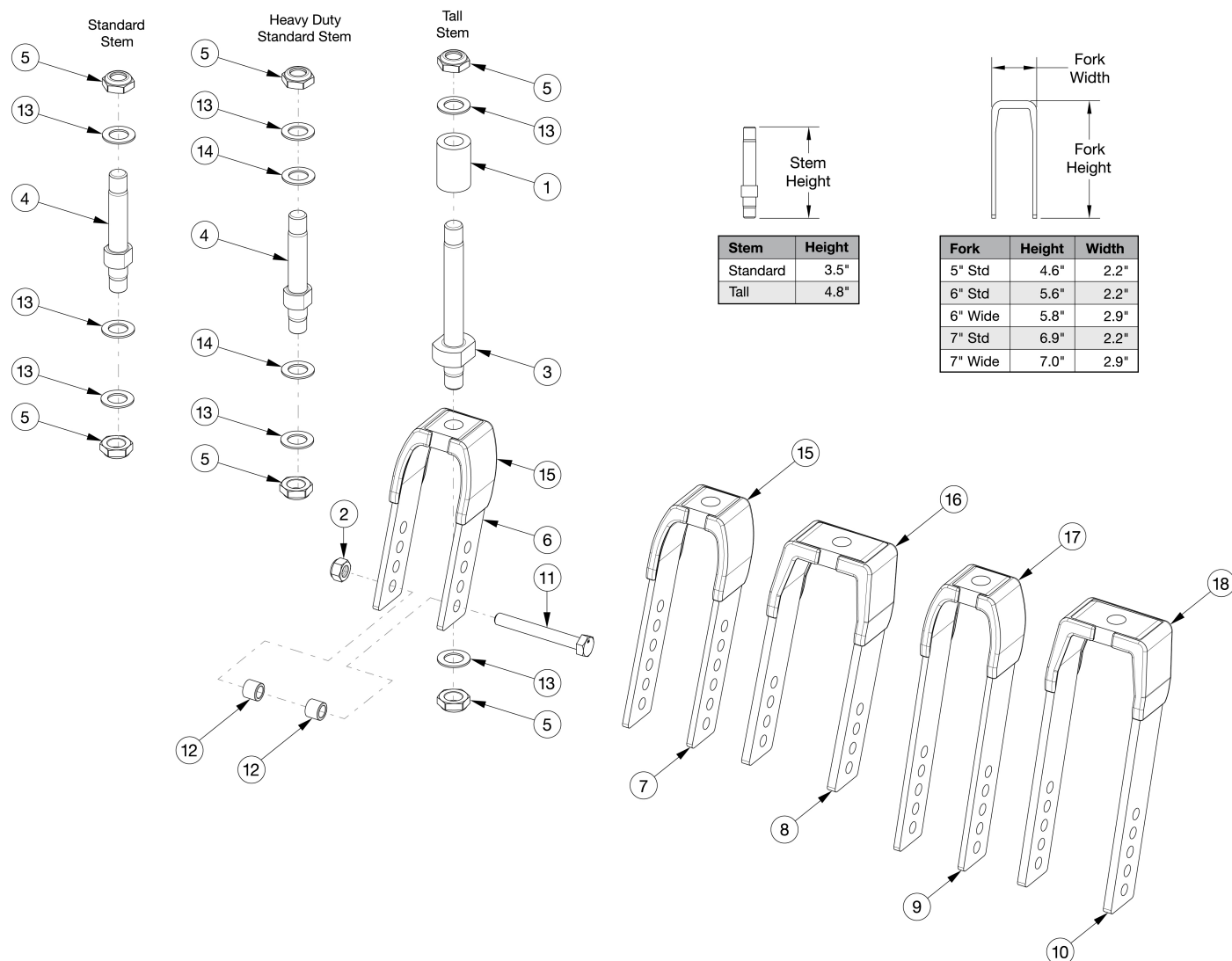

Item Number	Part Number	Description	Quantity
4a, 5, 6, 13, 15	123109	Transit Fork Assembly 5" Std Stem, Tilt <i>Used with standard weight capacity</i>	1
4a, 5, 7, 13, 15	123107	Transit Fork Assembly 6" Std Stem, Tilt <i>Used with standard weight capacity</i>	1
4a, 5, 8, 13, 16	123110	Transit Fork Assembly 6" Wide Std Stem, Tilt <i>Used with standard weight capacity. Only used with 2" wide casters.</i>	1
4a, 5, 9, 13, 17	123108	Transit Fork Assembly 7" Std Stem, Tilt <i>Used with standard weight capacity</i>	1
4a, 5, 10, 13, 18	123111	Transit Fork Assembly 7" Wide Std Stem, Tilt	1

Item Number	Part Number	Description	Quantity
		<i>Used with standard weight capacity. Only used with 2" wide casters.</i>	
1, 3, 5, 6, 13, 15	123127	Transit Fork Assembly 5" Tall Stem Tilt <i>Used with standard and heavy duty weight capacity</i>	1
1, 3, 5, 7, 13, 15	123128	Transit Fork Assembly 6" Tall Stem Tilt <i>Used with standard and heavy duty weight capacity</i>	1
1, 3, 5, 8, 13, 16	123129	Transit Fork Assembly 6" Wide Tall Stem Tilt <i>Used with standard and heavy duty weight capacity. Only used with 2" wide casters.</i>	1
1, 3, 5, 9, 13, 17	123130	Transit Fork Assembly 7" Tall Stem Tilt <i>Used with standard and heavy duty weight capacity.</i>	1
1, 3, 5, 10, 13, 18	123131	Transit Fork Assembly 7" Wide Tall Stem Tilt <i>Used with standard and heavy duty weight capacity. Only used with 2" wide casters.</i>	1
2, 11a, 12a	502653	Caster Attachment Hardware Kit CAT/FCS	1
2, 11b, 12b	502654	Caster Attachment Hardware Kit for 6" and 7" Wide Fork, CAT/FCS <i>Only used with 2" wide casters</i>	1
1	002476	Spacer, 3/4" OD x 1/2" ID x 1.23	1
2	100659	M8 Nylock Nut Blk Zn	1
1, 3, 5, 13	506374	Caster Stem Tall, Adult Tilt <i>Used with standard and heavy duty weight capacity</i>	1
4a, 5, 13	503819	Caster Stem Standard, Tilt <i>Used with standard weight capacity</i>	1
5	100529	M12 x 1.25mm Lock Nut Black	2
5, 6, 15	123112	Transit Fork 5" Tilt	1
5, 7, 15	123113	Transit Fork 6" Tilt	1
5, 8, 16	123114	Transit Fork 6" Wide Tilt <i>Only used with 2" wide casters</i>	1
5, 9, 17	123115	Transit Fork 7" Tilt	1
5, 10, 18	123116	Transit Fork 7" Wide Tilt <i>Only used with 2" wide casters</i>	1
11a	101829	M8 x 65 HHCS Blk Zn	1
11b	101831	M8 x 80 HHCS Blk Zn <i>Only used with 2" wide casters</i>	1
12a	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer	2

Item Number	Part Number	Description	Quantity
		<i>Used with all casters except 6x2 and 8x2</i>	
12b	001246	Caster Spacer Black 6 x 2 Poly .472ODX.319IDX.370LG <i>Only used with 6" x 2" casters</i>	2
13	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	3